APPROVED BY


Advanced Healthcare Products SDN BHD

(signature)

Tuan Hj Syed Al Aliffni, CEO
(name, position)

« 20 » November 20 16
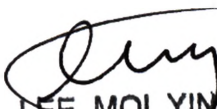
LEE MOI YIN
NOTARY PUBLIC
PETALING JAYA
SELANGOR
MALAYSIA

USER MANUAL

Powder-free nitrile examination gloves, non-sterile CONNECT
(Advanced Healthcare Products SDN BHD)

2016

Contents

Description	3
Intended use	3
Specification	4
Packing and marking	10
Information on sterilisation	12
Transportation, storage and in-service conditions.....	13
Information for healthcare professionals	13
Directions for use	14
Disposal procedure.....	14
Warranty.....	14

Description

Powder-free nitrile examination gloves, non-sterile CONNEC» (hereinafter – gloves).

Design styles:

1. Powder-free nitrile examination gloves, non-sterile CONNECT Regular, sizes: XS, S, M, L, XL.
2. Powder-free nitrile examination gloves, non-sterile CONNECT Long, sizes: XS, S, M, L, XL.
3. Powder-free nitrile examination gloves, non-sterile CONNECT Blaxx, sizes: XS, S, M, L, XL.
4. Powder-free nitrile examination gloves, non-sterile CONNECT Grande, sizes: XS, S, M, L, XL.

Manufacturer: Advanced Healthcare Products SDN BHD

Lot 60 & 61, Lorong Senawang 3/2, Senawang Industrial Estate, 70450 Seremban, Negeri Sembilan Darul Khusus, Malaysia.

Manufacturing site:

Advanced Healthcare Products SDN BHD

Lot 60 & 61, Lorong Senawang 3/2, Senawang Industrial Estate, 70450 Seremban, Negeri Sembilan Darul Khusus, Malaysia.

The manufacturer possesses a certified quality managements system, the manufacturing process and final quality control fully comply with the requirements established by EN ISO 13485:2012 (Certificate No. 49048 date of issue – 22.06.2016, valid until – 31.03.2019, Notified Body: NQA), EN ISO 9001:2008 (Certificate No. 49047 date of issue – 22.06.2016, valid until 14.09.2018, Notified Body: NQA).

Classification Code in accordance with Russian National Classification of Products OK-005-93: 25 1460.

Risk class in accordance with Classification List of Medical Devices approved by Order of Ministry of Health of the Russian Federation No. 4n dated 06 June, 2012 – 1.

Classification Code in accordance with Classification List of Medical Devices approved by Order of Ministry of Health of the Russian Federation No. 4n dated 06 June, 2012 – 185830.

Human body contact type: short-time contact with skin cover and mucous membranes.

Application environment: healthcare facilities.

Intended use

The gloves are used as two-way barrier to prevent healthcare consumer and healthcare professional from mutual introduction of infections during medical trials, medical diagnostic procedures and therapeutic measures as well as while handling pathological waste.

Specification

Materials used are shown in Table 1.

Table 1. Materials used


Design style	Material
Product:	
CONNECT Regular	• Synthetic rubber latex (acrylonitrile-butadiene) POLYLAC 582N CAS # 9003-18-3, produced by Shin Foong Chemical Industry Co., Ltd, Taiwan. • Colouring agent: Colanyl Blue A2R 100-CN, CAS # 109909-39-9, produced by Clariant Pigments (Tianjin) Ltd, China.
CONNECT Long	• Synthetic rubber latex (acrylonitrile-butadiene) POLYLAC 582N CAS # 9003-18-3, produced by Shin Foong Chemical Industry Co., Ltd, Taiwan. • Colouring agent: Colanyl Blue A2R 100-CN, CAS # 109909-39-9, produced by Clariant Pigments (Tianjin) Ltd, China.
CONNECT Blaxx	• Synthetic rubber latex (acrylonitrile-butadiene) POLYLAC 582N CAS # 9003-18-3, produced by Shin Foong Chemical Industry Co., Ltd, Taiwan. • Colouring agent: Farsperse Black PB705, CAS # 1333-86-4, produced by Farben Technique (M) Sdn Bhd, Malaysia
CONNECT Grande	• Synthetic rubber latex (acrylonitrile-butadiene) POLYLAC 582N CAS # 9003-18-3, produced by Shin Foong Chemical Industry Co., Ltd, Taiwan. • Colouring agent: Colanyl Blue A2R 100-CN, CAS # 109909-39-9, produced by Clariant Pigments (Tianjin) Ltd, China.
Dispenser (consumer package):	
CONNECT Regular	400 gsm Duplex Board, produced by Percetakan Tenaga Sdn Bhd, country of origin – Malaysia.
CONNECT Long	400 gsm Duplex Board, produced by Percetakan Tenaga Sdn Bhd, country of origin – Malaysia.
CONNECT Blaxx	400 gsm Duplex Board, produced by Percetakan Tenaga Sdn Bhd, country of origin – Malaysia.
CONNECT Grande	400 gsm Duplex Board, produced by Percetakan Tenaga Sdn Bhd, country of origin – Malaysia.

External appearance of the gloves is shown in Table 2

Table 2. External appearance

Design style	External appearance
CONNECT Regular	
CONNECT Long	

CONNECT
Blaxx

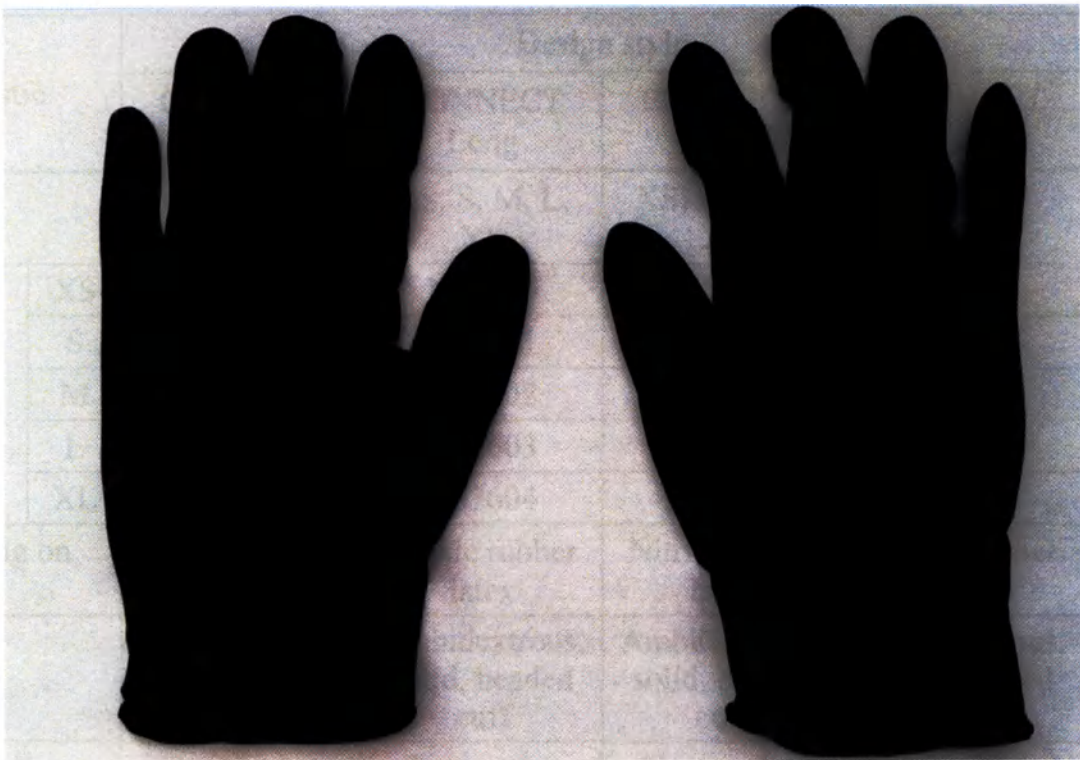CONNECT
Grande

Technical characteristics are shown in Table 3

Table 3. Technical characteristics

Characteristic		Design style			
		CONNECT Regular	CONNECT Long	CONNECT Blaxx	CONNECT Grande
Size		XS, S, M, L, XL	XS, S, M, L, XL	XS, S, M, L, XL	XS, S, M, L, XL
Reorder No. depending on the size	XS	N85500	N87600	NB4800	N86500
	S	N85501	N87601	NB4801	N86501
	M	N85502	N87602	NB4802	N86502
	L	N85503	N87603	NB4803	N86503
	XL	N85504	N87604	NB4804	N86504
Type (depending on the material)		Nitrile rubber latex	Nitrile rubber latex	Nitrile rubber latex	Nitrile rubber latex
Design		Ambidextrous, solid, beaded cuff	Ambidextrous, solid, beaded cuff	Ambidextrous, solid, beaded cuff	Ambidextrous, solid, beaded cuff
Surface finish	textured/ smooth	Textured at fingers	Textured at fingers	Textured at finger-tips	Textured at fingers
	powdered/ powder-free	Powder-free	Powder-free	Powder-free	Powder-free
Surface processing		Chlorination	Chlorination	Chlorination	Chlorination
Powder residue, mg/glove, not more		2	2	2	2
Colouring		Blue	Blue	Black	Blue

Design:

Overall dimensions are shown in Table 4.

Table 4. Overall dimensions

Design style	Size	width W, mm	length l, not less, mm	Single-wall thickness, mm			Thickness of bead, not more, mm
				finger, not less	palm, not more	cuff, not less	
CONNECT Regular	XS	≤ 80	240	0.13	0.22	0.08	2.50
	S	80 ± 10	240	0.13	0.22	0.08	2.50
	M	95 ± 10	240	0.13	0.22	0.08	2.50
	L	110 ± 10	240	0.13	0.22	0.08	2.50
	XL	≥ 110	240	0.13	0.22	0.08	2.50
CONNECT Long	XS	≤ 80	290	0.13	0.22	0.08	2.50
	S	80 ± 10	290	0.13	0.22	0.08	2.50
	M	95 ± 10	290	0.13	0.22	0.08	2.50
	L	110 ± 10	290	0.13	0.22	0.08	2.50
	XL	≥ 110	290	0.13	0.22	0.08	2.50
CONNECT Blaxx	XS	75 ± 5	240	0.13	0.22	0.07	2.50
	S	85 ± 5	240	0.13	0.22	0.07	2.50
	M	95 ± 5	240	0.13	0.22	0.07	2.50
	L	105 ± 5	240	0.13	0.22	0.07	2.50
	XL	115 ± 5	240	0.13	0.22	0.07	2.50
CONNECT Grande	XS	≤ 80	290	0.14	0.22	0.06	2.50
	S	80 ± 10	290	0.14	0.22	0.06	2.50
	M	95 ± 10	290	0.14	0.22	0.06	2.50
	L	110 ± 10	290	0.14	0.22	0.06	2.50
	XL	≥ 110	290	0.14	0.22	0.06	2.50

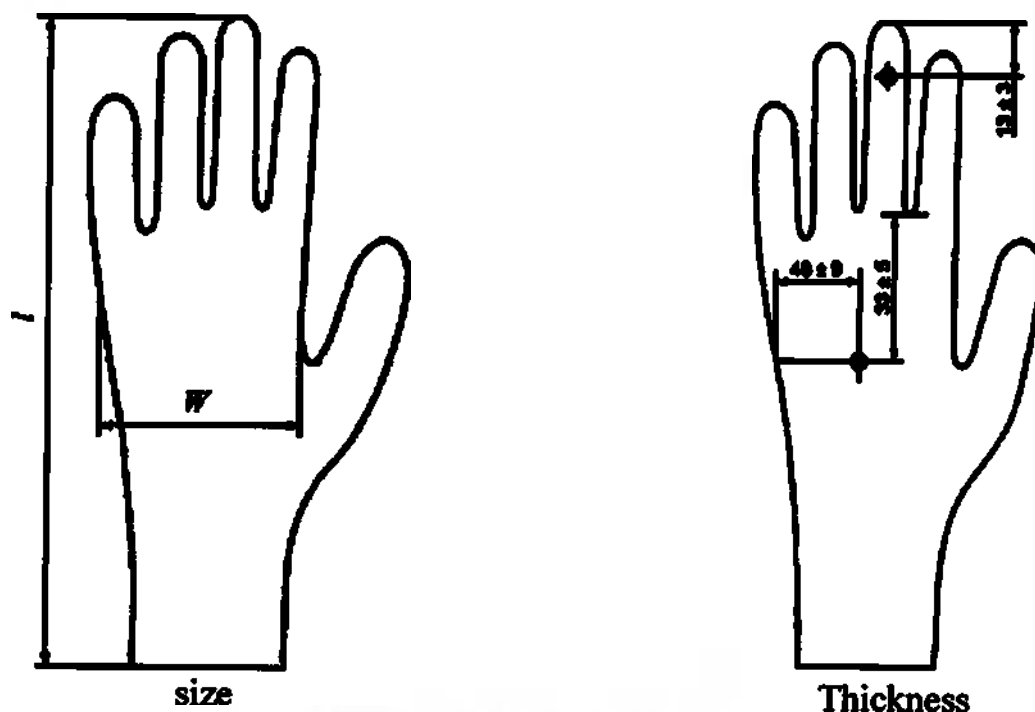
Figure 1 – Measurement points

Physical properties are shown in Table 5.

Table 5. Physical properties

Characteristic	Design style							
	CONNECT Regular		CONNECT Long		CONNECT Blaxx		CONNECT Grande	
	Before aging	After aging	Before aging	After aging	Before aging	After aging	Before aging	After aging
Elongation at break, %, not less	500	400	500	400	500	400	500	400
Force at break, N, not less	9	6	9	6	7	6	7	6
Tensile Strength, MPa, not less	14	14	14	14	14	14	14	14

Inspection level requirements, AQL requirements are shown in Table 6.

Table 6. Inspection level requirements, AQL requirements

Characteristic	Inspection level	AQL
Dimensions (length, thickness, width)	S-2	4,0
1000 water leak	G-1	2,5
Force and elongation at break (before and after aging)	S-2	4,0

Packing and marking

100 gloves by weight per dispenser (consumer package).

10 dispensers per outer carton (transportation package).

Outer carton secures safe transportation and storage.

Dispenser marking is as follows:

- name and address of the manufacturer in Russian and English;
- name and address of the manufacturer's representative in the Russian Federation;
- gloves name;
- «Examination gloves»;
- «Material: nitrile rubber latex»;
- «Disposable»;
- «Non-sterile»;
- «Ambidextrous»;
- «Liquid-tight»;
- «Solid»;
- «Powder-free»;
- «Beaded cuff»;
- «Textured at finger-tips» (for design style: CONNECT Blaxx);
- «Textured at fingers» (for design styles: CONNECT Regular, CONNECT Long, CONNECT Grande);
- «Extended cuff» (for design styles: CONNECT Long, CONNECT Grande);
- size;
- colour;
- quantity of gloves (by weight) per dispenser;
- lot number;
- reorder number;
- date of manufacture;
- shelf life;
- registration certificate number;
- application rules/warning;
- directions for use;
- «Made in Malaysia»;
- «GOST R 52239-2004»;
- symbols:

-  – Reorder number
-  – Lot number
-  – Date of manufacture
-  – Single use only
-  – Does not contain natural rubber latex
-  – Non-sterile
-  – Do not use if the package is damaged
-  – Manufacturer
-  – Avoid sunlight
-  – Avoid moisture
-  – Temperature limits (from 0 through +25 °C)
-  – Read user manual
-  – Warning! Read user manual

Outer carton marking is made in Russian and English as follows:

- name and address of the manufacturer;
- name and address of the manufacturer's representative in the Russian Federation;
- gloves name;
- size;
- quantity of gloves per dispenser;
- quantity of dispensers per outer carton;
- quantity of gloves per outer carton;
- lot number;

- reorder number;
- date of manufacture;
- shelf life;
- outer carton number;
- registration certificate number;
- warning «Store in cool dry place. Avoid ozon and direct sunlight»;
- «Made in Malaysia»;
- warning «This product is to be store in carton during transportation»;
- symbols:

REF

– Reorder number

LOT

– Lot number

– Date of manufacture

– Single use only

– single use only

– Non-sterile

– Do not use if the package is damaged

– Manufacturer

– Avoid sunlight

– Top

– Medical device. Handle with care

Delivery set:

- dispenser – 1 piece;

Abridged directions for use are printed on the dispenser.

Information on sterilisation

The gloves are non-sterile.

Single use only.

Reuse is prohibited.

Transportation, storage and in-service conditions

Storage conditions:

Gloves are to be stored packed in original manufacturer's package in cool and well-ventilated premises. Temperature – from 0°C through +25°C, relative humidity – $\leq 85\%$ (non-condensing) and atmospheric pressure – 500–1060 hPa, at a distance of not less than one meter from heat-radiating appliances.

While storing the gloves must be protected against exposure to direct sunlight, fluorescent lighting, atmospheric agents, ozone, greases, petroleum, paraffin and other substances destroying latex; the gloves should not be in direct contact with metals such as copper, manganese and iron.

Transportation conditions:

Temperature – from 0°C through +25°C, relative humidity – $\leq 85\%$ (non-condensing) and atmospheric pressure – 500–1060 hPa

Transportation of gloves packed into transportation package is effected by any means of roofed transport subject to all warnings indicated on the transportation package and haulage rules for the means of transport used.

After transportation at subzero temperatures gloves can be unpacked and used only after having been kept in the transportation package at the storage temperature for a period of 24 hours.

In-service conditions:

Temperature – from +5°C through +35°C, relative humidity – from 35% through 60% (non-condensing) and atmospheric pressure – 500–1060 hPa.

Do not use gloves after expiry date.

Single use only.

Information for healthcare professionals

Indications:

The gloves are used as two-way barrier to prevent healthcare consumer and healthcare professional from mutual introduction of infections during medical trials, medical diagnostic procedures and therapeutic measures as well as while handling pathological waste.

Counter indications:

Idiosyncrasy or allergy to gloves material.

Potential complications:

Allergic response.

Safety precautions:

- The gloves are intended for use as personal hand protective device.
- Do not use damaged gloves.
- Do not use gloves of improper size.
- Single use only.
- Do not use gloves after the expiry date.
- The gloves do not guarantee protection from punctures and cuts.

Directions for use

- Sanitize hands.
- Make sure the gloves are intact.
- Carefully put on the right and left gloves so that the gloves envelope the hands.
- Take off the gloves after use and put them into a special container suitable for transportation of used disposable medical devices.

Disposal procedure

The gloves are to be disposed of in accordance with Sanitary Regulations and Rules 2.1.7.2790-10 for class B waste (epidemiologically hazardous waste). After disinfection with special physical methods equipment and external appearance change ensuring impossibility of reuse of the device, class B waste can be stocked up, temporarily stored, transported, destroyed and buried together with class A waste (epidemiologically non-hazardous waste, close in composition to household waste). Package of disinfected class B waste must bear marking showing the waste is disinfected.

Warranty

Shelf life: 5 years from the date of manufacture.

The manufacturer guarantees the gloves sterility and full compliance with the specified requirements during the shelf life subject consumer following application, storage and transportation rules.

Manufacturer:

Advanced Healthcare Products SDN BHD

Lot 60 & 61, Lorong Senawang 3/2, Senawang Industrial Estate, 70450 Seremban, Negeri Sembilan Darul Khusus, Malaysia.

Tel.: +60-6-678-4109, fax: +60-6-678-4727

Manufacturer authorized representative in the Russian Federation:

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