APPROVED BY

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USER MANUAL

(ADDENDUM 1)

✦ Powder-free latex examination gloves, non-sterile CONNECT

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2017

Table. Further information on medical device overall dimensions and thickness

Design style	Size	min width for size XS, mm	max width for size XL, mm	max length, mm	max thickness at finger, mm*	min thickness at palm, mm*	max thickness at cuff, mm*	min thickness of bead, mm
CONNECT Basic	XS	60		260	0.16	0.08	0.12	1.00
	S			260	0.16	0.08	0.12	1.00
	M			260	0.16	0.08	0.12	1.00
	L			260	0.16	0.08	0.12	1.00
	XL		127	260	0.16	0.08	0.12	1.00
CONNECT Strong	S			310	0.45	0.17	0.21	1.00
	M			310	0.45	0.17	0.21	1.00
	L			310	0.45	0.17	0.21	1.00
	XL		119	310	0.45	0.17	0.21	1.00
CONNECT Grip	XS	60		260	0.17	0.11	0.12	1.00
	S			260	0.17	0.11	0.12	1.00
	M			260	0.17	0.11	0.12	1.00
	L			260	0.17	0.11	0.12	1.00
	XL		127	260	0.17	0.11	0.12	1.00
CONNECT Flex	XS	60		260	0.19	0.11	0.14	1.00
	S			260	0.19	0.11	0.14	1.00
	M			260	0.19	0.11	0.14	1.00
	L			260	0.19	0.11	0.14	1.00
	XL		127	260	0.19	0.11	0.14	1.00
CONNECT Poly-M	XS	60		260	0.20	0.11	0.15	1.00
	S			260	0.20	0.11	0.15	1.00
	M			260	0.20	0.11	0.15	1.00
	L			260	0.20	0.11	0.15	1.00
	XL		127	260	0.20	0.11	0.15	1.00

Design style	Size	min width for size XS, mm	max width for size XL, mm	max length, mm	max thickness at finger, mm*	min thickness at palm, mm*	max thickness at cuff, mm*	min thickness of bead, mm
CONNECT Essential	XS	60		260	0.19	0.11	0.15	1.00
	S			260	0.19	0.11	0.15	1.00
	M			260	0.19	0.11	0.15	1.00
	L			260	0.19	0.11	0.15	1.00
	XL		127	260	0.19	0.11	0.15	1.00
CONNECT Vital	XS	60		260	0.22	0.12	0.16	1.00
	S			260	0.22	0.12	0.16	1.00
	M			260	0.22	0.12	0.16	1.00
	L			260	0.22	0.12	0.16	1.00
	XL		127	260	0.22	0.12	0.16	1.00
CONNECT X-Blanc	XS	60		310	0.40	0.16	0.21	1.00
	S			310	0.40	0.16	0.21	1.00
	M			310	0.40	0.16	0.21	1.00
	L			310	0.40	0.16	0.21	1.00
	XL		127	310	0.40	0.16	0.21	1.00

* – single-wall thickness

Transportation conditions

Transportation of gloves packed into transportation package is effected by any means of roofed transport subject to all warnings indicated on the transportation package and

Transportation by roofed transport means the gloves are not exposed to direct sunlight and other atmospheric agents.

Climatic transportation conditions: temperature limits – from -30 °C through +30 °C, relative humidity – $\leq 80\%$.

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[Handwritten signature]

APPROVED BY

Advanced Healthcare Products SDN BHD

(signature)

pp Tuan Hj Syed Al Aliff n, CEO
(name, position)

« 20 » November 20 16

USER MANUAL

Powder-free latex examination gloves, non-sterile CONNECT

Advanced Healthcare Products SDN BHD

CERTIFIED TRUE COPY
OF THE ORIGINAL

No:..... -3 FEB 2017
LEE MOI YIN
NOTARY PUBLIC
PETALING JAYA

SPECIAL CERTIFICATE OF AUTHENTICITY

Contents

1 Description of the medical device	3
2 Manufacturer/Manufacturing site	3
3 Classification	3
4 Intended use, application environment	3
5 Indications, counter-indications and potential complications	4
6 Specification.....	4
7 Directions for use.....	13
8 Complete set	13
9 Packing.....	13
10 Marking.....	13
11 In-service conditions.....	16
12 Transportation.....	16
13 Storage	16
14 Service and maintenance	17
15 Ecological safety	17
16 Disposal procedure	17
17 Warranty	17
18 Контактная информация	17

1 Description of the medical device

Description: Powder-free latex examination gloves, non-sterile CONNECT (hereinafter – gloves)

Design styles:

1. Powder-free latex examination gloves, non-sterile CONNECT Basic, sizes XS, S, M, L, XL;
2. Powder-free latex examination gloves, non-sterile CONNECT Strong, sizes S, M, L, XL;
3. Powder-free latex examination gloves, non-sterile CONNECT Grip, sizes XS, S, M, L, XL;
4. Powder-free latex examination gloves, non-sterile CONNECT Flex, sizes XS, S, M, L, XL;
5. Powder-free latex examination gloves, non-sterile CONNECT Poly-M, sizes XS, S, M, L, XL;
6. Powder-free latex examination gloves, non-sterile CONNECT Essential, sizes XS, S, M, L, XL;
7. Powder-free latex examination gloves, non-sterile CONNECT Vital, sizes XS, S, M, L, XL;
8. Powder-free latex examination gloves, non-sterile CONNECT X-Blanc, sizes XS, S, M, L, XL.

2 Manufacturer/Manufacturing site

Advanced Healthcare Products SDN BHD

Lot 60 & 61, Lorong Senawang 3/2, Senawang Industrial Estate, 70450 Seremban, Negeri Sembilan Darul Khusus, Malaysia

3 Classification

- 3.1 Classification Code in accordance with Russian National Classification of Products OK-005-93: **251460**.
- 3.2 Risk class in accordance with Classification List of Medical Devices approved by Order of Ministry of Health of the Russian Federation No. 4n dated 06 June, 2012: **I**.
- 3.3 Classification Code in accordance with Classification List of Medical Devices approved by Order of Ministry of Health of the Russian Federation No. 4n dated 06 June, 2012: **122540**.

4 Intended use, application environment

- 4.1 The gloves are used as two-way barrier to prevent healthcare consumer and healthcare professional from mutual introduction of infections during medical trials, medical diagnostic procedures and therapeutic measures as well as while handling pathological waste.
- 4.2 *Application field:* broad spectrum.
- 4.3 *Application environment:* healthcare facilities.
- 4.4 *Human body contact type:* short-time contact with skin cover and mucous membranes.

5 Indications, counter-indications and potential complications

5.1 Indications:

The gloves are used as two-way barrier to prevent healthcare consumer and healthcare professional from mutual introduction of infections during medical trials, medical diagnostic procedures and therapeutic measures as well as while handling pathological waste.

Single use only.

5.2 Counter-indications:

Idiosyncrasy or allergy to natural rubber latex.

5.3 Potential complications:

Allergic response.

6 Specification

6.1 Gloves material:

- natural rubber latex, CAS # 9006-04-6, produced by Chip Yoo Latex Co., Ltd, Thailand.
- colouring agent: Colanyl Blue A2R 100-CN, CAS # 109909-39-9, produced by Clariant Pigments (Tianjin) Ltd, China – for design style CONNECT Strong.
- polymer coating: N-Uthan 67, produced by CCM Innovative Solutions Sdn Bhd, Malaysia – for design style CONNECT Poly-M.

6.2 Dispenser (consumer package) material:

- 400 gsm duplex board, produced by Percetakan Tenaga Sdn Bhd, Malaysia – for design style CONNECT Basic, CONNECT Grip, CONNECT Flex, CONNECT Poly-M;
- 450 gsm duplex board, produced by Percetakan Tenaga Sdn Bhd, Malaysia – for design style CONNECT Essential, CONNECT Vital;
- fluted cardboard 250BB/120M/120TL E flute, produced by Percetakan Tenaga Sdn Bhd, Malaysia – for design style CONNECT Strong;
- fluted cardboard 250BB/120M/150 E flute, produced by Percetakan Tenaga Sdn Bhd, Malaysia – for design style CONNECT X-Blanc.

6.3 Technical characteristics are shown in Table 1.

6.4 The gloves are solid. Physical properties are shown in Table 2.

6.5 Overall dimensions are shown in Table 3, Figure 1.

6.6 Bead thickness – not more than 2.5 mm.

6.7 Sterility requirements: the gloves are non-sterile.

6.8 Inspection level requirements, AQL requirements are shown in Table 4.

6.9 External appearance is shown in Table 5.

Table 1 – Technical characteristics

Characteristic		Design style							
		CONNECT Basic	CONNECT Strong	CONNECT Grip	CONNECT Flex	CONNECT Poly-M	CONNECT Essential	CONNECT Vital	CONNECT X-Blanc
Size		XS, S, M, L, XL	S, M, L, XL	XS, S, M, L, XL	XS, S, M, L, XL	XS, S, M, L, XL	XS, S, M, L, XL	XS, S, M, L, XL	XS, S, M, L, XL
Reorder number depending on the size	XS	L65800		L66700	L66500	L66800	L67200	L68200	L61400
	S	L65801	L61601	L66701	L66501	L66801	L67201	L68201	L61401
	M	L65802	L61602	L66702	L66502	L66802	L67202	L68202	L61402
	L	L65803	L61603	L66703	L66503	L66803	L67203	L68203	L61403
	XL	L65804	L61604	L66704	L66504	L66804	L67204	L68204	L61404
Type (depending on the material)		Natural rubber latex	Natural rubber latex	Natural rubber latex	Natural rubber latex	Natural rubber latex	Natural rubber latex	Natural rubber latex	Natural rubber latex
Design		Ambidextrous, solid, beaded cuff	Ambidextrous, solid, beaded cuff	Ambidextrous, solid, beaded cuff	Ambidextrous, solid, beaded cuff	Ambidextrous, solid, beaded cuff	Ambidextrous, solid, beaded cuff	Ambidextrous, solid, beaded cuff	Ambidextrous, solid, beaded cuff
Surface finish	textured/ smooth	Textured at finger tips	Textured at fingers	Textured all over	Textured all over	Textured all over	Textured at fingers	Textured at fingers	Textured at fingers
	powdered/ powder-free	Powder-free	Powder-free	Powder-free	Powder-free	Powder-free	Powder-free	Powder-free	Powder-free
Surface processing		Chlorination	Chlorination	Chlorination	-	Polymer coating	Chlorination	Chlorination	Chlorination

Characteristic	Design style							
	CONNECT Basic	CONNECT Strong	CONNECT Grip	CONNECT Flex	CONNECT Poly-M	CONNECT Essential	CONNECT Vital	CONNECT X-Blanc
Powder residue, mg/glove	≤2	≤2	≤2	≤2	≤2	≤2	≤2	≤2
Protein content, µg/dm ²	≤50	≤50	≤50	≤50	≤50	≤50	≤50	≤50
Colouring	Uncoloured	Blue	Uncoloured	Uncoloured	Uncoloured	Uncoloured	Uncoloured	Uncoloured

Table 2. Physical properties

Characteristic	Design style															
	CONNECT Basic		CONNECT Strong		CONNECT Grip		CONNECT Flex		CONNECT Poly-M		CONNECT Essential		CONNECT Vital		CONNECT X-Blanc	
	Before aging	After aging	Before aging	After aging	Before aging	After aging	Before aging	After aging	Before aging	After aging	Before aging	After aging	Before aging	After aging	Before aging	After aging
Elongation at break, %, not less	650	500	750	600	650	500	650	500	650	500	650	500	650	500	750	600
Force at break, N, not less	7	6	15	10	9	6	9	6	9	6	9	6	9	6	9	6

Table 3. Overall dimensions, thickness at measurement points

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*Single-wall thickness

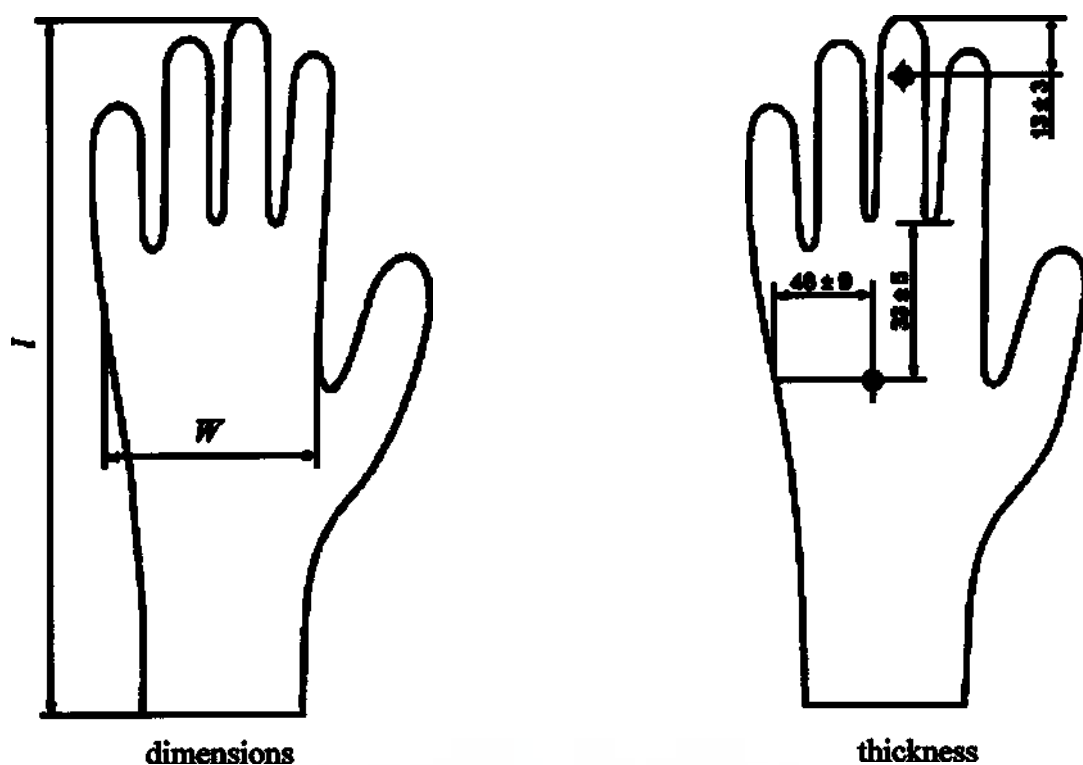
Figure 1 – Measurement points

Table 4. Inspection level requirements, AQL requirements

Characteristics	Inspection level	AQL
Dimensions (length, thickness, width)	S-2	4,0
1000 water leak	G-1	2,5
Force and elongation at break (before and after aging)	S-2	4,0

Table 5. External appearance

Design style	External appearance	
CONNECT Basic		
CONNECT Strong		

CONNECT
Grip

CONNECT
Flex

CONNECT
Poly-M

CONNECT
Essential

CONNECT
Vital

CONNECT
X-Blanc

7 Directions for use

7.1 Application rules

- The gloves are intended for use as personal hand protective device.
- Do not use damaged gloves.
- Do not use gloves of improper size.
- Single use only.
- Do not use gloves after the expiry date.
- The gloves do not guarantee protection from punctures and cuts.

7.2 Directions

- Sanitize hands.
- Make sure the gloves are intact.
- Carefully put on the right and left gloves so that the gloves envelope the hands.
- Take off the gloves after use and put them into a special container suitable for transportation of used disposable medical devices.

7.3 Warning

- The gloves are made of natural rubber latex which can cause allergic response.

8 Complete set

Delivery set:

– dispenser – 1 piece;

Abridged directions for use are printed on the dispenser.

9 Packing

100 gloves by weight per dispenser (consumer package) – for design styles: CONNECT Basic, CONNECT Grip, CONNECT Flex, CONNECT Poly-M, CONNECT Essential, CONNECT Vital.

50 gloves by weight per dispenser (consumer package) – for design styles: CONNECT Strong, CONNECT X-Blanc.

10 dispensers per outer carton (transportation package).

Outer carton secures safe transportation and storage.

10 Marking

10.1 Dispenser marking is as follows:

- name and address of the manufacturer in Russian and English;
- name and address of the manufacturer's representative in the Russian Federation;
- gloves name;
- «Examination gloves»;
- «Material: natural rubber latex»;
- «Disposable»;

- «Non-sterile»;
- «Ambidextrous»;
- «Liquid-tight»;
- «Solid»;
- «Powder-free»;
- «Beaded cuff»;
- «Textured at finger tips» (for design style CONNECT Basic);
- «Textured at fingers» (for design styles CONNECT Strong, CONNECT Essential, CONNECT Vital, CONNECT X-Blanc);
- «Textured all over» (for design styles CONNECT Grip, CONNECT Flex, CONNECT Poly-M);
- «Extra strong» (for design styles CONNECT Strong, CONNECT X-Blanc);
- «Extended cuff» (for design styles CONNECT Strong, CONNECT X-Blanc);
- size;
- colour;
- quantity of gloves (by weight) per dispenser;
- lot number;
- reorder number;
- date of manufacture;
- shelf life;
- registration certificate number;
- «The product is made of natural rubber latex which can cause allergic response»;
- application rules/warning;
- directions for use;
- «Made in Malaysia»;
- «GOST R 52239-2004»;
- symbols:

– Reorder number

– Lot number

– Date of manufacture

– Single use only

– Contains natural rubber latex

– Non-sterile

– Do not use if the package is damaged

-  – Manufacturer
-  – Avoid sunlight
-  – Avoid moisture
-  – Temperature limits (from 0 through +25 °C)
-  – Read user manual
-  – Warning! Read user manual

10.2 *Outer carton marking is made in Russian and English as follows:*

- name and address of the manufacturer;
- name and address of the manufacturer's representative in the Russian Federation;
- gloves name;
- size;
- quantity of gloves per dispenser;
- quantity of dispensers per outer carton;
- quantity of gloves per outer carton;
- lot number;
- reorder number;
- date of manufacture;
- shelf life;
- Carton number;
- registration certificate number;
- warning «Store in cool dry place. Avoid ozon and direct sunlight»;
- «Made in Malaysia»;
- warning «This product is to be store in carton during transportation»;
- symbols:

-  – – Lot number
-  – – Date of manufacture
-  – – Single use only
-  – – Contains natural rubber latex

– – Non-sterile

– – Manufacturer

– – Avoid sunlight

– – Avoid moisture

– – Top

– – Medical device. Handle with care

11 In-service conditions

Do not use gloves after expiry date.

Single use only.

Temperature – from +5°C through +35°C, relative humidity – from 35% through 60% (non-condensing) and atmospheric pressure – 500–1060 hPa.

After transportation at subzero temperatures gloves can be unpacked and used only after having been kept in the transportation package at the storage temperature for a period of 24 hours.

12 Transportation

Transportation of gloves packed into transportation package is effected by any means of roofed transport subject to all warnings indicated on the transportation package and haulage rules for the means of transport used.

After transportation at subzero temperatures gloves can be unpacked and used only after having been kept in the transportation package at the storage temperature for a period of 24 hours.

13 Storage

Gloves are to be stored packed in original manufacturer's package in cool and well-ventilated premises. Temperature – from 0°C through +25°C, relative humidity – $\leq 85\%$ (non-condensing) and atmospheric pressure – 500–1060 hPa, at a distance of not less than one meter from heat-radiating appliances.

While storing the gloves must be protected against exposure to direct sunlight, fluorescent lighting, atmospheric agents, ozone, greases, petroleum, paraffin and other substances destroying latex; the gloves should not be in direct contact with metals such as copper, manganese and iron.

14 Service and maintenance

The gloves are not subject to service and maintenance.

Re-use is prohibited.

15 Ecological safety

Gloves manufacturing is not to expose environment to any toxic discharges and have irritating and sensitizing effects on humans.

There is no environmental and human exposure while using, transporting and storing the gloves.

16 Disposal procedure

The gloves are to be disposed of in accordance with Sanitary Regulations and Rules 2.1.7.2790-10 for class B waste (epidemiologically hazardous waste). After disinfection with special physical methods equipment and external appearance change ensuring impossibility of reuse of the device, class B waste can be stocked up, temporarily stored, transported, destroyed and buried together with class A waste (epidemiologically non-hazardous waste, close in composition to household waste). Package of disinfected class B waste must bear marking showing the waste is disinfected.

17 Warranty

Shelf life:

3 years from the date of manufacture for design styles: CONNECT Basic, CONNECT Grip, CONNECT Flex, CONNECT Poly-M, CONNECT Essential, CONNECT Vital, CONNECT X-Blanc.

5 years from the date of manufacture– for design style: CONNECT Strong.

The manufacturer guarantees the gloves sterility and full compliance with the specified requirements during the shelf life subject to consumer following application, storage and transportation rules.

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Manufacturer authorized representative in the Russian Federation:

RTL-med, LLC

Russia, 117198, Moscow, Leninsky prospect, 113/1,

Tel.: +7 (495) 783-08-88, e-mail: general@rtl-med.com.

Manufacturer:

Advanced Healthcare Products SDN BHD

Lot 60 & 61, Lorong Senawang 3/2, Senawang Industrial Estate, 70450 Seremban, Negeri Sembilan Darul Khusus, Malaysia

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