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Email: rsood1963@gmail.com


www.innovativegloves.net

OPERATIONAL DOCUMENTATION FOR RUSSIA

TurboMAX ARCHDALE Gloves Nitrile Examination Non-sterile Powder free Textured Polymer coated
production

Innovative Gloves Co., Ltd

Approve:
Innovative Gloves Co., Ltd.
President & CEO
Rajeev Kumar Sood

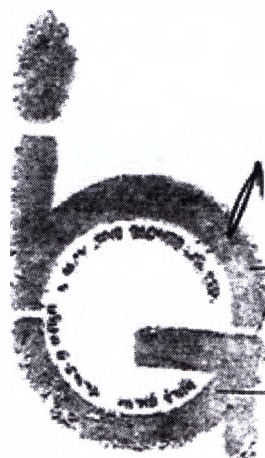Signature

Date
05.06.2017

Seal

No. 247493 9 JUN 2017
BKK100

Certified Genuine Signature(s) of

RAJEEV KUMAR SOOD

(MRS.SIRIMA SUTTI)

Finance and Accounting Officer
Experienced Level

Ministry of Foreign Affairs of Thailand

2129120

Section	Table of Content	Page
1	Product Information	3
1.1	Name and designation of MD	3
1.2	Manufacturer of medical device	3
2	Description and Operation	3
2.1	Brief information about the medical device, including the designation	3
2.2	Guaranteed technical parameters and characteristics by manufacturer	4
2.3	Composition (completeness of delivery) of medical device	5
2.4	Warnings, precautions and limitations when using MD	5
2.5	Description of packaging, labelling and warning labels on the device	5
3	Use of MD	6
3.1	Installation and commissioning	6
3.2	Operating procedure (intended use of product)	6
4	Maintenance and repair	7
5	Disposal	7
6	Guarantees	7
7	Indications and contraindications, side effects.	7

1. Product Information

1.1. Name and designation of MD

ARCHDALE Nitrile Examination Gloves Non-sterile Powder free Textured Polymer coated sizes XS, S, M, L, XL.

1.2. Manufacturer of medical device

Information about the manufacturer:

Name: Innovative Gloves Co., Ltd.

Address: 830 Moo 4, Sanambin-Ban Klang Road, Kuanlang, Hat Yai, Songkhla, Thailand

Telephone: (6674)502251

E-mail: Rsood1963@gmail.com

Web-site: www.innovativegloves.net

Information about the authorized representative of the manufacturer in the Russian Federation:

Ardeyl-Impex Limited Liability Company (Ardeyl-Impex LLC)

23 Kashyrskiy proezd, bld.5, office 3, Moscow, 115201

tel.: 8 495 777 1170, 8 495 777 1171, e-mail: abx@ardl.ru, INN 7724781425

2. Description and Operation

2.1. Brief information about the medical device, including the designation

The product is a non-sterile glove made of nitrile latex, intended for single use in medical research, diagnostic and therapeutic procedures to protect the patient and the researcher from mutual contamination.

Gloves have a polymeric coating inside to facilitate the process of putting on, including wet hands.

The working surface of gloves has a distinct texture that provides a secure grip and tool retention and prevents it from slipping in the hands while working in conditions of high humidity.

Classification of produced gloves:

- 1) depending on the material - nitrile rubber;
- 2) depending on the finishing -textured on the entire surface;
- 3) depending on the finishing - powder free;
- 4) depending on the cuff edge - with a rim on the edge;
- 5) depending on the colour - white due to a white pigment;
- 6) depending on the size: XS, S, M, L, XL.
- 7) depending on the shape - non-anatomic;


The inside surface of gloves is coated with a polymer coating.

2.2. Guaranteed technical parameters and characteristics by manufacturer Materials used for manufacture


Name of ingredient	Chemical Name	Purpose	%
Nitrile rubber	Akrilbutadienstirol	Raw material	97.92
Ammonia	Hydrogen nitride	Stabilizer	0.37
Zinc oxide	Zinc oxide	Activator	0.89
ZDEC	Zinc diethyldithiocarbamate	Accelerator	0.28
Sulfur	Sulfur	Cross-linking agent	0.54
Foamviper 7910	Oxyethylated nonylphenol	Antifoam agent	0.05
Pigment	Titanium dioxide	White Pigment	0.5-0.8
Acrylic	Acrylic polymer coating FLEKSIKOAT 99	Reduction of friction	1

Main Specifications

Points for measuring the width and length of gloves are shown in the figure below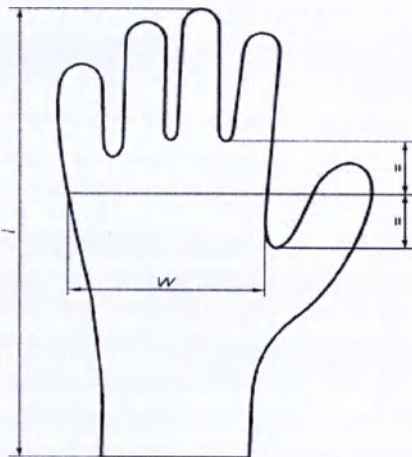

Points for measuring the wall thickness of gloves are shown in the figure below

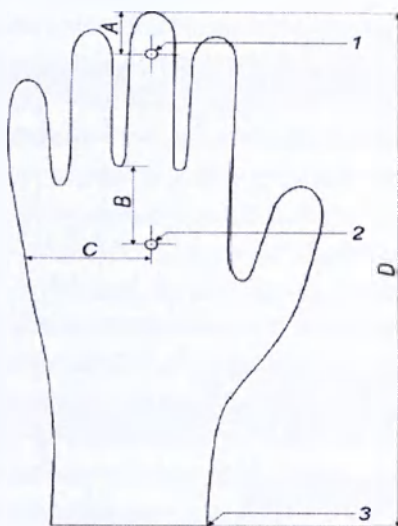1 – fingertip; 2 – palm; 3 – cuff edge;
 A = (13±3) mm; B = (33±5) mm; C = (48±9) mm.

The wall thickness of gloves at the cuff level is measured at a distance of 25 ± 5 from the cuff edge

Physical characteristics of gloves

Size	Short size name	Length (mm)	Thickness, one layer (mm)			Width (mm)
			At the level of fingertips	At the level of palm	At the level of cuff	
X-Small	XS	230±10	0.11±0.01	0.08±0.01	0.06±0.01	75±5
Small	S					85±5
Medium	M					95±5
Large	L					105±5
X-Large	XL					115±5

Weight of one glove is not more than 5 g (for all sizes)

Physical properties of gloves

Before accelerated aging		After accelerated aging	
Tensile strength, MPa, not less than	Elongation at break, N, not less than	Tensile strength, MPa, not less than	Elongation at break, N, not less than
22	700	22	550

Powder free gloves, a residual amount of powder agent < 2.0 mcg per glove in accordance with ASTM D 6124

Gloves are hermetic, the level of sealing quality (AQL) is not more than 1.5

Gloves with a circular section rim at the edge of cuff, the diameter of the rim is 1.8-2.2 mm

2.3. Composition (completeness of delivery) of medical device

One group (transport package) package contains ten consumer packages and one exploitative document.

2.4. Warnings, precautions and limitations when using MD Precautionary measures

- 1) Hand skin should be well-groomed, with no wounds, cracks and other damage.
- 2) Nails should be short and neatly trimmed. Do not use artificial nails.
- 3) Jewellery and watches should be removed when using gloves.
- 4) Gloves are designed exclusively for single use in medical research, diagnostic and therapeutic procedures to protect the patient and the researcher from mutual contamination.

2.5. Description of packaging, labelling and warning labels on the device Packaging

Consumer carton box, 100 (50 pairs) pieces of gloves in one carton.

The weight of the package with gloves (100 pcs.) Is not more than 700 g (regardless of the version).

The box allows you to store the gloves quality during transportation and storage.

Labelling

The labelling is reliable, verifiable and readable. The labelling is applied to the consumer package by any means (printing, embossing, stamping) to ensure its clarity, accuracy and readability. When using the printing method of marking, the scuffing is not allowed. The marking on the consumer package of gloves contains:

- name and location of the manufacturer;
- name and address of the representative in the country(s) of product distribution;
- inscription: "Does not contain natural rubber latex" or equivalent, and/or picture, clearly reflecting this indication;
- inscriptions: "ARCHDALE" and "TurboMAX";
- inscriptions describing the short and full name of the product: "NITRILE EXAMINATION GLOVES" and "NITRILE EXAMINATION NON-STERILE POWDER FREE TEXTURED POLYMER COATED GLOVES";
- inscription: "NON-STERILE" and/or picture, clearly reflecting this indication;
- inscription: "FOR SINGLE USE" and/or picture, clearly reflecting this indication;
- inscription: "EXAMINATION " and/or picture, clearly reflecting this indication;
- inscription: "NON-ANATOMIC SHAPE" and/or picture, clearly reflecting this indication;
- inscription: "TEXTURED" and/or picture, clearly reflecting this indication;
- inscription: "POWDER FREE" and/or picture, clearly reflecting this indication;
- inscription: "POLYMER COATED" and/or picture, clearly reflecting this indication;
- batch No.;
- date (month, year) of manufacture;
- shelf life;
- size and the number of pairs of gloves;
- information about the storage and transport conditions.

It is considered acceptable to supplement the marking other information, for example to use symbols and drawings, clarifying the consumer properties of gloves and their application. It is not allowed to put the information about the special properties of gloves without the appropriate documentary evidence.

The marking is made in the language of the country of product distribution, excluding proper name and other names.

3. Use of MD

3.1. Installation and commissioning

Open the group gloves package, take one pair of gloves and consistently wear gloves on the right, then on the left hand.

3.2. Operating procedure (intended use of product)

Consistently on the left and right gloves to turn back the edge so as the glove completely covers the coat collar.

Taking off gloves

To take the glove in the heel of the hand with fingers of the other hand.

To draw off the glove and hold it with the other hand.

Put fingers of a free hand in glove cuff on the other hand and draw off similarly.

Having turned out glove inside out, use it as a cover for other gloves.

To put gloves in the container gloves intended for them.

After removing the gloves wash your hands with soap and water and treat them with an alcohol-containing skin antiseptic.

You should change gloves:

- every two hours of continuous use;
- after discovery of visible defect;
- after a possible or really occurred puncture;
- if the glove starts to swell, stretch, stick or is not tight on the hand;
- in accordance with the rules adopted in the place of use of the these gloves.

Do not use the same pair of gloves changing one patient to another or one manipulation to another.

4. Maintenance and repair

Gloves refer to single use products and are not subject to maintenance.

Gloves are operated at a temperature (from +10 to +35) °C and humidity (80-100)%.

Store in the original packaging in a dry and well ventilated place, excluding the presence of dust, exposure to sunlight, moisture, X-rays at a temperature not lower than 50 °C and not above 40 °C and humidity (60-75%).

Protect from the effects of oils, gasoline and other substances that destroy rubber.

The product is transported by all types of covered vehicles in

In accordance with the rules of transport operating on this mode of transport at a temperature not lower than - 50 °C and not higher than 50 °C and humidity (60-75)%.

After transportation at minus temperature, unpack, soak for 24 hours at room temperature.

5. Disposal

Utilization and disposal of used products is carried out according to the requirements of SanPiN 2.1.7.2790-10, gloves used as intended refer to wastes of category B (epidemiologically hazardous wastes). Collection of used gloves should be carried out in marked (labeled) yellow bags.

6. Guarantees

Gloves refer to medical devices for single use.

The shelf life of gloves is 5 years from date of manufacture.

Guaranteed shelf life of gloves is 5 years.

7. Indications and contraindications, side effects.

Indications for single use during medical research, diagnostic and therapeutic procedures to protect the patient and the researcher from mutual contamination. Contraindications to use:

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Email: rsood1963@gmail.com

www.innovativegloves.net

contact dermatitis may occur during prolonged operation of individuals. Side effects: contact dermatitis (possible with prolonged work for individuals).

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