ROMA CAPITALE

Municipio **I^e**
Servizi Demografici

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INSTRUCTION FOR USE

*All in one solution for washing, cleaning and
storing of soft contact lenses, models: Orion,
Sodyal, UniVis. Version 1.1 May 2017*

INSTRUCTION FOR USE

MEDICAL DEVICE

«All in one solution for washing, cleaning and storing of soft contact lenses

models: Orion, Sodyal, UniVis,

production by «Omisan farmaceutici s.r.l.», Italy

All in one solution for washing, cleaning and storing of soft contact lenses, models: Orion, Sodyal, UniVis. Version 1.1 May 2017

ABOUT THE PRODUCT

The solution is versatile for washing, cleaning and storing soft contact lenses is available in three variants: Orion, Sodyal and UniVis. The product is intended for disinfection, treatment, storage, lubrication, removal of residues of the protein, moisture, wash all types of contact lenses. The product is intended for personal use by the patient at home. The solutions are available in bottles volume: 120 and 360 ml. All packages are equipped with a container for storing contact lenses.

To select one of the options of solution, the following information will help:

1. Orion solution has been designed for multi-functional care MCL. This dis-solution cleanses, destroys and removes accumulation of tear film, protein deposits, other deposits and debris from the contact lens surface. The solution destroys harmful microorganisms to the lens surface, and is also used for rinsing and storing lenses.
2. The solution Sodyal as the Orion, cleans, disinfects and removes protein deposits and is used for storage of lenses. But other than that, it is enriched with hyaluronic acid, and is designed for those who have dry eyes. It improves the comfort of wearing contact lenses for its high moisturizing properties.
3. The solution is further enriched UniVis hyaluronic acid and aloe vera, combining moisturizing properties of Hyaluronic Acid and soothing / calming properties of Aloe Vera, suitable for patients with dry eyes and irritation.

The compositions of solutions are given below.

Orion

RAW MATERIALS	%
Purified water	Up to 100
Boric acid	0,684 – 0,756
Sodium tetraborate	0,109 – 0,121
Sodium chloride	0,475 – 0,525
Disodium salt	0,104 – 0,116
PHMB	0,0017 – 0,0019

Sodyal

RAW MATERIALS	%
Purified water	Up to 100
Boric acid	0,684 – 0,756
Sodium tetraborate	0,109 – 0,121
Sodium chloride	0,475 – 0,525
Disodium salt	0,104 – 0,116
PHMB	0,0017 – 0,0019
Sodium hyaluronate	0,0048 – 0,0053
Hypromellose	0,0665 – 0,0735
Pluronic F-127	0,237 – 0,263
PVP K90	0,071 – 0,079

INSTRUCTION FOR USE

All in one solution for washing, cleaning and storing of soft contact lenses, models: Orion, Sodyal, UniVis. Version 1.1 May 2017

UniVis

RAW MATERIALS	%
Purified water	Up to 100
Boric acid	0,684 – 0,756
Sodium tetraborate	0,109 – 0,121
Sodium chloride	0,475 – 0,525
Disodium salt	0,104 – 0,116
PHMB	0,0017 – 0,0019
Sodium hyaluronate	0,0048 – 0,0053
Aloe vera gel 1: 1	0,095 – 0,105
Hypromellose	0,0665 – 0,0735
Pluronic F-127	0,38 – 0,42
PVP K90	0,0071 – 0,0079

Specifications

Chemical and physical parameters

Parameter	Standard value
pH	7,20 ± 0,15
osmolarity	300 ± 20 mOsm /

Packaging options

Parameter	120 ml	360 ml
The volume of solution	120 ml±3%	360 ml±3%
Weight of the bottle with the solution	131,5±3,6 g	388,5±10,8 g
Bottle weight	11,5±1,2 g	28,5±2,8 g
Dropper weight	1,2±0,12 g	1,2±0,12 g
Cover weight	2,0±0,2 g	2,0±0,2 g
Bottle sizes	13±0,3 cm 14,3±0,4 cm 4,27±0,2 cm	15,5±0,3 cm 17±0,4 cm 6,47±0,3 cm
Height without closure	2,5±0,1 cm 1,2±0,1 cm	2,5±0,1 cm 1,2±0,1 cm
Cap dimensions		
Height	3±0,2 cm	3±0,2 cm
Diameter	2,2±0,2 cm	2,2±0,2 cm

INSTRUCTION FOR USE

All in one solution for washing, cleaning and storing of soft contact lenses, models: Orion, Sodyal, UniVis. Version 1.1 May 2017

Container for contact lenses

Purpose: intended to storage contact lenses when they are not used by the user. Storage should only be carried out in a container filled with a solution for storing contact lenses.

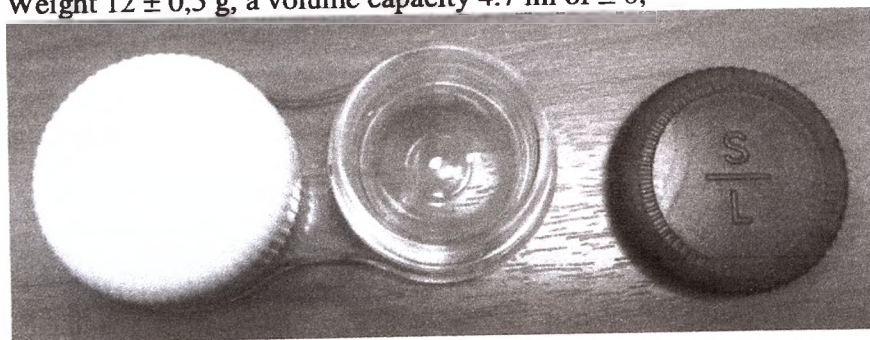
Application: For storage of contact lenses in a container fill it with a solution intended for disinfection, cleaning, storage, lubrication, removal of protein residues, moisturizing and washing contact lenses, after which a contact lens is placed in it. For storage of contact lenses, both one and two compartments of the container can be used. After removing contact lenses at the end of the storage period, the used solution must be disposed (pour out) immediately.

Marking: on the cover of the container the letters "L" and "R" or "S / L" and "D / R" are pressed in marking the left and right eyes respectively. Marking is intended in order that the user didn't forget which lens is placed in which container what is important when using lenses with different characteristics.

Disinfection: the solution used has disinfectant and cleaning properties, therefore the container does not require separate disinfection. After handling the container, it must be rinsed under a stream of clean water.

Disposal: Based on the morphological composition, the container does not have a negative impact on the environment and can be disposed of as solid household waste (MSW).

A container for storing contact lenses great for packing 360 ml.
Weight $12 \pm 0,5$ g, a volume capacity 4.7 ml of ± 0 ,

A container for storing contact lenses for small packaging of 120 ml.
Weight 7.5 ± 0.5 g, a volume capacity 2.30 ± 3 ml

INSTRUCTION FOR USE

All in one solution for washing, cleaning and storing of soft contact lenses, models: Orion, Sodyal, UniVis. Version 1.1 May 2017

Application solutions is similar in the future in the text is done once-ferences between them.

INDICATIONS:

- Disinfection;
 - Cleaning;
 - Storage;
 - Lubrication;
 - Remove residues of the protein;
 - Hydration; *
 - Rinsing;
- all types of contact lenses.

CONTRAINDICATIONS FOR USE

Allergic reaction to the components of the solution.

POSSIBLE ADVERSE EFFECTS:

Possible allergic reaction to separate components of solutions.

Feeling of discomfort.

Increased lacrimation.

Hyperemia of the eyes.

The above-mentioned possible adverse effects may also be associated with the use of contact lenses.

DIRECTIONS FOR USE

Wash your hands thoroughly.

Place the contact lens in the lens container, pour the solution and then close tightly. For complete disinfecting contact lenses desirable to leave at least 4 hour. Gently shake the container with lenses before using.

Clean contact lenses before use.

PRECAUTIONS

When cleaning and storing the lens, never use the solution several times; always take only a fresh portion.

After immersion of the lens in the container with the solution should be stored in this state no more than one month from the date of immersion.

Close the vial tightly after each use.

Keep the vial away from heat sources.

Do not swallow.

Do not touch the surface with a dropper to avoid contamination.

Keep away from children.

Sterile before opening.

Use within 90 days of first opening.

Do not use after expiry date.

Store at a temperature of +5 °C - +35 °C.

SYMBOLS

On the packaging contains information about the solution, manufacturer, expiration date, as well as informing and warning symbols.

	Manufacturer
	Lot number
	Temperature limitations
	Sterilization using aseptic processing techniques
	Use by (year, month)
	Refer to the instruction manual
	Sign of conformity to the requirements of the European Directive

GUARANTEES

The manufacturer guarantees the quality and safety of this product for 3 years from the date of issue, subject to consumer storage and handling conditions, the established by the manufacturer. The solution should be used within 90 days from the date of opening the bottle.

TRANSPORTATION AND STORAGE CONDITIONS

The solution can be transported in the original packaging by all types of covered ground transport at temperature of + 5°C - + 35°C and relative humidity up to 85%. Keep in closed heated warehouses under the same conditions.

RECYCLING

The product does not cause a direct impact on the environment and with the morphological shutter characterized as waste (Class A).

The remaining solution is used or poured down the drain.

The bottle and the container disposed of according to local regulations as a solid waste.

MANUFACTURER

Omisan farmaceutici s.r.l.

Via Galileo Galilei SNC-00012 Guidonia Montecelio (RM), Italy.

Tel.: 3964130370. Fax: 3964190550.

E-mail: omisan@omisan.com

AUTHORISED REPRESENTATIVE

Limited Liability Company «Vizoteque» (Vizoteque LLC)

Legal address: 117303, Russia, Moscow, ul. Odesskaya, 18, building 4, ap. 14.

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