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DIRECTIONS FOR USE

1. Open package and remove **Combat Gauze**. Keep the empty package.

2. Pack **Combat Gauze** into wound and use it to apply pressure directly over bleeding source. (More than one **Combat Gauze** may be required).

3. Continue to apply pressure for 3 minutes or until bleeding stops.

4. Wrap and tie bandage to maintain pressure. Seek medical care immediately. Show **PRODUCT REMOVAL** directions on package to medical personnel.

PRODUCT REMOVAL: 1. Gently remove gauze from wound. 2. Thoroughly irrigate wound.

Mixed by:
Za, LLC
41d Boulevard
Yord, CT 06492 USA
55.502.7930
03.294.0000
00.343.8656
203.303.7216
QuikClot.com

Do Not Reuse

WARNINGS: For external use only. Sterility not guaranteed if package is damaged or opened. Do not use if package is damaged. Avoid contact with eyes.

STERILE R

FOR USE BY TRAINED EMERGENCY PROVIDERS

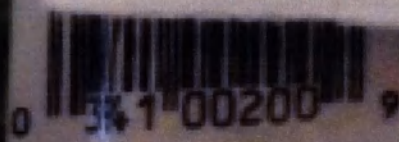
8 39241 0020 5

R200

6997

2021-06

FOR USE BY TRAINED EMERGENCY RESPONDERS

Manufactured by:
Z-Med Corporation
4 Fairview Rd.
Waltham, MA 01907
USA

Tel: +1-203-294-0000
Fax: +1-203-294-0668
www.z-medica.com
Z-Medica is a Z-Medica Corp.
Med. Device

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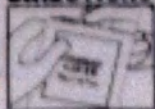
WARNINGS: For external use only. Sterility not guaranteed if package is damaged or opened. Discard package if damaged. Single use only. Do not resterilize. Avoid contact with eyes.

DIRECTIONS FOR USE

1. Open package and remove TraumaPad[®]. Keep the empty package.

2. Place TraumaPad into wound and use it to apply pressure directly over bleeding source. (More than one TraumaPad may be required.)

3. Continue to apply manual pressure for 3 minutes or until the bleeding stops.

4. Wrap and tie bandage to maintain pressure. Seek medical care immediately. Show product removal directions to medical personnel.

PRODUCT REMOVAL

1. Gently remove gauze from wound.
2. Thoroughly irrigate wound.

101413-01 Rev B

STERILE

Manufactured by:
Z-Medica, LLC
4 Fairfield Boulevard
Wallingford, CT 06492
USA

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FOR USE BY TRAINED EMERGENCY RESPONDER
DIRECTIONS FOR USE

1. Open package and remove Combat Gauze XL. Keep the empty package.

2. Pack Combat Gauze into wound and use it to apply pressure directly on bleeding source. (More than one Combat Gauze XL may be required).

3. Continue to apply pressure for 3 minutes or until bleeding stops.

4. Wrap and tie bandage maintain pressure. Seek medical care immediately. Show **PRODUCT REMOVAL** directions on package to medical personnel.

PRODUCT REMOVAL: 1. Gently remove gauze from wound 2. Thoroughly irrigate wound

Manufactured by:
Z Medica, LLC
 4 Fairfield Boulevard
 Wallingford, CT 06492
 USA
 Tel: +1-203-284-0800
 Fax: +1-203-284-9970
 • www.z-medica.com

• Patent 7804010, 8114433
 • 32, 8343837, 8383148

© Z Medica, LLC
 All rights reserved

STERILE

WARNING: For external use only.
 Sterility not guaranteed if package damaged or opened. Discard pack if damaged. Avoid contact with eyes.
 Do not re-sterilize.

DIRECTIONS FOR USE

1. Open package and remove **Combat Gauze LE**. Keep the empty package.

2. Pack **Combat Gauze LE** into wound and use it to apply pressure directly over bleeding source. More than one **Combat Gauze LE** may be required.

3. Continue to apply pressure for 3 minutes or until bleeding stops.

4. Wrap and tie bandage to maintain pressure. *Seek medical care immediately.* Show **PRODUCT REMOVAL** directions on package to medical personnel.

PRODUCT REMOVAL: 1. Gently remove gauze from wound. 2. Thoroughly irrigate wound.

Manufactured by:

Z-Medica, LLC

4 Fairfield Boulevard

Wallingford, CT 06492 USA

Tel: 1.877.750.0504, +1.203.294.0000

Fax: 1.800.343.8656, +1.203.303.7216

quikclot.com

Patents: z-medica.com/patents.aspx

© 2016 Z-Medica, LLC

All rights reserved.

100885-01 Rev E

Made in the USA

REF 350

LOT 7258

2022-02

Do Not Reuse

STERILE R

WARNINGS: For external use only. Sterility not guaranteed if package is damaged or opened. Discard package if damaged. Avoid contact with eyes. Reuse will cause risk of infection and loss of efficacy.

FOR USE BY TRAINED EMERGENCY RESPONDERS

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**HEAT
FREE**

QuikClot®
EMS
4x4 Dressing

One Sterile 4-Ply 4 in. x 4 in. (10 cm x 10 cm) Hemostatic Dressing
For Temporary External Use To Control Traumatic Bleeding

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TEAR HERE ▶

DIRECTIONS FOR USE

1. Open package and remove Rolled Gauze. Keep the empty package.

2. Pack Rolled Gauze into wound and use it to apply pressure directly over bleeding source. (More than one Rolled Gauze may be required)

3. Continue to apply pressure for 3 minutes or until bleeding stops.

4. Wrap and tie bandage to maintain pressure. Seek medical care immediately. Show **PRODUCT REMOVAL** directions on package to medical personnel.

PRODUCT REMOVAL: 1. Gently remove gauze from wound. 2. Thoroughly irrigate wound.

0 32941 00475 5

Do Not Reuse

WARNINGS: For external use only. Sterility not guaranteed if package is damaged or opened. Discard package if damaged. Avoid contact with eyes.

STERILE R

FOR USE BY TRAINED EMERGENCY RESPONDERS

Manufactured by:
Z-Medica, LLC
4 Fairfield Boulevard
Wallingford, CT 06492 USA
Tel: 1.855.502.7930, +1.203.294.0000
Fax: 1.800.343.8656, +1.203.303.7216
www.z-medica.com

U.S. Patents
7604819, 8114-33, 8257732, 8343537
© 2013 Z-Medica, LLC
All Rights Reserved
Made in USA

REORDER

101311-01 Rev A

LOT

6586

2018-10

QuikClot® EMS Rolled Gauze™

REF 475

◀ TEAR HERE

TEAR HERE ▶

HEAT
FREE

QuikClot®
EMS
Rolled Gauze™

**For Temporary External Use
To Control Traumatic Bleeding**
ONE STERILE 3 IN X 4 FT (7.5 CM X 1.2 M) HEMOSTATIC DRESSING

QUICKCLOT
75

2 x 2

Individually Packaged 2 in x 2 in
X-Ray Detectable Hemostatic Dressings

**X-Ray
Detectable**

Hx ONLY **STERILE**

QuikClot Inc.
1000 N. 10th St.
Bloomington, IL 61710 USA
Tel: 815 236 4000
Fax: 815 236 4001
www.quikclot.com
QuikClot is a registered trademark of QuikClot Inc.
© 2000 QuikClot Inc. All rights reserved.

QUICKCLOT

QuikClot
HEMOSTATIC DRESSING

REORDER
458

LOT

6711 2018-10

QuikClot®

HEMOSTATIC DRESSING

2 x 2

Ten (10) Individually Packaged 2 in x 2 in
Sterile X-Ray Detectable Hemostatic Dressings

Manufactured by
Z-Medica, LLC
4 Fairfield Boulevard
Wallingford, CT 06497 USA
Tel: 1 877 750 0904
Fax: 1 800 343 8696

www.z-medica.com
U.S. Patents 7524816, 8114415,
8257733, 8343537, 8203148
© 2013 Z-Medica, LLC
All Rights Reserved

**X-Ray
Detectable**

Rx ONLY **STERILE**

Z-MEDICA®

QuikClot

HEMOSTATIC DRESSING

One (1) 4-Ply 4 in x 4 in (10 cm x 10 cm) Dressing

STERILE

Rx ONLY

Do Not
Reuse

Read
Instructions

X-Ray
Detectable

CE
0086

EC REP

Emergc Europe
Molenstraat 15
2513 BH
The Hague
The Netherlands

Manufactured by:

Z-Medica, LLC

4 Fairfield Boulevard

Wallingford, CT 06492 USA

Tel: 1-877-750-0508, +1-203-294-0000

Fax: 1-800-343-8858, +1-203-301-1111

www.QuikClot.com

© 2015 Z-Medica, LLC

All rights reserved.

101621-01 Rev A

REF

459

Manufactured by:
Z-Medica, LLC
1000 Field Boulevard
Bristol, CT 06492 USA
Tel: 1.877.750.0506, 1.203.294.0000
Fax: 1.800.343.8656, 1.203.303.7216
www.z-medica.com

8343537, 8343538
© 2013 Z-Medica, LLC
All Rights Reserved
Made in USA

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