+49 (0)8152 700-0
+49 (0)8152 700-1366
Internet: www.3MESPE.com
E-Mail: info3MESPE@3M.com
WEEE-Reg.-Nr. DE 36963167
VAT-ID: DE 120679179

To Whom It May Concern

**Registration documents for RelyX Fiber Post
- Information for Use (IFU) -**

We, 3M Deutschland GmbH, Carl-Schurz-Str. 1, 41453 Neuss, herewith confirm that the following pages are an excerpt of the Information for Use.

Seefeld, April 10, 2017

Yours Sincerely,

3M Deutschland GmbH
3M Oral Care

Elfriede Zedler
Regulatory Affairs

Product Description

RelyX Fiber Post posts are radiopaque, translucent, glass fiber reinforced composite posts available in the four sizes* 0, 1, 2, and 3. Specific drills are used for removal of the root filling and preparation of the root canal; see the table below.

Combination post/drill				
Post size/Color code	0/White	1/Yellow	2/Red	3/Blue
Removal of the root filling	Universal drill	Universal drill	Universal drill	Universal drill
Preparation of the root canal	Drill White	Drill Yellow	Drill Red	Drill Blue
Technical data for the posts				
Diameter of apical post end (mm)	0.60	0.70	0.80	0.90
Diameter of coronal post end without retention (mm)	1.10	1.30	1.60	1.90
Taper	2.86° (5%)	3.44° (6%)	4.58° (8%)	5.72° (10%)
Length	20 mm	20 mm	20 mm	20 mm

* Not all sizes are available in all countries.

Use a dual-cure composite cement, e.g., RelyX U200, for the cementation of the posts.

The indicated 3M ESPE composite, e.g., Filtek Bulk Fill Posterior composite, is used as core build-up material. For processing details, please see the corresponding Instructions for Use of the cement and the core build-up material.

☞ These Instructions for Use must be kept for reference for the duration of product use.

Indications

- In case of insufficient residual tooth structure (<4 mm) the post is needed to support and secure the coronal restoration.

RelyX Fiber Post posts are not suitable for patients with bruxism or in maxillary anteriors with excessive vertical overlap or if the circular coronal tooth structure is less than 1.5 mm.

Precautionary Measures

3M MSDSs can be obtained from www.mmm.com, or contact your local subsidiary.

Precautions During Processing

- Selection of the correct drill and post size as well as their proper use are the responsibility of the user. The universal drill may be used solely to remove a root filling, and the color-coded drills may be used solely to prepare root canals. Do not continue to use drills which are dull or damaged. Replace the drills after 15 applications.
- If there has been a greater degree of destruction of the tooth, a dentin border 2 mm in width must be prepared apically of the core build-up which is covered by the definitive crown.
- Do not use any adhesives or composite cements which are light-cured only to cement the posts.

Preparatory Measures

- Treat the root canal endodontically as usual (root canal filling with guttapercha).
- Select the post size and the matching drill as indicated by the X-rays and according to the diameter of the prepared root canal.
 - To guarantee a precise fit, always use the drills and posts from a single system and do not combine them with products from other manufacturers (see color code).
- Determine the drilling depth with the help of the X-ray and according to the length of the prepared canal. 4 mm of the root filling must remain apically.
 - 2/3 of the post should be located in the root canal and 1/3 in the core build-up. This length ratio cannot always be achieved if the canal is crooked.
- Prepare the drills which will be used before every application (see "Cleaning and Disinfection").
- Have a self-adhesive or adhesive resin cement, e.g., RelyX U200 with an attached Endo Tip elongation tip, at hand for cementation of the post.
 - When other composite cements are used for the adhesive cementation of RelyX Fiber Post additional preparatory steps or other procedures are required. Please follow the Instructions for Use from the manufacturer in every case.
- We recommend using a cofferdam.

Prepare the Root Canal and the Post for Cementing

- As is standard when preparing the root canal for the insertion of a glass fiber post, begin by removing the guttapercha filling with a Gates-Glidden, Peeso, or Largo burr.
- Use the universal burr to remove any residual root filling (1000-2000 rpm), leaving 4 mm of the root filling apically.
- Use the appropriate drill for the post to widen the root canal (1000-2000 rpm); see table under Product Description. Avoid any expansion beyond the apex or the perforation of the root.
- Check the fitting of the post in the prepared canal; it should fit the root canal exactly, there must not be any kind of binding, and it must be easily removable.
- Mark the final length of the post or the point where it is to be cut.
- Shorten the post extraorally to the required length using a diamond disk and suction system; wear a mask.
 - Do not use scissors, scalpels, or similar tools; they will crush the post, ruining its structure and stability.
 - Do not use any post fragments which have been cut off.
- Before inserting the post, clean it with alcohol and dry it with air free of water and oil.
- If you do not use RelyX U200 cement, pre-treat the post surface according to the Instructions for Use from the cement manufacturer.
- As the final step before cementing, clean or disinfect the root canal with a 2.5-5.25% sodium hypochlorite solution (NaOCl).
 - Do not use hydrogen peroxide (H₂O₂) and other disinfectants and EDTA solutions as the residues can impair the adhesion strength and curing reaction of RelyX U200.
- Rinse immediately with water and dry with paper tips.

Cement the Post and Build Up the Core

- Protect the working area from contamination with water, blood, saliva, and sulcal fluid before beginning the cementing and through the setting phase.
- Cement the post as described in the Instructions for Use of the cement, e.g., with RelyX U200 and attached Endo Tip elongation tip or with another dual-cure self-adhesive or adhesive composite cement.
- Pre-treat the remaining post surface and the tooth surface with the bonding which belongs to the core build-up material, e.g., Single Bond Universal Adhesive.
- Use the desired core build-up material (e.g., Filtek Bulk Fill Posterior composite) according to the Instructions for Use to prepare the core buildup in a direct procedure.

After the Treatment

- Carefully check the sulcus of the treated teeth and the surrounding areas; remove any cement residues still remaining.

Revision

- Remove the restoration and the post superstructure.
- Use an X-ray to check the drilling depth and position of the post.
- Drill out the post with a diamond or trepan drill.
 - Insert a rubber dam to prevent the patient from inhaling any glass fiber dust. The person performing the treatment should wear a mask and use water spray and a suction unit.
 - The transition from the glass fiber post to the root dentin is barely notice able during the drilling operation; always proceed with extreme caution to avoid perforation of the root. Take X-rays at each step to check progress.

Cleaning the Posts

- The posts are intended for one-time use; their re-use is prohibited.
- Posts may be cleaned solely with alcohol and water; sterilization is not possible.

Preparing the Drills

General

The RelyX Fiber Post drill is in non-sterile condition when delivered and must be cleaned, disinfected, and sterilized before first use as well as before every re-use. Effective cleaning and disinfection are absolutely essential requirements for effective sterilization.

If possible, use a mechanical process (cleaning and disinfecting device) for cleaning and disinfecting the drills because the effectiveness and reproducibility are significantly lower for manual processes. Pre-treatment is required regardless of which process is used.

Material Resistance

Make sure that the cleaning and disinfectant agents you have chosen do not contain any of the following materials:

- Organic, mineral, and oxidizing acids (minimum acceptable pH value 5.5)
- Strong bases (maximum acceptable pH value 11; alkaline cleaners of about pH 10-11 are recommended)
- Organic solvents (e.g., alcohols, ethers, ketones, benzines)
- Oxidation agents (e.g., hydrogen peroxides)
- Halogens (chlorine, iodine, bromide)

- Aromatic/halogenized hydrocarbons
- Oils

Never use metal brushes or steel wool to clean the RelyX Fiber Post drills.

The drills must not be exposed to temperatures higher than 138 °C (280 °F).

The RelyX Fiber Post drill may be re-used up to 15 times, provided that it has been handled carefully and is undamaged and clean. Users go beyond this threshold or use damaged and/or contaminated RelyX Fiber Post drills solely on their own responsibility.

Pre-Treatment

Immediately after use (within a maximum of 2 hours), remove gross contaminations from the drills.

- Rinse the drills under running water (temperature <35 °C/95 °F) for at least 1 min.
- Use a clean, soft, lint-free cloth to manually remove all visible contamination.
- Rinse again for at least 1 min under running water.

Automatic Cleaning and Disinfection

- The chemicals being used must be compatible with the drills (see "Material Resistance").
- Cleaning agents and/or disinfectants must contain alkaline detergents (pH 10-11).

Follow manufacturer's instructions for concentrations, temperatures, development times, and rinsing off the cleaning and disinfectant agents exactly.

- Immerse the RelyX Fiber Post drills in the cleaning and disinfecting device (basket for small parts). Make sure that there is no contact between the drills.
- Start the cleaning cycle.
- When the cycle has finished, remove the drills from the cleaning and disinfecting device.
- Check and package the drills as quickly as possible after removing them from the device (see "Packaging"; if needed, after further drying in a clean place).

Manual Cleaning and Disinfection

Alternatively, cleaning and disinfecting can be conducted manually. Information about validated procedures can be obtained from 3M Deutschland GmbH.

Sterilization

Steam sterilization is the only sterilization method which may be used for the RelyX Fiber Post drills. Flash sterilization is prohibited. Follow carefully these instructions for the sterilization:

- Maximum sterilization temperature 134 °C (273 °F; tolerances pursuant to DIN EN ISO 17665).
- Sterilization time (exposure time at sterilization temperature) at least 5 min or 18 min (prior deactivation) at 132 °C (270 °F)/134 °C (273 °F).

Check

- Store the drills under dry and dust-free conditions.
- Examine the RelyX Fiber Post drill for damage, discoloration, and contamination before every use and dispose of damaged RelyX Fiber Post drills.
- Any RelyX Fiber Post drills which are still contaminated must be cleaned and disinfected a second time.

Maintenance

Never use instrument oils or greases.

Packaging

Package the RelyX Fiber Post drills in disposable sterilization packaging (single or double packaging). The packaging must meet the following requirements (material/process):

- DIN EN ISO/ANSI AAMI ISO 11607 (for USA: FDA clearance).
- Suitable for use in steam sterilization (temperature stability of at least 142 °C (288 °F), adequate steam permeability).
- Adequate protection for the drills and/or sterilization packaging from physical damage.

Additional information about preparation in accordance with DIN EN ISO 17664 is available from 3M Deutschland GmbH.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

Information valid as of April 2017

RUS: Authorized representative of the manufacturer in RF:

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