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certified copy

Carl-Schurz-Straße 1
41453 Neuss
Germany

Postal Address:
ESPE Platz
82229 Seefeld
Germany

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+49 (0)8152 700-0
+49 (0)8152 700-1366
Internet: www.3MESPE.com
E-Mail: info3MESPE@3M.com
WEEE-Reg.-Nr. DE 36963167
VAT-ID: DE 120679179

To whom it may concern

March 28, 2017

Registration documents for Express XT Putty Soft - IFU

We 3M Deutschland GmbH, Carl-Schurz-Str. 1, 41453 Neuss, herewith confirm that the following pages are an excerpt of the Information for Use.

3M Deutschland GmbH
3M Oral Care

Karin Schlichter
Regulatory Affairs

INSTRUCTION FOR USE

Product Description

Express™ XT impression materials are addition type silicones for one-step and two-step impression technique. **Express™ XT Putty Quick** and **Express™ XT Putty Soft** are kneadable materials for hand mixing. All other materials are available in the Garant cartridge and are statically mixed and directly applied with the **Garant™ Dispenser**.

Not all products are available in all countries.

Product Name	ISO 4823	ISO 4823 Consistency	Paste Color	Mixing Ratio (Vol.)	Mixing Device
Express XT Putty Quick	Type 0	putty	green	1:1	Manually
Express XT Putty Soft	Type 0	putty	ocher	1:1	Manually
Express XT Light Body Quick	Type 3	Light-bodied	pink	1:1	Garant Dispenser
Express XT Light Body	Type 3	Light-bodied	magenta	1:1	Garant Dispenser
Express XT Regular Body Quick	Type 2	Medium-bodied	caramel	1:1	Garant Dispenser
Express XT Regular Body	Type 2	Medium-bodied	petrol	1:1	Garant Dispenser

☞ These Instructions for Use should be kept for the duration of product use. For all additionally mentioned products please refer to the corresponding Instructions for Use

Indications

- All precision impressions (of e.g.: crown, bridge, inlay and onlay preparations).

Precautionary Measures

3M MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Preparation

- Block out undercuts or interdental spaces (as with any other impression material with increased shore hardness) sufficiently to aid impression removal from the teeth after setting,

as otherwise impression removal from the mouth will be difficult or could lead to the extraction of natural teeth or prostheses.

Impression Tray:

All rigid impression trays generally used for precision impressions are suitable.

- For sufficient adhesion, apply VPS Tray Adhesive on the tray and allow it to dry completely, 5-15 minutes drying time are ideal.

Garant Dispenser:

- Insert the cartridge with the wash impression material into the Garant Dispenser.
- Prior to application, check that both Garant cartridge openings are not clogged.
- When using new Garant cartridges, extrude a small amount of paste without the mixing tip until base paste and catalyst are evenly extruded.
- Attach a yellow Garant Mixing Tip to the cartridge.
- For intraoral application attach a yellow Garant Intraoral Tip onto the Garant Mixing Tip.
- If required: enlarge the exit opening for the intraoral tip by cutting it with a scalpel.
- Make sure that base paste and catalyst are mixed completely and are extruded in a uniform color.
 - Paste color: see "Product Description".
 - Material that has set inside the Garant Mixing Tip should not be extruded by force as this may damage the Garant cartridge and Garant Mixing Tip and result in the formation of leaks.

Retraction

- Suitable retraction agents include aluminum hydroxide chloride, aluminum sulfate or ferric sulfate solutions.
- Keep the areas from which the impression is to be taken dry during impression.
- The use of retraction cords may be required for subgingival preparations.
- Prior to the impression, thoroughly remove residual retraction material by rinsing and drying.

Dosage and Mixing of the Putty Material

- After opening a new container, remove and discard the foil.
- Use the pertinent dosage spoon to take equal volumes of the base paste and catalyst paste Express XT Putty Quick or Putty Soft from the container, whereby the dosage spoons must be filled level. The total volume depends on the impression tray size.
- Knead the pastes manually for 30 seconds. The mixed paste must display an even color without striping.
 - Paste color: please see "Product Description".

Times

Product	Working Time at 23°C/74°F from start of mixing* min:sec	Intraoral Syringe Time min:sec	Intraoral Setting Time min:sec
Express XT Putty Quick	1:30	-	2:30
Express XT Light Body Quick	1:30	0:40	2:30
Express XT Regular Body Quick	1:00	0:40	2:30
Intraoral setting time for combinations of quick setting materials and one-step technique:			2:30
Express XT Putty Soft	2:00	1:00	3:30
Express XT Light Body	1:30	0:40	3:30
Express XT Regular Body	2:00	1:00	3:30
Intraoral setting time for combinations of regular setting materials and one-step technique:			3:30

* Start of mixing = Entry of paste into the mixing tip.

These are the processing times for products that have been kept at a temperature of 23° C/74° F and 20-80% relative humidity. These times are shortened due to higher temperatures of the product or prolonged if the temperature of the product is lower

Impression Taking

One-step Technique:

Recommended product combinations	
Express XT Putty Soft	Express XT Light Body or Express XT Regular Body
Express XT Putty Quick	Express XT Light Body Quick or Express XT Regular Body Quick

- Fill Express XT Putty Quick or Putty Soft into the impression tray prepared with adhesive.
- While or after the tray is being loaded, apply the wash material around the cleaned and dried preparation from the bottom up. Keep the application nozzle permanently immersed in the paste to avoid the formation of air bubbles.
 - Do not exceed tray material and wash material processing times to avoid incomplete or inaccurate impressions.
- Slowly place the loaded tray in the mouth parallel to the long axes of the prepared teeth, and hold it in position passively.
- Remove the impression from the mouth after completed setting.

Two-step Technique:

Recommended product combinations	
Express XT Putty Soft or Express XT Putty Quick	Express XT Light Body or Express XT Light Body Quick

- Fill Express XT Putty Quick or Putty Soft into the impression tray prepared with adhesive.
- Slowly insert the loaded tray into the mouth parallel to the long axes of the prepared teeth, and hold it in place without exerting pressure.
- Remove the impression from the mouth after completed setting, clean and dry.
- Excise undercuts and interdental septa etc. and cut spillways, if required.
- Clean and dry the impression thoroughly to provide for optimal coalescence of the two materials.
- Insert the wash impression material into the preliminary impression at the appropriate sites. If required, apply around the cleaned and dried preparation, while keeping the application nozzle permanently immersed in the paste to avoid the formation of bubbles.
- Replace the preliminary impression in the mouth and hold it in place.
- Remove the impression from the mouth after completed setting.

After Impression Taking

Thoroughly examine and explore the sulcus of the prepared teeth and surrounding dentition.

Remove any residual cured impression material from the mouth.

Hygiene

- Place the impression in a standard disinfectant solution. The duration complies with the instructions of the manufacturer. Do not leave the impression submerged in the disinfection solution for more than the specified amount of time as otherwise the impression may be damaged.
- After disinfection, clean the impression of any remaining disinfection solution by rinsing under running water for approx. 15 seconds.

Model Preparation

- The impression can be poured two hours after removal from the mouth. There are no other time limitations for pouring.
- In order to attain a bubble-free model, rinse the impression with water shortly before pouring, remove excess water or use a silicone surfactant.
- Pour the impression with a standard dental die or model stone. The impressions can be silver or copper plated.

Cleaning

- **Garant Dispenser:** Remove paste that has not set using an alcohol-soaked cloth. The dispenser handle and plunger can be autoclaved up to a temperature of 135 °C/275 °F. Disassemble the dispenser before autoclaving. Glutaraldehyde-based solutions can be used for disinfection.
- **Impression Tray:** The adhesive can be removed from re-usable impression trays using acetone.

Notes

- **Completely** remove the oxygen inhibition layer of composite materials, e.g. fillings or core buildups, it may impair or retard the setting of silicone impression materials.
- **Keep** the filled Garant Mixing Tip on top of the cartridge to serve as a closure until the next use. Removing and replacing the spent Garant Mixing Tip may lead to carry-over of paste and ensuing formation of clogs.

- Vinyl polysiloxane impression materials **must not** be combined with condensation type silicones or polyether impression materials. Even small traces can affect the setting process.
- Disposable latex gloves impair the setting of silicone impression materials. Vinyl gloves are more suitable.
- Under no circumstances may impressions come into contact with solvent-containing fluids. This could result in expansion and inaccurate stone models.

Storage and Stability

Store the product at 15-25° C/59-77° F. Do not use after the expiration date. Store set impressions at temperatures below 30° C/86° F.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Information as of February 2012

RUS: Authorised representative of the manufacturer in RF: ZAO "3M Russia", Russia 121614, Moscow, Krylatskaya str. bldg.17/3 T. +74957847474

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