

SIEMENS

ПОДТВЕРЖДАЮ



Н.Ю. Чижова

2017 г.

По доверенности от Siemens Healthcare Diagnostics Inc.

500 GBC Drive, Mailstop 514 P.O. Box 6101

Newark, DE 19714, USA от 17 апреля 2017 г.

Фотографические изображения медицинского изделия для диагностики in vitro

«Картридж для определения аланинаминотрансферазы для биохимических анализаторов серии
«Dimension» (Dimension Alanine Aminotransferase Flex reagent cartridge (ALTI))»

производства «Сименс Хелскеа Диагностикс Инк.» (Siemens Healthcare Diagnostics Inc.), США

2017

Фото 1. Общий вид изделия: «Картридж для определения аланинаминотрансферазы для биохимических анализаторов серии «Dimension» (Dimension Alanine Aminotransferase Flex reagent cartridge (ALTI))»

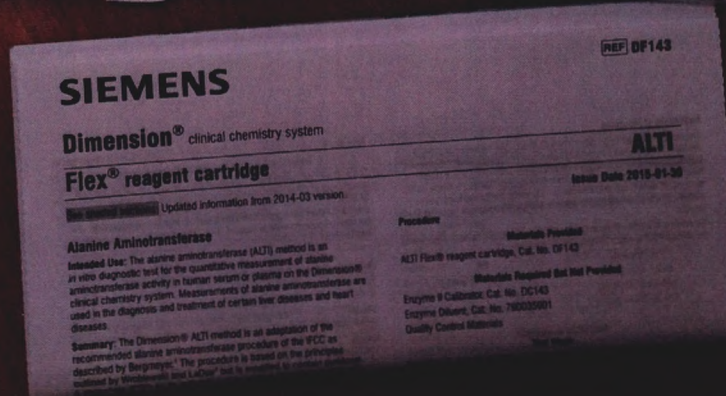


Фото 2. «Картридж для определения аланинаминотрансферазы для биохимических анализаторов серии «Dimension» (Dimension Alanine Aminotransferase Flex reagent cartridge (ALTI))», в упаковке

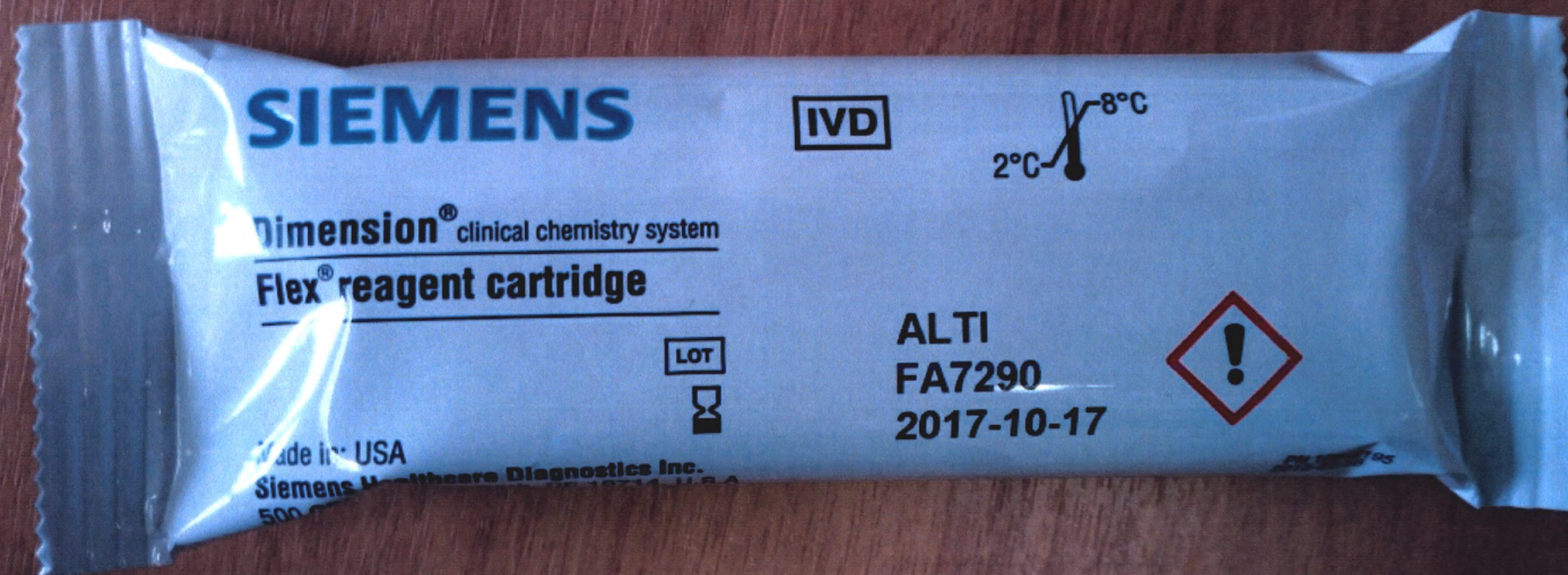


Фото 1. «Картридж для определения аланинаминотрансферазы для биохимических анализаторов серии «Dimension» (Dimension Alanine Aminotransferase Flex reagent cartridge (ALTI))», без упаковки



SIEMENS

REF DF143

Dimension[®] clinical chemistry system

Flex[®] reagent cartridge

ALTl

See shaded sections: Updated information from 2014-03 version.

Issue Date 2015-01-30

Alanine Aminotransferase

Intended Use: The alanine aminotransferase (ALTl) method is an *in vitro* diagnostic test for the quantitative measurement of alanine aminotransferase activity in human serum or plasma on the Dimension[®] clinical chemistry system. Measurements of alanine aminotransferase are used in the diagnosis and treatment of certain liver diseases and heart diseases.

Summary: The Dimension[®] ALTl method is an adaptation of the recommended alanine aminotransferase procedure of the IFCC as described by Bergmeyer.¹ The procedure is based on the principles outlined by Wroblewski and LaDue² but is modified to contain pyridoxal-5-phosphate (P5P) as an activator and to replace phosphate buffer with

Procedure

Materials Provided

ALTl Flex[®] reagent cartridge, Cat. No. DF143

Materials Required But Not Provided

Enzyme II Calibrator, Cat. No. DC143

Enzyme Diluent, Cat. No. 790035901

Quality Control Materials

Test Steps

Sampling,^{4a} reagent delivery, mixing, processing, and printing of results



Всего прошито/пронумеровано

5 листа (ов)

Н.Ю. Чистов / Н.Ю. Чистов
Подпись ФИО

