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EN Developed with Veinoplus® technology, Veinoplus® back is a stimulator designed to soothe back pain. It offers a unique programme of pain relief and relaxing massage which acts on the three main areas of the back. Compact and easy to use, you can take it anywhere!

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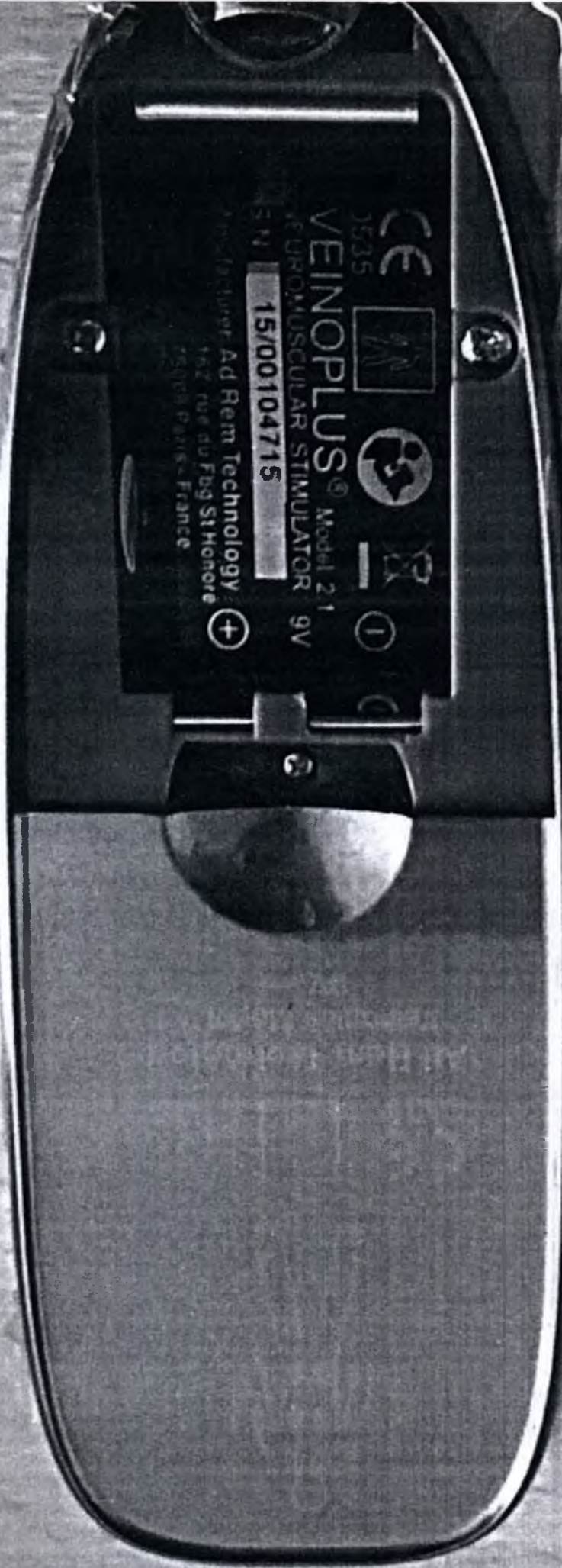
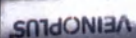
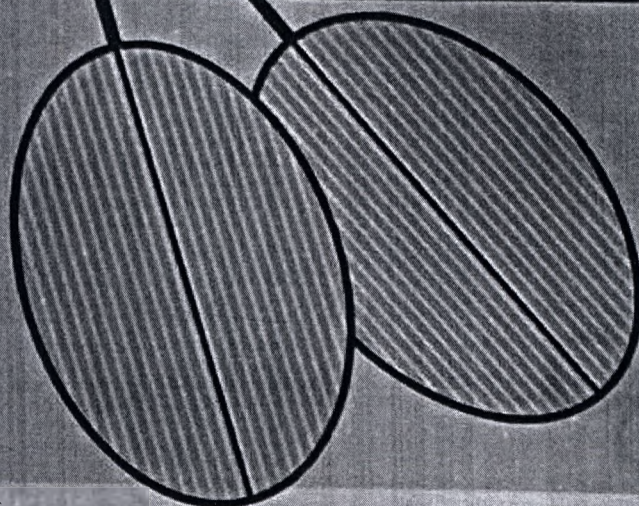
CE

PACK

Reusable
Self-Adhering
Electrodes

LATEX
FREE

Patented
GMDN code: 34374

Manufactured in USA for:

Ad Rem Technology

162, rue du Faubourg Saint Honoré

75008 Paris, France

www.adremtechnology.com

Certified ISO 13485 : 2012

EAC

INDICATIONS FOR USE:

Reusable, self-adhering, over-the-counter
Veinoplus® PACK Neurostimulation Electrodes
are indicated for use with Veinoplus® devices for
transcutaneous neuromuscular electrical
stimulation.

Transcutaneous Neurostimulation Electrodes are
passive devices serving as an interface between
a user's skin and a neurostimulation device.

For additional facts and electrode care information:

adremtechnology.com/our-products/electrodes-pack/


SIZE 8 x 13cm (3" x 5") OVAL QTY. 2

LOT

AM000010064

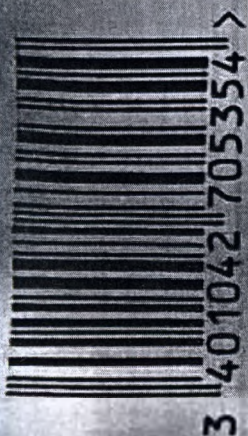2018-03

Instructions for Application, Operation and Removal:

1. Skin must always be clean and free from grease and lotion.
2. Do not apply to broken skin.
3. Attach lead wires to electrodes.
4. Remove the protective liner from the electrode. Save liner.
5. Apply electrode securely to skin. Apply center first, then smooth down to electrode edges.
6. Attach lead wires to Veinoplus device. Turn stimulator on; adjust according to Veinoplus instructions for treatment.
7. When stimulation session is complete, turn stimulator OFF.
8. Lift at the edge and peel to remove electrodes from skin.

Instructions for Electrode Care and Storage:

9. Replace the saved protective liner on the electrode. Be sure to place the ON side against the electrode gel surface.
- Disconnect the stimulator wires from the electrode lead wire connectors.
10. Replace electrodes in the original package.

If electrodes begin losing adhesion, gently rubbing one or two drops of water onto gel surface may extend usage. If not, replace with new electrodes.

Caution: Electrodes not firmly attached to the skin greatly increase the chance of burns and other adverse reactions.

Warnings and Precautions:

For all relevant hazards, contraindications, warnings, precautions and adverse side effects, refer to the Veinoplus device user's manual.

- Keep electrodes out of the reach of children.
- Electrodes are intended for single person use only.
- DO NOT remove electrode by pulling on lead wire.
- DO NOT use electrodes that are the wrong size or shape for your device, or incorrectly applied. This could result in skin irritation or thermal burns.

- Keep electrodes separated during treatment.
- DO NOT apply to broken skin. Should a skin rash or skin burn occur, immediately discontinue use and contact your physician.
- Only Veinoplus PACK electrodes MUST BE USED with Veinoplus stimulators.
- DO NOT apply electrodes across the head, or across the heart, or on the front of the neck.
- DO NOT stimulate while driving or operating machinery.

NOTE: The life of the electrode varies depending on skin conditions, skin preparation, type of stimulation, storage and climate. Please read the Veinoplus device user's manual thoroughly before use.

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Veinoplus[®] is a trademark of Ad Rem Technology
© Copyright 2013 Paris - Ad Rem Technology