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State of Minnesota

SECRETARY OF STATE

APOSTILLE

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1. Country: United States of America

This Public Document: Instructions for use (IFU) for Orthodontic Products

2. has been signed by Lori J Sheetz

3. acting in the capacity of Notary Public, State of Minnesota

4. bears the seal/stamp of Lori J Sheetz, Notary Public, State of Minnesota

CERTIFIED

5. at: St. Paul, Minnesota

6. Date: May 17, 2017

7. by: Secretary of State, State of Minnesota

8. File No: 9508486-11

9. Seal/Stamp:

10. Signature:

Steve Simon
Secretary of State

To Whom It May Concern:

In my capacity as Regulatory Affairs Manager, 3M Unitek, I certify that the attached English version of the Instructions for Use (IFU) for Orthodontic Products (Transbond XT Light Cure Adhesive, Transbond LR Light Cure Adhesive, Transbond MIP, Unitek Multi-Cure GI Band Cement, Transbond Plus Light Cure Band Adhesive, Unite Bonding Adhesive, Sondhi Rapid-Set Indirect Bonding Adhesive, and Wax Pak) intended for Russia is truthful and accurate.

Sincerely,


Mark Burville
Regulatory Affairs Manager
3M Unitek Corporation

11/17/2017
Date

STATE OF MINNESOTA)
COUNTY OF RAMSEY) ss

On May 11, 2017, before me, a Notary Public within and for said county, personally appeared Mark Burville, known by me to be the person described in and who executed the foregoing instrument and acknowledged that he executed the same as his own free act and deed.

Lori J. Sheetz, Notary Public
My Commission Expires, January 31, 2020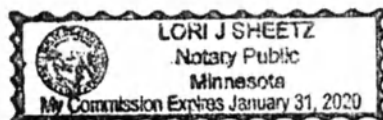

Instructions For Use

Indication: 3M Unitek products are intended for use in orthodontic treatment of malocclusions.

Intended use: This product is designed for direct bonding of ceramic orthodontic brackets and metal brackets.

Note: this product is not indicated for use with plastic (polycarbonate) brackets.

Contraindications

No known contraindications.

Scope of use

Orthodontics. Only for professional use.

Warning: This product contains acrylate monomers. Acrylate monomers are known to produce allergic skin reactions in certain sensitive individuals. May cause eye and skin irritation. Important: All resin-based primers that penetrate into the enamel rods may under certain conditions temporarily alter the appearance of the enamel. It does not discolor the enamel; rather it creates a contrast to the chalky appearance of the surrounding enamel.

Precautions: Avoid eye and skin contact. Wear gloves when handling this material. First Aid:

Eye contact: Immediately flush with plenty of water. See a physician if irritation persists.

Skin contact: Wash affected area with soap and water. See a physician if irritation persists.

Precaution: Porcelain Crowns or Facings: Care must be taken when bonding to porcelain crowns or facings as debonding may cause chipping, delamination or breakage of the crown. Do not bond to porcelain crowns that have thin facings or that appear to be compromised. Prepare the porcelain crown to be bonded by using a porcelain primer such as 3M™ ESPET™ RelyX™ Ceramic Primer. Follow instructions included with the porcelain primer.

A. Tooth Preparation

1. Isolate teeth using the Dry Field System or a combination of retractors, dri-angles and cotton rolls.
2. Prophyl teeth with an oil-free pumice or paste.
3. Rinse with water.
4. Air dry thoroughly using oil and moisture-free air source.

B. Acid Etching

1. If using Transbond™ Plus Self Etching Primer, proceed to Option 3: Priming Surfaces with Transbond Plus Self Etching Primer steps below. Otherwise, continue with phosphoric acid etching.
2. Apply Unitek™ Etching Gel (REF 712-039 or 712-044) to tooth surfaces following the instructions provided with the etching gel delivery system. If using other gel etching systems, consult the manufacturer's instructions for proper technique and recommended etching times.
3. Rinse with water.
4. Air dry thoroughly using oil and moisture-free air source.

C. Priming Teeth

Option 1: Priming Surfaces with Transbond™ XT Primer

1. Air dry tooth thoroughly.
2. Place small amount of Transbond™ XT Primer in well.
3. Apply thin uniform coat of primer on each tooth surface to be bonded. Note: Since Transbond XT primer acts as a wetting agent, only a very thin film of primer is necessary.

Option 2: Priming Surfaces with Transbond™ MIP Moisture Insensitive Primer

For detailed instructions on Transbond™ MIP Moisture Insensitive Primer, please refer to Instructions For Use REF 011-563.

Option 3: Priming Surfaces with Transbond™ Plus Self Etching Primer

Orthodontic Products

Transbond XT Light Cure Adhesive

For detailed instructions on Transbond™ Plus Self Etching Primer, please refer to REF 011-581.

D. Adhesive Application from Capsules

Note: Do not apply adhesive onto brackets until patient is ready to be bonded.

1. To insert capsule into dispenser, open dispenser handle as wide as possible. Pull plunger toward opened handle. Place capsule tip into the barrel slot.
2. Seat the capsule in the barrel by pushing the capsule forward (away from the handle) as far as possible. Push plunger towards capsule. Close handle grip until plunger contacts capsule. Remove the cap from the capsule.
3. Dispense a small amount of Transbond XT™ Adhesive paste onto bracket base using a slow, steady pressure. When completed wipe capsule tip clean and replace cap.
4. To remove the used capsule, open the dispenser handle as wide as possible. Pull plunger away from capsule. Push the capsule toward the plunger. Turn dispenser upside down to drop capsule into hand.

E. Adhesive Application from Syringes

Note: Do not apply adhesive onto brackets until patient is ready to be bonded.

With the syringe apply a small amount of Transbond XT adhesive paste onto bracket base. Use sparingly. When completed, wipe syringe tip clean and replace cap.

F. Placement and Curing

1. Immediately after applying adhesive, lightly place bracket onto tooth surface.
 2. Adjust bracket to final position. press firmly to seat bracket.
- Note:** If final positioning will be delayed, cover patient's mouth with a dark colored mask or other article to avoid premature curing by ambient light.
3. Gently remove excess adhesive from around the bracket base without disturbing the bracket.
 4. Hold the curing light stationary at a distance of 2-3 mm above interproximal contact for metal appliances and perpendicular to ceramic appliance surface.
- A technique tip for faster light curing of metal brackets is to position the curing light guide interproximal to two brackets. However, in order for the bracket to fully cure, both sides of the bracket must be illuminated.

Precaution: Please follow manufacturer's directions for the handling, proper use and recommendations for eye protection when using a curing light.

Refer to the chart to determine the curing conditions for optimal bond strength. If your curing light does not appear on this chart, please refer to your light manufacturer's instructions for curing conditions.

Appliance with Transbond™ XT Adhesive	Ortholux™ LED Curing Light (App. 1000 mW/cm ²) (LED)	Ortholux™ Luminous Curing Light (App. 1600 mW/cm ²) (LED)
Metal Brackets	5 seconds mesial + 5 seconds distal	3 seconds mesial + 3 seconds distal
Ceramic Brackets	5 seconds through the bracket	3 seconds through the bracket
Bondable Buccal Tubes	10 seconds mesial + 10 seconds occlusal	6 seconds mesial + 6 seconds occlusal

5. Archwires may be placed immediately after curing the last bracket.

Orthodontic Products

Transbond XT Light Cure Adhesive






G. Disinfection Information

Please refer to literature 011-650, "Reprocessing Instructions For Reusable Non-Sterile Devices" for cleaning and disinfection of 712-032, Adhesive Dispensing Gun.

Storage and Use

1. Do not expose materials to elevated temperatures or intense light. Bring to room temperature prior to use.
2. Do not store material in proximity to eugenol-containing products as this may inhibit the proper adhesive curing.
3. This system is designed to be used at room temperature (68-77°F, 20-25°C).
Storage between 35-80°F, 2-27°C
4. Shelf-life at room temperature is per the expiration date on the kit box. Rotate inventory to optimize shelf life.
5. Light Sensitive. Do not expose uncured Transbond XT Adhesive to ambient light for extended periods of time. When not in use, store syringes with caps at room temperature.

Orthodontic Products
Transbond XT Light Cure Adhesive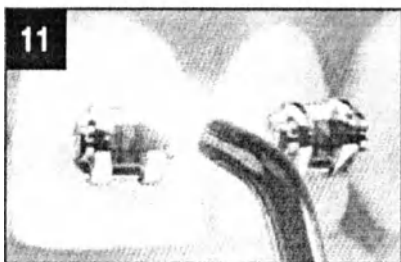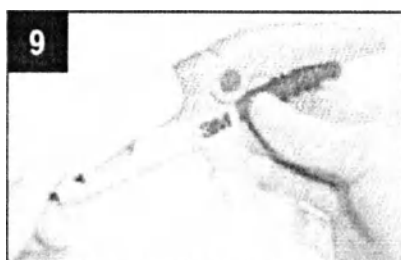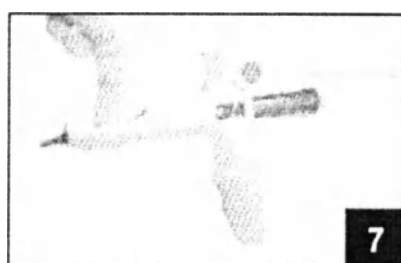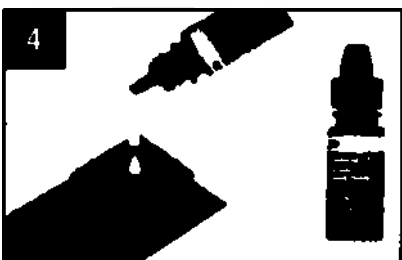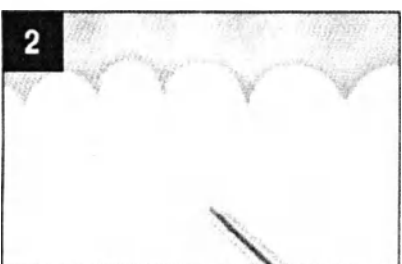

Orthodontic Products

Transbond XT Light Cure Adhesive

Shelf life

Shelf life at room temperature is 36 months from the date of manufacture.

Contents

Transbond XT Light Cure Adhesive Capsule Kit:

- Capsule (25 pcs.);
- Transbond XT Primer;
- Dispenser gun;
- Brush holder;
- Brush tips (60 pcs.).

Transbond XT Light Cure Adhesive Kit:

- Syringe (2 pcs.);
- Transbond XT Primer;
- Brush holder;
- Brush tips (60 pcs.).

Transbond XT Light Cure Adhesive Capsules (25 pieces per pack):

Transbond XT Light Cure Adhesive Syringes (2 or 4 pieces per pack)

Packaging

In Capsule Kit: capsule with adhesive: length 2.79 cm $\pm$ 0.13 cm, diameter 0.77 cm $\pm$ 0.01 cm.
Secondary packaging - box for capsules with adhesive: material: cardboard, dimensions 6.74 cm x 4.6 cm x 1.66 cm.

Tertiary packing - carton box material: cardboard, dimensions: 19.05 cm x 16.98 cm x 1.9 cm

In Capsules (25 pcs per pack): capsule with adhesive: length 2.79 cm $\pm$ 0.13 cm, diameter 0.77 cm $\pm$ 0.01 cm.

Secondary packaging - box for capsules with adhesive: material: cardboard, dimensions 6.74 cm x 4.6 cm x 1.66 cm.

In Kit: dispenser with adhesive: length: 12.9 cm $\pm$ 0.3 cm.

Secondary packaging-cardboard box: material: cardboard, dimensions 6.03 cm x 5.7 cm x 13.17 cm.

In Syringes (2 or 4 pcs per pack): syringe with adhesive: length: 12.9 cm $\pm$ 0.3 cm.

Secondary package - package: material: low density polyethylene, dimensions: 15.24 $\pm$ 0.6 cm x 25.4 $\pm$ 0.6 cm

Medicinal substances. The product does not use medicinal substances.

Biological safety. The product does not use materials of animal or human origin.

Requirements to the medical device maintenance and repair: Not applicable.

Disinfection and cleaning: Please refer to literature 011-650, "Reprocessing Instructions For Reusable Non-Sterile Devices" for cleaning and disinfection of 712-032, Adhesive Dispensing Gun.

Labelling. The products are supplied in a packaging with the labelling containing the following information: name, lot and reference number; name and address of the manufacturer; contents; shelf life; and symbols - Refer to Instructions for Use, EU compliance, storage / transportation temperature limitation, store away from sunlight, do not reuse, recyclable packaging, low density polyethylene packaging.

Disposal. Packaging/contents should be disposed in accordance with the local/regional/national/international regulations.

Medical devices should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class B).

Warranty

Orthodontic Products

Transbond XT Light Cure Adhesive

3M Company warrants this product will be free from defects in material and manufacture. 3M UNITEK MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Company's sole obligation shall be repair or replacement of the 3M Company product.

Name and address of the manufacturer: 3M Unitek Corporation, 2724 South Peck Road, Monrovia, California, 91016, USA

Authorized representative on the territory of the Russian Federation:

ZAO "3M Russia" Russia 121614, Moscow, Krylatskaya str. bldg.17/3. Tel.:+7 (495) 784 7474

Fax: +7 (495) 784 7475. 3mra@mmm.com

Orthodontic Products

Transbond LR Light Cure Adhesive

Instructions For Use

Warning: Single Use Only. Reuse may impair product performance and increase risk of patient injury.

Warning: Transbond™ LR Light Cure Adhesive contains acrylate monomers which are known to produce allergic skin reactions in certain sensitive individuals. May cause eye and skin irritation. Important: All resin-based primers that penetrate into the enamel rods may under certain conditions temporarily alter the appearance of the enamel. It does not discolor the enamel; rather it creates a contrast to the chalky appearance of the surrounding enamel.

Precautions: Avoid eye and skin contact. Wear gloves when handling this material.

First Aid:

Eye contact: Immediately flush with plenty of water. See a physician if irritation persists.

Skin contact: Wash affected area with soap and water. See a physician if irritation persists.

Caution: Follow manufacturer's directions on the handling, proper use, and recommendations for eye protection, when using a visible light curing unit.

Indication: 3M Unitek products are intended for use in orthodontic treatment of malocclusions.

Intended use: Transbond LR light cure adhesive in capsules is specifically designed for bonded lingual retainers. Transbond LR is not indicated for bonding of brackets because bracket drift or skating may occur.

Contraindications

No known contraindications.

Scope of use

Orthodontics. Only for professional use.

Direct Technique for Bonding a Lower 3-3 Lingual Retainer Tip – If you are concerned about serious lower incisor rotation, the lingual retainer may be placed before the total removal of the lower incisor brackets. Otherwise the application of the bonded lingual retainers can be accomplished following the debonding procedure.

Tooth Preparation. Adapt the wire of your choice to the lingual 3-3 surfaces and set aside.


Acid Etching

1. If using Transbond™ Plus Self Etching Primer, proceed to Option 3: Priming Surfaces with Transbond Plus self etching primer steps below. Otherwise, continue with prophyl.

2. Prophyl teeth with an oil-free pumice or paste. (Figure 1) Rinse with water. (Figure 2)

Orthodontic Products

Transbond LR Light Cure Adhesive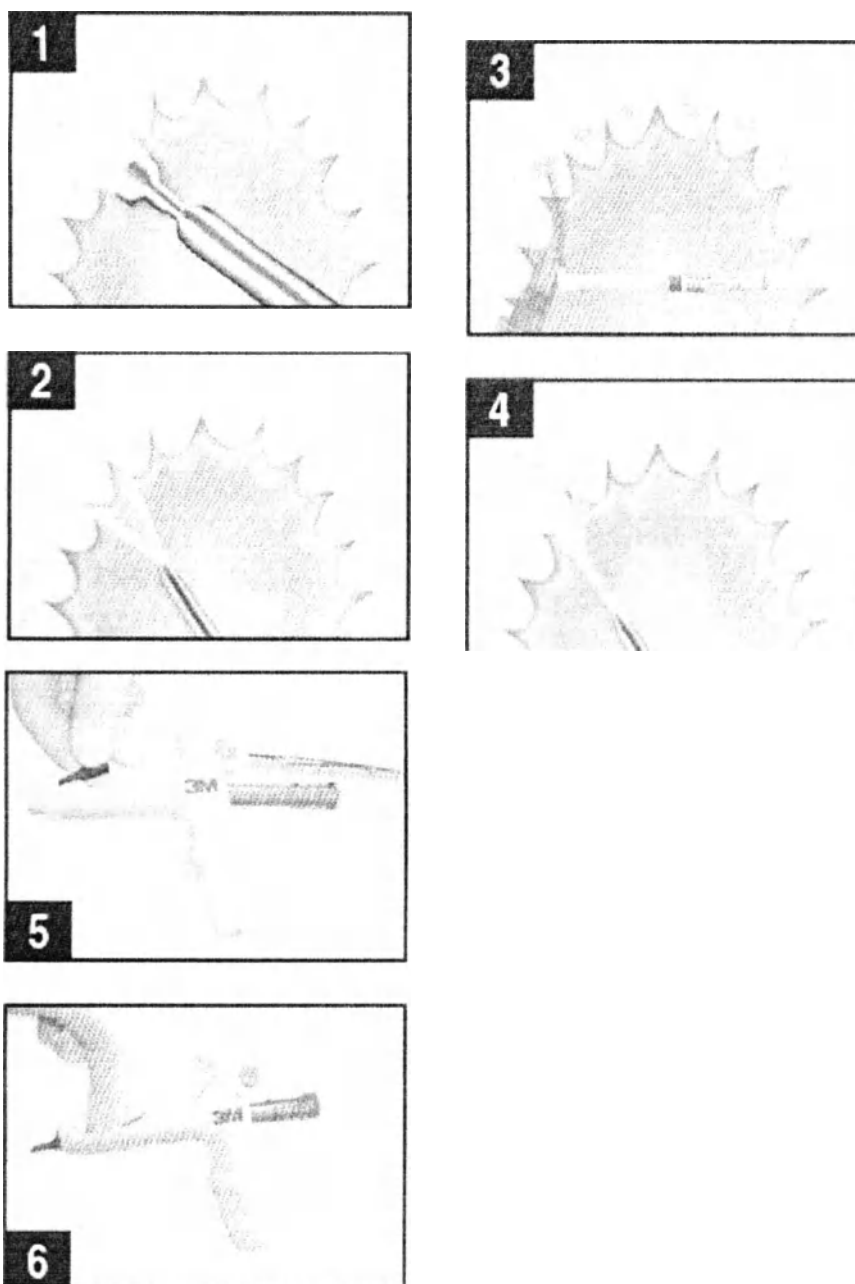

3. Isolate for etching the lower 3 -3 area by using long cotton rolls both buccal and lingual to the 3-3 area.

4. Apply Unitek™ Etching Gel (REF 712- 039 or 712- 044) to lingual teeth surfaces following the instructions. Rinse thoroughly to insure total removal of etchant, re-isolate using new cotton rolls. Cheek retractors are usually not required for the lower arch. (Figure 3)

5. Rinse with water. (Figure 4)

Priming Teeth

Option 1: Priming Surfaces with Transbond™ XT Primer. For detailed instructions on Transbond XT primer, please refer to REF 011-519.

Orthodontic Products

Transbond LR Light Cure Adhesive

Option 2: Priming Surfaces with Transbond™ MIP Moisture Insensitive Primer. For detailed instructions on Transbond MIP moisture insensitive primer, please refer to REF 011-563.

Option 3: Priming Surfaces with Transbond™ Plus self etching primer. For detailed instructions on Transbond Plus self etching primer, please refer to REF 011-581.

Lingual Retainer Bonding


1. To insert capsule into dispenser, open dispenser handle as wide as possible. Pull plunger toward opened handle. Place capsule tip into the barrel slot. (Figure 5)

2. Seat the capsule in the barrel by pushing the capsule forward (away from the handle) as far as possible. (Figure 6) Push plunger towards capsule. Remove the cap from the capsule.

3. Place floss threaders from the lingual through to the buccal in the interproximal between the laterals and the cuspids. Leave a slight "loop" on the lingual through which you will place the lingual wire (Figure 7)

Place preformed retainer wire in desired position. Wire can be held in place with a loop of floss placed in the interproximal between the laterals and the cuspids.

4. Pull the floss threader end so as to tighten the lingual wire against the lingual surfaces of the lower incisors and cuspids. You may dispense adhesive from capsule directly onto tooth or transfer adhesive from mixing pad. When completed, wipe capsule tip clean and replace cap. Let the material flow over and around the wire. (Figure 8)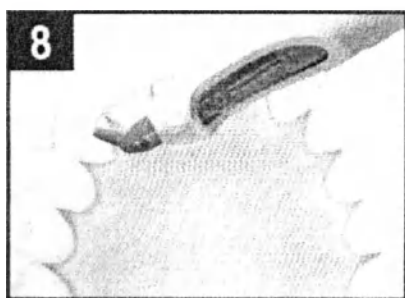

Apply a "button" of Transbond LR onto each end of the wire. Let the material flow until satisfied with the coverage. Remember, the advantage of light cure is that you have adequate time. Light cure each button for 10 seconds, trim if necessary. For maximum stability, repeat the application of bonding material to the lingual of each incisor.

Note: The above mentioned curing times apply only to the Ortholux™ XT Curing Light. Other curing lights, depending on their output, might require other curing times. (Figure 9)

5. Push the floss back through the interproximal area toward the lingual, enlarging the "loop". Cut the loop and pull the floss threader out from the buccal.

6. For maximum stability, repeat the application of bonding material to the lingual of each incisor. 7. Trim and polish if necessary.

Orthodontic Products

Transbond LR Light Cure Adhesive

8. To remove the used capsule from dispensing gun, open the dispenser handle as wide as possible. Pull plunger away from capsule. Push the capsule toward the plunger. (Figure 10) Turn dispenser upside down to drop capsule into hand.

Storage and Use

1. Do not expose materials to elevated temperatures or intense light. Bring to room temperature prior to use.
2. Do not store material in proximity to eugenol- containing products as this may inhibit the proper adhesive curing.
3. This system is designed to be used at room temperature (20 -25°C, 68-77°F). Storage between 2-27°C/35-80°F Light Sensitive
4. Shelf-life at room temperature is per the expiration date on the kit box. Rotate inventory to optimize shelf life.

Shelf life

Shelf life at room temperature is 36 months from the date of manufacture.

Contents

Transbond LR Light Cure Adhesive Capsule Kit:

- Capsule (25 pcs.);
- Transbond XT Primer;
- Dispenser gun;
- Brush holder;
- Brush tips (60 pcs.).

Transbond LR Light Cure Adhesive Capsules (25 pcs.).

Packaging

In Capsule Kit: capsule with adhesive: length 2.79 cm $\pm$ 0.13 cm, diameter 0.77 cm $\pm$ 0.01 cm.

Secondary packaging - box for capsules with adhesive: material: cardboard, dimensions: 6.74 cm x 4.6 cm x 1.66 cm.

Tertiary packing - cardboard box material: cardboard, dimensions: 19.05 cm x 16.98 cm x 1.9 cm

In Capsules (25 pcs per pack): capsule with adhesive: length 2.79 cm $\pm$ 0.13 cm, diameter 0.77 cm $\pm$ 0.01 cm.

Secondary packaging - box for capsules with adhesive: material: cardboard, dimensions: 6.74 cm x 4.6 cm x 1.66 cm.

Medicinal substances. The product does not use medicinal substances.

Biological safety. The product does not use materials of animal or human origin.

Requirements to the medical device maintenance and repair: Not applicable.

Disinfection and cleaning: Please refer to literature 011-650, "Reprocessing Instructions For Reusable Non-Sterile Devices" for cleaning and disinfection of 712-032, Adhesive Dispensing Gun.

Labelling. The products are supplied in a packaging with the labelling containing the following information: name, lot and reference number; name and address of the manufacturer; contents; shelf life; and symbols - Refer to Instructions for Use, EU compliance, storage / transportation temperature limitation, store away from sunlight, do not reuse, recyclable packaging, low density polyethylene packaging.

Disposal. Packaging/contents should be disposed in accordance with the local/regional/national/international regulations.

Medical devices should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class B).

Warranty

Orthodontic Products

Transbond LR Light Cure Adhesive

3M Company warrants this product will be free from defects in material and manufacture. 3M UNITEK MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Company's sole obligation shall be repair or replacement of the 3M Company product.

Name and address of the manufacturer: 3M Unitek Corporation, 2724 South Peck Road, Monrovia, California, 91016, USA

Authorized representative on the territory of the Russian Federation:

ZAO "3M Russia" Russia 121614, Moscow, Krylatskaya str. bldg.17/3, Tel.:+7 (495) 784 7474 Fax: +7 (495) 784 7475, 3mra@mmm.com

Orthodontic Products

Transbond MIP Moisture Insensitive Primer

Instructions for Use

Indication: This product is intended for use in orthodontic treatment of malocclusions.

Intended for use: The primer prepares the surface of tooth for bonding orthodontic appliances.

Contraindications

No known contraindications.

Scope of use

Orthodontics. Only for professional use.

Warning Acrylate monomers are known to produce allergic skin reactions in certain sensitive individuals. May cause eye and skin irritation.

Important: All resin-based primers that penetrate into the enamel rods may under certain conditions temporarily alter the appearance of the enamel. It does not discolor the enamel; rather it creates a contrast to the chalky appearance of the surrounding enamel.

Precaution: Avoid eye and skin contact. Wear gloves when handling this material.

First Aid

Eye contact: Immediately flush with plenty of water. See a physician if irritation persists.

Skin contact: Wash affected area with soap and water. See a physician if irritation persists.

Note: Transbond™ MIP Moisture Insensitive Primer has been formulated to work in both a moist and dry environment. If bonding is delayed and moisture contamination occurs after the application of primer onto the tooth surface, apply a fresh coat of primer over the contaminated area, agitate (rub) for 3 seconds on the tooth surface, deliver a gentle air burst for 1-2 seconds and bond immediately.

Tooth Preparation

1. Prophyl teeth with an oil-free pumice or paste. (Figure 1). Rinse with water. Isolate teeth.
2. Air dry thoroughly using oil and moisture-free air source. (Figure 2).

Acid Etching

3. Apply Transbond™ XT Etching Gel (REF 712-039) to teeth surfaces following the instructions with the etching system. (Figure 3). If using other etching systems, consult the manufacturer's instructions for proper technique and recommended etching times.
4. Rinse thoroughly with water to ensure total removal of etchant. (Figure 4). Remove excess water, but drying of tooth surface is not necessary.

Priming Surfaces with Transbond MIP

Note: For best results, closely follow priming instructions.

5. Dispense three drops of Transbond MIP moisture insensitive primer (REF 712-021) in primer well. (Figure 5). Tightly cap primer bottle to prevent evaporation.
6. Dip and completely saturate the brush tip. (Figure 6). Apply one liberal coat, covering entire etched surface of tooth. (Figure 7). Avoid touching the gingival area.
7. Re-dip brush in primer for each tooth to be primed. (Figure 6).
8. Gently blow air on each tooth for 2-5 seconds, aiming the air stream perpendicular to the labial surface of the tooth. (Figure 8).

Adhesive Bonding

Follow APC™ Adhesive Coated Appliance, Transbond™ XT or LR Light Cure Adhesive, Concise™ Orthodontic Bonding Adhesive or Unite™ Adhesive instructions for appropriate bonding procedures.

Storage and Use









1. Do not expose materials to elevated temperatures or intense light. Bring to room temperature prior to use.

Orthodontic Products

Transbond MIP Moisture Insensitive Primer

2. Do not store material in proximity to eugenol-containing products as this may inhibit the proper adhesive curing.
3. This system is designed to be used at room temperature (20°- 25°C, 68°-77°F).
Storage between 2°-27°C/35°-80°F Light Sensitive
4. Shelf life at room temperature is per the expiration date on the package label. Rotate inventory to optimize shelf life.

Orthodontic Products
Transbond MIP Moisture Insensitive Primer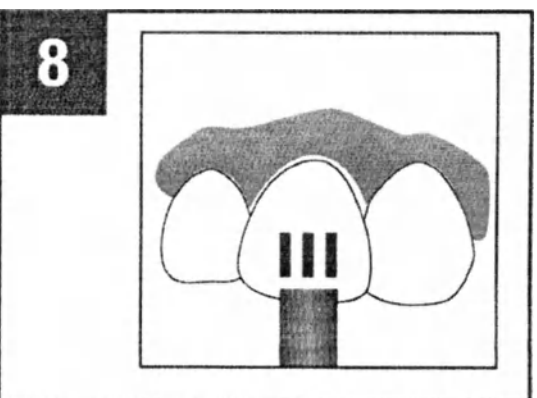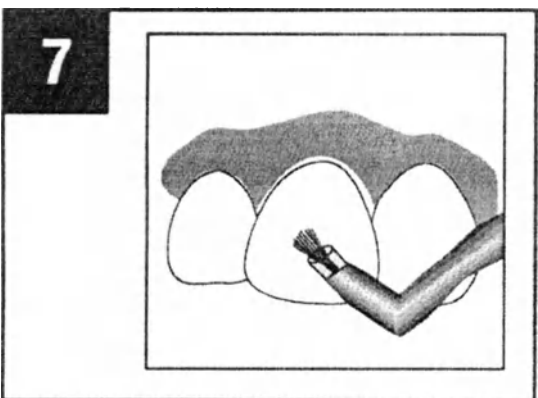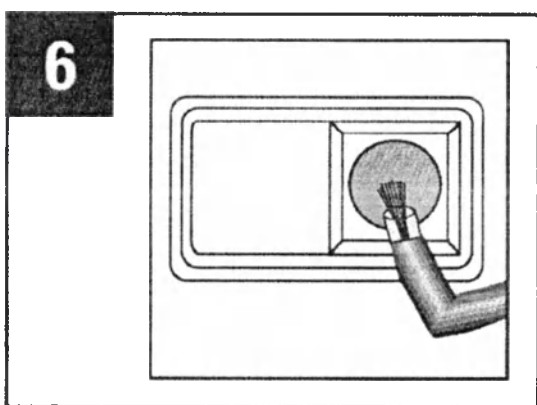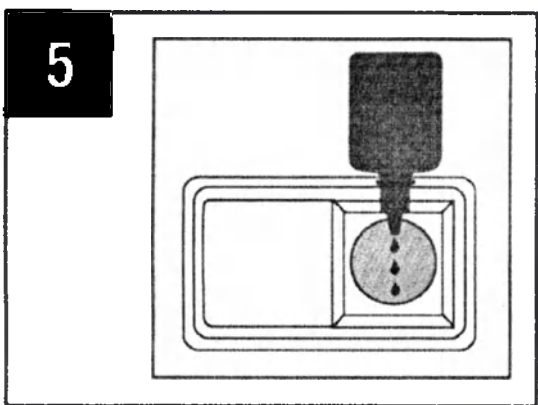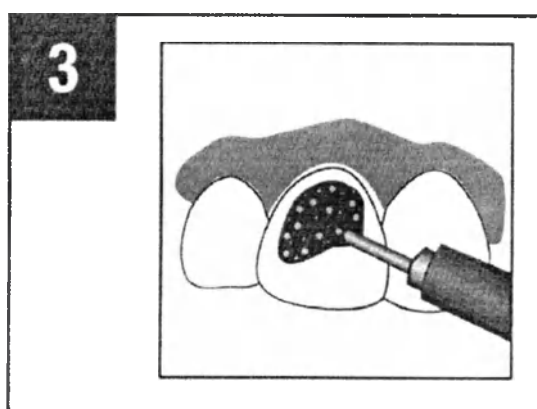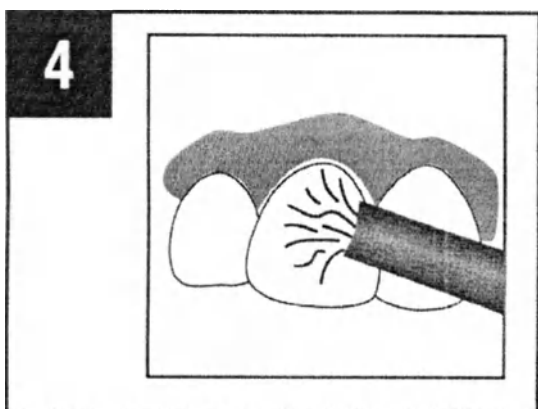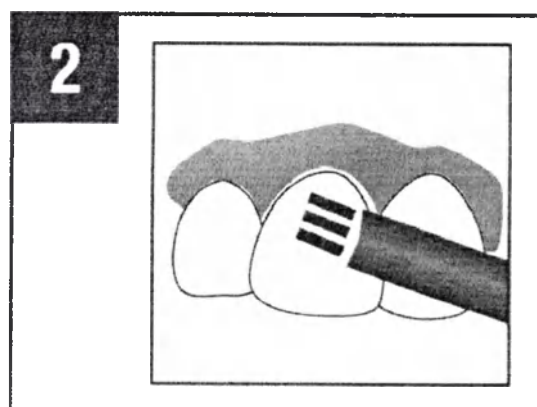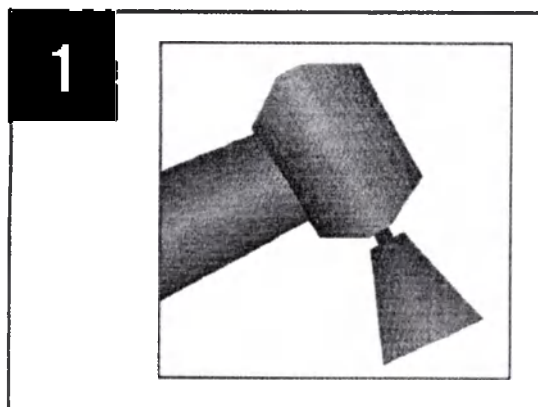

Orthodontic Products

Transbond MIP Moisture Insensitive Primer

Shelf life

Shelf life at room temperature is 36 months from the date of manufacture.

Contents

Transbond MIP Moisture Insensitive Primer Kit:

- Transbond MIP Primer;
- Light Sensitive Wells (1);
- Disposable Well (20 psc);
- Brush holder;
- Brush tips (60 pcs.).

Transbond MIP Moisture Insensitive Primer in a separate packaging

Packaging

In Kit: Primer Bottle: Height 4.57 cm $\pm$ 0.07 cm, Diameter 1.76 cm $\pm$ 0.04 cm.

Secondary packaging - carton box material: cardboard, dimensions: 19.05 cm x 16.98 cm x 2.2 cm.

In a separate packaging: Primer Bottle: Height 4.57 cm $\pm$ 0.07 cm, Diameter 1.76 cm $\pm$ 0.04 cm.

Secondary packaging - package: material: low density polyethylene, dimensions: 12.7cm x 20.32cm.

Medicinal substances. The product does not use medicinal substances.

Biological safety. The product does not use materials of animal or human origin.

Requirements to the medical device maintenance and repair: Not applicable.

Disinfection and cleaning: Not applicable.

Labelling. The products are supplied in a packaging with the labelling containing the following information: name, lot and reference number; name and address of the manufacturer; contents: shelf life; and symbols - Refer to Instructions for Use, EU compliance, storage / transportation temperature limitation, store away from sunlight, do not reuse, recyclable packaging, low density polyethylene packaging.

Disposal. Packaging/contents should be disposed in accordance with the local/regional/national/international regulations.

Medical devices should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class B).

Warranty

3M Company warrants this product will be free from defects in material and manufacture. 3M UNITEK MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Company's sole obligation shall be repair or replacement of the 3M Company product.

Name and address of the manufacturer: 3M Unitek Corporation, 2724 South Peck Road, Monrovia, California, 91016, USA

Authorized representative on the territory of the Russian Federation:

ZAO "3M Russia" Russia 121614, Moscow, Krylatskaya str. bldg.17/3, Tel.: +7 (495) 784 7474 Fax: +7 (495) 784 7475, 3mra@minm.com

Orthodontic Products

Unitek Multi-Cure Glass Ionomer Band Cement

Instructions For Use

Storage between 2°C-27°C/35°F-80°F

Light Sensitive

General Information: Unitek™ Multi-Cure Glass Ionomer Orthodontic Band Cement is a two- part. powder/liquid glass ionomer cement for orthodontic banding. The powder is a radiopaque, fluoroaluminosilicate glass. The liquid is a light sensitive, aqueous solution of a modified polyalkenoic acid. The chemical properties of the glass ionomer cements in general are such that fluoride leaches out from the material. Using the recommended powder/liquid ratio, this kit contains enough material to place a minimum of 120 bands.

Unitek Multi-Cure Glass Ionomer Orthodontic Band Cement will set by exposure to visible light. It also has self-curing mechanisms to provide a relatively rapid set where light does not penetrate.

Warning: Liquid contains HEMA (2-hydroxyethyl methacrylate). HEMA is severely irritating to the eye and is a known contact allergen. A small percentage of the population is known to have an allergic response to acrylate resins. To reduce the risk of allergic response, minimize exposure to these materials. In particular, exposure to uncured resin should be avoided. Use of protective gloves and a no-touch technique is recommended. If skin contacts liquid or powder/liquid mix, wash skin immediately with soap and water. Acrylates may penetrate commonly used gloves. If glove contacts liquid or powder/liquid mix, remove and discard glove, wash hands immediately with soap and water, and then reglove.

Liquid and powder/liquid mix may cause eye irritation upon contact and may be mildly irritating to oral soft tissue upon contact. Avoid contact with eyes and minimize contact with oral soft tissue. If contact with eyes occurs, flush immediately with large amounts of water. If irritation persists, consult a physician.

Warning: Multi-Cure Glass Ionomer Band Cement is intended for use only as an orthodontic band cement. Do not use for cementing preformed stainless steel crowns, as damage to the tooth may occur upon debonding.

Indication: 3M Unitek products are intended for use in orthodontic treatment of malocclusions.

Intended use: Unitek Multi-Cure Glass Ionomer Band Cement is intended for use as an orthodontic band cement.

Caution: This product has been designed to increase band retention. If you experience difficulty upon debanding, the use of a band splitter may be indicated for ease of removal.

Follow manufacturer's direction on the handling, proper use and recommendations for eye protection when using a visible light curing unit.

Scope of use

Orthodontics. Only for professional use.

Contraindications

No known contraindications.

Preparation

1. After removing separators, scale the gingival margins of teeth to be banded in order to remove any plaque.
2. Prophyl teeth with an oil free pumice or paste.
3. Rinse with water.
4. Trial fit each band to be placed on the patient.
5. After verifying proper fit, remove the trial fitted bands and clean.

Tip: To avoid getting cement in the buccal tube, wipe its mesial and distal ends with lip balm or cement guard.

Dispensing of powder and liquid: One level powder scoop (large) and three drops of liquid or one level powder scoop (small) and one drop of liquid (1.9/1 ratio of powder to liquid by weight) will provide an adequate amount of material for placing one large or small band respectively. Additional powder can be incorporated to obtain a thicker consistency mix. Additional liquid can be incorporated to obtain a thinner consistency mix.

Shake the bottle to fluff the powder before dispensing. Insert the powder scoop into the bottle, overfill it

Orthodontic Products

Unitek Multi-Cure Glass Ionomer Band Cement

with loosely packed powder and withdraw it against the plastic leveler to remove excess powder and obtain a level scoop. Dispense the desired number of powder scoops onto the mixing pad.

To best obtain a proper liquid drop size, hold the liquid vial vertically with the dropper tip down and without the tip contacting the mixing pad. Squeeze the vial to dispense the desired number of liquid drops onto the mixing pad.

Mixing

1. Using a large cement spatula, first mix one-half of the powder into the liquid, and then add the remainder of the powder. Continue mixing until all of the powder is incorporated into the liquid. **Do not mix longer than 45 seconds.**

2. Working time of the standard powder/liquid ratio is 3 minutes from the start of mix at a room temperature of 73° F (23° C). **Higher temperatures will shorten working time. Lower temperatures will lengthen working time.**

Cementing

1. Isolate teeth to be banded. For lower bandings, also place cotton rolls between the tongue and molars.

2. Load the band with cement and seat on the tooth.

Curing: Unitek Multi-Cure Glass Ionomer Orthodontic Band Cement cures by either exposing it to visible light or by chemical reaction.

• **Light Cure:** After removing excess material with a scaler, light cure the Unitek Multi-Cure Glass Ionomer Orthodontic Band Cement by placing the light guide 1-2 mm above the occlusal surface of the tooth being banded and curing the material for 20 seconds using the Ortholux™ LED Curing Light. After the initial light cure, the chemical cures will continue the hardening of the material.

Note: The above mentioned curing time applies only to Ortholux™ LED Curing Light. Other curing lights, depending on their output intensity might require longer curing times.

• **Chemical Cure:** After placing the band on the tooth and cleaning up the excess, place a cotton roll over the band and ask the patient to bite on it gently to maintain position during the setting process. Self cure set time is 5 minutes from the end of mixing at **oral cavity temperature.**

After placing all the bands, remove cotton rolls and dry angles.

Caution: Spray dry angles with water before removing to prevent sticking to the patient's cheek.

Storage & Use

1. See outer package for use by date.

2. The glass ionomer system is designed to be used at room temperatures of approximately (21° - 24° C) 70° - 75° F.

3. The glass ionomer liquid is a light sensitive material. Protect it from ambient light exposure by dispensing just prior to use and replace vial cap immediately after dispensing.

4. The glass ionomer powder is sensitive to high humidity and light. Store with bottle cap securely tightened.

Shelf life

Shelf life at room temperature is 36 months from the date of manufacture.

Contents

Unitek Multi-Cure Glass Ionomer Band Cement Kit:

- Powder Bottle;
- Liquid Bottle;
- Dispensing spoon (1);
- Mixing Pad (2 pcs.);

Packaging

In Kit: Bottle with powder: Height 94 mm ± 1.7 mm, Base diameter 38.6 mm ± 0.7 mm, Top diameter 25 mm, Thread pitch on the cap 28-400 ST. Bottle with liquid: Height (without cap) 7.7 cm ± 0.1 cm, Width larger 4.38 cm ± 0.13 cm, Width smaller 2.35 cm ± 0.13 cm, Height of the cap 1.18 cm ± 0.02 cm, Pitch thread on the cap 2.117 mm.

Secondary packaging - carton box: material: cardboard, dimensions: 8.25 cm x 8.33 cm x 10.4 cm.

Medicinal substances. The product does not use medicinal substances.

Biological safety. The product does not use materials of animal or human origin.

Orthodontic Products

Unitek Multi-Cure Glass Ionomer Band Cement

Requirements to the medical device maintenance and repair: Not applicable.

Disinfection and cleaning: Not applicable.

Labelling. The products are supplied in a packaging with the labelling containing the following information: name, lot and reference number; name and address of the manufacturer; contents; shelf life; and symbols - Refer to Instructions for Use, EU compliance, storage / transportation temperature limitation, store away from sunlight, recyclable packaging, low density polyethylene packaging.

Disposal. Packaging/contents should be disposed in accordance with the local/regional/national/international regulations.

Medical devices should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class B).

Warranty

3M Company warrants this product will be free from defects in material and manufacture. 3M UNITEK MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Company's sole obligation shall be repair or replacement of the 3M Company product.

Name and address of the manufacturer: 3M Unitek Corporation, 2724 South Peck Road, Monrovia, California, 91016, USA

Authorized representative on the territory of the Russian Federation:

ZAO "3M Russia" Russia 121614, Moscow, Krylatskaya str. bldg.17/3, Tel.:+7 (495) 784 7474 Fax: +7 (495) 784 7475, 3mra@mmm.com

Orthodontic Products

Transbond Plus Light Cure Band Adhesive

Instructions For Use

Warning: Transbond™ Plus Light Cure Band Adhesive contains acrylate monomers. Acrylate monomers are known to produce allergic skin reactions in certain sensitive individuals. May cause eye and skin irritation.

Precautions: Avoid eye and skin contact. Wear gloves when handling this material.

First aid:

Eye contact: Immediately flush with plenty of water. See a physician if irritation persists.

Skin contact: Wash affected area with soap and water. See a physician if irritation persists.

Precaution: Please follow manufacturer's directions on the handling, proper use and recommendations for eye protection, when using a curing light.

Precaution: Care must be taken when bonding to porcelain crowns or facings as debonding may cause chipping, delamination or breakage of the crown. Do not bond to porcelain crowns that have thin facings or that appear to be compromised.

Indication: 3M Unitek products are intended for use in orthodontic treatment of malocclusions.

Intended for use: Transbond Plus Light Cure Band Adhesive is designed for banding orthodontic appliances.

Scope of use

Orthodontics. Only for professional use.

Contraindications

No known contraindications.

Tooth Preparation

1. After removing separators, scale the gingival margins of teeth to be banded in order to remove any plaque. (Figure 1)
2. Prophyl teeth with an oil-free pumice or paste.
3. Rinse with water. (Figure 2)
4. Air dry thoroughly using oil and moisture-free air source. (Figure 3)
5. Trial fit each band to be placed on the patient. (Figure 4)
6. After verifying proper fit, remove the trial fitted bands and clean. (Figure 5)

Note: To avoid getting cement in the buccal tube, apply lip balm or cement guard to the mesial and distal ends.

Cementing

7. Isolate teeth to be banded. For lower bandings, also place cotton rolls between the tongue and molars. (Figure 6)
8. Insert a disposable dispensing tip securely onto the syringe. (Figure 7)
9. Load the band with adhesive and seat on the tooth. (Figure 8)

Note: Ensure a new tip is used for each patient to avoid cross-contamination.

Light Curing

10. Remove excess Transbond Plus Light Cure Band Adhesive with a scaler. (Figure 9)
 11. Light cure the Transbond Plus Light Cure Band Adhesive by placing the light guide 1-2 mm above the occlusal surface of the tooth being banded and curing the material for 30 seconds. (Figure 10)
- Note:** The above mentioned curing time applies only to Ortholux™ and Ortholux XT Visible Light Curing Units. Other curing lights, depending on their output intensity might require longer curing times.
12. Remove and discard used dispensing tip. Replace storage cap on the Transbond Plus syringe (Figure 11). Place Adhesive in refrigerator with 2°C-7°C/35°F-45°F.

Storage and Use:

1. Bring to room temperature prior to use.
2. Do not expose to elevated temperatures or intense light.
3. Do not store material in proximity to eugenol-containing products as this may inhibit the proper adhesive curing.
4. This system is designed to be used at room temperature (68° F - 77° F, 20° C - 25° C). Storage between 2°C-7°C/35°F-45°F Light Sensitive

Orthodontic Products

Transbond Plus Light Cure Band Adhesive

5. Shelf-life with refrigeration is per the expiration date on the kit box. Rotate inventory to optimize shelf life.

Shelf life

Shelf life at recommended storage temperature is 24 months from the date of manufacture.

Contents

Transbond Plus Light Cure Band Adhesive Kit:

- Syringe (5);
- Disposable tip (2 boxes of 25 pcs each);

Transbond Plus Light Cure Band Adhesive Introductory Kit:

- Syringe (1);
- Disposable tip (10 pcs);

Packaging

In Kit: Syringe: length: 12.9 cm $\pm$ 0.3 cm.

Secondary packaging-carton box: material: cardboard, dimensions: 16.9cm x 19cm x 1.9cm.

In Starter Kit: Syringe: length: 12.9 cm $\pm$ 0.3 cm.

Secondary package - package: material: low density polyethylene, dimensions: 23.5 $\pm$ 1.3 cm x 12.7 $\pm$ 0.6 cm

Medicinal substances. The product does not use medicinal substances.

Biological safety. The product does not use materials of animal or human origin.

Requirements to the medical device maintenance and repair: Not applicable.

Disinfection and cleaning: Not applicable.

Labelling. The products are supplied in a packaging with the labelling containing the following information: name, lot and reference number; name and address of the manufacturer; contents; shelf life; and symbols - Refer to Instructions for Use, EU compliance, storage / transportation temperature limitation, store away from sunlight, do not reuse, recyclable packaging, low density polyethylene packaging.

Disposal. Packaging/contents should be disposed in accordance with the local/regional/national/international regulations.

Medical devices should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class B).

Warranty

3M Company warrants this product will be free from defects in material and manufacture. 3M UNITEK MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Company's sole obligation shall be repair or replacement of the 3M Company product.


Name and address of the manufacturer: 3M Unitek Corporation, 2724 South Peck Road, Monrovia, California, 91016, USA

Authorized representative on the territory of the Russian Federation:

ZAO "3M Russia" Russia 121614, Moscow, Krylatskaya str. bldg.17/3. Tel.:+7 (495) 784 7474 Fax: +7 (495) 784 7475. 3mra@mmm.com

Orthodontic Products

Transbond Plus Light Cure Band Adhesive

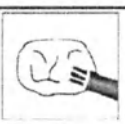- 1**
Remove separators and scale gingival margins 
- 2**
Prophy and rinse teeth to be banded 
- 3**
Air dry thoroughly 
- 4**
Trial fit each band 
- 5**
Remove trial fitted bands and clean 
- 6**
Isolate teeth to be banded 
- 7**
Insert disposable tip securely onto the syringe 
- 8**
Load the band with adhesive 
- 9**
Remove excess adhesive with a scaler 
- 10**
Light cure adhesive for 30 seconds 
- 11**
Discard used dispensing tip. 

Instruction for use

Indication: 3M Unitek products are intended for use in orthodontic treatment of malocclusions.

Intended for use: Unite™ Bonding Adhesive is indicated for bonding orthodontic appliances to teeth for orthodontic treatment.

Scope of use

Orthodontics. Only for professional use.

Contraindications

No known contraindications.

Warning: Acrylate monomers are known to produce allergic skin reactions in certain sensitive individuals. May cause eye and skin irritation.

Precautions: Avoid eye and skin contact. Wear gloves when handling this material.

First Aid:

Eye contact: Immediately flush with plenty of water. See a physician if irritation persists.

Skin contact: Wash affected area with soap and water. See a physician if irritation persists.

Tooth Preparation:

- 1) Prophyl teeth with an oil-free pumice or paste.
- 2) Rinse with water.
- 3) Air dry thoroughly and maintain a dry field. Isolate teeth with cotton rolls.
- 4) Apply Unitek™ Etching Gel (REF 712-039) or (REF 712-044). See manufacturer's instructions for etching gel or liquid before applying.
 - Etch for 15 seconds.
- 5) Rinse thoroughly with water and suction. Do not allow patient to rinse. Thoroughly dry etched surface with oil-and-moisture-free air. Surface should appear frosty white. If not, repeat etch process.
- 6) Option 1: Priming with Unite Primer
 - Place small amount of primer on mixing pad.
 - Brush a thin uniform amount of primer on each tooth surface to be bonded.
 - Brush a thin uniform amount of primer on each bracket base to be bonded.Option 2: Priming with Transbond™ MIP Moisture Insensitive Primer and Unite Primer
 - Dispense three drops of Transbond™ MIP Moisture Insensitive Primer in well.
 - Dip and completely saturate brush tip.
 - Apply one liberal coat of Transbond MIP primer, covering entire etched surface of tooth. Avoid touching the gingival area.
 - Redip brush in primer for each tooth to be primed.
 - Gently blow air on each tooth for 2-5 seconds, aiming the air stream perpendicular to the labial surface of the tooth.
 - Place a small amount of Unite primer on mixing pad.
 - Brush a thin uniform amount of primer on each bracket base to be bonded.

Note: If bonding is delayed and moisture contamination occurs after the application of primer to the tooth surface, apply one more fresh coat of primer over the contaminated area, blow air for 2-5 seconds and bond immediately.

Adhesive Bonding (Syringe):

- 7) Syringe small amount of Unite adhesive paste onto primed bracket base. Use sparingly. When completed express a small amount of adhesive from tip, wipe clean, replace tip cap.

Bracket Placement:

Orthodontic Products

Unite Bonding Adhesive

- 8) Place bracket on tooth and adjust within 20 seconds. Press bracket against tooth surface to secure bonding. Remove flash – do not move bracket. Hold in final position for 5-10 seconds. Do not reposition.

Archwire Placement:

- 9) Allow adhesive to set 4 minutes before inserting archwire.

NOTE: With Dyna-Lock™ Brackets, allow 8-10 minutes before inserting archwire.

Storage and Use

1. Refrigerate when not in use. Storage between 2°C-7°C/35°F-45°F. Do not freeze.
2. Do not expose materials to elevated temperatures or intense light. Bring to room temperature prior to use.
3. This system is designed to be used at room temperature (68°F-77°F, 20°C-25°C).

Debonding:

Metal Brackets: Use Unitek™ Debonding Instrument (REF 444-761).

Ceramic Brackets: Follow bracket manufacturer instructions.

Plastic Brackets: Follow bracket manufacturer instructions.

Shelf life

Shelf life at recommended storage temperature is 24 months from the date of manufacture.

Contents

Unite Bonding Adhesive Kit:

- Syringe (3 pcs);
- Primer;
- Brush holder (1);
- Mixing pad (1);
- Brush tips (10 pcs);

Unite Bonding Adhesive in a separate package (3 pcs per pack)

Unite Bonding Adhesive Primer in a separate package

Packaging

In Kit: Syringe with adhesive: Length 11cm ± 0.07 cm, Width 2.9cm ± 0.005 cm.

Secondary packaging - carton box: material: cardboard, dimensions: 21.59 cm x 2.54 cm x 16.98 cm.

In a separate package (3 pcs per pack): Syringe with adhesive: Length 11 cm ± 0.07 cm. Width 2.9 cm ± 0.005cm.

Secondary package - package: material: low density polyethylene, dimensions: 20.32 ± 0.3 cm x 15.24 ± 0.3 cm.

In a separate package (Primer): Primer Bottle: Height without cap 6.4 cm ± 0.05 cm. Base width 2.4 cm ± 0.05 cm. Top width 1.47cm ± 0.01 cm. Cap height 1cm ± 0.05cm, Thread pitch on the cap 15-415 ST.

Secondary package - package: material: low density polyethylene, dimensions: 12.7 ± 0.3 cm x 12.7 ± 0.3 cm.

Medicinal substances. The product does not use medicinal substances.

Biological safety. The product does not use materials of animal or human origin.

Requirements to the medical device maintenance and repair: Not applicable.

Disinfection and cleaning: Not applicable.

Orthodontic Products

Unite Bonding Adhesive

Labelling. The products are supplied in a packaging with the labelling containing the following information: name, lot and reference number; name and address of the manufacturer; contents; shelf life; and symbols - Refer to Instructions for Use, EU compliance, storage / transportation temperature limitation, store away from sunlight, do not reuse, recyclable packaging, low density polyethylene packaging.

Disposal. Packaging/contents should be disposed in accordance with the local/regional/national/international regulations.

Medical devices should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class B).

Warranty

3M Company warrants this product will be free from defects in material and manufacture. 3M UNITEK MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Company's sole obligation shall be repair or replacement of the 3M Company product.

Name and address of the manufacturer: 3M Unitek Corporation, 2724 South Peck Road, Monrovia, California, 91016, USA

Authorized representative on the territory of the Russian Federation:

ZAO "3M Russia" Russia 121614, Moscow, Krylatskaya str. bldg.17/3. Tel.: +7 (495) 784 7474
Fax: +7 (495) 784 7475. 3mra@mmm.com


Orthodontic Products

Unite Bonding Adhesive


1
Prophy teeth,
rinse with
water.

2
Air dry
thoroughly.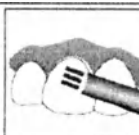

3
Etch
enamel for
15 seconds.

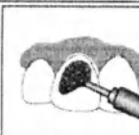
4
Rinse with
water and air
dry thoroughly.

Option 1

5
Apply Unite primer
to tooth surface and
bracket base.

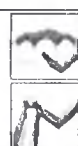Option 2

5A
Apply Transbond MIP
primer to tooth surface.
Apply Unite primer to
bracket base.

6
Apply
adhesive to
bracket base
over primer.


7
Place
adhesive on
mixing pad.

8
Seat and
position
bracket on
the tooth.

9
Wait
4 minutes.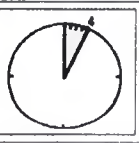

Orthodontic Products

Sondhi Rapid-Set Indirect Bonding Adhesive

Instructions For Use

Warning: Acrylate monomers are known to produce allergic skin reactions in certain sensitive individuals. May cause eye and skin irritation.

Precautions: Avoid eye and skin contact. Wear gloves when handling this material.

First aid:

Eye contact: Immediately flush with plenty of water. See a physician if irritation persists.

Skin contact: Wash affected area with soap and water. See a physician if irritation persists.

Precaution: Care must be taken when bonding to porcelain crowns or facings as debonding may cause chipping, delamination or breakage of the crown. Do not bond to porcelain crowns that have thin facings or that appear to be compromised. Prepare the porcelain crown to be bonded by using a porcelain primer such as 3M's Scotchbond™ Ceramic Primer. Follow instructions included with the porcelain primer.

Indication: 3M Unitek products are intended for use in orthodontic treatment of malocclusions.

Intended use: Sondhi™ Rapid-Set Indirect Bonding Adhesive is intended for indirect bonding of orthodontic brackets. The quick set time of the adhesive does not allow sufficient time for standard direct bonding techniques.

Scope of use

Orthodontics. Only for professional use.

Contraindications

No known contraindications.

Indirect Tray Preparation:


Follow your current laboratory procedure for preparing a stone model, positioning and bonding brackets onto the model, forming a custom resin base and preparing the indirect tray.

Note: Indirect Bonding Using APC™ PLUS Adhesive

- Because of the hydrophilic nature of the APC™ PLUS Adhesive, best results in indirect bonding will be achieved when the stone model is thoroughly dried before placing the APC PLUS adhesive coated brackets. After pouring the ortho stone into the impression, the stone model should be dried overnight in an oven at 110°F (43°C) or under ambient conditions.
- After removing the indirect bonding tray from the stone model, rinsing and drying, you may observe a cloudy white layer on the custom resin base. This white color is caused by a very thin surface layer of APC PLUS adhesive resin. Subsequent microetching or cleaning (e.g. using a toothbrush) of the custom resin base will ensure adequate bond strength.
- It is strongly recommended that a secondary cure of the custom resin bases be performed by directly exposing the tray to a light source to achieve a complete cure.

Tooth Preparation


1. Prophyl teeth with an oil-free pumice or paste (Figure 1).
2. Rinse with water.
3. Isolate teeth with cotton rolls.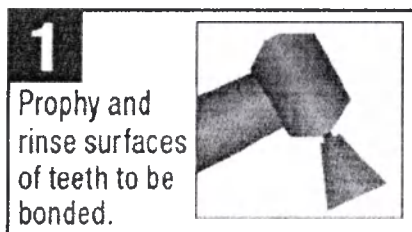

Orthodontic Products


Sondhi Rapid-Set Indirect Bonding Adhesive

4. Air dry thoroughly using oil and moisture-free air source. (Figure 2)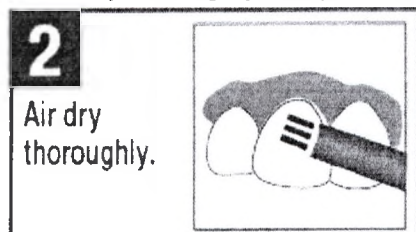


Acid Etching


5. Apply Unitek™ Etching Gel (REF 712-039) to teeth surfaces following the instructions provided with the etching gel delivery system. If using other gel etching systems, consult the manufacturer's instructions for proper technique and recommended etching times. (Figure 3)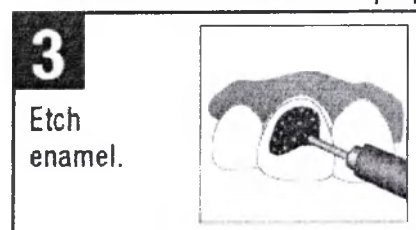

6. Rinse with water. (Figure 4)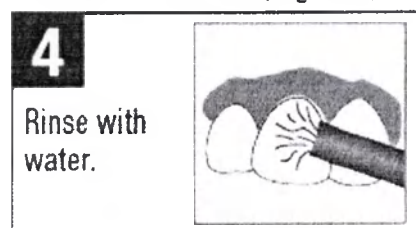

7. Air dry thoroughly. (Figure 5)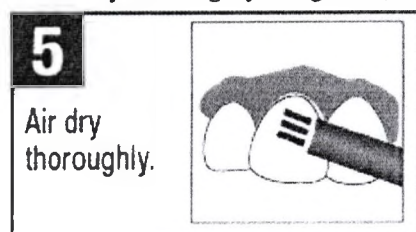


Adhesive Bonding

Optional: If using Transbond™ MIP Moisture Insensitive Primer (REF 712-021), include the following steps.

8. Dispense three drops of Transbond™ MIP Primer in well. (Figure A)


Optional Transbond MIP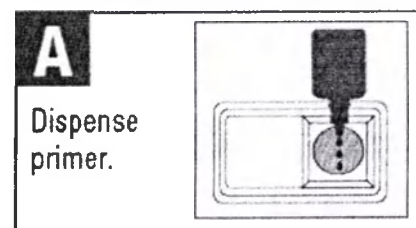


Orthodontic Products

Sondhi Rapid-Set Indirect Bonding Adhesive

9. Dip and completely saturate brush tip. (Figure B)

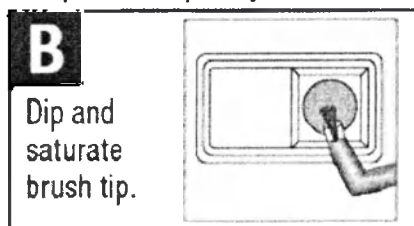
10. Apply one liberal coat of Transbond MIP primer, covering entire etched surface of tooth. Avoid touching the gingival area. (Figure C)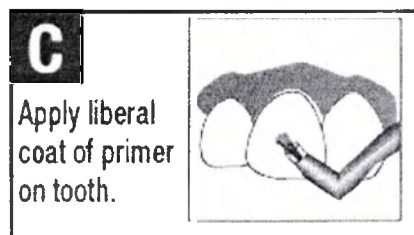


11. Re-dip brush in primer for each tooth to be primed. (Figure B)


12. Gently blow air on each tooth for 2-5 seconds, aiming the air stream perpendicular to the labial surface of the tooth. (Figure D) Note: If the application of Resin A is delayed and moisture contamination occurs after the application of primer to the tooth surface, apply one more fresh coat of primer over the contaminated area, blow air for 2-5 seconds and start the bonding procedure immediately.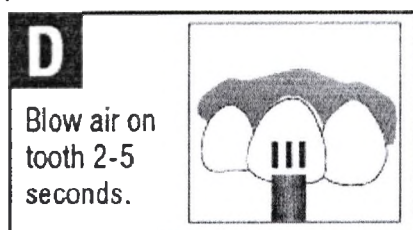

13. Dispense 2 to 3 drops of Resin A into one side of the mixing well and 2 to 3 drops of Resin B into the other side of the mixing well. Care should be taken not to mix the two liquids together. (Figure 6)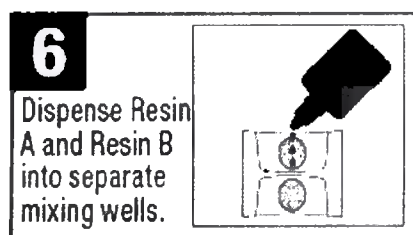


14. Apply a thin coat of Resin B to each custom resin base in the transfer tray. Use a brush insert with the orange applicator handle. (Figure 7)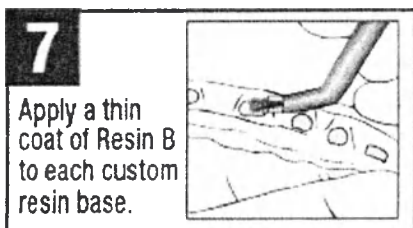

Orthodontic Products

Sondhi Rapid-Set Indirect Bonding Adhesive

15. Apply a thin coat of Resin A to the tooth surface. (Figure 8A Labial Bonding, Figure 8B Lingual Bonding) Note: A small amount of Resin A and Resin B is sufficient for successful bonding. Excess resin can be scaled off from around the brackets after removing the trays.

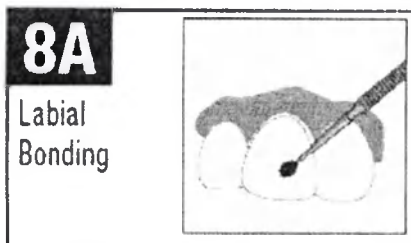
Caution: Do not allow Resin A and Resin B to make contact until ready to seat the indirect tray in the patient's mouth. In order to reduce clean-up issues apply a very thin layer of Resin B to the bracket bases.


16. The tray is then seated and held in place for 30 seconds. (Figure 9A Labial Bonding, Figure

17. The tray can be removed in two minutes. Remove the tray using a scaler to peel the tray from the lingual to buccal. Use extreme care when removing the tray from around the bracket wings and hooks. (Figure 10)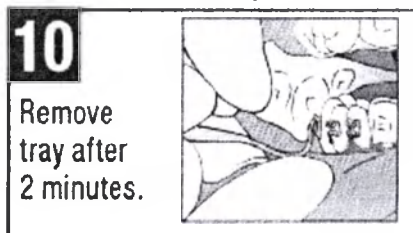

18. Scale the excess resin from around the brackets and from the interproximal contacts. Use dental floss to check that all contacts are open. (Figure 11)

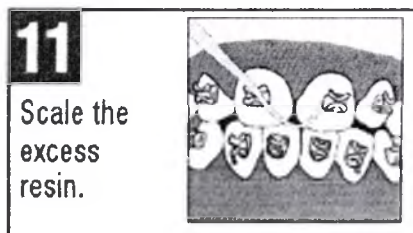19. Archwires can be immediately tied in.

Storage and Use

1. Refrigerate when not in use.
2. Do not expose materials to elevated temperatures or intense light. Bring to room temperature prior to use.
3. This system is designed to be used at room temperature (68°F-77°F, 20°C-25°C). Storage between 2°C-7°C/ 35°F-45°F.

Orthodontic Products

Sondhi Rapid-Set Indirect Bonding Adhesive

4. Shelf-life is per the expiration date on the kit box. Rotate inventory to optimize shelf life.

Shelf life

Shelf life at recommended storage temperature is 24 months from the date of manufacture.

Contents

Sondhi Rapid-Set Indirect Bonding Adhesive Kit:

- Resin Bottle A;
- Resin Bottle B;
- Brush holder (2 pieces) (1 orange and 1 yellow holder);
- Brush tips (60 pcs.);
- 2-cavity dispensing well.

Sondhi Rapid-Set Indirect Bonding Adhesive Resin A Bottle in a separate package.

Sondhi Rapid-Set Indirect Bonding Adhesive Resin B Bottle in a separate package.

Packaging

In Kit: Resin A Bottle: Height 6.2cm $\pm$ 0.1cm, Width 2.7cm $\pm$ 0.03cm, Cap Height 0.4cm $\pm$ 0.02cm, Thread pitch on the cap 2mm, Thread length 13mm.

Resin B Bottle: Height 6.2cm $\pm$ 0.1cm, Width 2.7cm $\pm$ 0.03cm, Cap height 0.4cm $\pm$ 0.02cm, Thread pitch on the cap 2mm, Thread length 13mm.

Secondary package - plastic case: material: polyethylene, dimensions: 21.7 cm x 19.6 cm x 7.9 cm.

In a separate package (Resin A Bottle): Resin A Bottle: Height 6.2cm $\pm$ 0.1cm, Width 2.7cm $\pm$ 0.03cm, Cap Height 0.4cm $\pm$ 0.02cm, Thread pitch on the cap 2mm, Thread Length 13 mm.

Secondary package - package: material: low density polyethylene, dimensions: 23.5 $\pm$ 1.3 cm x 12.7 $\pm$ 0.6 cm.

In a separate package (Resin B Bottle): Resin B Bottle: Height 6.2cm $\pm$ 0.1cm, Width 2.7cm $\pm$ 0.03cm, Cap Height 0.4cm $\pm$ 0.02cm, Thread pitch on the cap 2mm, Thread length 13 mm.

Secondary package - package: material: low density polyethylene, dimensions: 23.5 $\pm$ 1.3 cm x 12.7 $\pm$ 0.6 cm.

Medicinal substances. The product does not use medicinal substances.

Biological safety. The product does not use materials of animal or human origin.

Requirements to the medical device maintenance and repair: Not applicable.

Disinfection and cleaning: Not applicable.

Labelling. The products are supplied in a packaging with the labelling containing the following information: name, lot and reference number; name and address of the manufacturer; contents; shelf life; and symbols - Refer to Instructions for Use. EU compliance, storage / transportation temperature limitation, store away from sunlight, do not reuse, recyclable packaging, low density polyethylene packaging.

Disposal. Packaging/contents should be disposed in accordance with the local/regional/national/international regulations.

Medical devices should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class B).

Warranty

3M Company warrants this product will be free from defects in material and manufacture. 3M UNITEK MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is

Orthodontic Products

Sondhi Rapid-Set Indirect Bonding Adhesive

defective within the warranty period, your exclusive remedy and 3M Company's sole obligation shall be repair or replacement of the 3M Company product.

Name and address of the manufacturer: 3M Unitek Corporation, 2724 South Peck Road, Monrovia, California, 91016, USA

Authorized representative on the territory of the Russian Federation:

ZAO "3M Russia" Russia 121614, Moscow, Krylatskaya str. bldg.17/3, Tel.:+7 (495) 784 7474
Fax: +7 (495) 784 7475, 3mra@mmm.com

Orthodontic Products

Wax Pak

Instructions for Use

Wax Pak – orthodontic wax is intended to cover orthodontic devices to prevent oral mucosa irritation.

Indication

3M Unitek products are intended for use in orthodontic treatment of malocclusions.

Contraindication

There is no known contraindication.

Scope of use

Orthodontic. Only for professional use.

Potential side effects

There are no known potential side effects.

Instruction for Use

1. Open wax container.
2. Remove one wax stick from container.
3. Take the required amount of wax. In most cases, wax in size of a pea is enough.
4. Apply wax on bracket or clip-tube. Evenly distribute wax over the elements you wish to isolate.
5. If necessary, the amount of wax can be increased.
6. Close wax container and store in a dry dark place.

Contents: Wax Pak is packaged in a plastic container. Container size: Length 5.2 cm $\pm$ 0.3 cm, Width 3 cm $\pm$ 0.3 cm, Height 1 cm $\pm$ 0.3 cm.

Conditions of use/storage/transportation

This system is designed to be used at room temperature (68°F-77°F, 20°C-25°C). Values of humidity and pressure are not regulated.

Storage conditions: Values of temperature, humidity and pressure are not regulated.

During transportation, the products can be exposed to short-term temperature changes, which do not affect the safety and products specifications. The device can be transported by any kind of transport.

Shelf life at room temperature is 5 years from the date of manufacture.

This medical device is not subject to maintenance and repair.

Drug products: This medical device does not contain drug products or does not introduce medicinal substances.

Human blood derivatives: Medical device does not contain and does not deliver human blood derivatives.

Materials of animal origin: Materials of animal origin are not intentionally added in the product.

Disposal. Packaging/contents should be disposed in accordance with the local/regional/national/ international regulations. Medical devices should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class B).

Warranty. Company 3M warrants this product will be free from defects in material and manufacture. COMPANY 3M MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Company 3M's sole obligation shall be replacement of the product.

Name and address of the manufacturer. 3M Unitek Corporation, 2724 South Peck Road, Monrovia, California, 91016, USA

Authorized representative on the territory of the Russian Federation:

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