State of Minnesota

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SECRETARY OF STATE

APOSTILLE

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1. Country: United States of America

This Public Document: 3M Comply 1251 Chemical Indicator Strip

2. has been signed by Irma Vargas

3. acting in the capacity of Notary Public, State of Minnesota

4. bears the seal/stamp of Irma Vargas, Notary Public, State of Minnesota

CERTIFIED

5. at: St. Paul, Minnesota

6. Date: May 23, 2017

7. by: Secretary of State, State of Minnesota

8. File No: 9515947-4

9. Seal/Stamp:

10. Signature:

Steve Simon

Steve Simon
Secretary of State

3M Health Care Business

3M Center

2510 Conway Ave., Bldg. 275-5W-06


St. Paul, MN 55144 U.S.A.

651 733 1110

3M

To whom it may concern


In my capacity as International Regulatory Affairs Associate, 3M Health Care Business, St. Paul, MN, USA, I certify that the attached Instruction for Use (IFU) for 3M™ Comply™ EO 1251 Chemical Indicator Strip in English is truthful and accurate.

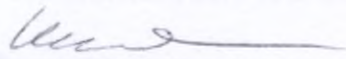
Regina Feferman
International Regulatory Affairs Associate
3M Health Care

5/22/2017
Date

State of Minnesota)
County of Ramsey) SS

Subscribed and sworn to before me this 22nd day of May 2017.
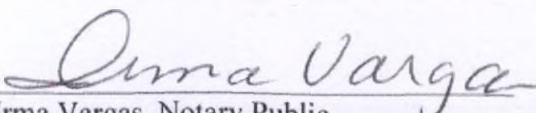
Irma Vargas, Notary Public
My Commission Expires: January 31, 2020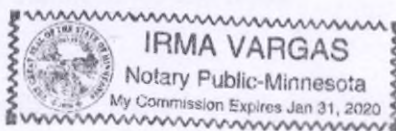

3M™ Comply™ 1251 Chemical Indicator Strip

Description: The 3M™ Comply™ 1251 EO Chemical Indicator Strip is a paper strip 1.5 cm wide by 20 cm long (0.625 inches wide by 8 inches long) printed with a chemical indicator ink that turns from red to green when exposed to the critical parameters of the ethylene oxide (EO) sterilization process (i.e., gas concentration, RH, time and temperature) It is designed to indicate whether EO sterilization conditions have been met at the point of placement of the strip, which should be placed in the area determined to be the least accessible to EO penetration. Each strip is perforated at the center if a shorter internal chemical indicator is preferred. An internal chemical indicator should be used inside each package to be sterilized.

Indications for Use

The Comply™ 1251 EO Chemical Indicator Strip is designed to monitor all warm and cool cycles of 100% EO and EO/HCFC (hydrochlorofluorocarbon) gas mixture sterilizer cycles.

Contraindications

None.

Potential complications

No potential complications have been identified for this product.

Precautions

Do not use Comply™ 1251 EO Chemical Indicator Strip to monitor steam dry heat EO/carbon dioxide mixtures or other low temperature sterilization processes.

Directions for Use

- Place a Comply™ 1251 EO Chemical Indicator Strip inside each pack, peel pouch container system or tray to be EO sterilized in the area determined to be the least accessible to EO penetration.
- After processing, the red chemical indicator line will turn green when exposed to ethylene oxide.

Storage/Shelf Life

- Best stored in a dry (<50% relative humidity) condition at room temperature (15-30°C [59-86°F]) and protected from light. Do not store near strong alkaline or acidic products such as cleaning/disinfection agents.
- This medical device isn't subject to maintenance and repair.

Utilization

Dispose of the device in accordance with applicable local/regional/national/international regulations. Medical devices in Russian Federation must be disposed of in the local public entities in accordance with SanPiN 2.1.7.2790-10 "Sanitary-epidemiological requirements for the handling of medical waste" to the waste class B).

Transport conditions

Shelf life of the products is guaranteed as labeled, if stored according to the recommended storage conditions. During transportation the products may experience short term temperature variations without affecting product safety and performance.

Transportation of medical device is carried out by all type of transport.

Explanation of Symbols

Caution, see instructions for use

Do not reuse

Use by date

Batch code

Manufacturer

Date of manufacture

Product is designed for use with ethylene oxide sterilization cycles.

Catalogue Number

Made in U.S.A. for

3M Health Care

2510 Conway Ave.

St. Paul, MN 55144 U.S.A.

1-800-228-3957

3M.com/infectionprevention

3M and Comply are trademarks of 3M.

Used under license in Canada.

© 2016, 3M. All rights reserved

Authorised representative of the
manufacturer in Russia:

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