3M

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certified copy

Carl-Schurz-Straße 1
41453 Neuss
Germany

Postal Address:
ESPE Platz
82229 Seefeld
Germany

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+49 (0)8152 700-0
+49 (0)8152 700-1366
Internet: www.3MESPE.com
E-Mail: info3MESPE@3M.com
WEEE-Reg.-Nr. DE 36963167
VAT-ID: DE 120679179

To Whom It May Concern

**Registration documents for Express™ Ultra-Light Body
- Information for Use (IFU) -**

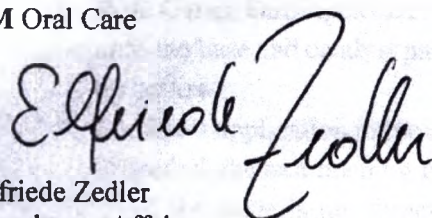
We, 3M Deutschland GmbH, Carl-Schurz-Str. 1, 41453 Neuss, herewith confirm that the following pages are an excerpt of the Information for Use.

Seefeld, April 7, 2017

Yours Sincerely,

3M Deutschland GmbH
3M Oral Care

Elfriede Zedler
Regulatory Affairs

Product Description

Express™ Ultra-Light Body is a hydrophilic silicone addition polymerization material of light-bodied consistency (ISO 4823 Type 3) in the Garant™ cartridge. Mixing ratio is 1 volume base paste:1 volume catalyst.

Not all products are available in all countries.

- For details on all products mentioned, please refer to the corresponding Instructions for Use. Instructions for Use should not be discarded for the duration of product use.

Indications

- All precision impressions (e.g. of crown, bridge, inlay, and onlay preparations)

Precautionary Measures

3M MSDSs can be obtained at www.mmm.com or from your local subsidiary.

Preparation

Impression Trays:

Particularly suitable are rigid unperforated and perforated metal trays.

- For sufficient adhesion, apply a thin layer of VPS Tray Adhesive to the tray and allow to dry completely (at least 5 min, 15 min drying time is ideal).

Garant Dispenser/Cartridge:

- Place the Express Ultra-Light Body in the Garant dispenser, and check shortly before application to ensure that both Garant cartridge openings are not clogged.
- With Garant cartridges that have not been previously used, extrude a small amount of paste until the base and catalyst paste flow out evenly. Subsequently, attach a new Garant mixing tip yellow.
- For direct application in the mouth, attach a Garant intraoral tip yellow onto the mixing tip. If needed, the exit opening for the intraoral tip can be enlarged with a scalpel.
If the paste is not directly applied using the Garant dispenser, a separate Elastomer Syringe can be filled directly from the Garant mixing tip (without an intraoral tip)
- Material that has set inside the Garant mixing tip should not be extruded by force as this may damage the Garant cartridge and Garant mixing tip and result in the formation of leaks.

Retraction

Retraction cords as well as solutions and pastes based on aluminum chloride, aluminum sulfate, or iron sulfate are suitable retraction agents. Recommendation: 3M ESPE Astringent Retraction Paste/Retraction Capsule. The paste containing aluminum chloride is directly applied from the retraction capsule into the sulcus. The paste can be used as an alternative to, or in combination with, retraction cords or other retraction methods, depending on the clinical situation and the practitioner's application technique.

- Areas from which impressions are to be taken should be kept dry.
- Prior to taking the impression, completely remove all residual retraction agents by rinsing and drying.

Dosing and Mixing

Express Ultra-Light Body is dosed and statically mixed in the Garant dispenser.

Times

	Processing time from start of mixing*	Intraoral syringe time	Residence time mouth
	min:sec	min:sec	min:sec
Express Ultra-Light Body	1:30	0:40	2:30

These are the processing times at 23°C/74°F. Higher temperatures shorten, and lower temperatures prolong the overall processing time.

* Start of mixing = Entry of paste into the Penta or Garant mixing tip

Impression Taking

- If necessary for removal of the set impression from the mouth appropriately block-out deep undercuts in interdental areas.

Two-Step Impressions:

- Prepare preliminary impression using Express XT Penta Putty (see IFU for Express XT Penta Putty)
- Place Express Ultra-Light Body in the appropriate locations in the preliminary impression. If needed, apply material around the prepared teeth, keeping the mixing tip constantly immersed in the paste to avoid air bubble formation. Stay within the given processing times.
- Replace the preliminary impression in the mouth.
- After the residence time in the mouth, remove the impression from the mouth.

One-Step Technique:

- Load the impression tray prepared with adhesive with Express XT Penta Putty (see IFU of Express XT Penta Putty)
- While the assistant is loading the tray, the dentist applies the Express Ultra-Light Body material around the dried preparation from the bottom up. Stay within the given processing times.
- Slowly place the loaded tray in the mouth parallel to the long axes of the prepared teeth before the lapse of the processing time and hold the tray in position without applying pressure.
 - The tray should be positioned in the mouth within 1:15 from start of tray filling.
- Remove the impression from the mouth, once the residence time in the mouth has elapsed.

After Impression Taking

- Thoroughly examine and explore the sulcus of the prepared teeth and surrounding dentition. Remove any residual cured impression material from the mouth.

Hygiene

- Place the impression in a standard disinfectant solution for the period of time recommended by the manufacturer. Excessive disinfection may damage the impression.
- After disinfection, rinse the impression under running water for approx. 15 seconds.

Model Preparation

- In order to obtain a bubble-free model, rinse the impression briefly and dry before pouring, or use a silicone surfactant.
- Prepare a cast from the impression using a commercial specialized stone plaster no earlier than 2 hours after impression taking. There are no other time limitations.

Cleaning

- Garant Dispenser: Remove paste that has not set using an alcohol- soaked cloth. The dispenser handle and plunger can be autoclaved up to a temperature of 135° C/275° F. Disassemble the dispenser before autoclaving. Glutaraldehyde-based solutions can be used for disinfection.
- Impression Tray: The adhesive can be removed from impression trays using acetone.

Notes

- The oxygen inhibition layer of composite materials, e.g. in fillings or core buildups, may impair the setting of silicone impression materials, and should therefore be removed completely.
- Keep the filled Garant mixing tip on top of the cartridge to serve as a closure until the next use. Removing and replacing the spent Garant mixing tip may lead to carry-over of paste and ensuing formation of clogs.
- Vinyl polysiloxane impression materials must not be combined with condensation polymerization silicones or polyether impression materials. Even trace amounts impair the setting process.
- Disposable latex gloves impair the setting of silicone impression materials. Vinyl gloves are more suitable.
- In no case should the impressions come into contact with solvent-containing liquids, as this may result in swelling and imprecise modelling.

Storage and Stability

Store the product at 15-25° C/59-77° F.

Do not use after the expiration date.

Store impressions at temperatures below 30° C/86° F.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and workmanship. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Information valid as of April 2017

RUS: Authorised representative of the manufacturer in RF:

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