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State of Minnesota

# SECRETARY OF STATE

## APOSTILLE

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1. Country: United States of America

This Public Document: Documents for 3M Unitek

2. has been signed by Angela Maxcine Draper

3. acting in the capacity of Notary Public, State of Minnesota

4. bears the seal/stamp of Angela Maxcine Draper, Notary Public, State of Minnesota

## CERTIFIED

5. at: St. Paul, Minnesota

6. Date: July 19, 2017

7. by: Secretary of State, State of Minnesota

8. File No: 9580763-7

9. Seal/Stamp:

10. Signature:

*Steve Simon*

Steve Simon  
Secretary of State

To Whom It May Concern:

In my capacity as Regulatory Affairs Manager, 3M Unitek, I certify that the attached English version of the Instructions for Use (IFU) for Ortholux Luminous Curing Light intended for Russia is truthful and accurate.

Sincerely,

Mark Burville  
Regulatory Affairs Manager  
3M Unitek

18 JUL 2017  
Date

STATE OF MINNESOTA)  
COUNTY OF RAMSEY ) ss

On July 18, 2017, before me, a Notary Public within and for said county, personally appeared Mark Burville, known by me to be the person described in and who executed the foregoing instrument and acknowledged that he executed the same as his own free act and deed.

Angela Maxcine Draper, Notary Public  
My Commission Expires: January 31, 2019

## **Ortholux™ Luminous Curing Light**

### **Safety**

Prior to installation and start-up of the unit, please read these instructions carefully!

As with all electrical devices, the proper function and safe operation of this unit depend on the user's compliance with the standard safety procedures as well as the specific safety recommendations presented in these Instructions for Use.

1. Use of the device is restricted to trained personnel in accordance with the instructions below. The manufacturer assumes no liability for any damage resulting from the use of this unit for any other purpose.
2. Prior to start-up of the unit, ensure that the operating voltage stated on the rating plate is compatible with the available main voltage. Operation of the unit at a different voltage may damage the unit.
3. Position the unit so that the power plug is accessible at all times. The power plug is used to turn the unit on and off. To disconnect the charger from the main voltage, remove the power plug from the electrical outlet.
4. Use supplied 3M Unitek charger. The use of any other charger can result in damage to the battery.
5. **CAUTION!** The Ortholux™ Luminous Curing Light must be aimed away from the eyes in order to avoid serious health consequences due to irradiation of the eyes. Use the eye shield. Exposure must be restricted to the area of the oral cavity in which clinical treatment is intended.
6. **CAUTION!** In cases where the Ortholux Luminous Curing Light will be used outside of the oral cavity additional eye protection may be needed. The reflected light from curing surfaces when not confined to the oral cavity may be more intense than intra-orally. It is recommended that protective orange glasses be worn by everyone in close proximity of the extraoral curing in addition to the use of the eye-shield on the light.
7. **CAUTION!** The Ortholux Luminous Curing Light generates high intensity light. The light emitted should be aimed directly above the material to be cured - exposure of the soft tissues (gingiva, oral mucosa, and skin) to high-intensity light should be avoided as such exposure may cause damage or irritation. If exposure of soft tissues cannot be avoided, adjust the polymerization process to the light level, e.g., by shortening the polymerization times or increasing the distance between the light guide exit and the material to be cured. When utilizing the light in Extended Mode (15s), do not keep the light in one position. Longer exposure in the region of the pulp can lead to injury or irritation of the pulp, which is why the specified exposure times must not be exceeded.
8. Ortholux Luminous Curing Light may be operated only with the supplied light guide or 3M Unitek replacement light guide. The light guide has to be seen as an applied part. The use of other light guides may result in a reduction or increase in the light intensity. The product's warranty does not cover any damage resulting from the use of third-party light guides.
9. Condensation resulting from the unit being transferred from a cold to a warm environment may be a potential risk. Hence, the unit should be turned on only after it is completely equilibrated to ambient temperature.
10. In order to avoid electric shock do not introduce any objects into the unit with the exception of replacement parts handled in accordance with the Instructions for Use.
11. Use only genuine 3M Unitek parts when replacing defective components as directed in these Instructions for Use. The product's warranty does not cover any damage resulting from the use of third-party replacement parts.
12. Should you have any reason to suspect the safety of the unit to be compromised, the unit must be taken out of operation and labeled accordingly to prevent third parties from inadvertently using a possibly defective unit. Safety may be compromised, e.g., if the unit malfunctions or is noticeably damaged.
13. Keep solvents, flammable liquids, and sources of intense heat away from the unit as they may damage the plastic housing of the charger, the seals, or the cover on the operating buttons.
14. Do not operate the unit in the proximity of flammable mixtures.
15. Do not allow cleaning agents to enter the unit as this may short-circuit the unit or cause potentially dangerous malfunction.
16. Only service centers authorized by 3M Unitek may open the unit housing and repair the device.

17. Ortholux Luminous Curing Light must not be used for patients, or by users, with heart pacemaker implants who have been advised to be cautious with regard to their exposure to small electrical devices.
18. Do not use Ortholux Luminous Curing Light for patients with a history of photobiological reactions (including individuals with urticaria solaris or erythropoietic protoporphyria) or who are currently on photosensitizing medication (including 8-methoxypsoralen or dimethylchlorotetracycline).
19. Individuals with a history of cataract surgery may be particularly sensitive to the exposure to light and should be discouraged from exposure to the Ortholux Luminous Curing Light unless adequate safety measures, such as the use of protective goggles to filter blue light, are undertaken.
20. Individuals with a history of retinal disease should seek advice from their ophthalmologist prior to operating the unit. In operating the Ortholux Luminous Curing Light, this group of individuals must take extreme care and comply with any and all safety precautions (including the use of suitable light-filtering safety goggles).
21. This unit has been developed and tested in accordance with the relevant EMC regulations and standards. It is in conformity with legal requirements. Since various factors such as power supply, wiring, and the ambient conditions at the place of operation can affect the EMC properties of the unit, the possibility that, under unfavorable conditions, there will be EMC disruptions cannot be completely excluded. If you should notice problems in the operation of this or other units, move the unit to a different location.
22. Prior to each use of the unit ensure that the emitted light intensity is sufficient to safely guarantee polymerization. Use the light testing area integrated into the charger for this purpose.

### Glossary of Symbols

Attention, Consult Accompanying Documents

Follow instructions for use.

Type B Equipment - Protection against electric shock

Class II Equipment - double insulated

Establishes that the product conforms to all applicable legal requirements

Battery power level

Icon to identify electric and electronic devices. The unit must be collected and disposed of separately.

Temperature limitation


Humidity limitation

Atmospheric pressure limitation

**Labelling of the medical products contains the following information:**

Products are supplied in a package with labelling containing the following information: product name and reference number; name and address of the manufacturer; contents; LOT number; information about trade mark; symbols of storage/transport conditions – Temperature, humidity and pressure ranges, Equipment type B, CE conformity mark; Ecological package, Recyclable.

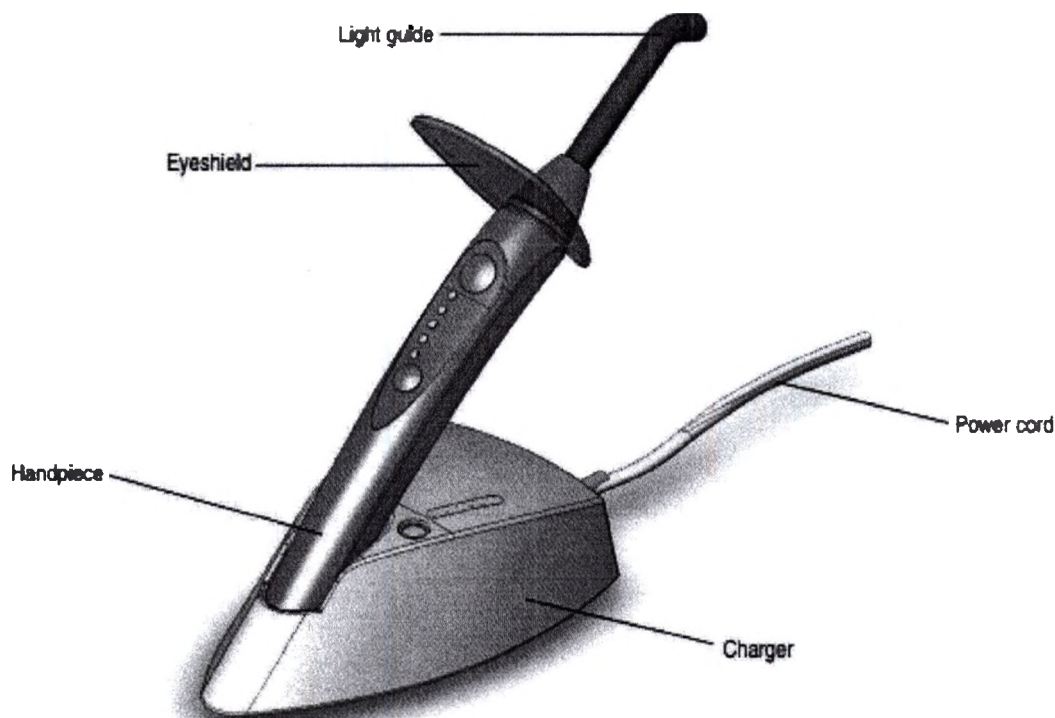**Product Description****Name of product**

Ortholux Luminous Curing Light

1. Ortholux Luminous Curing Light contents:

1. Handpiece.
2. Eye Shield.
3. Battery.
4. Charger.
5. Light Guide.

Ortholux Luminous Curing Light, is a high-performance light source for intraoral polymerization of orthodontic adhesives. It consists of a charger and a cordless handpiece powered by a rechargeable battery. The device is a medical electrical device in accordance with IEC 60601-1 and is available as a tabletop device. Wall mounting is not possible. The light source is a high-performance light-emitting diode (LED). The emitted light covers the light wavelengths between 430 and 480 nm.

Settable exposure times:

- 3, 6, 9, or 12 seconds.
- Extended mode (15 seconds)
- Tack cure mode (1 second)

The charger is equipped with an integrated light intensity testing area.

The unit is shipped with an 8 mm diameter light guide. It is not permissible to use light guides from other manufacturers.

The handpiece is equipped with a sleep mode to minimize the unit's energy consumption. The handpiece switches to sleep mode once it is placed in the charger or if left unused for approximately 5 minutes outside the charger.

NOTE: Do not discard these Instructions for the duration of product use.

### Indication

Ortholux Luminous Curing Light is intended for the polymerization of light-curing orthodontic adhesives with photoinitiators in the wavelength range of 430-480 nm.

### Scope of use:

Orthodontia. Only for professional use.

### Contraindications

Not identified

The high intensity light output of the Ortholux Luminous Curing Light allows for fast curing times. Refer to the table under Recommended Cure Techniques with 3M Unitek Adhesives.

- Though the majority of light-curing orthodontic adhesives are responsive in this range of wavelengths, you may wish to contact the manufacturer of the material in question.
- Ortholux Luminous Curing Light is not intended for use with dental adhesives or other light curing procedures in a dental office.

### Technical Data

| Characteristic                                    | Value                                                                                      |
|---------------------------------------------------|--------------------------------------------------------------------------------------------|
| <b>Charger</b>                                    |                                                                                            |
| Operating voltage                                 | 100-127 V 50/60 Hz or 230 V 50/60 Hz<br>(See rating plate for factory-set voltage.)        |
| Power input                                       | 0.045 A (230 V) 0.1 A (100-127 V)                                                          |
| Length                                            | 170 ±0.7 mm                                                                                |
| Width                                             | 95±0.5 mm                                                                                  |
| Height                                            | 50 ±0.4 mm                                                                                 |
| Weight                                            | 650±2 g                                                                                    |
| Classification                                    | IP II,  |
| <b>Handpiece</b>                                  |                                                                                            |
| Power supply                                      | Lithium-ion battery, nominal voltage 3.7 V                                                 |
| Utilizable wavelength range                       | 430-480 nm                                                                                 |
| Wavelength peak                                   | 455 nm +/- 10 nm                                                                           |
| Light intensity (between 400 and 515 nm)          | 1600 mW/cm <sup>2</sup> ± 10% (independent of battery power level)                         |
| Optically active light emission area              | Usually 42 mm <sup>2</sup> (optically active)                                              |
| Intermittent operation                            | Typically 7 minutes at a room temperature of 23°C                                          |
| Total exposure time (new, fully charged battery)  | 60 min                                                                                     |
| Diameter                                          | 28±0.1/-0.2 mm                                                                             |
| Length                                            | 270±2.4 mm                                                                                 |
| Weight                                            | 250 g (including Light Guide)                                                              |
| Total height of Handpiece inserted in the Charger | 180 mm                                                                                     |
| <b>Eye Shield</b>                                 |                                                                                            |
| Dimensions                                        | (64.4±0.38*89.3±0.44*3.2±0.2) mm                                                           |
| <b>Battery</b>                                    |                                                                                            |
| Dimensions                                        | (84.5±0.44*Ø21.9±0.25) mm                                                                  |
| <b>Light Guide</b>                                |                                                                                            |
| Dimensions                                        | (82.2±2*21±2*Ø8±0.25) mm                                                                   |

### Operating conditions

Temperature: 10 °C – 40 °C  
Relative humidity: 30%-75%  
Atmospheric pressure: 700 -1060 hPa

#### **Transport and Storage Conditions:**

##### **Ambient temperature range:**

-20°C to 40°C (-4°F to 104°F)

##### **Relative humidity:**

30% to 75%

##### **Atmospheric pressure:**

700 hPa to 1060 hPa

Ortholux Luminous Curing Light can be transported by all types of transport.

#### **Installation of the Unit**

##### **Factory Settings**

The factory settings of the unit are as follows:

When first turned on the unit is in Extended Mode, i.e. only the Power level LED is lit on the hand piece.

#### **Initial Steps**

##### **Charger**

1. Please ensure first that the voltage stated on the rating plate corresponds to the existing main supply voltage. The rating plate is attached to the bottom of the charger.
2. Place the charger on a level surface.
3. Connect the power cable of the charger to the power supply.


The green LED on the charger showing that the unit is ready for operation comes on. This shows that the unit is ready for operation; please refer to the section "Operating Status Display of the Charger."

##### **Light Guide/Handpiece**

1. Never place the handpiece in the charger without the battery inserted in the handpiece!
2. Place the eye shield on the front of the unit.
3. Autoclave the light guide prior to first use.
4. Insert the light guide firmly into the handpiece.

##### **Removing and Inserting the Light Guide from/into the Handpiece**


- The light guide has a magnetic holder. Remove the light guide from the handpiece by pulling it towards the front.
- Attach the light guide to the handpiece until it is firmly in place.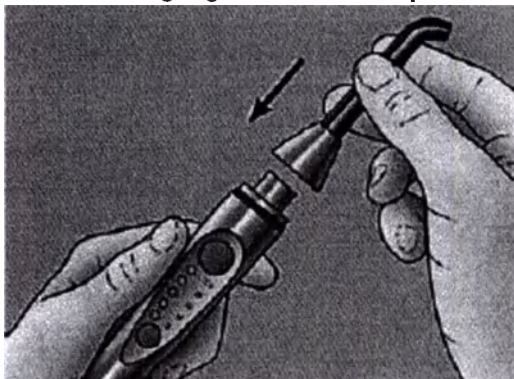

##### **Insert the battery**

1. Remove the protective cap from the battery and keep with the unit packaging.

2. Slowly insert the battery with the thread side towards the metal housing into the handpiece until it stops. Screw the battery clockwise by hand until the seal is pressed firmly against the metal housing. The unit will not function properly if the battery is not screwed in completely!
3. If the unit does not function properly, remove the battery and reinsert it as described above.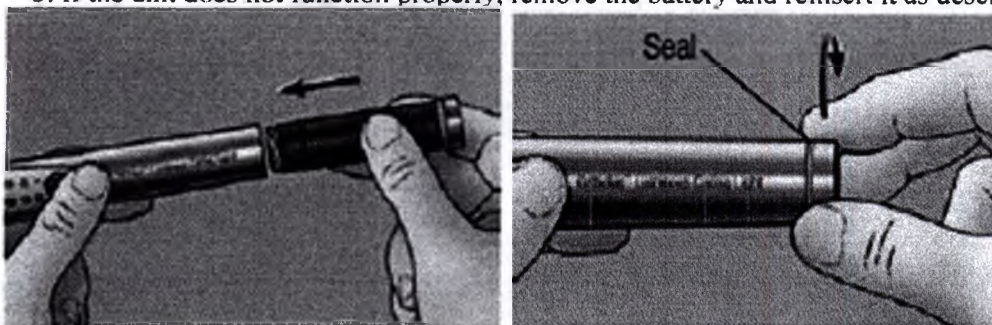


### Battery Charging

1. The unit contains a powerful lithium-ion rechargeable battery. The battery does not have any memory effect and can therefore be recharged at any time by placing it in the charger (see "Power Level Display of the Handpiece" below).
  2. Alternatively, the battery in the unit can be replaced by a spare battery (available separately) which has been charged.
  3. Place the handpiece on the charger for approximately 1.5 hours before the first use, in order to charge the new battery completely for the first time.
- The status LED showing the operating status of the charger flashes green during the charging operation; see below "Operating Status Display of the Charger."


### Operating Status Display of the Charger

| Status LED showing operating status | Operating status                      |                                  |
|-------------------------------------|---------------------------------------|----------------------------------|
|                                     | Handpiece/battery outside the charger | Handpiece/battery in the charger |
| Steady green light                  | Charger ready for operation           | Charging has been completed      |
| Flashes green                       | n/a                                   | Battery is being charged         |
| Steady red light                    | Charge contact pins are wet           | Charge contact pins are wet      |
| Flashes alternately red and green   | Malfunction in the charger            | Problem during charging          |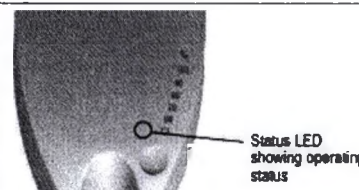


### Power Level Display of the Handpiece

| Power Level LED    | Operating status                                                                               |                                                                |
|--------------------|------------------------------------------------------------------------------------------------|----------------------------------------------------------------|
|                    | Handpiece outside the charger                                                                  | Handpiece in the charger                                       |
| Steady green light | Battery is charged, handpiece ready for operation                                              | Not possible, handpiece is in sleep mode                       |
| Steady red light   | Warning for low battery capacity, remaining capacity adequate typically for 20 3-second cycles | Not possible, handpiece is in sleep mode                       |
| Flashes red        | Battery fully discharged, exposure cycle will be completed or, if in extended mode, stopped    | Problem in charging, battery is defective or cannot be charged |

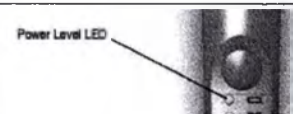## Operation
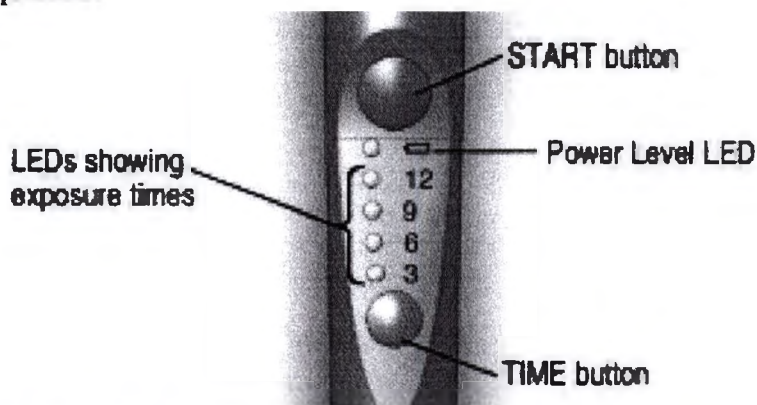
### Activating the Handpiece

If the Power Level LED is off, briefly press the START button to activate the hand piece. The Power Level LED will come on and a short audible signal is emitted. The hand piece is now ready for operation; the handpiece displays the latest set exposure time setting.

### Selection of Exposure Time

Press the TIME button to cycle through the following settings from left to right. Press the TIME button again to go back to 3s settings from the Extended Mode.

| 3s | 6s | 9s | 12s | Extended Mode (15s) |
|----|----|----|-----|---------------------|
|    |    |    |     |                     |

**CAUTION!** High intensity light always involves the production of heat. When utilizing the light in Extended Mode (15s), do not keep the light in one position. Longer exposure in the region of the pulp can lead to injury or irritation of the pulp, which is why the specified exposure times must not be exceeded.

### Activating and Deactivating the Light Emission

Briefly press the START button to activate the light emission for the Exposure Time set as described above.

- Once the emission has begun, the active Exposure Time LEDs will count down every 3s with an acoustical signal indicating the elapsed time, except for the Extended Mode.
- When the emission is completed, the Exposure Time LEDs will return as set before. Example with 12s Exposure Time setting:

| Beginning of Emission | 3 sec Elapsed | 6 sec Elapsed | 9 sec Elapsed | End of Emission |
|-----------------------|---------------|---------------|---------------|-----------------|
|                       |               |               |               |                 |
| ON Signal             | 1 beep        | 2 beeps       | 3 beeps       | OFF Signal      |

- For Extended Mode, an acoustical signal is emitted every 3 seconds for a total of 15 seconds.
- If desired, press the START button to stop the light emission before the set time is over.

### Tack Cure Function

Hold down the START button to activate the Tack Cure function regardless of the set Exposure Time. The unit emits a single one-second light pulse which allows for a short exposure to tack cure brackets. The light will return to standard operation when the START button is no longer depressed.

### Recommended Cure Techniques with 3M Unitek Adhesives

- Rotate the light guide into the desired position for polymerization.
- For most 3M Unitek adhesives, reference the table below for recommended cure technique

| Appliance                         | Adhesive                                                                                                                            | Ortholux™ Luminous Curing Light                       |
|-----------------------------------|-------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------|
| Metal Brackets                    | APC™ Flash-Free Adhesive Coated Brackets                                                                                            | 6 seconds mesial x 6 seconds distal                   |
| Ceramic Brackets                  |                                                                                                                                     | 3 seconds through the bracket                         |
| Buccal Tubes (direct bond)        |                                                                                                                                     | 6 seconds mesial x 6 seconds occlusal                 |
| Metal Brackets                    | Transbond™ XT Adhesive, Transbond™ PLUS Color Change Adhesive, APC™ II Adhesive Coated Brackets, APC™ PLUS Adhesive Coated Brackets | 3 seconds mesial x 3 seconds distal                   |
| Ceramic Brackets                  |                                                                                                                                     | 3 seconds through the bracket                         |
| Buccal Tubes (direct bond)        |                                                                                                                                     | 6 seconds mesial x 6 seconds occlusal                 |
| Lingual Retainers                 | Transbond™ LR Adhesive                                                                                                              | 3 seconds mesial x 3 seconds distal                   |
| Molar Bands                       | Transbond™ Plus Band Adhesive Unitek™ Multi-Cure Glass Ionomer Band Cement                                                          | 12 seconds (3 seconds per cusp)                       |
| Indirect Trays (Metal Brackets)   | Transbond™ Supreme LV Low Viscosity Light Cure Adhesive                                                                             | 6 seconds mesial x 6 seconds distal through the trays |
| Indirect Trays (Ceramic Brackets) |                                                                                                                                     | 6 seconds through the bracket and trays               |

- For other 3M Unitek adhesives not specifically mentioned, please refer to the adhesive Instructions for Use.
- If using other light cure adhesive systems, consult the adhesive manufacturer's instructions.
- The light guide tip should be placed approximately 1-2 mm above the appliance and pointed perpendicular towards the tooth surface.
- The recommendations are applicable with 100% light output intensity, measured at the intensity test area on the charging base.
- A technique tip for faster light curing of metal brackets is to position the light guide interproximal to two brackets. In order for the bracket to fully cure, both sides of the bracket must be exposed to the light.
  - Keep the light guide clean at all times to obtain full light intensity. Cured primer/ adhesive on the light guide will reduce the intensity resulting in incomplete cure.
  - Damaged light guides substantially reduce the light intensity and must be replaced immediately, sharp edges may cause serious injury!

NOTE: Holding the light away from the tooth, moving the light guide and turning the light guide tip at an angle may cause incomplete cure of the material.


### Sleep Mode

Once the handpiece is placed in the charger, all internal functions and LEDs are automatically turned off as the handpiece switches to sleep mode. This reduces the power consumption of the battery to a minimal level. Outside the charger, the handpiece is also switched to sleep mode if it is not used for approximately 5 minutes.

### Measurement of Light Intensity

The light intensity of the Ortholux Luminous Curing Light handpiece can only be reliably determined using the light intensity testing area on the Ortholux Luminous Curing Light charger! The circular testing

area is located below the Status LED on the charger. Measurements carried out using other units will produce incorrect results due to the different light sources and the varying arrangement of the components of the units. **The light intensity is independent of the battery charge level.**

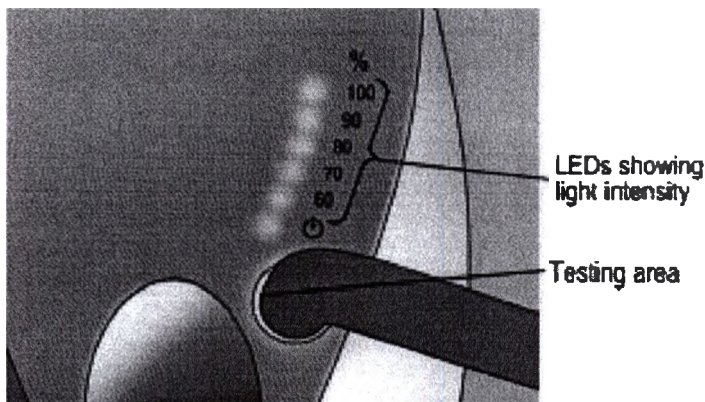#### CAUTION!

- Clean the testing area with a damp cloth.
- Without applying pressure, place the light guide on the testing area such that the light-emitting opening of the light guide is flush with the testing area.
- Activate the lamp by pressing the START button.  
The number of illuminated LEDs on the charger is indicative of the measured light intensity:  
5 LEDs = 100%, 4 LEDs = 90%, 3 LEDs = 80%, 2 LEDs = 70%, 1 LED = 60%.
- If the light intensity is below 80% (fewer than 3 LEDs are ON), check the light guide and handpiece for contamination or defects.
- Remove and clean the light guide of any contamination. Replace a defective light guide with a new light guide.

#### Acoustical Signals - Handpiece

*An acoustical signal is emitted:*

- every time a button is pressed
- every time the light is turned ON or OFF
- 1 time after 3 seconds exposure time, 2 times after 6 seconds, 3 times after 9 seconds

Exception: in Extended Mode; an acoustical signal is emitted every 3 seconds

- every time the sleep mode is terminated by pressing the START button.

*A 2 second-error signal is emitted, if:*

- the handpiece overheats
- the battery lacks sufficient charge

The acoustical signals from the handpiece can be turned off (except for the 2-second error signal). Put the handpiece in sleep, e.g., by setting it in the charger. Take the device from the charger; press and hold the TIME button, then press the START button. This changes the sleep mode of the handpiece back to active and the status "Acoustical signals activated" to "Acoustical signals deactivated". The acoustical signals can be reactivated by following the same procedure.

#### Troubleshooting

| Error                                                         | Cause                                                                                    | Solution                                                      |
|---------------------------------------------------------------|------------------------------------------------------------------------------------------|---------------------------------------------------------------|
| The charge level display on the handpiece glows red steadily. | The remaining battery charge is adequate only for typically 20 3-second exposure cycles. | Place the handpiece in the charger and re-charge the battery. |

|                                                                                                                                                                                                                                        |                                                                                                                            |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                              |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The power level display of the handpiece flashes red.<br>The ongoing exposure is interrupted (light off signal is emitted) followed by a 2 second-error signal; the handpiece switches to "sleep" mode and resists further activation. | The battery lacks sufficient charge.                                                                                       | Place the handpiece in the charger and re-charge the battery.                                                                                                                                                                                                                                                                                                                                                                                                                                |
| The power level display of the handpiece flashes red while the handpiece is in the charger.                                                                                                                                            | Charging problem. The battery is defective or at the end of its useful life.                                               | Remove and re-insert the battery to reset the unit.<br>Replace the battery.                                                                                                                                                                                                                                                                                                                                                                                                                  |
| The handpiece has not been used for a long time and now it cannot be turned on.                                                                                                                                                        | There is not enough charge in the battery to turn on the handpiece.                                                        | Place the handpiece in the charger and re-charge the battery.                                                                                                                                                                                                                                                                                                                                                                                                                                |
| The light emission does not start when the START button is pressed; an error signal sounds for 2 seconds.                                                                                                                              | The handpiece has become overheated in the course of use. The handpiece can be used again once it has cooled down briefly. | Allow the handpiece to cool down. The handpiece can be used again, once pressing the START button successfully activates the light. During the cooling-off period immediately after the acoustical signal has gone off, 2 minute cooling-off time in the air typically corresponds to 1 minute operating time.                                                                                                                                                                               |
| During light emission in extended mode, an error signal sounds for 2 seconds, the emission is stopped, and the handpiece changes to sleep mode.                                                                                        | The handpiece has become overheated in the course of use. The handpiece can be used again once it has cooled down briefly. | Allow the handpiece to cool down. - Immediately after the temperature control has turned off the handpiece, it must cool down a minimum of 1.5 minutes in the air before a new light emission can be started. The handpiece can be used again, once pressing the START button successfully activates the light. - During the cooling-off period immediately after the acoustical signal has gone off, 2 minute cooling-off time in the air typically corresponds to 1 minute operating time. |
| The LED showing the operating status in the charger flashes alternately red and green. The handpiece is not in the charger.                                                                                                            | Charger is defective.                                                                                                      | Unplug and plug in the charger to reset.<br>Have the charger repaired.                                                                                                                                                                                                                                                                                                                                                                                                                       |
| The LED showing the operating status in the charger flashes alternately red and green. The handpiece is in the charger.                                                                                                                | Malfunction during charging.                                                                                               | <ul style="list-style-type: none"> <li>• Remove and re-insert the battery to reset the unit.</li> <li>• The battery is damaged.</li> <li>• Replace the battery.</li> </ul>                                                                                                                                                                                                                                                                                                                   |
| The LED showing the operating status in the charger does not come on although the plug has been connected to an electrical outlet.                                                                                                     | Power outlet carries no voltage.<br>Charger is defective.                                                                  | Use a different power outlet.<br>Have the charger repaired.                                                                                                                                                                                                                                                                                                                                                                                                                                  |
| The light intensity is too low.                                                                                                                                                                                                        |                                                                                                                            | Clean the light guide and the protecting glass in the light guide mounting hole (please refer to "Cleaning the Light Guide").                                                                                                                                                                                                                                                                                                                                                                |

|                                                                         |                                                                  |                                                                                                      |
|-------------------------------------------------------------------------|------------------------------------------------------------------|------------------------------------------------------------------------------------------------------|
| The LED showing the operating status in the charger glows red steadily. | The charge contact pins in the handpiece or the charger are wet. | Dry the charge contact pins. Be careful not to bend the flexible charge contact pins in the charger. |
|-------------------------------------------------------------------------|------------------------------------------------------------------|------------------------------------------------------------------------------------------------------|

### **Maintenance and Care**

The Ortholux Luminous Curing Light is maintenance free with the exception of cleaning the light guide and replacing batteries as needed. The light output should be checked on a regular basis, i.e. daily, using the integrated light intensity testing area on the charging base.

See the information contained in this chapter to secure problem-free operation.

There are no user serviceable parts on the unit. If the unit malfunctions and the trouble shooting steps above do not remedy the situation, call 3M Unitek Customer Service.

### **Inserting/Removing the Battery**

Never place the handpiece in the charger without the battery inserted in the handpiece! Use 3M Unitek batteries only! The use of other manufacturer batteries or non-rechargeable batteries/primary batteries is a potential hazard and may damage the unit.

1. Remove the battery from the handpiece by turning it counter-clockwise as shown.
2. Insert the (new) battery into the handpiece and turn it clockwise until the seal is firmly pressed against the metal housing.
3. Place the handpiece in the charger for 1.5 hours in order to fully charge the new battery for the first time.

- The LED showing the operating status on the charger flashes green; see also "Operating Status Display of the Charger."

### **Handpiece/Battery Care**

1. Use only the 3M Unitek charger which is provided with the unit; failure to do so could lead to damage to the battery!
2. Do not immerse the battery in water or incinerate! Please also observe the chapter on "Safety".

### **Cleaning the Light Guide**

Clean and disinfect the light guide before every use. The light guide is not sterile when delivered and must be sterilized before being used for the first time.

#### Material Resistance

Make sure that the cleaning and disinfectant agents you have chosen do not contain any of the following materials:

- Organic, mineral, and oxidizing acids (minimum acceptable pH value 5.5)
- Bases (maximum acceptable pH value 8.5)
- Oxidation agents (e. g., hydrogen peroxide)
- Halogens (chlorine, iodine, bromide)
- Aromatic/halogenized hydrocarbons

Please check the manufacturer's information about the cleaning and disinfecting agents.

The light guide must not be exposed to temperatures higher than 134 °C (273 °F).

The light guide has been tested for up to 500 sterilization cycles.

#### Pre-Treatment

The pre-treatment must be carried out before either automatic or manual cleaning and disinfecting.

- Immediately after using (within a maximum of 2 hours), remove gross contaminations from the light guide.
- Rinse the light guide off thoroughly (at least 10 sec) under running water or use a suitable disinfectant solution without any aldehyde (disinfectant should not contain any aldehyde to prevent blood from becoming fixed).
- Use a soft brush or a soft cloth to manually remove contaminations. Adhering polymerized composite should be removed with alcohol, a plastic spatula may help in removing the material. Do not use any sharp or pointed tools to protect the surface of the light guide from scratching.

#### Manual Cleaning and Disinfection of the Light Guide

Place the light guide for the specified application time into the solution, making sure that it is completely covered (as needed, using ultrasonic support or careful brushing with a soft brush).

- A neutral enzymatic cleaning agent is recommended (e.g., Cidezyme/Enzol from Johnson & Johnson).
- Remove the light guide from the solution and rinse thoroughly (at least 10 sec) in water with low germ count.
- To disinfect, place the cleaned light guide for the specified application time into the solution, making sure that it is completely covered. Disinfectants containing o-phthalaldehyde are recommended (e. g., Cidex OPA from Johnson & Johnson).
- Remove the light guide from the solution and rinse thoroughly (at least 10 sec) in water with low germ count.
- Dry the light guide with a clean cloth.
- Check the light guide (see section "Check").

#### Automatic Cleaning/Disinfection

(Disinfector/CDD (Cleaning and Disinfection Device))

Alternatively, cleaning and disinfecting can be conducted automatically. Information about validated procedures can be obtained from 3M Deutschland GmbH.

#### Sterilization

Effective cleaning and disinfection are absolutely essential requirements for effective sterilization.

Only steam sterilization is approved as a sterilization procedure:

- Maximum sterilization temperature 134 °C (273 °F)
- Sterilization time (exposure time at sterilization temperature) at least 20 min. at 121°C (250 °F) or at least 3 min. at 132 °C (270 °F) /134 °C (273 °F)

#### Check

Before using the light guide again, check it for damaged surfaces, discoloration, and contamination; do not use damaged light guides. If the light guide is still contaminated, repeat the cleaning and disinfection.

#### **Cleaning the Charger, Handpiece, and Eye Shield**

- Clean all components with a soft cloth and, if necessary, a mild cleaning agent (e.g. dishwashing detergent). Solvents or abrasive cleaners may not be used in any case, as they can damage the plastic components!

- Cleaning agents must not enter the unit!

To disinfect all components, spray the disinfectant on a towel and use it to disinfect the unit. Do not spray the disinfectant directly on the handpiece or the charger.

- Disinfection agents must not enter the unit!

Dry residual disinfectants on the device with a soft and fluff-free cloth, as they damage the plastic components. If necessary, ask the manufacturer of the disinfectant if its constant use will damage plastic surfaces.

Additional Notes: Make sure that charge contact pins remain dry and are not contacted by metallic or greasy parts. Do not bend the charge contact pins during drying. Wet charge contact pins will cause an operating error (malfunction message: the LED showing the operating status of the charger flashes red).

#### **Storage of the Handpiece during Extended Periods of Non-Use**

1. If the handpiece is not to be used for an extended period of time (e.g., during vacation) fully charge the battery prior to departure or keep the handpiece inserted in the operational charger. A safety switch within the battery prevents a total discharge.
3. Discharged or nearly discharged batteries must be recharged as soon as possible.

#### **Disposal**

Your new unit contains a powerful lithium-ion rechargeable battery. Dispose of defective batteries and units in accordance with local legal regulations!

Separate collection is a condition for recycling and use, due to which it is possible to ensure careful use of renewable and non-renewable natural resources in production.

When disposed your Ortholux Luminous should not be disposed with household waste. For these purposes 3M Unitek has created opportunities for disposal. For more information on the country-specific current procedure of actions available contact 3M subsidiary responsible for this issue.

Medical products should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class Γ).

**Customer Information**

No person is authorized to provide any information that deviates from the information provided in this instruction sheet.

**Description of packaging**

Primary packaging: polystyrene insert. Secondary packaging: folding box

Length: (280± 3) mm, Width: (230± 3) mm, Height: (110± 3) mm.

Handpiece and Battery are additionally placed in protective plastic bag.

**Contents**

Ortholux Luminous contents: 1 Handpiece, 1 Eye Shield, 1 Battery, 1 Charger, 1 Light Guide, 1 Instruction for Use.

**Warranty**

3M Unitek warrants this product will be free from defects in material and manufacture for a period of two years from date of purchase.

User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Unitek's sole obligation shall be repair or replacement of the 3M Unitek product.

**Limitation of Liability**

Except where prohibited by law, 3M Unitek will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**Name and address of the manufacturer:**

3M Unitek Corporation, 2724 South Peck Road, Monrovia, CA 91016, USA

**Authorized representative on the territory of the Russian Federation:**

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