	EXTRACTS FROM OSTOMY CARE PRODUCTS TECHNICAL FILE	Document: KS TF 001 Revision No: 02 Date: 02.10.2017
---	--	---

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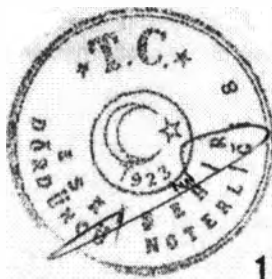
« _____ » 201

KONFORT GİZEL SAĞLIK HİZMETLERİ
TIBBİ MALZEME SANAYİ VE TİC. LTD. ŞTİ.
CŞŞ. B. Cd. No:32 Dumanpazarı/ŞEKİR
Tel:0222 226 03 27 Faks:0222 226 03 20
Eskişehir M.D. Başkentler 535 018 1237
Marsis No:57850 22-0200114 T.C.Sic. No:22049
www.comfortnisi.com
Sermayesi:100.000,00 TL

EXTRACT FROM MEDICAL DEVICE TECHNICAL FILE

Class 1 Medical Devices

Technical information demonstrating conformity of products produced by Konfort Ozel Saglik Hizmetleri Tıbbi Malz. San. Tic. Ltd. Şti. prepared following review of the Essential Requirements of Annex 1 of the Medical Directive. These products have been classified to fall within Class 1 (Annex IX of the Directive) - non sterile and non measuring.

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1. Medical device:

“One-piece and two-piece ostomy pouches Confort in different production variants”

I) One-piece ostomy pouches Confort with the following features:

- 1) One-piece drainable ostomy pouches Confort transparent, with filter, with cut-to fit opening up to 32 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 2) One-piece drainable ostomy pouches Confort opaque, with filter, with cut-to fit opening up to 32 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 3) One-piece drainable ostomy pouches Confort opaque, without filter, with cut-to fit opening up to 32 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 4) One-piece drainable ostomy pouches Confort transparent, without filter, with cut-to fit opening up to 32 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 5) One-piece closed ostomy pouches Confort transparent, with filter, with cut-to fit opening up to 32 mm.
- 6) One-piece closed ostomy pouches Confort opaque with filter, with cut-to fit opening up to 32mm.
- 7) One-piece closed ostomy pouches Confort transparent, without filter, with cut-to fit opening up to 32 mm.
- 8) One-piece closed ostomy pouches Confort opaque, without filter, with cut-to fit opening up to 32 mm.
- 9) One-piece drainable ostomy pouches Confort transparent, with filter, with cut-to fit opening up to 55 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 10) One-piece drainable ostomy pouches Confort opaque, with filter, with cut-to fit opening up to 55 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 11) One-piece drainable ostomy pouches Confort opaque, without filter, with cut-to fit opening up to 55 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 12) One-piece drainable ostomy pouches Confort transparent, without filter, with cut-to fit opening up to 55 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 13) One-piece closed ostomy pouches Confort transparent, with filter, with cut-to fit opening up to 55mm.
- 14) One-piece closed ostomy pouches Confort opaque with filter, with cut-to fit opening up to 55 mm.
- 15) One-piece closed ostomy pouches Confort transparent, without filter, with cut-to fit opening up to 55 mm.

- 16) One-piece closed ostomy pouches Confort opaque, without filter, with cut-to fit opening up to 55 mm.
- 17) One-piece drainable ostomy pouches Confort transparent, with filter, with cut-to fit opening up to 60 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 18) One-piece drainable ostomy pouches Confort opaque, with filter, with cut-to fit opening up to 60 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 19) One-piece drainable ostomy pouches Confort opaque, without filter, with cut-to fit opening up to 60 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 20) One-piece drainable ostomy pouches Confort transparent, without filter, with cut-to fit opening up to 60 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 21) One-piece closed ostomy pouches Confort transparent, with filter, with cut-to fit opening up to 60 mm.
- 22) One-piece closed ostomy pouches Confort opaque with filter, with cut-to fit opening up to 60 mm.
- 23) One-piece closed ostomy pouches Confort transparent, without filter, with cut-to fit opening up to 60 mm.
- 24) One-piece closed ostomy pouches Confort opaque, without filter, with cut-to fit opening up to 60 mm.
- 25) One-piece drainable ostomy pouches Confort transparent, with filter, with cut-to fit opening up to 70 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 26) One-piece drainable ostomy pouches Confort opaque, with filter, with cut-to fit opening up to 70 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 27) One-piece drainable ostomy pouches Confort opaque, without filter, with cut-to fit opening up to 70 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 28) One-piece drainable ostomy pouches Confort transparent, without filter, with cut-to fit opening up to 70 mm, including
-clamp for drainable pouches Confort (supplied on request).
- 29) One-piece closed ostomy pouches Confort transparent, with filter, with cut-to fit opening up to 70 mm.
- 30) One-piece closed ostomy pouches Confort opaque with filter, with cut-to fit opening up to 70 mm.
- 31) One-piece closed ostomy pouches Confort transparent, without filter, with cut-to fit opening up to 70 mm.
- 32) One-piece closed ostomy pouches Confort opaque, without filter, with cut-to fit opening up to 70 mm.

- 33) One-piece drainable ostomy pouches Confort transparent, with filter, with cut-to fit opening up to 80 mm, including
- clamp for drainable pouches Confort (supplied on request).
- 34) One-piece drainable ostomy pouches Confort opaque, with filter, with cut-to fit opening up to 80 mm, including
- clamp for drainable pouches Confort (supplied on request).
- 35) One-piece drainable ostomy pouches Confort opaque, without filter, with cut-to fit opening up to 80 mm, including
- clamp for drainable pouches Confort (supplied on request).
- 36) One-piece drainable ostomy pouches Confort transparent, without filter, with cut-to fit opening up to 80 mm, including
- clamp for drainable pouches Confort (supplied on request).
- 37) One-piece closed ostomy pouches Confort transparent, with filter, with cut-to fit opening up to 80 mm.
- 38) One-piece closed ostomy pouches Confort opaque with filter, with cut-to fit opening up to 80 mm.
- 39) One-piece closed ostomy pouches Confort transparent, without filter, with cut-to fit opening up to 80 mm.
- 40) One-piece closed ostomy pouches Confort opaque, without filter, with cut-to fit opening up to 80 mm.
- 41) One-piece drainable ostomy pouches Confort transparent, with filter, with cut-to fit opening up to 100 mm, including
- clamp for drainable pouches Confort (supplied on request).
- 42) One-piece drainable ostomy pouches Confort opaque, with filter, with cut-to fit opening up to 100 mm, including
- clamp for drainable pouches Confort (supplied on request).
- 43) One-piece drainable ostomy pouches Confort opaque, without filter, with cut-to fit opening up to 100 mm, including
- clamp for drainable pouches Confort (supplied on request).
- 44) One-piece drainable ostomy pouches Confort transparent, without filter, with cut-to fit opening up to 100 mm, including
- clamp for drainable pouches Confort (supplied on request).
- 45) One-piece closed ostomy pouches Confort transparent, with filter, with cut-to fit opening up to 100 mm.
- 46) One-piece closed ostomy pouches Confort opaque with filter, with cut-to fit opening up to 100 mm.
- 47) One-piece closed ostomy pouches Confort transparent, without filter, with cut-to fit opening up to 100 mm.
- 48) One-piece closed ostomy pouches Confort opaque, without filter, with cut-to fit opening up to 100 mm.
- 49) One-piece drainable ostomy pouches Confort transparent, with filter, with Velcro closure, with cut-to fit opening up to 32 mm.

- 50) One-piece drainable ostomy pouches Confort opaque, with filter, with Velcro closure, with cut-to fit opening up to 32 mm.
- 51) One-piece drainable ostomy pouches Confort opaque, without filter, with Velcro closure, with cut-to fit opening up to 32 mm.
- 52) One-piece drainable ostomy pouches Confort transparent, without filter, with Velcro closure, with cut-to fit opening up to 32 mm.
- 53) One-piece drainable ostomy pouches Confort transparent, with filter, with Velcro closure, with cut-to fit opening up to 55 mm.
- 54) One-piece drainable ostomy pouches Confort opaque, with filter, with Velcro closure, with cut-to fit opening up to 55 mm.
- 55) One-piece drainable ostomy pouches Confort opaque, without filter, with Velcro closure, with cut-to fit opening up to 55 mm.
- 56) One-piece drainable ostomy pouches Confort transparent, without filter, with Velcro closure, with cut-to fit opening up to 55 mm.
- 57) One-piece drainable ostomy pouches Confort transparent, with filter, with Velcro closure, with cut-to fit opening up to 60 mm.
- 58) One-piece drainable ostomy pouches Confort opaque, with filter, with Velcro closure, with cut-to fit opening up to 60 mm.
- 59) One-piece drainable ostomy pouches Confort opaque, without filter, with Velcro closure, with cut-to fit opening up to 60 mm.
- 60) One-piece drainable ostomy pouches Confort transparent, without filter, with Velcro closure, with cut-to fit opening up to 60 mm.
- 61) One-piece drainable ostomy pouches Confort transparent, with filter, with Velcro closure, with cut-to fit opening up to 60 mm.
- 62) One-piece drainable ostomy pouches Confort opaque, with filter, with Velcro closure, with cut-to fit opening up to 70 mm.
- 63) One-piece drainable ostomy pouches Confort opaque, without filter, with Velcro closure, with cut-to fit opening up to 70 mm.
- 64) One-piece drainable ostomy pouches Confort transparent, without filter, with Velcro closure, with cut-to fit opening up to 70 mm.
- 65) One-piece drainable ostomy pouches Confort transparent, with filter, with Velcro closure, with cut-to fit opening up to 80 mm.
- 66) One-piece drainable ostomy pouches Confort opaque, with filter, with Velcro closure, with cut-to fit opening up to 80 mm.
- 67) One-piece drainable ostomy pouches Confort opaque, without filter, with Velcro closure, with cut-to fit opening up to 80 mm.
- 68) One-piece drainable ostomy pouches Confort transparent, without filter, with Velcro closure, with cut-to fit opening up to 80 mm.

69) One-piece drainable ostomy pouches Confort transparent, with filter, with Velcro closure, with cut-to fit opening up to 100 mm.

70) One-piece drainable ostomy pouches Confort opaque, with filter, with Velcro closure, with cut-to fit opening up to 100 mm.

71) One-piece drainable ostomy pouches Confort opaque, without filter, with Velcro closure, with cut-to fit opening up to 100 mm.

72) One-piece drainable ostomy pouches Confort transparent, without filter, with Velcro closure, with cut-to fit opening up to 100 mm.

II) Two-piece ostomy pouches Confort with the following options:

1) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 32 mm.
- Closed two-piece ostomy pouches Confort, transparent, without filter, flange 32 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

2) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 35 mm.
- Closed two-piece ostomy pouches Confort, transparent, without filter, flange 35 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

3) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 38 mm.
- Closed two-piece ostomy pouches Confort, transparent, without filter, flange 38 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

4) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 40 mm.
- Closed two-piece ostomy pouches Confort, transparent, without filter, flange 40 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

5) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 45 mm.
- Closed two-piece ostomy pouches Confort, transparent, without filter, flange 45 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

6) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 50 mm.

- Closed two-piece ostomy pouches Confort, transparent, without filter, flange 50 mm (supplied on request no more than 10 pieces).
- Ostomy belt Confort (supplied on request).
- 7) Two-piece ostomy pouches Confort including:
 - Flanges for two-piece ostomy pouches Confort, flange 55 mm.
 - Closed two-piece ostomy pouches Confort, transparent, without filter, flange 55 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).
- 8) Two-piece ostomy pouches Confort including:
 - Flanges for two-piece ostomy pouches Confort, flange 57 mm.
 - Closed two-piece ostomy pouches Confort, transparent, without filter, flange 57 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).
- 9) Two-piece ostomy pouches Confort including:
 - Flanges for two-piece ostomy pouches Confort, flange 60 mm.
 - Closed two-piece ostomy pouches Confort, transparent, without filter, flange 60 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).
- 10) Two-piece ostomy pouches Confort including:
 - Flanges for two-piece ostomy pouches Confort, flange 70 mm.
 - Closed two-piece ostomy pouches Confort, transparent, without filter, flange 70 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).
- 11) Two-piece ostomy pouches Confort including:
 - Flanges for two-piece ostomy pouches Confort, flange 80 mm.
 - Closed two-piece ostomy pouches Confort, transparent, without filter, flange 80 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).
- 12) Two-piece ostomy pouches Confort including:
 - Flanges for two-piece ostomy pouches Confort, flange 100 mm.
 - Closed two-piece ostomy pouches Confort, transparent, without filter, flange 100 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).
- 13) Two-piece ostomy pouches Confort including:
 - Flanges for two-piece ostomy pouches Confort, flange 32 mm.
 - Closed two-piece ostomy pouches Confort, opaque without filter, flange 32 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).

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14) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 35 mm.
- Closed two-piece ostomy pouches Confort, opaque, without filter, flange 35 mm (supplied on request

no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

15) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 38 mm.
- Closed two-piece ostomy pouches Confort, opaque, without filter, flange 38 mm (supplied on request

no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

16) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 40 mm.
- Closed two-piece ostomy pouches Confort, opaque, without filter, flange 40 mm (supplied on request

no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

17) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 45 mm.
- Closed two-piece ostomy pouches Confort, opaque, without filter, flange 45 mm (supplied on request

no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

18) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 50 mm.
- Closed two-piece ostomy pouches Confort, opaque, without filter, flange 50 mm (supplied on request

no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

19) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 55 mm.
- Closed two-piece ostomy pouches Confort, opaque, without filter, flange 55 mm (supplied on request

no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

20) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 57 mm.
- Closed two-piece ostomy pouches Confort, opaque, without filter, flange 57 mm (supplied on request

no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

21) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 60 mm.

- Closed two-piece ostomy pouches Confort, opaque, without filter, flange 60 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

22) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 70 mm.

- Closed two-piece ostomy pouches Confort, opaque, without filter, flange 70 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

23) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 80 mm.

- Closed two-piece ostomy pouches Confort, opaque, without filter, flange 80 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

24) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 100 mm.

- Closed two-piece ostomy pouches Confort, opaque, without filter, flange 100 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

25) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 32 mm.

- Closed two-piece ostomy pouches Confort, transparent, with filter, flange 32 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

26) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 35 mm.

- Closed two-piece ostomy pouches Confort, transparent, with filter, flange 35 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

27) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 38 mm.

- Closed two-piece ostomy pouches Confort, transparent, with filter, flange 38 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

28) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 40 mm.

- Closed two-piece ostomy pouches Confort, transparent, with filter, flange 40 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

29) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 45 mm.
- Closed two-piece ostomy pouches Confort, transparent, with filter, flange 45 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

30) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 50 mm.
- Closed two-piece ostomy pouches Confort, transparent, with filter, flange 50 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

31) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 55 mm.
- Closed two-piece ostomy pouches Confort, transparent, with filter, flange 55 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

32) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 57 mm.
- Closed two-piece ostomy pouches Confort, transparent, with filter, flange 57 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

33) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 60 mm.
- Closed two-piece ostomy pouches Confort, transparent, with filter, flange 60 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

34) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 70 mm.
- Closed two-piece ostomy pouches Confort, transparent, with filter, flange 70 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

35) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 80 mm.
- Closed two-piece ostomy pouches Confort, transparent, with filter, flange 80 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

36) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 100 mm.

- Closed two-piece ostomy pouches Confort, transparent, with filter, flange 100 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

37) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 32 mm.

- Closed two-piece ostomy pouches Confort, opaque with filter, flange 32 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

38) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 35 mm.

- Closed two-piece ostomy pouches Confort, opaque, with filter, flange 35 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

39) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 38 mm.

- Closed two-piece ostomy pouches Confort, opaque, with filter, flange 38 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

40) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 40 mm.

- Closed two-piece ostomy pouches Confort, opaque, with filter, flange 40 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

41) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 45 mm.

- Closed two-piece ostomy pouches Confort, opaque, with filter, flange 45 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

42) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 50 mm.

- Closed two-piece ostomy pouches Confort, opaque, with filter, flange 50 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

43) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 55 mm.

- Closed two-piece ostomy pouches Confort, opaque, with filter, flange 55 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

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44) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 57 mm.
- Closed two-piece ostomy pouches Confort, opaque, with filter, flange 57 mm (supplied on request

no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

45) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 60 mm.
- Closed two-piece ostomy pouches Confort, opaque, with filter, flange 60 mm (supplied on request

no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

46) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 70 mm.
- Closed two-piece ostomy pouches Confort, opaque, with filter, flange 70 mm (supplied on request

no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

47) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 80 mm.
- Closed two-piece ostomy pouches Confort, opaque, with filter, flange 80 mm (supplied on request

no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

48) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 100 mm.
- Closed two-piece ostomy pouches Confort, opaque, with filter, flange 100 mm (supplied on request

no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

49) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 32 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, flange 32 mm (supplied on

request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

50) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 35 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, flange 35 mm (supplied on

request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

51) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 38 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, flange 38 mm (supplied on request no more than 10 pieces).

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

52) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 40 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, flange 40 mm (supplied on request no more than 10 pieces).

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

53) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 45 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, flange 45 mm (supplied on request no more than 10 pieces).

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

54) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 50 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, flange 50 mm (supplied on request no more than 10 pieces).

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

55) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 55 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, flange 55 mm (supplied on request no more than 10 pieces).

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

56) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 57 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, flange 57 mm (supplied on request no more than 10 pieces).

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

57) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 60 mm.

- Drainable two-piece ostomy pouches Confort, transparent, without filter, flange 60 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

58) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 70 mm.

- Drainable two-piece ostomy pouches Confort, transparent, without filter, flange 70 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

59) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 80 mm.

- Drainable two-piece ostomy pouches Confort, transparent, without filter, flange 80 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

60) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 100 mm.

- Drainable two-piece ostomy pouches Confort, transparent, without filter, flange 100 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

61) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 32 mm.

- Drainable two-piece ostomy pouches Confort, transparent, with filter, flange 32 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

62) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 35 mm.

- Drainable two-piece ostomy pouches Confort, transparent, with filter, flange 35 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

63) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 38 mm.

- Drainable two-piece ostomy pouches Confort, transparent, with filter, flange 38 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

64) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 40 mm.
- Drainable two-piece ostomy pouches Confort, transparent, with filter, flange 40 mm (supplied on request no more than 10 pieces).

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

65) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 45 mm.
- Drainable two-piece ostomy pouches Confort, transparent, with filter, flange 45 mm (supplied on request no more than 10 pieces).

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

66) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 50 mm.
- Drainable two-piece ostomy pouches Confort, transparent, with filter, flange 50 mm (supplied on request no more than 10 pieces).

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

67) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 55 mm.
- Drainable two-piece ostomy pouches Confort, transparent, with filter, flange 55 mm (supplied on request no more than 10 pieces).

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

68) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 57 mm.
- Drainable two-piece ostomy pouches Confort, transparent, with filter, flange 57 mm (supplied on request no more than 10 pieces).

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

69) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 60 mm.
- Drainable two-piece ostomy pouches Confort, transparent, with filter, flange 60 mm (supplied on request no more than 10 pieces).

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

70) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 70 mm.
- Drainable two-piece ostomy pouches Confort, transparent, with filter, flange 70 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.

- Ostomy belt Confort (supplied on request).

71) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 80 mm.

- Drainable two-piece ostomy pouches Confort, transparent, with filter, flange 80 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.

- Ostomy belt Confort (supplied on request).

72) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 100 mm.

- Drainable two-piece ostomy pouches Confort, transparent, with filter, flange 100 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.

- Ostomy belt Confort (supplied on request).

73) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 32 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, flange 32 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.

- Ostomy belt Confort (supplied on request).

74) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 35 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, flange 35 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.

- Ostomy belt Confort (supplied on request).

75) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 38 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, flange 38 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.

- Ostomy belt Confort (supplied on request).

76) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 40 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, flange 40 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

77) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 45 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, flange 45 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

78) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 50 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, flange 50 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

79) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 55 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, flange 55 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

80) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 57 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, flange 57 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

81) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 60 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, flange 60 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

82) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 70 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, flange 70 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

83) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 80 mm.
- Drainable two-piece ostomy pouches Confort, opaque, without filter, flange 80 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

84) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 100 mm.
- Drainable two-piece ostomy pouches Confort, opaque, without filter, flange 100 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

85) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 32 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, flange 32 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

86) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 35 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, flange 35 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

87) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 38 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, flange 38 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

88) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 40 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, flange 40 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

89) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 45 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, flange 45 mm (supplied on request

no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

90) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 50 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, flange 50 mm (supplied on request

no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

91) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 55 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, flange 55 mm (supplied on request

no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

92) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 57 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, flange 57 mm (supplied on request

no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

93) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 60 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, flange 60 mm (supplied on request

no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

94) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 70 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, flange 70 mm (supplied on request

no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

95) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 80 mm.

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- Drainable two-piece ostomy pouches Confort, opaque, with filter, flange 80 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

96) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 100 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, flange 100 mm (supplied on request no more than 10 pieces.

- Clamp for drainable pouches Confort.
- Ostomy belt Confort (supplied on request).

97) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 32 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, with Velcro closure, flange 32 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

98) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 35 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, with Velcro closure, flange 35 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

99) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 38 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, with Velcro closure, flange 38 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

100) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 40 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, with Velcro closure, flange 40 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

101) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 45 mm.
- Drainable two-piece ostomy pouches Confort, transparent, without filter, with Velcro closure, flange 45 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

102) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 50 mm.

- Drainable two-piece ostomy pouches Confort, transparent, without filter, with Velcro closure, flange 50 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

103) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 55 mm.

- Drainable two-piece ostomy pouches Confort, transparent, without filter, with Velcro closure, flange 55 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

104) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 57 mm.

- Drainable two-piece ostomy pouches Confort, transparent, without filter, with Velcro closure, flange 57 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

105) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 60 mm.

- Drainable two-piece ostomy pouches Confort, transparent, without filter, with Velcro closure, flange 60 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

106) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 70 mm.

- Drainable two-piece ostomy pouches Confort, transparent, without filter, with Velcro closure, flange 70 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

107) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 80 mm.

- Drainable two-piece ostomy pouches Confort, transparent, without filter, with Velcro closure, flange 80 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

108) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 100 mm.

- Drainable two-piece ostomy pouches Confort, transparent, without filter, with Velcro closure, flange 100 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

109) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 32 mm.

- Drainable two-piece ostomy pouches Confort, transparent, with filter, with Velcro closure, flange 32 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

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- 110) Two-piece ostomy pouches Confort including:
- Flanges for two-piece ostomy pouches Confort, flange 35 mm.
 - Drainable two-piece ostomy pouches Confort, transparent, with filter, with Velcro closure, flange 35 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).
- 111) Two-piece ostomy pouches Confort including:
- Flanges for two-piece ostomy pouches Confort, flange 38 mm.
 - Drainable two-piece ostomy pouches Confort, transparent, with filter, with Velcro closure, flange 38 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).
- 112) Two-piece ostomy pouches Confort including:
- Flanges for two-piece ostomy pouches Confort, flange 40 mm.
 - Drainable two-piece ostomy pouches Confort, transparent, with filter, with Velcro closure, flange 40 mm (supplied on request no more than 10 pieces).
- 113) Ostomy belt Confort (supplied on request). Two-piece ostomy pouches Confort including:
- Flanges for two-piece ostomy pouches Confort, flange 45 mm.
 - Drainable two-piece ostomy pouches Confort, transparent, with filter, with Velcro closure, flange 45 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).
- 114) Two-piece ostomy pouches Confort including:
- Flanges for two-piece ostomy pouches Confort, flange 50 mm.
 - Drainable two-piece ostomy pouches Confort, transparent, with filter, with Velcro closure, flange 50 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).
- 115) Two-piece ostomy pouches Confort including:
- Flanges for two-piece ostomy pouches Confort, flange 55 mm.
 - Drainable two-piece ostomy pouches Confort, transparent, with filter, with Velcro closure, flange 55 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).
- 116) Two-piece ostomy pouches Confort including:
- Flanges for two-piece ostomy pouches Confort, flange 57 mm.
 - Drainable two-piece ostomy pouches Confort, transparent, with filter, with Velcro closure, flange 57 mm (supplied on request no more than 10 pieces).
 - Ostomy belt Confort (supplied on request).
- 117) Two-piece ostomy pouches Confort including:
- Flanges for two-piece ostomy pouches Confort, flange 60 mm.
 - Drainable two-piece ostomy pouches Confort, transparent, with filter, with Velcro closure, flange 60 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

118) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 70 mm.
- Drainable two-piece ostomy pouches Confort, transparent, with filter, with Velcro closure, flange 70 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

119) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 80 mm.
- Drainable two-piece ostomy pouches Confort, transparent, with filter, with Velcro closure, flange 80 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

120) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 100 mm.
- Drainable two-piece ostomy pouches Confort, transparent, with filter, with Velcro closure, flange 100 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

121) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 32 mm.
- Drainable two-piece ostomy pouches Confort, opaque, without filter, with Velcro closure, flange 32 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

122) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 35 mm.
- Drainable two-piece ostomy pouches Confort, opaque, without filter, with Velcro closure, flange 35 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

123) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 38 mm.
- Drainable two-piece ostomy pouches Confort, opaque, without filter, with Velcro closure, flange 38 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

124) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 40 mm.
- Drainable two-piece ostomy pouches Confort, opaque, without filter, with Velcro closure, flange 40 mm (supplied on request no more than 10 pieces).

- Ostomy belt Confort (supplied on request).

125) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 45 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, with Velcro closure, flange 45 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

126) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 50 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, with Velcro closure, flange 50 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

127) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 55 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, with Velcro closure, flange 55 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

128) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 57 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, with Velcro closure, flange 57 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

129) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 60 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, with Velcro closure, flange 60 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

130) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 70 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, with Velcro closure, flange 70 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

131) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 80 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, with Velcro closure, flange 80 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

132) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 100 mm.

- Drainable two-piece ostomy pouches Confort, opaque, without filter, with Velcro closure, flange 100 mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

133) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 32 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, with Velcro closure, flange 32 mm

(supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

134) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 35 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, with Velcro closure, flange 35 mm

(supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

135) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 38 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, with Velcro closure, flange 38 mm

(supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

136) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 40 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, with Velcro closure, flange 40 mm

(supplied on request no more than 10 pieces.

137) Ostomy belt Confort (supplied on request). Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 45 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, with Velcro closure, flange 45 mm

(supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

138) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 50 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, with Velcro closure, flange 50 mm

(supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

139) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 55 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, with Velcro closure, flange 55 mm

(supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

140) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 57 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, with Velcro closure, flange 57 mm

(supplied on request no more than 10 pieces.

26

- Ostomy belt Confort (supplied on request).

141) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 60 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, with Velcro closure, flange 60 mm

(supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

142) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 70 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, with Velcro closure, flange 70 mm

(supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

143) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 80 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, with Velcro closure, flange 80 mm

(supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

144) Two-piece ostomy pouches Confort including:

- Flanges for two-piece ostomy pouches Confort, flange 100 mm.
- Drainable two-piece ostomy pouches Confort, opaque, with filter, with Velcro closure, flange 100

mm (supplied on request no more than 10 pieces.

- Ostomy belt Confort (supplied on request).

2. INTENDED USE OF MEDICAL DEVICE SET BY THE MANUFACTURER

Stoma – is surgically formed opening of intestines and located at abdominal wall with purpose of evacuation of intestinal output or urine.

Stoma doesn't have sphincters and cannot control intestinal output discharge. Stoma doesn't have any nerves, so the patients do not feel any pain. Sometimes occurring feelings of itchiness or pain are associated with peristomal skin irritation or intestinal peristaltic.

Stoma surgery is performed when part of intestines removed or not functional due to inherited defect, trauma, or other disease.

Depending on kind of surgery performed there are different kinds of stomas: colostomy if large intestines are involved, ileostomy if small intestines were used to form stoma.

Choice of ostomy care appliances depends on the type and location of stoma, form of

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24

intestinal output, and patient's type of skin and personal preferences.

One-piece and two-piece ostomy pouches Confort in different production variants are used for care of the ostomy patients to compensate missing function and provide comfort conditions in stoma care procedures.

One-piece ostomy pouches Confort in all the production variety used for collection of intestinal stoma output.

Adhesive ostomy wafers Confort are used as a part of two piece system, are attached to peristomal skin, in order to protect is from stoma output. Two-piece drainable and closed pouches fixed to the wafers.

Two-piece pouches Confort in all production variety are indented for usage with Confort wafers (fixed to the wafers) in order to collect stoma output.

Clip for drainable pouches Confort is used to close bottom drainage hole of the drainable pouch.

Confort ostomy belt is used for additional fixation of two-piece ostomy pouches.

3. TECHNICAL DESCRIPTION

3.1. General description of medical device

3.1.1. One-piece ostomy pouch Confort – is a closed or drainable ostomy pouch with integrated adhesive plate and a clip, closure (if applicable). One-piece ostomy pouches are divided to drainable and notdrainable. Choice of the pouch depends on type and localization of stoma, peristomal skin condition.

Notdrainable (closed) pouches are commonly used by the patients with colostoma. 1-pc closed pouches are disposable, and supposed to replace when filled.

Drainable (open) pouches are commonly used by the patients with ileostoma. It's recommended to replace open pouches once a day. In case if it required to replace 1-pc drainable pouches more often it's recommended to switch to 2-pc systems. Drainable 1-pc pouches supplied with clips or have integrated Velcro closure in order to secure draining hole.

There's a carbon filter in inner side of the pouches effectively neutralizing odor and removing excessive gas from the pouch. The pouches manufactured with filter or without filter.

The pouches have backing from body side, made of unwoven material, which contributes to better air circulation, removal of sweat from the surface of the body.

Ostomy pouches are made of 5-layers film and water impermeable, inner surface of the pouches is smooth with low friction coefficient. The pouches are attached to the skin by integrated hydrocolloid adhesive plate, which provides adhesion to the skin all the period of usage untill replacement, hydrocolloid material allows to remove the plate without any adhesive residual.

Adhesive plate is made of nonwoven material with adhesive at one side and hydrololloid adhesive ring in the center, diameter of adhesive ring can comply with the diameter of the plate. Protection paper has a ring marks showing diameter of the starter hole and maximal possible opening, and also allows the patient make cut-to –fit opening.

Detailed information on materials used is in Appendix A "Materials"

Schematic picture of the closed (not drainable) one-piece ostomy pouch Confort is presented at picture 1, Appendix B.

Schematic picture of open (drainable) one-piece ostomy pouch Confort is presented at picture 2, Appendix B

3.1.2. Adhesive ostomy wafers Confort for two-piece ostomy system wafers. Adhesive wafers are attached to the skin around stoma to provide maximal protection. Modern materials used allow fast adhesion to the skin and ease of application. The wafer is easely adapted to any body type and abdomen structure.

Adhesive plate is made of nonwoven material with adhesive at one side and hydrololloid adhesive ring in the center, diameter of adhesive ring can comply with the diameter of the plate.

Protection paper from body side has sizing contours in order to facilitate cutting opening according to stoma size and shape.

Backside of the wafer has additional neck connected to the flange ring. Two-piece pouches Confort are attached to the wafers. Reliable fixation of the pouch to the flange allows not to remove the pouch during bath, shower or sport. The wafer is usually worn from two to five days and should be usually replaced once or twice a week.

Detailed information on materials used is in Appendix A “Materials”

3.1.3. Two-piece ostomy system Confort consists of adhesive wafers and pouches which are connected using reliable flange ring, size of the ring at the wafer should be same with the ring size of the pouch. The flange ring have connectors for belt. Two-piece ostomy pouches Confort can be open (drainable) or closed (not drainable). Drainable pouches are supplied with clip or have intergrated Velcro closure in order to secure drain hole. Closed pouches have filter at inner side, which effectively neutralizes odor and removes eccessiver gas to prevent pouch ballooning. The pouches manufactured with filter and without filter options. Both drainable and notdrainable pouches can be transparent and opaque. Transparent pouches allow control stoma output.

Soft comfort backing from nonwoven material contributes to better air circulation, removal of sweat from the surface of the body. Nonwoven backing materialis absords skin moisture and easy to dry after shower. The pouches are made of 5-layers film and water impermeable, inner surface of the pouches is smooth with low friction coefficient.

Detailed information on materials used is in Appendix A “Materials”

3.1.4. Clip is used to close drainable pouches. It allows to close the pouches after draining and drain the pouches without detaching them from skin.

3.1.5. Confort ostomy belt is used for additional fixation of two-piece ostomy appliances. In two piece ostomy system bags are fixed to wafers by flange ring, which has special connectors for the belt. Belt usage allows to decrease load to the wafer and increase wearing time as well as increase self-confidence of the patients.

Detailed information on materials used is in Appendix A “Materials”

3.2 Manufacturing specification.

3.2.1. One-piece ostomy drainable and notdrainable pouches comply to the following sizing and other requirements:

Length of drainable and not-drainable pouches, with closed clip or Velcro closure is 240mm. Allowed length deviation ±10%;

Length of drainable pouches with straight drainage hole not using clip or Velcro closure is – 280mm. Allowed length deviation ±10%;

Width of ostomy pouches is 135mm to 150mm. Allowed width deviation is ±10%; Width of the drain hole of the drainable pouches is 35mm. Allowed width deviation is ±10%;

Size of the carbon filter is 65mm (±6mm) x 13mm (±2mm).

Recommended volume capacity of 1-pc ostomy pouches is 650 ml (± 50ml).

Weight of 1-pc ostomy pouch – 20,0 g (±2,0g).

Diameter of the starter hole - 10 mm (± 2mm).

Maximall diameter of the cut-to-fit hole is - 32 mm, 55 mm, 60 mm, 70 mm, 80 mm and 100 mm (± 1mm).

Diameter of the plate (microporous adhesive) – 100 to 150 mm.

Thickness of hydrocolloid of of 1-pc ostomy pouches is 0,6 mm (± 0,2mm).

Diameter of hydrocolloid adhesive ring of 1-pc ostomy pouches 55 to 130 mm:

Maximal diameter of cut-to-fit hole, mm*	Diameter of hydrocolloid ring, mm**
---	-------------------------------------

32	55
55	90
60	90
70	100
80	110
100	130

* ($\pm 1\text{mm}$)

** ($\pm 2\text{mm}$)

3.2.2. Two-piece ostomy wafers Confort comply to the following sizing and other requirements:

Starter hole diameter - 10 mm ($\pm 2\text{mm}$).

Maximall diameter of the cut-to-fit hole is - 32 mm, 35 mm, 38 mm, 40 mm, 45 mm, 50 mm, 55 mm, 57 mm, 60 mm, 70 mm, 80 mm and 100 mm ($\pm 1\text{mm}$).

Diameter of the plate (diameter of microporous adhesive) – 62 to 130 mm.

Diameter of hydrocolloid adhevis 60 to 140 mm.

Thickness of hydrocolloid adhesive of 2-pc wafers is 0,6 mm ($\pm 0,2\text{mm}$).

Inner diameter of the flange ring is 30 to 100 mm.

Size (diameter) of the flange-ring wafer comlies to the size of the pouch flange-ring.

Diameter of hydrocolloid adhesive ring of 2-pc ostomy pouches depends on size of the wafer:

Maximal diameter of cut-to-fit hole, mm*	flange microporous adhesive diameter, mm**
32	62
35	65
38	68
40	70
45	75
50	80
55	85
57	100
60	100
70	100
80	110
100	130

* ($\pm 1\text{mm}$)

** ($\pm 2\text{mm}$)

Weight of the wafer 8,0g ($\pm 0,8\text{g}$).

3.2.3. Two-piece ostomy pouches Confort comply to the following sizing and other requirements:

Length of drainable and not-drainable pouches, with closed clip or Velcro closure is 240mm. Allowed length deviation $\pm 10\%$;

Length of drainable pouches with straight drainage hole not using clip or Velcro closure is – 280mm. Allowed length deviation $\pm 10\%$;

Width of ostomy pouches is 135mm to 150mm. Allowed width deviation is $\pm 10\%$; Width of the drain hole of the drainable pouches is 60mm. Allowed width deviation is $\pm 10\%$;

Size of the carbon filter is 65mm ($\pm 6\text{mm}$) x 13mm ($\pm 2\text{mm}$).

Recommended volume capacity of 1-pc ostomy and urostomy pouches is 650 ml ($\pm 50\text{ml}$).

Inner diameter of the flange ring is 30 to 100 mm.

Size (diameter) of 2 p-c ostomy pouches flange-ring complies to the size (diameter) of the Confort adhesive wafer flange-ring.

Inner diameter of flange ring on the pouch complies to the appropriate size of the ring on the wafer:

- 32mm, 35mm, 38mm, 40mm, 45mm, 50mm, 55mm, 57mm (± 1 mm),
- 60mm, 70mm, 80mm, 100mm (± 2 mm).

Weight of 2-pc ostomy pouch – 8,0 g ($\pm 1,0$ g).

3.2.4. Clip for drainable pouches Confort comply to the following sizing and other requirements:

Length – 90mm (± 9 mm), width – 10mm (± 1 mm), weight 1,5r ($\pm 0,5$ g).

3.2.5. Ostomy belt Confort comply to the following sizing and other requirements:

Maximal length 140 cm (± 10 cm), width – 30mm (± 3 mm), weight – 22,0g ($\pm 2,2$ g). Maximal stretch of Ostomy belt Confort not less than 50%. Rupture force of the Ostomy belt Confort over 98N. Ostomy belt locks strength is over 78N.

3.2.6. Clip provides safe closure and resists over 20 opening closure cycles with angle 0° to 180° . Minimal effort to open/close the clip for drainable pouches Confort is over 1N.

3.2.7. Adhesion strength of the hydrocolloid and microporous adhesive of 1-pc ostomy or urostomy pouch and 2-pc wafer Confort should be over 70 N/m.

3.2.8. Connection of the plate with pouch of 1-pc ostomy or urostomy pouch and connection of the flange of 2-pc wafer with the pouch resists load over 20N.

3.2.9. One-piece ostomy pouches and two-piece flange and wafer (connected), closed Velcro closures, clips provide impermeability for output.

3.2.10. Velcro lock strength in the working direction not less than 30N.

3.2.11. Maximal pouch rupture resistance in dry and wet condition of the pouch – not less than 10MPa.

4. MEDICAL DEVICE USAGE

4.1. Medical device usage conditions

One-piece and two-piece ostomy pouches Confort in different production variants are used in-patient and for home care of ostomy patients.

4.2. Medical device application area

One-piece and two-piece ostomy pouches Confort in different production variants are technical rehabilitation devices replacing function of missing organ in patients with intestinal stomas after surgery to remove part of whole intestines.

4.3. Indications and contrindication for usage

4.3.1. Indications

To collect intestinal stoma output.

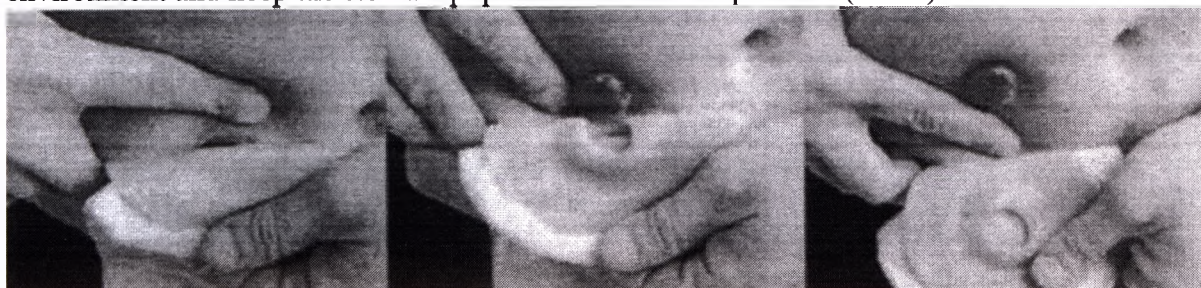
4.3.2. Contrindications

Individual intolerance

4.4. Usage instructions

4.4.1. USAGE INSTRUCTION OF ONE PIECE CLOSED (NOT DRAINABLE) OSTOMY POUCHES CONFORT, TRANSPARENT, OPAQUE, WITH FILTER, WITHOUT FILTER.

Collect your materials. Make sure that all necessary materials are ready for use. Collect a measuring guide, a pair of scissors, a pen, wet and dry gauze and a pouch. Warm up the hydrocolloid to body temperature to make sure that the hydrocolloid bonds well to the skin. Choose the right size. Measure your stoma with a measuring guide and cut (if necessary) the flange to the right size of the stoma. Make sure that you choose the right size so that the skin or stoma will stay in healthy shape. Allow a maximum of 2 mm of space between your stoma and flange for a safe and correct fit. If the size is too big, stoma output will damage the skin. If the size is too small, the flange will damage the stoma. Over time your stoma may change shape or size, so periodic measurements may be required to prevent leakage. Prepare your skin. Wash the skin surrounding your stoma with warm and unperfumed water. Do not use anything that contains alcohol, cologne or ether to clean your skin as it can cause damage. Do not rub the skin too hard as this might lead to skin damage. Dry the skin carefully and thoroughly to ensure that your pouch adheres well. If your skin is irritated and sore you can use skin barrier cream or wipes. Apply the skin barrier cream or wipe gently to the surrounding skin. Apply your pouch. Take your pouch and make sure that your pouch is secure and control the complete seal of the pouch with your fingers. When you change 1-pc pouch, support the skin with one hand while removing the pouch with other hand. Clean your stoma and the area around the stoma on the pouch. Replace your new pouch and control the complete seal of the pouch with your fingers. Dispose your pouch. Remove your pouch and dispose of in the trash. Do not flush your used pouch down the toilet because this will block your sewage system. Store your pouch dry. Always store your stoma equipment in a dry environment and keep the stoma equipment in a safe temperature (0-27°).

4.4.2. USAGE INSTRUCTION OF ONE PIECE OPEN (DRAINABLE) OSTOMY POUCHES CONFORT, TRANSPARENT, OPAQUE, WITH FILTER, WITHOUT FILTER, WITH VELCRO CLOSURE

Collect your materials Make sure that all necessary materials are ready for use. Collect a measuring guide, a pair of scissors, a pen, wet and dry gauze and a stoma and pouch. Warm up the hydrocolloid to body temperature to make sure that the hydrocolloid bonds well to the skin. Choose the right size. Measure your stoma with a measuring guide and cut (if necessary) the flange to the right size of the stoma.

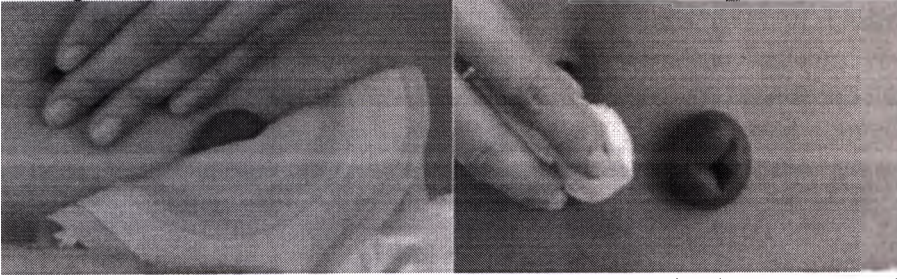
Make sure that you choose the right size so that the skin or stoma will stay in healthy shape. Allow a maximum of 2 mm of space between your stoma and flange for a safe and correct fit. If the size is too big, stoma output will damage the skin. If the size is too small, the flange will damage the stoma. Over time your stoma may change shape or size, so periodic measurements may be required to prevent leakage. Prepare your skin. Support the skin and remove the hydrocolloid carefully, do not tear but push the hydrocolloid of the skin with wet gauze. Wash the skin

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surrounding your stoma with warm and unperfumed water. Do not use anything that contains alcohol, cologne or ether to clean your skin as it can cause damage.

Do not rub the skin too hard as this might lead to skin damage. Dry the skin carefully and thoroughly to ensure that your pouch adheres well. If your skin is irritated and sore you can use skin barrier cream or wipes. Apply the skin barrier cream or wipe gently to the surrounding skin. Close Velcro closure, by rolling up the top of the pouch and firmly press it to ensure that the closure is secure, or close the clamp. Apply your pouch. Take your pouch and make sure that your pouch is secure and control the complete seal of the pouch with your fingers. When you drain the pouch. Lift up the top and unroll the Velcro closure or open the clamp. Empty the contents of your pouch into the toilet. Make sure that Velcro closure is clean, then roll it up and press to ensure Velcro closure is secure or close the clip and control that the clip is secure. When you change 1-pc pouch, support the skin while removing the pouch with your other hand. Clean your stoma and the area around the stoma on the pouch. Replace your new pouch and control the complete seal of the pouch with your fingers.

Dispose your pouch Remove your pouch and dispose of in the trash. Do not flush your used pouch down the toilet because this will block your sewage system. Store your pouch dry. Always store your stoma equipment in a dry environment and keep the stoma equipment in a safe temperature (0-27°)

4.4.3. USAGE INSTRUCTION OF TWO PIECE CLOSED (NOT DRAINABLE) OSTOMY POUCHES CONFORT, TRANSPARENT, OPAQUE, WITH FILTER, WITHOUT FILTER.

Collect your materials Make sure that all necessary materials are ready for use. Collect a measuring guide, a pair of scissors, a pen, wet and dry gauze and a stoma base plate and pouch. Warm up the base plate to body temperature to make sure that the hydrocolloid bonds well to the skin. Choose the right size Measure your stoma with a measuring guide and cut (if necessary) the flange to the right size of the stoma. Make sure that you choose the right size so that the skin or stoma will stay in healthy shape. Allow a maximum of 2 mm of space between your stoma and flange for a safe and correct fit. If the size is too big, stoma output will damage the skin. If the size is too small, the flange will damage the stoma. Over time your stoma may change shape or size, so periodic measurements may be required to prevent leakage. Prepare your skin.

Support the skin and remove the base plate carefully, do not tear but push the flange of the

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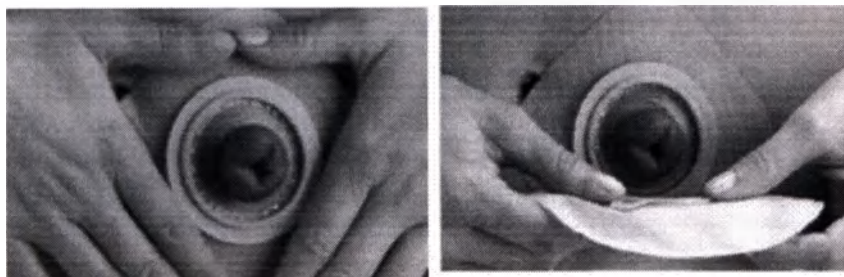
02

skin with wet gauze. Wash the skin surrounding your stoma with warm and unperfumed water. Do not use anything that contains alcohol, cologne or ether to clean your skin as it can cause damage. Do not rub the skin too hard as this might lead to skin damage. Dry the skin carefully and thoroughly to ensure that your base plate adheres well. If your skin is irritated and sore you can use skin barrier cream or wipes. Apply the skin barrier cream or wipe gently to the surrounding skin. Fit the wafer. Remove the release paper from the wafer. Stretch the skin of your abdomen with a flat hand and place the base plate over your stoma into the right position (start at the bottom and work your way to the top).

Work your fingers around the base plate so that the hydrocolloid bonds well onto the skin and make sure that the plate is secure. Fit your pouch. Take your pouch and click the pouch onto the ring of base plate. Work your way from the bottom to the top of the ring. Make sure that your pouch is secure and control the complete seal of the pouch with your fingers. Change your pouch. If you change your pouch, make sure that your base plate will stay secure. Support the wafer with one hand while removing the pouch with another hand. Clean your stoma and the area around the stoma on the base plate. Replace your new pouch onto the ring of the base plate and control the complete seal of the pouch with your fingers. Dispose your pouch. Remove your pouch and dispose of it in the trash. Do not flush your used pouch down the toilet because this will block your sewage system. Store your pouch dry. Always store your stoma equipment in a dry environment and keep the stoma equipment in a safe temperature (0-27°).

4.4.4. USAGE INSTRUCTION OF TWO PIECE OPEN (DRAINABLE) OSTOMY POUCHES CONFORT, TRANSPARENT, OPAQUE, WITH FILTER, WITHOUT FILTER, WITH CLIP, WITH VELCRO CLOSURE

Collect your materials. Make sure that all necessary materials are ready for use. Collect a measuring guide, a pair of scissors, a pen, wet and dry gauze and a stoma base plate and pouch. Warm up the base plate to body temperature to make sure that the hydrocolloid bonds well to the skin. Choose the right size. Measure your stoma with a measuring guide and cut (if necessary) the flange to the right size of the stoma. Make sure that you choose the right size so that the skin or stoma will stay in healthy shape. Allow a maximum of 2 mm of space between your stoma and flange for a safe and correct fit. If the size is too big, stoma output will damage the skin. If the size is too small, the flange will damage the stoma. Over time your stoma may change shape or size, so periodic measurements may be required to prevent leakage. Prepare your skin. Support the skin and remove the base plate carefully, do not tear but push the flange of the skin with wet gauze. Wash the skin surrounding your stoma with warm and unperfumed water. Do not use anything that contains alcohol, cologne or ether to clean your skin as it can cause damage. Do not rub the skin too hard as this might lead to skin damage. Dry the skin carefully and thoroughly to ensure that your base plate adheres well. If your skin is irritated and sore you can use skin barrier cream or wipes. Apply the skin barrier cream or wipe gently to the surrounding skin. Fit your base plate. Remove the release paper of the base plate. Stretch the skin of your abdomen with a flat hand and place the base plate over your stoma into the right position (start at the bottom and work your way to the top). Work your fingers around the base plate so that the hydrocolloid bonds well onto the skin and make sure that the plate is secure. Fit your pouch. Take your pouch and click the pouch onto the ring of base plate. Work your way from the bottom to the top of the ring. Make sure that your pouch is

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51

secure and control the complete seal of the pouch with your fingers. Close Velcro closure, by rolling up the tap of the pouch and firmly press it to ensure that the closure is secure, or close the clamp. Empty your pouch. Lift up the tap and unroll the Velcro closure or open the clamp. Empty the contents of your pouch into the toilet. Make sure that Velcro closure is clean, then roll it up and press to ensure Velcro closure is secure or close the clamp and control that the clamp is secure. Change your pouch. If you change your pouch, make sure that your base plate will stay secure. Support the base plate with your hand while removing the pouch with your other hand. Clean your stoma and the area around the stoma on the base plate. Replace your new pouch onto the ring of the base plate and control the complete seal of the pouch with your fingers. Dispose your pouch. Remove your pouch and dispose of in the trash. Do not flush your used pouch down the toilet because this will block your sewage system. Store your pouch dry. Always store your stoma equipment in a dry environment and keep the stoma equipment in a safe temperature (0-27°).

4.4.5. CONFORT WAFERS USAGE INSTRUCTION

4.4.5.1. Cut the wafer according to your stoma size and shape using sizing guide. Remove protection paper from the hydrocolloid before applying the wafer to the skin. Stretch the skin of your abdomen with a flat hand and place the base plate over your stoma into the right position (start at the bottom and work your way to the top). Attach the wafer to even skin. Work your fingers around the base plate so that the hydrocolloid bonds well onto the skin and make sure that the wafer is secure. Connect pouch of proper size to the wafer. The wafer may stay attached to the skin for several days, Duration of usage depends on type of stoma and skin. If the plate is firmly attached to the skin and there are no leakages it should not be replaced when the pouches are drained or replaced.

4.4.5.2. You may take shower not removing the base plate carefully wipe the plate with soft tissue after the shower.

4.4.5.3. Plate removal – plate should be replaced if it starts detaching from the skin or you feel skin irritation because of stoma output leakage under the plate. When you use two-pieces pouches, first remove the pouch. Then remove the plate carefully, pull the plate with one hand, holding skin with another.

Attention:

- In order to avoid skin damage do not remove the plate with a sharp movement.
- The plates are disposable and can't be reused.

Always store your stoma appliances in a dry environment and in a safe temperature (0-27°).

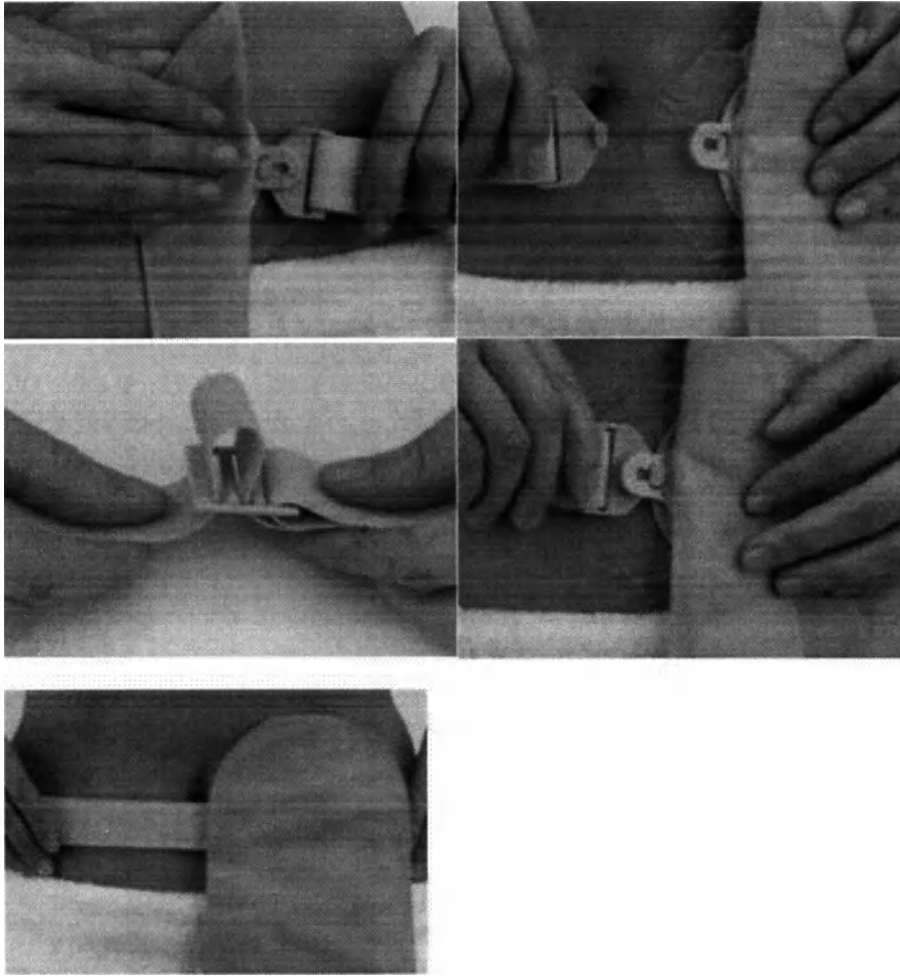
4.4.6. CONFORT OSTOMY ACCESSORIES USAGE INSTRUCTION

4.4.6.1. CLIP FOR DRAINABLE POUCHES

Remove protection paper from sticky side of the clip. Place the clip at the center of drainable pouch lower part. The clip should be placed from outer side of the pouch not from body side. Roll up drainable part of the pouch 3 times directing from yourself. Firmly press the edges of the clip. Take open clip with one hand, bend open edge of the pouch towards inner part of the clip and press till click confirming the clip is safely closed.

4.4.6.2. OSTOMY BELT CONFORT

Connect the belt to the ring. Adjust length of the belt if needed. Make sure that the belt connected to other end of the ring. Connect the belt to the pouch.

5. Packaging

One-piece ostomy pouches Confort are packed into cardboard boxes, 30 pieces in a box.

Usage instruction and stoma measuring guide inserted into all boxes.

Wafers for 2-pc ostomy pouches Confort are packed 5 pieces in a cardboard box.

2-pc ostomy pouches Confort packed into cardboard boxes 20 pieces in a box.

Clips Confort are packed 20 pieces per cardboard box.

Confort ostomy belt – 1 piece in a cardboard box.

6. Package

The package is a cardboard box and provides protection of the medical devices during transportation and storage. Individual cardboard boxes packed into corrugated cardboard transport boxes, 40, 60 pieces per box or according customer's demand.

7. LABELING

7.1. Each item has the following labeling:

- Manufacturer
- item code
- batch code (same as a production date)

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25

Batch codes allow to identify production information connected to the product, including day of production, all the materials batch numbers. Batch numbers allow identify market destination of the batch.

7.2. Cardboard labeling (individual package):

- Manufacturer's name and contacts;
- Distributor's in Russia name and contacts;
- Item, model;
- Item code (special symbol is marked);
- Batch code (special symbol is marked);
- Production date (special symbol is marked);
- Expiration date (special symbol is marked);
- Storage and handling instructions (special symbols marked);
- Quantity of individual packages in a box (special symbol is marked);
- Disposal information;
- Compliance mark;
- CE mark;
- Registration certificate number.

7.3. Corrugated cardboard (transport package) labeling:

- Manufacturer's name and contacts;
- Distributor's in Russia name and contacts;
- Item, model;
- Item code (special symbol is marked);
- Batch code (special symbol is marked);
- Production date (special symbol is marked);
- Expiration date (special symbol is marked);
- Storage and handling instructions (special symbols marked);
- Quantity of individual packages in a box (special symbol is marked);
- Disposal information;
- Compliance mark;
- CE mark;
- Registration certificate number.

Symbols used:

item numner

batch number (same as a manufacturing date)

expiration date...

storage temperature (from 0 to +27°C)

humidity (45 to 70 %)

air pressure (84,0 to 106,7 kPa)

avoid direct sunlight

protect from humidity

quantity in a box, transportation box

boxes in a stack

compliance mark

CE - mark

8. TRANSPORTATION, STORAGE, DISPOSAL

8.1. Transportation

The product can be transported by any closed transportation except not heated airplane compartments.

UN no. None

ADR None

IMDG None

8.2. Storage

Store the product in dry premises, not closer than 1 m to heating devices, without direct sunlight, with temperature 0-27°, and relative humidity 45 to 80 %.

8.3. Disposal

8.3.1. Ecotoxicological data have not been determined specifically for this product. The product is not biodegradable. The product is stable in the environmental conditions. Extreme heating may cause toxic fumes.

8.3.2. Used ostomy appliances should be wrapped into plastic bag and put into garbage container. Strictly not recommended to throw into sewage to avoid obstruction.

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24

Product poses no immediate hazard, but can give rise to toxic fumes in a fire. **Fire**

fighting risks: Smoke, soot, CO/CO₂, and other toxic fumes.

Extinguishing media: Water, carbon dioxide or foam.

Fire fighting safety measures: Keep personnel removed from and upwind of fire. Wear full fire fighting turnout gear and respiratory protection.

The products are intended to be attached to the skin according usage instructions. **Do not dispose into natural water and canalization.**

9. MANUFACTURERS WARRANTY

9.1. Manufacturer guarantees compliance of the products to the standards requirements and production documents on condition of proper transportation, storage and usage.

9.2. One-piece ostomy pouches and two-piece ostomy pouches and wafers are disposable products. Ostomy belt and clip are multiple usage products. Expiration term is five years after production on condition to transportation, storage, usage requirements.

9.3. Manufacturer doesn't take responsibility for any harm caused by product usage until it proved.

9.4. Manufacturer doesn't take responsibility for any harm caused by reuse of disposable products.

9.5. Manufacturer doesn't take responsibility for any harm caused by usage of expired products.

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Phone: 00 90 222 236 03 27 – 00 90 222 236 03 29 Fax: 00 90 222 236 03 23

www.confortostomi.com

Warranty obligations and claims in Russia should be addressed to the representative in charge
“S.P.HELPIK” ltd., 117997, Moscow, Profsouznaya st., 86 b. 2.

Phone +7-496-334-8269, Fax +7-496-334-9509.

IR 4TH
PUBLIC
BAŞOL

CADDESİ
SARAYI
85 KAT:1
UNPAZARI/
ŞEHİR
222313511
222333176

I hereby certify the signature under this certification transaction (Notary Public) the signature of **SEÇİL ÇOBAN ERGÜN**, Personal Identification Number: **28006668826**; acting as **COMPANY EXECUTIVE** for "**KONFORT ÖZEL SAĞLIK HİZMETLERİ TIBBİ MALZEMELER SANAYİ TİCARET LİMİTED ŞİRKETİ**" Co., Ltd. (VAT Reg No: 5760481237); whose father's name is **Osman**, mother's name is **Selma**; date of birth is **1/1/1983**, place of birth is **Eskişehir**; registered to civil registry at **Isparta Province, Isparta Centre County, Kurtuluş Quarter/village, 21 volume, 20 family serial no, 141 sequence no** according to the Identity Card bearing photograph issued by **Odunpazarı Civil Registry Office** on the date **01/08/2012**, with **9787** registration, **A12** serial and **840087** number and who declares that she still resides at above mentioned address and she is literate and she signed before me in my office. Year Two Thousand and Seventeen, Twelfth day of October-Thursday- **12/10/2017**.

SUPPORTING DOCUMENT: By the examination of list of authorized signatures certified by **ESKİŞEHİR 4th Notary Public** with date **12/10/2012** and **23350** roll number, it is seen that person named **SEÇİL ÇOBAN ERGÜN** is authorized to represent the company named **KONFORT ÖZEL SAĞLIK HİZMETLERİ TIBBİ MALZEMELER SANAYİ TİCARET LİMİTED ŞİRKETİ** for a period of ten years from **15/12/2011** to **15/12/2021**.

Seal and signature
Deputy of **ESKİŞEHİR 4th NOTARY PUBLIC**
GÜLTEN BAŞOL
Head Clerk
Mustafa KOSKU
12th October 2017

Vat, FEE, Stamp Tax and Valuable Paper cost were collected against voucher.

I hereby certify this document is true and accurate English translation of the original Turkish document.

Translator

I hereby certify the signature under is Turkish to English translation translated in conformity with the original Turkish document according to the

Signature

Signature

NOTARY CERTIFICATION TEXT

I hereby certify the signature under is Turkish to English translation translated in conformity with the original Turkish document according to the signature of the translator is the signature of the translator of our office, **ERKAN UYSAL**, T.R Id Number: **54409285254**, who was born in **1978**, son of **MESUT**, registered to birth at **Kütahya Province, Domaniç county, HİSAR quarter, 315 family sequence**, who resides in **Vişnelik Serap Sokak No:1 Daire 3-Odunpazarı-Eskişehir**.

A copy of the text is enclosed to this translation at my request. Concerned Person

ESKİŞEHİR 4TH NOTARY PUBLIC
GÜLTEN BAŞOL
(Official seal and signature)

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