State of Minnesota

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SECRETARY OF STATE

APOSTILLE

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1. Country: United States of America

This Public Document: Instructions for Use (IFU) for 3M ESPE Clinpro

2. has been signed by Lori J Sheetz

3. acting in the capacity of Notary Public, State of Minnesota

4. bears the seal/stamp of Lori J Sheetz, Notary Public, State of Minnesota

CERTIFIED

5. at: St. Paul, Minnesota

6. Date: March 22, 2017

7. by: Secretary of State, State of Minnesota

8. File No: 9419642-12

9. Seal/Stamp:

10. Signature:

Steve Simon

Steve Simon
Secretary of State

3M ESPE
Dental Products

2510 Conway Avenue
St. Paul, MN 55144-1000

To Whom It May Concern:

In my capacity as Regulatory Affairs Specialist, 3M ESPE Dental Products, St. Paul, MN, USA, **I certify that the attached English version of the Instructions for Use (IFU) for 3M ESPE Clinpro™ XT Varnish intended for Russia, is truthful and accurate.**

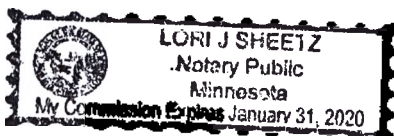
Shari L. Myszka
Shari L. Myszka, R.Ph. (Pharm.D.)
Regulatory Affairs Specialist
3M ESPE Dental Products

March 20, 2017
Date

STATE OF MINNESOTA)
COUNTY OF RAMSEY) ss

On March 20, 2017, before me, a Notary Public within and for said county, personally appeared Shari L. Myszka, known by me to be the person described in and who executed the foregoing instrument and acknowledged that she executed the same as her own free act and deed.

Lori J. Sheetz
Lori J. Sheetz, Notary Public
My Commission Expires, January 31, 2020

General Information

3M™ ESPE™ Clinpro™ XT Varnish Durable Fluoride-Releasing Coating is a site- specific, protective coating for enamel and dentin tooth surfaces. It is a light-cured glass ionomer material in a two-part liquid/paste system. The liquid/paste materials are contained in the 3M™ ESPE™ Clicker™ Dispensing System. This dispensing system provides simultaneous dispensing of each component for a consistent mix. The paste contains a radiopaque fluoro-aluminosilicate glass. The liquid contains a modified polyalkenoic acid. Clinpro XT varnish provides the major benefits of glass ionomer materials including adhesion to tooth structure and sustained fluoride and calcium phosphate release. Clinpro XT varnish offers a combination of a prolonged working time with a short set time achieved by light curing.

Indications

- Protective coating for treatment of tooth sensitivity caused by exposed root surfaces
- Site specific protective coating for high risk, caries prone areas (e.g. around orthodontic brackets, non-cavitated lesions, acid erosion prone surfaces)
- Temporary protective coating for immature enamel (e.g. newly erupting molars)

Contraindications

- Not intended as a replacement for long-term conventional sealants
- Not to be used under orthodontic brackets
- Not to be used in patients with dry mouth (e.g. patients who may awaken with a dry mouth due to open-mouth breathing)

Precautionary Information for Patients

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known acrylate allergies. If prolonged contact of uncured material with oral soft tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product.

Precautionary Information for Dental Personnel

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If product contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed.

3M ESPE MSDS information can be obtained from www.3MESPE.com or contact your local subsidiary.

Instructions for Use

Preparation for Exposed Root Surface Sensitivity Applications

1. Thoroughly clean tooth surface(s) to remove plaque and debris. Rinse thoroughly with water.

Note: Do not use any cleaning medium that may contain oils.

2. Remove pooled water and saliva, but leave tooth slightly moist.
3. Avoid water and saliva contamination during application of this material.

Preparation for site specific tooth surface coating applications; around orthodontic brackets, newly erupting teeth, non-cavitated lesions, and acid erosion prone surfaces

Note: This product is to be used after the orthodontic bracket is light cured or allowed to completely self cure. It is not to be used under or prior to orthodontic bracket placement.

1. Thoroughly clean tooth surface(s) to remove plaque and debris. Rinse thoroughly with water.

Note: Do not use any cleaning medium that may contain oils.

2. Remove pooled water and saliva, but leave tooth slightly moist.
3. Avoid water and saliva contamination during application of this material.
4. Apply a generous amount of 35% phosphoric acid etchant to the enamel surface ensuring entire area is etched.
5. Etch for a minimum of 15 seconds, but no longer than 60 seconds.
6. Thoroughly rinse teeth with air/water spray to remove etchant.
7. Optional: For etch verification, the dry etched surface should appear matte and frosty.

Dispensing

1. Remove cap from the Clicker dispenser by holding down the cap lever and sliding the cap off of the dispenser.
2. Dispense a small amount of material onto a mix pad to ensure even dispensing of both pastes. Discard this material.
3. Fully depress the lever of the Clicker dispenser to dispense "1 click" of the varnish on to the mix pad. Release lever when paste stops extruding (1-2 seconds). Repeat dispensing process for additional material.
4. Wipe the dispenser tips clean with alcohol-dampened gauze.
5. Replace protective cap immediately after dispensing. Hold the sides of the Clicker dispenser cartridge and slide cap into place until securely latched as indicated by the audible "click."
- Do not depress the lever of the Clicker dispenser during cap removal and/or replacement.
- Do not advance the dispenser plunger during cap removal and/or replacement.
- An approximate number of Clinpro XT coating applications remaining in the Clicker Dispensing System are indicated on the underside of the plunger adjacent to the black arrow on the cartridge.

Mixing

1. Using a small spatula, mix paste/liquid components together rapidly (10-15 seconds). The mixed material should have a smooth consistency and glossy appearance.
2. To minimize water evaporation and maximize working time, confine mixing of the paste/liquid to a small area on the mixing pad, about one inch (2.5 cm) in diameter.

Application

Note: Avoid water and saliva contamination during application.

1. Apply a thin layer (1/2mm or less) of the mixed material to the tooth surface using a ball applicator, brush, or other suitable instrument. **Note:** Avoid prolonged contact of uncured material with soft tissues. If accidental contact occurs, immediately wipe off with wet cotton roll.
2. This product has a minimum working time of 2 minutes 30 seconds at a room temperature of about 73°F (23°C). Higher temperatures will shorten working time.

Light Curing

1. Light cure material by exposing layers of 1/2mm or less for 20 seconds to a 3M ESPE curing light, or other curing unit of comparable intensity.
2. **The delayed** auto-setting mechanism of this material will ensure an eventual cure of material shielded from light polymerization such as undercut areas.
3. Trim any ledges or overhangs with a blade or bur. If necessary, the coating can be removed with the use of a coarse prophylactic paste or pumice.

Storage, use and shelf life

This material is light sensitive. Protect from ambient light exposure by dispensing just prior to use and replacing Clicker dispenser cap immediately after dispensing.

Do not store materials in proximity to eugenol containing products.

This product is designed for use at room temperature. If stored in cooler, allow product to reach room temperature prior to use. Shelf life at room temperature is 24 months. Ambient temperature routinely higher than 27°C/80°C may reduce shelf life. See outer package for storage and expiration dates. Do not use the product after expiration date.

Disinfect the capped Clicker dispenser using an intermediate level disinfection process (liquid contact) as recommended by the CDC (Center for Disease Control) and endorsed by the ADA (**American Dental Association**). **Guidelines for Infection Control in Dental Health-Care Settings** - 2003 (Vol 52; No. RR-17), Centers for Disease Control and Prevention.

Conditions of use of medical device:

The medical device should be used only by qualified dental professionals who have experience with similar products. The medical device should be used in specialized premises in accordance with good aseptic technique.

Operating conditions:

This material is light sensitive. Protect from ambient light exposure by dispensing just prior to use and replacing Clicker dispenser cap immediately after dispensing. Do not store materials in proximity to eugenol containing products.

Storage conditions:

This product is designed for use at room temperature. If stored in cooler, allow product to reach room temperature prior to use. Shelf life at room temperature is 24 months. Ambient temperatures routinely higher than 27°C/80°F may reduce shelf life. See outer package for storage and expiration dates.

This medical device isn't subject to maintenance and repair.

Transportation conditions:

Shelf life of the products is guaranteed as labeled, if stored according to the recommended storage conditions. During transportation the products may experience short term temperature variations without affecting product safety and performance. Transportation of medical device is carried out by all type of transport.

Disposal methods:

Dispose of contents/container in accordance with the local/regional/national/ international regulations.

Composition

Paste A

Methacrylates, Water, Silanized glass, substituted amino benzoate

Paste B

Methacrylates, Water, Calcium Glycerophosphate, camphoroquinone, iodonium salt, butylated hydroxyl toluene

The dimensions of Clicker dispenser:

Size 198,12±5,1 x 25,4±1,3 x 35±5,1 mm

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture.

3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the products for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Authorised representative of the manufacturer in RF

ZAO "3M Russia", Russia 121614, Moscow, Krylatskaya str. Bldg. 17/3
T +74957847474

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