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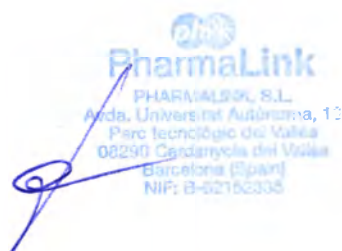
e-mail: info.ru@xantispharma.com

**Patient information leaflet
Medical device user guide**

“Nasal hypertonic spray SANORIN AQUA SEA WATER, 30 ml”

Medical device name

Nasal hypertonic spray SANORIN AQUA SEA WATER, 30 ml

Composition and description of medical device

Seawater. Aqua purified. Non-sterile.

Prescription

Used for prevention and complex treatment of rhinitis, including SARS.

For adults and children over 6 years.

Used in hospitals and at home.

Domain of usage

Otolaryngology

Indications for use

- Prevention and complex treatment of rhinitis, including SARS. To restore nasal breathing in rhinitis and nasal congestion.
- Moisturizing nasal mucosa under conditions of excessively dry air in the Central heating systems and air-conditioning systems during air travel.
- Preparation of nasal mucosa for nasal application of medicaments.
- For daily hygiene of the nasal cavity and nasopharynx (including the unfavorable ecological situation).
- Prevention of runny nose and protect nasal cavity from irritating environmental factors (dust, tobacco smoke, allergens).

Application method

When cold it is necessary to clear the nose before doing the procedure.

- Open the package and remove the protective cap;
 - Gently lift the nasal spray to move;
 - Click on the adapter of the spray and spray its contents;
 - Allow to drain the excess solution, and then, if necessary, blow your nose;
 - After cleaning the nostrils, follow the same procedure for the second nasal passage, which ensures complete removal of mucus and provides a good breath.
- Spend 2-6 sprays daily in each nostril.

Contraindications

Use is contraindicated in patients with a previously installed hypersensitivity to any components included in the composition of the medical device.

Side effect

Discontinue use if irritation develops.

Consult a physician if irritation persists or there is a need to use medical device "Nasal hypertonic spray SANORIN AQUA SEA WATER, 30 ml" regularly.

National standards used by manufacturer

Medical device made in accordance with the standards:

- NS-EN ISO 13485:2012/ISO 13485:2003 No. 153346-2014-AQ-IBE-NA, the certificate gives the right to develop, manufacture and sale of non-sterile solutions to cleanse the nose and outer ear, as well as on the production of nasal sprays for children. The certificate is issued by DNV Business Assurance organization located at Veritasveien 1, 1322 Høvik, Norway
- DIRECTIVE of EUROPEAN COUNCIL 93/42/EEC of 1993 regarding medical products (MDD 93/42/EEC) Amended by DIRECTIVE 2007/47/EEC of 2007.
- The state Pharmacopoeia of the Russian Federation XIII.
- European Pharmacopoeia 8.0 (similar indicators of the global Fund of the Russian Federation XIII).

Precautions for use

Before applying, it is needed to read the instructions.

For hygienic reasons and to prevent the transmission of pathogenic microorganisms, one person should only use the bottle.

Use only in an upright position.

Product description and packaging

Spray in a bottle with a volume of 30 ml, instructions for use are placed in carton packaging.

Conditions of transportation and storage

Store in a dry, inaccessible for children place at temperature from +5°C to +25°C.

Shelf life

3 years.

Do not use the solution more than 1 month after opening the bottle.

Do not use after the expiry date.

Prescription status

No prescription.

Utilization

In accordance with Sanitary Regulations and Standards 2.1.7.2790-10 "Sanitary-epidemiological requirements for the handling of medical waste" medical device "Nasal hypertonic spray SANORIN AQUA SEA WATER, 30 ml" refers to A class risk.

Collection class A waste is carried out in reusable containers or disposable bags. Color packs can be any except yellow and red. In the home is disposed of as household waste.

Manufacturer

«PHARMALINK S. L.», Avinguda Universitat Autònoma 13, 08290 Cerdanyola del Valles, Barcelona, (Spain) telephone number +34 666 570 401.

Organization authorized by the manufacturer to receive claims from consumers

LLC «Xantis Pharma»

Legal address: Russia, 117186, Moscow, Nagornaya str., build. 20, 1

Current address: 115035, Moscow, Sadovnicheskaya nab., 75

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This Apostille only certifies the authenticity of the signature and the capacity of the person who has signed the public document, and, where appropriate, the identity of the seal or stamp which the public document bears.

This Apostille does not certify the content of the document for which it was issued.

[This Apostille is not valid for use anywhere within Spain]

[To verify the issuance of this Apostille, see <https://eregister.justicia.es>]

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