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INSTRUCTIONS

for use of optical frames
with demo lenses: metal, plastic, combi-
nation, rimless , semi-rimless with acces-
sories: pattern, tape for insertion, nylon
cord

1. INTENDED USE:

Optical frames with demo lenses: metal,
plastic, combination, rimless , semi-
rimless with accessories: pattern, tape for
insertion, nylon cord « Wenzhou Dong-
tian Glasses Manufacturing Co.,LTD.»
company production are intended for lens
fixation and correct lens position in rela-
tion to eyes.

2. TECHNICAL SPECIFICATIONS AND DESIGN

Optical frames consist of front, temples,
pads, hinges. Hinges can be single and
spring hinges.

Optical frames are characterized by 3 main
geometric parameters: width (a), bridge
width (b) and length of temples to ear bend
(L).

Sizes used for frames of « Wenzhou
Dongtian Glasses Manufacturing
Co.,LTD.» company production are given
in the table below:

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There shall be no sharp edges and burrs at the frame surface, shells, scratches, blow-holes, solder beads seen by unaided eye.

Temples shall turn smoothly without sticking along the axis.

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Accessories:

Pattern - technological element, which is installed in the machine allows you to grind the lens to the desired shape, to be installed in a frame without gaps and tightness.

Height $17 \pm 0,2$ mm, width $55 \pm 0,2$ mm, height $17 \pm 0,2$ mm, width $51 \pm 0,2$ mm, height $17 \pm 0,2$ mm, width $50 \pm 0,2$ mm, height $16 \pm 0,2$ mm, width $48 \pm 0,2$ mm, thickness $1,5 \text{ mm} \pm 0,2 \text{ mm}$.

Nylon cord - frames element needed for attaching the lens. It fits into the holes in the frame in front of the lens insert.

Length 90 ± 5 mm, diameter $0,7 \pm 0.05$ mm.

Tape for insertion - technological element, which allows the line to fill in the groove made on the lens.

Length 90 ± 10 mm, width 5 ± 2 mm, thickness $0,5 \text{ mm} \pm 0,02 \text{ mm}$

Optical frames are selected considering anthropometric data of a patient.

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3. Indications for use:

Correction of defects in the optics of the eyes:

- myopia (myopia) - short-sightedness,
- Presbyopia (presbyopia) - hyperopia
- astigmatism (astigmatismus)
- Hypermetropia (hypermetropia), a weak type of clinical refraction
- anisometropia (difference in refractions on the eyes)
- keratoconus and others.

К The following can be considered as COUNTERINDICATION FOR USE:

- idiosyncrasy of vision correction during glasses wearing;
- impossibility of vision correction during glasses wearing.

4. APPLIED MATERIALS:

Plastic frames are produced from cellulose acetate Ecco Leaves JB 21000. Metal, combination and rimless frames are produced from copper nickel alloy monel KA 500 or Beta -Titanium (74% Ti - Titanium, 22% V- Vanadium, 4 % Al – Aluminium), temple tips are produced from cellulose acetate Ecco Leaves JB 21000, pads are produced from silicone LSR50 shore.

Semi-rimless frames are produced from copper nickel alloy monel KA 500 or Beta - Titanium (74% Ti - Titanium, 22% V- Vanadium, 4 % Al –Aluminium), temple tips are produced from cellulose acetate Ecco Leaves JB 21000, pads are produced from silicone LSR50 shore, retaining cord is produced from nylon polymer Nilor (nylon line),

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patterns are produced from Polycryl Clear, tape for insertion is produced from nylon polymer Nilor.

Demo lenses are produced from Polycryl Clear acryl.

For production of hinges, screws, nuts and rivets, use:

Copper-nickel alloy (Monel KA 500), Beta-Titanium (74% Ti- Titanium, 22% V- Vanadium, 4% Al-Aluminum).

Materials meet biological safety requirements.

5. MARKING:

Optical frames of « Wenzhou Dongtian Glasses Manufacturing Co.,LTD.» company production are marked on temples. Manufacturer's name is applied on the right temple. Technical parameters are applied on the left temple: horizontal width of the installed lens (a), bridge width (b) and full length of temples; mode code or description; color or coating code.

Due to the diversity of models and different sizes of temples are not may be point out the manufacturer on the temples at the specified model code (in accordance with EU rules).

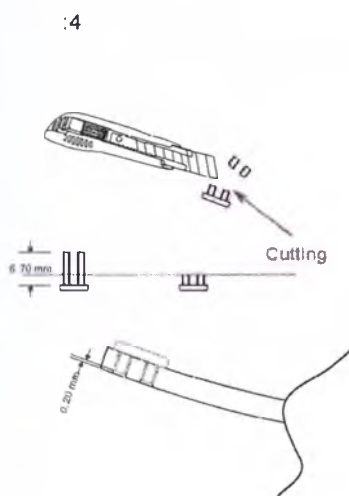
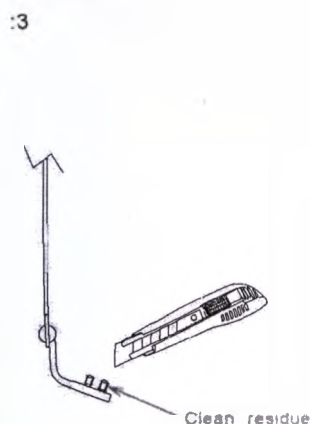
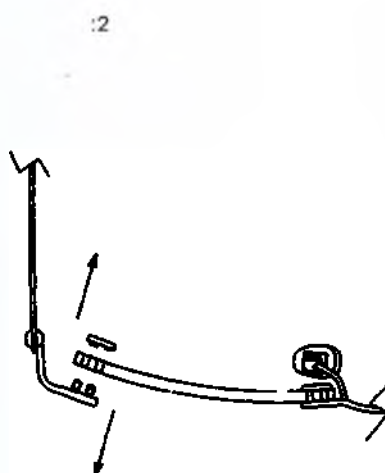
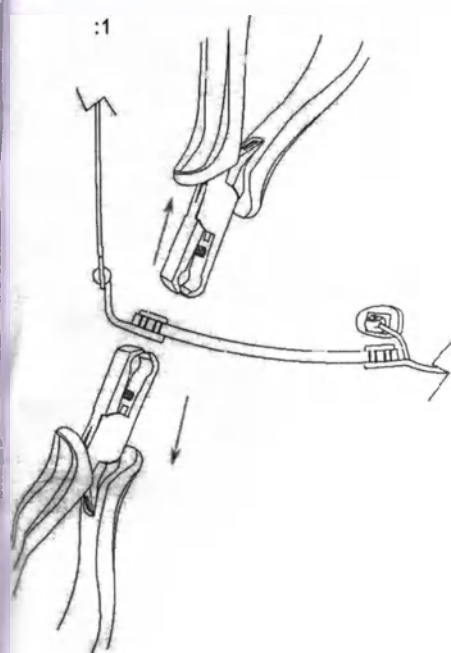
6.PRODUCT PREPARATION FOR USE:

Optical frames shall not lose the initial form following lenses insertion into it. Eye wires shall be symmetrical. Chipping, scratches, gaps between spectacle lenses and eye wires as well as other defects of glasses assembly are not allowed. Spectacle lenses inserted in the frame shall not displace and fall out from the lens shapes upon standard use of glasses.

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

Assembling pin connection of rimless
frame

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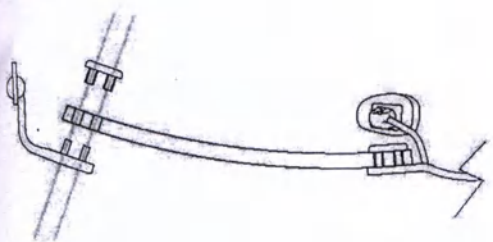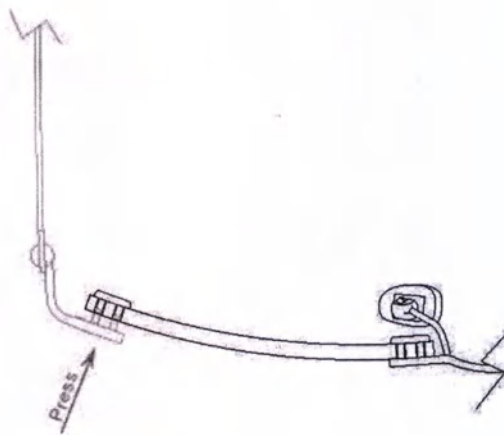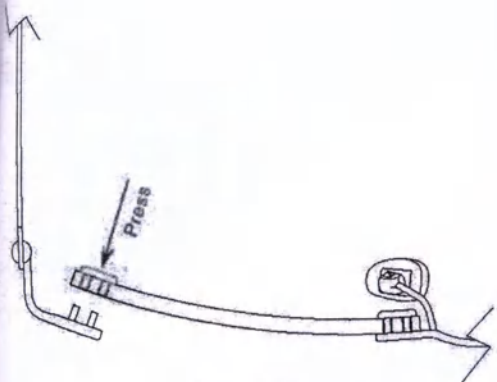
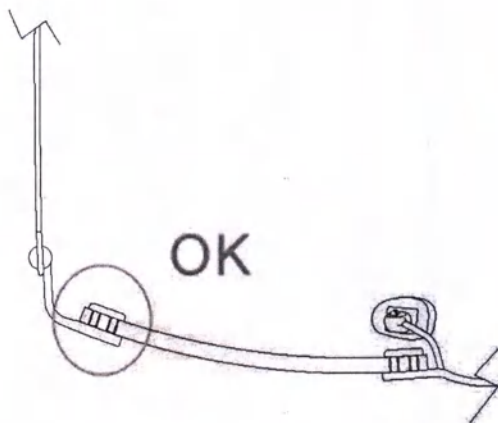
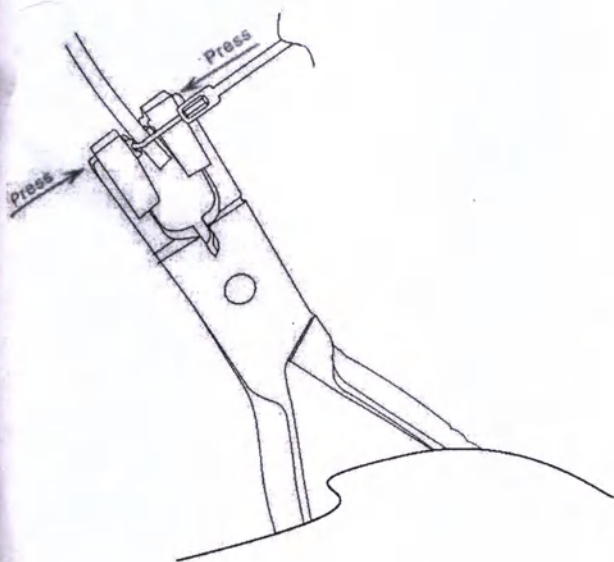
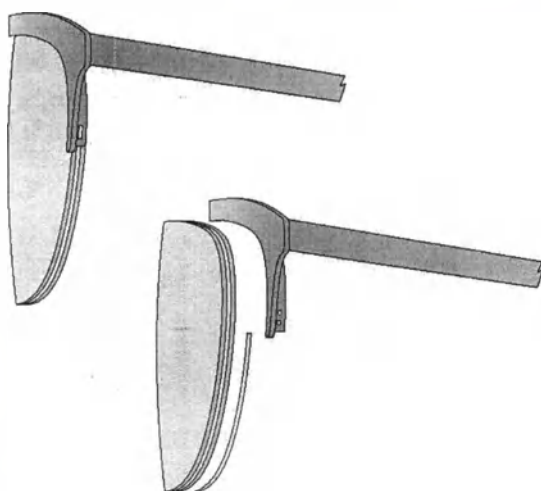
Attaching a lens to the frame by means of
retaining cord in frame semi-rimless.

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- significant mechanical forces shall not be
applied to the frame;

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- the frame should be kept in a case or oth-
er package excluding possibility of me-
chanical damage;

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- total service time of the frame – 3 years
minimum.

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- Storage conditions: in heated premises
upon ambient temperature from +5° C to
+50° C, 80% relative air humidity, atmos-
pheric pressure - 700-1060 hPa.

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- impact of aggressive substances (acids,
alkalis) and organic solvents or their fumes
is not allowed during storage.

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Optical frames are applied in all climatic
zones, at any time of day or night.

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9. SAFETY PRECAUTIONS

Upon use of optical frames the care should be taken upon sports activities and work performance intending sharp head movements, inclinations, jumps and impacts.

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10. POTENTIAL SIDE EFFECTS WHEN APPLYING

No side effects upon use of the medical product have been revealed.

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11. INFORMATION ON POTENTIAL EFFECTS OF MEDICAL PRODUCT ON ABILITY TO DRIVE AND USE MACHINES

For drivers having problems with sight the optical frames enable them to drive and use machines (upon fabrication of corrective glasses from frames).

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13. MAINTENANCE AND REPAIR

Optical frames shall be repaired in special laboratories licensed for production, maintenance and repair.

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14. WARRANTY LIABILITIES

Frames shelf life: plastic, combined frames — 5 years; metal frames — 10 years.

Total frames service time is 3 years minimum with maximum 30000 number of temple swings.

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25.10.2017

15. TRANSPORTATION

Transportation of optical frames is per-
formed using all transportation modes in
covered vehicles as per haulage rules ef-
fective on the given transportation mode.

Temperature - 45 ° C to + 50 ° C

Relative humidity 98%

Atmospheric pressure 700-1060 hPa.

16. DISPOSAL

Optical frames shall be disposed in the
same way as household waste.

17. CERTIFICATES

The manufacturer uses an internal quality
system, which is annually reviewed and
updated.

18. MANUFACTURER

« WENZHOU DONGTIAN GLASSES
MANUFACTURING CO.,LTD.»

Registered address: NO.16 Jugao Road,
Juguang Garden, Wenzhou, Zhejiang, Chi-
na,

tel: 0086 577 5557 5736

fax: 0086 577 8606 2033.

19. AUTHORISED REPRESENTATIVE:

«RICO Ltd.»,

registered address :

Russian Federation, Moscow, 121096,
2-nd Filevskaya street 7, bld., 6

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(2017) WZ.Z. No. 5335

Applicant: Wenzhou Dongtian Glasses Manufacturing Co., Ltd.

Address: No.16, Jugao Road, Juguangyuan, Wenzhou City

Legal Representative: Tian Zhiwu, male, born on October 17, 1965, ID Card No. 330302 19651017 1612

Issue under Notarization: seal and signature

This is to certify that Tian Zhiwu, legal representative of Wenzhou Dongtian Glasses Manufacturing Co., Ltd., came to our notary public office, and affixed his signature and company seal to the foregoing INSTRUCTIONS for *use of optical frames* in the presence of me, the notary, this 27th day of October 2017.

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(2017) WZ.Z. No. 5336

Applicant: Wenzhou Dongtian Glasses Manufacturing Co., Ltd.

Address: No.16, Jugao Road, Juguangyuan, Wenzhou City

Legal Representative: Tian Zhiwu, male, born on October 17, 1965, ID Card No. 330302 19651017 1612

Issue under Notarization: true and exact translation

This is to certify that the attached English translation of NOTARIAL CERTIFICATE (2017) WZ.Z. No. 5335 conforms to the original document in Chinese.

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