

Bisco

CE0459

TheraCal[®] LC

Light-
Cured

Resin-Modified Calcium Silicate Pulp Protectant/Liner

Instructions for Use
and
Material Safety Data Sheet
(MSDS)







BISCO
 BISCO, Inc.
 1100 W. Irving Park Rd.
 Schaumburg, IL 60193
 U.S.A.
 847-534-6000
 1-800-BIS-DENT

IN THE USA
 1-800-BIS-DENT

Outside the USA
 847-534-6000

Product of BISCO, Inc.
 1100 W. Irving Park Rd.
 Schaumburg, IL 60193
 U.S.A.

Product of BISCO, Inc.
 1100 W. Irving Park Rd.
 Schaumburg, IL 60193
 U.S.A.

Product of BISCO, Inc.
 1100 W. Irving Park Rd.
 Schaumburg, IL 60193
 U.S.A.

BISCO CE0459

TheraCal LC® Light-Cured
 Resin-Modified Calcium Silicate Pulp Protection Liner

TECHNIQUE 1 Pulp Exposure (Direct Pulp Capping)

1 Under rubber dam isolation, complete cavity preparation.

2 Core seal dressing by placing a TheraCal LC liner over the pulp exposure. Blot preparation with cotton pellet. Leave visibly moist.

Bisco

BISCO, Inc.
 1100 W. Irving Park Rd.
 Schaumburg, IL 60193
 U.S.A.
 847-534-6000

BISCO France
 125, avenue de la Chapelle
 93000 La Chapelle
 France
 33-1-40-40-62-62



BISCO, Inc.
1100 W. Irving Park Rd.
Schaumburg, IL 60193
U.S.A.
647-534-6000

BISCO France
120, allée de la Coudoulette
13680 Lançon de Provence
France
33-4-90-42-92-92

BG-100R7

