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## English

### Intended use

Assay for the quantitative in vitro determination of platelet function triggered by arachidonic acid. The reagent is intended for use in platelet function testing with whole blood samples on the Multiplate analyzer.

### Summary

Arachidonic acid is an omega-6 fatty acid that is found in many of the body's cell membranes. The conversion of arachidonic acid to thromboxane is regulated by the enzyme cyclooxygenase (COX). This enzyme exists in two forms: COX-1 the constitutive form found in all tissues and COX-2 which (COX). Cette enzyme existe sous deux formes: COX-1, la forme constitutive em thromboxane é régulé

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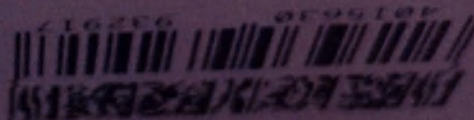
English

Intended use

Assay for the quantification of platelet aggregation following stimulation of the platelet adenosine diphosphate (ADP) receptors. The reagent is intended for use in platelet aggregation testing with whole blood samples on the Multiplate analyzer.

## Summary

Adenosine diphosphate (ADP) is a nucleotide that is stored in the dense granules of platelets. Upon activation, platelets undergo shape change and release the contents of their alpha and dense granules. The release

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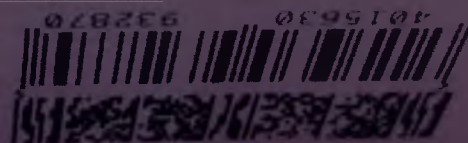
## English

### Intended use

Assay for the quantitative in vitro determination of platelet aggregation by collagen. The reagent is intended for use in platelet function tests with whole blood samples on the Multiplate analyzer.

### Summary

Collagen is a protein found in the connective tissue of humans. It is exposed to connective tissue such as the skin or a blood vessel, collagen is exposed and freely circulating platelets immediately adhere to it.

(COL test)

# COLtest

REF 06675824 190

REF 06675832 190

1 x → 1.0 mL

3 x → 1.0 mL

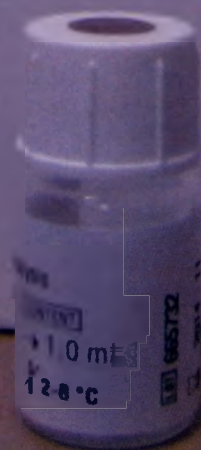
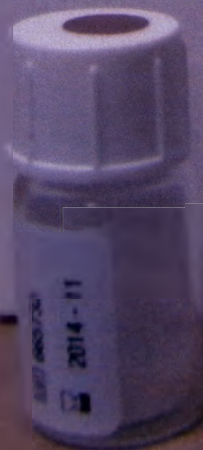
## English

### Intended use

Assay for the quantitative in vitro determination of platelet function triggered by collagen. The reagent is intended for use in platelet function testing with whole blood samples on the Multiplate analyzer.

### Summary

Collagen is a protein found in the connective tissue of humans. Upon injury to connective tissue such as the skin or a blood vessel, collagen becomes exposed and fresh circulating platelets immediately attach themselves

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English

## Intended use

Assay for the quantitative in vitro determination of von Willebrand Factor (VWF)- and glycoprotein Ib (GPIb)-dependent platelet aggregation with whole blood samples on the Multiplate analyzer.

## Summary

Ristocetin is an antibiotic known to induce thrombocytopenia and platelet agglutination. Platelets bind to von Willebrand Factor (VWF) by means of glycoprotein Ib (GPIb) receptors in the presence of ristocetin. In vitro ristocetin forms complexes with VWF which bind to GPIb and trigger platelet aggregation. In vitro, ristocetin forms complexes with VWF which bind to GPIb and trigger platelet aggregation. In vitro, ristocetin forms complexes with VWF which bind to GPIb and trigger platelet aggregation.

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English

## Intended use

Assay for the quantitative in vitro determination of platelet function triggered by TRAP-6. The reagent is intended for use in platelet function testing with whole blood samples on the Multiplate analyzer.

## Summary

Thrombin is capable of activating platelets, which is mediated primarily by the hydrolysis of a G-protein-coupled receptor on the platelet membrane, referred to as protease-activated receptor 1 (PAR-1) and a second receptor (PAR-4) peu sensible à la thrombine.

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English

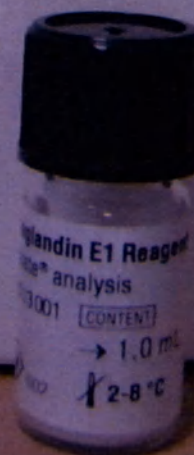
### Intended use

Reagent for the quantitative in vitro determination of platelet function triggered by adenosine diphosphate and Prostaglandin E1 Reagent (in conjunction with reagent ADPtest). The reagent is intended for use in platelet function testing with whole blood samples on the Multiplate analyzer. For the assessment of ADPtest HS (high sensitivity). For the assessment of positive (i.e. abnormal) controls of the ADPtest.

### Summary

Adenosine diphosphate (ADP) induced platelet aggregation is achieved through

Activation platelet aggregation is achieved through Adenosine diphosphate (ADP) so that

066745-42001V3

# ASA Reagent

REF 06675913 190

REF 06675921 190

English

1 x → 1.0 mL

3 x → 1.0 mL

## Intended use

The ASA Reagent (acetylsalicylic acid) is intended for use as quality control in platelet aggregation function testing with whole blood samples on the Multiplate analyzer. Addition of ASA Reagent to the blood sample leads to reduced aggregation responses in ASPItest and COLtest.

## Summary

Acetylsalicylic acid (ASA) is an acetyl derivative of salicylic acid that is a white, crystalline, weakly acidic substance. It is useful in the relief of headache and muscle and joint aches. Acetylsalicylic acid is also effective in reducing

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