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CERTIFICATE

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT:the seal of MDF
Instruments Medifriend Inc. on the annexed USER'S
MANUAL is genuine.

China Council for the Promotion
of International Trade

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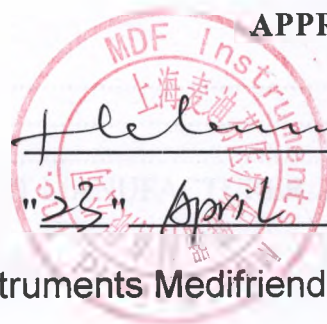
Authorized Cai Yiming
Signature:

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(Date:Apr. 25, 2018)

APPROVED

Heleum GM
"23" April 2018

MDF Instruments Medifriend Inc.

**USER'S MANUAL
on medical device**

**Sphygmomanometers MDF with accessories,
manufactured by MDF Instruments Medifriend Inc.**

Manufacturing address:

(1898 Lai Yin Road, building 6, floor 3 (including the 2nd floor, Building 6 and the
1st floor, Building 2),

Jiu Ting Town, Song Jiang District, Shanghai 201615, China)

Tel.: + 86.21.3763.3530, fax+ 86.21.3763.3532

<https://www.mdfinstruments.com>

Table of contents

REFERENCES.....	3
NAME OF MEDICAL DEVICE	3
MEDICAL DEVICE MANUFACTURER.....	3
AUTHORIZED REPRESENTATIVE OF THE MEDICAL PRODUCT MANUFACTURER.....	3
DESIGNATION OF MEDICAL DEVICE.....	3
SUPPLY PACKAGE	4
PRODUCT DESCRIPTION AND DESIGN.....	6
DESCRIPTION OF ACCESSORIES:.....	11
INDICATIONS FOR USE.....	11
CONTRAINDICATIONS	11
POSSIBLE ADVERSE REACTIONS	11
CONDITIONS AND OPERATING RULES	12
BLOOD PRESSURE MEASUREMENT.....	12
MEDICAL DEVICE CLASSIFICATION	14
PRECAUTIONS	14
VALUES OF MAIN PARAMETERS, CHARACTERISTICS (PROPERTIES) GUARANTEED BY MANUFACTURER.....	14
MINOR REPAIRS	16
TECHNICAL MAINTENANCE.....	16
PACKAGING	18
LABELLING	19
STORAGE AND TRANSPORTATION.....	23
DISPOSAL CONSIDERATIONS AND REQUIREMENTS FOR ENVIRONMENTAL PROTECTION.....	23
OPERATING CONDITIONS.....	24
WARRANTY LIABILITIES.....	24

ences

standards that were the basis for medical product and documentation developed and performing tests:

- GOST P 50444-92;
- GOST 31515.1-2012;
- GOST 31515.1-2012.

Name of medical device

MDF sphygmomanometers with accessories.

Medical device manufacturer

Organizational and legal form and the full name of the legal entity	MDF Instruments Medifriend Inc., China
Address (location) of the legal entity	1898 Lai Yin Road, building 6, floor 3 (including the 2nd floor, Building 6 and the 1st floor, Building 2), Jiu Ting Town, Song Jiang District, Shanghai 201615, China
Phone numbers	+ 86.21.3763.3530
E-mail address of the legal entity	https://www.mdfinstruments.com

AUTHORIZED REPRESENTATIVE OF THE MEDICAL PRODUCT MANUFACTURER

Organizational and legal form and the full name of the legal entity	Limited liability company "Medexpert Consulting"
The abbreviated name of the legal entity (if available)	LLC "Medexpert Consulting"
Address (location) of the legal entity	101000, Moscow, 10, Myasnitskaya str., building 1
Phone numbers	tel.: (495) 679-84-25
E-mail address of the legal entity	plotnikov@medexpert.ru

Designation of medical device

Device for non-invasive blood pressure measurement

...sphygmomanometer operation principle: when the pressure inside the cuff begins to exceed the blood pressure, the cuff itself begins to clamp the artery and the blood flow stops somewhat. When the pressure in the cuff begins to decrease, the artery opens and the blood flow through the arteries restores. If the artery lumen is small, the blood swirls begin to create Korotkoff's sounds, the pulsation of which is clearly heard in the stethoscope. When the artery is completely opened, sounds disappear.

Application conditions - the product can be used in all premises, including hospital and home premises.

A potential consumer, both a medical worker and an individual for self-sustained use, independently from medical personnel.

Scope of use - For using as a personal means of monitoring systolic and diastolic blood pressure, as well as for dynamic monitoring these parameters in medical centres and during home observation.

Supply package

MDF sphygmomanometers with accessories.

MDF sphygmomanometers, design versions:

1. MDF sphygmomanometer 808M consisting of:

- pressure gauge;
- compressed air pump;
- cuff MDF 2100-450

Accessories:

- wallet

2. Sphygmomanometer MDF 808B consisting of:

- pressure gauge;
- compressed air pump;
- cuff MDF 2100-450

Accessories:

- wallet

3. Sphygmomanometer MDF 848XPD consisting of:

- pressure gauge;
- compressed air pump;
- cuff MDF 2030-421

Accessories:

- wallet

Sphygmomanometer MDF 848XPD consisting of:
- pressure gauge;
- compressed air pump;
- cuff MDF 2100-451
Accessories:
- wallet

5. Sphygmomanometer MDF848XPD consisting of:
- pressure gauge;
- compressed air pump;
- cuff MDF 2100-451
Accessories:
- wallet

6. Sphygmomanometer MDF848IC consisting of:
- pressure gauge;
- compressed air pump;
- cuff MDF 2030-421
Accessories:
- wallet

7. Sphygmomanometer MDF848IC consisting of:
- pressure gauge;
- compressed air pump;
- cuff MDF 2100-451
Accessories:
- wallet

PRODUCT DESCRIPTION AND DESIGN

Product design

The sphygmomanometer consists of a pressure gauge, which measures the air pressure; a cuff fastened around the patient's arm; and an air pump with an adjustable release valve.

The devices are identical by circuit design and the components used.

The devices have the same designation.

The products are manufactured at the same manufacturer-enterprise, have the same metrological characteristics, differ in design features, furnishing and weight as discussed below:

MDF 808M, MDF 808B - pocket aneroid sphygmomanometer, with a cuff having two hoses, a pressure gauge and a pneumatic pressure injector (bulb) are connected to the cuff separately;

MDF 848XPD, MDF 848AR, MDF 848IC - manual aneroid sphygmomanometer, combined with a pneumatic pressure injector (bulb).


Sealing MDF sphygmomanometers with accessories is not provided.

No software.

Products are supplied non-sterile, sterilization is not required.

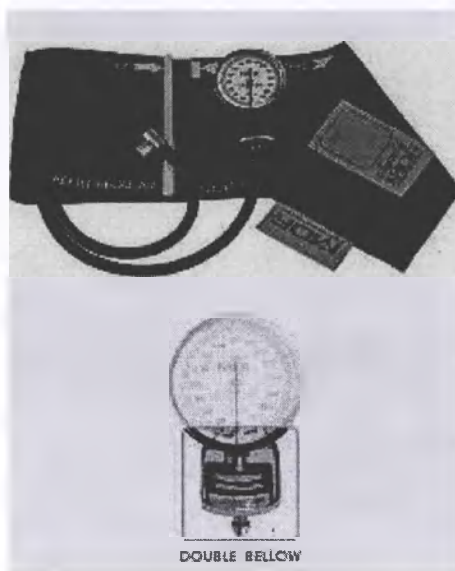
MDF 808M

MDF 808B

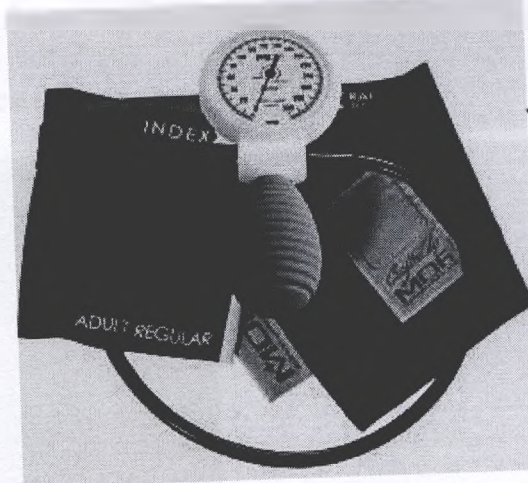MDF 848XPD

MDF 848AR

MDF 848IC

1. A cuff for blood pressure measurement

Photo 1

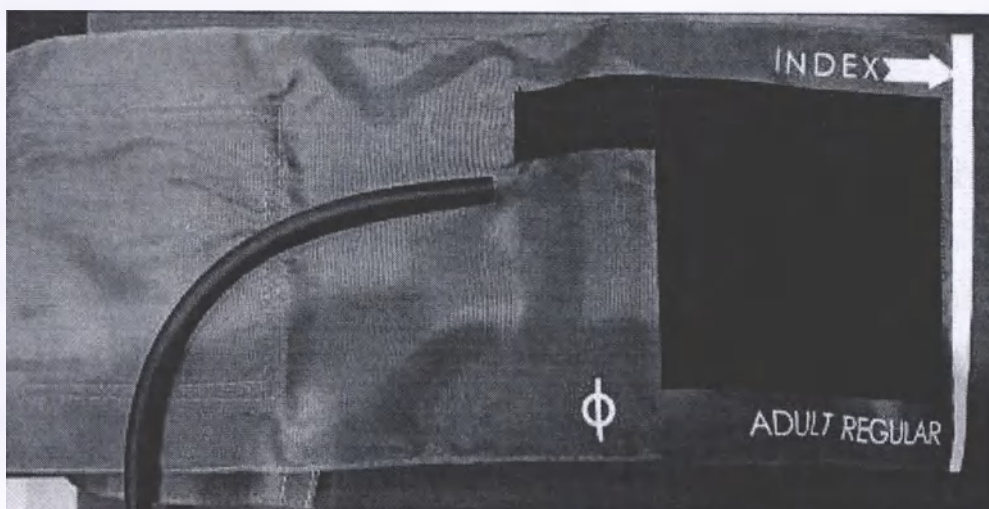
Photo 2

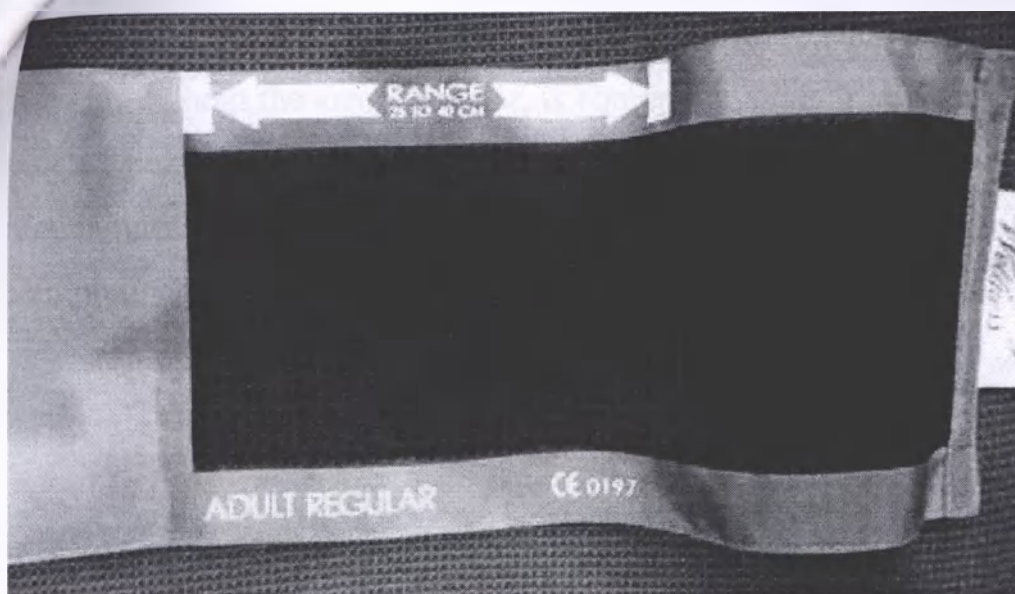
Photo 3

MDF cuff for blood pressure measurement consists of a cuff and a non-latex chamber. The cuff is resistant to abrasion, chemical exposure (chlorine and peroxide) and moisture. It is made of high-molecular nylon polymer. Cuff dimensions correspond to the American Heart Association recommendations, cuff labelling with

an indication of the artery position  (photo 2) and patient's limb size


(photo 3), allows fixing the cuff properly, ensuring correct compression of the artery. A seamless inflation chamber with one or two tubes is made of a crack resistant, non-sticky, high density, non-latex PVC material not causing allergic reactions. The universal tube of the camera ensures the compatibility of the cuff with all MDF models and the products of the key blood pressure measuring system manufacturers.

MDF non-latex cuff for blood pressure measurement provides greater comfort for patient and high performance. It is a hand-made device. MDF non-latex cuff for blood pressure measurement is designed to measure blood pressure by non-invasive method on healthy shoulder skin or thigh area.

2. Manometer

precise, certified pressure gauge with a range of up to 300 mm Hg with an error ± 3 mm Hg, without the arrow stopper, is equipped with an easily readable high-contrast clock-type dial. The cast zinc body of the pressure gauge has textured baked enamel coating and is equipped with a heavy-duty clamp, which is easily attached to the cuff clamping.

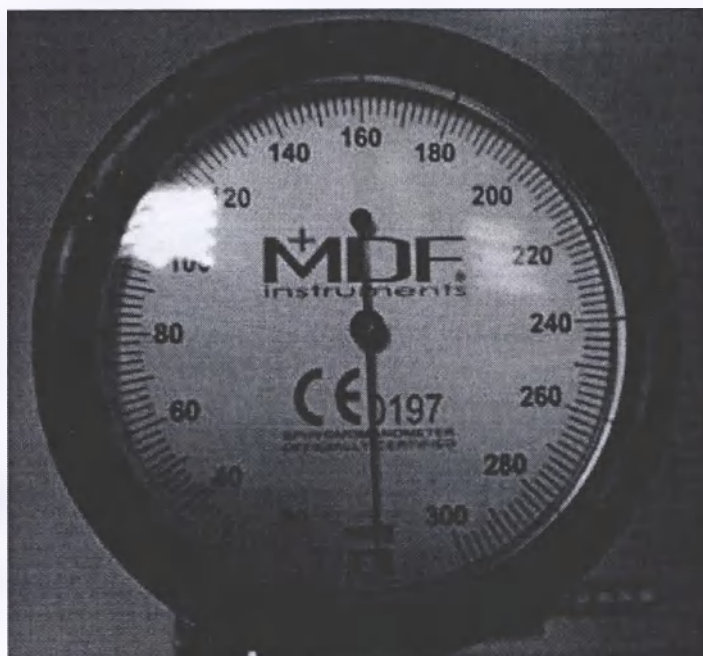Manometer


3. Air pump with an adjustable release valve

Chrome-plated copper screw valve provides the necessary air release rate. Air inflating bulb is made of a crack resistant, non-sticky, high density, non-latex PVC material not causing allergic reactions.

DESCRIPTION OF ACCESSORIES:

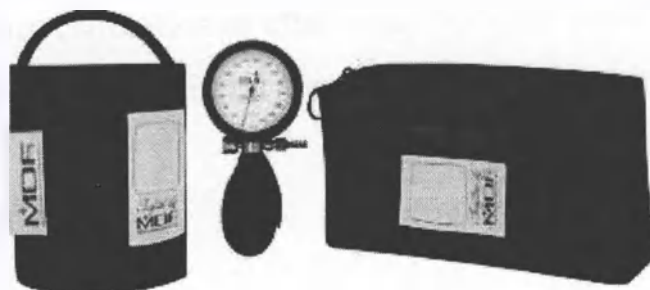Wallet

It is a cloth bag with a zipper for protection in storing and transposition of sphygmomanometers.

Indications for Use

Preventive measures and treatment of arterial hypertension.

Prevention and treatment of cardiovascular system diseases.

Therapeutic examinations.

Control of human medical status.

Contradictions

No.

Possible adverse reactions

Skin allergy in users.

Incorrect measurement can affect the result precision.

patient will feel uncomfortable if the sphygmomanometer is under pressure exceeding 270 mm Hg. If the pressure exceeds 300 mm Hg, the device will be damaged.

Conditions and operating rules

For the commissioning of sphygmomanometers, no special installation, assembling, adjustment, calibration or other operations are needed.

There are no special requirements for rooms where the sphygmomanometer will be used.

Selection of cuff size

Nylon cuffs with stickers.

On one side of the nylon cuffs a soft side of the touch fastener is located, and on the other there is a stiff side with hooks. This helps fasten and unfasten the cuffs quickly. All cuff models are equipped with level lines. To make sure that you have correctly selected the cuff size, verify that the white line on the worn cuff is between the arrows. If the white line does not reach the interval between the arrows, then the cuff is small. If the line is beyond this interval, then the cuff is too large.

Precise readings of pressure measurements can only be reached if the cuff size is correctly matched.

Blood pressure measurement

It is necessary to measure the blood pressure in quiet and comfortable environment for the patient, at an average room temperature, after the patient has adapted to the climatic conditions of the doctor's office, after staying there for at least 5-10 minutes. 1 hour before starting to measure the blood pressure, it is necessary to exclude any food intake, 1.5-2 hours before the measuring no smoking and taking tonic drinks, tea, coffee or alcohol.

- Put the cuff on the bare left forearm 2-3 cm above the elbow fold, then fasten the touch fastener.
- After you put on the cuff, pump it up by squeezing the bulb until the

... in the cuff exceeds 20 mm Hg (expected systolic blood pressure). The pressure reading is easy to read by means of a pressure gauge with a pressure scale and an arrow.

- Make sure that the patient whose blood pressure is measured is relaxed and seats in a comfortable position. It is necessary to bend the arm at the elbow, the wrist should be at the level of the heart with the inner side facing up.
- Attach the stethoscope olive to the artery under the cuff (in models for self-sustained use, the stethoscope olive is built into the cuff).
- In order to measure the blood pressure, loosen the trigger on the bulb by turning it counter-clockwise. Ideally, one air release run should correspond to 2-3 mm Hg per second. To achieve this, it is necessary to slowly turn the valve. Watch for the air release, monitoring the scale: the arrow speed should be 1-1.5 divisions per second.
- When the upper blood pressure value (systolic pressure) is reached, you will hear repetitive beats.
- When the lower pressure value (diastolic pressure) is reached, the beats are no longer audible.
- Then open the valve completely to release air from the cuff.
- The procedure for measuring the pressure is finished.

It should be pointed out that only a doctor can accurately evaluate measurements results that you have obtained.

Medical device classification

Depending on the potential risk of usage, the device is classified as Class 1.

Precautions

- It is forbidden to pump over 300 mm Hg pressure.
- Do not allow the cuff contact with sharp objects.
- Sphygmomanometer disassembling is forbidden.
- Avoid exposure or bumping to sphygmomanometer.
- Always empty the air completely after measurement and before storage.
- Avoid prolonged exposure to excessive heat, cold, solvents, moisture and

oils.

- It is forbidden to leave the inflated cuff on the shoulder or the thigh.
- Do not use more than 2 minutes.
- It is not allowed to use by children without adult supervision.
- Before each use, make sure that all parts are securely and correctly fastened.

For the accuracy and complete blood pressure picture, always use a stethoscope.

- Always consult a therapist.
- Only for medical purposes.

VALUES OF MAIN PARAMETERS, CHARACTERISTICS (PROPERTIES) GUARANTEED BY MANUFACTURER

13.1 Sphygmomanometer technical specifications:

	MDF 808M	MDF 808B	MDF 848XPD	MDF 848AR	MDF 848IC
Assembled device weigh, g, (not more)	510	525	420	300	460
Shipping weigh, g, (not more)	620	635	625	610	790
Pressure gauge overall dimensions, mm (± 2 mm)	$\varnothing 50$	$\varnothing 50$	$\varnothing 73$	$\varnothing 70$	$\varnothing 73.5$
Pressure gauge faceplate size, mm (± 2 mm)	$\varnothing 47.5$	$\varnothing 47.5$	$\varnothing 59.5$	$\varnothing 60$	$\varnothing 60$
Pressure gauge body thickness (± 2), mm	41	41	45	33	40
Length of connecting tubes,	600 - 2 pcs.		600 - 2 pcs.		

	MDF 808M	MDF 808B	MDF 848XPD	MDF 848AR	MDF 848IC
Outer diameter, mm (± 10 mm)					
Outer connecting tube diameter, mm	9		9		
Connecting tubes wall thickness, mm	2		2		
Cuff length, cm (± 0.5 cm)	560				
Cuff width, cm (± 0.5 cm)	145				
Pressure units millimeters of mercury	mm Hg				
Manometer scale graduation value, mm Hg	2				
Pressure measuring range, mm Hg	From 0 to 300				
Basic error of pressure measurement, mm Hg (not more)	+/- 3				
Additional error of pressure measurement, mm Hg (not more)	+/- 3				
Inflated air maximum pressure, mm Hg	300				
Maximum error in pressure measurement in the cuff at any point of the scale range, mm Hg.	± 3 (± 0.4 kPa)				
Decrease in readings (pressure drop) when the valve is closed, mm Hg per minute, not more	4 (0.5 kPa)				
Depressurization rate: regulation of depressurization rate in the cuff with the air bleed valve, mm Hg/s, not more	3 (0.4 kPa/s)				
Fast relief: pressure reduction time from 260 to 15 mm Hg (from 35 to 2 kPa) during rapid relief in the pneumatic system with the pressure relief valve fully open, s, not more	8				

13.2 Cuff characteristics:

Cuff Model	Circumference, cm (±1 cm)	Pneumatic chamber size LxW, mm	Weight, g (±5 g)
MDF 2030-421	19 - 25	190±10 x100±5	50
MDF 2100-450	25 - 40	220±20 x130±10	76
MDF 2100-451	25 - 40	220±20 x130±10	76

13.3 Specifications of the wallet for carrying the models MDF 808M, MDF 808B, MDF 848XPD, MDF 848AR, MDF 848IC:

Overall dimensions, mm (± 5 mm) - 230x60x135.

Weight, g, not more than 95.

13.4 Materials contacting (short-term contact) with intact human skin:

- nylon, manufactured by MDF: cuff sleeve;
- PVC (polyvinyl chloride), produced by MDF: air pump, flexible tubes.

Minor repairs

Device may only be repaired by a service centre authorized by the manufacturer.
Do not make repair yourself.

TECHNICAL MAINTENANCE

It is necessary to regularly check and remove dust from the screen filter of the inlet valve at the end of the pumping bulb.

General rules for cleaning and care:

Manometer

Prior to each use, carry out thorough cleaning of sphygmomanometer.

To clean the foreign matters on sphygmomanometer and stethoscope, use only clean cotton cloth soaked in the isopropyl alcohol solution or the water and mild detergent mixture. Water should not get inside the sphygmomanometer.

Cuff sleeve

The cuff sleeve may be hand washed in warm water with mild detergent. Dry in air. Mechanical drying is prohibited.

Disinfection with 3% hydrogen peroxide is allowed.

If necessary, the user can easily replace the sphygmomanometer, the chamber or the inflating bulb by simply unhooking them from the connector and replacing with another part.

Attention!

Do not handle the nylon surface of the cuff and the sticker with the iron!

Never expose cuffs to direct sunlight!


Never touch cuffs and latex parts with sharp tools, as they can damage the surface!

Packaging



Packing characteristics:

Model	Number/individual package	Number/paperboard container for inner packaging	The size of the individual package (LxWxH), mm, ± 2 mm
MDF 808M	1	12	235x150x64
MDF 808B	1	12	235x150x64
MDF 848XPD	1	14	235x150x64
MDF 848AR	1	14	235x150x64
MDF 848IC	1	14	235x150x64

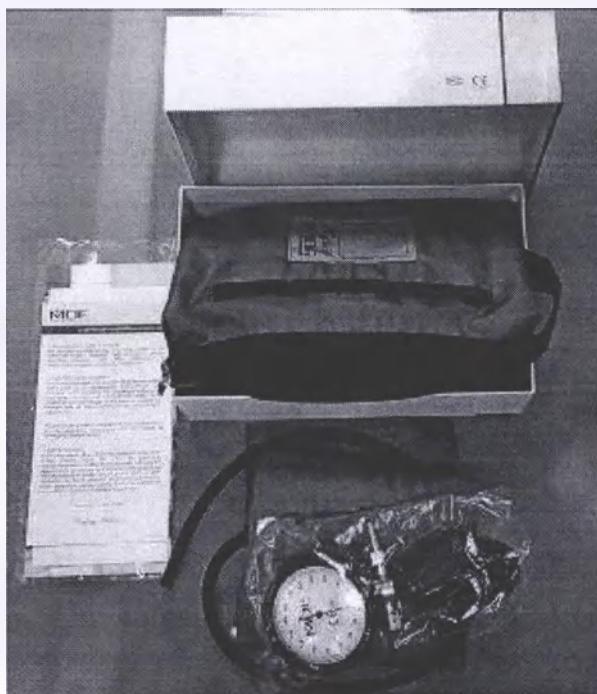Sphygmomanometer components are packed in a cloth bag, which is in turn inserted in a cardboard box not more than 2 mm thick.

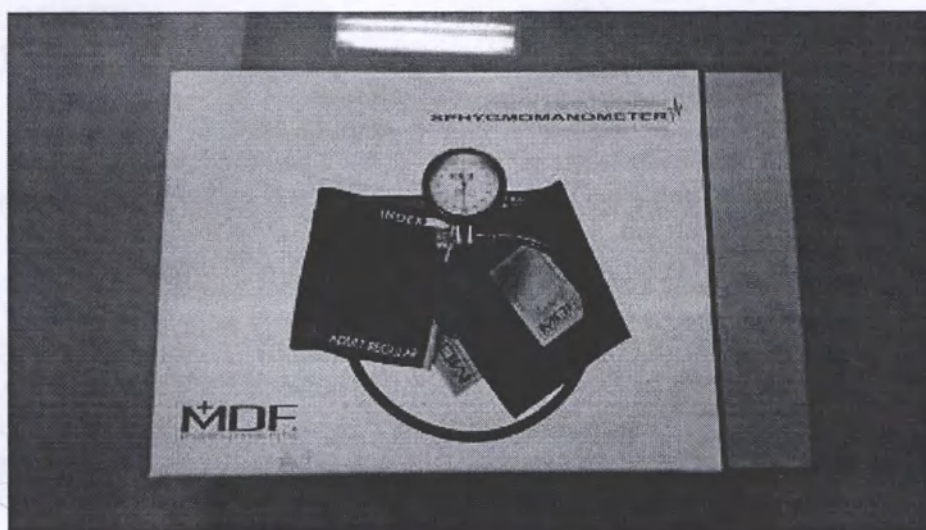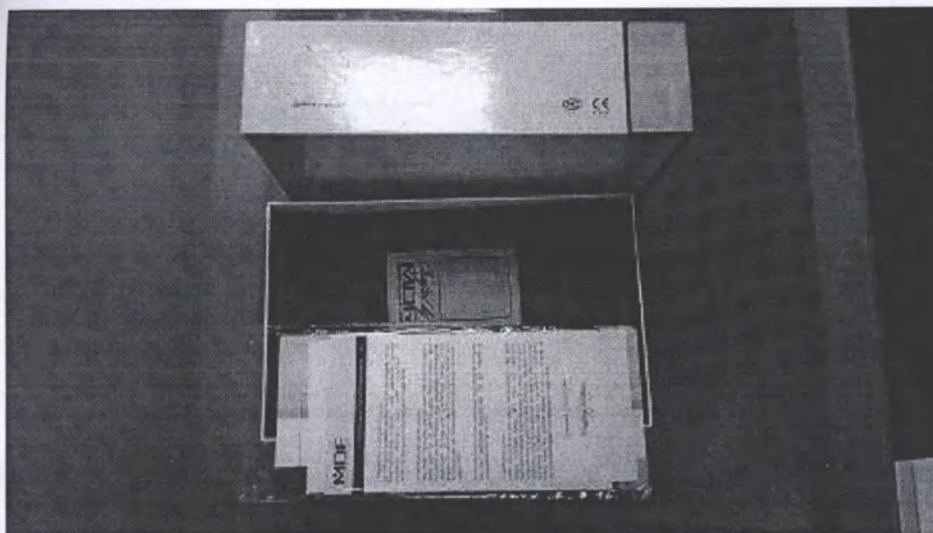
The instruction is also enclosed in the bag.

Transport packaging:

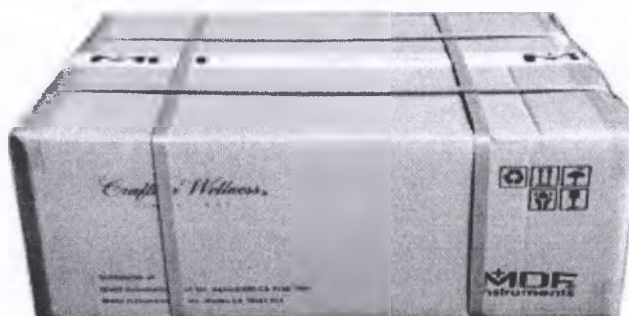The device is packed in a cardboard box and placed in it on a plastic pallet to prevent movement during transportation

Labelling

Nameplate on the package:

ITEM #: MDF848XP16 (serial number)	LOT	
MODEL: MDF 848XP (model name)	(batch code)	
SIZE: Adult (for adults)		
QTY: 1 (number of products in the package)		Barcode
Color: Real Teal Colour: Teal (emerald) MDF16		
Mdfinstruments.com (manufacturer's website)	(manufacturing date)	

assembled
in China
designed
in Los Angeles, the USA

the project of individual package labelling in Russian:

MDF Instruments Medifriend Inc,
1898 Lai Yin Road, building 6, floor 3 (including the
2nd floor, Building 6 and the 1st floor, Building 2), Jiu
Ting Town, Song Jiang District, Shanghai 201615,
China

MDF
instruments®

MDF 848XP Sphygmomanometer

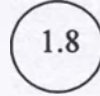
Serial number MDF848XP16

LOT

V1116179

Model: MDF 848XP

Size: For adults

Number of products in the
package:

1

Barcode

Colour: Emerald

Manufacturer's website MDFinstruments.com

2016-11

Manufacturer's authorized representative in the territory of the
Russian Federation: Limited liability company "MedExpert", 101000, Moscow, 10,
Myasnitskaya str., building 1, tel. (495) 679-84-25, e-mail: plotnikov@medexpert.ru

The used devices and packaging materials must be utilised as Class A
waste according to SanPiN 2.1.7.2790-10.

User's manual is attached

Manufactured in accordance with EN ISO 81060-1:2012, EN 980:2008, EN 1041:2008,
ISO 10993-1:2009, ISO 10993-5:2009, ISO 10993-10:2010, EN ISO 13485:2012+AC:2012,
EN ISO 14971:2012, EN 62366:2008.

The project of transportation package labelling in Russian:

MDF Instruments Medifriend Inc,
1898 Lai Yin Road, building 6, floor 3 (including the
2nd floor, Building 6 and the 1st floor, Building 2), Jiu
Ting Town, Song Jiang District, Shanghai 201615,
China

MDF
instruments®

MDF sphygmomanometers

Number of products in the
package:

10

V1116179

LOT

20 °C

+70 °C

Brutto 10.1 kg

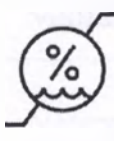
Netto

9.6 kg

Barcode

0%

85%

59x26X33 cm

0.05 m3

Manufacturer's website MDFinstruments.com

Manufacturer's authorized representative in the territory of the
Russian Federation: Limited liability company "MedExpert", 101000, Moscow, 10,

bol legends in the manual text and in the individual package:

SYMBOL DEFINITION:

 Batch Code

 Arterial Indicator

 Date of Manufacture

MDF Registered Trademark of Medfriend and MDF Instruments USA

 Temperature Limitation

 Conforms to European Standard

 Caution, Consult Accompanying Documents

 Authorized Representative in the European Community

AUTHORIZED REPRESENTATIVE IN EUROPEAN COMMUNITY
 Medical Technology Promedif Consulting GmbH
 Altenhofstrasse 80, 64386 St. Ingbert, Germany
 e: info@mt-procons.com

CE
0197

MDF
Instruments

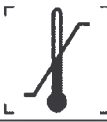
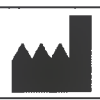
Distributed by:

MDFinstruments.com

 MDF Instruments USA, Inc. 24665 Seagull Way Suite 117 Malibu CA 90263 USA

 MDF Instruments GmbH, 5016 Chestnut Road, Suite 100 Agoura Hills, CA 91301 USA

MDF100 (11/12)

Symbol	Determination
	Batch code
	Arterial indicator
	Manufacturing date
	Temperature range
	Humidity range
	The product complies with the requirements of European Directive EC 0197
	Caution! Refer to the user's manual
	Authorized representative in the European Community
	Manufacturer's data
	National register symbol by PR 50.2.009

Symbol	Determination
1.8	Accuracy class for measuring instruments

Additionally on transport packaging:

Symbol	Determination
	Top of the product
	Fragile, handle with care
	Protect from moisture
	Use with caution
	Recycling symbol

Storage and transportation

The device should be stored at a temperature from -20°C to $+40^{\circ}\text{C}$ and at a relative humidity of not more than 85%.

It is allowed to be transported by all types of transport in covered vehicles under temperature from -20°C to $+70^{\circ}\text{C}$ and relative humidity up to 85%.

DISPOSAL CONSIDERATIONS AND REQUIREMENTS FOR ENVIRONMENTAL PROTECTION

The class of danger of medical waste based on morphological composition characteristics is A class.

If it is necessary to dispose of the product and accessories, all SanPiN 2.1.7.2790-10 requirements and current regulations should be observed.

Disposal of packaging materials should be carried out according to the type of materials pursuant to the state and local requirements and regulations.

Operating conditions

At the ambient temperature from 10 ° C to 40 ° C and relative humidity from 20% to 85% (without condensation).

WARRANTY LIABILITIES

MDF Instruments Medifriend Inc. guarantees the quality of its products and compliance with the declared characteristics in case of compliance with this instruction.

The warranty period is 3 years from the date of manufacture specified on the package.

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