G. Gmüld

Pen-injector for insulin injections for individual use

RinsaPen® I

with accessories:

Basic device:

Pen-injector for insulin injections for individual use RinsaPen® I – 1

Accessories:

1. Instructions for use (complete) – 1
2. Quick guide (summary) – 1
3. Textile pen case – 1

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
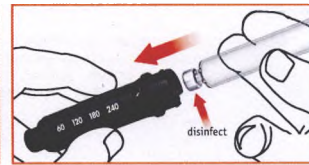
Pen preparation

Setting and changing cartridges

- 1** Remove the pen cap.
- 2** Remove the cartridge holder by twisting. Remove any empty cartridge if necessary.
- 3** Disinfect the rubber seal of the cartridge with the alcohol wipe and insert the cartridge in the cartridge holder with the aluminum end first.


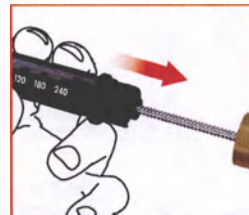
- 4** With the top of your finger push the threaded rod as far as it will go. It is also possible to push the threaded rod using the cartridge holder with the cartridge inserted in it.
- 5** Screw in the cartridge holder with the cartridge inserted in the pen-injector housing. Before fitting the needle disinfect the open part of the cartridge holder (use alcohol wipe).


Fitting the pen needle

Follow the instruction for use for your pen needle.

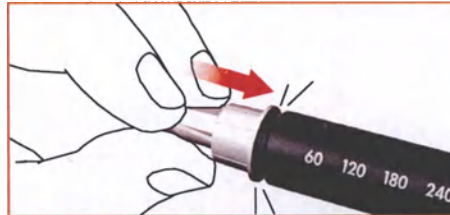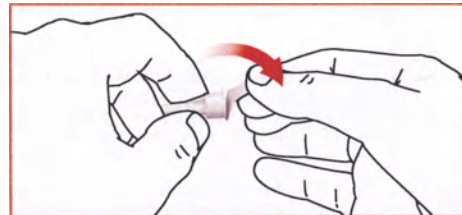- 1** Remove the sealing foil from the outer needle cap.
- 2** Using outer needle cap set the needle on the cartridge holder. Make sure the pen needle is correctly positioned.

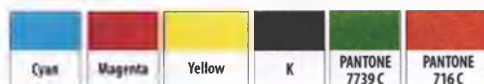
- 3** Remove the outer needle cap. Keep it so that you can use it later for safe removal and disposal of the pen needle.
- 4** Remove the inner needle cap and dispose of it.

Priming/ performance test

A performance test of the pen-injector must be conducted before each injection in order to remove air bubbles from the insulin cartridge and check that the pen needle is not clogged.

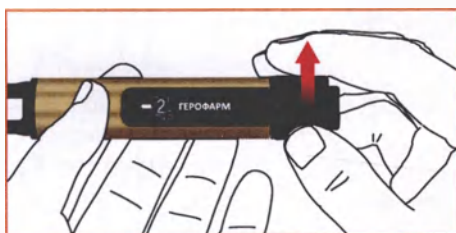
- 1 Set a dose of minimum 2 I.U. by turning the dosage knob clockwise. Check the dose set by looking at the display window vertically from above.


- 2 Holding the pen-injector with the pen pointing up carefully tap the cartridge holder with your finger so that any air bubbles can rise to the top.

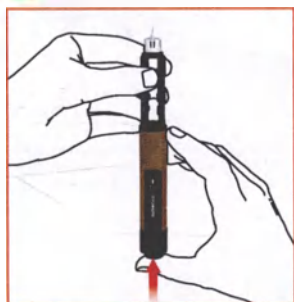- 3 Press the push-button as far as it will go.

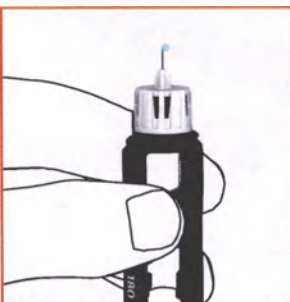
- 4 A few drops of insulin should emerge from the top of the pen needle.

If no drops emerge, repeat steps 1 to 4. If after conducting several performance checks still no drops emerge, change the pen needle and repeat steps 1 to 4.

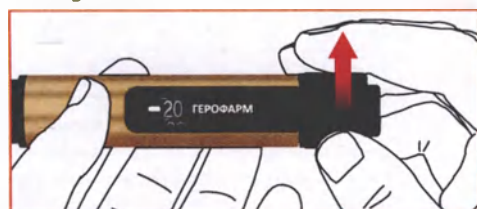Setting the dose

- 1 Dial the required insulin dose by turning the dosage knob clockwise. Correct the dose by turning counter clockwise.

- 2 Check the dialed dose by looking vertically from above at the display window.

Important! When using the pen-injector it is not possible to set a dose that is larger than the quantity of insulin still available in the cartridge. If the amount of insulin left in the cartridge is no longer sufficient for the desired insulin dose, the pen will stop when turning clockwise.

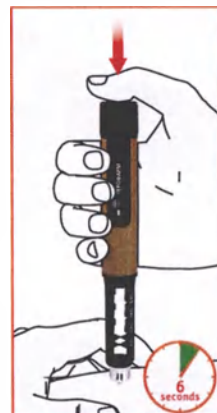
Injection

Consult your healthcare professional to choose the injection technique.


- 1 Disinfect (alcohol wipe is recommended for use) the intended place of injection, about 4-5 cm in diameter. Leave it to dry (~1 minute). Insert the pen needle. Press the push-button as far as it will go and keep it fully pressed for the whole time of injection.

- 2 When the entire insulin dose has been delivered, keep the push-button pressed for another 6 seconds to make sure the entire dose has been delivered. Remove the pen needle slowly. Disinfect the place of injection with alcohol wipe. Check to make sure the display is '0'.

Important! You can interrupt the injection by letting go of the push-button. In the display window you can read off the remaining insulin dose that has not been administered, it can be administered by pressing the push-button again.

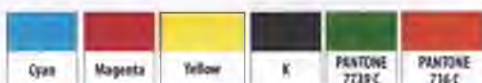Important! Read the Instructions for use of the pen-injector for insulin injections for individual use RinsaPen® I carefully for detailed instructions before use. If you have any concerns, please call Customer Service of GEROPHARM, Ltd., use toll free number: 8 800 333 43 76 (free throughout Russia).

LEGALISATION

Martin Bürgi, notary public, registered in the Register of Notaries Public of the Canton of Berne, with office in Kirchberg (Switzerland), certifies:

that the Ypsomed AG, with registered office at CH-3400 Burgdorf (Switzerland), has submitted today to the notary the foregoing document (two pages).

Certified at the office of the notary public at Kirchberg on the twenty-sixth of March two thousand and eighteen;

26.03.2018

The notary public:

A large, stylized handwritten signature in blue ink, written over the text "The notary public:".

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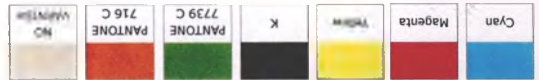
STAMPING

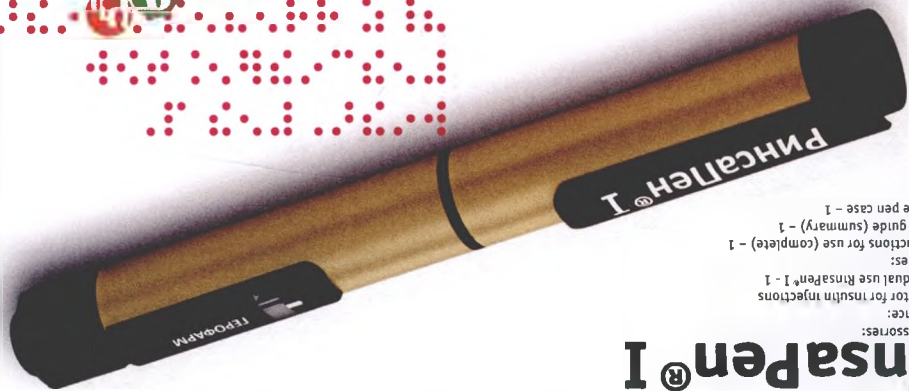
YPSOMED
SELF CARE SOLUTIONS

RinsaPen® I, pen injector for insulin injections
is manufactured by Ypsomed AG (Switzerland)
specifically for Group of companies "GEROPHARM"

Start
first use
before:
Back:

Registration certificate (№, date)

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RinsaPen® I
Pen-injector for insulin injections for individual use
with accessories:

- Basic device:
Pen-injector for insulin injections
for individual use RinsaPen® I - 1
- Accessories:
1. Instructions for use (complete) - 1
2. Quick guide (summary) - 1
3. Textile pen case - 1

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with accessories:

Basic device:

Pen-injector for insulin injections
for individual use RinsaPen® I - 1

Accessories:

- 1. Instructions for use (complete) - 1
- 2. Quick guide (summary) - 1
- 3. Textile pen case - 1

Important! Cartridges and pen needles are not provided in the package content.

Read the Instructions for use carefully before use.

RinsaPen® I is intended for the subcutaneous injection of insulin
and insulin analogues from 3 ml cartridges.

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Insulin dose accuracy and safety of use are assured only with these cartridges used.

YPSOMED
Brunnengasse 6
3401 Burgdorf / Switzerland

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that the Ypsomed AG, with registered office at CH-3400 Burgdorf (Switzerland), has submitted today to the notary the foregoing document (one page).

Certified at the office of the notary public at Kirchberg on the fifth of April two thousand and eighteen;

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A smaller, handwritten signature in blue ink, located to the right of the main signature.

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