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Instruction Manual

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General warning	..II
Photographs of the equipment	..III
Controls, displays, connectors and their uses	..IV
Symbols employed	..V

General information	..2
Device description	..2

Safety instructions	..3
Warnings and cautions	..3
Intended use	..7
User qualification	..7
Safety precautions at the site of installation	..8
Safety precautions when operating the system	..8

Installation and operating instructions	..9
Unpacking the equipment	..9
Basic equipment	..9
Installation	..9
Operating the cold light fountain	..10

Maintenance	..14
Lamp replacement	..14
Fuse replacement	..16
Cleaning and care of the unit	..17
Maintenance	..17
Servicing and repair	..17
Repair program	..18
Limitation of liability	..19
Manufacturer's warranty	..19

Technical description	..20
Troubleshooting	..20
Technical data	..22
Technical documentation	..23

Spare parts, recomm. accessories	24
List of spare parts	..24
Accessories	..24

Appendix	..29
Cleaning agents and disinfectants	..29
Electromagnetic compatibility information for the HALOGEN 250-2 twin	..31

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Warnings and cautions

Please read this manual and follow the instructions carefully. The words **Warning**, **Caution** and **Note** convey special meanings. Wherever they are used in this manual, they should be carefully reviewed to ensure the safe and effective operation of the device.

Definitions

Warning: A Warning indicates that the personal safety of the patient or physician may be involved. Disregarding a Warning could result in injury to the patient or physician.

Caution: A Caution indicates that particular service procedures or precautions must be followed to avoid possible damage to the unit.

Note: A Note indicates special information about operating the unit.

Warning: Read this instruction manual thoroughly and be familiar with its contents prior to using this equipment. Read the section on safety instructions carefully to avoid putting your patients, personnel, or yourself at risk.

Before using the unit on the patient it is imperative that you be acquainted with how the unit operates and is controlled.

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Safety instructions Warnings and cautions

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Caution: Always adjust the cold light fountain to the minimum illumination intensity necessary to achieve optimum illumination of the endoscopic scene, either by direct vision or when coupled to a video camera.

Caution: Only operate the unit with the voltage stated on the unit's identification plate.

Caution: When replacing fuses, use only fuses of the same rating.

Caution: Do not store liquids on or above the device. Avoid allowing fluids to enter the unit.

Caution: Do not touch glass bulb with your bare fingers, use the protective cover. Touching the lamp with bare fingers may result in starting difficulty.

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Intended use

The cold light fountain HALOGEN 250-2 twin Model 20 1134 20 is used as a light source in conjunction with rigid and flexible endoscopes and other instruments that are designed for use with a cold light fountain.

Use of the unit in fields other than those indicated above is not allowed for safety reasons.

The cold light fountain HALOGEN 250-2 twin may only be used with accessories which have been designated by KARL STORZ as suitable for the unit or the safe use of which is proven.

Note: KARL STORZ light cables are recommended for use with this unit. Some manufacturer's cables may be damaged when this unit is operated at higher settings.

Unauthorized conversions or modifications to the unit are not allowed for safety reasons.

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User qualification

The cold light fountain HALOGEN 250-2 twin may only be used by physicians and medical assistants who have a corresponding specialized qualification and who have been instructed in use of the unit.

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Installation and operating instructions

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Unpacking the equipment

Carefully unpack the cold light fountain and remove the unit and its accessories from their packaging. Check for missing items and evidence of shipping damage. File any complaints with the manufacturer or supplier immediately. If possible, retain the original packing materials for later use; these can come in handy if the unit has to be transported.

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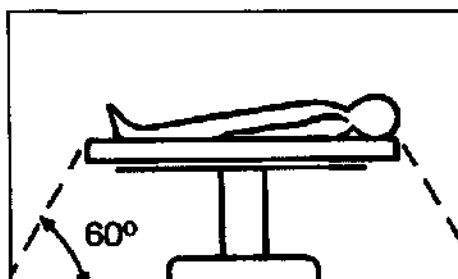
Basic equipment

- 1 Cold light fountain HALOGEN 250-2 twin
- 1 Power cord 400 A
- 2 Instruction manuals
- 1 Wallet for instruction manual
- 1 Operating instruction quick reference guide (self-adhesive)

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Installing and connecting the system

Note: The cold light fountain HALOGEN 250-2 twin may be used only in medical facilities having electrical installations conforming to the applicable national, state, and local electrical codes.

Warning: This unit is not intended for use in hazardous zones. Do not operate the unit within the hazard zone shown in the diagram while explosive anesthetic gases are in use.

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Insert the power cord into the unit's receptacle ⑩,
pushing it firmly into place.

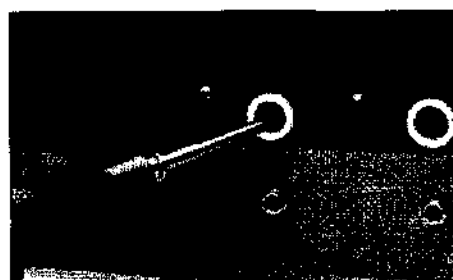
Warning: Insert the power cord only into
electrical outlets located outside areas
subject to explosion hazards.

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Plug the light cable into one of the sockets ④ as
far as it will go. Handle the light cable by the
grip piece only, never pull on the cable itself. If
necessary, 2 light cables can be connected
simultaneously.

Note: The light exits ④ are fitted with an anti-
glare cap which prevents the emergence of any
light, even with the lamp switched on. Light is
emitted only when the light cable is plugged in.

Warning: Danger of blindness. Never
look into the open end of a light cable.

Warning: Danger of burns. Contact with
the open end of a light cable connected
to a cold light fountain can possibly lead
to burns.

Warning: Light cable socket may be
very hot; particularly if the temperature
switch has been triggered and the unit
has switched off.

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Adjust the required luminous intensity with the intensity switch (3) (3 levels).

Note: The unit always starts with the last luminous intensity setting.

Warning: Danger of glare. Never look into the open end of a light cable.

Warning: Danger of burns. Contact with the open end of a light cable connected to a cold light fountain can possibly lead to burns.

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If one lamp fails, the spare lamp will be activated automatically.

If halogen lamps are defective, the lamp defective indicator (2) lights up red.

The unit automatically switches to the spare lamp. The defective lamp should, nevertheless, be changed as quickly as possible (see the "Lamp replacement" section, pages 14/15.)

The spare lamp is monitored. If there are 2 defective lamps, the lamp defective indicator (2) flashes.

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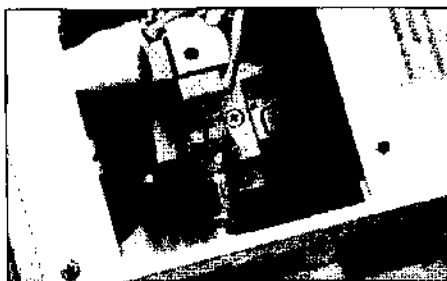
Remove the defective halogen lamp from its holder and dispose of it in accordance with the regulations.

Note: Only the defective lamp can be reached. Therefore, when replacing two defective lamps, these must be changed one at a time. After changing the first lamp, the unit must be started up and shut down again before changing the second lamp.

Warning: Danger of burns. Lamp may be hot.

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Note: Do not touch the glass bulb of the new halogen lamp with your bare fingers. (If necessary, clean the lamp with a cotton swab soaked in alcohol before operation).

Please use the protective cover provided with the new lamp. Insert new halogen lamp (24 V/250 W) into the contact socket until firmly in place. The lamp is automatically adjusted.

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Reinstall the lamp compartment lid.

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Refit the housing cover and tighten the thumb-screws.
Reconnect the power cord.
Carry out function test.

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Warning: Always pull out power plug before cleaning! Make sure the lamp has cooled down.

Caution: Avoid allowing fluids to enter the unit.

The exterior of the unit can be cleaned with a conventional surface disinfectant.

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Maintenance

Preventive maintenance is not essential. Regular maintenance can, however, contribute to identifying potential problems before they become serious, thus enhancing the instrument's reliability and extending its useful operating life.

The warnings, cautions and notes contained in the safety instructions must be observed.

Maintenance services can be obtained from your local representative or from the manufacturer.

Regardless of the accident prevention regulations or testing intervals for medical devices prescribed in different countries, we recommend a functional or safety test of the unit at least once a year. For detailed instructions please refer to the version of the Service Manual currently valid.

Note: To ensure safe operation of the system, you should perform the following procedures every 12 months:

1. Check leakage current from chassis < 100 µA.
2. Check ground impedance < 0.1 ohms.
3. Check power consumption ≤ rated power.

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Servicing and repair

Defective items of equipment must be serviced and repaired exclusively by persons authorized by us; all repair work must employ original KARL STORZ parts only.

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Limitation of liability

KARL STORZ shall be liable for failure or deterioration in the safe operation, operational reliability, and performance of this equipment only subject to the following conditions:

- all assembly operations, system expansions, readjustments, modifications, or repairs have been performed by a person or persons duly authorized by KARL STORZ;
- all electrical installations at the location of use meet applicable national and local electrical codes, and
- the unit has been used in accordance with its operating instructions at all times.

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Manufacturer's warranty

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship. In so doing we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Please fill out the attached warranty card on the last page and return it as soon as possible to:

KARL STORZ GmbH & Co. KG
Postfach 230
78503 Tuttlingen, Germany

Note: The unit must always be sent to the following address (also during the warranty period; where applicable, with warranty card):

KARL STORZ GmbH & Co. KG
Abt. Reparaturservice
Dr. Karl-Storz-Str. 34
78532 Tuttlingen, Germany

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Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall relieve us of any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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Technical description
Troubleshooting

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Symptom:
Red pilot lamp is illuminated.
Possible causes:
One lamp is defective.
Remedy:
Change lamp.

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Symptom:
Too little light.
Possible causes:
- Luminous intensity is set too low (Stage 1 or 2).
- Ends of light cable and/or endoscope are soiled.
- Light cable is defective.
- Unit is no longer set (e.g. due to shaking during transportation).
Remedy:
- Set the luminous intensity to a higher stage (2 or 3) using the + button (8).
- Clean ends of light cables, and light entrances and exits on the endoscope.
- Replace light cable.
- In order to reset, hold down the + button (8) and then switch on the unit.

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Symptom:
No light output in spite of changing the lamp.
Possible causes:
- The inserted lamp is defective.
- Automatic cutout (9) has operated.
- Lamp base is defective.
Remedy:
- Fit a new lamp.
- Activate the automatic cutout (9) by pressing.
- Have the unit repaired.

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Technical documentation

On request, the manufacturer will provide those circuit diagrams, itemized parts listings, descriptions, sets of adjustment instructions, and other items of available documentation to suitably qualified user personnel duly authorized by the manufacturer for their use in repairing those components of the unit that have been designated by their respective manufacturers as repairable.

Supply of such technical documentation relating to the unit shall not be construed as constituting manufacturer's authorization of user's personnel, regardless of their level of technical training, to open or repair the unit.

Explicitly exempted herefrom are those maintenance and repair operations described in this manual.

Note: We reserve the right to make engineering modifications in the interest of promoting technological progress and generating performance improvements without obligation on our part to submit prior notice thereof.

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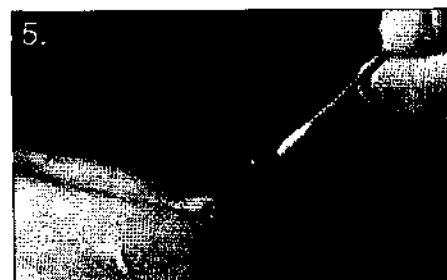
Cleaning the light cable

Note: Comprehensive information on cleaning light cables can be found in the “Light cable” instructions.

- 1. Manual cleaning:** The light cables may be cleaned using a cleaning agent/disinfectant which is specially approved for endoscopes. Wipe the outside of the light cables with a sponge or cloth soaked in cleaning solution.
- 2. Disinfection:** immerse in a disinfection solution (see Appendix, page 29, 30).
- 3. Thermal decontamination by machine** (up to 93 °C)

Caution: When preparing light cables in a cleaning and disinfection machine, particular care is necessary in the choice of chemical agents and machine programs.

- 4.** The light cable must be rinsed thoroughly after every cleaning or disinfection cycle. Completely demineralized water is recommended for rinsing.
- 5.** Clean the fiber surfaces and optical end faces with a soft cloth, sponge or cotton swab, soaked with 70% alcohol. Stubborn contaminants and deposits may be removed using cleaning paste (Art. no. 27661).
- 6. Sterilization:** Steam sterilization up to 134 °C (+3 °C) STERIS SYSTEM1* processor or gas or plasma sterilization.

Note: The instructions for “Cleaning, Sterilization and Care of KARL STORZ Instruments” must be followed. There the procedures for cleaning, disinfection and sterilization are explained in detail.

Use a cleaning agent/disinfectant approved by KARL STORZ for cleaning/disinfection (see Appendix). You will also find the current list of approved products on our Web site (www.karlstorz.de or www.karlstorz.com).

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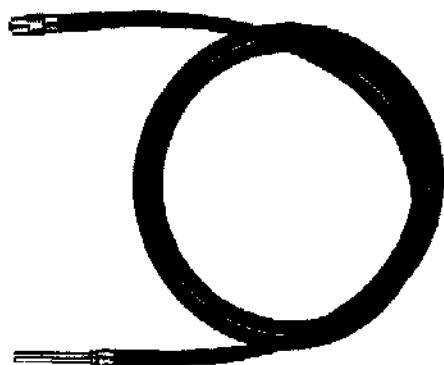
Fluid light cables

In fluid light cables the light is not transmitted through glass fibers, but through a special liquid contained in the light cable. Fluid light cables are stiffer than glass fiber light cables and bending them too much will cause a severe loss of light. However, they do transmit a brighter light than fiberoptic light cables with the same diameter. For this reason, fluid light cables are ideally suited for endoscopic photography.

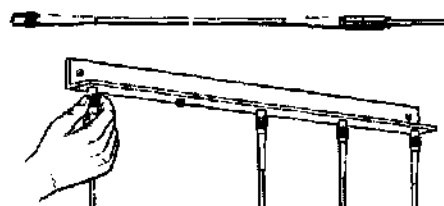
Caution: Damage to the elastomer coat will impair the light cable.

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- 1. Manual cleaning:** The light cables may be cleaned using a cleaning agent/disinfectant which is specially approved for endoscopes. Wipe the outside of the light cables with a sponge or cloth soaked in cleaning solution.
- 2. Disinfection:** immerse in a disinfection solution (see Appendix, page 29, 30).
- 3. Machine decontamination**

Caution: Carry out chemical disinfection by machine at max. 65°C.

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Type

- 1 = Cleaner
- 2 = Disinfectant
- 3 = Suitable for ultrasound baths

Note: You will also find the current list of approved products on our Web site (www.karlstorz.de or www.karlstorz.com).

Note: When agents other than those named are used, KARL STORZ assumes no liability for any damage which may occur.

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Chemicals for the preparation of telescopes and endoscopic instrument sets

Manual cleaning and disinfection/ultrasound
The following instruments and telescopes produced by KARL STORZ must not be fully immersed in a liquid: HAMOU® telescopes with ocular focusing, magnifiers with focusing, motors and their handles, IMPERATOR drill handles, prismatic light deflectors.

The following instruments produced by KARL STORZ must not be placed in an ultrasound cleaning bath: rigid telescopes, flexible telescopes, other instruments with optical components made of glass.

When preparing and using the solutions, follow the solution manufacturer's instructions, paying close attention to proper dilution and recommended immersion times. Regarding the microbiological effectiveness please ask the manufacturer. The following cleaning and disinfectant solutions have been approved regarding material compatibility:

Ecolab GmbH & Co. OHG	Sekusept forte	2, 3
	Sekusept Plus	1, 2, 3
	Sekusept Pulver Classic	1, 2, 3
	Sekusept Pulver Classic + Aktivator	1, 2, 3
Estee GmbH	Ultrasedmit Keeg	2, 3
	Acid	1, 2
Fresenius AG	Acid plus (new)	1, 2
	Sporcid FF	1, 2
Indaba Industria E Comercio LTDA	Letahdeido	2
Jose Collado S.A.	Dardor 4000 Liquido	1
	Dardor 8000	2
	Dardor Singledryd 2000	2
	Aldasan 2000	1, 2
Lysotorm Dr. Rosemann GmbH	Almyrol	1, 2
	Desaform	1, 2
	Lysotormin 3000	1, 2
	Edisonite Super	3
Merz+Co.	Mucadent-15	2, 3
	Mucadent-Zymaktiv	1, 3
	Mucocit-I	1, 2, 3
Orochemie	A 10	2
	ID 210	2
Promagent AB	Waveclip	2
	Gigasept	2, 3
Schäcke & Mayr GmbH	Gigasept FF	2, 3
	Lysol AF	1, 2, 3
	Lysol V	2
Schuhmacher, Dr. GmbH	Desocion forte	2
	Perfektan Endo	2
Starie	EnzyCam 2	1, 3
	neodisher LM 2	1
Weigert, Dr. GmbH & Co.	neodisher mediclean	1
	neodisher mediclean forte	1, 3
	neodisher mediclean	1, 3
Whitney Industries PTY Ltd	Alkal Plus	2

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*** Caution:** The use of CAUSTIC SODA can cause changes to the surface of aluminium parts (including coated parts), plastics and soldered connections, and may reduce the service life of the instrument set. For flexible endoscopes, only immerse the examination shaft, not the housing/handle part.

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WARNING: Medical electrical equipment needs special precautions regarding Electromagnetic Compatibility (EMC). Observe the EMC instructions in this appendix, as well as the instruction manual, during installation and commissioning.

The HALOGEN 250-2 twin model 201134 20 corresponds to EN IEC 60601-1-2 :2001 [CISPR 11 Class B] and therefore meets the EMC requirements of the Medical Device Directive (MDD) 93/42/EEC.

These limits are designed to provide reasonable protection against the typical electromagnetic interference to be expected in a medical environment. The HALOGEN 250-2 twin model 201134 20 is a Group 1 unit (as per CISPR 11). Group 1 contains all the 'equipment and systems which generate or use RF energy only for their internal functioning'.

NOTE: The tables and guidelines that are included in this appendix provide information to the customer or user that is essential in determining the suitability of the equipment or system for the electromagnetic environment of use, and in managing the electromagnetic environment of use to permit the equipment or system to perform its intended use without disturbing other equipment and systems or non-medical electrical equipment. If this equipment does cause harmful interference with other devices, the user is encouraged to try to correct the interference by one or more of the following measures:

- reorient or relocate the receiving device
- increase the separation between the equipment
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) is connected.

If you have any further questions, please contact your local representative or our service department.

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Table 201		
Guidance and manufacturer's declaration – electromagnetic emissions		
HALOGEN 250-2 twin model 20 1134 20 is intended for use in the electromagnetic environment specified below. The customer or user of the HALOGEN 250-2 twin model 20 1134 20 should ensure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	HALOGEN 250-2 twin model 20 1134 20 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The HALOGEN 250-2 twin model 20 1134 20 is suitable for use in all establishments (e.g. hospitals and doctors' offices) including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

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Table 202 Guidance and manufacturer's declaration – electromagnetic immunity			
HALOGEN 250-2 twin model 201134 20 is intended for use in the electromagnetic environment specified below. The customer or user of the HALOGEN 250-2 twin model 201134 20 should ensure that it is used in such an environment.			
Immunity test	EN IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	Complies ± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Complies ± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	Complies ± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U_T * (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles <5% U_T (>95% dip in U_T) for 5 sec	Complies <5% U_T * (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 0.5 cycle See guidelines 70% U_T (30% dip in U_T) for 1 cycle See guidelines See guidelines	Mains power quality should be that of a typical commercial or hospital environment. If the user of the equipment requires continued operation during voltage dips, short interruptions, voltage variations on power supply input lines, or operation at <5% U_T (>95% dip in U_T) for 5 seconds, it is recommended that the equipment or system be powered from an uninterruptible power supply.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	Complies 3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

* Note: U_T is the a.c. mains voltage prior to application of the test level.

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Table 206

**Recommended separation distances between portable and mobile RF communications equipment and the
HALOGEN 250-2 twin model 201134 20**

The HALOGEN 250-2 twin model 201134 20 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the unit can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the unit as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter [W]	Separation distance "d" [m] according to frequency of transmitter		
	150 kHz to 80 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{3} \right] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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WARRANTY

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship.

In doing so we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Complete warranty card upon purchase/delivery and return immediately to:

KARL STORZ GmbH & Co. KG
Postfach 230
78503 Tuttlingen/Germany

To be completed by the supplier/importer:

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KARL STORZ
Endoscopy China Ltd.
21 F, Tianjin Building
167 Connaught Road West
Hong Kong, China
Phone: +8 52 28652411
Fax: +8 52 28654114

OOO KARL STORZ Endoscopy - WOSTOK
Gogolevski Blvd. 3/1/2, Quartier No. 3
119019 Moskau, Russia
Phone: +7 095 7378113
Fax: +7 095 7378114

KARL STORZ
Endoscopy India Private Ltd.
C-126, Okhla Industrial Area
Phase-1,
New Delhi 110020, India
Phone: +91 11 26815445-51
Fax: +91 11 26812986

KARL STORZ Endoskope
Regional Center for Endoscopy S.A.L.
St. Charles City Center, 5th Floor
Phoenicia Street, Mina Elhosh
2020 0908 Beirut - Lebanon
Phone: +961 1 368181
Fax: +961 1 365151

KARL STORZ Industrial
Gedik Is Merkezi B Blok
Kat 5, D 38-39
Bagdat Cad. No: 162
Maltepe Istanbul, Turkey
Phone: +90 216 442 95 00
Fax: +90 216 442 90 30
Sales for Industrial Endoscopy

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