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Thank you for your expression of confidence in the KARL STORZ brand name. Like all of our other products, this product is the result of years of experience and great care in manufacture. You and your organization have decided in favor of a modern high quality piece of equipment from KARL STORZ.

These operating instructions are intended as an aid to correct installation, connection and operation of the POWERSHAVER SL system consisting of*:

- POWERSHAVER SL 20721020-1
- Standard shaver handpiece (8K) 28721035
- Dual-pedal footswitch 20012831
- Triple-pedal footswitch 20012832
- High-speed shaver handpiece (15K) 28721030
- Small joint shaver handpiece (6K) 28721040
- Multifunction handpiece 28721036 for use with optional attachments 28721031/37/32/33/34

* The POWERSHAVER SL system can consist of various components; the footswitch depends on the set; the handpieces are optional.

All essential details of the power unit and all actions required on your part are clearly presented and explained. We ask that you read this manual carefully before using the POWERSHAVER SL. Keep this manual available for reference in a convenient and conspicuous location near the POWER UNIT.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

General warning

Users of KARL STORZ devices should be thoroughly trained in their use and the applicable medical procedures. Use of these devices and accessories should not be undertaken until the user has fully familiarized himself/herself with the instructions for use, assembly and care. Instruction manuals should be carefully studied and be available to the endoscopic team during the procedure; it is essential to follow the instructions contained in the instruction manual, paying particular attention to the Cautions and Warnings.

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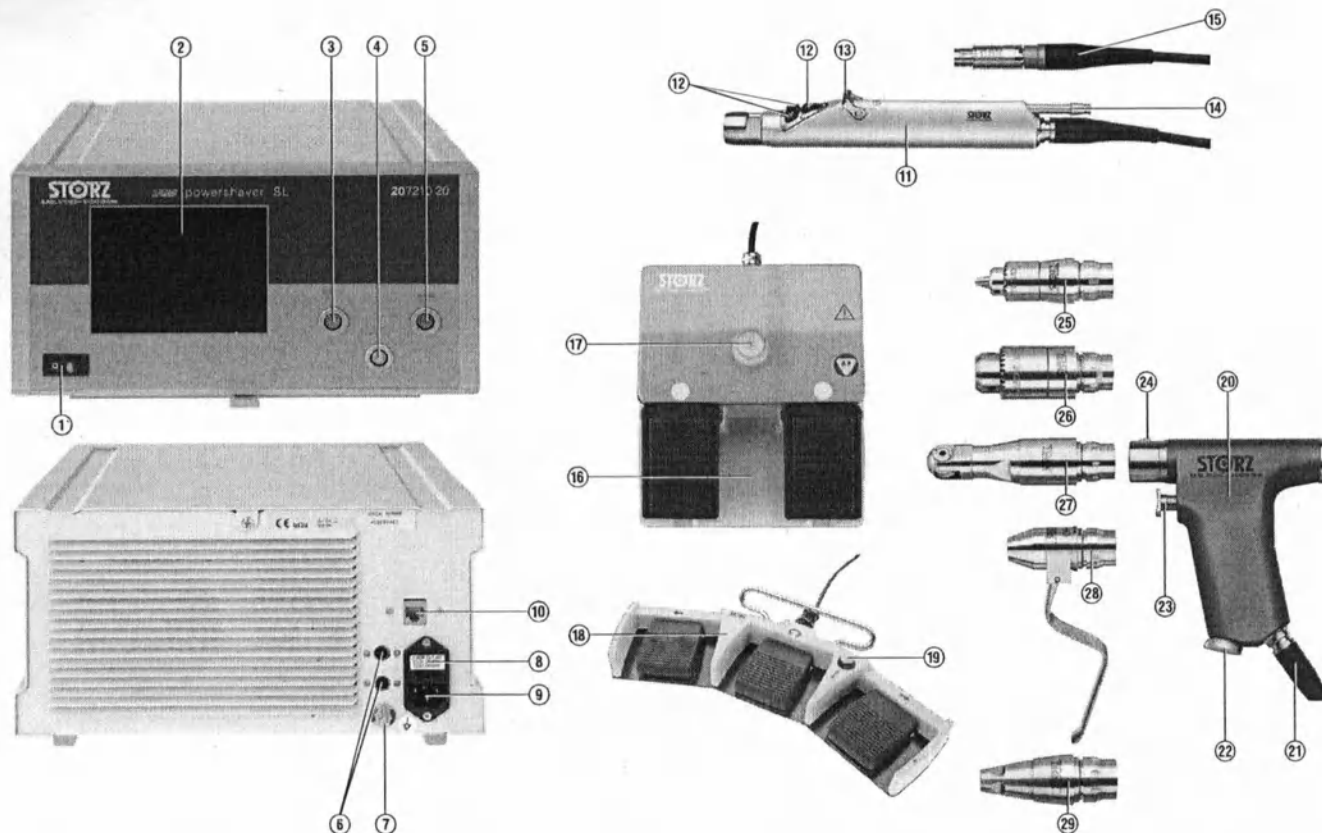
Controls, displays, connectors, and their uses

- ① Power switch
- ② LCD display
- ③ Socket for multifunction handpiece
- ④ Socket for footswitch
- ⑤ Socket for shaver handle
- ⑥ SCB connector*
- ⑦ Potential equalization connector
- ⑧ Power fuses
- ⑨ Power supply cord receptacle
- ⑩ LAN connection (for use during manufacture and software updates only**)
- ⑪ Shaver handpiece (optional)
- ⑫ Control buttons (functions, see page 22)
- ⑬ Rocker lever for suction
- ⑭ Connection point for suction tube
- ⑮ Connecting cable (integrated)
- ⑯ Dual-pedal footswitch (see page 23)
- ⑰ Footswitch top button
- ⑱ Triple-pedal footswitch (see page 24)
- ⑲ Footswitch top button
- ⑳ Multifunction handpiece (optional)
- ㉑ Connecting cord (integrated)
- ㉒ Forward/reverse lever
- ㉓ Trigger
- ㉔ Button for releasing attachments
- ㉕ Jacobs chuck 28721031 (optional)
- ㉖ Jacobs chuck 28721037 (optional)
- ㉗ Sagittal saw 28721032 (optional)
- ㉘ Pin driver 28721033 (optional)
- ㉙ Synthes driver 28721034 (optional)

*The KARL STORZ Communication Bus (SCB) interface connection may only be connected to KARL STORZ IEC-60601-1 SCB-enabled equipment. The SCB interface, based on the CAN bus, permits remote control of equipment functions, as well as remote display of equipment parameters.

**The LAN connector must NOT be connected during patient operations and normal use.

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Terminal for connecting ground cable

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Read the instructions carefully before operating
the equipment

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DANGER: Risk of explosion if used in the
presence of flammable anaesthetics.

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CAUTION: To avoid the risk of electrical
shock, do not open the unit.

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Refer servicing to qualified service personnel.
Refer to instruction manual for sterilization
instructions.

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Do not store liquids on or above the unit.
Keep the unit out of reach of patients.

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If the unit is disconnected at sockets ③, ④,
①, the voltage supply to the connected unit is
interrupted.

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General warning	II
Photographs of the equipment	IV
Controls, displays, connectors and their uses	V
Symbols employed	VI

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General information	2
Description of the system	2

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Safety precautions when using the POWERSHAVER SL system	5
Intended use	7
User qualification	7
Safety precautions at the site of installation	8
Safety precautions when operating the system	8

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Installation and operating instructions	9
Unpacking the equipment	9
Basic equipment	9
Installing and connecting the system	9
Operation	13
Operation via the touch screen/LCD	16
Operation of the handpiece	22
Small joint shaver handpiece	23
Footswitch operation	24
Multifunction handpiece (drill/saw)	26

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Maintenance	28
Fuse replacement	28
Cleaning and sterilization	29
Maintenance	39
Servicing and repair	39
Repair program	40
Limitation of liability	41
Manufacturer's warranty	41

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Spare parts, recomm. accessories	49
Appendix	58
Single-use blades and burs	58
Cleaning agents and disinfectants	60
Electromagnetic Compatibility (EMC) Information	62

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Description of the system

The POWERSHAVER SL system consisting of*

- POWERSHAVER SL unit 20721020-1
- Standard shaver handpiece (8K) 28721035 (optional)
- Dual-pedal footswitch 20012631 or Three-pedal footswitch 20012632
- High-speed shaver handpiece (15K) 28721030 (optional)
- Small joint shaver handpiece (6K) 28721040 (optional)
- Multifunction handpiece 28721036 (optional)

is used in conjunction with a shaver blade or burr (see Accessories page 49 ff.) to perform surgical interventions under endoscopic control in the field of arthroscopy.

* The system can consist of various components; the footswitch depends on the set; all handpieces are optional.

With the corresponding blade, the system is designed to provide controlled cutting, shaving, and abrading of bone and tissue during arthroscopic surgical procedures of the knee, shoulder, ankle, elbow, wrist, hip, and TMJ.

The ergonomically designed and lightweight shaver handpiece has a suction channel for connection to a suction system. Using a rocker lever, the suction channel can be opened or closed.

The easily visible LCD touch screen on the control unit of the POWERSHAVER SL provides information on the most important operating parameters. The touch screen provides control of the operating parameters.

With the optionally available multifunction handpiece and its accessories, such as the drill attachment and the saw attachment, the POWERSHAVER SL becomes a multifunctional instrument that can be used in a wide range of areas in orthopedics.

The control unit automatically recognizes the shaver handpieces or the multifunction handpiece as soon as they are connected.

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Special features

- The shaver handpieces (6K, 8K and 15K) have two modes of operation: 'Normal' and 'Oscillation'.
- The rotational speed of the small joint shaver handpiece (6K) ranges from 1,000 rpm to 6,000 rpm.
- The rotational speed of the standard shaver handpiece (8K) ranges from 1,000 rpm to 6,000 rpm and the high-speed shaver handpiece (15K) can be used from 1,000 rpm to 15,000 rpm.
- The rotational speed of the standard, high-speed and small joint shaver handpiece in oscillation mode ranges from 500 rpm to 3,000 rpm.
- Programmable user profiles with varying parameters can be set up, modified, and saved by the user.
- The speed and mode of the shaver handpieces may be controlled by either the touch screen, the handpiece (except 6K), or the footswitch.
- Fast and simple blade change using a precise, quick-release coupling.
- POWERSHAVER SL is equipped with the KARL STORZ Communication Bus (SCB).
- The control unit automatically recognizes the shaver handpiece and the multifunction handpiece connected in parallel.

* Note: The recommended maximum speed is 4,000 rpm, or 1,500 rpm in oscillation mode.

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Warnings and cautions

Please read this manual and follow the instructions carefully. The words **Warning**, **Caution** and **Note** convey special meanings. Wherever they are used in this manual, they should be carefully reviewed to ensure the safe and effective operation of the device.

Definitions

 **Warning:** A Warning indicates that the personal safety of the patient or physician may be involved. Disregarding a Warning could result in injury to the patient or physician.

 **Caution:** A Caution indicates that particular service procedures or precautions must be followed to avoid possible damage to the unit.

Note: A Note indicates special information about operating the unit.

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WARNINGS:

Warning: Read this instruction manual thoroughly and be familiar with its contents prior to using this equipment. Read the section on safety instructions carefully to avoid putting your patients, personnel, or yourself at risk.

Warnings: The POWERSHAVER SL must only be used in medical facilities that conform to applicable national, state and local electrical codes.

Warning: Keep out of reach of patients.

Warning: The instructions and interface specifications for medical devices and/or system components used in combination must be observed precisely.

Warning: Combinations of medical devices are only assured to be safe if they are identified as such in the respective instruction manuals or the intended purpose and interface specifications of the devices used in combination permit this (cf. IEC 60601-1-1).

Warning: Connect the device to a properly grounded "Hospital Grade" or "Hospital Use" electrical outlet. Check ground continuity regularly. Routinely inspect the power cord and plug. Do not use if either the plug or cord is damaged.

Warning: Test this equipment prior to each surgical procedure to ensure that it functions correctly.

Warning: Risk of explosion if used in the presence of flammable anesthetics.

Warning: When using HF surgical devices, the shaver handle and shaver blade must be put out of reach of the patient.

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Warning: The LAN connector Ⓢ is only for use by the manufacturer and is not to be connected during patient operations and normal use.

Warning: Always unplug the unit before performing any maintenance including cleaning.

Warning: To avoid the risk of electrical shock, do not remove the unit's cover. Refer servicing to the manufacturer or qualified personnel.

Warning: Removal of the cover by non-authorized personnel will void the warranty.

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Caution: Only operate the unit with the voltage stated on the unit's identification plate.

Caution: When replacing fuses, use only fuses of the same rating.

Caution: The POWERSHAVER SL may only be used with accessories and disposable items which have been designated for use by KARL STORZ.

Caution: Do not store liquids on or above the device. Avoid allowing liquids to enter the unit.

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Notes: Any damage to the unit resulting from incorrect operation is not covered by the manufacturer's warranty.

Notes: This unit has been marked in accordance with the European Directive 2002/96/EC on waste electrical and electronic equipment (WEEE). At the end of its useful operating life, dispose of the unit as electronic scrap.

Please ask either KARL STORZ GmbH & Co. KG, a KARL STORZ subsidiary or your distributor for information on your local collection point. Within the scope of application of this Directive, KARL STORZ GmbH & Co. KG is responsible for the proper disposal of this unit.

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Before using the POWERSHAVER SL system on a patient it is imperative that you be acquainted with how the system operates and is controlled.

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Intended use

The POWERSHAVER SL is intended to provide controlled cutting, shaving, and abrading of bone and tissue during arthroscopic surgical procedures of the knee, shoulder, ankle, elbow, wrist, hip, and TMJ conducted by qualified surgeons.

Use of the POWERSHAVER SL in fields other than those indicated above is not permitted.

The POWERSHAVER SL may only be used with accessories, replacement parts, and disposable items which have been designated as suitable for the unit by KARL STORZ.

Unauthorized conversions or modifications to the system are not permitted.

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User qualification

The POWERSHAVER SL system may only be used by physicians and qualified medical personnel who have been trained in use of the device.

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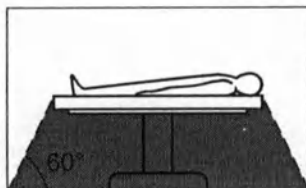
Safety precautions at the site of installation

The POWERSHAVER SL system may only be used in medical rooms of which the installations conform to applicable national regulations.

The POWERSHAVER SL is equipped with a connector for potential equalization. It should be connected in accordance with state, local and national electrical codes.

The POWERSHAVER system is not intended for use in potentially explosive atmospheres. This means, for example, that, when using easily combustible and explosive inhalation anesthetics or mixtures of these, the Powershaver system must not be operated inside the demarcated hazard zone. Examples of such substances are: anesthetic ether (diethyl ether, cyclopropane) as well as combustible, volatile skin cleansers and skin disinfectants which may create an explosive atmosphere (e.g. detergent ether, petroleum ether).

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Safety precautions when operating the system

Warning: The POWERSHAVER SL must be tested for operational safety and proper condition prior to each surgical procedure.

During surgery with the POWERSHAVER SL, the patient must be treated and kept under observation with the usual medical care. This includes keeping a check on the progress of treatment, as well as monitoring vital signs and the anesthetic.

Warning: Do not operate the POWERSHAVER handpiece unless the blade/burr is visible.

Warning: When using HF surgery, the shaver handpiece and shaver blade must not come in contact with the patient.

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Unpacking the equipment

Carefully remove the unit and its accessories from their packaging. Check for missing items and evidence of shipping damage.

File any complaints with the manufacturer or supplier immediately.

If possible, retain the original packing materials for later use; these can be used if the unit has to be transported.

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Hinweis: The handpieces 28721035, 28721030, 28721040 and 28721036 are optionally available (refer to Accessories).

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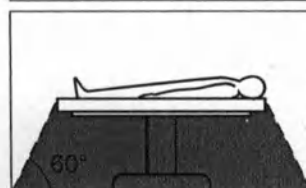
Installing and connecting the system

Note: The POWERSHAVER SL and any accessories connected to it may be used only in medical facilities that conform to the applicable state, local and national electrical codes.

Warning: The POWERSHAVER SL is not intended for use in potentially explosive atmospheres. Do not operate the system within the demarcated hazard zone while explosive anesthetic gases are in use.

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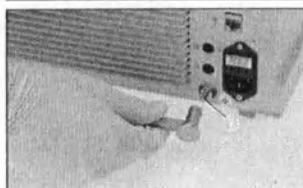
Place the POWERSHAVER SL on a level surface.

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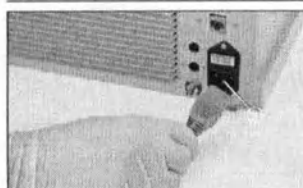
The POWERSHAVER SL is equipped with a connector for potential equalization ⑦. The unit's ground line should be installed by a qualified electrician.

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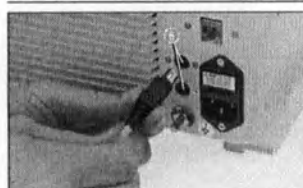
Before connecting the POWERSHAVER SL to the electrical supply line, verify that the line voltage stated on the unit's identification plate is the same as that of the electrical supply line to be used. Insert the mains cord into the unit's receptacle ⑧, pushing it firmly into place.

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Warning: Insert the mains cord only into electrical outlets located outside areas subject to explosion hazards.

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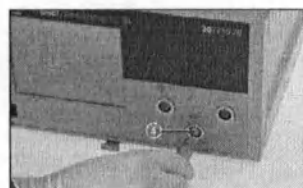
KARL STORZ-SCB*
Note: To prevent the SCB connecting cable from being pulled out accidentally, the SCB connector possesses a protection device. Pull back the protection device of the SCB connector and insert the connector into one of the SCB sockets ⑨. Connect the other end of the cable to the KARL STORZ-SCB* (KARL STORZ Communication Bus) control unit or other SCB units (see instruction manual KARL STORZ-SCB* System).

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Connecting the footswitch
To use the footswitch ⑭, insert the connecting cord of the footswitch into the socket for the footswitch ④ on the front of the POWERSHAVER SL unit. Align the red dot on the connecting cord with the red dot on the footswitch socket.

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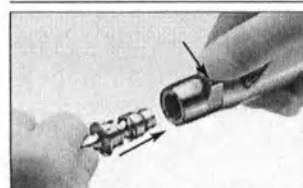
Connecting the shaver handpiece
Connect the handpiece cable ⑮ to the socket for the shaver handle ⑤ on the front of the control unit. Align the red dot on the shaver handpiece cord and the socket to ensure proper fit.

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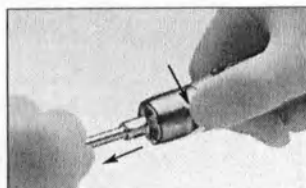
Attaching blade to the shaver handpiece
Note: Insert the blade either with direction setter, and thus with the blade opening facing upwards, or rotated through 180°, with the opening facing downwards.
Insert the blade with the direction setter into one of the two notches on the lock. Turn the lock on the lever counterclockwise and hold it.
Slide the blade into the lock as far as it will go. Finally, let go of the lever and make sure that the blade is securely attached by pulling it.
Note: If disposable products are used, follow the instructions on page 58 ff.

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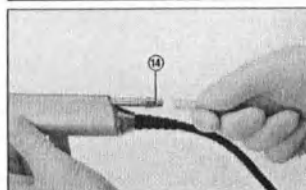
**Removal of blade from
shaver handpiece**

Turn the lock on the lever counterclockwise and hold in this position.
Pull the blade out of the handpiece.

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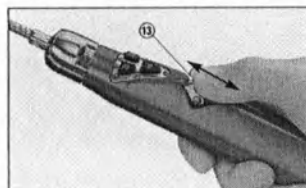
Attach the suction tube to the connection ⑭ on the handpiece and connect the tubing to a suction pump.

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Suction is controlled using the rocker lever ⑮. If the rocker lever is at the rear limit stop, the suction channel is closed; at the front limit stop, the suction channel is open (maximum suction).

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Operation

Switch the power switch ① to the On (I) position. (Power switch lights up.)

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Basic operation

The touch screen/LCD ② will be automatically activated when the control unit initializes and will remain in operation as long as power is supplied to the unit. The POWERSHAVER SL will stop operating when the control unit is powered down.

The touch screen/LCD ② will display the initial screen, which includes the KARL STORZ logo and the software version codes as shown in the screen to the left.

The touch screen/LCD ② will then display the normal 'run mode' screen. At this point all plugged in handpieces can be operated by the user.

Connected and activated handpieces and footswitches are indicated by the icons on the touch screen/LCD display ②.

The shaver handpieces will be disabled when the user either exits the screen or switches to another screen. The shaver handpieces will only be operational in the 'run mode' screen.

The multifunction handpiece cannot be operated from the touch screen or the footswitch. The control unit supplies power to the multifunction handpiece, which is operated using the trigger on the handpiece.

Note: The handpiece buttons may be manually disabled by the user to allow footswitch control. Please refer to page 21 for instructions.

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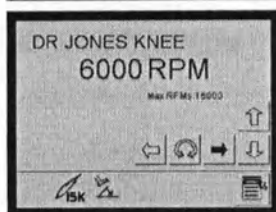
The screen displayed to the left shows the normal 'run mode' screen, when no specific set of parameters (physician profile) has been selected.
The touch screen/LCD display ① confirms that the control unit is in the oscillation mode with a selected speed of 1500 rpm. The icons in the lower left corner confirm that the 15K handpiece and footswitch are ready for use.

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The screen displayed to the left shows a physician profile with the name 'DR JONES KNEE' which has been programmed and is ready for use. Selection and programming of physician profiles is discussed later in this manual.
The touch screen/LCD display ① confirms that the control unit is in forward mode with a selected speed of 6000 rpm. The icons in the lower left corner confirm that the 15K handpiece and footswitch are ready for use.

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The control unit monitors the buttons ① on a shaver handpiece ②, footswitch ③, SCB commands (on SCB units), the multifunction handpiece, and the touch screen/LCD ④. The SCB indicator is located on the lower left-hand side of the screen. The SCB icon appears if the SCB Bus system is functioning (please also refer to the instructions on page 20).
The touch screen/LCD ④ indicates the accessory (handpiece) that is connected to the control unit.

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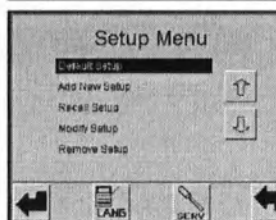
Operation via the touch screen/LCD

If the 'Up' (↑) button is pressed, the displayed speed increases.
If the 'Down' (↓) button is pressed, the displayed speed decreases.
If the 'Reverse direction/Mode' (↺) button is pressed, the blade/burr will turn counterclockwise.
If the 'Forward direction/Mode' (↻) button is pressed, the blade/burr will turn clockwise.
If the 'Oscillation Mode' button (○) is pressed, the blade oscillates (the direction of rotation alternates).
Press the 'Menu' icon in the lower right hand corner of the screen to call up the Setup Menu.

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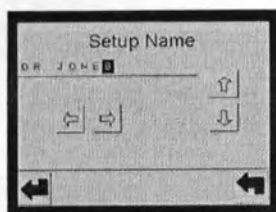
When the 'Menu' icon is pressed, the touch screen/LCD displays the Setup Menu screen shown to the left.
Selecting the 'Go Back' (↵) arrow in the lower right-hand corner will return the user to the previous screen.
From the Setup Menu the user will be able to access the following menus:
Default Setup
Recall Setup
Add New Setup
Modify Setup
Remove Setup
Select Language
Service Mode

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When 'Default Setup' is selected, the factory defined default settings will be loaded. When the 'Enter' (↵) icon is pressed, the touch screen will return to the normal 'run mode' screen and display 'Default' in the upper left corner of the screen. The user may now operate any handpiece using the default settings.

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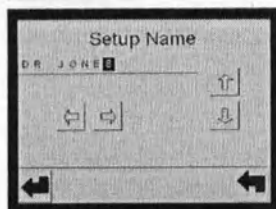
Add New Setup allows the programming of physician profiles. When the 'Add New Setup' icon is selected by scrolling with the (↑/↓) arrows and 'Enter' (↵) is pressed, the screen shown to the left appears. Selecting the 'Go Back' (←) arrow will return the user to the previous screen.

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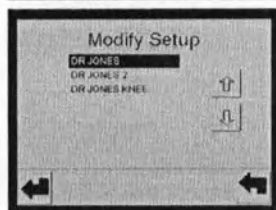
To add a physician profile the user may enter a name by holding the up/down (↑/↓) arrows down until the required character is displayed. Select the right/left (→/←) arrows to move the cursor to the next position. Repeat to enter a profile name. Press the 'Enter' (↵) icon in the lower left corner of the screen to advance to the next screen.

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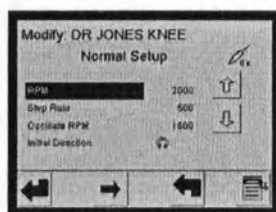
By pressing the 'Modify Setup' icon in the Setup Menu, the user can edit a profile. Use the (↑/↓) arrows to highlight the physician profile to be modified and press 'Enter' (↵).

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The screen displayed to the left will appear to enable the 8K handpiece settings to be modified. Press the up/down (↑/↓) arrows to highlight the setting to be modified (RPM, StepRate, Initial Direction). Once highlighted, press 'Enter' (↵) to select the value to be modified. Press the (↑/↓) arrows to change the value. Press 'Menu' to return to the Setup menu screen.

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To modify the 6K or 15K handpiece profile settings, press the (↵) icon, then follow the instructions for the 8K handpiece.

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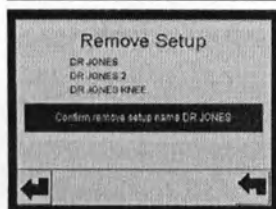
Use the up/down (↑/↓) arrows to scroll to 'Recall Setup'. Press the 'Enter' (↵) icon for profile selection. Press the up/down (↑/↓) arrows to highlight the required physician profile and press the 'Enter' (↵) icon. The touch screen will display the parameters of the physician profile selected. The handpiece and control unit are now ready for use with this physician profile.

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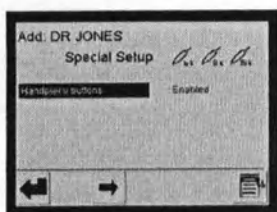
When the 'Remove Setup' icon is pressed in the Setup Menu, the screen shown to the left appears.

Use the up/down (↑/↓) arrows to select the physician profile to be removed. Press the 'Enter' (↵) icon. 'Confirm remove setup name Dr. Jones' will appear. Press 'Enter' (↵) to confirm the removal of the profile. Selecting the 'Go Back' (←) arrow will return the user to the previous screen.

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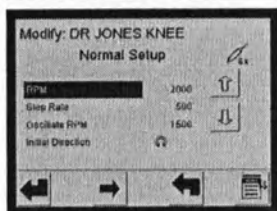
The screen shown to the left appears. By displaying the two icons, the touch screen/LCD confirms that the 6K, 8K and 15K handpieces are ready for use. Press the (↵) icon to continue. "Confirm remove setup name Dr. Jones" will appear.

Note: For information on how to disable handpiece buttons, please refer to the instructions on page 21.

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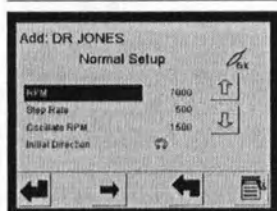
The screen to the left appears displaying the current settings for the 8K handpiece. To highlight the setting to be changed (RPM, StepRate, Initial Direction), scroll with the (↑/↓) arrows. Press the 'Enter' (↵) icon to select the value. Use the (↑/↓) arrows to modify the value. Press the (↵) icon to continue. Selecting the 'Go Back' (↶) arrow will return the user to the previous screen.

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The screen to the left appears displaying the current settings for the 15K handpiece. Proceed as described for the 8K shaver handpiece to change the settings.

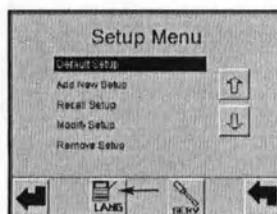
Press the (↵) icon to move to the next handpiece (e.g., 6K shaver handpiece). To change the settings, proceed as described for the 8K shaver handpiece.

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Multi-Language Support

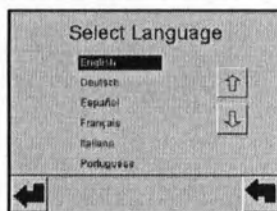
When the 'Menu' icon is pressed, the touch screen/LCD will display the Setup menu screen shown to the left.

From the Setup menu, the user will also be able to access the 'Select Language' menu.

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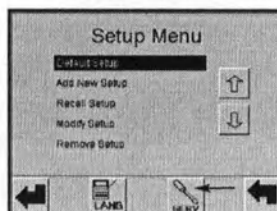
When the 'Language' icon is pressed (see arrow), the screen to the left appears.

Use the (↑/↓) arrows to scroll to the language of choice. Press the 'Enter' (↵) icon to select.

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Service Mode

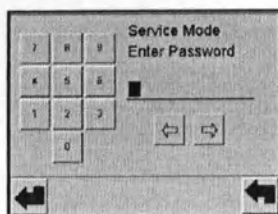
When the 'Menu' icon is pressed, the touch screen/LCD will display the Setup menu screen shown to the left.

From the Setup menu, the user will also be able to access the 'Service Mode' menu.

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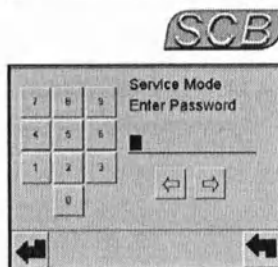
When the 'Service icon' is pressed (see arrow), the screen to the left appears.

The 'Service Mode' should only be used by trained, qualified service personnel familiar with its operation. This mode is password protected. Please refer to the service manual for more details.

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SCB capability

POWERSHAVER SL software version 3.0 and higher supports the SCB (KARL STORZ Communication Bus) interface. SCB status is displayed in the lower left-hand corner of the touch screen/LCD display. The icons are explained in the following table.

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SCB Online: The POWERSHAVER SL is physically connected to the SCB network and communication with other SCB units has been established.

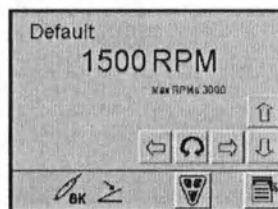
SCB Offline: The POWERSHAVER SL was connected to the SCB network but there is currently no communication with other SCB units. The unit is waiting for the SCB connection to be re-established.

no display: The POWERSHAVER SL does not have SCB capability or the unit is waiting for the SCB connection to be established. Since the unit was powered up there has been no communication via the SCB interface.

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Disabling Handpiece Button Operation

The buttons on the 6K, 8K and 15K handpieces may be disabled if the user only wants to operate the POWERSHAVER via the footswitch. This can be specified for each physician in physician profiles.

To disable the buttons on the handpieces, press the 'Handpiece button' icon.

Handpiece operation is only disabled in the current physician profile. This setting remains in effect until a new handpiece is connected, another physician profile is selected, or the unit is powered down.

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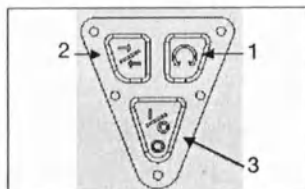
A flashing button icon with a line through it on the screen shows that the handpiece buttons have been disabled in the current physician profile.

To enable the handpiece buttons again, press the 'Handpiece button' icon again.

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Operation of the handpiece

The POWERSHAVER SL system features shaver handpieces which can be operated from the touch Screen/LCD of the POWERSHAVER SL, the handpiece buttons or the footswitch. All handpieces (except small joint shaver handpieces) are equipped with three buttons. The functions are identical on both handpieces. Below is a definition of the button functionality:

Button 1: right (Oscillation) (+)

When the handpiece has stopped, pressing this button will change the handpiece to Oscillation mode. Each time the button is pressed the blade will oscillate for one second.
When the handpiece is running, pressing button 1 will increase the speed of the blade.

Button 2: left (Forward/Reverse) (-)

When the handpiece has stopped, pressing button 2 will change the direction of the blade rotation from forward to reverse.
When the handpiece is running, pressing button 2 will decrease the speed of the blade.

Button 3: center

This button starts and stops the handpiece.

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Small joint shaver handpiece

The small joint shaver handpiece is operated using the LCD touch screen of the POWERSHAVER SL or the footswitch. It only has one button, the function of which will be explained in the following section:

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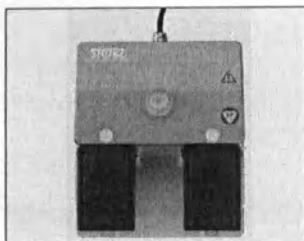
Note: The small joint handpiece 28 7210 40 is only compatible with the POWERSHAVER SL together with software version 2.4 or higher.

Check the software version of the POWERSHAVER SL:
If necessary, turn the POWERSHAVER SL off (the software version is only displayed when the device is switched on).
Turn the POWERSHAVER SL on (the power switch lights up green).
The software version number is shown on the screen (touch screen/LCD) (see image on the left).

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Footswitch operation

The shaver handpieces can be controlled using a footswitch.
The dual-pedal footswitch 200128 31 has three "buttons" in the form of a left pedal, right pedal and a top mounted switch. Below is a definition of the footswitch functionality:

Left pedal:
Reverse (Left) < Counterclockwise

Right pedal:
Forward (Right) > Clockwise

Left and right pedal simultaneously:
Oscillate ○

Button on top of footswitch:
Pressing the button on the top of the footswitch once switches the footswitch to speed adjust mode, whereby the foot pedals can now adjust the speed up and down.
Pressing the right foot pedal increases the speed (by the programmed rate) and pressing the left foot pedal decreases the speed (by the programmed rate).
By pressing the top button again, the unit returns to normal operating mode with the new speed setting.

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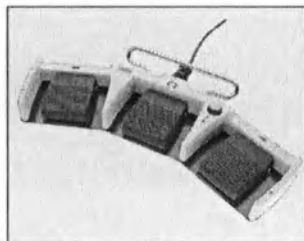
The triple-pedal footswitch 200128 32 has three distinct pedals for mode selection and a button for speed control. Below is a definition of the triple-pedal footswitch functionality:

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Button on top of footswitch:
Pressing the button on the top of the footswitch once switches the footswitch into speed adjust mode, whereby the foot pedals can now adjust the speed up and down.
Pressing the right foot pedal increases the speed (by the programmed rate) and pressing the left foot pedal decreases the speed (by the programmed rate).
By pressing the top button again, the unit returns to normal operating mode with the new speed setting.

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**Multifunction handpiece
(drill/saw)**

The POWERSHAVER SL control unit is capable of driving a multifunction handpiece. This handpiece may be attached to drill chucks, sagittal saws, etc. A list of available accessories is shown in a separate section of this manual.

Each accessory is connected to the handpiece by attaching the accessory and handpiece together until solid contact is made. Press the rear of the button on the top (2) of the handpiece to remove the accessory.

The motor is cannulated and accepts pins up to 0,125 inches (3,2 mm) in diameter.

The POWERSHAVER SL allows the speed of the handpiece to be controlled by moving the trigger (3). The speed (rpm) of the handpiece is proportional to the position of the trigger. Pushing the trigger all the way in toward the handpiece provides the maximum speed.

The maximum and minimum speed of the multifunction handpiece depends on the attachment and its design.

Warning: The multifunction handpiece requires additional exposure to steam for sterilization. See sterilization instructions.

Warning: EtO sterilization is NOT recommended for sterilization of the multifunction handpiece. See page 38 for proper steam sterilization parameters.

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Предупреждение. Sterilization of multifunction handpiece requires additional exposure to steam for sterilization. See sterilization instructions (see section "Sterilization").

Предупреждение. EtO-sterilization for multifunction handpiece is not recommended. See page 38 of this manual for the appropriate steam sterilization parameters (see section "Sterilization").

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Note: The multifunction handpiece 28721036 is only compatible with the POWERSHAVER SL software version 2.3 or higher.

Check the software version of the POWERSHAVER SL:

If necessary, turn the POWERSHAVER SL off (the software version is only displayed when the device is switched on).

Turn the POWERSHAVER SL on (the power switch lights up green).

The software version number is shown on the screen (touch screen/LCD) (see image on the left).

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Multifunction Handpiece Operation

The multifunction handpiece contains a three position switch that selects motor direction or standby operation:

- Forward motor direction – switch moved to the right (FWD) position
- Reverse motor direction – switch moved to the left (REV) position
- Standby – switch in the center position

If the multifunction handpiece is connected to the receptacle on the front of the POWERSHAVER unit, and the switch is set to the right or left position, all the handpieces (6K, 8K, 15K) and the footswitch are disabled.

Note: The switch should be in the standby (center) position when connecting the multifunction handpiece to the receptacle on the POWERSHAVER SL, otherwise the handpiece will not be recognized by the POWERSHAVER SL. In such a case, the multifunction handpiece icon is not displayed on screen.

Moving the switch from standby to the FWD or REV position enables the multifunction handpiece and disables all other handpieces (6K, 8K, 15K) and the footswitch, as depicted on the screen to the left.

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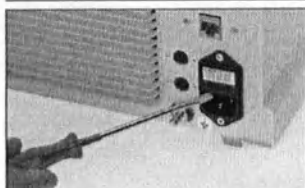
Fuse replacement

Switch off the control unit and remove the power plug from the electrical outlet.

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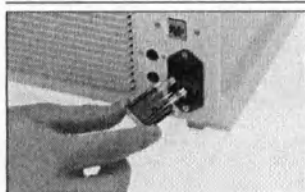
Remove the power fuse holder ① with a screwdriver or other tool.

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Caution: Only use fuses of the correct rating.

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Insert new fuses.
Refit the power fuse holder

Reconnect the mains cord.
Test the unit for proper operation.

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Cleaning and sterilization

Note: The instructions „Cleaning, sterilization and care of KARL STORZ instruments“ must be followed. The procedures for cleaning, disinfection and sterilization are explained there in detail.

Warning: Always remove the power plug before cleaning!
Caution: Avoid allowing liquids to enter the control unit.

The exterior of the unit can be cleaned with a conventional surface disinfectant.

Cleaning instructions for the shaver handpieces and reusable blades

Warning: The POWERSHAVER handpieces and reusable blades/burrs must be thoroughly cleaned and sterilized according to validated preparation procedures prior to use and subsequent reuse.

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* The use of alkaline cleaning agents can lead to changes in the surface of the handpieces. It is thus preferable to use neutral or enzymatic cleaning agents.

** Single-use products may not be prepared again.

Note: Use a cleaning agent approved by KARL STORZ for cleaning (see Appendix, pages 60, 61). You will also find the current list of approved products on our Web site (www.karlstorz.com).

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Preparation for cleaning and sterilization

Place the instruments in a container and soak with a cleaning solution (according to the manufacturer's instructions) immediately after use to prevent contaminants from drying onto the instruments.

Water quality requirements

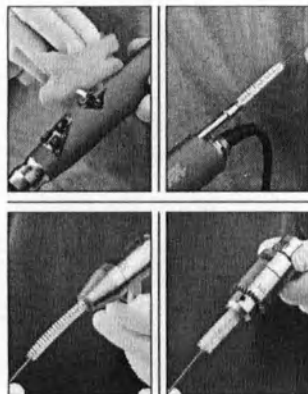
Completely demineralized water is recommended for cleaning, rinsing and preparing the cleaning/disinfectant solution.

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Cleaning

Manual Precleaning

Caution: Wear protective gloves, clothing and a face mask when cleaning contaminated handpieces and blades.

Caution: When preparing and using solutions, follow the manufacturer's instructions, paying close attention to proper dilution and recommended immersion times. Prolonged immersion may result in tarnishing or corrosion.

Shaver handpiece/Multifunction handpiece and accessories

Remove blade/burr from the handpiece. All moving parts must be in the open position. Remove the suction tube from the handpiece. Remove the accessory (e.g. Jacobs chuck) from the multifunction handpiece. Clean the handpiece, including the suction channel and the area where the blades/burrs connect into the handpiece, mechanically with water or a cleaning and disinfectant solution (see Appendix, page 60) and a soft brush (Art. no. 27650 E). Clean the accessory for the multifunction handpiece (e.g. Pin Driver) with a sponge and brush. Do not use hard or sharp cleaning tools. In order to avoid material damage, carry out the final flushing with completely demineralized water. Finally dry with filtered compressed air.

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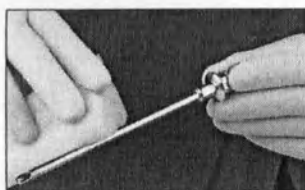
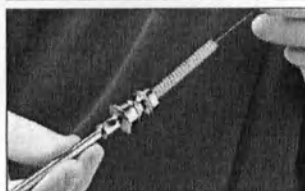
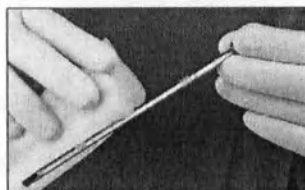
Shaver blades/burrs (attachments)

Thoroughly rinse the instruments to remove all gross debris. Remove contaminants with brushes, sponges, soft cloths or a cotton applicator. Cleaning accessories are available from KARL STORZ (see catalog).

Cleaning brushes must be cleaned and disinfected immediately.

Remove the insert from the outer sheath of the shaver blade. Thoroughly clean the lumens of the reusable blades/burrs with a cleaning brush. The outer sheath as well as the insert of the shaver blade must then be rinsed out with a strong jet of water or a cleaning and disinfectant solution until no more residue can be seen. In order to avoid material damage, carry out the final flushing with completely demineralized water. Finally dry with filtered compressed air.

If required, the blade can be cleaned in an ultrasound bath in the disassembled state with an exposure time of approx. 3-5 min.

After cleaning, inspect the handpieces, blades, and burrs for cleanliness and damage. The sheaths and tips of the blades/burrs must not have any grooves, scratches or dents.

Afterwards the handpiece (and, if applicable, accessories) as well as the shaver blades/burrs can be thermally disinfected at 93 °C in the washer. Finally, the parts can be steam sterilized at 134 °C or gas sterilized (see also sterilization instructions for steam sterilization of the multifunction handle, page 38).

Machine cleaning and disinfection

Note: Use high-level disinfectant solutions which are specially approved for endoscopes from KARL STORZ (see Appendix, pages 60, 61).

* Shaver blades/burrs: Care must be taken to ensure that the sheaths are thoroughly rinsed during machine cleaning and disinfection (special connection in the cleaning and disinfection machine).

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* Shaver handpiece: To ensure the proper functioning of the handpiece, steam sterilization with fractionated prevacuum should be performed after the washer cycle, in particular to remove any moisture left in the area around the electric connections.

Sterilization

Prior to sterilization, lubricate the hub of the reusable blade/burr. Put a very small amount of grease on the hub of the blade. Insert the inner tube into the outer tube and rotate several times. The blades must then be disassembled and re-assembled after sterilization.

Caution: Before putting into operation after completing sterilization, allow the shaver handpiece to cool down to room temperature under ambient conditions.

Caution: When dismantling the shaver attachments (blades/burrs) for cleaning and sterilization purposes, please pay attention to the following: Do not mix up the individual inner and outer attachment parts. Please note the printed article numbers! Incorrect assembly can lead to friction on the tip of the shaver attachments and increased heat generation.

Sterilization

Routine steam sterilization is recommended for initial and subsequent sterilization of all handpieces and reusable blades/burrs. The instruments may also be sterilized with ethylene oxide (EIO). To achieve the desired sterility assurance level (SAL) of 10⁶, KARL STORZ recommends only the following gas and steam sterilization methods.

Note: Before sterilization, the items must be thoroughly cleaned (see pages 31, 32). Then pack the items into appropriate containers (Art. no. 39315 A).

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Warning: Sterilization is only possible on clean surfaces.
Warning: The recommended sterilization parameters are only valid with sterilization equipment that is properly maintained and validated.
Warning: Any deviations from the recommended parameters for sterilization should be validated by the user.

Warning: KARL STORZ recommends that, in order to assure sterility, all blades/burs should be sterilized while disassembled and then reassembled in a sterile field.

Warning: Before sterilization, the handpieces and reusable blades/burs must be thoroughly cleaned and all residues completely removed.

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Steam sterilization

Sterilization trays should be positioned in the sterilizer so that there is adequate circulation and penetration of steam, and so that the air can escape and the condensate flow away. Load the sterilizer according to its instruction manual.

After completion of the steam sterilization cycle, the sterilized instruments must be allowed to cool down slowly.

Warning: Risk of burns! Parts are hot after steam sterilization. Allow them to cool down.

KARL STORZ has validated the following steam sterilization methods:

- Prevacuum
- Gravity displacement
- Fractionated prevacuum

Note: Due to the design of the handpieces suitable for the POWERSHAPER system, we recommend the "fractionated prevacuum procedure".

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Prevacuum method

Prevacuum or high vacuum sterilization consists of four phases: a conditioning phase, a sterilization phase, an evacuation phase and a drying phase. In the conditioning phase, a vacuum is generated in the sterilization chamber; then the instruments are heated by injecting steam. Sterilization occurs during the sterilization phase when the chamber reaches a temperature of 134 °C (+ 3 °C) and a pressure of 2 bar. The sterilization phase lasts 4 minutes. The evacuation phase removes the steam from the chamber. In the drying phase, the instruments are dried in a vacuum for approximately 20 minutes. The following sterilization conditions have been validated in a prevacuum sterilizer.

Temperature: 134 °C (+ 3 °C)
Pressure (p_b): 2 bar
Sterilization time: 4 minutes

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Gravity displacement method

Gravity displacement sterilization also consists of four phases which are similar to the prevacuum type of sterilization. During the conditioning phase, steam is injected into the chamber and the air is forced out through the drain. Sterilization occurs when the temperature in the chamber reaches 121 °C (+ 3 °C) and the pressure reaches 1 bar. The sterilization time in a gravity displacement sterilizer is longer than in a prevacuum sterilizer (45 minutes). Gravity displacement sterilization can also be done at 134 °C (+ 3 °C) and 2 bar. The higher temperature reduces the sterilization time required for sterilization to 10 minutes. The steam is removed from the chamber during the evacuation phase by allowing the steam to escape down the drain. The sterilized items remain in the chamber at atmospheric pressure to dry by the heat given off by the autoclave jacket. The following sterilization conditions have been validated using a gravity displacement sterilizer.

Temperature: 121 °C or 134 °C (+ 3 °C)
Pressure (p_b): 1 bar or 2 bar
Sterilization time: 45 minutes or 10 minutes

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Fractionated prevacuum method

The fractionated prevacuum method also consists of the four previously mentioned phases. In the conditioning phase, a vacuum is generated up to four times. Steam is then blown in and sterilization takes place at 134°C (+ 3°C) 5 minutes (minimum time). Before the final drying phase, the steam is removed from the chamber during the evacuation phase. Drying takes place under vacuum.

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(HCFC-124) 90 % (Gewichtsprozent)

Ethylene oxide (EtO) gas sterilization

Warning: KARL STORZ recommends that all blades/burs be sterilized with ethylene oxide while disassembled and then reassembled in a sterile field.

Warning: Sterility of KARL STORZ blades/burs sterilized with ethylene oxide CANNOT be assured if the instruments are sterilized while assembled.

All sterilization cycles must include a

pre-conditioning cycle:

Temperature: 54 °C ± 2 °C

Time: 30 minutes

KARL STORZ has validated ethylene oxide (EtO)

sterilization using the following parameters:

Gas mixture: EtO: HCFC*

Temperature: 54 °C ± 2 °C

Relative humidity: 60 % ± 20 %

Pressure: 0.58-0.70 bar (8-10 psi)

Exposure time: 120 minutes

EtO concentration: 600 ± 30 mg/l

* Gas mixture composed of 10% ethylene oxide

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Gas sterilization with formaldehyde (FO)

When using formaldehyde the instruments can be used immediately after the sterilization cycle has been completed. An additional ailing time is not necessary.

Notes: Observe national laws and regulations for the gas sterilization method.

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Grundemann, B.J. and Meeker, M.H. Alexander's Care of the Patient in Surgery, 7th edition. The C.V. Mosby Company, St. Louis, Mo. 1983.

Association for the Advancement of Medical Instrumentation. Designing, Testing and Labeling Reusable Medical Devices for Reprocessing in Health Care Facilities: A Guide for Device Manufacturers. AAMI TIR No. 12-1994.

Descoteaux, J.-G., Poulin, E.C., Julien, M. and Guidoin, R. Residual Organic Debris on Processed Surgical Instruments. AORN Journal, Vol. 62, No. 1, pp. 23-29, 1995.

References for cleaning and sterilization

Grundemann, B.J. and Meeker, M.H. Alexander's Care of the Patient in Surgery, 7th edition. The C.V. Mosby Company, St. Louis, Mo. 1983.

Association for the Advancement of Medical Instrumentation. Designing, Testing and Labeling Reusable Medical Devices for Reprocessing in Health Care Facilities: A Guide for Device Manufacturers. AAMI TIR No. 12-1994.

Descoteaux, J.-G., Poulin, E.C., Julien, M. and Guidoin, R. Residual Organic Debris on Processed Surgical Instruments. AORN Journal, Vol. 62, No. 1, pp. 23 - 29, 1995.

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Steam Sterilization Instructions for the Multifunction Handpiece

 **Warnings:** The multifunction handpiece requires additional exposure to steam for sterilization. The validated sterilization cycles are listed below.

 **Warning:** In order to guarantee sterility, KARL STORZ recommends removing all attachments from the multifunction handpiece, and sterilizing these separately.

Prevacuum:
The following sterilization conditions have been validated for the multifunction handpiece and its accessories in a prevacuum sterilizer:

Temperature: 134°C (+3°C) (270 to 272°F)
Pressure: 2 bar (29 psi)
Exposure Time: 16 minutes

 **Caution:** Please note increased sterilization time for the prevacuum method.

Gravity Displacement Method:
The following conditions have been used to validate sterilization procedures for the multifunction handpiece and its accessories in gravity autoclaves:

Temperature: 134°C (+3°C) (270 to 272°F)
Pressure: 2 bar (29 psi)
Exposure Time: 35 minutes

 **Caution:** Please note increased temperature for the gravity displacement method.

Sterilization trays should be positioned in the sterilizer so that there is adequate circulation and penetration of steam, air removal, and condensate drainage. A loosely loaded sterilizer allows the best penetration of sterilant. On completion of the steam sterilization cycle, the instruments should not be used until adequately cooled.

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Maintenance

Preventive maintenance is not essential. Regular maintenance can, however, contribute to identifying potential problems before they become serious, thus enhancing the instrument's reliability and extending its useful operating life. The warnings, cautions and notes contained in the safety instructions must be observed.

Maintenance services can be obtained from your local representative or from the manufacturer. Regardless of the accident prevention regulations or testing intervals for medical devices prescribed in different countries, we recommend a functional or safety test of the unit at least once a year. For detailed instructions please refer to the version of the Service Manual currently valid, catalog number SV 3525 (English version).

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Servicing and repair

Defective items of equipment must be serviced and repaired exclusively by persons authorized by us; all repair work must employ original KARL STORZ parts only.

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Disposal

This unit has been marked in accordance with the European Directive 2002/96/EC on waste electrical and electronic equipment (WEEE). At the end of its useful operating life, dispose of the unit as electronic scrap.

Please ask either KARL STORZ GmbH & Co. KG, a KARL STORZ subsidiary or your specialist dealer for information on your local collection point. Within the scope of application of this Directive, KARL STORZ GmbH & Co. KG is responsible for the proper disposal of this unit.

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Repair program

Usually to bridge the repair period, you will receive a unit on loan which you then return to KARL STORZ as soon as you receive the repaired unit.

In Germany you can refer repairs direct to

KARL STORZ GmbH & Co. KG
Abt. Reparaturservice
Dr. Karl-Storz-Straße 34
78532 Tuttlingen, Germany.

In other countries please contact your local KARL STORZ branch or authorized dealer.

Important information

To avoid the spread of infectious illnesses among hospital staff or among KARL STORZ employees, telescopes, instruments and equipment must be cleaned and sterilized/disinfected before they are sent for repair. We reserve the right to return contaminated instruments/equipment to the sender.

Repairs, changes or expansions which are not carried out by KARL STORZ or by experts authorized by KARL STORZ will invalidate all guarantee rights.

KARL STORZ gives no guarantee on the correct functioning of equipment or instruments which have been repaired by unauthorized third parties.

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Limitation of Liability

KARL STORZ shall be liable for failure or deterioration in the safe operation, operational reliability, and performance of this equipment only subject to the following conditions:

- all assembly operations, system expansions, readjustments, modifications, or repairs have been performed by a person or persons duly authorized by KARL STORZ;
- all electrical installations at the location of use meet applicable national and local electrical codes, and
- the unit has been used in accordance with its operating instructions at all times.

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Manufacturer's warranty

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship. In so doing we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Please fill out the attached warranty card on the last page and return it as soon as possible to:

KARL STORZ GmbH & Co. KG
Postfach 230
78503 Tuttlingen/Germany

Note: The unit must always be sent to the following address (also during the warranty period, where applicable, with warranty card):

KARL STORZ GmbH & Co. KG
Abt. Reparaturservice
Dr. Karl-Storz-Str. 34
78532 Tuttlingen/Germany

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Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall relieve us of any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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General

In the event of the unit malfunctioning, the troubleshooting list provided in this chapter will assist the user in finding the possible cause. The guide explains how to rectify the various malfunctions.

Warning: High voltage is present in the unit's interior. The unit must only be opened by an authorized KARL STORZ Customer Service technician.

Warning: Maintenance work that is not mentioned in this chapter must only be carried out by KARL STORZ authorized technical personnel.

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Problem
Instrument totally inoperative.

Possible cause
Outage in electrical supply.

Recommended remedy
Check the electrical supply.

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Problem
Complete failure of the unit.

Recommended remedy
Notify Customer Service.

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Problem
POWER-ON lamp does not light up.

Possible cause
1. No power from the power line.
2. Defective power fuse.
3. Defective POWER-ON lamp.

Recommended remedy
1. Check that there is electricity to the wall outlet.
2. Change fuses as described in the instruction manual. Make sure you fit the correct type of fuse.
3. Notify Customer Service.

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Problem
1. POWER-ON lamp lights up.
2. No indication of rotation.
3. No energy output when operating the footswitch/pushbuttons.
4. Speed cannot be regulated.

Possible cause
1. Motor control electronics faulty.
2. Connection to motor interrupted.
3a. Faulty handpiece.
3b. Handpiece blocked or jammed.
4. Faulty motor control electronics.

Recommended remedy
1, 2, 4 Notify Customer Service.
3a. Connect another handpiece.
3b. Clean handpiece.

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44

POWERSHAVER SL Error Messages

Message	Screen	Description	Remedy
48 Volt Power Supply Failure	Main Run Screen	Hardware failure - motor power supply voltage.	Restart the unit by switching the power off and on again. Notify KARL STORZ Technical Support if the problem persists.
ERROR - Duplicate Setup name	Setup Name (Add Setup)	A setup with the same name exists.	Press the Enter button on the screen and enter a different Setup name.
ERROR - Default setup error	Setup Name (Add Setup)	A checksum error occurred in default profile memory (ram).	Restart the unit by switching the power off and on again. Notify KARL STORZ Technical Support if the problem persists.
ERROR - Invalid password	Enter Password (Advanced Setup)	An incorrect password was entered.	Acknowledge the message by pressing the Enter button. Reenter the password or press the "Go Back" button to return to Normal Setup.
Handpiece calibration needed	Version Screen	The system created a default calibration data file and has not been calibrated since.	Notify KARL STORZ Technical Support.
Release handpiece button to continue	Main Run Screen	The system detected that a handpiece button was pressed at startup.	Release the button. If the message remains on the screen, attempt the following before notifying KARL STORZ Technical Support: 1. Reconnect the handpiece. 2. Connect another handpiece. 3. Restart the unit by switching power off and on.
Release footswitch center button to continue	Main Run Screen	The system detected that the button in the center of the footswitch was active at startup.	Release the button. If the message remains on the screen, attempt the following before notifying KARL STORZ Technical Support: 1. Reconnect the footswitch. 2. Restart the unit by switching power off and on.
Release footpedal to continue	Main Run Screen	The system detected that a footpedal was active at startup.	Release the pedal. If the message remains on the screen, attempt the following before notifying KARL STORZ Technical Support: 1. Reconnect the footswitch. 2. Restart the unit by switching power off and on.
Release multifunction handpiece trigger to continue	Main Run Screen	The trigger on the multifunction handpiece was pressed at startup.	Release the trigger. If the message persists, then attempt the following before notifying KARL STORZ Technical Support: 1. Reconnect the handpiece. 2. Restart the unit by switching power off and on.
Unknown device - Please remove handpiece	Main Run Screen	The system did not recognize the ID of the device attached to the BK/1SK handpiece connector.	Remove and re-attach the handpiece. If the message persists, then restart the unit by switching power off and on. Notify KARL STORZ Technical Support if the message reappears.
Unknown device - Please remove multifunction handpiece	Main Run Screen	The system did not recognize the ID of the device attached to the multifunction handpiece connector.	Remove and re-attach the handpiece. If the message persists, then restart the unit by switching power off and on. Notify KARL STORZ Technical Support if the message reappears.
Motor Stalled	Main Run Screen	The motor was engaged but motor speed was below the minimum threshold.	Check and clean the handpiece blade if necessary. Restart the motor. If the message persists, disconnect the handpiece and notify KARL STORZ Technical Support.

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Technical data

Standard compliance (for 20721020-1)

According to IEC 60601-1, UL 2801,
CAN/CSA C22.2 No. 601.1-M90:
• Type of protection against electric shocks:
Protection Class I
• Degree of protection against electric shocks:
Applied part of type B

Warning: When using HF surgery, the shaver (handpiece and shaver blade) must not come in contact with the patient.

According to IEC 60601-1-2 :2001:

Please read the Electromagnetic Compatibility Information in the Appendix (p. 62-73).

Directive compliance (for 20721020-1)

This medical device belongs to Class IIa

This medical device bears the CE mark in accordance with the Medical Device Directive (MDD) 93/42/EEC.

A code number after the CE mark indicates the responsible notified body.

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Technical documentation

On request, the manufacturer will provide those circuit diagrams, itemized parts listings, descriptions, sets of adjustment instructions, and other items of available documentation to suitably qualified user personnel duly authorized by the manufacturer for their use in repairing those components of the instrument that have been designated as repairable by their respective manufacturers.

Supply of such technical documentation relating to the instrument shall not be construed as constituting manufacturer's authorization of user's personnel, regardless of their level of technical training, to open or repair the instrument.

Explicitly exempted herefrom are those maintenance and repair operations described in this manual.

Note: We reserve the right to make engineering modifications in the interest of promoting technological progress and generating performance improvements without obligation on our part to submit prior notice thereof.

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Spare parts, recommended accessories

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List of spare parts/accessories POWERSHAVER SL

Item	Cat. no.
Shaver handpiece BK	28721035
Shaver handpiece 15K	28721030
Small Joint Handpiece BK	28721040
Dual-pedal footswitch	20012831
Triple-pedal footswitch	20012832
Multifunction handpiece	28721036
Jacobs chuck 4 mm	28721031
Jacobs chuck 6.5 mm	28721037
Sagittal Saw	28721032
Pin driver	28721033
Synthes style driver	28721034
Power fuse	100-127 VAC 2 x T 5.0 A 250V, IEC 127 format 220-250 VAC 2 x T 2.5 A 250V, IEC 127 format
SCB Cable	20080170
Mains cord (grounded)	400 A
'Hospital Grade' mains cord (USA)	400 B
Instruction manual	96156009D

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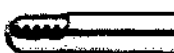
28205 AD

Recommended shaver blades and burrs

Aggressive cutter,
working length 120 mm, for removal of degenerative meniscus tissue and filaments of adhesion.

Item	Cat. no.
Reusable, sterilizable	
Diameter 3.5 mm, color code green	28205 AB
Diameter 4.2 mm, color code green	28205 AC

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28205 AD

Diameter 5.5 mm,
color code green
*Sterile, single use,
pack of 6*

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28205 ABS

Diameter 3.5 mm, color code
green/light gray

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28205 ADS

Diameter 5.5 mm, color code
green/dark gray

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End cutter,
working length 120 mm, for removal of degenera-
tive meniscus tissue, cicatrized areas of syno-
vial membrane; for removing large flaps of
cartilage.

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28205 CBS

Diameter 3.5 mm,
color code yellow/light gray

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28205 CDS

Diameter 5.5 mm,
color code yellow/dark gray

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Aggressive full radius resector,
working length 120 mm, for partial, subtotal, and
total synovectomy, for removal of degenerative
meniscus tissue and adhesion filaments.

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28205 DD

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**Spare parts,
recommended accessories**

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Sterile, single use,
pack of 6
Diameter 3.5 mm,
color code blue/light gray 28205 DBS
Diameter 4.2 mm,
color code blue/gray 28205 DCS
Diameter 5.5 mm,
color code blue/dark gray 28205 DDS

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Curved full radius resector,
distal end angled at 15°, for partial, subtotal,
and total synovectomy, for removal of degenerative
meniscus tissue and adhesion filaments. The
curvature enables inaccessible meniscus and
cartilage areas, and also synovial areas to be
reached.

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Sterile, single use,
pack of 6
Diameter 4.2 mm,
color code light blue/gray 28205 EGS

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Curved aggressive full radius resector,
distal end angled at 15°, for partial, subtotal,
and total synovectomy, for removal of degenerative
meniscus tissue and adhesion filaments. The
curvature enables inaccessible meniscus and
cartilage areas, and also synovial areas to be
reached.

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Sterile, single use,
pack of 6
Diameter 4.2 mm,
color code light green/gray 28205 EHS

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Round burr,
working length 120 mm, for removal of osteo-
phytes, for smoothing bone irregularities, for
milling the subchondral bone during subchondral
abrasion, for induction of fibrous cartilage

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Finish barrel burr
(10 flutes, 4 notches),
working length 120 mm, for removing large
areas, e.g. during subacromial decompression
at the shoulder; for removing osteophytes or a
notchplasty; for smoothing bone irregularities, for
milling the subchondral bone during subchondral
abrasion, for induction of fibrous cartilage; for
removing large areas of cartilage, e.g. in sub-
acromial decompression

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Aggressive barrel burr
(8 flutes, 4 notches),
working length 120 mm, for removing large
areas, e.g. during subacromial decompression
at the shoulder; for removing osteophytes or a
notchplasty; for smoothing bone irregularities; for
milling the subchondral bone in subchondral
abrasion, for induction of fibrous cartilage; for
removing large areas of cartilage, e.g. in sub-
acromial decompression

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**Recommended shaver blades and
burrs for small joint handpiece**

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Aggressive cutter,
working length 70 mm, for removal of degenera-
tive meniscus tissue and filaments of adhesion.

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Full radius resector,
working length 70 mm, for partial, subtotal, and
total synovectomy, for removal of degenerative
meniscus tissue and adhesion filaments.

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Aggressive full radius resector,
working length 70 mm, for partial, subtotal, and
total synovectomy, for removal of degenerative
meniscus tissue and adhesion filaments.

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Slotted Whisker,
working length 70 mm, for partial, subtotal, and
total synovectomy, for removal of degenerative
meniscus tissue and adhesion filaments.

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Accessories for pin driver

Drill wire, diameter 2.4 mm, spiral-ground	28729 D
Kirschner wire, diameter 2.4 mm, triangular-ground	28729 E

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Accessories for sagittal saw

Saw blade, blade width 10 mm, blade thickness 0.4 mm, cutting depth 0.6 mm, working length 50 mm	28207 SLA
Same, blade width 14 mm	28207 SLB

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Saw blade, blade width 10 mm, blade thickness 0.4 mm, cutting depth 0.6 mm, with stopper at 15 mm, working length 50 mm	28207 SLC
Same, blade width 6 mm	28207 SLD
Same, blade width 8 mm	28207 SLE

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Saw blade, blade width 6 mm, blade thickness 0.4 mm, cutting depth 0.6 mm, working length 31 mm	28207 SLG
Same, blade width 10 mm	28207 SLH

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Single-use blades and burrs
Normal use

The single-use sterile blades/burrs are intended for use with the POWERSHAVER SL.

Use of the single-use sterile blades/burrs in any way other than that described is not permitted.

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Use of the single-use products and explanation of the symbols employed

 **Warning:** Before use, check the expiration date and ensure that the packaging is undamaged. If the expiration date has passed or the packaging is damaged, the content of the pack must not be used under any circumstances.

 **Warning:** This product is intended for single use and must not be re-sterilized. Dispose of the product after use. Do not allow the blade to run 'dry'. The recommended speed of rotation for the blades is 4,000 rpm.

Note: This product is individually wrapped in protective packaging and sterilized with gamma radiation.

Manufacturer:
Manufacturer in accordance with Directive 93/42/EEC:
KARL STORZ GmbH & Co. KG
Mittelstraße 8, 78532 Tuttlingen, Germany

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Traceability

To ensure traceability, the self-adhesive identification labels enclosed in the original packaging can be kept and stuck in the operation report and in the patient dossier.

The person responsible for stores at the hospital can maintain a record of the products currently in store, including article no. (REF) and lot no. (LOT) so that rapid action can be taken in the event of changes or recalls.

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WARNING: Medical electrical equipment needs special precautions regarding Electromagnetic Compatibility (EMC). Observe the EMC instructions in this appendix during installation and commissioning.

The POWERSHAVER SL model 20721020-1 corresponds to ENIEC 60601-1-2:2001 (CISPR 11 Class B) and therefore meets the EMC requirements of the Medical Device Directive (MDD) 93/42/EEC.

These limits are designed to provide reasonable protection against the typical electromagnetic interference to be expected in a medical environment. The POWERSHAVER SL model 20721020-1 is a Group 1 unit (as per CISPR 11). Group 1 contains all the "equipment and systems which generate or use RF energy only for their internal functioning".

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NOTE: The tables and guidelines that are included in this appendix provide information to the customer or user that is essential in determining the suitability of the equipment or system for the electromagnetic environment of use, and in managing the electromagnetic environment of use to permit the equipment or system to perform its intended use without disturbing other equipment and systems or non-medical electrical equipment. If this equipment does cause harmful interference with other devices, the user is encouraged to try to correct the interference by one or more of the following measures:

- reorient or relocate the receiving device
- increase the separation between the equipment
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) is connected.

If you have any further questions, please contact your local representative or our service department.

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WARNING: The POWERSHAVER SL model 20721020-1 should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the equipment or system should be observed to verify normal operation in the configuration in which it will be used.

WARNING: The use of portable and mobile RF equipment may have an impact on this and other pieces of medical equipment.

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Table 201 Guidance and manufacturer's declaration – electromagnetic emissions		
POWERSHAVER SL model 207210 20-1 is intended for use in the electromagnetic environment specified below. The customer or user of the POWERSHAVER SL model 207210 20-1 should ensure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	POWERSHAVER SL model 207210 20-1 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The POWERSHAVER SL model 207210 20-1 is suitable for use in all establishments (e.g. hospitals and doctors' offices) including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

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Table 202 Guidance and manufacturer's declaration - electromagnetic immunity			
POWERSHAVER SL model 207210 20-1 is intended for use in the electromagnetic environment specified below. The customer or user of the POWERSHAVER SL model 207210 20-1 should ensure that it is used in such an environment.			
Immunity test	EN/IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	Complies ± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Complies ± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	Complies ± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U _n * (>95% dip in U _n) for 0.5 cycle 40% U _n (80% dip in U _n) for 5 cycles 70% U _n (30% dip in U _n) for 25 cycles <5% U _n (>95% dip in U _n) for 5 sec	Complies <5% U _n * (>95% dip in U _n) for 0.5 cycle Complies 40% U _n (80% dip in U _n) for 5 cycles Complies 70% U _n (30% dip in U _n) for 25 cycles Complies <5% U _n (>95% dip in U _n) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the equipment or system requires continued operation during power mains interruptions, it is recommended that the equipment or system be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-6	3 A/m	Complies 3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

* Note: U_n is the a.c. mains voltage prior to application of the test level.

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68

Table 204 Guidance and manufacturer's declaration - electromagnetic immunity - for equipment and systems that are not life-supporting			
POWERSHAVER SL model 207210 20-1 is intended for use in the electromagnetic environment specified below. The customer or user of the POWERSHAVER SL model 207210 20-1 should ensure that it is used in such an environment.			
Immunity test	EN/IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6 Radiated RF IEC 61000-4-3	3 V _{ms} 150 kHz to 80 MHz 3 V/m 80 MHz to 2.5 GHz	3 V _{ms} 3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the POWERSHAVER SL model 207210 20, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = \sqrt[3]{5/3 \cdot P}$ $d = \sqrt[3]{5/3 \cdot P}$ 80 MHz to 800 MHz $d = \sqrt[3]{1/3 \cdot P}$ 800 MHz to 2.5 GHz Where P is the maximum output power rating of the transmitter in watts [W] according to the transmitter manufacturer and d is the recommended separation in meters [m]. Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey*, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 
Note 1: At 80 MHz and 800 MHz, the higher frequency range applies Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection by structures, objects and people.			
a Field strengths from fixed transmitters, such as base stations for radio [cellular/mobile] telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the unit is used exceeds the applicable RF compliance level above, the unit should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the unit. b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

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71

Table 206 Recommended separation distances between portable and mobile RF communications equipment and the POWERSHAVER SL model 20 7210 20-1			
The POWERSHAVER SL model 20 7210 20-1 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the unit can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the unit as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter [W]	Separation distance d [m] according to frequency of transmitter		
	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2.5 GHz
	$d = \left[\frac{3.5}{3} \right] \sqrt{P}$	$d = \left[\frac{3.5}{3} \right] \sqrt{P}$	$d = \left[\frac{7}{3} \right] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

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WARRANTY

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship.
In doing so we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Complete warranty card upon purchase/delivery and return immediately to:

KARL STORZ GmbH & Co. KG
Postfach 230
78503 Tuttlingen/Germany

To be completed by the supplier/importer:
Company stamp/signature:

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REPLY CARD
KARL STORZ GmbH & Co. KG
Postfach 230
78503 Tuttlingen/Germany

Please attach
sufficient
postage

To be filled out by instrument owner:
Ratun address/company stamp:
Field of application:
Type of instrument: Serial no.:
Purchase date:
Signature/Date:

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KARL STORZ Sports Medicine
 1 Pine Hill Drive, Suite 105
 Batterymarch Park,
 Quincy MA 02169, USA
 Phone: +1 866 5 50 26 63
 Fax: +1 617 3 76 21 07

KARL STORZ Veterinary Endoscopy America, Inc.
 175 Cremona Drive
 Goleta, CA 93117, USA
 Phone: +1 805 968-7776
 Fax: +1 805 685-2588
 E-Mail: info@karlstorzvet.com

KARL STORZ Endoscopia Latino-America, Inc.
 815 N. W. 57th Avenue, Suite 480
 Miami, FL 33126-2042, USA
 Phone: +1 305 262-8980
 Fax: +1 305 262-8986
 E-Mail: info@kseas.com

KARL STORZ Endoscopy Canada Ltd.
 2345 Argentia Road, Suite 100
 Mississauga, ON, L5N 8K4
 Phone: +1 905 816-8100
 Fax: +1 905 858-0933
 E-Mail: info@karlstorz.ca

KARL STORZ Endoscopia
 Miramar Trade Center
 Edificio Jerusalem, Oficina 308 A
 La Habana, Cuba
 Phone: +53 7204 1097
 Fax: +53 7204 1098

KARL STORZ Endoscopia México S.A. de C.V.
 Lago Constanza No 326
 Col. Chapultepec Morales,
 México, D.F.C.P. 11520, Mexico
 Phone: +52 55 52 50 56 07
 Fax: +52 55 55 45 01 74

KARL STORZ Endoscopia Argentina S.A.
 Cerviño 4449 Piso 10°
 1425 Buenos Aires C. F., Argentina
 Phone: +54 11 47 72 45 45
 Fax: +54 11 47 72 44 33
 E-Mail: marketing@karlstorz.com.ar

KARL STORZ Endoscopy (UK) Ltd.
 392 Edinburgh Avenue, Slough
 Berkshire, SL1 4UF, Great Britain
 Phone: +44 1753 503500
 Fax: +44 1753 578124
 E-Mail: customerservice@karlstorz-uk.com

KARL STORZ Endoscopy Sverige AB
 Storsåtravägen 14,
 12739 Skärholmen, Sweden
 Phone: +46 8 50 56 4800
 Fax: +46 8 50 56 48 48
 E-Mail: kundservice@karl-storz.se

KARL STORZ Endoskopi Danmark A/S
 Skovlystvej 33
 2840 Holte, Denmark
 Phone: +45 45 162600
 Fax: +45 45 162609
 E-Mail: marketing@karlstorz.dk

KARL STORZ Endoscopy Belgium N.V.
 St. Annalaan 3 bus 1
 1853 Strombeek-Bever, (Grimbergen), Belgium
 Phone: +32 22 67 38 10
 Fax: +32 22 67 45 01
 E-Mail: info@stopler.be

KARL STORZ Endoscopia Nederland B.V.
 Ursulinenhof 1, 4133 DA Vianen, Netherlands
 Phone: +31 34 7373151
 Fax: +31 34 7370658
 E-Mail: karlstorz@planet.nl

KARL STORZ Endoscopia France S.A.
 12, rue Georges Guynemer
 Quartier de l'Europe
 78280 Guyancourt, France
 Phone: +33 1 30 48 42 00
 Fax: +33 1 30 48 42 01
 E-Mail: marketing@karlstorz.fr

KARL STORZ Endoskop Austria GmbH
 Landstraßer Hauptstr. 146 11 18
 1030 Wien, Austria
 Phone: +43 1 71 56 04 70
 Fax: +43 1 71 56 04 79
 E-Mail: storz-austria@karlstorz.at

KARL STORZ Endoscopia Italia S. r. l.
 Via dell'Artigianato, 3
 37135 Verona, Italy
 Phone: +39 045 822 2000
 Fax: +39 045 822 2001
 E-Mail: info@karlstorz.it

KARL STORZ Endoscopia Ibérica S. A.
 Parque Empresarial San Fernando
 Edificio Francia - Planta Baja
 28830 Madrid, Spain
 Phone: +34 91 677 1051
 Fax: +34 91 677 2981
 E-Mail: marketing@karlstorz.es

KARL STORZ South Africa, (Pty) Ltd.
 P. O. Box 3039,
 Cape Town 8000, South Africa
 Phone: +27 11 781 5781
 Fax: +27 11 781 2660
 E-Mail: info@karlstorz.co.za

KARL STORZ Endoscopy (S) Pte. Ltd.
 #05 - 08 San Centre,
 171 Chin Swee Road
 Singapore 0316, Singapore
 Phone: +65 65 32 55 48
 Fax: +65 65 32 38 32
 E-Mail: karlstorz@pacific.net.sg

KARL STORZ Endoscopy Japan K. K.
 Bois Hongo Building 6F1
 3-42-5 Hongo
 Bunkyo-ku, Tokyo 113-0033, Japan
 Phone: +81 3 5802-3966
 Fax: +81 3 5802-3988
 E-Mail: info@karlstorz.co.jp

KARL STORZ Endoscopy China Ltd.
 Suite 1601-2, 16/F
 Chinachem Exchange Square,
 1-7 Hoi Wan Street, Quarry Bay,
 Hong Kong, China
 Phone: +8 52 28 65 24 11
 Fax: +8 52 28 65 41 14
 E-Mail: kstorzhk@netvigator.com

KARL STORZ GmbH & Co. KG
 Resident Representative Office
 80/33 (44/19) Dang Van Ngu
 F 10 Q Phu Nhuan, Ho Chi Minh City, Vietnam
 Phone: +84 89 91 84 42
 Fax: +84 88 44 03 20

OOO KARL STORZ Endoscopy - WOSTOK
 Gogolevskiy Blvd. 3/1/2
 119019 Moscow, Russia
 Phone: +7 095 73 78113
 Fax: +7 095 73 78114
 E-Mail: kste-wostok@karlstorz.ru

KARL STORZ Endoscopy India Private Ltd.
 C-126, Okhla Industrial Area, Phase-1,
 New Delhi 110020, India
 Phone: +91 11 26 81 54 45-51
 Fax: +91 11 26 81 29 86
 E-Mail: karlstorz@vsnl.com

KARL STORZ Endoskope
 Regional Center for Endoscopy S.A.L.
 St. Charles City Center, 5th Floor
 Phoenixia Street, Mina Elhosn
 2020 0908 Beirut - Lebanon
 Phone: +961 1 368181
 Fax: +961 1 365151

KARL STORZ Industrial
 Gedik Is Merkezi B Blok
 Kat 5, D 38-39
 Bagdat Cad. No: 162
 Maltepe Istanbul, Turkey
 Phone: +90 216 442 95 00
 Fax: +90 216 442 90 30
 Sales for Industrial Endoscopy