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Instruction Manual

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**Important information for users of
KARL STORZ instruments..... II**

Information regarding this instruction manual	II
Pictures of the instruments	III
Controls, connectors and their uses	V

Safety instructions 3

Normal use.....	3
User qualification	3
Safety precautions when using the endoscopes	4
Warnings and cautions	4

Handling 7

1.0 Scope of supply.....	7
2.0 Use of the endoscope	8
2.1 Preparation of the light source	8
2.2 Connecting the light conductor, air/water con- nection and the suction channel	8
2.3 Preparation of the angle of the distal tip.....	8
2.4 Suction	9
2.5 Insufflation and light	9
2.6 Water rinsing.....	9
2.7 Connecting the videoscop to a light source ...	10
3.0 Preparation for use	11
3.1 Test of the various functions	11
3.2 Dismantling and assembling the Albarran Unit.....	14
3.3 Correct maintenance of the endoscope.....	17
3.4 Recommendations for use.....	18
3.5 How does the endoscope buttons work.....	19

**Cleaning, care
and sterilization20**

General information	20
Preparation for cleaning.....	22
Cleaning procedure	23
Leakage test.....	25

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Cleaning of the Albarran Module	26
Cleaning the endoscope: hand cleaning	27
Cleaning the endoscope: By machine	32
Recommended procedure for the preparation of video endoscopes	33
Cleaning accessories.....	34
Cleaning the water bottle.....	34

Disinfection/Sterilization	36
Disinfection of the endoscope	36
Gas sterilization	37
Storage	39

Maintenance	40
Servicing and repair	40
Limitation of liability	40
Warranty	40
Directive compliance	40
Important information	41

Appendix	42
Trouble Shooting.....	42
Chemicals for the preparation of Telescopes and endoscopic instrument sets	44

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Please read these safety instructions carefully before using the instrument. Make certain you are well acquainted with the function and operation of the instrument before initial use on the patient.

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Normal use

The model KARL STORZ Video-Gastroscope has been developed for endoscopic diagnosis and treatment in the upper digestive tract, including the oesophagus, the stomach and the duodenum.

The KARL STORZ Video-Duodenoscopes were developed for endoscopic diagnosis and treatment in the duodenum.

The model KARL STORZ Video-Sigmoidoscope and KARL STORZ Video-Colonoscope have been developed for endoscopic diagnosis and treatment in the lower digestive tract, including the rectum and large bowel. These instruments should not under any circumstances be used for other purposes.

Contraindications:

No contraindications are known which directly relate to the product. The video endoscope must not be used if, in the opinion of a qualified physician, such an application would endanger the patient, e.g. due to the patient's general condition, or if the endoscopic method as such is contraindicated.

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User qualification

KARL STORZ video endoscopes may only be used by persons with an appropriate medical qualification and who are acquainted with the endoscopic technique. The information given in these instructions only serves to instruct in the correct handling and cleaning of the video endoscopes. It is not intended as an introduction to the endoscopic technique.

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Safety precautions when using the video endoscopes

Video endoscopes must be used according to the medical rules and procedures of endoscopy recognized for this method.

Warnings and cautions

Please read this manual and follow its instructions carefully. The words, **Warning**, **Caution** and **Note** convey special meanings. Wherever they are used in this manual, the text should be carefully reviewed to ensure the safe and effective operation of this product. To make the words **Warning**, **Caution** and **Note** stand out more clearly, they are accompanied by a pictogram.

Warning: A Warning indicates that personal safety of the patient or physician may be involved. Disregarding Warning could result in injury to the patient or physician.

Caution: A Caution indicates that particular service procedures or safety precautions must be followed to avoid possible damage to the product.

Note: A Note indicates special information about operating the product, or clarifies important information.

Warning: Before using the video endoscope, read the following safety instructions carefully, to avoid putting your patients, personnel or yourself at risk.

Warning: Check the video endoscope prior to each surgical procedure to ensure that it functions correctly and has been properly cleaned, disinfected and/or sterilized.

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Warning: Verify the sound condition of the video endoscope as well as of accessories used in combination before every application. Damaged video endoscopes or damaged accessories must not be used.

Warning: Combinations of medical devices are only then assured to be safe if

- they are identified as such in the respective instruction manual or
- the intended purpose and the interface specifications of the devices used in combination permit this.

Pay careful attention to the instructions and interface specifications of medical products used in combination.

Warning: High energy light radiated through endoscopes may give rise to high temperatures in front of the light outlet, at the end faces of the light guide and at the tip of the endoscope.

Warning: The video endoscope is delivered unsterilized and must be disinfected/sterilized before the initial use as well as before any further application.

Warning: Remove the pressure compensation cap from the vent port:

- for every endoscopic intervention
- before immersion in liquid/
chemical disinfection.

Warning: Before removal from the patient, the distal tip of the video endoscope must be returned to the neutral, nondeflected position (L and U on top of each other and positioned between the valves) to prevent damage to the instrument and/or injury to the patient.

Warning: If a malfunction should occur during application on a patient, stop the application immediately. Place the distal tip of the video endoscope in the neutral, nondeflected position and remove the video endoscope slowly and carefully from the patient.

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Caution: Do not steam sterilize (autoclave) the video endoscopes.

Caution: The maximum permissible temperature for decontamination, sterilization, transport and storage is 60°C.

Caution: Never immerse video endoscopes in physiological saline solution, since only brief contact leads to pitting and corrosion.

Caution: Place the pressure compensation cap on the vent port before

- gas sterilization/plasma sterilization
- shipping

Caution: Only use KARL STORZ accessories (guide probes, cleaning brushes etc.). Accessories from other manufacturers may damage the video endoscopes.

Caution: Only use guide probes with flexible ends. Other types of guide probe may cause considerable damage to video endoscopes.

Caution: Make certain to read the "Cleaning, sterilization and care of KARL STORZ instruments" instructions. Methods of cleaning, disinfection and sterilization are explained in detail there.

Note: Any damage to the video endoscope resulting from incorrect operation is not covered by the guarantee.

Note: Follow local governing ordinances and recycling plans regarding disposal or recycling of device components.

Note: Federal law restricts this device to sale by or on the order of a physician (USA only).

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2.0 Use of the video endoscope

2.1. Preparation of the light source

We recommend our light source with 100 Watt Xenon lamp. The minimum requirement is a 150 watt halogen lamp or a 100 Watt xenon lamp.

Please read the full instructions in the light source instruction manual (observe the precautions for using Xenon lights).

1. Install the light source on a solid even surface
2. Insert the power cable into the socket provided

2.2. Connecting the light conductor, air/water connection and the suction channel

1. Insert the plug into the light source (Fig. 1).
2. Connect video connector (Fig. 2).
3. Connect the tube from the water bottle to the endoscope supply plug (rinse bottle connector, fig. 3). The water bottle holder automatically positions itself correctly and cannot be adjusted manually (see page 10).
4. Connect the tube from the suction pump to the endoscope plug (suction pump connection) (see page 10).

IMPORTANT: To fill/empty the water bottle, remove the container from the light source and undo the cover.

2.3. Preparation of the angle of the distal tip

Rotating the large control wheel, inscribed "U/D", clockwise lowers the distal tip. Rotation anticlockwise raises the tip.

Right and left movements are controlled using the smaller control wheel (inscribed "L/R"). Clockwise = right/ anticlockwise = left (fig. 4)

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Warning: Do not rotate the wheels forcibly, so as not to damage the instrument or traumatize the patient! Before attempting to introduce or remove the instrument, release the control wheel brakes!

2.4. Suction

Connect the suction pump. To aspirate, press on the suction knob (with the red ring) on the endoscope.

Suction can also be used if the biopsy forceps are placed in the biopsy channel and the biopsy forceps valve is mounted. During suction, the forceps valve seals off the forceps.

2.5. Insufflation and light

Start the light source, connect the air pump.

The air pump forces air from the opening on the upper side of the air/water valve (with the blue ring).

For insufflation, cover the air control opening with a finger (do not depress the valve, since this starts water rinsing).

Note: To avoid patient discomfort, use insufflation sparingly. Excessive insufflation may cause cramps, spasms and injuries!

Note: To increase the amount of air in internal organs, while the endoscope is being withdrawn, the air valve must be activated at intervals.

2.6 Water rinsing

Pressing on the air/water valve starts the water flow and/or lense cleaning. Different valve positions allow various combinations.

IMPORTANT: Only fill the water bottle 2/3 (1/3 is required to cushion air pressure).

DO NOT PULL DIRECTLY ON THE CAP VALVE!

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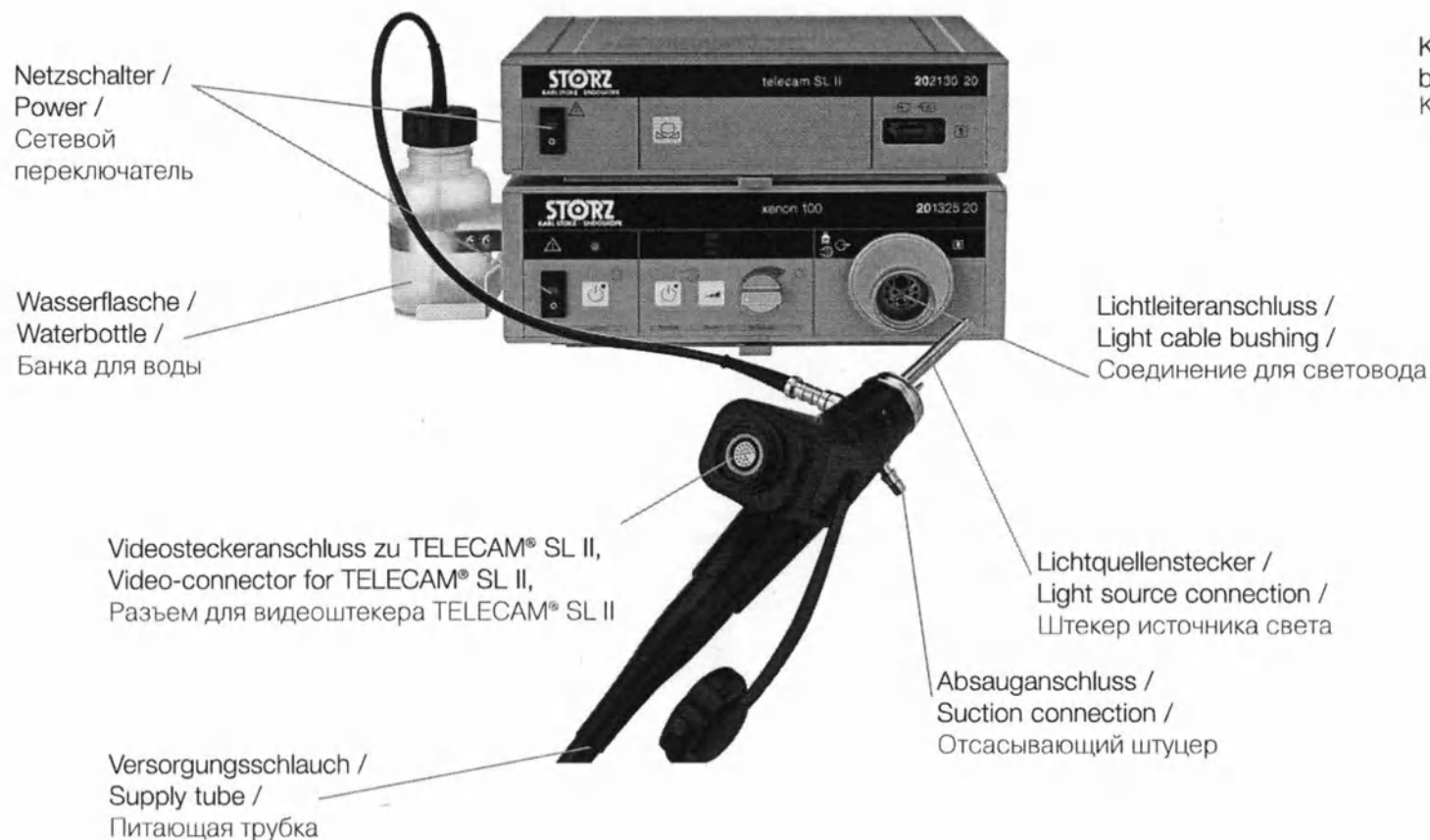
2.7 Connecting the Videoscope to a light
source illustrated with the
KARL STORZ XENON 100

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Connection of light source connector with XENON 100 and water bottle (at Modell 13910, 13911, 13807, 13806 there is also a Jet – nozzle at the connector)

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3.0 Preparation for use

Note: Before use for the first time, the instrument should undergo a complete cleaning cycle, including preliminary cleaning!

3.1 Test the various functions

1. Prepare water/light sources as described in 2.1.
2. Connect supply connector, water bottle and suction tube according to 2.2.
3. Check the supply tube for damage (e.g. breaks, tears, stretched or squashed spots...) and the supply connectors for loose connections.
4. Examine the surface of the flexible shaft by eye and by touch for tears, cuts, bulges, foreign bodies or other damage. **Do not press too hard on the bending section!!**

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3.2 Check the movement and function of the control wheels

a) Testing up and down control.

The large control wheel controls up and down movements. To test up and down movements, release the up and down brake. Slowly rotate the large control wheel clockwise – the end piece should move downwards. Rotating the control wheel counterclockwise causes upwards movement.

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b) Inspection of right/left control

The smaller control wheel controls right/left movement. To test right/left movement, release the right/left brake. Slowly rotate the small control wheel clockwise – the end piece should move to the right. Rotating the control wheel counterclockwise causes movement to the left.

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c) Inspection of the brake function
Release the brakes and test their action. Activate the brakes by turning the brake control counterclockwise – ensure that the angled position of the endpiece is fixed in the desired position. The control wheels and brakes should turn freely when the brakes are released.
Do not twist or bend the bending piece by hand!

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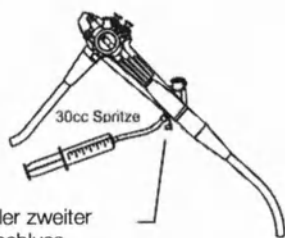
6. Check the air supply. Attach the air pump and hold the distal end of the instrument under water. Cover the air control opening in the air valve (blue ring) with a finger. Bubbles should emerge from the air nozzle.

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7. Check the water supply:
a) Depress the air/water knob (blue ring) fully. Water should flow out of the water nozzle (distal). **IMPORTANT:** when you first depress the knob, it may take some seconds for water to emerge.
b) Using a syringe, inject water manually through the manual rinsing connector (custom made). Water should flow out of the water nozzle in the distal end.

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8. Test the suction function. With the instrument and the suction pump correctly installed, place the distal end under water again. Depress the suction knob (red ring) fully. Water should flow through the instrument and into the water bottle of the suction equipment.

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9. Test the biopsy channel, by introducing biopsy forceps.

Caution: it should be possible to guide the biopsy forceps smoothly and without problems through the entire channel.

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Accessories such as biopsy forceps that are folded or bent will not function properly, are difficult to guide through the channel and may cause severe damage to it. They must therefore be replaced!

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3.2 Dismantling and assembling the Albarran Unit (Video-duodenoscopes only)

As conventional duodenoscopes are difficult to clean and maintain due to their particular construction, KARL STORZ has developed a modular Albarran system.

The patented Albarran Module is easy to dismantle, which facilitates thorough cleaning. It is no longer necessary to dismantle the entire endoscope to remedy technical malfunctions. As a rule, repairs can be carried out by dismantling the Albarran unit.

Dismantling the Albarran unit

1. Remove the cable as shown in Fig. 1 by turning the rotating knob on the proximal endoscopic housing. Unscrew the rotating knob (2) from the proximal housing.
2. Remove the released collet (11) (Fig. 2)
3. Releasing the Albarran module from the distal endoscope head (Fig. 3)
The locking screw (6) of the Albarran module unit is indicated with two red spots.
The red spots are turned counterclockwise towards each other using a special screwdriver (7).
The Albarran module is now released (Fig. 4).
4. Removal of the Albarran module with Albarran cable (Fig. 5)
Separate the Albarran module (3) from the endoscope with the thumbs and carefully remove with the cable from the endoscope (4). Do not bend or kink the cable!
5. Removal of the Albarran cable guide (Fig. 6)
Bend the distal tip of the endoscope by turning the small control wheel to the right ("R").
The Albarran cable guide (5) is now visible at the distal endoscope head and can be removed.

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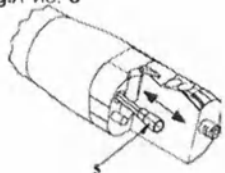
3. Attach the Albarran module (Fig. 2) by turning the locking screw (6) clockwise with the screwdriver (7)

The two red spots must now be opposite each other (Fig. 9).

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4. Testing the module locking: Locking of the Albarran module must be tested by trying to press on the head of the Albarran module with the thumbs.


Caution: Do not turn the locking screw in the anti-clockwise direction – risk of damage!

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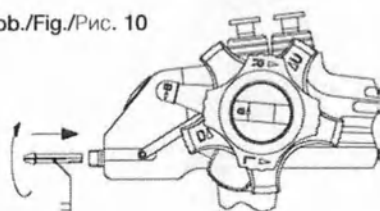
5. Place the shaft of the endoscope on a table.
6. Introduce the collet (11) into the proximal housing while twisting them until they engage and cannot be turned any further (Fig. 10).

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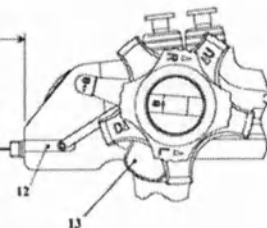
7. Lock the control carriage (12) onto the endoscope housing as shown in Fig. 11.
8. Screw up and lock the rotating knob (2) as seen in Fig. 12.

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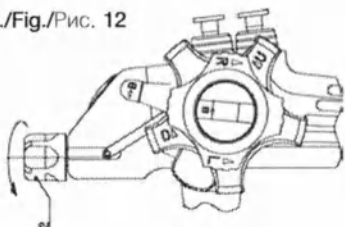9. Testing Albarran function: Grasp the Albarran lever (10) shown in Fig. 11 at the distal end with the thumbs and at the same time work the proximal Albarran control (Fig. 12).

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Significant movement pressure must be felt in the Albarran lever through the thumbs. The Albarran unit is functioning correctly.

Advice: This documentation covers only handling and care of the Albarran system.

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3.3 Correct maintenance of the endoscope

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Control wheels for angling (the direction of movement is always in the direction of the arrow/ brakes are activated in the direction of the arrows)

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Correct maintenance of the endoscope

With the control handle of the instrument in the left hand, the thumb, index and middle fingers can easily control the large control wheel. In this position, the suction valve can be depressed with either the index or the ring finger. The air/water valve can also be covered with these fingers.

The smaller projecting control wheel can be controlled completely with the fingers of the left hand, or alternatively with the thumb and fingers of the right hand. It is necessary and desirable to activate both control wheels at the same time.

3.4 Recommendations for use

1. Use a water-soluble lubricant for the distal end of the tube.
2. Observe the lumen the whole time. If you become disoriented, withdraw the instrument gently until vision is restored.

Note: Do not put lubricant on the lens to avoid clouding!

Note: Do not use petroleum-, silicone- or Vaseline-based media, to avoid damage to the endoscope material.

Note: If contamination of the objective occurs during the procedure, this can normally be cured by a combination of water and insufflation. Excessive water or air can be removed by simultaneous pressure on the suction knob.

3. Suction is possible while the biopsy channel is in use and the biopsy valve is mounted. The forceps valve completely seals off the forceps.

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4. Bending the tip is described in 3.1.- 5a and b (Picture: Page 11/17).
5. Before withdrawing the instrument, ensure that the brakes are off.

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3.5 Operating the endoscope buttons

Each button can be individually programmed. E.g. One short press triggers the freeze function (still image), another short press returns to the video function and a longer press (5-7 seconds) prints the freeze image (only if a printer is configured in the system).

Note: „Red-out“, a blurred red picture in the lens caused by the instrument head resting directly on the organ surface > withdraw the instrument gently until vision is restored.

Warning: Pushing the instrument forwards despite a „red-out“ effect may result in serious injury to the patient.

Note: Only use insufflation sparingly. Excessive insufflation may cause cramps/spasms and may injure the patient. To remove excessive air, carry out suction repeatedly.

Note: If angling of the tip does not function properly, stop the examination immediately. Release the brakes; place the valves in the neutral position. Align the control wheels with „U“ and „L“ (see page 11/17), and withdraw the instrument carefully observing the process through the instrument.

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General information

Warning: The listed cleaning, disinfection and sterilization methods alone do not guarantee disinfection or sterility. This can only be achieved if the staff follows recognized and validated preparation methods.

Warning: For all cleaning of contaminated endoscopes and instruments, liquid-proof protective gloves must be worn. When using a cleaning gun, safety goggles are necessary. Aerosols should be removed using a suction device.

Warning: Remember that sterilization is only then successful if

- the instruments have been freed of all organic material and cleaning residues. Only clean surfaces can be reached by the sterilization.
- work proceeds according to recognized and validated methods.
- any deviation from recommended procedures must be validated by the user.
- the recommended sterilization parameters are maintained. These are only valid with sterilization equipment that is properly maintained and calibrated.

Caution: The instructions "Cleaning, sterilization and care of KARL STORZ Instruments" must be followed. There, the cleaning, disinfection and sterilization procedures are explained in detail. The list of KARL STORZ approved materials can be found on the Internet at "www.karlstorz.com" or "www.karlstorz.de" or in the Annex of this manual.

Caution: For the production and application of solutions, precisely observe the information from the manufacturer regarding mixing ratios and immersion times. Prolonged immersion and an incorrect concentration can lead to damage. Pay attention to the microbiological range of action of the chemicals used.

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Caution: Never clean video endoscopes with a metal brush.

Caution: Constantly changing between different preparation methods causes particular wear on the materials and must therefore be avoided. A once-only change in methods however, e.g. when acquiring new cleaning equipment, is perfectly safe.

Caution: Video endoscopes must not be cleaned in an ultrasonic bath.

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Caution: Never immerse video endoscopes in physiological saline solution, since even brief contact can lead to pitting and corrosion.

Caution: Video endoscopes must only be rinsed with demineralized water, as mineral crystals may prevent successful sterilization.

Caution: Cleaning in an instrument washing machine is only possible following suitable procedures.

Caution: The maximum permissible temperature for decontamination, sterilization, transport and storage is 65 °C.

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Preparation for cleaning

1. The preliminary cleaning of the endoscope is performed at the place of examination. The main cleaning is carried out in the preparation room.

Immediately after use, clean the exterior of the video endoscope of heavy soiling (secretions, etc.).

To do this use a disposable cloth soaked with disinfectant or rinse off the video endoscope with a hand spray. Submerge the distal end in a container with cleaning solution and activate the suction to clean the working channel.

2. Put the cleaning valve into the air/water channel 2. (this valve is not for normal use!)

3. Check that the water bottle is filled (max 2/3). Start the pump. Now water and air are passed through the air channel as well as through the water channel. (The process should last for at least 10 seconds).

4. Pull out the video plug and put on the sealing cap for the video cable connector and screw it on tightly (see also 5.1)

It is recommended that you pump up the endoscope before cleaning at a max. of 160 mmHg, releasing the pressure only when cleaning and/or disinfection is completed.

5. Remove instrument from pump and light source. Place the cleaning valve in solution.

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Cleaning Procedure

Cleaning should only be undertaken by authorised trained staff. All protective measures should be observed (gloves, protective glasses, masks, clothing...).

Thorough cleaning of the endoscope and all its accessories after every use ensures problem-free working and the best results for disinfection or sterilisation procedures (gas).

The basic guidelines for cleaning and care of instruments (for example RKI, ESGE/ESGENA) should be observed.

Caution: Before soaking in disinfectant, check the instrument for leaks. Otherwise fluid penetrating the video endoscope could cause damage.

Caution: Do not use autoclaving for the endoscope or accessories (with the exception of accessories that are specifically designated as autoclavable, e.g. forceps).

Note: The endoscope must be cleaned immediately after every use. If material/accessories are allowed to remain and dry in the instrument, complete or partial blockage of the channel may result. Thorough cleaning is a prerequisite for effective disinfection and sterilization. This will avoid clots/ blockages and the resulting repairs and instrument failures. Suitable enzymatic cleaners should preferably be used for the preliminary cleaning for reasons of material compatibility. Also, cleaning agents containing white oils (Zylokain) should not be used.

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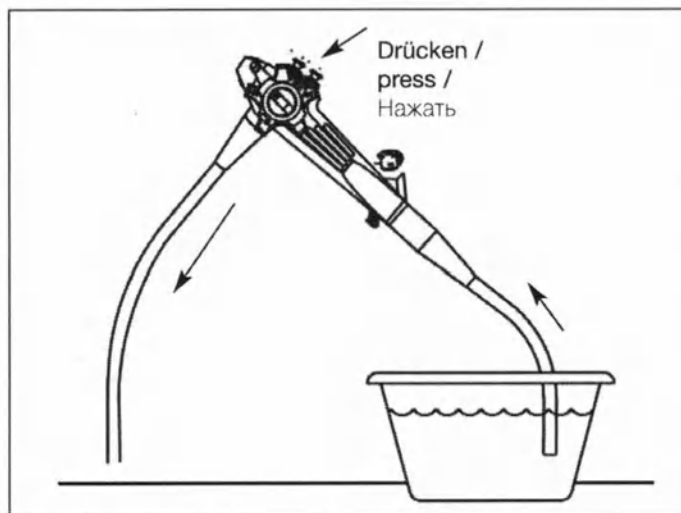
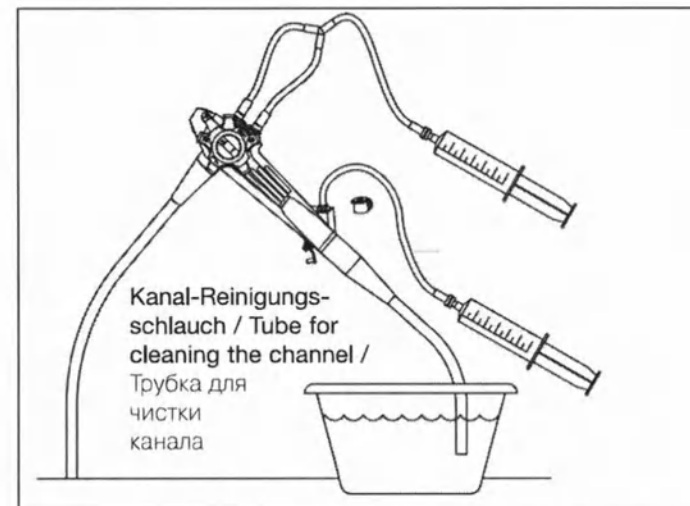
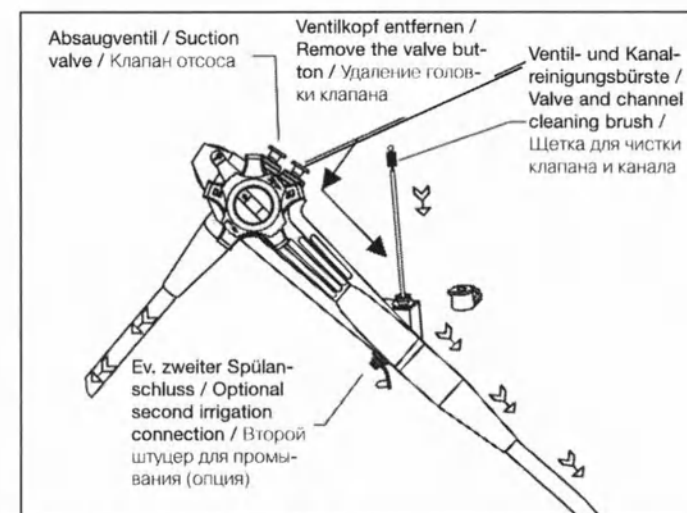
Leakage test

Before any procedure test the endoscope for leaks.

1. Remove all valves.

2. Close the video cable connector. (Only when the top closes tightly to the video connectors body (blue rubber can't be seen) the endoscope is waterproof), (blue rubber seal should be removed and slightly moistened with instrument oil).
3. The connection tube for the leakage test and the test connector on the endoscope must be dry. Place the tester connection cap tightly on the tester connector and turn clockwise through 90°.

4. Inflate the seal tester until the maximum test pressure of 160 mmHg is reached. The rubber case on the distal angle piece of the endoscope inflates slightly (if this is not possible, contact KARL STORZ).
5. Wait five minutes. If the pressure does not fall by more than 10 mmHg, the instrument is watertight and can be immersed. Start cleaning and disinfecting the endoscope. After cleaning, depress the red lever on the end of the seal tester to let down the pressure within the instrument. (Before further use of the seal tester, the red lever must be reset to the initial position again.) The test pressure must fall to 0. Remove the seal tester.

Note: During inflation, make certain that the red pressure relief button is completely closed.

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If there is a loss in pressure, the videoscope has a leak and must not be used or immersed in liquid.

In this case, send the sterilized videoscope to KARL STORZ for repair, or contact KARL STORZ. Please attach a sterilization indicator to the documentation.

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Warning: Danger of infection during dispatch! Contaminated instruments may cause serious or life-threatening illnesses during transportation or on delivery to the recipient. Before dispatching a video scope, subject it to gas or plasma sterilization with the pressure compensation cap fitted.

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Cleaning of the Albarran Module

The following components can be cleaned in the ultrasound bath:

- Albarran module with cable
- Albarran cable guide
- Collet
- Rotating knob

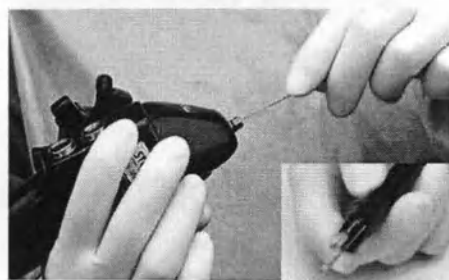
Carefully clean the Albarran module at the distal end with a soft brush

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After examination, the instrument can be rinsed with the irrigation tube (Art. No. 13991 PS) and brushed with the cleaning brush (Art. No. 27651 F) and cleaning agent starting with the proximal housing.

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Starting from the proximal housing area, brush using the cleaning brush (Art. no. 27651 H) and cleaning agent. Before retracting the brush head that emerges at the distal end, clean it using another brush or compressed air.

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Then blow dry the channels with compressed air.

Caution: The pressure must not exceed 50 kPa (0.5 bar/7.25 psi). The cleaning gun might therefore have to be operated with a pressure reducer.

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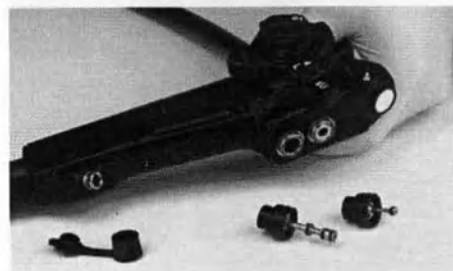
Cleaning/disinfection of the video endoscope: Hand cleaning

Please adhere to the general standards, e.g. ESGENA guidelines or DEGEA/RKI recommendations for cleaning, disinfection and sterilization, as well as to the instructions of the cleaning agent manufacturer.

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1. Remove forceps valve, suction valve and air/water valve and place in solution.

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2. Pull out the video plug and put on the sealing cap for the video cable connector and screw it on tightly.
It is recommended that you pump up the endoscope before cleaning at a max. of 160 mmHg, releasing the pressure only when cleaning and/or disinfection is completed.

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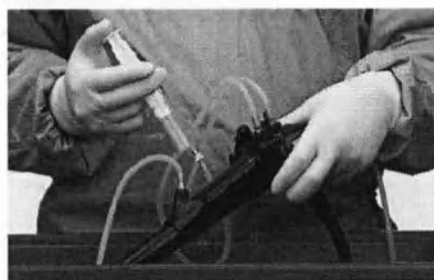
3. Wipe the housing, shaft, and tube carefully with lint-free cloth soaked in an enzyme cleaner (use a suitable cleaning agent or disinfectant in accordance with the list of approved chemicals, see page 39). Avoid shocks and pulling movements so as not to damage the shaft. Rinse the endoscope carefully with clean water.

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4. Rinse the endoscope. (see page 25). To do so, aspirate approx. 100 ccm cleaning solution through the suction channel. Then rinse with clean water.

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Caution: Only immerse instrument parts once they have successfully passed a leakage test.

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5. Introduce the brush supplied (Page 6) into the suction channel and push it forwards until it is visible in the forceps channel. Clean the brush head appearing at the distal end with another brush or compressed air before pulling back.

Caution: The brush is now in the forceps channel, it cannot be pushed further into the forceps channel!!

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6. Now thread the brush through the forceps channel inlet to the distal end of the shaft.

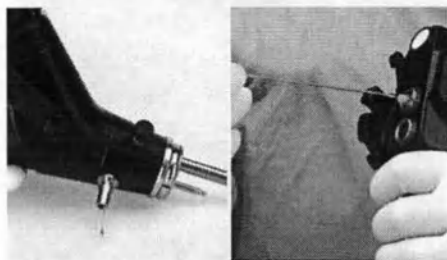
Caution: With regard to model 13800, the thinner brush 27651 G is designed for brushing the biopsy channel; the thicker brush 27651 H is designed exclusively for cleaning the supply segment.

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
7. Brush out the whole suction channel (shaft segment and supply segment) and the biopsy channel several times.

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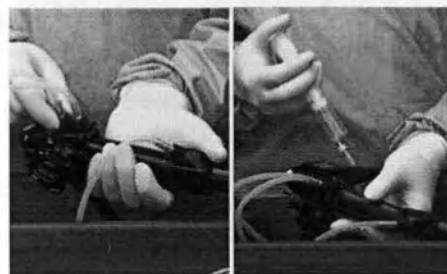8. Connect the Y-rinsing tube instead of the air/water valves, and pump cleaning fluid through the channels using a syringe. Finally, rinse with clean water.

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9. Connect the channel cleaning tube to the biopsy inlet. Immerse the free end of the cleaning tube and the shaft in the cleaning solution. Aspirate about 100-200 ml solution. Repeat the suction process with clean water to rinse.

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10. Place the whole instrument in cleaning solution. Carefully rub all external surfaces. Watch for regularly formed bubbles for a period of 5 minutes, since this is evidence of a leak. In this case, remove the instrument and contact KARL STORZ.

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11. Clean all particles of dirt from the distal end. It may be necessary to clean the air/water valve with a brush (27652) with soft bristles, in order to remove hardened contaminants. Never remove dirt with a needle or other sharp objects, because these can cause severe damage.

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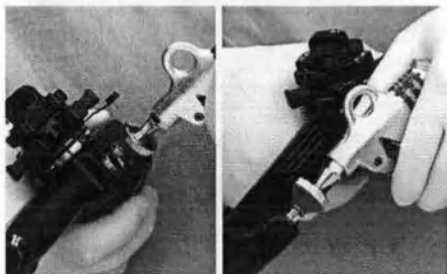
12. Remove the instrument from the solution and rinse with clear water (demineralized water).
13. Carefully clean all valves with the cleaning brush (27652) and then rinse.

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14. After the final step in the cleaning process has been completed, remove the entire endoscope from the solution. Thoroughly dry all other external surfaces of the endoscope, as well as the channels, preferably with compressed air at a max. of 0.5 bar/7.25 psi.

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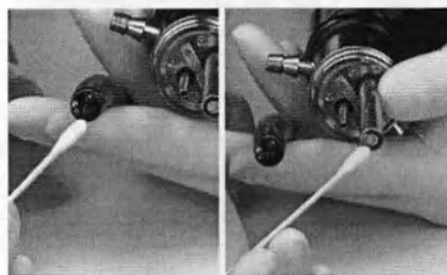
15. Clean the distal lens and the eyepiece with a cotton-wool bud soaked in lens cleaner or 70% isopropyl alcohol. During cleaning of the distal lens, always wipe away from the air/water channel so as not to introduce any particles into this opening.

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Caution: Do not autoclave the endoscope or its accessories (with the exception of accessories that are specifically designated as autoclavable, e.g. forceps etc.)

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**Cleaning/disinfection of the video
endoscope: by machine**

All KARL STORZ videoscopes can be connected to equipment for semi- or fully automatic cleaning. The manufacturer of the cleaning machine must be consulted when selecting the cleaning programs. Usually, special programs for fiberscopes are available which are suited for cleaning and material preservation. For machine cleaning, make certain that flexible and semiflexible telescopes as well as miniature endoscopes are not heated above 60 °C. Depending on the soiling, preliminary brushing of the instrument channel may be necessary.

Caution: Check the videoscope for leaks before any cleaning in semi or fully automatic equipment.

Lay the endoscope carefully in the machine basket. Connect the irrigation tubing to the channels. Proceed with the endoscope cleaning program according to the regulations of the machine manufacturer.

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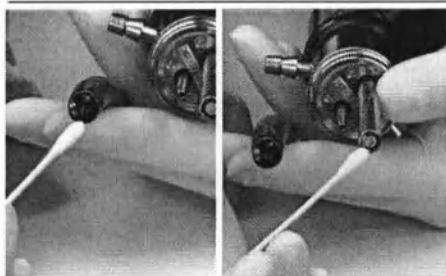
Caution: After the program run, the fiberscopes are greatly at risk of corrosion in the preparation machine.

Once the program is complete, preferably remove the fiberscopes from the machine immediately.

Then if necessary dry off the optical surfaces with a cotton swab soaked with 70% alcohol.

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**Recommended procedure for the
preparation of video endoscopes**

**Preliminary cleaning directly after the exami-
nation (see page 20)**

- The outer casing should be wiped with a lint-free damp cloth from top to bottom.
 - Aspirate sufficient amounts of cleaning solution alternately through the distal end and the cleaning extension.
 - Apply the air-water supply for a few seconds
 - Connect the cleaning adapter
 - Separate the flexible endoscope from the light source
- Caution:** Seal off the video plug with the sealing cap.

Cleaning in the preparation place

- Carry out leakage test.
- Immerse the flexible endoscope
- Clean the channels with special cleaning brushes
- Clean the complete air/water system thoroughly (in the case of duodenoscopes, the Albarran unit must also be disassembled, see page 14)
- Rinse the channels

**Preparation in the endoscope washing
machine (RDA)**

- The machine must fulfil all the disinfection and material compatibility requirements.
- Make certain that the environmental conditions with regard to temperature and pressure are not exceeded: +60 °C (tolerance +/- 5%) and 1 bar for the channels, 0.3 bar leak test.
- Recommendation:
Before placing the endoscope in the machine, clean all channels (instrument channel, water channel, Albarran channels, air/water channels) whether or not they have been used.
- It is not recommended to switch between manual and machine preparation.

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Cleaning accessories

1. Basic cleaning is required before disinfection or sterilisation. Clean biopsy forceps and brushes carefully (preferably with enzymatic cleaners) and using a soft brush. Remove any deposits, such as blood, excrement, and mucus from biopsy forceps and needles – ultrasound bath.
2. Rinse in clean water and dry (compressed air can be used).
3. Before sterilisation, carefully oil moveable parts. Use only suitable, moisture-resistant, physiologically safe, residue-free oil (e.g., 27656 B). After oiling, open and close the forceps several times.
4. Disinfect accessories and where appropriate sterilise with ETO gas.
5. Always test instruments after cleaning to exclude malfunction and any damage.

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Cleaning of water bottle

1. Empty
2. Carefully clean the container, the lid and the tube.
3. Rinse and dry
4. Disinfect and where appropriate sterilise with ETO gas.

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Inspection after cleaning

Before assembly all parts and surfaces must be checked for damage, complete cleaning and drying.

Replace defective parts or send in for repair. Before sending in for repair, all parts must be cleaned and sterilized/disinfected.

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If residue is still present after cleaning, prepare the parts again, subject them to special treatment or discard or repair.

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Check plastic parts for discoloration, porosity and flexibility. If the parts have changed substantially from their new condition, they must be replaced. This especially applies to seals and rubber caps. Test the instruments for proper operation: check the telescope for a clear view, the instrument for steerability and all canals for patency (see page 11).

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Disinfection/Sterilization

Warning: Chemical disinfection for patient application is only recommended for instruments which come in contact solely with mucous membranes or minor skin wounds or are inserted into natural, unsterilized orifices. In principle, sterilization is preferable from the point of view of hygiene.

Caution: All deviations from the recommended disinfection parameters must be validated by the user.

Caution: Before disinfection, all parts of the instrument must be thoroughly cleaned and rinsed as well as dried. All parts of KARL STORZ video endoscopes can be disinfected in solution. Only the materials approved by KARL STORZ are recommended.

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Gas sterilization

Sterilization using formaldehyde or ethylene oxide are gentle and therefore recommended methods of sterilization.

Note: Observe national laws and regulations for the gas sterilization method.

Caution: If the instrument is to be sterilized, the entire system must be opened by fitting the EtO cap on the pressure compensation part to ensure pressure compensation.

Warning: Before commencing gas sterilization, the instrument must be completely dry or else sterile conditions will not be achieved.

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For gas sterilization with ethylene oxide, the following airing times must be observed for instruments due to gas absorption of the materials:

	at 20°C	at 42°C	at 55°C
Telescopes	24 hrs	18 hrs	12 hrs
Rubber items	7 days	4 days	3 days
Video endoscopes/ Plastics	5 days	2 days	30 hrs

(minimum time in simple transparent packaging and an application time on the patient of more than 30 minutes).

Maximum acceptable levels (per 21 CFR 812.100; Federal Register, Vol. 43, No. 122, June 23 1978) of residues following EtO sterilization are as follows:

Ethylene oxide:	25 ppm* / 250 ppm**
Ethylene chlorohydrine:	25 ppm* / 250 ppm**
Ethylene glycol:	250 ppm* / 5000 ppm**

** for instruments in contact with skin and mucous membrane,
* for instruments in contact with blood.

Statements as to the efficiency of sterilization for the respective type of instrument (depending on the length and diameter of the lumina) may only be made by the manufacturer.

Caution: Attention must be paid to the information on cleaning, sterilization and storage in the "Cleaning, sterilization and care of KARL STORZ instruments" instructions. The methods for cleaning, disinfection and sterilization are explained there in detail.

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Storage

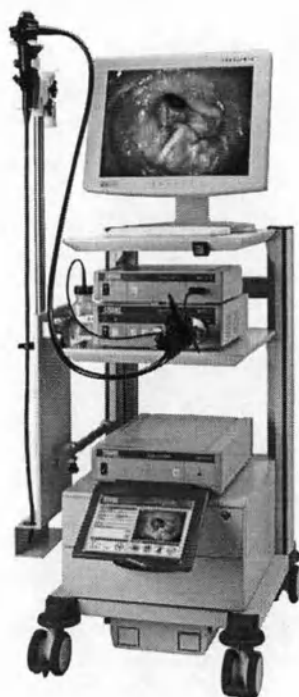
Caution: Do not expose video endoscopes to direct sunlight or excessive heat.

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Video endoscopes should always be packed and stored in sterile cloths or suitable containers acc. to EN 868 and must be absolutely dry before placing in storage.

The video endoscope is most suitably stored suspended vertically with the distal end pointing down (e.g. endoscope holder 29003 FT, see picture).

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Storage must take place in facilities with normal temperature and air humidity levels.

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Servicing and repair

Defective items of equipment must be serviced and repaired exclusively by persons authorized by us; all repair work must employ original parts.

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Limitation of liability

KARL STORZ GmbH & Co. KG shall be liable for failure or deterioration in the safe operation, operational reliability, and performance of this equipment only subject to the conditions that all assembly, operations, system expansions, readjustments, modifications, or repairs to same have been performed by KARL STORZ, and that the instrument has been used in accordance with its operating instructions at all times.

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Warranty

For two years as of delivery to the end customer, we agree to replace the goods free of charge if proof can be provided of faulty materials or faulty workmanship. In doing so we cannot accept the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply. Please fill out the attached warranty card on the last page and return it as soon as possible to:

KARL STORZ GmbH & Co. KG
Postfach 230
D-78503 Tuttlingen/Germany

Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall release us from any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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Directive compliance

This medical product bears the CE mark in accordance with the Medical Device Directive (MDD) 93/42/EEC. A code number after the CE mark indicates the responsible notified body.

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Important information

To avoid the spread of infectious illnesses among hospital staff or among KARL STORZ employees, telescopes, instruments and equipment must be cleaned and sterilized/disinfected before they are sent for repair. We reserve the right to return contaminated instruments/equipment to the sender.

Repairs, changes or expansions which are not carried out by KARL STORZ or by experts authorized by KARL STORZ will invalidate all warranty rights.

KARL STORZ gives no warranty on the correct functioning of equipment or instruments which have been repaired by unauthorized third parties.

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Type

- 1 = Cleaner
- 2 = Disinfectant
- 3 = Suitable for ultrasound baths

Note: You will also find the current list of approved products on our Web site (<http://www.karlstorz.de> or www.karlstorz.com).

Note: When agents other than those named are used, KARL STORZ assumes no liability for any damage which may occur.

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Chemicals for the preparation of video endoscopes
Manual cleaning and disinfection/ultrasound

When preparing and using the solutions, follow the solution manufacturer's instructions paying, close attention to proper dilution and recommended immersion times. Regarding the microbiological effectiveness please ask the manufacturer. The following cleaning and disinfectant solutions have been approved regarding material compatibility:

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Firmenstempel/Unterschrift:

WARRANTY

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship.

In doing so we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Complete warranty card upon purchase/delivery and return immediately to:

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To be completed by the supplier/importer:

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KARL STORZ GmbH & Co. KG
Postfach 230
78503 Tuttlingen/Германия

To be filled out by instrument owner:

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