State of Minnesota - Secretary of State

This Certificate is not valid for use anywhere within the United States of America, its territories or possessions.

Apostille

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1. Country (Pays): United States of America

This public document (Le présent acte public) Addendum to Instruction for Use for medical device

2. has been signed by (a été signé par) Irma Vargas

3. acting in the capacity of (agissant en qualité de) Notary Public, State of Minnesota

4. bears the seal / stamp of (est revêtu du sceau / timbre de) Irma Vargas, Notary Public, State of Minnesota

Certified Attesté

5. at (à) St. Paul, Minnesota

6. Date (Date) 10/29/2018

7. by (par) Secretary of State, State of Minnesota

8. File No (Sous n°) 1042070000195

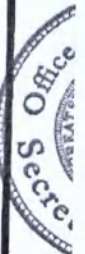
9. Seal / Stamp (Sceau / Timbre):

10. Signature (Signature):

Steve Simon

Steve Simon
Secretary of State
State of Minnesota

This Apostille only certifies the authenticity of the signature and the capacity of the person who has signed the public document, and, where appropriate, the identity of the seal or stamp which the public document bears.

This Apostille does not certify the content of the document for which it was issued.

To verify the issuance of this Apostille, see <https://apostille.sos.state.mn.us>.

This certificate does not constitute an Apostille under the Hague Convention of 5 October 1961, when it is presented in a country which is not a party to the convention. In such cases, the certificate should be presented to the consular section of the mission representing that country.

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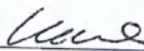
To whom it may concern:

In my capacity as International Regulatory Affairs Associate, 3M Health Care Business, St. Paul, MN, USA, I certify that the attached addendum to Instruction for Use (IFU) for "3M Tegaderm Foam High Performance Foam Non-Adhesive Dressing" in English is truthful and accurate.


Regina Feferman
International Regulatory Affairs Associate
3M Health Care


10/29/18
Date

State of Minnesota)
County of Ramsey) SS

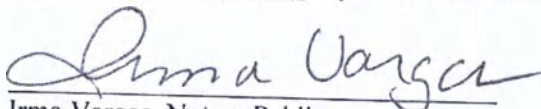Subscribed and sworn to before me this 29th day of October 2018.

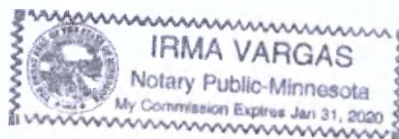Irma Vargas, Notary Public
My Commission Expires: January 31, 2020

3M Health Care Business

3M Center
2510 Conway Ave., Bldg. 275-SW-06
St. Paul, MN 55144 U.S.A.
651 733 1110

Addendum to Instruction for Use for medical device

***3M Tegaderm Foam High Performance
Foam Non-Adhesive Dressing***

3M Health Care Business

3M Center
2510 Conway Ave., Bldg. 275-5W-06
St. Paul, MN 55144 U.S.A.
651 733 1110

Section «Indications for Use» is to be replaced by the following:

Indications for Use

Tegaderm High Performance Foam Non-Adhesive Dressing is indicated for use as a primary or secondary dressing for low- to highly-exuding partial and full thickness wounds such as pressure ulcers, venous leg ulcers, neuropathic ulcers, arterial ulcers, skin tears, surgical wounds, and abrasions. The dressing is suitable for use with compression therapy. The dressing is not designed, sold or intended for use except as indicated. Tegaderm Foam Dressing is not intended for use in pressure reduction or as a surgical sponge.

3M Tegaderm High Performance Foam Non-Adhesive Dressing can be supplied in the following sizes:

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2510 Conway Ave., Bldg. 275-5W-06
St. Paul, MN 55144 U.S.A.
651 733 1110

To whom it may concern:

In my capacity as International Regulatory Affairs Associate, 3M Health Care Business, St. Paul, MN, USA, I certify that the attached addendum to Instruction for Use (IFU) for 3M Tegaderm High Performance Foam Non-Adhesive Dressing (catalog number 90605) in English is truthful and accurate.

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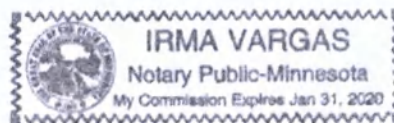
***3M Tegaderm Foam High Performance Foam
Non-Adhesive Dressing
(Catalog number 90605)***

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Indications for Use

Tegaderm High Performance Foam Non-Adhesive Dressing (catalog number 90605) is indicated for use as a primary or secondary dressing for low- to highly-exuding partial and full thickness wounds such as pressure ulcers, venous leg ulcers, neuropathic ulcers, arterial ulcers, skin tears, surgical wounds and abrasions. The dressing is suitable for use with compression therapy.

The dressing is not designed, sold or intended for use except as indicated. Tegaderm Foam Dressing is not intended for use in pressure reduction or as a surgical sponge.

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Apostille

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1. Country (Pays): **United States of America**

This public document (Le présent acte public) **Addendum to Instruction for Use for medical device**

2. has been signed by (a été signé par) **Irma Vargas**

3. acting in the capacity of (agissant en qualité de) **Notary Public, State of Minnesota**

4. bears the seal / stamp of (est revêtu du sceau / timbre de)
Irma Vargas, Notary Public, State of Minnesota

Certified Attesté

5. at (à) **St. Paul, Minnesota**

6. Date (Date) **10/29/2018**

7. by (par) **Secretary of State, State of Minnesota**

8. File No (Sous n°) **1042070000179**

9. Seal / Stamp (Sceau / Timbre):

10. Signature (Signature):

Steve Simon

Steve Simon

Secretary of State
State of Minnesota

This Apostille only certifies the authenticity of the signature and the capacity of the person who has signed the public document, and, where appropriate, the identity of the seal or stamp which the public document bears.
This Apostille does not certify the content of the document for which it was issued.

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3M

To whom it may concern:

In my capacity as International Regulatory Affairs Associate, 3M Health Care Business, St. Paul, MN, USA, I certify that the attached addendum to Instruction for Use (IFU) for "3M Tegaderm IV Transparent Film Dressing with Border" and "3M Tegaderm Transparent Film Dressing Frame Style" in English is truthful and accurate.

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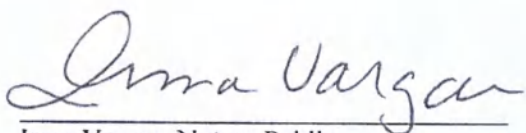
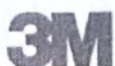
Section «Warnings» is to be replaced by the following:

Warnings:

Do not use the dressings as a replacement for sutures and other primary wound closure methods.

Using the dressings to cover a drug delivery patch could lead to faster absorption, increasing the risk for overdose.

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2510 Conway Ave., Bldg. 275-5W-06
St. Paul, MN 55144 U.S.A.
651 733 1110

To whom it may concern:

In my capacity as International Regulatory Affairs Associate, 3M Health Care Business, St. Paul, MN, USA, I certify that the attached addendum to Instruction for Use (IFU) for "3M Tegaderm Foam Adhesive Dressing" in English is truthful and accurate.

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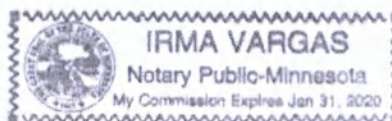
Indications for Use:

Tegaderm Foam Adhesive; High Performance Foam Adhesive Dressing is designed to allow easy, one-handed application to difficult body contours, including heels and elbows, without the need for cutting. The dressing is indicated or use as a primary dressing for low to highly exuding, partial and full thickness dermal wounds, including:

- pressure ulcers
- venous leg ulcers
- abrasions
- arterial ulcers
- skin tears
- neuropathic ulcers

Additional applications of Tegaderm High Performance Foam Adhesive Dressing include:

- use as a secondary (cover) dressing in conjunction with wound fillers (such as gauze or alginate dressings).
- use under compression wrap systems for venous leg ulcer treatment.

This product is not designed, sold or intended for use except as indicated. Tegaderm Foam Adhesive dressing (High Performance Foam Adhesive Dressing) is not intended for use in pressure reduction or as a surgical sponge.

3M Tegaderm High Performance Foam Adhesive Dressing can be supplied in the following sizes: 8,8cm x 8,8cm, 10cm x 11cm 14,3cm x 14,3cm, 14,3cm x 15,6cm, 6,9cm x 7,6cm, 19cm x 22,2cm, 13,9cm x 13,9cm.

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