INSTRUCTION

For Use

1. **PURPOSE:** Ophthalmic spectacle frames made of metal and combined produced by "Leonardo D. Exclusive Brillenmode GmbH", meant for holding the lenses in a correct fixed position over the eyes.

2. MAIN TECHNICAL CHARACTERISTICS AND CONSTRUCTION

Ophthalmic spectacle frames, depending on design features and manufacturing technology, are divided into types indicated in Table 1

Table 1.

Designation of frame type	Name of frame type	Type and designation of temples
FM	Metal optical frames	Metal rigid PR; Metal elastic PE.
FC	Combined optical frames	Metal rigid MR; Metal elastic PE; Plastic rigid PR; Plastic elastic PE

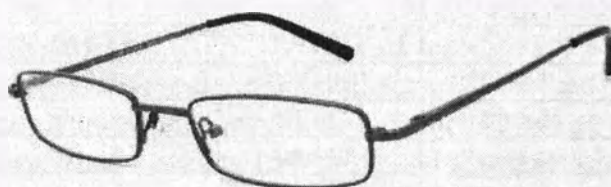
Fig. 1. Metal ophthalmic spectacle frames (FM)

Fig. 2. Combined ophthalmic spectacle frames (FC)

Ophthalmic spectacle frames consist of a font, temples, nose pads, hinges. Hinges may be simple and spring-based.

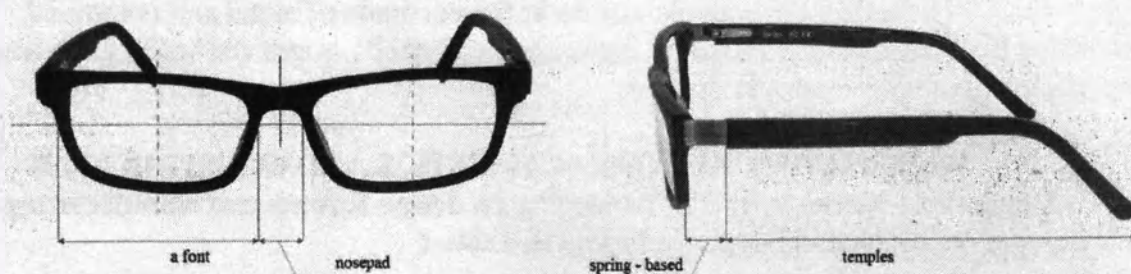
Fig. 3. Structure ophthalmic spectacle frames

Ophthalmic spectacle frames are characterized by 3 main geometric parameters: width (a), bridge width (b) and temple length to the curve (L).

The sizes used for spectacle frames by "Leonardo D. Exclusive Brillenmode GmbH" are given in the table 2:

Ophthalmic spectacle frames		Distance between the vertical tangents to the facet groove of the rim (a). (mm)	Minimum distance between the vertical tangents to the bottom of the facet groove of the rim (b). (mm)	Temple length (L) (mm)	Temple length from the hinge axis to the curve (L.) (mm)	Temple mass without lenses (g)
Ophthalmic spectacle frames (metal)	men	$53 \pm 0,2$	$17 \pm 0,2$	$155 \pm 1,0$ HET	$115 \pm 1,0$	$22 \pm 1,0$
	men	$56 \pm 0,2$	$20 \pm 0,2$	$150 \pm 1,0$ 140	$110 \pm 1,0$	$22 \pm 1,0$
	men	$58 \pm 0,2$	$22 \pm 0,2$	$130 \pm 1,0$ 140	$90 \pm 1,0$	$22 \pm 1,0$
	women	$54 \pm 0,2$	$17 \pm 0,2$	$135 \pm 1,0$ 140	$95 \pm 1,0$	$20 \pm 1,0$
	women	$53 \pm 0,2$	$19 \pm 0,2$	$135 \pm 1,0$ 140	$95 \pm 1,0$	$20 \pm 1,0$
	women	$56 \pm 0,2$	$20 \pm 0,2$	$145 \pm 1,0$ 140	$105 \pm 1,0$	$20 \pm 1,0$
	unisex	$55 \pm 0,2$	$18 \pm 0,2$	$135 \pm 1,0$ 135	$95 \pm 1,0$	$20 \pm 1,0$
	unisex	$56 \pm 0,2$	$20 \pm 0,2$	$150 \pm 1,0$ 135	$110 \pm 1,0$	$22 \pm 1,0$
	unisex	$58 \pm 0,2$	$22 \pm 0,2$	$130 \pm 1,0$ 135	$90 \pm 1,0$	$22 \pm 1,0$
Ophthalmic spectacle frames (combined)	men	$54 \pm 0,2$	$22 \pm 0,2$	$140 \pm 1,0$ HET	$100 \pm 1,0$	$17 \pm 1,0$
	men	$57 \pm 0,2$	$18 \pm 0,2$	$140 \pm 1,0$ 140	$100 \pm 1,0$	$20 \pm 1,0$
	men	$53 \pm 0,2$	$17 \pm 0,2$	$145 \pm 1,0$ 140	$105 \pm 1,0$	$20 \pm 1,0$
	women	$48 \pm 0,2$	$22 \pm 0,2$	$140 \pm 1,0$ 135	$100 \pm 1,0$	$16 \pm 1,0$
	women	$53 \pm 0,2$	$18 \pm 0,2$	$130 \pm 1,0$ 135	$90 \pm 1,0$	$18 \pm 1,0$
	women	$54 \pm 0,2$	$18 \pm 0,2$	$145 \pm 1,0$ 135	$105 \pm 1,0$	$20 \pm 1,0$
	unisex	$50 \pm 0,2$	$22 \pm 0,2$	$140 \pm 1,0$ 140	$100 \pm 1,0$	$17 \pm 1,0$
	unisex	$52 \pm 0,2$	$18 \pm 0,2$	$145 \pm 1,0$ 140	$105 \pm 1,0$	$18 \pm 1,0$
	unisex	$56 \pm 0,2$	$18 \pm 0,2$	$145 \pm 1,0$ 140	$105 \pm 1,0$	$20 \pm 1,0$

On the surface of the frames there may be no sharp edges or burrs, shells, scratches, bubbles, sagging solder visible to the naked eye.

Temples should rotate on the axis smoothly, without jamming.

Decorative trim (if any) is from nickel silver (N18 CUNI18ZN20).

Ophthalmic spectacle frames correspond to the European norm (standards):

EN ISO 12870, EN ISO 7998, EN ISO 8596, EN ISO 8624, EN ISO 9456, EN ISO 11380, EN ISO 11381, EN ISO 11715

Ophthalmic spectacle frames are chosen according to the anthropometric parameters of the patient. It's necessary to consult a medical worker to decide whether to apply this method of vision correction.

3. INDICATIONS for application

Correction of defects of the optics of eyes:

- myopia
- presbyopia
- astigmatismus
- hypermetropia (small type of clinical refraction)
- anisometropia (difference of refraction on eyes)
- keratoconus and others.

4. The following may be considered CONTRAINDICATIONS:

- individual intolerance to sight correction by means of spectacles;
- impossibility of sight correction by means of spectacles.

5. MATERIALS:

Frames are manufactured from materials which can't cause irritation, allergic or toxic reactions in the process of exploitation on contact with human skin.

Metal frames are produced of nickel silver (N18 CUNI18ZN20) or clean titanium (number CAS 7740-32-6, number EINECS 241-036-9), hinges and screws are produced of nickel silver (N18 CUNI18ZN20). Tips of temples are produced of cellulose acetate (cellulose acetate polymer (69-75%) and diethyl phthalate (25-31%).

Combined frames are produced of nickel silver (N18 CUNI18ZN20) and cellulose acetate (cellulose acetate polymer (69-75%) and diethyl phthalate (25-31%), hinges and screws are produced of nickel silver (N18 CUNI18ZN20). Tips of temples are produced of cellulose acetate (cellulose acetate polymer (69-75%) and diethyl phthalate (25-31%). Nose pads are produced of silicone LSR50.

The transport insert made of acrylic Polycryl Clear.

Materials comply with the requirements of biological safety.

6. MARKING:

The right temple indicates the manufacture's name, conformity mark.

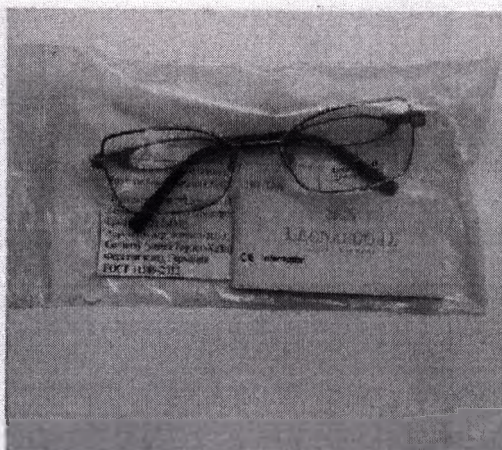
Figure 6. Individual plastic packaging.

Plastic packaging of each frame or the label placed into plastic packaging should contain:

- nominal value of basic sizes;
- model number;
- model colour;
- name of product;
- manufacturer;
- date of manufacturing;
- storage life;
- conformity mark,
- goverment standard.

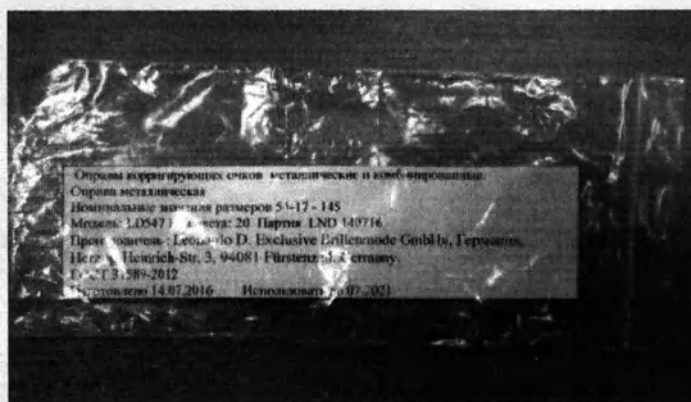
Figure 7. Label attached to plastic packaging

Frames in plastic packaging should be compactly placed into group packaging box. Group packaging box should contain frames of the same type with the same basic sizes, in quantity of 8 - 10 pcs.

Figure 8. Group packaging box

Each group packaging box should have a sticker of the manufacturer on it, which should contain following inscriptions:

- model number;
- model colour;
- nominal value of basic sizes;
- quantity of frames

label
tain:

Dimensions of the packaging box, in cm (length x width x height): 23 x 15 x 5,8.

Maximum load on the group packaging box is 10 kg.

Transport packaging is made only by agreement with the consumer.

Frames packed in transport packaging should be resistant to transport shaking with a maximum acceleration of 30 m/s² and frequency of 2 to 3 Hz.

Marking of transport packaging with application of the handling mark "Caution, fragile".

Set of delivery: transport inserts, frame.

7. PREPARING THE ITEM FOR USE: The spectacle frame should not lose its original shape after fitting the lenses. The rims should be symmetrical. No dents, scratches or gaps between spectacle lenses and rims, nor any other assembling defects are allowed.

Spectacle lenses fixed in the frame should not move or fall out of the rims during normal spectacle use.

8. TERMS OF STORAGE AND USE:

It is not recommended to leave the rim in direct sunlight at high temperatures (for example, car control panel);

- Need to avoid the fall of the rim on hard surfaces, and mechanical loads that can damage the rim;

- Storage frames need to use a case or pouch, is not permitted the wearing of eyeglasses in the pockets of clothing or bags in order to avoid scratches on the cover and mechanical damage;

- To put on and take off glasses with both hands is necessary - this prevents their deformation;

- Need to avoid wearing eyeglasses during sleep;

- Are not allowed to apply to the rim significant mechanical forces;

- Caution should be exercised when doing sports and the production of works involving sudden movements of the head, the slopes, jumps and bumps.

- Do not apply significant mechanical efforts to the frame;

- Keep the frame in a case or any other package to protect it from mechanical damage;

- Full average term of use is no less than 3 years..

- Terms of storage: in heated storages under the environmental temperature from + 5°C to + 50°C, relative air moisture 80%, atmospheric pressure 700-1060 hPa

- No impact of aggressive substances (acids, alkali) and organic solvents and their fumes is allowed during storage.

9. TERMS OF USE

Ophtalmic spectacle frames are used in all climate zones at any time of the day.

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Steuernummer:
153/131/30069
Ust-ID Number:
DE 811596823

10. PRECAUTIONS FOR USE

While using ophthalmic spectacle frames it is necessary to exercise caution practising sports and activities involving sudden head movements, bends, jumps and bumps.

11. POSSIBLE SIDE EFFECTS

Allergic reaction in case of individual intolerance of frames material is possible.

12. INFORMATION ON THE POSSIBLE IMPACT OF THE USE ON ABILITY TO DRIVE VEHICLES, MACHINERY

Drivers with vision problems may drive vehicles and machinery using spectacle frames (in the manufacture of frames for corrective glasses).

13. STERILITY

Unsterile.

14. MAINTENANCE AND REPAIRS

Ophthalmic spectacle frames should be repaired in special laboratories licensed for production, maintenance and repairs.

15. WARRANTY OBLIGATIONS

Storage life: combined frames – 5 years, metal frames – 10 years.

Full storage life – not less than 3 years with temple bends not more than 30 000.

16. TRANSPORTATION

Ophthalmic spectacle frames are transported by all kinds of transport in covered vehicles in accordance with the rules of transportation of goods applicable to this kind of transport.

Temperature from - 45°C to + 50°C

Relative air moisture up to 98%

Atmospheric pressure 700-1060 hPa

17. DISPOSAL

Ophthalmic spectacle frames do not contain harmful substances and components that pose a threat to human health and the environment during and after the end of life and disposal. Ophthalmic spectacle frames should be disposed of as household waste.

Wastes of production and parts with expired year of storage are classified as class A medical waste.

18. CERTIFICATES

The manufacturer uses its own quality system that is subject to verification and updating on an annual basis.

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Legal address: 111024, Moscow, Andronovskoe highway, 26, build. 6, off. 11B
Phone number 8 (495) 660-01-25

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