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CERTIFICATE

中国国际贸易促进委员会暨中国国际商会
China Council for the Promotion of International Trade is China Chamber of International Commerce

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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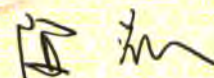
THIS IS TO CERTIFY THAT: the seal of Shanghai HBM Healthcares, Inc. on the annexed DOCUMENT is genuine.

China Council for the Promotion
of International Trade

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Shanghai HBM Healthcare, Inc.

No. 902-904, No.255, New Golden Bridge,
Shanghai, China 201206

Tel: 021-5135 2699 Fax: 021-5135 2600

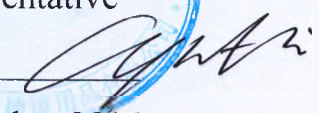
Email: gaochen@HBMChina.com

www.HBMChina.com

APPROVED by

Shanghai HBM Healthcare, Inc.

Legal Representative

Cynthia Gao 

Date: 12 October 2016

DISPOSABLE STERILE DIALYSIS KITS MEDISPO-M

(including Starting Kit and Ending Kit)

INSTRUCTION OF USE

Introduction

This Instruction of Use has been prepared on 12 October 2016 for the purpose of registration of the medical device in the Russian Federation. The scope of information included in the extract is in compliance with the Rules of Registration of Medical Devices (RF Government Regulation No. 1416 dtd. 27 December 2012).

Table of Contents

1 The name of the medical product	4
2 Details of Product Designer:	6
3 Details of Product Manufacturer and Manufacturing Site	7
4 Intended Use of Product established by Manufacturer:	7
5 Conditions of use	7
6 List of indications for use, contraindications, possible side effects	7
7 Description and Specifications	8
8 Directions of Use	19
9 Description of the working principle.....	19
10 Routine Maintenance Information.....	20
11 Scope of supply	20
12 Packaging.....	23
13 Marking and Labelling	24
14 Operating Conditions.....	27
15 Transport Information.....	27
16 Storage Information.....	27
17 Manufacturer's Warranty	27
18 Product Disposal Procedure.	28
19 List of International Standards which the product meets	28
20 Contacts	28

1 The name of the medical product

The name of the medical Product - Disposable Sterile Dialysis Kits Medispo-M (including Starting Kit and Ending Kit) in the following models:

1 Kit No.1

Starting Kit No.1

- Latex exam gloves, powder-free, size S - 1 pair;
- Absorbing sheet, 40×60 cm – 1 pc.;
- Gauze swabs (balls), dia. 30–40 mm – 5 pcs.;
- Gauze layered swabs (sheet), 3.5×7.0 cm – 5 pcs.;
- Band-Aid plaster, 2×12 cm – 6 pcs.;
- Disposable sterile injection syringe, 10 ml, with injection needle 0.8×40 mm – 2 pcs.;
- Disposable sterile injection syringe, 20 ml, with injection needle 0.8×40 mm – 1 pc.

Ending Kit No.1

- Latex exam gloves, powder-free, size S - 1 pair;
- Gauze swabs (balls), dia. 30–40 mm – 5 pcs.;
- Gauze layered swabs (sheet), 3.5×7.0 cm – 5 pcs.;
- Band-Aid plaster, 2×12 cm – 6 pcs.;
- Elastic bandage, 8.0 cm × 2.0 m – 2 pcs.

2 Kit No.2

Starting Kit No.2

- Latex exam gloves, powder-free, size S - 1 pair;
- Absorbing sheet, 40×60 cm – 1 pc.;
- Gauze swabs (balls), dia. 30–40 mm – 5 pcs.;
- Gauze layered swabs (sheet), 3.5×7.0 cm – 5 pcs.;
- Band-Aid plaster, 2×12 cm – 6 pcs.;

- Disposable sterile injection syringe, 20 ml, with injection needle 0.8×40 mm – 1 pc.

Ending Kit No.2

- Latex exam gloves, powder-free, size S - 1 pair;
- Gauze swabs (balls), dia. 30–40 mm – 5 pcs.;
- Gauze layered swabs (sheet), 3.5×7.0 cm – 5 pcs.;
- Elastic bandage, 8.0 cm × 2.0 m – 2 pcs.

3 Kit No.3

Starting Kit No.3

- Latex exam gloves, powder-free, size S - 1 pair;
- Absorbing sheet, 40×60 cm – 1 pc.;
- Gauze swabs (balls), dia. 30–40 mm – 5 pcs.;
- Gauze layered swabs (sheet), 3.5×7.0 cm – 5 pcs.;
- Band-Aid plaster, 2×12 cm – 6 pcs.;
- Disposable sterile injection syringe, 20 ml, with injection needle 0.8×40 mm – 1 pc.

Ending Kit No.3

- Latex exam gloves, powder-free, size S - 1 pair;
- Gauze swabs (balls), dia. 30–40 mm – 5 pcs.;
- Gauze layered swabs (sheet), 3.5×7.0 cm – 5 pcs.;
- Band-Aid plaster, 2×12 cm – 6 pcs.;
- Elastic bandage, 8.0 cm × 2.0 m – 2 pcs.

4 Kit No.4:

Starting Kit No.4

- Absorbing sheet, 40×60 cm – 1 pc.;
- Gauze swabs (balls), dia. 30–40 mm – 5 pcs.;
- Gauze layered swabs (sheet), 3.5×7.0 cm – 5 pcs.;
- Band-Aid plaster, 2×12 cm – 6 pcs.;

- Disposable sterile injection syringe, 20 ml, with injection needle 0.8×40 mm – 1 pc.

Ending Kit No.4

- Gauze swabs (balls), dia. 30–40 mm – 5 pcs.;
- Gauze layered swabs (sheet), 3.5×7.0 cm – 5 pcs.;
- Elastic bandage, 8.0 cm × 2.0 m – 2 pcs.

5 Kit No.5

Starting Kit No.5

- Absorbing sheet, 40×60 cm – 1 pc.;
- Gauze swabs (balls), dia. 30–40 mm – 5 pcs.;
- Gauze layered swabs (sheet), 3.5×7.0 cm – 5 pcs.;
- Band-Aid plaster, 2×12 cm – 6 pcs.

Ending Kit No.5

- Gauze swabs (balls), dia. 30–40 mm – 5 pcs.;
- Gauze layered swabs (sheet), 3.5×7.0 cm – 5 pcs.;
- Elastic bandage, 8.0 cm × 2.0 m – 2 pcs.

6 Kit No.6

Starting Kit No.6

- Absorbing sheet, 40×60 cm – 1 pc.;
- Gauze swabs (balls), dia. 30–40 mm – 5 pcs.;
- Gauze layered swabs (sheet), 3.5×7.0 cm – 5 pcs.;
- Band-Aid plaster, 2×12 cm – 6 pcs.

Ending Kit No.6

- Gauze swabs (balls), dia. 30–40 mm – 5 pcs.;
- Elastic bandage, 8.0 cm × 2.0 m – 2 pcs.

2 Details of Product Designer:

HBM Group, Inc.,

HMB Building, 98-02 218 St., Queens Village, New York 11429, USA,

Tel.: 1-718-776 6666, Fax: 1-718-776 4666,

E-mail: hbmusa@medispo.us

3 Details of Product Manufacturer and Manufacturing Site

Shanghai HBM Healthcares, Inc.,

902-904, No. 255, New Golden Bridge Road, Shanghai 201206, China,

Tel.: 021-5135 2699, Fax: 021-5135 2600,

E-mail: gaochen@HBMChina.com, www.HBMChina.com

4 Intended Use of Product established by Manufacturer:

Disposable Sterile Dialysis Kits (including Starting Kit and Ending Kit) (hereinafter - Kits) are available in a number of models intended for the use by trained medical personnel during the procedure of dialysis in specialized dialysis centers or other medical institutions.

5 Conditions of use

The products can be used in specialized dialysis centers and other health care institutions.

6 List of indications for use, contraindications, possible side effects

6.1 Indications for use

The Kits are intended for use by trained medical personnel during the procedure of dialysis in specialized dialysis centers or other medical institutions.

6.2 Contraindications, possible side effects

Individual intolerance or allergic reactions to natural latex, which is made of a set of gloves examination latex powder free.

7 Description and Specifications

7.1 Content

Kits are used during the treatment of dialysis – a method of extrarenal blood purification method. All products included in the kit are disposable. Kits are sterile. The Kits allow a comprehensive approach to the dialysis treatment, thereby easing the work of medical personnel.

The main reason for the development and production of kits, saving your time of qualified medical personnel to gather all items necessary for the procedure of dialysis. All the necessary products are packaged together in one sterile sealed tray. No need to package each item separately (except for the individual packaging of the syringe). Therefore, less packaging material is needed and less waste generated.

Table 1 below shows the content of different models including Starting Kit and Ending Kit.

Table 1 - Content of Kits, different models

Content	Q-ty, pcs.	Starting Kit						Q-ty, pcs.	Ending Kit					
		1	2	3	4	5	6		1	2	3	4	5	6
Latex exam gloves, powder-free, size S	1 pair	+	+	+	/	/	/	1 pair	+	+	+	/	/	/
Absorbing sheet, 40 × 60 cm	1	+	+	+	+	+	+	—	/	/	/	/	/	/
Gauze swabs (balls), dia. 30–40 mm	5	+	+	+	+	+	+	5	+	+	+	+	+	+
Gauze layered swabs (sheet), 3.5 × 7.0 cm	5	+	+	+	+	+	+	5	+	+	+	+	+	/
Band-Air plaster 2 × 12 cm	6	+	+	+	+	+	+	6	+	/	+	/	/	/
Disposable sterile injection syringe, 10 ml, with injection needle 0.8 × 40 mm	2	+	/	/	/	/	/	—	/	/	/	/	/	/

Content	Q-ty, pcs.	Starting Kit						Q-ty, pcs.	Ending Kit					
		1	2	3	4	5	6		1	2	3	4	5	6
Disposable sterile injection syringe, 20 ml, with injection needle 0.8 × 40 mm	1	+	+	+	+			—						
Elastic bandage, 8.0 cm × 2.0 m	—							2	+	+	+	+	+	+

7.2 Description and Functions

Products (Fig. 1) offer complete support for skilled health care professionals (doctor or nurse) while providing a hemodialysis treatment to a patient.

Latex exam gloves, powder-free, size S (gloves) protect the skin of health care professionals (doctor/ nurse) from the contact with the patient (Fig. 2).

Absorbing sheet, 40×60 cm (absorbent towel) serves to absorb the excess of fluids or blood (Fig. 3).

Gauze swabs (balls), dia. 30–40 mm can be used to remove the excess of fluids at the fistula needle puncture site (Fig. 4).

Gauze swabs (sheet), 3.5×7.0 cm (gauze pad) can be used to remove the injection site and absorb the blood after injection.

Band-Aid plaster, 2×12 cm secures the fistula needle in place.

Disposable sterile injection syringe, 10 ml, with injection needle 0.8×40 mm and disposable sterile injection syringe, 20 ml, with injection needle 0.8×40 mm (syringes) are used for injections.

Elastic bandage, 8.0 cm × 2.0 m covers the fistula need puncture spot and/or injection site after the treatment is complete.

Fig. 1 – Disposable Sterile Dialysis Kits MEDISPO-M in the Manufacturer's container

Fig. 2 - Latex exam gloves, powder-free, size S

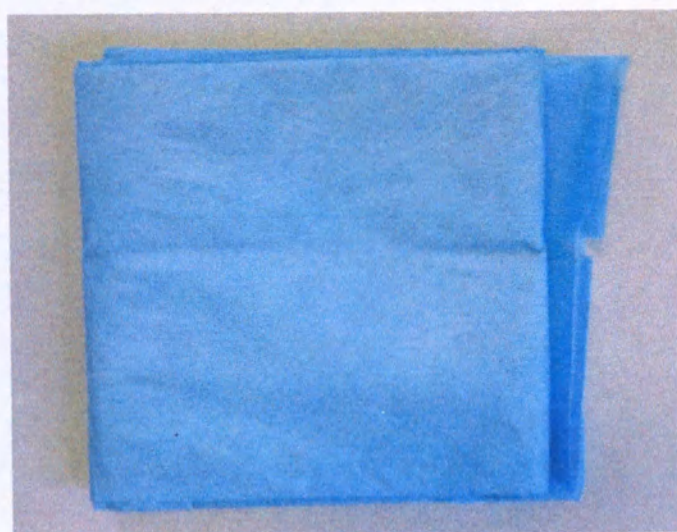
Fig. 3 – Adsorbing sheet, 40x60 cm

Fig. 4 - Gauze swabs (balls), dia. 30–40 mm

Fig. 5 – Gauze layered swabs (sheet), 3.5×7.0 cm

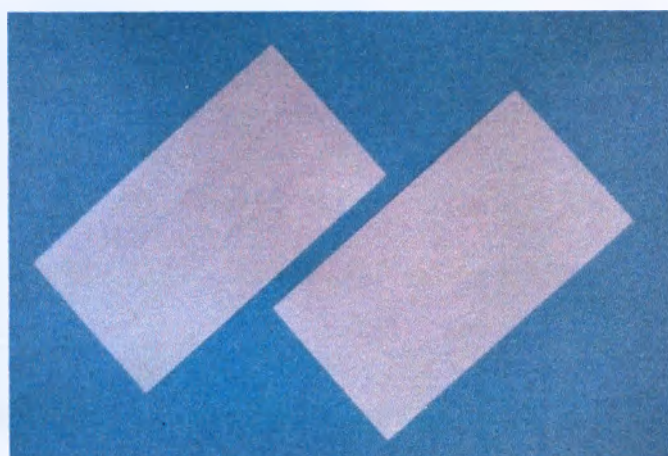
Fig. 6 – Band-Air plaster, 2×12 cm

*Fig. 7 – Disposable sterile injection syringe, 10 ml, with injection needle 0.8×40 mm,
Disposable sterile injection syringe, 20 ml, with injection needle 0.8×40 mm*

Fig. 8 – Elastic bandage, 8.0 cm × 2.0 m

7.3 Salient features

Overall dimensions:

- Starting kit – 271 × 217 × 25 mm, tolerance ± 5mm;
- Ending Kit – 231 × 217 × 25 mm, tolerance ± 5mm;

Weight:

- Starting Kit:
 - Kit No.1 – (92 ± 5) g;
 - Kit No.2 – (74 ± 5) g;
 - Kit No.3 – (74 ± 5) g;

- Kit No.4 – (62 ± 5) g;
- Kit No.5 – (48 ± 5) g;
- Kit No.6 – (48 ± 5) g;
- Ending Kit:
 - Kit No.1 – (58 ± 5) g;
 - Kit No.2 – (58 ± 5) g;
 - Kit No.3 – (58 ± 5) g;
 - Kit No.4 – (42 ± 5) g;
 - Kit No.5 – (42 ± 5) g;
 - Kit No.6 – (42 ± 5) g;

The description and specifications of the product components are shown in Table 2.

Table 2 - Description and specifications of the product components

Sl. No.	Description	Salient features	Material	Fig. No.
1	Latex exam gloves, powder-free, size S	Seamless five-finger gloves Smooth surface, powder-free Rolled cuff. Size S Dimensions: – palm centre width – (80 ± 10) mm; – length - min. 220 mm. Thickness - min. 0.08 mm. Thickness (exact palm centre) - max. 0.22 mm. Fluid-tight gloves. Mechanical properties: - Tensile strength before aging - min. 7.0 N; - Elongation before aging - min. 650%;	Natural Rubber Latex 60% D.R.C. H.A., total solids - min. 61.5%, dry rubber content - min. 60%, ammonia content (for total weight) 0.70±0.05%, VFA - max. 0.05, specific weight at 25°C - min. 0.94 Brand: Huangchunfa, manufactured by Chinalight Rubber (Thailand) Co. Ltd. /Thailand	2

S. No.	Description	Salient features	Material	Fig. No.
		- Tensile strength after aging - min. 6.0 N; - Elongation after aging - min. 500%;		
2	Absorbing sheet, 40×60 cm	Size (length × width) - 40 × 60 cm, tolerance ± 2 cm. Density – (110 ± 10) g/m². Absorption – min. 200 g/m². The rate of absorption – not more than 10 sec. Strength characteristics: – tensile strength in dry and in wet state in the longitudinal and transverse direction - not less than 20 N	Non-woven – spunbond, 100% polypropylene, density (20°C) ~ 0.9 g/cm³, white (bleached, CI 77891). Brand: Jin Gang, manufactured by Jiangyin Jin Gang Non-woven Cloth Co., Ltd. /China + PE (polyethylene). Brand: Shengfeng, blue colorant CI 74160. Manufactured by Shanghai Shengfeng Industrial Fabric Co., Ltd. /China +Wood-pulp paper, hydrogen peroxide bleached. Brand: Henglin, manufactured by Shandong Henglin Meilian /China	3
3	Gauze swabs (balls), dia. 30–40 mm	Made of bleached gauze Dia. - 30 - 40 mm. Size when unfolded (length × width) - 25×25 cm, tolerance ± 5 cm. Gauze breaking strength: – warp - min. 78.0 N; – weft - min. 34.0 N;	100% cotton. Manufactured by Xinjiang Production &Construction Corps /China	4

Sl. No.	Description	Salient features	Material	Fig. No.
		Gauze surface weight - min 36 g/m², max. tolerance - minus 5%. Thread count per 10 cm - within: - warp - minus 2%; - weft- minus 3%; Plus-tolerances of the surface density are not limited. Physical and chemical properties: – reaction of aqueous extract - neutral; – wettability - max. 10 s; – capillary capacity - min. 10 cm/h; – moisture - (5.0 – 8.5)%; – ash content - max. 0.3%; – whiteness -max. 80%; – whiteness stability -max. 10%		
4	Gauze swabss (sheet), 3.5×7.0 cm	Made of bleached gauze Size when folded (length × width) - 3.5×7.0 cm, tolerance ± 0.2 cm. Size when unfolded (length × width) - 7.0×28.0cm, tolerance ± 0.2 cm. Gauze breaking strength: – warp - min. 78.0 N; – weft - min. 34.0 N; Gauze surface weight - min 36 g/m², max. tolerance - minus 5%. Thread count per 10 cm - within: - warp - minus 2%;	100% cotton. Manufactured by Xinjiang Production &Construction Corps /China	5

Sl. No.	Description	Salient features	Material	Fig. No.
		- weft- minus 3%; Plus-tolerances of the surface density are not limited. Physical and chemical properties: - reaction of aqueous extract - neutral; - wettability - max. 10 s; - capillary capacity - min. 10 cm/h; - moisture - (5.0 – 8.5)%; - ash content - max. 0.3%; - whiteness - max. 80%; - whiteness stability - max. 10%		
5	Band-Aid plaster, 2×12 cm	Size (width × length) - 2 × 12 cm, tolerance ± 0.2 cm. W/o strong odour. No visible defects. Adhesive layer - white or clear. Anti-adhesion coat - yes. Adhesive layer peel resistance – (25 - 200) N/m.	PE (polyethylene). Brand: Shengfeng, white - semi-opaque micro perforated polyethylene foil, manufactured by Shanghai Shengfeng Industrial Fabric Co., Ltd. / China (adhesive tape base) + Cellucotton, anti-adhesive. Brand: Xinfeng, manufactured by Wenzhou Xinfeng Composite Materials Co., Ltd. /China (protective coat) + Polyacrylate glue, non-sensitizing (ethyl-α-cyanoacrylate 80%, butyl acrylate 8.5% and sulfolane methacrylate 11.5%). Brand: Zhejiang,	6

Sl. No.	Description	Salient features	Material	Fig. No.
			manufactured by Zhejiang Satellite Petro Chemical Co. Ltd./China (adhesive layer)	
6	Syringe Disposable sterile injection syringe, 10 ml, with injection needle 0.8×40 mm	Syringe is in conformity with the requirements of ISO ISO 7886-1:1993 Sterile hypodermic syringes for single use – Part 1: Syringes for manual use. Capacity - 10 ml. Luer lock syringe with 6% taper. Hypodermic needle is in conformity with the requirements of ISO 7864:1993 Sterile hypodermic needles for single use. Hypodermic needle size - 0.8×40 mm (21G × 1 ½"). Each syringe along with a hypodermic needle is placed in an individual pouch bag.	Polypropylene (PP). Brand: R3260T, colourless (barrel, rod, plunger, rod stop, tip), manufactured by Hongji Petrochemical Co., Ltd./ China + Rubber (gasket). Brand: JL, manufactured by Changzhou Jinlin /China + Polydimethylsiloxane CAS:9006-65-9 (lubricant), Brand: WH, manufactured by Changzhou Jinlin Medical Device Co., Ltd. /China + Black dye CI 77266 (scale) + Stainless Steel, grade 1Cr18Ni9Ti (needle tube), manufactured by Zhejiang Oujian Medical Apparatus Co.,Ltd. /China + Green dye 2,5G4/8 (needle hub), manufactured by Shanghai Yunzhu Chemical Co., Ltd./China + UF glue, HT3311 - cyanoacrylate glue, colourless, density at 25°C 1.08 g/cm3, manufactured by Yantai Hightite Chemical Co., Ltd./China Syringes along with hypodermic needles	7
7	Disposable sterile injection syringe, 20 ml, with injection needle 0.8×40 mm	Syringe is in conformity with the requirements of ISO ISO 7886-1:1993 Sterile hypodermic syringes for single use – Part 1: Syringes for manual use. Luer lock syringe with 6% taper. Capacity - 20 ml. Hypodermic needle is in conformity with the requirements of ISO 7864:1993 Sterile hypodermic needles for single use. Hypodermic needle size - 0.8×40 mm (21G × 1 ½"). Each syringe along with a hypodermic		7

Sl. No.	Description	Salient features	Material	Fig. No.
		needle is placed in an individual pouch bag.	are placed in individual pouch bags: thermally bonded composite packages: Soft PP film, colourless. Brand: Zshine + Cellucotton, brand: Zshine, manufactured by Hangzhou Zshine Medical packing co., LTD./China +Dyes (label) Black - Pantone Black C	
8	Elastic bandage, 8.0 cm × 2.0 m	Size (width × length) - (8.0 ± 0.2) cm × (2.0 ± 0.1) m. The stretched length of the bandage is given. Surface weight – (75 ± 5) g/m ² . Elongation – not less than 50 %. Breaking strength – min. 59.0N. Breaking elongation – no less than extensibility value.	Stretch cotton, content: – Chengfeng cotton 75% + Qianxi spandex 25% , manufactured by Zhangjiagang Henlongxiang Co., ltd./China.	8

The products are non-toxic, sterile, pyrogen-free.

Each syringe is placed along with a needle in a individual package. The packaging is sealed.

Each product is placed in a sterile tray, which is a thermally bonded composite package: Hard PP film, colourless. Brand: Zshine +

+ Cellucotton, brand: Zshine, manufactured by Hangzhou Zshine Medical packing co., LTD./China + Dyes (label):

Red - Pantone Red 032C, Green - Pantone Green C. The tray is sealed.

7.4 The kits are single-use sterile products.

The sterilization method is ethylene oxide.

8 Directions of Use

8.1 General Provisions

Before use, read the Instructions for Use.

The product is intended for use by skilled health care professionals (doctor and/or nurse).

8.2 Safety Requirements

The product is for single use only. Do not re-use the product.

Do not use if the sterile package (tray) has been damaged.

Do not use after the expiry date.

Open the sterile package (tray) directly before use.

While using the product, strictly adhere to the aseptic regulations applicable in health care institutions.

8.3 Indications, Contraindications, Possible Side Effects

8.3.1 Indications

The product is intended for the use by skilled health care professionals in hemodiafiltration treatment in specialized dialysis centres and other health care institutions.

8.3.2 Contraindications, possible side effects

Intolerance or allergic reaction to natural latex that is used to make powder-free exam gloves included in the kit.

9 Description of the working principle

9.1 Open the sealed tray according to the sign.

9.2 Put in place of Absorbing sheet.

9.3 Wear gloves.

9.4 Secure the patch introduced a fistula needle.

9.5 Start dialysis.

9.6 An injection of a drug with a syringe, if it is assigned to the patient.

9.7 After completion of the dialysis procedure to close the fistula area of injection needles and/or the injection site with a bandage.

9.8 Use gauze swabs (balls), gauze swabs (sheet) for the absorption or removal of all unnecessary liquids, processing areas of injection.

9.9 Collect the products mentioned above and disposed of.

10 Routine Maintenance Information

No routine maintenance is required. The product is non-maintainable

11 Scope of supply

11.1 Contents of delivery:

- Product placed in a tray - 25 pcs.;
- Instruction for use - 1 pc.;
- Carton - 1 pc.

11.2 The contents of each kit placed in a tray is in accordance with the product model, see Table 3.

Table 3 - Content of Kits, different models

Description	Q-ty, pcs.
1. Disposable Sterile Dialysis Kit MEDISPO-M - Kit No.1	1
1.1 Starting Kit No.1	1
– Latex exam gloves, powder-free, size S	1 pair
– Adsorbing sheet, 49×60 cm	1
– Gauze swabss (balls), dia. 30–40 mm	5
– Gauze layered swabs (sheet), 3.5×7.0 cm	5
– Band-Air plaster, 2×12 cm	6
– Disposable sterile injection syringe, 10 ml, with injection needle 0.8×40 mm	2
– Disposable sterile injection syringe, 20 ml, with injection needle 0.8×40 mm	1

Description	Q-ty, pcs.
1.2 Ending Kit No. 1	1
– Latex exam gloves, powder-free, size S	1 pair
– Gauze swabs (balls), dia. 30–40 mm	5
– Gauze layered swabs (sheet), 3.5×7.0 cm	5
– Band-Air plaster, 2×12 cm	6
– Elastic bandage, 8.0 cm × 2.0 m	2
2. Disposable Sterile Dialysis Kit MEDISPO-M - Kit No.2	1
2.1 Starting Kit No.2	1
– Latex exam gloves, powder-free, size S	1 pair
– Adsorbing, 40×60 cm	1
– Gauze swabs (balls), dia. 30–40 mm	5
– Gauze layered swabs (sheet), 3.5×7.0 cm	5
– Band-Air plaster, 2×12 cm	6
– Disposable sterile injection syringe, 20 ml, with injection needle 0.8×40 mm	1
2.2 Ending Kit No.2	1
– Latex exam gloves, powder-free, size S	1 pair
– Gauze swabs (balls), dia. 30–40 mm	5
– Gauze layered swabs (sheet), 3.5×7.0 cm	5
– Elastic bandage, 8.0 cm × 2.0 m	2
3. Disposable Sterile Dialysis Kit MEDISPO-M - Kit No.3	1
3.1 Starting Kit No.3	1
– Latex exam gloves, powder-free, size S	1 pair
– Adsorbing sheet, 40×60 cm	1

Description	Q-ty, pcs.
– Gauze swabs (balls), dia. 30–40 mm	5
– Gauze layered swabs (sheet), 3.5×7.0 cm	5
– Band-Air plaster , 2×12 cm	6
– Disposable sterile injection syringe, 20 ml, with needle injection 0.8×40 mm	1
3.2 Ending Kit 3	1
– Latex exam gloves, powder-free, size S	1 pair
– Gauze swabs (balls), dia. 30–40 mm	5
– Gauze layered swabs (sheet), 3.5×7.0 cm	5
– Band-Air plaster, 2×12 cm	6
– Elastic bandage, 8.0 cm × 2.0 m	2
4. Disposable Sterile Dialysis Kit MEDISPO-M – Kit No. 4	1
4.1 Starting Kit No.4	1
– Adsorbing sheet, 40×60 cm	1
– Gauze swabs (balls), dia. 30–40 mm	5
– Gauze layered swabs (sheet), 3.5×7.0 cm	5
– Band-Air plaster, 2×12 cm	6
– Disposable sterile injection syringe, 20 ml, with injection needle 0.8×40 mm	1
4.2 Ending Kit No.4	1
– Gauze swabs (balls), dia. 30–40 mm	5
– Gauze layered swabs (sheet), 3.5×7.0 cm	5
– Elastic bandage, 8.0 cm × 2.0 m	2
5. Disposable Sterile Dialysis Kit MEDISPO-M - Kit No.5	1
5.1 Starting Kit No.5	1

Description	Q-ty, pcs.
– Adsorbing sheet, 40×60 cm	1
– Gauze swabs (balls), dia. 30–40 mm	5
– Gauze layered swabs (sheet), 3.5×7.0 cm	5
– Band-Air plaster, 2×12 cm	6
5.2 Ending Kit No.5	1
– Gauze swabs (balls), dia. 30–40 mm	5
– Gauze layered swabs (sheet), 3.5×7.0 cm	5
– Elastic bandage, 8.0 cm × 2.0 m	2
6. Disposable Sterile Dialysis Kit MEDISPO-M - Kit No.6	1
6.1 Starting Kit No.6	1
– Adsorbing sheet, 40×60 cm	1
– Gauze swabs (balls), dia. 30–40 mm	5
– Gauze layered swabs (sheet), 3.5×7.0 cm	5
– Band-Air plaster, 2×12 cm	6
6.2 Ending Kit No.6	1
– Gauze swabs (balls), dia. 30–40 mm	5
– Elastic bandage, 8.0 cm × 2.0 m	2

12 Packaging

Each syringe along with a needle is placed in a individual package, which is is a thermally bonded composite package made of soft PP film and medical crepe paper.

The product is placed in a tray, which is a thermally bonded composite package made of hard PP film and a medical crepe paper tray cover. The tray is sealed.

25 pcs. trays are placed in a corrugated fireboard box (carton). Instructions for use are also included in the carton.

13 Marking and Labelling

13.1 Marking and Labelling of Retail Packages

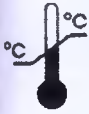
Each paper tray cover is labelled. The starting kits are labelled green, the

Ending Kits are provided with red labels.

Each tray is labelled with the following information:

- designer's name, trade mark, and address;
- manufacturer's name and address;
- distributor's name, logo and address;
- product description;
- content of product;
- indication "NON-TOXIC, PYROGEN-FREE";
- indication "The gloves are made of natural rubber latex that may cause allergic reaction";
- reference No. (symbol applied as appropriate);
- batch code (symbol applied as appropriate);
- date of manufacture (symbol applied as appropriate);
- indication "use by..." (symbol applied as appropriate);
- indication "consult instructions for use" (symbol applied as appropriate);
- indication "Do not use if the package has been damaged" (symbol applied as appropriate);
- indication that the product is for single-use (symbol applied as appropriate);
- indication "Sterilization using ethylene oxide" (symbol applied as appropriate);
- storage temperature range (symbol applied as appropriate);
- disposal (symbol applied as appropriate);
- conformity mark(symbol applied as appropriate);
- CE marking; (symbol applied as appropriate);
- registration certificate No.
- the indication "open here".

Symbols:

	– Manufacturer
	– Reference No.
	– Batch code
	– Date of manufacture
	– Use by ...
	– Consult Instructions for use
	– Do not use if the package has been damaged
	– Do not re-use
	– Ethylene oxide sterilization
	– Storage temperature range
	– Special disposal
	– Conformity mark
	– CE marking
	– Open here

13.2 Marking and Labelling of Shipping Packages

Every carton is labelled with the following information:

- designer's name, trade mark, and address;
- manufacturer's name and address;
- distributor's name, logo and address;
- product description;
- content of product;
- indication "NON-TOXIC, PYROGEN-FREE";
- indication "The gloves are made of natural rubber latex that may cause allergic reaction";
- reference no. (symbol applied as appropriate);
- batch code (symbol applied as appropriate);
- date of manufacture (symbol applied as appropriate);
- indication "use by..." (symbol applied as appropriate);
- indication "Consult Instructions for Use" (symbol applied as appropriate);
- indication "do not use if the package has been damaged" (symbol applied as appropriate);
- No. of trays per carton (symbol applied as appropriate);
- indication that the product is for single-use (symbol applied as appropriate);
- indication "sterilization using ethylene oxide" (symbol applied as appropriate);
- storage temperature range (symbol applied as appropriate);
- transport temperature range (symbol applied as appropriate);
- conformity mark (symbol applied as appropriate);
- CE marking (symbol applied as appropriate);
- registration certificate No.

14 Operating Conditions

Directly before use or in case of transportation at freezing temperatures, leave the product (without opening and/ or in transportation package) at room temperature at least for 24 hours at +25 °C or higher. Do not use if the sterile package (pouch bag) has been damaged.

Temperature: $(+10 - +25) \pm 2^\circ\text{C}$

Humidity: from 45 to 80 %.

Atmospheric pressure: $(84.0 - 106.7)$ kPa [(630-800) mm Hg.].

15 Transport Information

The packaged products can be transported by any closed means of transportation in accordance with the appropriate shipping rules.

Transport conditions are as follows: at temperature $(-30 - +30) \pm 2^\circ\text{C}$.

16 Storage Information

Throughout the shelf life, keep the products in original package in dry ventilated rooms away from sunlight and atmospheric precipitation at temperature $(+5 - +25) \pm 2^\circ\text{C}$ and humidity max. 70% (from 45 to 70 %), 1 m away from heating surfaces.

Keep the products away from sunlight and atmospheric precipitation, do not expose to oil, gasoline, kerosene, or other similar substances.

17 Manufacturer's Warranty

17.1 The Manufacturer warrants that the products are in compliance with the requirements of the standards and Manufacturer's documentation provided that the prescribed transport, storage, and operation conditions are observed by the Customer.

17.2 The expiry date is 3 years from the date of manufacture (sterilization).

17.3 The Manufacturer shall not be held liable for any loss or damage caused during the use of the products until their fault is established.

17.4 The Manufacturer shall not be held liable for any loss or damage caused by the re-use of the products.

17.5 The Manufacturer shall not be held liable for any loss or damage caused after the expiry date marked on the package.

18 Product Disposal Procedure.

Products, which had contact with blood and/or body fluids are to be discarded and/or destroyed in accordance with SanPiN 2.1.7.2790-2010 class B.

Unused product (not having contact with blood and/or bodily fluids), including those with expired shelf life, must be recycled and/or disposed of according SanPiN 2.1.7.2790-2010 in class A.

19 List of International Standards which the product meets

The medical device "Disposable Sterile Hemodialysis Kits (including Starting Kit and Ending Kit) MEDISPO-M, different models" are in conformity with international standards for products: EN 455-1:2000, EN 455-2:2000, EN 455-3:2006, EN 11737-1:2006, EN 11737-2:2009, EN 11737-3:2004, EN 13726-1:2002, EN 13726-2:2002, EN 13726-3:2002, EN 13726-4:2002, EN 13726-6:2002, EN 13795-1:2002, EN 14079:2003, EN 20594-1:1995 (ISO 594-1:1986), EN ISO 11607-1:2006, EN ISO 14971:2000, EN ISO 15223-1:2012, ISO 11135-1:2007, ISO 11138-2:2006, ISO 11607-1:2006, ISO 14155-1:2003.

20 Contacts

For any questions related to the quality of the medical device "Disposable Sterile Hemodialysis Kits (including Starting Kit and Ending Kit) MEDISPO-M of various models" manufactured by Shanghai HBM Healthcare, Inc., China, please contact:

An authorized representative of the manufacturer on the territory of the Russian Federation - Limited Liability Company Holding UNONA (LLC Holding UNONA)

28 Kirova / 43 Toledova St., lit. 3, office 3, the city of Yekaterinburg,
Sverdlovsk Region, 620028, Russia

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