continuous innovation for pathology

APPROVED BY
Medical Device Manufacturer
Sakura Finetek Europe B.V.

The undersigned authority legalizes the
signature of the signing proxy(ies).

THE NETHERLANDS CHAMBER OF COMMERCE AND INDUSTRY

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Accu-Edge® Microtome Blades

APOSTILLE

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- 1 Country: THE NETHERLANDS
This public document
- 2 has been signed by R.D.M. van den Dool
- 3 acting in the capacity of official of the Chamber of
Commerce and Industry
- 4 bears the seal/stamp of aforesaid Chamber of
Commerce and Industry

Certified

- 5 in Den Haag
- 6 on 30-01-2018
- 7 by the registrar of the district court of Den Haag
- 8 no. 2018-1094

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Accu-Edge® Microtome Blades

Design options:

1. Accu-Edge® Microtome Blades S-35 "Routine", 50 pcs.
2. Accu-Edge® Microtome Blades R-35 "Ribbon Section", 50 pcs.
3. Accu-Edge® Microtome Blades A-22 "Paraform", 50 pcs.
4. Accu-Edge® Microtome Blades A-35 "Extreme thin Section", 50 pcs.
5. Accu-Edge® Microtome Blades N-35 "Hard Tissue", 50 pcs.
6. Accu-Edge® Microtome Blades N-35HR "Extra Hard Tissue", 50 pcs.
7. Accu-Edge® Microtome Blades S-22 "Thin Section", 50 pcs.
8. Accu-Edge® Microtome Blades HIGH-PROFILE, 50 pcs.
9. Accu-Edge® Microtome Blades C-35 "Cryo", 20 pcs.

2. Indications.

Not applicable to this medical device.

3. Contraindications and possible side-effects.

Not applicable to this medical device.

4. Intended Use

Laboratory microtome blades are intended for sectioning paraffin-embedded or frozen tissue for making pathologic specimens and subsequent microscopy.

5. Product Description and Principle of Operation

5.1. Operation

Accu-Edge® Microtome Blades S-35 "Routine"

For thin sectioning of paraffin-embedded tissue specimens. It has earned a high reputation for superior cutting performance and durability.

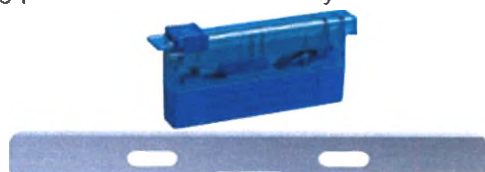
Figure 1 – Accu-Edge Microtome Blades S-35 "Routine"

Accu-Edge® Microtome Blades R-35 “Ribbon Section”

For thin sectioning of paraffin-embedded tissue specimens. Especially suited for routine microtomy operations. Double spattering increases edge strength and durability even when sectioning right after trimming with the same blade position.

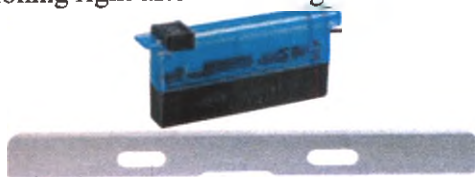
Figure 2 – Accu-Edge® Microtome Blades R-35 “Ribbon Section”

Accu-Edge® Microtome Blades A-22 “Paraform”

For extra thin sectioning of paraffin-embedded tissue specimens. Especially suited for fibered tissues/ biopsy.

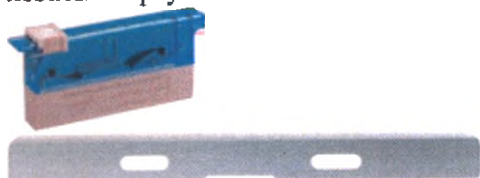
Figure 3 – Accu-Edge® Microtome Blades A-22 “Paraform”

Accu-Edge® Microtome Blades A-35 “Extreme thin Section”

For thin sectioning of paraffin-embedded tissue specimens. Especially suited for hard tissue block sectioning and ribbon sectioning. Durable and stable life after trimming.

Figure 4 – Accu-Edge® Microtome Blades A-35 “Extreme thin Section”

Accu-Edge® Microtome Blades N-35 “Hard Tissue”

For thin sectioning of paraffin-embedded tissue specimens. Long blade life, no need to move the blade in real sectioning after trimming.

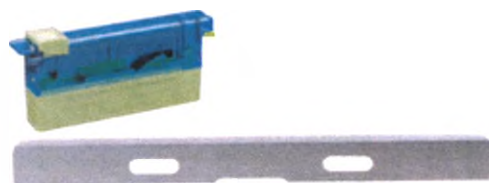
Figure 5 – Accu-Edge® Microtome Blades N-35 “Hard Tissue”

Accu-Edge® Microtome Blades N-35HR “Extra Hard Tissue”

For thin sectioning of paraffin-embedded tissue specimens. Long blade life, no need to move the blade in real sectioning after trimming.

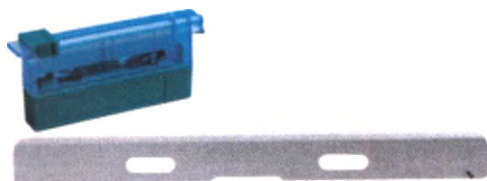
Figure 6 – Accu-Edge® Microtome Blades N-35HR “Extra Hard Tissue”

Accu-Edge® Microtome Blades S-22 “Thin Section”

For extra thin sectioning of paraffin-embedded tissue specimens. Especially recommended for laboratory use. Under some circumstances, blade life may be limited because of the ultra-thin edge.

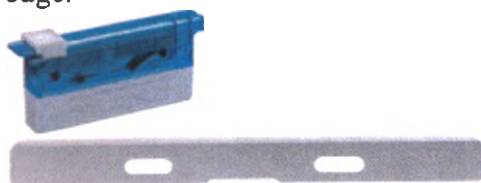
Figure 7 – Accu-Edge® Microtome Blades S-22 “Thin Section”

Accu-Edge® Microtome Blades HIGH-PROFILE

For thin sectioning of paraffin-embedded tissue specimens. Especially suited for ribbon sectioning with rotary microtome.

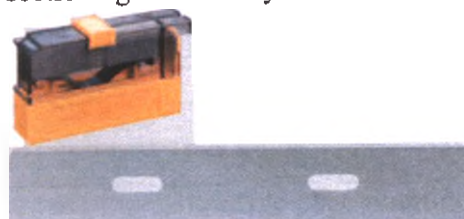
Figure 8 – Accu-Edge® Microtome Blades HIGH-PROFILE (LARGE)

Accu-Edge® Microtome Blades C-35 “Cryo”

For thin cryo-sectioning. Specially ground for cryostat use to eliminate curling or wrinkling problems.

Figure 9 – Accu-Edge® Microtome Blades C-35 “Cryo”

5.2 Product Specification

1. Accu-Edge® Microtome Blades S-35 “Routine”

Blades in a pack	50 pcs.
Material	Stainless Steel
Thickness x Width x Length	0.254 x 8 x 80 mm
Edge angle	35°
Blade hardness	52 HRC
Edge width	2 µm
Allowable edge blunting	0.05

2. Accu-Edge® Microtome Blades R-35 “Ribbon Section”

Blades in a pack	50 pcs.
Material	Stainless Steel
Thickness x Width x Length	0.254 x 8 x 80 mm
Edge angle	35°
Blade hardness	52 HRC
Edge width	2 µm
Allowable edge blunting	0.05

3. Accu-Edge® Microtome Blades A-22 "Paraform"

Blades in a pack	50 pcs.
Material	Stainless Steel
Thickness x Width x Length	0.254 x 8 x 80 mm
Edge angle	22°
Blade hardness	52 HRC
Edge width	2 µm
Allowable edge blunting	0.05

4. Accu-Edge® Microtome Blades A-35 "Extreme thin Section"

Blades in a pack	50 pcs.
Material	Stainless Steel
Thickness x Width x Length	0.254 x 8 x 80 mm
Edge angle	35°
Blade hardness	52 HRC
Edge width	2 µm
Allowable edge blunting	0.05

5. Accu-Edge® Microtome Blades N-35 "Hard Tissue"

Blades in a pack	50 pcs.
Material	Stainless Steel
Thickness x Width x Length	0.254 x 8 x 80 mm
Edge angle	35°
Blade hardness	52 HRC
Edge width	2 µm
Allowable edge blunting	0.05

6. Accu-Edge® Microtome Blades N-35HR "Extra Hard Tissue"

Blades in a pack	50 pcs.
Material	Stainless Steel
Thickness x Width x Length	0.254 x 8 x 80 mm
Edge angle	35°
Blade hardness	52 HRC
Edge width	2 µm
Allowable edge blunting	0.05

7. Accu-Edge® Microtome Blades S-22 "Thin Section"

Blades in a pack	50 pcs.
Material	Stainless Steel
Thickness x Width x Length	0.254 x 8 x 80 mm
Edge angle	22°
Blade hardness	52 HRC
Edge width	2 µm
Allowable edge blunting	0.05

8. Accu-Edge® Microtome Blades HIGH-PROFILE

Blades in a pack	50 pcs.
Material	Stainless Steel
Thickness x Width x Length	0.31 x 14 x 75.7 mm
Edge angle	35°

Blade hardness	52 HRC
Edge width	2 μm
Allowable edge blunting	0.05

9. Accu-Edge® Microtome Blades C-35 “Cryo”

Blades in a pack	20 pcs.
Material	Carbon Steel
Thickness x Width x Length	0.254 x 8 x 80 mm
Edge angle	35°
Blade hardness	52 HRC
Edge width	2 μm
Allowable edge blunting	0.05

One blade can be used to make from 20 to 40 sections of tissue paraffin block before its disposal.

This device is disposable, which means that under the natural loss of functionality during operation, the blade cannot be re-sharpened but must be disposed according to accepted norms and regulations.

The blade can be used to section different blocks including blocks containing biomaterial of different patients. When going to section a new block it is recommended to clean the blade of paraffin residues with special brushes. Treat the blade with disinfectant solutions if necessary.

6. Analytical characteristics

Accu-Edge® Microtome Blades

Laboratory microtome blades intended for sectioning paraffin-embedded or frozen tissue for making pathologic specimens and subsequent microscopy.

Accu-Edge® Microtome Blades are suitable for microtomes of all kinds if equipped with a disposable microtome blade holder, compatible with the microtome.

7. Precautions

When using Accu-Edge® Microtome Blades follow the rules of safe operation and personal hygiene.

This product is disposable. To avoid contamination it shall be disposed after use as a medical waste.

Disposal is performed according to the norms, rules and regulations of the country where this medical device is used.

8. Preparation for Use

Accu-Edge® Microtome Blades are non-sterile disposable medical devices. For routing operations sterility of this expendable component is not required.

Accu-Edge® Microtome Blades are medical devices that require no special preparation.

9. Analysis

Accu-Edge® Microtome Blades

Microtomy:

WARNING! Before you start make sure that all moving parts of microtome are locked with safety stoppers and their spontaneous movement is impossible.

1. Remove plastic dispenser out of the paper box.
2. Move carriage forward.
3. Remove the blade appearing in the slit at the top of dispenser and then, depending on the design of microtome/cryotome (sliding or rotary), and in strict accordance with operating instructions fix the blade at the working part of the microtome (holder, frame, etc., depending on the construction).
4. Return the carriage to initial position.
5. After use remove the blade out of the holder and put it into the waste tray in the bottom of the dispenser for disposal.

10. Operating and Storage Environment

Protect Accu-Edge® Microtome Blades from rough handling or mechanical shock.

Average service life - 2 years.

Store in a dry, well-ventilated area, avoiding direct sunlight.

Recommended shelf life – 5 years.

Storage temperature: from -10 °C to +50 °C.

11. Environmental Requirement. Disposal Conditions.

Accu-Edge® Microtome Blades are clean and environmental-friendly.

To avoid contamination Accu-Edge® Microtome Blades shall be disposed after use as a medical waste.

Disposal is performed according to the norms, rules and regulations of the country where this medical device is used.

12. Packaging

The blades are packed in a plastic dispenser that provides ease of use and protects against injury.

The dispenser is a 110 x 45 x 23 mm (97 x 55 x 26 mm) plastic container with fittings for easy storage and safe removal of the blades.

The medical devices are shipped in paper boxes.

Design options	Transport packaging	Weight of a single medical device, g	Weight of the packaging, g
Accu-Edge® Microtome Blades S-35 "Routine"	Paper box Dimensions: Plastic dispenser Dimensions: Blades in pack: 50 pcs	1.1 +/- 0.1	97.6+/-0.5
Accu-Edge® Microtome Blades R-35 "Ribbon Section"	Paper box Dimensions: Plastic dispenser Dimensions: Blades in pack: 50 pcs	1.1 +/- 0.1	97.6+/-0.5
Accu-Edge® Microtome Blades A-22 "Paraform"	Paper box Dimensions: Plastic dispenser Dimensions: Blades in pack: 50 pcs	1.1 +/- 0.1	97.6+/-0.5
Accu-Edge® Microtome Blades A-35 "Extreme thin Section"	Paper box Dimensions: Plastic dispenser Dimensions: Blades in pack: 50 pcs	1.1 +/- 0.1	97,6+/-0.5
Accu-Edge® Microtome Blades N-35 "Hard Tissue"	Paper box Dimensions: Plastic dispenser Dimensions: Blades in pack: 50 pcs	1,1 +/- 0.1	97,6+/-0,5
Accu-Edge® Microtome Blades N-35HR "Extra Hard Tissue"	Paper box Dimensions: Plastic dispenser Dimensions: Blades in pack: 50 pcs	1.1 +/- 0.1	97,6+/-0,5
Accu-Edge® Microtome Blades S-22 "Thin"	Paper box Dimensions:	1.1 +/- 0.1	97.6+/-0.5

Section"	Plastic dispenser Dimensions: Blades in pack: 50 pcs		
Accu-Edge® Microtome Blades HIGH-PROFILE	Paper box Dimensions: Plastic dispenser Dimensions: Blades in pack: 50 pcs	2.2 +/- 0.2	172 +/- 5
Accu-Edge® Microtome Blades C-35 «Cryo»	Paper box Dimensions: Plastic dispenser Dimensions: Blades in pack: 20 pcs	1.1 +/- 0.1	64+/-3

13. Transportation

The blades are packed in a plastic dispenser that provides ease of use and protects against injury.

The dispenser is a 110 x 45 x 23 mm (97 x 55 x 26 mm) plastic container with fittings for easy storage and safe removal of the blades.

The medical devices are shipped in paper boxes.

The shipping package contains 10 packs of blades.

Shipping in all kinds of enclosed transport does not require special temperature regime.

14. Product Labeling

Product label contains the following information: reference number, name, packaging, lot number (LOT), information signs *, product information (number of blades in the package), production date, storage conditions, information about manufacturer and supplier and contacts in case of complaints.

DO NOT REUSE

MANUFACTURER

In-vitro diagnostic medical device

Certifies that the device complies with essential requirements of EC directives and harmonized standards, and that this product has passed directive compliance verification.

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14. International Declarations and Certificates

Accu-Edge® Microtome Blades have the following certificates:

EN ISO 13485:2012/AC:2012

EN ISO 14971:2012

EN 980:2008

EC 98/79/EC

Repair and Maintenance Requirements

Accu-Edge® Microtome Blades are maintenance free and cannot be repaired. In case of failure medical device shall be disposed of.

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FEATHER SAFETY RAZOR CO., LTD. MINO SITE 600-1, Matsumori, Mino-City, Gifu, 501-3753 JAPAN