Department of State

## APOSTILLE

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1. Country: United States of America

This public document

2. has been signed by GABRIEL COLON

3. acting in the capacity of Notary Public of Florida

4. bears the seal/stamp of Notary Public, State of Florida

Certified

5. at Tallahassee, Florida

6. the Sixteenth day of November, A.D., 2017

7. by Secretary of State, State of Florida

8. No. 2017-125918

9. Seal/Stamp:

10. Signature:

*Ken Detzner*

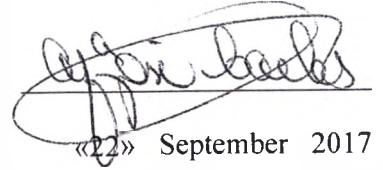
Secretary of State

DSDE 99 (2/12)

This document contains a true watermark. Hold up to light to see "SAFE" and "VERIFY FIRST."


«Certify»  
Manager Regulatory Affairs  
Maria Jose Carles

  
«22» September 2017

State of Florida  
County of Dade

Sworn to and subscribed before me  
This 13 Day of September  
By Maria Jose Carles

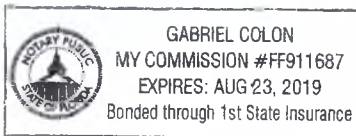Type of Identification presented  
Business License  
Gabriel Colon

## Remineralizing gels “Sherbet”

### Instructions for use

## **1. The name of the medical device**

Remineralizing gels «Sherbet» in performance

## **2. Information about the developer, manufacturer and place of manufacture of the medical device**

### **Developer**

Dharma Research, Inc.

5220 N.W. 72nd Avenue, Bay 15, Miami, Florida, 33166 USA

### **Manufacturer**

Dharma Research Inc.

5220 N.W. 72nd Avenue, Bay 15, Miami, Florida, 33166 USA

### **Place of production**

Dharma Research Inc.

5220 N.W. 72nd Avenue, Bay 15, Miami, Florida, 33166 USA

## **3. Determination, composition and basic principles of the functioning of a medical device**

### **Determination**

The Remineralizing gels "Sherbet" are used in dentistry in order to prevent tooth decay and to reduce the increased dental sensitivity by remineralizing process.

### **Composition and basic principles of the functioning of the medical device**

**The Remineralizing gel Sherbet APF Thixotropic Gel** - is an oral gel containing 1.23% fluoride ions, with different flavors: Chewing Bubble, Cherry Pie, Strawberry Heaven, Chunky Mango, Mint Mousse, French Vanilla, French Vanilla Dye-Free.

The Remineralizing gels Sherbet APF Thixotropic Gel; the flavors of Chewing Bubble and Strawberry Heaven are supplied in 250 ml and 500 ml bottles.

The Remineralizing gels Sherbet APF Thixotropic Gel; the flavor of Cherry Pie are supplied in 250 ml bottles; 500 ml; 3.8 liters.

The Remineralizing gels Sherbet APF Thixotropic Gel the flavors of Mint Mousse, Chunky Mango and French Vanilla, Dye-Free are supplied in 500 ml bottles.

**The Remineralizing gels Sherbet Neutral Thixotropic Gel** - is an oral gel containing 2% fluoride ions, with different flavors: Mint Mousse, Strawberry Heaven and French Vanilla Dye- Free).

The Remineralizing gels "Sherbet" Neutral Thixotropic Gel the flavors of Mint Mousse and French Vanilla Dye-Free are supplied in 500 ml bottles.

The Remineralizing gels "Sherbet" Neutral Thixotropic Gel; the flavor of Strawberry Heaven are supplied in bottles with a volume of 250 ml; 500 ml.

**The Remineralizing gels Sherbet Neutral Fluoride Gel 1.1%** - is an oral gel containing 1.1% fluoride ions, with different flavors: Mint Mousse, Chewing Bubble. Supplied in a tube of 120 g.

The composition of the Remineralizing gels Sherbet has the property of thixotropy. This means that it becomes more fluid when the tray is put on the teeth under pressure, and then again takes the form of the heavy gel as soon as the tray is fixed. The main function of thixotropy is to prevent material from flowing into the patient's larynx.

The main active component of the Remineralizing Sherbet gels are fluoride ions contained in hydrofluoric acid, which remineralize the enamel of the teeth.

### **Indications**

Prophylaxis of dental caries, hypersensitivity of teeth.

### **Contraindications**

Revealed hypersensitivity to fluorides.

Do not use in patients with ulcerative necrotic gingivitis and stomatitis.

The materials contain dyes. Make sure that the patient is not allergic to this compound.

When the material is used, patients should not use other fluoride medications on the same day. The routine use of fluoride tablets by patients should be discontinued within a few days after application.

For use by children under 6 years old - consult a doctor.

### **Possible side effects**

It is necessary to limit the use of fluoride-containing drugs in areas with high fluorine content in water (the critical concentration of fluoride for children is more than 1.5 mg / L, for adults - 6 mg / L), as this leads to the accumulation of fluoride in the body. Constant exposure to fluorides can cause the development of fluorosis until the bone is damaged. In large quantities (on average more than 20 mg), fluorine can exhibit its toxic properties, namely, the violation of fat and carbohydrate metabolism, i.e. A decrease in the body's ability to absorb sugar.

### **Terms of Use of the Medical Device**

The Remineralizing gels «Sherbet» designed for professional use in hospitals.

#### 4. Composition and materials used

The composition of the Remineralizing gels Sherbet APF Thixotropic Gel is shown in Table 1..

**Table 1**

| <b>Composition</b>                          | <b>%</b> |
|---------------------------------------------|----------|
| Food coloring                               | 0-1      |
| Aroma                                       | 1-2      |
| Hydrofluoric acid CAS # 7664-39-3           | 4-8      |
| Potassium hydroxide (E525) CAS # 1310-58-3  | 2-6      |
| Magnesium aluminum silicate CAS # 1327-43-1 | 0-4      |
| Polysorbate 20 CAS # 9005-64-5              | 0-5      |
| Purified Water CAS # 7732-18-5              | 75-88    |
| Saccharin sodium CAS # 81-07-2              | 0-5      |
| Sweetness enhancer (Glutamate sodium E 621) | 0-2      |
| Titanium dioxide CAS # 13463-67-7           | 0-5      |
| Xanthan gum (E415) CAS # 11138-66-2         | 0-4      |

Depending on taste and color the Remineralizing gels Sherbet APF Thixotropic Gel may contain following dyes and flavors:

|                             |
|-----------------------------|
| <b>Flavor</b>               |
| The taste Chewing Bubble    |
| The taste Cherry Pie        |
| The taste Chunky Mango      |
| The taste Mint Mousse       |
| The taste Strawberry Heaven |
| The taste French Vanilla    |
| <b>Food coloring</b>        |
| FD&C Red #40 E129           |
| FD&C Green #3 E143          |
| FD&C Yellow #5 - E102       |

The composition of the Remineralizing gels Sherbet Neutral Thixotropic Gel is shown in Table 2.

**Table 2**

| <b>Composition</b>                         | <b>%</b> |
|--------------------------------------------|----------|
| Potassium hydroxide (E525) CAS # 1310-58-3 | 4-8      |
| Food coloring                              | 0-1      |

|                                             |       |
|---------------------------------------------|-------|
| Aroma                                       | 1-2   |
| Hydrofluoric acid CAS # 7664-39-3           | 4-8   |
| Magnesium aluminum silicate CAS # 1327-43-1 | 0-4   |
| Polysorbate 20 CAS # 9005-64-5              | 0-5   |
| Purified Water CAS # 7732-18-5              | 75-88 |
| Saccharin sodium CAS # 81-07-2              | 0-5   |
| Sweetness enhancer (Glutamate sodium E 621) | 0-2   |
| Titanium dioxide CAS # 13463-67-7           | 0-5   |
| Xanthan gum (E415) CAS # 11138-66-2         | 0-4   |

Depending on taste and color the composition of the Remineralizing gels Sherbet Neutral Thixotropic Gel may contain the following dyes and flavors:

|                             |
|-----------------------------|
| <b>Flavor</b>               |
| The taste Mint Mousse       |
| The taste Strawberry Heaven |
| The taste French Vanilla    |
| <b>Food coloring</b>        |
| FD&C Red #40 E129           |
| FD&C Green #3 E143          |
| FD&C Yellow #5 E102         |

Composition of the Remineralizing gels Sherbet Neutral Fluoride Gel 1.1% is shown in Table 3.

**Table 3**

| <b>Composition</b>                      | <b>%</b> |
|-----------------------------------------|----------|
| Water CAS # 7732-18-5                   | 45-68    |
| Sodium hydroxide (E524) CAS # 1310-73-2 | 3-8      |
| Hydrofluoric acid CAS # 7664-39-3       | 3-8      |
| Potassium nitrate CAS # 7757-79-1       | 4-9      |
| Carbopol CAS # 9003-01-4                | 9-15     |
| Aroma                                   | 1-2      |
| Saccharin sodium CAS # 81-07-2          | 2-5      |

Depending on the taste the Remineralizing gels "Sherbet" Neutral Fluoride Gel 1.1% may contain the following flavors:

|                |
|----------------|
| <b>Flavor</b>  |
| Chewing Bubble |
| Mint Mousse    |

## 5. Technical specifications

Technical Specifications of the Remineralizing gels «Sherbet» presented in Table 5.

**Table 5**

| <b>Remineralizing gel Sherbet APF Thixotropic Gel</b>       |                                                                                     |
|-------------------------------------------------------------|-------------------------------------------------------------------------------------|
| Appearance                                                  | A thick, non-transparent gel of uniform color, visually without external impurities |
| Acidity (pH) at a temperature of 20-25 °C:                  | 4.05 - 5.00                                                                         |
| Viscosity at 25 °C                                          | 10,000 cP - 45,000 cps                                                              |
| Fluoride ion test:                                          | 1.11 - 1.35% (in weight ratio)                                                      |
| Total number of microorganisms:                             | <1 CFU / ml                                                                         |
| Weight of dry residue (per volume 500ml)                    | 50-100 g                                                                            |
| <b>Remineralizing gel "Sherbet" Neutral Thixotropic Gel</b> |                                                                                     |
| Appearance                                                  | A thick, non-transparent gel of uniform color, visually without external impurities |
| Acidity (pH) at a temperature of 20-25 °C:                  | 6.5 - 8.5                                                                           |
| Viscosity at 25 °C                                          | 10,000 cP - 45,000 cps                                                              |
| Fluoride ion test:                                          | 1.8% - 2.2% of the total weight                                                     |
| Total number of microorganisms:                             | <1 CFU / ml                                                                         |
| Weight of dry residue (per volume 500ml)                    | 50-100 g                                                                            |
| <b>Remineralizing gel Sherbet Neutral Fluoride Gel 1.1%</b> |                                                                                     |
| Appearance                                                  | A thick white gel, visually without external impurities                             |
| Acidity (pH) at 20-25 °C:                                   | 5.2 - 6.4                                                                           |
| Viscosity at 25 °C                                          | 51,300-62,700 cps                                                                   |
| Sodium Fluoride Sample                                      | 0.99 - 1.21% of total weight                                                        |
| Sample of potassium nitrate                                 | 4.5 - 5.5% of the total weight                                                      |
| Total number of microorganisms                              | <1 CFU / ml                                                                         |
| Weight of dry residue (per volume 500ml)                    | 100-150 g                                                                           |

## 6. Methods and methods of disinfection

The Remineralizing gels «Sherbet» are supplied non-sterile.

The packaging system for non-sterile products should preserve the product without deterioration at the level of purity provided.

## 7. Packing

Each material is packed in the manufacturer's original packaging. Packaging of the product ensures safety during storage, transportation and handling. Helps avoid damage and deterioration of product properties.

## 8. Information on product labeling

On each package there is a clear marking, containing:

- trade or firm name of the product;
- address and name of the manufacturer or supplier;
- storage condition;
- serial number (Lot);
- the composition of the product;
- the end date of the use of "use up to ..."

The interpretation of the symbols used in the marking is presented in Table 9

**Table 9**

| Symbol                                                                              | Explanation                                |
|-------------------------------------------------------------------------------------|--------------------------------------------|
|    | Batch number                               |
|    | Shelf life                                 |
|    | Temperature limitation                     |
| Re-order:                                                                           | Article number                             |
|    | Attention, read the instructions for use!  |
|    | Protect from moisture                      |
|  | Dispose of separately from household waste |
|  | up                                         |
|  | fragile                                    |

## 9. Precautionary measures

- Keep out of the reach of children
- Do not swallow. In case of ingestion of the drug inside in quantities applicable when using the drug, promptly seek medical help or contact a poison control center.
- For professional use only.
- Keep away from direct sunlight.
- In case of contact with eyes, rinse immediately with water and consult a doctor.

## 10. Method of use

### 10.1 Method of use of the Remineralizing gel Sherbet APF Thixotropic Gel

Shake before use.

After a thorough prophylactic cleaning of the tooth surface, fill two single or one double fluoride tray for a 1/3 by gel. Air dry the surface of the teeth and enter the tray into the oral cavity. Instruct patient to slightly bite the trays and keep them in this state for 1 minute (80% efficiency) or for 4 minutes (100% efficiency). Remove trays and instruct the patient to spit the gel remnants and do not eat for at least 30 minutes.

For best results, use twice a year.

## **10.2 Method of use of the Remineralizing gel Sherbet Neutral Thixotropic Gel**

Shake before use.

After a thorough prophylactic cleaning of the tooth surface, fill two single or one double fluoride tray for a 1/3 by gel. Air dry the surface of the teeth and enter the trays into the oral cavity. Instruct the patient to gently bite the trays and hold them in this state for 4 minutes. Remove trays and instruct the patient to spit the gel remnants and do not eat for at least 30 minutes.

For best results, use twice a year.

## **10.3 Method of use of the Remineralizing gel Sherbet Neutral Fluoride Gel 1.1%**

### **Method of application 1 (using an application spoon)**

After a thorough prophylactic cleaning of the tooth surface, fill two single or one double fluoride tray with 1/3 gel. Air dry the surface of the teeth and enter the trays into the oral cavity. Instruct the patient to gently bite the trays and hold them in this state for 4 minutes. Remove trays and instruct the patient to spit the gel remnants and do not eat for at least 30 minutes.

### **Method of application 2**

After a thorough prophylactic cleaning of the surface of the teeth, squeeze 2.5 cm of gel onto a toothbrush with soft bristles and spread over the entire surface of the teeth. Leave the gel on the teeth for 1 minute. Spit and rinse your mouth. Make sure that all surfaces that have increased sensitivity are treated with a brush. It is applied twice a day as recommended by a doctor.

## **11. Conditions of storage and transportation**

Recommendations on storage:

Temperature of 15-30 ° C

Relative air humidity from 30 to 75% non-condensing

Do not freeze.

Shelf life 36 months.

The Remineralizing gels "Sherbet" are transported by all types of covered vehicles in accordance with the rules of transportation of goods operating on the appropriate mode of transport and regulatory documents for loading and securing cargo. Conditions of transportation coincide with storage conditions.

## **12. Disposal of a medical device**

Disposal is carried out in accordance with regulatory documents and requirements for the disposal of this type of medical devices adopted in the country of distribution.

Do not discharge excess product into the sewer system, do not dispose of it in a public garbage container; Use the device for the professional disposal of chemical waste.

### **13. Warranty**

The manufacturer guarantees that the products produced and supplied are free from defects in terms of materials and workmanship. The warranty period is 36 months.

The manufacturer shall not be liable in case of improper use, misuse, negligence and violation of storage conditions. The warranty does not apply to products that have been damaged as a result of improper or incorrect use or if repairs have been made or any changes to the product have been made by any person other than the Manufacturer.

### **14. Complaint**

The organization accepting in the territory of the Russian Federation claims from the consumer for the quality of the product:

Limited Liability Company "StaiNo"

LLC " StaiNo ", Russian Federation, 119415, Moscow, ul. Udaltsova, 12, room.160

Tel .: +7 (495) 540-57-10, E-mail: [info@staino.ru](mailto:info@staino.ru), Website: <http://staino.ru>

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A black and white copy of this document is not official

## State of Florida

Department of State

## APOSTILLE

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1. Country: United States of America

This public document

2. has been signed by

GABRIEL COLON

3. acting in the capacity of

Notary Public of Florida

4. bears the seal/stamp of

Notary Public, State of Florida

Certified

5. at Tallahassee, Florida6. the Twenty-Fifth day of March, A.D., 20187. by Secretary of State, State of Florida8. No. 2018-21674

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State of Florida  
County of Dade

Sworn to and subscribed before me  
This 23 day of MARCH 2018  
By MARIA J. CARLES  
Type of Identification Produced DRIVERS LICENSE

«Certify»  
Manager Regulatory Affairs  
Maria Jose Carles

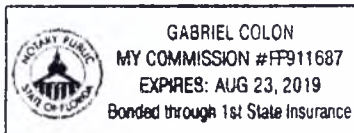
Maria Jose Carles  
«23» March 2018

State of Florida  
County of Miami-Dade

The foregoing instrument was acknowledged before  
me this 23 day of March, 2018, by  
MARIA J. CARLES

Gabriel Colon  
Notary Signature

Personally Known \_\_\_\_\_ OR Produced Identification \_\_\_\_\_  
Type of Identification Produced Driver License

Remineralizing gels «Sherbet»

in performance

Supplement to the Instructions for Use

*This supplement is designed to make the following changes to the instructions for the use of the Remineralizing gels "Sherbet" in performance:*

Section 3 "**Determination, composition and basic principles of the functioning of a medical device**" subsection "**Terms for use of Medical Device**" read as follows:

Sherbet remineralizing gels are intended only for professional use by dentists in health facilities.

Section 10 "**Method of application**", subsection 10.3 "**Method of application of Remineralizing pH-Neutral gel 1.1% of Sherbet Neutral Fluoride Gel 1.1%**", the method of application 2 shall be read as follows:

After a thorough prophylactic cleaning of the surface of the teeth, squeeze 2.5 cm of gel onto a toothbrush with soft bristles and spread over the entire surface of the teeth. Leave the gel on the teeth for 1 minute. Spit and rinse your mouth. Make sure that all surfaces that are sensitive are treated with a brush.

All other sections of the instructions for the use of the medical device "Remineralizing Gels "Sherbet" in performance" remain unchanged.

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