Department of State

APOSTILLE

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1. Country: United States of America

This public document

2. has been signed by GABRIEL COLON

3. acting in the capacity of Notary Public of Florida

4. bears the seal/stamp of Notary Public, State of Florida

Certified

5. at Tallahassee, Florida

6. the Sixteenth day of November, A.D., 2017

7. by Secretary of State, State of Florida

8. No. 2017-125915

9. Seal/Stamp:

10. Signature:

Ken Peterson

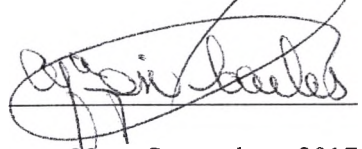
Secretary of State

DSDE 99 (2/12)

This document contains a true watermark. Hold up to light to see "SAFE" and "VERIFY FIRST."

«Certify»
Manager Regulatory Affairs
Maria Jose Carles

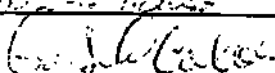
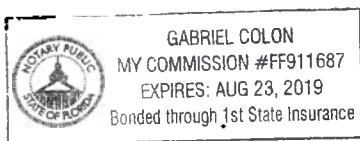
«22» September 2017

State of Florida
County of Dade

Sworn to, and subscribed before me
This _____ Day of _____
By _____

Type of Identification presented

Prophylaxis paste “Sherbet”

Instructions for use

1. The name of the medical product

Prophylaxis paste Sherbet (details in Annex)

2. Information about the developer, manufacturer and place of production of medical products

Developer

Dharma Research, Inc.

5220 N.W. 72nd Avenue, Bay 15, Miami, Florida, 33166 USA

Manufacturer

Dharma Research, Inc.

5220 N.W. 72nd Avenue, Bay 15, Miami, Florida, 33166 USA

The place of production

Dharma Research, Inc.

5220 N.W. 72nd Avenue, Bay#15, Miami, Florida, 33166 USA

3. Appointment, composition and basic principles of the functioning of a medical device

Appointment

Prophylaxis paste "Sherbet" are used in dentistry for professional removal of dental deposits and prevention of dental caries.

Composition and basic principles of the functioning of the medical device

Sherbet Prophylaxis paste is a preventive professional toothpaste with different tastes (Chewing Bubble, Cherry Pie, Chunky Mango, Raspberry Jelly and Assorted flavors) and varying degrees of grit (Fine, Medium, Coarse) which allows doctor to carry out procedures from rough surface cleaning to fine polishing, both teeth and fillings. It can contain 1.23% fluoride ions.

Sherbet Prophylaxis paste, Assorted Flavors taste includes 4 types of taste: Chewing Bubble, Cherry Pie, Chunky Mango, Raspberry Jelly.

Sherbet Prophylaxis paste can be supplied with individual cups (unidoses): 100 unidose (200g) per package and 200 units (400g) in a package or in jars of 100g, 170g, 340g.

Sherbet Prophylaxis paste in unidoses are supplied with Ring -holder Propy Ring. Ring - holder Propy Ring - is a ring with a holder for unidose, which is put on the finger of

the hand to ensure comfortable work of the doctor. Sherbet Prophylaxis paste in unidoses are supplied with Sherbet Disclosing Solution. Sherbet Disclosing Solution - is a solution used to detect plaque. The indicator of dental plaque - Sherbet Disclosing Solution is supplied in a 30 ml vial.

The main active substance of Sherbet Prophylaxis paste is pumice, due to its abrasiveness when the plaque is removed from the teeth.

The degree of granularity of the prophylactic paste depends on the size of the abrasive particles, which mechanically cleans the enamel of the teeth from plaque and tartar. The abrasive is 35-55% of the entire contents of the paste. The larger the particle size of the abrasive, the higher the abrasiveness of the prophylaxes paste.

Indications

- Removal of soft and mineralized plaque from the surface of the teeth.
- Elimination of discolorations caused by the deposition of food pigments on the tooth surface (coffee, tea, tobacco)
- The need of reducing the volume of oral microorganisms, thereby preventing gingivitis and periodontal disease

Contraindications

Identified hypersensitivity to fluoride.

Possible side effects

- Abrasion hard tooth tissues.
 - Should limit the use of fluoride-containing drugs in areas with a high content of fluoride in water (the critical concentration of fluoride for children is more than 1.5 mg / l, for adults - 6 mg / l), as this leads to the accumulation of fluoride in the body. Constant exposure to fluorides can cause the development of fluorosis until the bone is damaged.
- In large quantities (on average more than 20 mg), fluorine can exhibit its toxic properties, namely, the violation of fat and carbohydrate metabolism, i.e. A decrease in the body's ability to absorb sugar.

Terms of use of medical device

Sherbet Prophylaxis pastes are designed for professional use only in health facilities.

Precautions

- Keep away from children.
- Do not swallow. Excessive ingestion of fluorides may cause nausea.
- For professional use only.

4. The composition and materials used

Prophylaxis paste Sherbet presented in Table 1

Table 1

Composition	%
Hydrofluoric acid	4-8

Potassium hydroxide	2-6
Xanthan gum	10-15
Pumice	35-55
Purified Water	20-25
Sodium saccharin	0-5
Flavour	1-2
Food coloring	0-1

Depending on the taste and colors of the Prophylaxis paste Sherbet may contain the following dyes and fragrances (table 2):

Table 2

Flavours
Chewing Bubble
Cherry Pie
Chunky Mango
Raspberry Jelly
Food coloring
FD&C Yellow #5
FD&C Red #40
FD&C Red #28
FD&C Green #3
FD&C Red #33
FD&C Blue #1

Composition of the Sherbet Prophylaxis paste - Fluoride-Free is presented in Table 3

Table 3

Composition	%
Xanthan gum	10-18
Pumice	35-50
Purified Water	20-25
Sodium saccharin	0-5
Flavour	1-2
Food coloring	0-1

Depending on the taste and colors of the Sherbet Prophylaxis paste Fluoride-Free may contain the following colorants and flavorings (table 4):

Table 4

Flavours
Chewing Bubble
Cherry Pie
Chunky Mango
Food coloring
FD&C Yellow #5
FD&C Red #40
FD&C Red #28
FD&C Green #3
FD&C Red #33
FD&C Blue #1

The composition of the Sherbet Prophylaxis paste is a Dye-Free are presented in Table 5

Table 5

Composition	%
Hydrofluoric acid	4-8
Potassium hydroxide	2-6
Xanthan gum	10-15
Pumice stone	35-55
Purified Water	20-25
Saccharin sodium	0-5
Flavor (Taste Cherry Pie)	1-2

The composition of the Sherbet Prophylaxis paste Fluoride-Free & Dye-Free are presented in Table 6

Table 6

Composition	%
Xanthan gum CAS # 11138-66-2	13-18
Pumice stone CAS # 1332-09-8	35-60
Purified Water CAS # 7732-18-5	20-25
Saccharin sodium CAS # 81-07-2	0-5
Aroma	1-2

The composition of the Disclosing Solution Sherbet presented in Table 7

Table 7

Composition	%
Purified water	50-60
Propylene glycol	30-40

Flavour Cherry pie	5-6
The Red dye	4-5
Saccharin sodium	1-2

5. Specifications

Technical characteristics of the Prophylaxis paste Sherbet presented in Table 8

Table 8

Appearance	A thick homogenous paste of uniform color, without visual external impurities
Acidity (pH) at a temperature of 20-25 C	9.00 – 10.70
Identification of fluoride	The presence of fluorides installed
A sample of the Fluoride ion	1.11 – 1.35% (in terms of weight)
The total number of microorganisms	<1 CFU/ml
The size of abrasive grains microns, not more than:	
Fine	44
Medium	51
Coarse	75

Technical characteristics of the Sherbet Prophylaxis paste Fluoride-Free are presented in Table 9

Table 9

Appearance	A thick homogenous paste of uniform color, without visual external impurities
Acidity (pH) at a temperature of 20-25 C	9.00 – 10.70
Identification of fluoride	The presence of fluorides installed
A sample of the Fluoride ion	1.11 – 1.35% (in terms of weight)
The total number of microorganisms	<1 CFU/ml
The size of abrasive grains microns, not more than:	
Fine	44
Medium	51
Coarse	75

Specifications led dental plaque Disclosing Solution Sherbet presented in Table 10

Table 10

Appearance	A liquid solution is homogeneous red color, visually free from foreign substances
Acidity (pH) at a temperature of 20-25 C	6.00 – 7.00
The total number of microorganisms	<1 CFU/ml

6. The methods and techniques of disinfection

Sherbet Prophylaxis paste are supplied non-sterile.

All tools that cannot be sterilized and used during operation should be disinfected according to current regulations, and instructions for their use.

Packaging system for non-sterile devices must keep the product without deterioration at the level of cleanliness stipulated.

7. Packaging

Sherbet Prophylaxis paste is packed in the original manufacturer's packaging.

The packaging ensures safety during storage, transportation and handling from damage and the deterioration of the properties of the products.

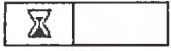
8. Information on product labeling

On each package there is a clear marking, containing:

- trade or firm name of the product;
- address and name of the manufacturer or supplier;
- storage condition;
- serial number (Lot);
- the end date of the use of "use up to ..."

The interpretation of symbols used in the marking shown in Table 11

Table 11

Symbol	Transcript
	The batch number
	Shelf life
Re-order:	The article number
	Without fluoride
	Keep away from moisture
	Temperature limiting
	Do not dispose of together with household garbage
	International universal recycling code for polyethylene of high density)

	International universal recycling code for polyethylene of low density)
	International universal recycling code for ABS plastic
	up
	fragile
	International universal recycling code for Polypropylene

9. Precautions

- Keep away from children.
- Do not swallow. Excessive ingestion of fluorides may cause nausea.
- For professional use only.

10. Method of application

10.1 Application of dental material Prophylaxis paste Sherbet for teeth cleaning in unidoses

Before using the Sherbet Disclosing Solution, shake.

Add 10 drops to 30 ml. water. Apply to the surface of the teeth with a cotton swab or ask the patient to rinse the mouth for 5 seconds, then spit it out. Do not swallow. Dental plaque will turn bright red. After marking the location of the plaque, rinse the mouth thoroughly with water and then use a Sherbet paste.

Use a preventative tip or preventative rubber cups. Fill the cup, pressing it on the paste at a slow speed. Maintain the required amount of paste in the cup to avoid too much dissolving the paste with saliva or swallowing it. It is recommended that you use a saliva ejector during the procedure.

10.2 Method of use Dental material for professional teeth cleaning Sherbet Prophylaxis paste in packages

Use a preventative tip or preventative rubber cups. Fill the cup, pressing it on the paste at a slow speed. Maintain the required amount of paste in the cup to avoid too much dissolving the paste with saliva or swallowing it. It is recommended that you use a saliva ejector during the procedure.

11. The conditions of storage and transport

Storage conditions:

- ambient temperature 15-30 ° C;

- non-condensing relative humidity 30 -75 %

Do not freeze.

Keep jars closed when it not in use. Store in the original packaging provided by the manufacturer.

The shelf life of the Sherbet Disclosing Solution is 36 months.

Shelf life of the dental material for professional cleaning of teeth Sherbet Prophylaxis paste under suitable storage conditions is 60 months.

Prophylaxis pastes Sherbet are transported by all types of covered vehicles in accordance with the rules of transportation of goods operating on the appropriate mode of transport and regulatory documents for loading and securing cargo. Conditions of transportation coincide with storage conditions.

12. Disposal of a medical device

Disposal is carried out in accordance with regulatory documents and requirements for the disposal of this type of medical devices adopted in the country of distribution.

Do not discharge excess product into the sewer system, do not dispose of it in a public garbage container; Use the device for the professional disposal of chemical waste.

Prophylaxis paste Sherbet belong to the class D on the classification of the hazard of medical waste specified in SanPiN 2.1.7.2790-10.

The used materials are disposed of according to the specified class of danger of medical waste according to SanPin

13. Warranty

The manufacturer guarantees that the products produced and supplied are free from defects in terms of materials and workmanship. The warranty period is 60 months for Sherbet Prophylaxis paste and 36 months for the indicator of dental plaque - Sherbet Disclosing Solution.

The manufacturer shall not be liable in case of improper use, misuse, negligence and violation of storage conditions. The warranty does not apply to products that have been damaged as a result of improper or incorrect use or if repairs have been made or any changes to the product have been made by any person other than the Manufacturer.

14. Complaint

The company accepting in the territory of the Russian Federation claims from the consumer for the quality of the product:

ООО «StaiNo», Russian Federation, 119415, Moscow, Udaltsova Str., H. 12, room 160

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