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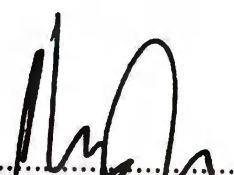
NOTARIAL CERTIFICATE


I, **CHOUNG YOUN SIN**, a Notary Public officiating at Klang, Malaysia hereby certify that the **INSTRUCTIONS MANUAL** and other documents annexed hereto were presented to me and are to my personal knowledge which information I verily believe to be true belong to that of **TOP GLOVE SDN. BHD. (COMPANY NO: 220483-T)** of Lot 4969, Jalan Teratai, Batu 6, Off Jalan Meru, 41050 Klang, Selangor Darul Ehsan, Malaysia.

IN FAITH AND TESTIMONY WHEREOF I, CHOUNG YOUN SIN have hereunto set my hand and affixed my seal of office at Klang aforesaid on 28th March 2018.

.....
CHOUNG YOUN SIN
Notary Public
Klang, Selangor

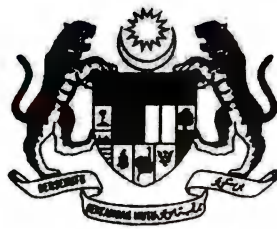Attorney General approval reference number **WKL/25/39/97** dated 20th November 2015 and is valid until 14th February under Notary Public Act 1959

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
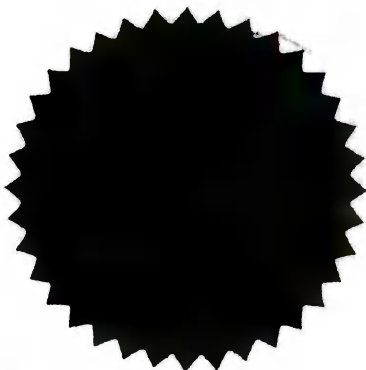
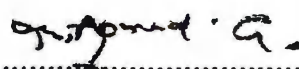
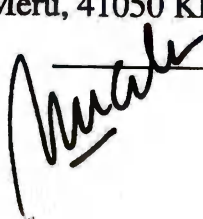
INSTRUCTIONS MANUAL

**Medical natural latex examination gloves Albens,
powder-free, non-sterile, ambidextrous**

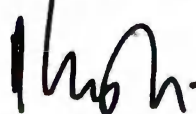NOTARISED BY

CHOUNG YOUN SIN
ADVOCATE & SOLICITOR
NOTARY PUBLIC
1A, LINTANG GELUGOR,
41050 KLANG,
SELANGOR D.E.

28 MAR 2018

1. INTRODUCTION

Dear Customer,

Thank you for the confidence you have given us by purchasing examination gloves Albens of natural latex, powder-free, non-sterile, ambidextrous.

All gloves are tested by the manufacturer for safety and functionality, and leave the plant in top condition.

This manual contains information about the functions necessary for everyday use of natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous.

"TOP GLOVE SDN. BHD"

2. DESCRIPTION AND FUNCTIONS

Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous are used by medical personnel to provide protection upon contact with biological fluids and industrial chemicals and from contamination by microorganisms and viruses at carrying out of diagnostic and other medical procedures.

2.1. Assignment of the medical device

Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous (hereinafter referred to as **examination gloves Albens**) intended for single use in medical research, diagnostic and therapeutic procedures to protect the patient and the researcher from mutual contamination, as well as for use in dealing with contaminated medical materials.

2.2. Composition of the medical device

Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous;

Versions:

- smooth, single chlorinated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- smooth, single chlorinated, high tactile sensitivity; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- smooth, double chlorinated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- smooth, polymer-coated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- smooth, polymer-coated, high tactile sensitivity; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- smooth, chlorinated and polymer-coated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- textured, single chlorinated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- textured, single chlorinated, high tactile sensitivity; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- textured, double chlorinated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- textured, polymer-coated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- textured, polymer-coated, high tactile sensitivity; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- textured, chlorinated and polymer-coated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- textured, single chlorinated, long length and strength; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.

Explanation: The detailed names of versions of the medical device shall be considered

as:

Versions:

- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, single chlorinated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, single chlorinated, high tactile sensitivity; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, double chlorinated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, polymer-coated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, polymer-coated, high tactile sensitivity; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, chlorinated and polymer-coated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, single chlorinated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, single chlorinated, high tactile sensitivity; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, double chlorinated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, polymer-coated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, polymer-coated, high tactile sensitivity; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.

- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, chlorinated and polymer-coated; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.
- Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, single chlorinated, long length and strength; Sizes: XS, S, M, L, XL; Colors: natural, blue, green.

1. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, single chlorinated.

These gloves are powder-free, non-sterile, smooth, ambidextrous (same form for left and right hand), have beaded cuff with roller, providing full fixation on the wrist.

The surface modified by a single chlorination (subject to a single wash with chlorine-containing solution) to prevent sticking of gloves and to facilitate the process of donning.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

For chlorination is used PROCHLOR CL99 (Liquid Chlorine).

The amount of chlorine for a single chlorination is 0.1% - 0.12%, depending on the weight of the glove. The amount of chlorine in single chlorinated gloves for different sizes is as follows: XS - 0.1%; S - 0.1%; M - 0.11%; L - 0.12%; XL - 0.12%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79)

Fig. 1 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, single chlorinated, color: natural, blue, green

2. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, single chlorinated, high tactile sensitivity.

These gloves are powder-free, non-sterile, smooth, ambidextrous (same form for left and right hand), have beaded cuff with roller, providing full fixation on the wrist.

The surface modified by a single chlorination (subject to a single wash with chlorine-containing solution) to prevent sticking of gloves and to facilitate the process of donning.

Thinner glove walls in the finger area provide a high tactile sensitivity.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

For chlorination is used PROCHLOR CL99 (Liquid Chlorine).

The amount of chlorine for a single chlorination is 0.1% - 0.12%, depending on the weight of the glove. The amount of chlorine in single chlorinated gloves for different sizes is as follows: XS - 0.1%; S - 0.1%; M - 0.11%; L - 0.12%; XL - 0.12%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79)

Fig. 2 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, single chlorinated, high tactile sensitivity, color: natural, blue, green

3. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, double chlorinated.

These gloves are powder-free, non-sterile, smooth, ambidextrous (same form for left and right hand), have beaded cuff with roller, providing full fixation on the wrist.

The surface modified by a double chlorination (subject to a double wash with a chlorine-containing solution) to flush out of latex proteins, that helps reduce allergenicity, and to prevent sticking of gloves and to facilitate the process of donning.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

For chlorination is used PROCHLOR CL99 (Liquid Chlorine).

The amount of chlorine for double chlorination is 0.15% - 0.2%, depending on the weight of the glove. The amount of chlorine for double chlorinated gloves in different sizes is as follows: XS - 0.15%; S - 0.15%; M - 0.175%; L - 0.2%; XL - 0.2%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79).

Fig. 3 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, double chlorinated, color: natural, blue, green

4. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, polymer-coated.

These gloves are powder-free, non-sterile, smooth, ambidextrous (same form for left and right hand), have beaded cuff with roller, providing full fixation on the wrist.

The surface modified by a polymer - coating to prevent sticking of gloves and to facilitate the process of donning.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

Material of the polymer - coating: acrylic (type: Flexicoat 99A).

The amount of polymer - coating is up to 2.2%, depending on the weight of the glove. The amount of polymer in a glove for different sizes is as follows: XS - 1.7%; S - 1.8%; M - 2%; L - 2.1%; XL - 2.2%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79)

Fig. 4 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, polymer - coated, color: natural, blue, green.

5. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, polymer-coated, high tactile sensitivity.

These gloves are powder-free, non-sterile, smooth, ambidextrous (same form for left and right hand), have beaded cuff with roller, providing full fixation on the wrist.

The surface modified by a polymer – coating to prevent sticking of gloves and to facilitate the process of donning.

Thinner glove walls in the finger area provide a high tactile sensitivity.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

Material of the polymer - coating: acrylic (type: Flexicoat 99A).

The amount of polymer - coating is up to 2.2%, depending on the weight of the glove. The amount of polymer in a glove for different sizes is as follows: XS - 1.7%; S - 1.8%; M - 2%; L - 2.1%; XL - 2.2%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79).

Fig. 5 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, polymer - coated, high tactile sensitivity, color: natural, blue, green.

6. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, chlorinated and polymer-coated.

These gloves are powder-free, non-sterile, smooth, ambidextrous (same form for left and right hand), have beaded cuff with roller, providing full fixation on the wrist.

The surface is modified by a single chlorination followed by subsequent coating with polymers to prevent sticking of the glove and facilitating the process of putting on.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

For the chlorination with polymer-coating, chlorine is used (type: PROCHLOR CL99 (Liquid Chlorine)) and polymer: acrylic (type: Flexicoat 99A).

The amount of chlorine - 0.1% - 0.15%, polymer - 0.15% - 0.2% for poly-chlorinated coating, is depending on the weight of the glove. The amount of chlorine and polymer in the poly-chlorinated gloves for different sizes is as follows: XS: chlorine - 0.1%, polymer - 0.15%; S: chlorine - 0.1%, polymer - 0.15%; M: chlorine - 0.125%, polymer - 0.175%; L: chlorine - 0.15%, polymer - 0.2%; XL: chlorine - 0.15%, polymer-0.2%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79).

Fig. 6 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, chlorinated and polymer - coated, color: natural, blue, green.

7. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, single chlorinated.

These gloves are powder-free, non-sterile, textured, ambidextrous (same form for left and right hand), have beaded cuff with roller, providing full fixation on the wrist.

The surface modified by a single chlorinated (subject to a single wash with chlorine-containing solution) to prevent sticking of gloves and to facilitate the process of donning.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

For chlorination is used PROCHLOR CL99 (Liquid Chlorine).

The amount of chlorine for a single chlorination is 0.1% - 0.12%, depending on the weight of the glove. The amount of chlorine for a single chlorination in a gloves of different sizes is as follows: XS - 0.1%; S - 0.1%; M - 0.11%; L - 0.12%; XL - 0.12%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79).

Fig. 7 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, single chlorinated, color: natural, blue, green.

8. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, single chlorinated, high tactile sensitivity.

These gloves are powder-free, non-sterile, textured, ambidextrous (same form for left and right hand), have beaded cuff with roller, providing full fixation on the wrist.

The surface modified by a single chlorinated (subject to a single wash with chlorine-containing solution) to prevent sticking of gloves and to facilitate the process of donning.

Thinner glove walls in the finger area provide a high tactile sensitivity.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

For chlorination is used PROCHLOR CL99 (Liquid Chlorine).

The amount of chlorine for a single chlorinated is 0.1% - 0.12%, depending on the weight of the glove. The amount of chlorine in single chlorinated gloves for different sizes is as follows: XS - 0.1%; S - 0.1%; M - 0.11%; L - 0.12%; XL - 0.12%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79).

Fig. 8 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, single chlorinated, high tactile sensitivity, color: natural, blue, green.

9. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, double chlorinated.

These gloves are powder-free, non-sterile, textured, ambidextrous (same form for left and right hand), have beaded cuff with roller, providing full fixation on the wrist.

The surface modified by a double chlorinated (subject to a double wash with chlorine-containing solution) to flush out of latex proteins, that helps to reduce allergenicity, and to prevent sticking of gloves and to facilitate the process of donning.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

For chlorination is used PROCHLOR CL99 (Liquid Chlorine).

The amount of chlorine for double chlorination is 0.15% - 0.2%, depending on the weight of the glove. The amount of chlorine for double chlorination in a gloves in different sizes is as follows: XS - 0.15%; S - 0.15%; M - 0.175%; L - 0.2%; XL - 0.2%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79).

Fig. 9 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, double chlorinated, color: natural, blue, green

10. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, polymer - coated.

These gloves are powder-free, non-sterile, textured, ambidextrous (same form for left and right hand), have beaded cuff with roller, providing full fixation on the wrist.

The surface is modified by a polymer - coating to prevent sticking of gloves and to facilitate the process of donning.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

Material of the polymer - coated: acrylic (type: Flexicoat 99A).

The amount of polymer coated is up to 2.2%, depending on the weight of the glove. The amount of polymer in a glove for different sizes is as follows: XS - 1.7%; S - 1.8%; M - 2%; L - 2.1%; XL - 2.2%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79).

Fig. 10 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, polymer - coated, color: natural, blue, green

11. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, polymer - coated, high tactile sensitivity.

These gloves are powder-free, non-sterile, textured, ambidextrous (same form for left and right hand), have beaded cuff with roller, providing full fixation on the wrist.

The surface modified by a polymer - coating to prevent sticking of gloves and to facilitate the process of donning.

Thinner glove walls in the finger area provide a high tactile sensitivity.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

Material of the polymer - coated: acrylic (type: Flexicoat 99A).

The amount of polymer coated is up to 2.2%, depending on the weight of the glove. The amount of polymer in a glove for different sizes is as follows XS - 1.7%; S - 1.8%; M - 2%; L - 2.1%; XL - 2.2%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79).

Fig. 11 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, polymer - coated, high tactile sensitivity, color: natural, blue, green

12. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, chlorinated and polymer-coated.

These gloves are powder-free, non-sterile, textured, ambidextrous (same form for left and right hand), have beaded cuff with roller, providing full fixation on the wrist.

The surface modified by a single chlorination followed by subsequent coating with polymers to prevent sticking of the glove and facilitating the process of putting on.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

For the chlorination with polymer-coating, chlorine is used (type: PROCHLOR CL99 (Liquid Chlorine) and polymer: acrylic (type: Flexicoat 99A).

The amount of chlorine - 0.1% - 0.15%, polymer - 0.15% - 0.2% for poly-chlorinated coating, is depending on the weight of the glove. The amount of chlorine and polymer in the poly-chlorinated gloves for different sizes is as follows: XS: chlorine - 0.1%, polymer - 0.15%; S: chlorine - 0.1%, polymer - 0.15%; M: chlorine - 0.125%, polymer - 0.175%; L: chlorine - 0.15%, polymer - 0.2%; XL: chlorine - 0.15%, polymer-0.2%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79).

Fig. 12 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, chlorination and polymer - coated, color: natural, blue, green

13. Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, single chlorinated, long length and strength.

Gloves have increased length and strength and are powder-free, non-sterile, textured, ambidextrous (there is no separation between the right and left gloves), have a reinforced cuff providing a durable fixation of the glove on the wrist.

The surface is modified by a single chlorinated (subject to a single wash with chlorine-containing solution) to prevent sticking of the glove and to facilitate the process of putting on.

Material: natural rubber latex of hevea (type: 60% DRC High Ammonia (HA) Latex Concentrate) (Color: natural). The color of the latex itself - natural, the color can have a shade from white to yellow.

Chlorine is chlorinated with PROCHLOR CL99 (Liquid Chlorine).

The amount of chlorine for a single chlorination is 0.1% - 0.12%, depending on the weight of the glove. The amount of chlorine in single chlorinated gloves for different sizes is as follows: XS - 0.1%; S - 0.1%; M - 0.11%; L - 0.12%; XL - 0.12%.

Colorants: blue (type: Blue L7100), green (type: Flexobrite Green 1211/79).

Fig. 13 - Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, single chlorinated, long length and strength, color: natural, blue, green

2.3. Technical description

Technical characteristics of examination gloves Albens are given in Tables 1-3.

Table 1. Quality Standards

№	Characteristics	Level of control	Quality indicators	
			Acceptable quality level (AQL)	Compliance with standards
1	Sizes (width, length, thickness)	S2	4.0	ASTM D3578-05
2	The tensile properties (elongation and gain at break)	S2	4.0	ASTM D3578-05
3	Tightness	GI	1.5	Manufacturer internal standard
4	Visual defects: - main - minor	GI	2.5 4.0	Manufacturer internal standard
5	Packing defects: - regular - encountered - maximum allowable (incl. calculation of gloves)	GI GI S2	- 4.0 4.0	Manufacturer internal standard

Table 2. Sizes

Description	Size	The requirement of ASTM D3578-05	Manufacturer's standard
Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, single chlorinated			
Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small)	70 ± 10	76 ± 3
	S (small)	80 ± 10	84 ± 3
	M (medium)	95 ± 10	94 ± 3
	L (large)	111 ± 10	105 ± 3
	XL (extra large)	N/A	113 ± 3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.11 ± 0.01 Palm: 0.10 ± 0.01 Roller: ≤ 2.50

Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, single chlorinated, high tactile sensitivity

Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small) S (small) M (medium) L (large) XL (extra large)	70 ± 10 80 ± 10 95 ± 10 111 ± 10 N/A	76 ± 3 84 ± 3 94 ± 3 105 ± 3 113 ± 3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.10 ± 0.01 Palm: 0.10 ± 0.01 Roller: ≤ 2.50

Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, double chlorinated

Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small) S (small) M (medium) L (large) XL (extra large)	70 ± 10 80 ± 10 95 ± 10 111 ± 10 N/A	76 ± 3 84 ± 3 94 ± 3 105 ± 3 113 ± 3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.13 ± 0.01 Palm: 0.12 ± 0.01 Roller: ≤ 2.50

Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, polymer - coated

Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small) S (small) M (medium) L (large) XL (extra large)	70 ± 10 80 ± 10 95 ± 10 111 ± 10 N/A	76 ± 3 84 ± 3 94 ± 3 105 ± 3 113 ± 3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.11 ± 0.01 Palm: 0.10 ± 0.01 Roller: ≤ 2.50

Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, polymer - coated, high tactile sensitivity

Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small) S (small) M (medium) L (large) XL (extra large)	70 ± 10 80 ± 10 95 ± 10 111 ± 10 N/A	76 ± 3 84 ± 3 94 ± 3 105 ± 3 113 ± 3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.10 ± 0.01 Palm: 0.10 ± 0.01 Roller: ≤ 2.50

Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, smooth, chlorination and polymer - coated

Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small) S (small) M (medium) L (large)	70 ± 10 80 ± 10 95 ± 10 111 ± 10	76 ± 3 84 ± 3 94 ± 3 105 ± 3

	XL (extra large)	N/A	113±3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.15 ± 0.01 Palm: 0.12 ± 0.01 Roller: ≤ 2.50
Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, single chlorinated			
Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small) S (small) M (medium) L (large) XL (extra large)	70 ± 10 80 ± 10 95 ± 10 111 ± 10 N/A	76 ± 3 84 ± 3 94 ± 3 105 ± 3 113 ± 3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.11 ± 0.01 Palm: 0.10 ± 0.01 Roller: ≤ 2.50
Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, single chlorinated, high tactile sensitivity			
Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small) S (small) M (medium) L (large) XL (extra large)	70 ± 10 80 ± 10 95 ± 10 111 ± 10 N/A	76 ± 3 84 ± 3 94 ± 3 105 ± 3 113 ± 3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.10 ± 0.01 Palm: 0.10 ± 0.01 Roller: ≤ 2.50
Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, double chlorination			
Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small) S (small) M (medium) L (large) XL (extra large)	70 ± 10 80 ± 10 95 ± 10 111 ± 10 N/A	76 ± 3 84 ± 3 94 ± 3 105 ± 3 113 ± 3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.13 ± 0.01 Palm: 0.12 ± 0.01 Roller: ≤ 2.50
Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, polymer - coated			
Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small) S (small) M (medium) L (large) XL (extra large)	70 ± 10 80 ± 10 95 ± 10 111 ± 10 N/A	76 ± 3 84 ± 3 94 ± 3 105 ± 3 113 ± 3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.13 ± 0.01 Palm: 0.12 ± 0.01 Roller: ≤ 2.50

Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, polymer – coated, high tactile sensitivity			
Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small) S (small) M (medium) L (large) XL (extra large)	70 ± 10 80 ± 10 95 ± 10 111 ± 10 N/A	76 ± 3 84 ± 3 94 ± 3 105 ± 3 113 ± 3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.12 ± 0.01 Palm: 0.12 ± 0.01 Roller: ≤ 2.50
Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, chlorinated and polymer - coated			
Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small) S (small) M (medium) L (large) XL (extra large)	70 ± 10 80 ± 10 95 ± 10 111 ± 10 N/A	76 ± 3 84 ± 3 94 ± 3 105 ± 3 113 ± 3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.15 ± 0.01 Palm: 0.12 ± 0.01 Roller: ≤ 2.50
Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous, textured, single chlorinated, long length and strength			
Length (mm.)	XS, S M, L, XL	≥ 220 ≥ 230	min 240
Width (mm.)	XS (extra small) S (small) M (medium) L (large) XL (extra large)	70 ± 10 80 ± 10 95 ± 10 111 ± 10 N/A	76 ± 3 84 ± 3 94 ± 3 105 ± 3 113 ± 3
Thickness (mm.)	For all sizes	Fingers: ≥ 0.08 Palm: ≥ 0.08 Roller: ≤ 2.50	Fingers: 0.22 ± 0.01 Palm: 0.20 ± 0.01 Roller: ≤ 2.50

Table 3. Strength properties

Description	The requirement of ASTM D3578-05		Manufacturer's standard	
	Before aging	After aging	Before aging	After aging
For all types				
Elongation at break (elasticity), %	≥ 650	≥ 500	≥ 650	≥ 500
Force of break, N	≥ 7.0	≥ 6.0	≥ 7.0	≥ 6.0
Tensile Strength, MPa	≥ 18	≥ 14	≥ 18	≥ 14

Tightness

Sampling and testing gloves at sampling inspection performed by the manufacturer in accordance with the sampling plan ISO 2859-1. Levels controls and acceptable levels of quality specified in the table 1 "Quality standards".

Visual defects

The selection and testing of gloves to visual defects (main and minor) by the manufacturer carried out in accordance with a plan of sampling ISO 2859-1. Levels controls and acceptable levels of quality specified in the table 1 "Quality standards".

2. PROPER USE

Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous intended for single use in medical research, diagnostic and therapeutic procedures to protect the patient and the researcher from mutual contamination, as well as for use in dealing with contaminated medical materials.

3.1. Indications

Examination gloves Albens used by medical personnel during evaluation and treatment of patients and other medical procedures that do not require compliance with the conditions of sterility, and upon contact with biological fluids and industrial chemicals at carrying out of laboratory tests and other sanitary procedures.

3.2. Contraindications

The presence of allergic reactions to latex at professionals and / or patients.

The presence of allergic reactions to chlorine and its traces at professionals and / or patients.

3.3. Possible side effects

Possible individual intolerance of materials, the appearance of an allergic reaction.

3.4. Preparation for use

When using gloves, the recommendations in section 3.5 should be followed. Use of the product.

3.5. Use of the product

Medical natural latex examination gloves Albens, powder-free, non-sterile, ambidextrous intended for single use in medical research, diagnostic and therapeutic procedures to protect the patient and the researcher from mutual contamination, as well as for use in dealing with contaminated medical materials.

3.5.1. Preparation of hands before use

The nails of medical workers must be carefully short cut. Should avoid the use of a nail varnish as it can cause undesirable dermatologic reactions, which are often the result of secondary infections.

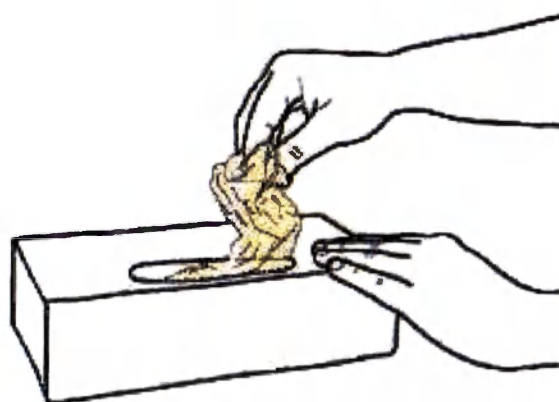
Note that the wearing of wedding rings, signet rings and other jewelry may increase the microbial load when wearing gloves, complicating the process of donning gloves and increase the likelihood of damage.

Preparing to use the product includes the carrying out of hygienic antiseptic treatment of hands, depending on the suspected manipulation or procedures.


5.2. Donning of the gloves

After the preparation of the hands, wear **examination gloves Albens**, using the following procedure:


1. Take the first glove from the packaging;

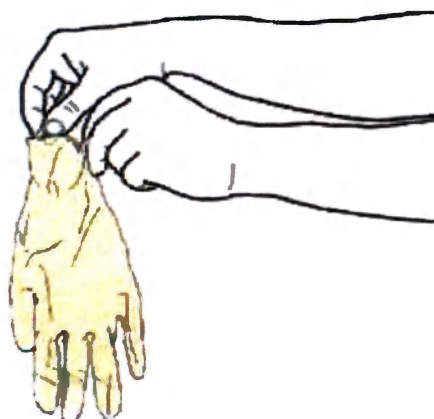2. Take the upper edge of the glove cuff;

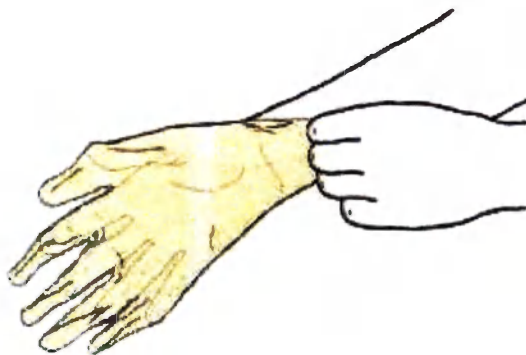
3. Tighten the first glove on the hand;

4. Get a second glove from packaging;

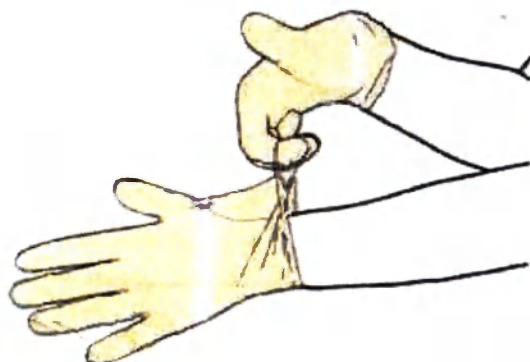5. Turn inside out the outer surface of a glove so that the pull of her bent fingers of the hand in the first glove;


6. Finally, correct gloves, ensures comfortable use.


5.3. Removing of gloves

After completing the procedures **examination gloves Albens** should be removed by using the following procedure:

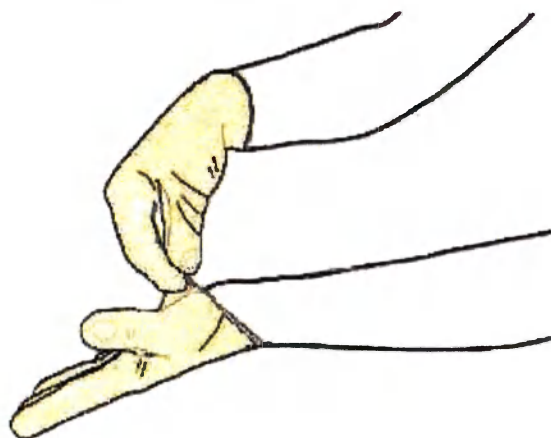
1. Grab first glove by fingers at the wrist, without touching the skin of the forearm, and pull it with your hands so that the glove turned inside out;

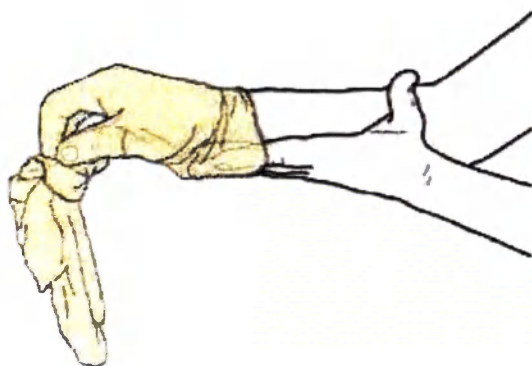
2. Holding the first glove in hand with a glove, draw the fingers of free hand under the other glove;

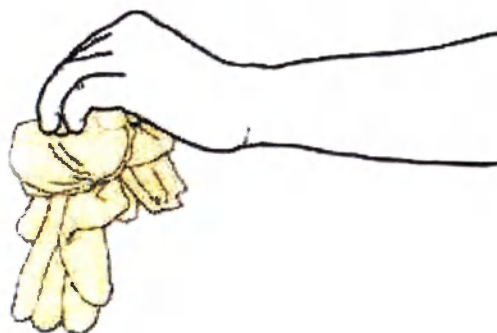3. Pull the second glove in hand, twisted it inside out and using that as the first bag gloves;

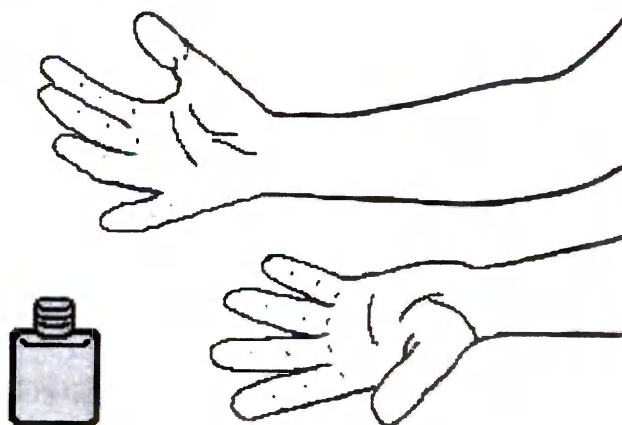4. Place the used gloves in destined place for their capacity and carry out the hygienic antiseptic treatment of hands.

3.5.4. Requirements for the work in gloves

Examination gloves Albens are powder-free, and therefore do not require additional processing before carrying out medical procedures.

Examination gloves Albens is necessary to change:

- every 2 hours of continuous operation with one patient;
- when moving from one patient to another;
- after the discovery of visible defects;
- after the alleged or occurred puncture;
- in the case of contamination of gloves;
- in accordance with the rules adopted in the health facility.

Examination gloves Albens are single use item; processing and reuse of gloves not allowed.

Damaged or corrupted items are unsuitable for use.

It is necessary to strictly observe safety precautions when working with sharp piercing objects and tools.

3.6. Terms of use

- The main factors influencing the choice of a particular kind of gloves and their use:
 - the presence of actual production risks (biological, mechanical, chemical);
 - sanitary and epidemiological conditions (presence or absence of the need to observe of sterility);
 - features of the manipulation (types of the operations and procedures, physical efforts, duration of work in gloves, the need to preserve tactile sensitivity and so forth.);
 - condition of the hands's skin of health worker, the presence of allergy in medical staff and patients, and a number of other factors.
- Use of **examination gloves Albens** does not replace hand hygiene. It is necessary to treat the hands according to the mode of manipulation.
- Use gloves of suitable sizes.
- The surface of the **examination gloves Albens** may be modified in four possible ways: single chlorinated, double chlorinated, polymer coated and a mixed method. This characteristic must be considered when selecting a particular kind of gloves.
- During use, must be maintained the following parameters:
 - Temperature: from +10°C to +30°C;
 - Relative humidity: from 65% to 85%;
 - Atmospheric pressure: from 500 to 1060 gPa.

3.7. Security measures

- Do not use **examination gloves Albens** in the presence of allergy to natural rubber latex at medical workers and / or patient.
- Do not use gloves with visible defects.
- Do not use a one pair of gloves when moving from one patient to another.
- **Examination gloves Albens** are single use item; processing and reuse of gloves not allowed.
- When using the product it is recommended to use the techniques of removal / donning presented in Section 3.5. **Use of the product** of this Application instructions.

4. METHODS AND MEANS FOR DISINFECTION AND CLEANING BEFORE STERILIZATION

Examination gloves Albens are single use items and should not be treated.

Self sterilization of gloves is not allowed. If necessary, use items, sterilized by the manufacturer.

5. REPAIR AND SERVICE

Examination gloves Albens beyond repair and maintenance.

6. STORAGE AND SERVICE LIFE

Examination gloves Albens should be stored in its original packaging in a closed room.

Storage must be carried out under the following conditions:

Temperature: from +10°C to +30°C.

Relative humidity: from 65% to 85%.

Atmospheric pressure: from 500 to 1060 GPa.

Gloves in storage must be protected from direct sunlight.

Store packaged gloves should be away from heating appliances (at a distance of not less than 1 meter).

Avoid joint storage gloves substances having a strong odor, and able to destroy latex (gasoline, kerosene, oil, etc.).

Shelf life of examination gloves Albens – 5 years from date of production under the conditions of storage, transportation and use in accordance with the rules of this Application instructions.

7. TRANSPORTATION

Examination gloves Albens can be transported any kinds of transport, ensuring protection against mechanical damage and atmospheric precipitation during handling operations.

Temperature: from +10°C to +40°C.

Relative humidity: from 65% to 85%.

Atmospheric pressure: from 500 to 1060 GPa.

8. PACKING

Examination gloves Albens packed in a cardboard box or polypropylene packaging. At the request of the client, the number of gloves in the block can vary - 2 pcs; 4 pcs; 6 pcs; 10 pcs; 20 pcs; 50 pcs; 100 pcs. The size of the cardboard box may differ depending on the version of the glove, the size of the gloves and the number of pieces – 50 pcs or 100 pcs.

Packing material: single-layer cardboard (type: 275.M112.275 AF).

At the customer's request, the number of gloves in polypropylene packaging can vary - 2 pcs; 4 pcs; 6 pcs; 10 pcs, 20 pcs. The size of the polypropylene packaging may differ depending on the version of the gloves, the size of the gloves and the number of pieces in the package.

Packing material: polypropylene (type: 9003-07-0).

Table 4. Types of packages used, depending on quantity of pieces inside


Quantity of pieces in a package	Type of package
2 pcs in a package	a polypropylene packaging
4 pcs in a package	a polypropylene packaging
6 pcs in a package	a polypropylene packaging
10 pcs in a package	a polypropylene packaging
20 pcs in a package	a polypropylene packaging
50 pcs in a package	a cardboard box
100 pcs in a package	a cardboard box

9. MARKING

Packing example (a cardboard box):

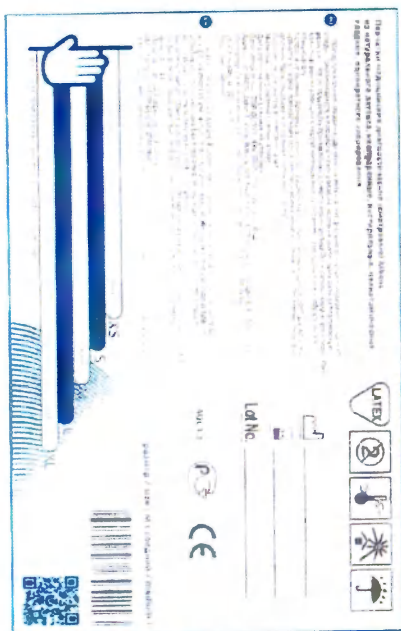Packing example (a polypropylene packaging):

Information on individual packaging:

- product name;
- manufacturer's name and address;
- country of manufacturer;
- the name and address of the importer;
- warning about the possibility of an allergic reaction to natural rubber latex;
- sign informing about the content of natural latex in the product design;
- sign that indicates that the medical device is intended for single use;
- sign indicating the temperature range within which the medical device must be stored;
- sign that indicates that the medical device must be protected from exposure to sunlight;
- sign that indicates that the medical device must be protected from moisture
- manufacture date;
- the expiry date;
- lot number;
- AQL value (acceptable level of quality);

- Size.

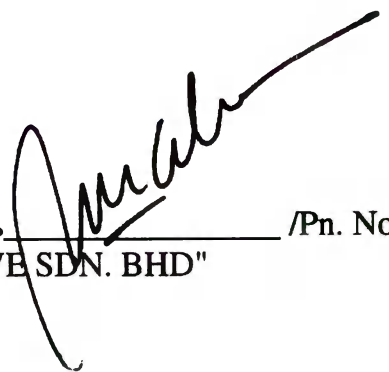
Packing on the other kinds is similar.

10. PROCEDURE OF DESTRUCTION AND DISPOSAL


After performing the necessary medical procedures, after the expiration date or damage to the gloves of diagnostic Albens, which excludes the possibility of further use, the user should dispose of the product after disinfection according to SanPin 2.1.7.2790-10 and art. 22 of the Federal Law "On the circulation of medical devices."

11. COMPLAINTS

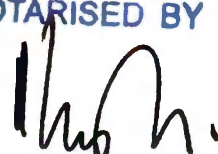If a manufacturing fault detected, the user can refer to the manufacturer's authorized distributor in Russia - The Company Vendors Ltd. (125362, Russia, Moscow, Svobody str. 35, bld. 5), phone: (495) 792-31-91 (multichannel), e-mail: info@torgavtomat.ru.

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