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YH

Introduction

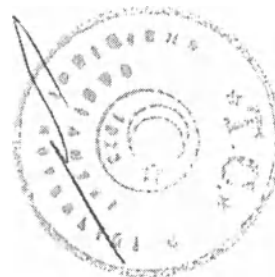
This manual was prepared on 05 April 2017 for the purpose of registration of the medical device in the Russian Federation. This is version 1 (v.1/05.04.2017) of the revised instruction manuals for registration of the medical device in the Russian Federation. The scope of information included in the manual is in compliance with the Rules of Registration of Medical Devices (RF Government Regulation No. 1416 dtd. 27 December 2012) and Order No. 11H of the Ministry of Health Care of the Russian Federation dtd. 19 Jan 2017 "Concerning the approval of requirements to the scope of Manufacturer's technical documentation and instruction manuals for medical devices" (registered with the Ministry of Justice of the Russian Federation on 10 March 2017, Reg. No. 45896).

The original manual for the medical device: Diapers, Baby Medical, Different Types:

- Giggles Diapers, Baby Medical, sizes: «New Baby», «Mini», «Midi», «~~Maxi~~», «Junior», «Junior+»;
- Giggles Premium Diapers, Baby Medical, sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior», «Junior+»;
- Jusy Diapers, Baby Medical, sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior»,

Manufacturer: SEVINCLER SAĞLIK URUNLERI SANAYI VE TICARET ANONIM SİRKETİ, was issued on 28 September 2011.

The contents of this manual was verified against the Requirements to the scope of the Manufacturer's instruction manuals for medical devices as contemplated in Order No. 11H of the Ministry of Health Care of the Russian Federation dtd. 19 Jan 2017. See the compliance matrix in Schedule 2. The compliance of this manual with the Requirements to the scope of the Manufacturer's instruction manuals for medical devices as contemplated in Order No. 11H of the Ministry of Health Care of the Russian Federation dtd. 19 Jan 2017.

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Table of Contents

Table of Contents	3
1. Product Description	5
2. Details of Product Manufacturer:.....	5
2.1 Full Name	5
2.2 Short Name	5
2.3 Form of Incorporation	5
2.4 Address	5
2.5 Contact Info	5
3. Manufacturing Site.....	5
4. Intended Use, Indications and Contraindications, Possible Side Effects, Potential Users	5
4.1 Intended Use of the Product established by Manufacturer:.....	5
4.2 Diapers, Baby Medical. Functional Principle.....	5
4.3 Indications	5
4.4 Contraindications and Possible Side Effects	6
4.5 Potential Users	6
5. Classification.....	6
6. Specifications.....	6
6.1 Composition of Diapers, Baby Medical	6
6.2 Diapers, Baby Medical, Types	7
6.3 Appearance of Diapers, Baby Medical.....	8
6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features.....	8
6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features	12
6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features	18
7. Diapers, Baby Medical. Quality Parameters.....	22
7.1 Microbiological Safety Parameters	22
7.2 Organoleptic, Toxicological, Chemical Safety Parameters.....	22
7.3 Giggles Diapers, Baby Medical. Quality Parameters	23
7.4 Giggles Premium Diapers, Baby Medical. Quality Parameters	23
7.5 Jusy Diapers, Baby Medical. Quality Parameters	23
7.6 No in-use ink set off is allowed if the label is printed on the product.....	24
8. Labelling	24
8.1 Retail Package Labelling.....	24
8.2 Shipping Package Labelling	26
9. Packaging	26
9.1 Retail Package	26
9.2 Shipping Package	27
10. Disposal Information.....	27
10.1 Caution Messages	28
10.2 Disposal Information	28
11. Environmental	28
12. Transportation, storage, disposal.....	29
12.1 Transport Rules.....	29
12.2 Storage Information:.....	29
12.3 Disposal.....	29
13. Use.....	29

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Diapers, Baby Medical. Instruction for Use.....	29
14. Manufacturer's Warranty.....	29
15. Contacts.....	29
16. Schedule 1 Photographic images of the appearance of Diapers, Baby Medical.....	30
17. Schedule 2. The compliance of this manual with the Requirements to the scope of the Manufacturer's instruction manuals for medical devices as contemplated in Order No. 11Н of the Ministry of Health Care of the Russian Federation dtd. 19 Jan 2017.	31

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Hereinafter, the terms: "Diapers, Baby Medical", "Diapers, Baby Medical, different types", "products", "diapers" shall mean this medical device.

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These diapers absorb the child's bodily fluids. The products consist of a special layer made of superabsorbent polymer (SAP) and fluffed pulp that takes up and concentrates the fluids, as well as a protective layer and barrier elements that help lock all the fluids accumulated inside the diaper to avoid the leaks.

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4.5 Potential Users

Diapers, Baby Medical, manufactured by Sevinçler Sağlık Ürünleri San. ve Tic. A. Ş can be used by: Health care professionals in care for infants, preschool- and school-age children in care providing institutions.

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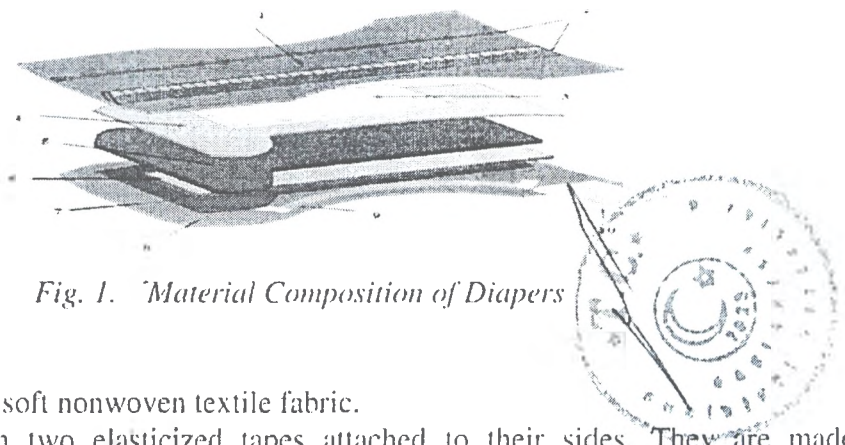
where:

- 1- The top layer made of a soft nonwoven textile fabric.
- 2- Protective barriers with two elasticized tapes attached to their sides. They are made of nonwoven and provide leakage protection.

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[Handwritten signature]

- 3- Absorbing and Distribution Layer (ADL), made of nonwoven. Colour: green. ADL's function is to accelerate the fluid absorption and distribute the fluid over the diaper in order to prevent its accumulation in one spot.
 - 5- Absorbent core arranged under ADL. This layer comprises a mixture of a superabsorbent polymer (SAP) and fluffed pulp. The absorbent core is sandwiched between the nonwoven sheets (top barrier sheet (-4) and bottom backsheet (-6)), which keep the absorbent core in place even when the diaper is full.
 - 7- Protective layer. It covers the absorbent core and the nonwoven sheets. It is a polyethylene layer with images and lettering imprinted on it (the appearance depends on the size and type of the products. For details see p. 6.2 - Diapers, Baby Medical, Types and p. 6.3 - Appearance of Diapers, Baby Medical). This layer helps lock all the fluid accumulated inside the diaper and avoid the leaks.
 - 8- The bottom nonwoven layer provides an exceptional wearing comfort of the products.
 - 9- Protective side bands. These are three rubber bands that fit snugly around the wearer's leg without putting pressure on the skin for a comfort of use.
 - 10- Velcro tapes, made of a propylene film that is coated with nonwoven on the outside and contains pressure-sensitive adhesive and the hooks system, on the inside. Each product has one Velcro tape per rear side strap. You can open and close the diaper as many times as necessary.
- The appearance of Diapers, Baby Medical, is shown in Fig. 2.

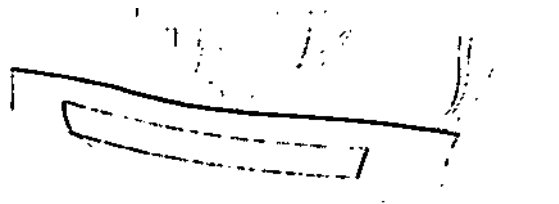
Fig. 2. Appearance of Diapers, Baby Medical

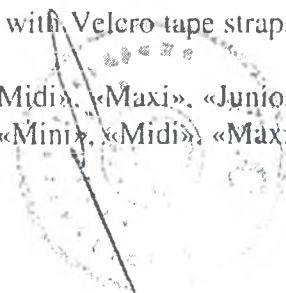
where:

- 1- Side straps made of soft nonwoven textile fabric. The straps ensure a no-pressure comfortable fit. Giggles and Jusy diapers have side straps that are part of one-piece design, two straps per side. Giggles Premium Diapers, Baby Medical have side straps that are glued to the protective layer.
 - 2- Stretchable elastic side straps, made of nonwoven and flexible film, two pieces per product glued to the protective layer covered with nonwoven. This design ensures a no-pressure comfortable fit. The design is used in Giggles Premium diapers.
 - 3- Frontal tape, made of polyethylene covered with the loop system material. The tape keeps the diaper securely fastened and makes the opening of the fasteners by the baby very unlikely. The tape is located at the front and glued to the bottom backsheet layer, one piece per product.
- Therefore, it can be seen that Diapers, Baby Medical manufactured by Sevinçler Sağlık Ürünleri San. ve Tic. A. Ş., consist of the top protective layer, ADL, absorbent core, protective layer, bottom backsheet, barrier elements, fasteners.

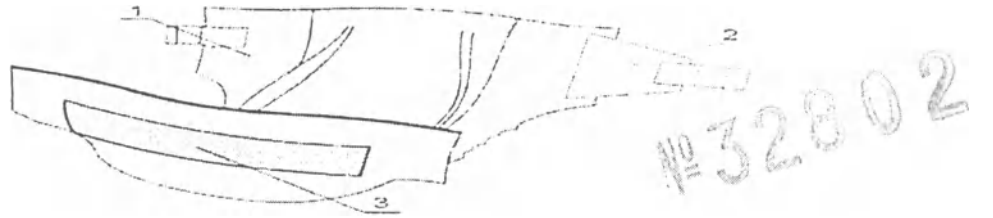
6.2 Diapers, Baby Medical, Types

The products have a comfortable underwear-style cut provided with Velcro tape straps and come in the following types:

Giggles Diapers, Baby Medical, sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior», «Junior+»;
Giggles Premium Diapers, Baby Medical, sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior», «Junior+»;

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6.3 Appearance of Diapers, Baby Medical

Diapers, Baby Medical, manufactured by Sevinçler Sağlık Ürünleri San. ve Tic. A. S., meet the following requirements:

No external defects are allowed in the products including mechanical damage (broken edges, cuts, damaged fasteners, etc.), stains of various origin, inclusions visible to a naked eye. The layers are glued together using a high-performance adhesive to ensure a good bond between the diaper's layers (seams). The seams must be continuous.

The printed symbols on the products must be clear, without blurring. No surface lifting or ink set off is allowed.

The composition of Diapers, Baby Medical must be in accordance with the description given in p. 6.1 Composition of Diapers, Baby Medical.

The configuration and different types of the products are shown in p. 6.2 Diapers, Baby Medical, Types

The linear dimensions, design, and structural features of different types must be as required in p. 6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, p. 6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, p. 6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features.

6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features

6.4.1 Table 2 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Diapers, Baby Medical, size: «New Baby», 12 pcs. per pack.

Table 2. Parameters of Giggles Diapers, Baby Medical . size: «New Baby»

Sl. No.	Parameter	Value/ Description
1	SAP ¹ , g	4 ± 0.5
2	Fluffed pulp, g	9 ± 0.5
3	Product length, mm	360 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	314 ± 5
7	Width of absorbent core (across the top of the product), mm	105 ± 5
8	Width of absorbent core (in the centre), mm	90 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(50 × 25) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	205 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet.

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Table 2. Parameters of Giggles Diapers, Baby Medical , size: «New Baby»

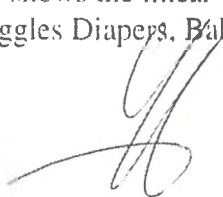
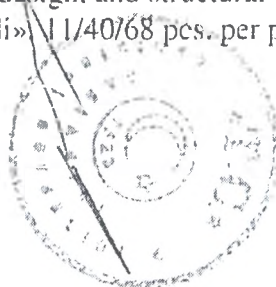
Sl. No.	Parameter	Value/ Description
		Images of animals: bunnies; colours: yellow, violet. Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

6.4.2 Table 3 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Diapers, Baby Medical, size: «Mini», 12 / 44/ 76 pcs.per pack.

Table 3. Parameters of Giggles Diapers, Baby Medical , size: «Mini»

Sl. No.	Parameter	Value/ Description
1	SAP, g	4 ± 0.5
2	Fluffed pulp, g	9 ± 0.5
3	Product length, mm	375 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	315 ± 5
7	Width of absorbent core (across the top of the product), mm	105 ± 5
8	Width of absorbent core (in the centre), mm	90 ± 5
9	Dimensions of Velcro tapes (length x width), mm	$(50 \times 25) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	205 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: – Style of printed lettering: Giggles; colour: violet. – Images of animals: bunnies; colours: yellow, violet. – Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

6.4.3 Table 4 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Diapers, Baby Medical, size: «Midi» 11/40/68 pcs. per pack.

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Table 4. Parameters of Giggles Diapers, Baby Medical, size: «Midi»

Sl. No.	Parameter	Value/ Description
1	SAP, g	6 ± 0.5
2	Fluffed pulp, g	10 ± 0.6
3	Product length, mm	430 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	370 ± 5
7	Width of absorbent core (across the top of the product), mm	110 ± 5
8	Width of absorbent core (in the centre), mm	90 ± 5
9	Dimensions of Velcro tapes (length x width), mm	$(50 \times 30) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	255 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Images of animals: bunnies; colours: yellow, violet. Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

6.4.4 Table 5 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Diapers. Baby Medical, size: «Maxi», 10/36/60 pcs. per pack.

Table 5. Parameters of Giggles Diapers, Baby Medical, size: «Maxi»

Sl.No.	Parameter	Value/ Description
1	SAP, g	8 ± 0.5
2	Fluffed pulp, g	11 ± 0.7
3	Product length, mm	475 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	410 ± 5
7	Width of absorbent core (across the top of the product), mm	115 ± 5
8	Width of absorbent core (in the centre), mm	90 ± 5
9	Dimensions of Velcro tapes (length x width), mm	$(50 \times 30) \pm 5$

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Table 5. Parameters of Giggles Diapers, Baby Medical, size: «Maxi»

Sl.No.	Parameter	Value/ Description
	width), mm	
10	Side barrier (length of elasticized tapes, non-stretched), mm	255 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Images of animals: bunnies; colours: yellow, violet. Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

6.4.5 Table 6 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Diapers, Baby Medical, size: «Junior», 8/28/46 pcs. per pack.

Table 6. Parameters of Giggles Diapers, Baby Medical, size: «Junior»

Sl.No.	Parameter	Value/ Description
1	SAP, g	10.5 ± 0.5
2	Fluffed pulp, g	13 ± 0.7
3	Product length, mm	510 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	450 ± 5
7	Width of absorbent core (across the top of the product), mm	120 ± 5
8	Width of absorbent core (in the centre), mm	100 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(50 × 30) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	285 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Images of animals: bunnies; colours: yellow, violet. – Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

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6.4.6 Table 7 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Diapers, Baby Medical, size: «Junior+», 40 pcs. per pack.

Table 7. Parameters of Giggles Diapers, Baby Medical, size: «Junior+»

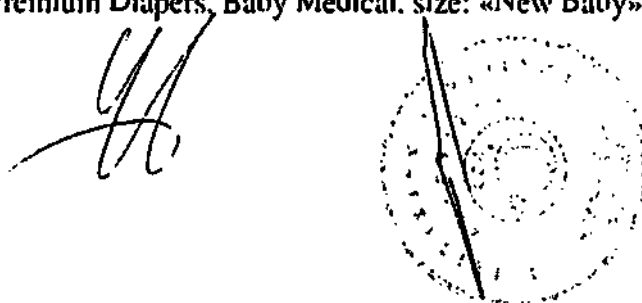
Sl.No.	Parameter	Value/ Description
1	SAP, g	11 ± 0.5
2	Fluffed pulp, g	15 ± 0.7
3	Product length, mm	550 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	510 ± 5
7	Width of absorbent core (across the top of the product), mm	120 ± 5
8	Width of absorbent core (in the centre), mm	100 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(50 × 30) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	285 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Images of animals: bunnies; colours: yellow, violet. Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

Giggles products contain no harmful substances that could affect the human health. Giggles Diapers, Baby Medical, are made of the following materials: hotmelt adhesive, polyethylene, polypropylene, nonwoven, polyester fiber, pulp, polyacrylate (SAP), elastic, green, violet, and yellow dyes.

Photographic images of the appearance of Giggles Diapers, Baby Medical, are shown in Schedule 1. Photographic images of the appearance of Diapers, Baby Medical.

6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features

6.5.1 Table 8 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Premium Diapers, Baby Medical, size: «New Baby», 56 pcs. per pack.

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Table 8. Parameters of Giggles Premium Diapers, Baby Medical , size: «New Baby»

Sl.No.	Parameter	Value/ Description
1	SAP, g	4.5 ± 0.5
2	Fluffed pulp, g	8 ± 0.5
3	Product length, mm	360 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	314 ± 5
7	Width of absorbent core (across the top of the product), mm	100 ± 5
8	Width of absorbent core (in the centre), mm	85 ± 5
9	Dimensions of Velcro tapes (length $\times$ width), mm	$(50 \times 25) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	205 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: – Style of printed lettering: Giggles; colour: violet. – Images of animals: bunnies; colours: yellow, violet. – Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	Instead of being part of a one-piece design, the front and stretchable rear side straps are glued thereto.

6.5.2 Table 9 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Premium Diapers, Baby Medical, size: «Mini», 12/40/58 pcs. per pack.

Table 9. Parameters of Giggles Premium Diapers, Baby Medical , size: «Mini»

Sl.No.	Parameter	Value/ Description
1	SAP, g	4.5 ± 0.5
2	Fluffed pulp, g	8 ± 0.5
3	Product length, mm	375 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	315 ± 5
7	Width of absorbent core (across the top of the product), mm	100 ± 5
8	Width of absorbent core (in the centre), mm	85 ± 5

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Table 9. Parameters of Giggles Premium Diapers, Baby Medical , size: «Mini»

Sl.No.	Parameter	Value/ Description
	mm	
9	Dimensions of Velcro tapes (length × width), mm	(50 × 25) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	175 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: - Style of printed lettering: Giggles; colour: violet. - Diaper No. (2), colour: violet. - Decorative frame around the symbol of the product type and no; colour: pink. - Images of hearts of different size; colours: yellow, violet, pink. - Geometric pattern along the sides; colours: yellow, violet, pink. - Diamond pattern formed with dots of different size, placed over the product surface; colours: violet. - Images of animals: big and small bunnies; colours: yellow, violet, pink. - Rectangle pattern along the top and bottom of the diaper; colours: violet, yellow. Pantones of dyes used: violet Pantone 2587C, yellow Pantone 122C, pink - Pantone 204C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	Instead of being part of a one-piece design, the front and stretchable rear side straps are glued thereto.

6.5.3 Table 10 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Premium Diapers, Baby Medical, size: «Midi», 11/36/48 pcs. per pack.

Table 10. Parameters of Giggles Premium Diapers, Baby Medical , size: «Midi»

Sl.No.	Parameter	Value/ Description
1	SAP, g	6.5 ± 0.5
2	Fluffed pulp, g	9 ± 0.6
3	Product length, mm	430 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	370 ± 5
7	Width of absorbent core (across the top of the product), mm	105 ± 5
8	Width of absorbent core (in the centre), mm	90 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(50 × 25) ± 5

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Table 10. Parameters of Giggles Premium Diapers, Baby Medical, size: «Midi»

Sl.No.	Parameter	Value/ Description
10	Side barrier (length of elasticized tapes, non-stretched), mm	175 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Diaper No. (3), colour: violet. Decorative frame around the symbol of the product type and no; colour: pink. Images of hearts of different size; colours: yellow, violet, pink. Geometric pattern along the sides; colours: yellow, violet, pink. Diamond pattern formed with dots of different size, placed over the product surface; colours: violet. Images of animals: big and small bunnies; colours: yellow, violet, pink. Rectangle pattern along the top and bottom of the diaper; colours: violet, yellow. Pantones of dyes used: violet Pantone 2587C, yellow Pantone 122C, pink - Pantone 204C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	Instead of being part of a one-piece design, the front and stretchable rear side straps are glued thereto.

6.5.4 Table 11 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Premium Diapers, Baby Medical, size: «Maxi», 10/32/44 pcs. per pack.

Table 11. Parameters of Giggles Premium Diapers, Baby Medical, size: «Maxi»

Sl.No.	Parameter	Value/ Description
1	SAP, g	9 ± 0.5
2	Fluffed pulp, g	10 ± 0.7
3	Product length, mm	475 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	410 ± 5
7	Width of absorbent core (across the top of the product), mm	110 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(50 × 30) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	175 ± 5

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Table 11. Parameters of Giggles Premium Diapers, Baby Medical, size: «Maxi»

Sl.No.	Parameter	Value/ Description
11	Design of protective layer	Printed on the white background of the protective layer: – Style of printed lettering: Giggles; colour: violet. – Diaper No. (4), colour: violet. – Decorative frame around the symbol of the product type and no; colour: pink. – Images of hearts of different size; colours: yellow, violet, pink. – Geometric pattern along the sides; colours: yellow, violet, pink. – Diamond pattern formed with dots of different size, placed over the product surface; colours: violet. – Images of animals: big and small bunnies; colours: yellow, violet, pink. – Rectangle pattern along the top and bottom of the diaper; colours: violet, yellow. Pantones of dyes used: violet Pantone 2587C, yellow Pantone 122C, pink - Pantone 204C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	Instead of being part of a one-piece design, the front and stretchable rear side straps are glued thereto.

6.5.5 Table 12 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Premium Diapers, Baby Medical, size: «Junior» 8/25/36 pcs. per pack.

Table 12. Parameters of Giggles Premium Diapers, Baby Medical, size: «Junior»

Sl.No.	Parameter	Value/ Description
1	SAP, g	10.5 ± 0.5
2	Fluffed pulp, g	13 ± 0.7
3	Product length, mm	510 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	450 ± 5
7	Width of absorbent core (across the top of the product), mm	120 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(50 × 30) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	175 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Diaper No. (5), colour: violet.

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Table 12. Parameters of Giggles Premium Diapers, Baby Medical, size: «Junior»

Sl.No.	Parameter	Value/ Description
		Decorative frame around the symbol of the product type and no; colour: pink. Images of hearts of different size; colours: yellow, violet, pink. Geometric pattern along the sides; colours: yellow, violet, pink. Diamond pattern formed with dots of different size, placed over the product surface; colours: violet. Images of animals: big and small bunnies; colours: yellow, violet, pink. Rectangle pattern along the top and bottom of the diaper; colours: violet, yellow. Pantones of dyes used: violet Pantone 2587C, yellow Pantone 122C, pink - Pantone 204C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	Instead of being part of a one-piece design, the front and stretchable rear side straps are glued thereto.

6.5.6 Table 13 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Diapers, Baby Medical, size: «Junior+», 32 pcs. per pack.

Table 13. Parameters of Giggles Premium Diapers, Baby Medical, size: «Junior+»

Sl.No.	Parameter	Value/ Description
1	SAP, g	11 ± 0.5
2	Fluffed pulp, g	15 ± 0.7
3	Product length, mm	550 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	510 ± 5
7	Width of absorbent core (across the top of the product), mm	120 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(50 × 30) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	285 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Images of animals: bunnies; colours: yellow, violet. Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent

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Table 13. Parameters of Giggles Premium Diapers, Baby Medical, size: «Junior+»

Sl.No.	Parameter	Value/ Description
13	Configuration of side straps	Instead of being part of a one-piece design, the front and stretchable rear side straps are glued thereto.

Giggles Premium products contain no harmful substances that could affect the human health. Giggles Premium Diapers, Baby Medical, contain hotmelt adhesive, polyethylene, polypropylene, nonwoven, polyester fiber, pulp, polyacrylate (SAP), elastic, green, violet, yellow, and pink dyes.

Photographic images of the appearance of Giggles Premium Diapers, Baby Medical, are shown in Schedule 1. Photographic images of the appearance of Diapers, Baby Medical.

6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features

6.6.1 Table 14 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Jusy Diapers, Baby Medical, size: «New Baby», 12/40/44 pcs. per pack.

Table 14. Parameters of Jusy Diapers, Baby Medical, size: «New Baby»

Sl.No.	Parameter	Value/ Description
1	SAP, g	3 ± 0.5
2	Fluffed pulp, g	10 ± 0.7
3	Product length, mm	360 ± 5
4	Product width (across the top of the back panel including side straps), mm	260 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	305 ± 5
7	Width of absorbent core (across the top of the product), mm	100 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5
9	Dimensions of Velcro tapes (length $\times$ width), mm	$(60 \times 25) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	205 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Jusy; colours: violet, green. Printed lettering: Baby; colours: violet, green. Images of small birds; colours: green, violet. Images of a teddy bear sleeping on a crescent moon over a cloud; colours: green, violet. Geometric pattern along the sides; colours: green, violet. Geometric pattern - shaded circle; colour: violet. Pantones of dyes used: violet Pantone 266C, yellow Pantone 375C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

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6.6.2 Table 15 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Jusy Diapers, Baby Medical, size: «Mini». 12/40/44 pcs. per pack.

Table 15. Parameters of Jusy Diapers, Baby Medical, size: «Mini»

Sl.No.	Parameter	Value/ Description
1	SAP, g	3 ± 0.5
2	Fluffed pulp, g	10 ± 0.7
3	Product length, mm	375 ± 5
4	Product width (across the top of the back panel including side straps), mm	260 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	305 ± 5
7	Width of absorbent core (across the top of the product), mm	100 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5
9	Dimensions of Velcro tapes (length $\times$ width), mm	$(60 \times 25) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	205 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Jusy; colours: violet, green. Printed lettering: Baby; colours: violet, green. Images of small birds; colours: green, violet. Images of a teddy bear sleeping on a crescent moon over a cloud: colour: green, violet. Geometric pattern along the sides; colours: green, violet. Geometric pattern - shaded circle; colour: violet. Pantones of dyes used: violet Pantone 266C, yellow Pantone 375C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

6.6.3 Table 16 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Jusy Diapers, Baby Medical, size: «Midi». 11/40 pcs. per pack.

Table 16. Parameters of Jusy Diapers, Baby Medical, size: «Midi»

Sl.No.	Parameter	Value/ Description
1	SAP, g	5 ± 0.5
2	Fluffed pulp, g	11 ± 0.6
3	Product length, mm	430 ± 5
4	Product width (across the top of the back panel including side straps), mm	310 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	370 ± 5

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Table 16. Parameters of Jusy Diapers, Baby Medical, size: «Midi»

Sl.No.	Parameter	Value/ Description
7	Width of absorbent core (across the top of the product), mm	100 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(60 × 25) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	255 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Jusy; colours: violet, green. Printed lettering: Baby; colours: violet, green. Images of small birds; colours: green, violet. Images of a teddy bear sleeping on a crescent moon over a cloud; colour: green, violet. Geometric pattern along the sides; colours: green, violet. Geometric pattern - shaded circle; colour: violet. Pantones of dyes used: violet Pantone 266C, yellow Pantone 375C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

6.6.4 Table 17 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Jusy Diapers, Baby Medical, size: «Maxi» 10/36/40 pcs. per pack.

Table 17. Parameters of Jusy Diapers, Baby Medical, size «Maxi»

Sl.No.	Parameter	Value/ Description
1	SAP, g	6 ± 0.5
2	Fluffed pulp, g	12 ± 0.7
3	Product length, mm	475 ± 5
4	Product width (across the top of the back panel including side straps), mm	310 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	410 ± 5
7	Width of absorbent core (across the top of the back panel), mm	120 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(60 × 25) ± 5

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Table 17. Parameters of Jusy Diapers, Baby Medical, size «Maxi»

Sl.No.	Parameter	Value/ Description
10	Side barrier (length of elasticized tapes, non-stretched), mm	255 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Jusy; colours: violet, green. Printed lettering: Baby; colours: violet, green. Images of small birds; colours: green, violet. Images of a teddy bear sleeping on a crescent moon over a cloud: colour: green, violet. Geometric pattern along the sides: colours: green, violet. Geometric pattern - shaded circle; colour: violet Pantones of dyes used: violet Pantone 266C, yellow Pantone 375C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

6.6.5 Table 18 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Jusy Diapers, Baby Medical, size: «Junior», 8/28 pcs. per pack.

Table 18. Parameters of Jusy Diapers, Baby Medical, size «Junior»

Sl.No.	Parameter	Value/ Description
1	SAP, g	8.5 ± 0.5
2	Fluffed pulp, g	13 ± 0.7
3	Product length, mm	510 ± 5
4	Product width (across the top of the back panel including side straps), mm	340 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	450 ± 5
7	Width of absorbent core (across the top of the back panel), mm	120 ± 5
8	Width of absorbent core (in the centre), mm	100 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(60 × 25) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	285 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Jusy; colours: violet, green. Printed lettering: Baby; colours: violet, green. Images of small birds; colours: green, violet.

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Table 18. Parameters of Jusy Diapers. Baby Medical. size «Junior»

Sl.No.	Parameter	Value/ Description
		Images of a teddy bear sleeping on a crescent moon over a cloud: colour: green, violet. Geometric pattern along the sides: colours:green, violet. Geometric pattern - shaded circle: colour: violet. Pantones of dyes used: violet Pantone 266C, yellow Pantone 375C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

Jusy products contain no harmful substances that could affect the human health. Jusy Diapers, Baby Medical, contain hotmelt adhesive, polyethylene, polypropylene, nonwoven, polyester fiber, pulp, polyacrylate (SAP), elastic, green, and violet dyes.

A photographic image of the appearance of Jusy Diapers, Baby Medical, is shown in Schedule 1 Photographic image of the appearance of Diapers, Baby Medical.

7. Diapers, Baby Medical. Quality Parameters

Since manufacturers of the disposable hygiene products are obliged to care about the health and well being of the users, Sevinçler Sağlık Ürünleri is committed to the strict adherence of the microbiological standards as shown in p. 7.1 - Microbiological Safety Parameters and p. 7.2 Organoleptic, Toxicological, Chemical Safety Parameters.

7.1 Microbiological Safety Parameters

Table 19 contains the microbiological safety parameters of Diapers, Baby Medical manufactured by Sevinçler Sağlık Ürünleri San. ve Tic. A. S.

Table 19. Microbiological Safety Parameters of Diapers, Baby Medical. Manufacturer: Sevinçler Sağlık Ürünleri San. ve Tic. A. S.

Description	Value
Total Microbial Count, CFU	Max. 1×10^2
Yeast-like moulds per 1 g	None
Bile-tolerant bacteria per 1 g	None
Pathogenic staphylococci per 1 g	None
Pseudomonas aeruginosa per 1 g	None

7.2 Organoleptic, Toxicological, Chemical Safety Parameters

Diapers, Baby Medical, Manufacturer: Sevinçler Sağlık Ürünleri San. ve Tic., meet all safety and operation standards as given in Table 20.

Table 18. Jusy Medikal Bebek Bezlerinin Parametreleri, boy: «Junior»

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Description	Value
Odour intensity from a diaper (aqueous extract). score, max.	5
Toxicity index, %, inclusive	70 – 120
pH value change, max	± 1.0

7.3 Giggles Diapers, Baby Medical. Quality Parameters

Table 21 shows the functional parameters that ensure the proper performance of Giggles Diapers, Baby Medical.

Table 21. Giggles Diapers, Baby Medical. Quality Parameters

Description	Value					
Diaper Type and Size	Giggles «New Baby» (1)	Giggles «Mini» (2)	Giggles «Midi» (3)	Giggles «Maxi» (4)	Giggles «Junior» (5)	Giggles «Junior+» (6)
Diaper Weight, g	18 ± 3	19 ± 3	23 ± 3	27 ± 3	32 ± 3	35 ± 3
Full moisture absorption, g, min	120.0	160.0	240.0	270.0	300.0	330.0
Absorbing time, s, max.	3.0	3.0	3.0	3.0	3.0	3.0
Resorption, g, max.	0.4	0.4	6	14	16	16

7.4 Giggles Premium Diapers, Baby Medical. Quality Parameters.

Table 22 shows the functional parameters that ensure the proper performance of Giggles Premium, diapers.

Table 22. Giggles Premium Diapers, Baby Medical. Quality Parameters.

Description	Value					
Diaper Type and Size	Giggles Premium «New Baby» (1)	Giggles Premium «Mini» (2)	Giggles Premium «Midi» (3)	Giggles Premium «Maxi» (4)	Giggles Premium «Junior» (5)	Giggles Premium «Junior+» (6)
Diaper Weight, g	18 ± 3	19 ± 3	23 ± 3	27 ± 3	32 ± 3	35 ± 3
Full moisture absorption, g, min	120.0	160.0	240.0	270.0	300.0	330.0
Absorbing time, s, max.	3.0	3.0	3.0	3.0	3.0	3.0
Resorption, g, max.	0.4	0.4	6	14	16	16

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Table 23. Jusy Diapers, Baby Medical. Quality Parameters

Description	Value				
	(1)	(2)	(3)	(4)	(5)
Diaper Weight, g	18 ± 3	19 ± 3	23 ± 3	27 ± 3	32 ± 3
Full moisture absorption, g, min	120.0	160.0	240.0	270.0	280.0
Absorbing time, s, max.	3.0	3.0	3.0	3.0	3.0
Resorption, g, max,	0.4	0.4	6	14	16

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8. Labelling

8.1 Retail Package Labelling

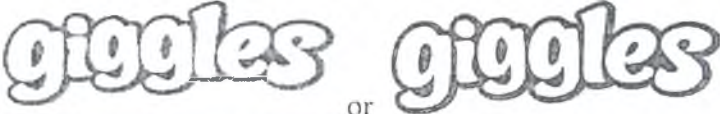
Package labelling for Diapers, Baby Medical, manufactured by Sevinçler Sağlık Ürünleri San. ve Tic. A. S. must be accurate, verifiable, and legible. Labels must be printed on the package, no ink set off is allowed.

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1) Front and rear views:

– Type of diaper in English. images of letterings for different types are shown in Table 24.

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Sl. No.	Diaper Type	Image of Lettering to be included into Label
1	Giggles Diapers. Baby Medical	
2	Giggles Premium Diapers. Baby Medical	
3	Jusy Diapers. Baby Medical	

Description of the product in Russian: «Подгузники детские медицинские» (Diapers, Baby Medical); Photographic image of a mother with a child; photographic image of a child for products size «Junior+»;

View of configuration of a product made as an image, see Fig. 3.

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– Symbol of the age group (size), child's weight range, diaper No., number of diapers per pack:

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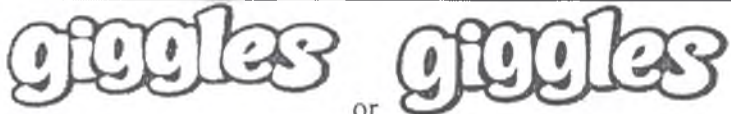
2) Top view:

Type of diaper in English, images of letterings for different types are shown in Table 24.
Description of the product in Russian: «Подгузники детские медицинские» (Diapers, Baby Medical);
image of an arrow and text: "Open here" to indicate the recommended package opening spot;
only for Giggles products: symbol of the age group (size), child's weight range, diaper No., number of
diapers per pack; image of a mother with a child;

3) View from below:

No special label is provided, as this part is reserved for seams;
however, the labelling from the front and/ or the back of the package may continue into the bottom part
of the package;

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List of ingredients:

Batch no. containing the date of manufacture (DD/MM/YY);

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Details of Product Manufacturer (name in English and Russian; Manufacturer's address in English
including the country in English and Russia, tel. No., e-mail address, website, customer service tel.
No.);

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List of all sizes available for the product type in the form of a coloured chart (see Table 25).

The image of the products included in the package is highlighted with a different colour.

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Sl.No.	Diaper Type	Image of the coloured chart labelled on the retail package.
1	Giggles Diapers. Baby Medical	
2	Giggles Premium Diapers. Baby Medical	
3	Jusy Diapers. Baby Medical	

6) Right-side view 1 (to the left from the front view):

– Instructions for use (diagrams and text);

7) Right-side view 2 (to the left from the rear view):

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- Symbol "Do not deposit into toilets";
- Symbol "Keep away from sunlight";
- Symbol "Keep dry";
- Symbol "Storage temperature range";
- Details of dealership in Russia (name, address, tel. No.);
- Details of Registration in Russia: certificate No./ date of issue;
- 8) Additional graphic images required to clarify the label info:
- For Giggles Premium products, the front and rear parts of the package labelling contain a picture of stretchable side straps included in the product. See the scheme in Fig. 4.

Fig. 4. Picture of stretchable side straps

No information regarding the product's special properties may only be included in the package labelling without a certificate attestation.

8.2 Shipping Package Labelling

Shipping packages must be labelled with handling symbols, main, auxiliary, and explanatory text.

The labels must include the following handling symbols:

- Symbol "Keep away from sunlight";
- Symbol "Keep dry".

The main signs must contain:

- Consignee's full or conventional name registered, as appropriate;
- Destination including station or port of transit, where necessary;
- Serial no. of the unit within shipment/ total number of package units per shipment.

The explanatory text must contain the information included in the retail package labelling: Name of product (text in Russian: «подгузники детские медицинские» (Diapers, Baby Medical)), Product type (in English, must be in accordance with the lettering shown in Table 24). Symbol of age group (size), Number of products per pack (pcs.), Batch no. containing the date of manufacture (DD/MM/YY). Expiry date. Details of registration in Russia: certificate No./ date of issue. Manufacturer's name. Consignor's address. Manufacturer's country (text «Изготовлено в Турции» (Made in Turkey)).

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9.2 Shipping Package

Giggles diapers are placed in a polyethylene bag, 20 packs or 2 packs per bag (in case of 8-12-pcs. packs), 5 packs or 3 packs per bag (in case of 28-44-pcs. packs), 4 packs per bag (in case of 40-76-pcs. packs). Otherwise, diapers are shipped in cartons, 4 packs per carton.

Giggles Premium diapers are placed in a polyethylene bag, 20 packs or 2 packs per bag (in case of 8-12-pcs. packs), 5 packs or 3 packs per bag (in case of 25-40-pcs. packs), 4 packs per bag (in case of 32-56-pcs. packs). Otherwise, diapers are shipped in cartons, 4 packs per carton.

Jusy diapers are placed in a polyethylene bag, 20 packs or 2 packs per bag (in case of 8-12-pcs. packs), 5 packs or 3 packs per bag (in case of 28-44-pcs. packs). Otherwise, diapers are shipped in cartons, 4 packs per carton.

The shipping package is in conformity with the applicable standards. It is designed to protect the products against damage during shipping by different means of road and sea transport, transshipment, storage and handling, provided that the proper handling practices are applied.

10. Disposal Information

Products manufactured by Sevinçler Sağlık Ürünleri San. ve Tic. A. S., meet the following requirements:

No external defects are allowed in the products including mechanical damage (broken edges, cuts, damaged fasteners, etc.), stains of various origin, inclusions visible to a naked eye. The layers are glued together using an high-performance adhesive to ensure a good bond between the diaper layers (seams).

The seams must be continuous,

The printed symbols on the diapers must be clear, without blurring or spaces. No surface lifting or ink set off is allowed.

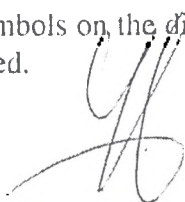
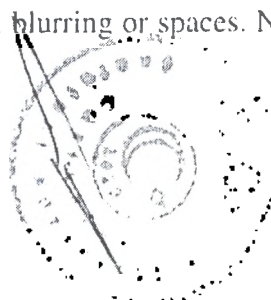
Table 26. Bebek Bezi Baby Medikal, tek paket

Marka	Ölçü	Paket adet sayısı	Marka	Ölçü	Paket ad. sayısı
Giggles	«Yeni Dogan»	12	Giggles Premium	« Yeni Dogan »	56
	«Mini»	12		«Mini»	12
		44			40
		76			58
	«Midi»	11		«Midi»	11
		40			36
		68			48
	«Maxi»	10		«Maxi»	10
		36			32
		60			44
	«Junior»	8		«Junior»	8
		28			25
		46			36
	«Junior+»	40		«Junior+»	32
Jusy	«Yeni Dogan»	12	Jusy	«Midi»	11
		40			40
		44		«Maxi»	10
	«Mini»	12			36
		40			40
		44		«Junior»	8
					28

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The composition of Diapers, Baby Medical must be in accordance with the description given in p. 6.1 – Composition of Diapers, Baby Medical.

The configuration and different types of the products must be as described in p. 6.2 – Diapers, Baby Medical. Types.

The linear dimensions, design, and structural features of different types of diapers must be as required in p. 6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, p. 6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, p. 6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features.

The products contain no toxic substances that could affect the human health.

10.1 Caution Messages

To ensure the safe use of the products, please adhere to the following rules:

Do not use the products having external defects or mechanical damage (broken edges, cuts, damaged fasteners, etc.), stains of various origin, inclusions visible to a naked eye.

Do not use the products having surface lifting or ink set off.

Do not use the products if the package is broken.

Do not use the products with ink set off. The printed symbols on the diapers must be clear, without blurring or spaces.

Do not re-use the products. These products are non-reusable.

Adhere to the instruction for use.

Change when necessary in order to avoid diaper rash.

Do not use after the expiry date.

Keep away from open flame.

Storage Temperature: +18°C - +25°C.

Keep dry.

10.2 Disposal Information

Discard after use.

Depositing the used products into toilets is strictly prohibited.

After use, place the product into a container with a lid, to avoid spreading of bacteria and foul odour that can attract insects.

Do not keep the used diapers at home for a long time.

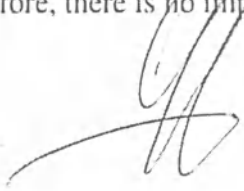
Use specialized anti-bacterial containers for the used diapers.

Dispose of the products in accordance with good medical practice as specified in Sanitary Rules and Regulations SanPiN 2.1.7.2790, as well as local, national, and federal laws and regulations.

11. Environmental

During the manufacture, all the waste is disposed of in accordance with the national environmental law. No hazardous chemical substances form in the manufacture cycle. The only waste the company generates is the waste of raw materials or products that do not meet the specifications. Such waste is collected and sent to recycling.

Sevinçler Sağlık Ürünleri San. ve Tic. A. S. does not use any water in the manufacture, so not water waste is generated. Besides, Sevinçler Sağlık Ürünleri San. ve Tic. A. S. does not generate any gaseous waste, therefore, there is no impact on the environment.

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11. Çevre

To preserve the environment, do not deposit the used products into toilets. use specialized waste containers instead.

12. Transportation, storage, disposal

12.1 Transport Rules

1. The transport must be clean, dry, without any odours.
2. The products can be shipped on or without pallets.
3. The temperature inside the transport must be from -10 C to +25 C.

12.2 Storage Information:

Store the product in a dark well-ventilated place. Keep out of reach of children. away from heaters and flame. Protect against rain and moisture. At warehouses, store the products on pallets.

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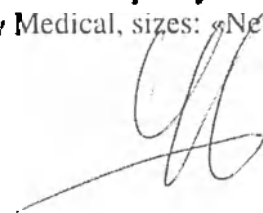
Diagram	Description
	Place a diaper underneath the baby so that the frontal tape is at the front, and the fasteners are at the back.
	Make sure that the protective barriers fold inwards.
	Secure the side tapes on the frontal tape using the fasteners, symmetrically on both sides.
	You can open and close the diaper as many times as necessary. After use, place the diaper into a specialized container. Do not flush.

14. Manufacturer's Warranty

Provided that they have been transported, stored, and used in a proper manner, the quality of the products will be in conformity with the Manufacturer's certificates for 4 (four) year from the date of manufacture.

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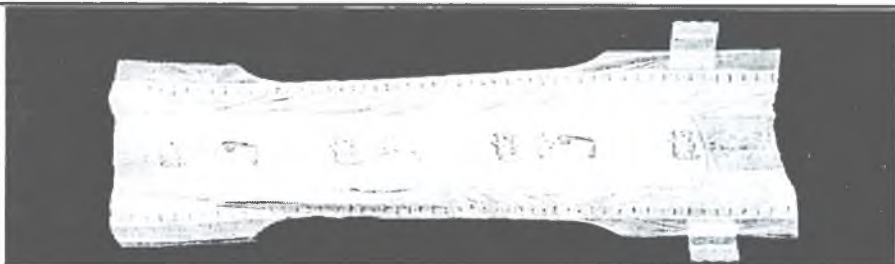
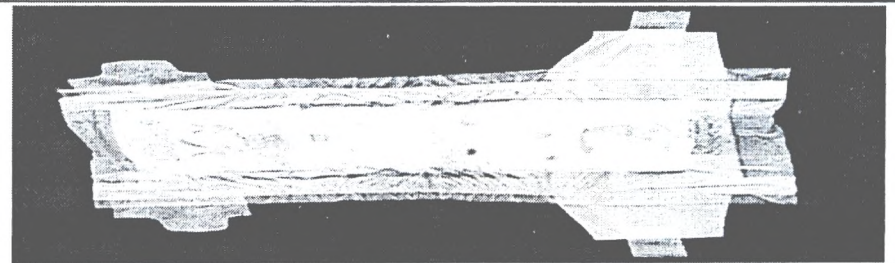
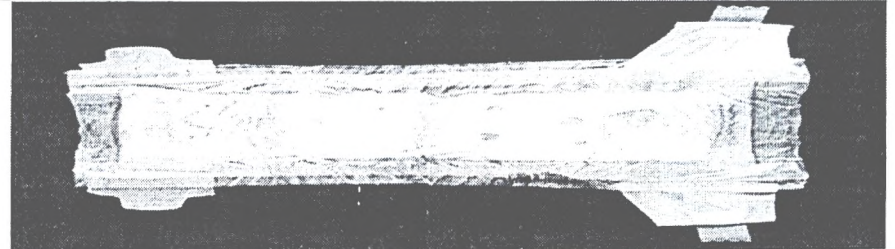
Tel.: +7 (343) 270-75-00, Fax: +7 (343) 246-91-56.

Toll-free number: 8 800 222 3788.

16. Schedule 1 Photographic images of the appearance of Diapers, Baby Medical.

Table 27. Photographic images of the appearance of Diapers, Baby Medical.

Sl.No.	Type	Photographic image of the appearance
1	2	3
1	Giggles Diapers, Baby Medical, sizes: «New Baby» «Mini» «Midi» «Maxi» «Junior» «Junior+»	
2	Giggles Premium Diapers, Baby Medical, size «New Baby»	
3	Giggles Premium Diapers, Baby Medical, size «Mini»	
4	Giggles Premium Diapers, Baby Medical, size «Midi»	

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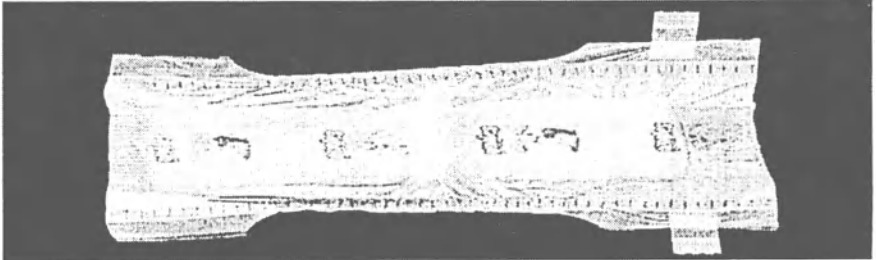
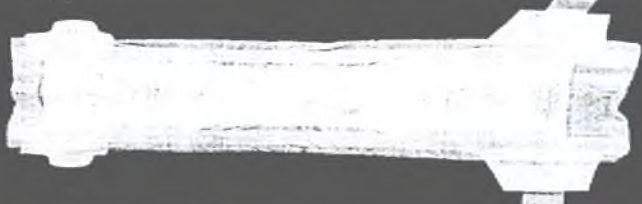
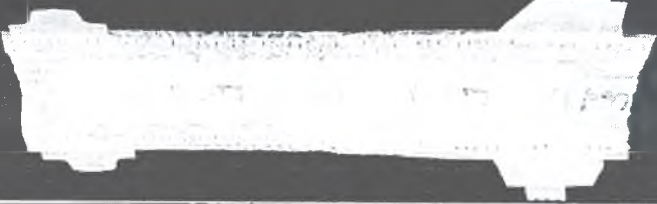
Table 27. Photographic images of the appearance of Diapers, Baby Medical.

Sl.No.	Type	Photographic image of the appearance
1	2	3
1	Giggles Bebek Bezi Medikal, ölçü: «New Baby» «Mini» «Midi» «Maxi» «Junior» «Junior+»	
2	Giggles Premium Bebek Bezi Medikal, ölçü «New Baby»	

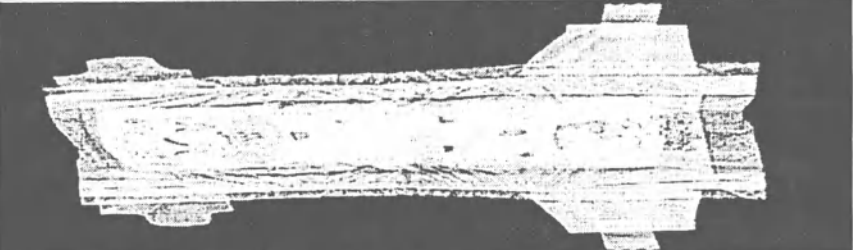
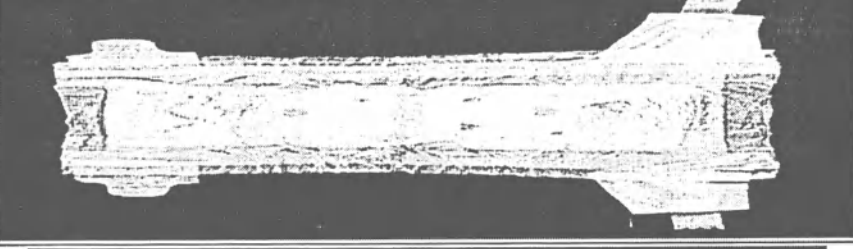
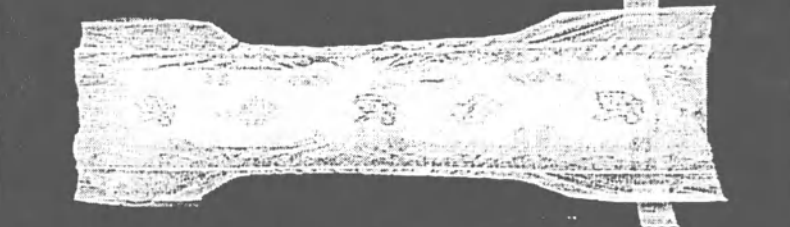
Handwritten signature

5	Giggles Premium Diapers, Baby Medical, size «Maxi»	
6	Giggles Premium Diapers, Baby Medical, size «Junior»	
7	Giggles Premium Diapers, Baby Medical, sizes: «Junior+»	
8	Jusy Diapers, Baby Medical, sizes: «New Baby» «Mini» «Midi» «Maxi» «Junior».	

17. Schedule 2. The compliance of this manual with the Requirements to the scope of the Manufacturer's instruction manuals for medical devices as contemplated in Order No. 11H of the Ministry of Health Care of the Russian Federation dtd. 19 Jan 2017.

Table 28. The compliance of this manual with the Requirements to the scope of the instruction manuals for medical devices as contemplated in Order No. 11H dtd. 19 Jan 2017.

Sl.No.	Order No. 11H Section	Manual Section
1	1) Product Description:	1. Product Description
2	2) Product Manufacturer's info: full and short name (if any) Dealership info: full and....	2. Details of Product Manufacturer. 3. Manufacturing Site 15. Contacts
3	3) Intended use of Product, potential users (e.g. health care professional);	4.1 Intended Use of Product established by Manufacturer 4.5 Potential Users
4	4) Functionality and application of the product:	4.2 Diapers, Baby Medical. Functional Principle 7. Diapers, Baby Medical. Quality Parameters
5	5) Risks from the use of the Product, contraindications, expected and foreseen side effects associated with the intended use of the	Class of potential risk from the use of the Product, in accordance with the Medical device Classification – 1 – low-risk medical devices.

3	Giggles Premium Bebek Bezi Medikal, ölçü «Mini»	
4	Giggles Premium Bebek Bezi Medikal, ölçü «Midi»	
5	Giggles Premium Bebek Bezi Medikal, ölçü «Maxi»	
6	Giggles Premium Bebek Bezi Medikal, ölçü «Junior»	
7	Giggles Premium Bebek Bezi Medikal, ölçü: «Junior+»	
8	Jusy Bebek Bezi Medikal, ölçü: «New Baby» «Mini» «Midi» «Maxi» «Junior».	

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Table 28. The compliance of this manual with the Requirements to the scope of the instruction manuals for medical devices as contemplated in Order No. 11H dtd. 19 Jan 2017.

Sl.No.	Order No. 11H Section	Manual Section
	Product:	4.4 Contraindications and Possible Side Effects Manufacturer's Specification: Diapers, Baby Medical, Different Types: Giggles, Giggles Premium, Jusy Risk analysis procedure Risk analysis and evaluation chart. Risk analysis report.
7	6) Product specification;	5. Classification 6. Specifications 7. Diapers, Baby Medical. Quality Parameters
8	7) Description of non-medical accessories or products to be used in combination with the Product (if any);	N/A
9	8) Details of medicinal products, materials of animal and/ or human origin, that the Product contains;	N/A
10	9) Details of installation, adjustment, calibration and other activities required to put the Product into operation;	13. Operation
11	10) Requirements to rooms where the Product is to be installed, requirements to qualification or skills of the installation and personnel (if any);	N/A
12	11) Details of procedure used to check the Product is properly installed and ready for safe operation, including;	13. Operation
13	a) Scope and intervals of maintenance activities including cleaning and disinfection of the Product;	The products are non-reusable, non-sterile, and not intended for sterilization.
14	b) Product Manufacturer's data, keys, passwords, software which are required for proper installation, operation and maintenance of the Product;	N/A.
15	c) List of consumables (components, chemicals), details of use and replacement;	These products are non-reusable.
16	d) Required calibration for safe and reliable operation of the Product throughout its life;	N/A
17	e) Risk mitigation methods for risks associated with the installation, calibration, or maintenance of the Product;	The products are non-reusable, non-sterile, and not intended for sterilization. 10.1 Caution
18	f) Details of installation, commissioning, adjustment, calibration, and other required activities to put into operation and operate	13. Use

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Table 28. The compliance of this manual with the Requirements to the scope of the instruction manuals for medical devices as contemplated in Order No. 11H dtd. 19 Jan 2017.

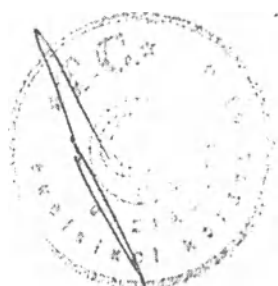
Sl.No.	Order No. 11H Section	Manual Section
	(use) the Product):	
19	g) Details of operational (in-use) characteristics of the Product, storage and transport conditions (e.g., air temperature and humidity, lighting, etc):	13. Use 10. Disposal Information 12. Transportation, storage, disposal
20	h) List of national standards adhered to by the Manufacturer:	The information included in the Product's File (not part of this manual): 1) List of codes and standards
21	12) Details of the Product's sterile condition, sterilization method, what to do in case of damage found in sterile package (if the Product is sterile as supplied), or, where the Product is not sterile when supplied, required sterilization procedure.	The products are non-reusable, non-sterile, and not intended for sterilization.
22	13) Details of preparation of the Product for re-use including cleaning, disinfection, packaging, and, if necessary, re-sterilization (for re-usable products), as well as the Product unfit-for-use criteria:	The products are non-reusable, non-sterile, and not intended for sterilization.
23	14) Product identification information (for products to be used in combination with other medical devices and/ or accessories):	N/A.
24	15) Nature, type, as well as intensity and distribution of emission (electromagnetic, ionizing, other) from the Product, protection methods	N/A.
25	15) Safety precautions for the following situations: a) Breakdowns, operation failures, abnormal functioning of the Product that can affect the Product safety, including found from visual inspection; b) External factors, associated with the use of the Product in combination with other medical devices and/ or equipment, which affect the Product functioning, as well as other predictable factors such as electromagnetic fields, electrostatic discharge, emission (electromagnetic, ionizing, other), atmospheric pressure and its changes, ambient air temperature and humidity; c) Risk of electromagnetic interference caused by the Product that affects other medical devices, equipment, communication systems as a result of diagnostics, treatment or use for	10. Disposal Information

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Table 28. The compliance of this manual with the Requirements to the scope of the instruction manuals for medical devices as contemplated in Order No. 11H dtd. 19 Jan 2017.

Sl.No.	Order No. 11H Section	Manual Section
	intended purposes (e.g. electromagnetic emission from the Product that affects other equipment):	
26	17) Caution messages and/ or precautions to be used by the consumer while using a drug-containing product	N/A.
27	18) Caution messages and/ or precautions to be used by the consumer while disposing of the Product, accessories.....environmental or physical danger caused by the Product:	10.2 Disposal Information 11. Environmental
28	19) Situations where the consumer must seek medical advice:	4.4 Contraindications and Possible Side Effects
29	20) Details of the original document and latest revision of instruction manuals:	Introduction
30	21) Product disposal procedure and conditions.	12.3 Disposal

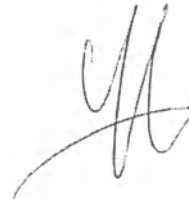
12328 02

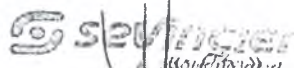
Manual for Medical Device
Diapers, Baby Medical, Different Types
v.1/05.04.2017

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april 2017

EXTRACT FROM TECHNICAL DOCUMENTATION FOR MEDICAL DEVICE:

Diapers, Baby Medical, Different Types:

- Giggles Diapers, Baby Medical, sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior», «Junior+»
- Giggles Premium Diapers, Baby Medical, sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior», «Junior+»
- Jusy Diapers, Baby Medical, sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior»

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2017

Introduction

This extract from Technical Documentation was prepared on 05 April 2017 for the purpose of registration of the medical device in the Russian Federation. This is version 1 (v.1/05.04.2017) of the revised technical documentation for registration of the medical device in the Russian Federation. The scope of information included in this document is in compliance with the Rules of Registration of Medical Devices (RF Government Regulation No. 1416 dtd. 27 December 2012) and Order No. 1111 of the Ministry of Health Care of the Russian Federation dtd. 19 Jan 2017 "Concerning the approval of requirements to the scope of Manufacturer's technical documentation and instruction manuals for medical devices" (registered with the Ministry of Justice of the Russian Federation on 10 March 2017, Reg. No. 45896).

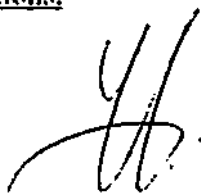
The original specification for the medical device: Diapers, Baby Medical. types:

- Giggles Diapers, Baby Medical, sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior», «Junior+»
- Giggles Premium Diapers, Baby Medical, sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior», «Junior+»
- Jusy Diapers, Baby Medical, sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior»

Manufacturer: SEVINCLER SAGLIK URUNLERI SANAYI VE TICARET ANONIM SIRKETI. was issued on 28 September 2011.

The contents of Extract from Technical Documentation was verified against the Requirements to the scope of the Manufacturer's technical documentation for medical devices as contemplated in Order No. 1111 of the Ministry of Health Care of the Russian Federation dtd. 19 Jan 2017. See the compliance matrix in Schedule 4. The compliance of Extract from Technical Documentation with the requirements to the scope of the Manufacturer's technical documentation for medical devices as contemplated in Order No. 1111 of the Ministry of Health Care of the Russian Federation dtd. 19 Jan 2017.

The contents of this Extract from Technical Documentation was verified against the requirements of the Russian national standard GOST P 52557 – 2011 Paper Children's Diapers. General Specifications. See the compliance matrix in Schedule 5. The compliance of this Extract from Technical Documentation with the requirements of the Russian national standard GOST P 52557 – 2011 Paper Children's Diapers. General Specifications.

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Table of Contents

Table of Contents	3
1. Product Description	5
2. Details of Product Manufacturer:	5
2.1 Full Name	5
2.2 Short Name	5
2.3 Form of Incorporation	5
2.4 Address	5
2.5 Contact Info	5
3. Manufacturing Site	5
4. Intended Use, Indications and Contraindications, Possible Side Effects, Potential Users	5
4.1 Intended Use of the Product established by Manufacturer:	5
4.2 Diapers, Baby Medical. Functional Principle	5
4.3 Indications	5
4.4 Contraindications and Possible Side Effects	5
4.5 Potential Users	6
5. Classification	6
6. Specifications	6
6.1 Composition of Diapers, Baby Medical	6
6.2 Diapers, Baby Medical. Types	7
6.3 Appearance of Diapers, Baby Medical	8
6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features	8
6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features	12
6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features	18
7. Diapers, Baby Medical. Quality Parameters	22
7.1 Microbiological Safety Parameters	22
7.2 Organoleptic, Toxicological, Chemical Safety Parameters	23
7.3 Giggles Diapers, Baby Medical. Quality Parameters	23
7.4 Giggles Premium Diapers, Baby Medical. Quality Parameters	23
7.5 Jusy Diapers, Baby Medical. Quality Parameters	24
7.6 No in-use ink set off is allowed if the label is printed on the product.	24
8. Labelling	24
8.1 Retail Package	24
8.2 Shipping Package Labelling	26
9. Packaging	27
9.1 Retail Package	27
9.2 Shipping Package	27
10. Disposal Information	28
10.1 Caution Messages	28
10.2 Disposal Information	28
11. Environmental	29
12. Acceptance Criteria	29
12.1 The products are accepted for inspection provided the following conditions are met:	29
12.2 The products must pass the following acceptance tests	29
13. Test Methods	30
13.1 In-line Quality Control	30
13.2 Pre-packaging Inspection	30

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13.3 Visual Inspection at Lab	30
13.4 Product Weighing Method	30
13.5 Determining the Full Moisture Absorption	30
13.6 Determining the Absorption Time	31
13.7 Determining the Resorption	31
13.8 Safety Parameters	32
13.8.1 Microbiological Quality Control	32
13.8.2 Organoleptic Parameters	33
13.8.3 Toxicological Parameters	33
13.8.4 Chemical Parameters	33
13.9 Determining the Ink Set off	34
14. Transportation, Storage, Disposal	34
14.1 Transport Rules	34
14.2 Storage Information:	34
14.3 Disposal	34
15. Use	34
Diapers, Baby Medical. Instruction for Use	34
16. Manufacturer's Warranty	34
17. Contacts	34
18. Schedule 1 Materials	36
19. Schedule 2 Manufacturing Process Flow Chart	40
20. Schedule 3 Photographic images of the appearance of Diapers, Baby Medical.	41
21. Schedule 4 The compliance of Extract from Technical Documentation with the requirements to the scope of the Manufacturer's technical documentation for medical devices as contemplated in Order No. 11H of the Ministry of Health Care of the Russian Federation dtd. 19 Jan 2017	42
22. Schedule 5 The compliance of this Extract from Technical Documentation with the requirements of the Russian national standard GOST P 52557 – 2011 Paper Children's Diapers. General Specifications.	45

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1. Product Description

Diapers. Baby Medical. Different Types:

Giggles Diapers, Baby Medical. sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior», «Junior+»

Giggles Premium Diapers, Baby Medical. sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior», «Junior+»

Jusy Diapers, Baby Medical. sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior».

Hereinafter, the terms: "Diapers. Baby Medical", "Diapers. Baby Medical. different types", "products", "diapers" shall mean this medical device.

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Tel: +90(342) 3375700, 4444150

info@sevinclersaglik.com.tr, www.sevincler.com.tr

4. Intended Use, Indications and Contraindications, Possible Side Effects, Potential Users

4.1 Intended Use of the Product established by Manufacturer:

These diapers are designed to receive and retain urine and fecal matter of infants or children with incontinence. The products are intended for child care and come in different types and sizes. The products are non-reusable, non-sterile, and not intended for sterilization.

These products can be used in care providing institutions and/ or at home.

4.2 Diapers, Baby Medical. Functional Principle

These diapers absorb the child's bodily fluids. The products consist of a special layer made of superabsorbent polymer (SAP) and fluffed pulp that takes up and concentrate the fluids, as well as a protective layer and barrier elements that help lock all the fluids accumulated inside the diaper to avoid the leaks.

4.3 Indications

The products are designed to receive and retain bodily fluids of infants, preschool- or school-age children that are not able/ trained to control the urination or bowel movements. The products can be used at home and/ or in care providing institutions (including non-parental care and educational institutions).

4.4 Contraindications and Possible Side Effects

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There are no contraindications/ side effects if the product is handled and used in a standard manner. In rare cases, individual intolerance of ingredients of the products can be observed.

4.5 Potential Users

Diapers, Baby Medical, Manufacturer: Sevinçler Sağlık Ürünleri San. ve Tic. A.Ş. can be used by: Health care professionals in care for infants, preschool- and school-age children in care providing institutions.

Family members of infants, preschool- or school-age children that are not able/ trained to control the urination or bowel movements, who use the products to collect and retain the child's bodily fluids, at home and/ or in care providing institutions.

Care workers providing child care services for infants, preschool- or school-age children at home and/ or in care providing institutions.

5. Classification

The products fall into the following categories based on the child age:

Table 1. Diaper Size and Weight Chart

Age Group	Child Weight, kg	Size	Diaper No.
Newborn (0 to 28 days)	2 – 5	«New Baby»	1
For infants, preschool- and school-age children:			
- Small size	3 – 6	«Mini»	2
- Medium size	4 – 9	«Midi»	3
- Large size	7 – 20	«Maxi»	4
- Extra large size	20 – 25	«Junior»	5
- Extra extra large size	20 – 30	«Junior+»	6

6. Specifications

6.1 Composition of Diapers, Baby Medical

Diapers, Baby Medical, consist of a number of working layers and additional components that do not participate in the absorbing process but make the use of the products easy and comfortable.

The material composition of a diaper is shown in Fig. 1.

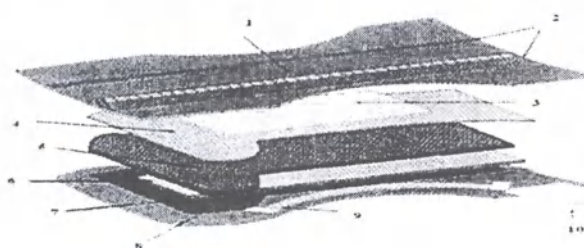
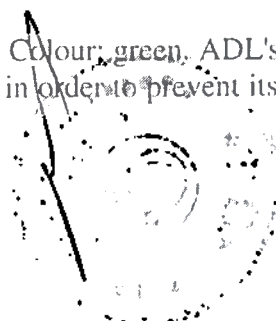
Fig. 1. Material Composition of Diapers

where:

- 1- The top layer made of a soft nonwoven textile fabric.
- 2- Protective barriers with two elasticized tapes attached to their sides. They are made of nonwoven and provide leakage protection.
- 3- Absorbing and Distribution Layer (ADL), made of nonwoven. Colour: green. ADL's function is to accelerate the fluid absorption and distribute the fluid over the diaper in order to prevent its accumulation in one spot.

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- 5- Absorbent core arranged under ADL. This layer comprises a mixture of a superabsorbent polymer (SAP) and fluffed pulp. The absorbent core is sandwiched between the nonwoven sheets (top barrier sheet (-4) and bottom backsheet (-6)), which keep the absorbent core in place even when the diaper is full.
- 7- Protective layer. It covers the absorbent core and the nonwoven sheets. It is a polyethylene layer with images and lettering imprinted on it (the appearance depends on the size and type of the products. For details see p. 6.2 - Diapers, Baby Medical, Types and p. 6.3 - Appearance of Diapers, Baby Medical). This layer helps lock all the fluid accumulated inside the diaper and avoid the leaks.
- 8- The bottom nonwoven layer provides an exceptional wearing comfort of the products.
- 9- Protective side bands. These are three rubber bands that fit snugly around the wearer's leg without putting pressure on the skin for a comfort of use.
- 10- Velcro tapes, made of a propylene film that is coated with nonwoven on the outside and contains pressure-sensitive adhesive and the hooks system, on the inside. Each product has one Velcro tape per rear side strap. You can open and close the diaper as many times as necessary.
- The appearance of Diapers, Baby Medical, is shown in Fig. 2.

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where:

1- Side straps made of soft soft nonwoven textile fabric. The straps ensure a no-pressure comfortable fit. Giggles and Jusy Diapers, Baby Medical, have side straps that are part of one-piece design, two straps per side.

Giggles Premium diapers have side straps that are glued to the protective layer.

2- Stretchable elastic side straps, made of nonwoven and flexible film, two pieces per product glued to the protective layer covered with nonwoven. This design ensures a no-pressure comfortable fit. The design is used in Giggles Premium diapers.

3- Frontal tape, made of polyethylene covered with the loop system material. The tape keeps the diaper securely fastened and makes the opening of the fasteners by the baby very unlikely. The tape is located at the front and glued to the bottom backsheet layer, one piece per product.

For more information on the materials, see Schedule 1. Materials

Therefore, it can be seen that Diapers, Baby Medical, manufactured by Sevinçler Sağlık Ürünleri San. ve Tic. A. Ş., consist of the top protective layer, ADL, absorbent core, protective layer, bottom backsheet, barrier elements, fasteners.

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- No external defects are allowed in the products including mechanical damage (broken edges, cuts, damaged fasteners, etc.), stains of various origin, inclusions visible to a naked eye. The layers are glued together using a high-performance adhesive (See Schedule 1. Materials), to ensure a good bond between the diaper layers (seams). The seams must be continuous.
- The printed symbols on the products must be clear, without blurring. No surface lifting or ink set off is allowed.
- The composition of Diapers, Baby Medical, must be in accordance with the description given in p. 6.1 Composition of Diapers, Baby Medical.
- The configuration and different types of the products are shown in p. 6.2 Diapers, Baby Medical. Types
- The linear dimensions, design, and structural features of different types must be as required in p. 6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, p. 6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, p. 6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features.

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Table 2. Parameters of Giggles Diapers, Baby Medical, size: «New Baby»

Sl.No.	Parameter	Value/ Description
1	SAP ¹ , g	4 ± 0.5
2	Fluffed pulp, g	9 ± 0.5
3	Product length, mm	360 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	314 ± 5
7	Width of absorbent core (across the top of the product), mm	105 ± 5
8	Width of absorbent core (in the centre), mm	90 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(50 × 25) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	205 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Images of animals: bunnies; colours: yellow, violet. Images of hearts which some bunnies hold in their

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Table 2. Parameters of Giggles Diapers, Baby Medical, size: «New Baby»

Sl.No.	Parameter	Value/ Description
		paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

6.4.2 Table 3 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Diapers, Baby Medical, size: «Mini», 12 / 44/ 76 pcs. per pack.

Table 3. Parameters of Giggles Diapers, Baby Medical, size: «Mini»

Sl. No.	Parameter	Value/ Description
1	SAP, g	4 ± 0.5
2	Fluffed pulp, g	9 ± 0.5
3	Product length, mm	375 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	315 ± 5
7	Width of absorbent core (across the top of the product), mm	105 ± 5
8	Width of absorbent core (in the centre), mm	90 ± 5
9	Dimensions of Velcro tapes (length $\times$ width), mm	$(50 \times 25) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	205 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Images of animals: bunnies; colours: yellow, violet. Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

6.4.3 Table 4 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Diapers, Baby Medical, size: «Midi», 11/40/68 pcs. per pack.

Table 4. Parameters of Giggles Diapers, Baby Medical, size: «Midi»

Sl. No.	Parameter	Value/ Description
1	SAP, g	6 ± 0.5

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Tablo 4. Giggles Medikal Bebek Bezi Parametreleri, boy: «Midi»

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Table 4. Parameters of Giggles Diapers, Baby Medical, size: «Midi»

Sl. No.	Parameter	Value/ Description
2	Fluffed pulp, g	10 ± 0.6
3	Product length, mm	430 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	370 ± 5
7	Width of absorbent core (across the top of the product), mm	110 ± 5
8	Width of absorbent core (in the centre), mm	90 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(50 × 30) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	255 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Images of animals: bunnies; colours: yellow, violet. Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

6.4.4 Table 5 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Diapers, Baby Medical, size: «Maxi», 10/36/60 pcs. per pack.

Table 5. Parameters of Giggles Diapers, Baby Medical, size: «Maxi»

Sl.No.	Parameter	Value/ Description
1	SAP, g	8 ± 0.5
2	Fluffed pulp, g	11 ± 0.7
3	Product length, mm	475 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	410 ± 5
7	Width of absorbent core (across the top of the product), mm	115 ± 5
8	Width of absorbent core (in the centre), mm	90 ± 5

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Table 5. Parameters of Giggles Diapers, Baby Medical, size: «Maxi»

Sl.No.	Parameter	Value/ Description
9	Dimensions of Velcro tapes (length × width), mm	$(50 \times 30) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	255 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Images of animals: bunnies; colours: yellow, violet. Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

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Table 6. Parameters of Giggles Diapers, Baby Medical, size: «Junior»

Sl.No.	Parameter	Value/ Description
1	SAP, g	10.5 ± 0.5
2	Fluffed pulp, g	13 ± 0.7
3	Product length, mm	510 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	450 ± 5
7	Width of absorbent core (across the top of the product), mm	120 ± 5
8	Width of absorbent core (in the centre), mm	100 ± 5
9	Dimensions of Velcro tapes (length × width), mm	$(50 \times 30) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	285 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Images of animals: bunnies; colours: yellow, violet. Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.

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Table 6. Parameters of Giggles Diapers, Baby Medical, size: «Junior»

Sl.No.	Parameter	Value/ Description
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

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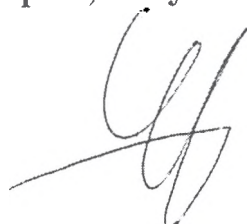
Giggles products contain no harmful substances that could affect the human health. Giggles Diapers, Baby Medical, are made of the following materials: hotmelt adhesive, polyethylene, polypropylene, nonwoven, polyester fiber, pulp, polyacrylate (SAP), elastic, green, violet, and yellow colourants. Photographic images of the appearance of Giggles Diapers, Baby Medical, are shown in Schedule 3. Photographic images of the appearance of Diapers, Baby Medical.

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Table 8. Parameters of Giggles Premium Diapers, Baby Medical, size: «New Baby»

Sl.No.	Parameter	Value/ Description
1	SAP, g	4.5 ± 0.5
2	Fluffed pulp, g	8 ± 0.5
3	Product length, mm	360 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	314 ± 5
7	Width of absorbent core (across the top of the product), mm	100 ± 5
8	Width of absorbent core (in the centre), mm	85 ± 5
9	Dimensions of Velcro tapes (length $\times$ width), mm	$(50 \times 25) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	205 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Images of animals: bunnies; colours: yellow, violet. Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	Instead of being part of a one-piece design, the front and stretchable rear side straps are glued thereto.

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Table 9. Parameters of Giggles Premium Diapers, Baby Medical, size: «Mini»

Sl.No.	Parameter	Value/ Description
1	SAP, g	4.5 ± 0.5
2	Fluffed pulp, g	8 ± 0.5
3	Product length, mm	375 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	315 ± 5
7	Width of absorbent core (across the top of the product), mm	100 ± 5

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Table 9. Parameters of Giggles Premium Diapers, Baby Medical, size: «Mini»

Sl.No.	Parameter	Value/ Description
	mm	
8	Width of absorbent core (in the centre), mm	85 ± 5
9	Dimensions of Velcro tapes (length $\times$ width), mm	$(50 \times 25) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	175 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Diaper No. (2); colour: violet. Decorative frame around the symbol of the product type and no; colour: pink. Images of hearts of different size; colours: yellow, violet, pink. Geometric pattern along the sides; colours: yellow, violet, pink. Diamond pattern formed with dots of different size, placed over the product surface; colours: violet. Images of animals: big and small bunnies; colours: yellow, violet, pink. Rectangle pattern along the top and bottom of the diaper; colours: violet, yellow. Pantones of dyes used: violet Pantone 2587C, yellow Pantone 122C, pink - Pantone 204C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	Instead of being part of a one-piece design, the front and stretchable rear side straps are glued thereto.

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Table 10. Parameters of Giggles Premium Diapers, Baby Medical, size: «Midi»

Sl.No.	Parameter	Value/ Description
1	SAP, g	6.5 ± 0.5
2	Fluffed pulp, g	9 ± 0.6
3	Product length, mm	430 ± 5
4	Product width (across the top of the back-panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	370 ± 5
7	Width of absorbent core (across the top of the product), mm	105 ± 5

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Table 10. Parameters of Giggles Premium Diapers, Baby Medical, size: «Midi»

Sl.No.	Parameter	Value/ Description
8	Width of absorbent core (in the centre), mm	90 ± 5
9	Dimensions of Velcro tapes (length $\times$ width), mm	$(50 \times 25) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	175 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Diaper No. (3); colour: violet. Decorative frame around the symbol of the product type and no; colour: pink. Images of hearts of different size; colours: yellow, violet, pink. Geometric pattern along the sides; colours: yellow, violet, pink. Diamond pattern formed with dots of different size, placed over the product surface; colours: violet. Images of animals: big and small bunnies; colours: yellow, violet, pink. Rectangle pattern along the top and bottom of the diaper; colours: violet, yellow. Pantones of dyes used: violet Pantone 2587C, yellow Pantone 122C, pink - Pantone 204C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	Instead of being part of a one-piece design, the front and stretchable rear side straps are glued thereto.

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Table 11. Parameters of Giggles Premium Diapers, Baby Medical, size: «Maxi»

Sl.No.	Parameter	Value/ Description
1	SAP, g	9 ± 0.5
2	Fluffed pulp, g	10 ± 0.7
3	Product length, mm	475 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	410 ± 5
7	Width of absorbent core (across the top of the product), mm	110 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5

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Table 11. Parameters of Giggles Premium Diapers, Baby Medical, size: «Maxi»

Sl.No.	Parameter	Value/ Description
9	Dimensions of Velcro tapes (length × width), mm	$(50 \times 30) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	175 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles; colour: violet. Diaper No. (4), colour: violet. Decorative frame around the symbol of the product type and no; colour: pink. Images of hearts of different size; colours: yellow, violet, pink. Geometric pattern along the sides; colours: yellow, violet, pink. Diamond pattern formed with dots of different size, placed over the product surface; colours: violet. Images of animals: big and small bunnies; colours: yellow, violet, pink. Rectangle pattern along the top and bottom of the diaper; colours: violet, yellow. Pantones of dyes used: violet Pantone 2587C, yellow Pantone 122C, pink - Pantone 204C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	Instead of being part of a one-piece design, the front and stretchable rear side straps are glued thereto.

6.5.5 Table 12 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Giggles Premium Diapers, Baby Medical, size: «Junior», 8/25/36 pcs. per pack.

Table 12. Parameters of Giggles Premium Diapers, Baby Medical, size: «Junior»

Sl.No.	Parameter	Value/ Description
1	SAP, g	10.5 ± 0.5
2	Fluffed pulp, g	13 ± 0.7
3	Product length, mm	510 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	450 ± 5
7	Width of absorbent core (across the top of the product), mm	120 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5

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Table 12. Parameters of Giggles Premium Diapers, Baby Medical, size: «Junior»

Sl.No.	Parameter	Value/ Description
9	Dimensions of Velcro tapes (length × width), mm	$(50 \times 30) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	175 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Giggles: colour: violet. Diaper No. (5), colour: violet. Decorative frame around the symbol of the product type and no: colour: pink. Images of hearts of different size: colours: yellow, violet, pink. Geometric pattern along the sides: colours: yellow, violet, pink. Diamond pattern formed with dots of different size, placed over the product surface: colours: violet. Images of animals: big and small bunnies: colours: yellow, violet, pink. Rectangle pattern along the top and bottom of the diaper: colours: violet, yellow. Pantones of dyes used: violet Pantone 2587C, yellow Pantone 122C, pink - Pantone 204C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	Instead of being part of a one-piece design, the front and stretchable rear side straps are glued thereto.

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Table 13. Parameters of Giggles Diapers, Baby Medical, size: «Junior+»

Sl.No.	Parameter	Value/ Description
1	SAP, g	11 ± 0.5
2	Fluffed pulp, g	15 ± 0.7
3	Product length, mm	550 ± 5
4	Product width (across the top of the back panel including side straps), mm	295 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	510 ± 5
7	Width of absorbent core (across the top of the product), mm	120 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5
9	Dimensions of Velcro tapes (length × width), mm	$(50 \times 30) \pm 5$

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Table 13. Parameters of Giggles Diapers, Baby Medical, size: «Junior+»

Sl.No.	Parameter	Value/ Description
10	Side barrier (length of elasticized tapes, non-stretched), mm	285 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: – Style of printed lettering: Giggles; colour: violet. – Images of animals: bunnies; colours: yellow, violet. – Images of hearts which some bunnies hold in their paws; colour: violet. Pantones of dyes used: violet Pantone 269C, yellow Pantone 122C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	Instead of being part of a one-piece design, the front and stretchable rear side straps are glued thereto.

Giggles Premium products contain no harmful substances that could affect the human health. Giggles Premium Diapers, Baby Medical, contain hotmelt adhesive, polyethylene, polypropylene, nonwoven, polyester fiber, pulp, polyacrylate (SAP), elastic, green, violet, yellow, and pink colourants.

Photographic images of the appearance of Giggles Premium Diapers, Baby Medical, are shown in Schedule 3. Photographic images of the appearance of Diapers, Baby Medical.

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Table 14. Parameters of Jusy Diapers, Baby Medical, size: «New Baby»

Sl.No.	Parameter	Value/ Description
1	SAP, g	3 ± 0.5
2	Fluffed pulp, g	10 ± 0.7
3	Product length, mm	360 ± 5
4	Product width (across the top of the back panel including side straps), mm	260 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	305 ± 5
7	Width of absorbent core (across the top of the product), mm	100 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(60 × 25) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	205 ± 5

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Table 14. Parameters of Jusy Diapers, Baby Medical, size: «New Baby»

Sl.No.	Parameter	Value/ Description
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Jusy: colours: violet, green. Printed lettering: Baby: colours: violet, green. Images of small birds: colours: green, violet. Images of a teddy bear sleeping on a crescent moon over a cloud: colours: green, violet. Geometric pattern along the sides: colours: green, violet. Geometric pattern - shaded circle: colour: violet. Pantones of dyes used: violet Pantone 266C, yellow Pantone 375C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

6.6.2 Table 15 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Jusy Diapers, Baby Medical, size: «Mini», 12/40/44 pcs. per pack.

Table 15. Parameters of Jusy Diapers, Baby Medical, size: «Mini»

Sl.No.	Parameter	Value/ Description
1	SAP, g	3 ± 0.5
2	Fluffed pulp, g	10 ± 0.7
3	Product length, mm	375 ± 5
4	Product width (across the top of the back panel including side straps), mm	260 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	305 ± 5
7	Width of absorbent core (across the top of the product), mm	100 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5
9	Dimensions of Velcro tapes (length × width), mm	$(60 \times 25) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	205 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Jusy: colours: violet, green. Printed lettering: Baby: colours: violet, green. Images of small birds: colours: green, violet.

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6.6.3 Table 16 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Jusy Diapers, Baby Medical, size: «Midi», 11/40 pcs. per pack.

Table 16. Parameters of Jusy Diapers, Baby Medical, size: «Midi»

Sl.No.	Parameter	Value/ Description
1	SAP, g	5 ± 0.5
2	Fluffed pulp, g	11 ± 0.6
3	Product length, mm	430 ± 5
4	Product width (across the top of the back panel including side straps), mm	310 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	370 ± 5
7	Width of absorbent core (across the top of the product), mm	100 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5
9	Dimensions of Velcro tapes (length $\times$ width), mm	$(60 \times 25) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	255 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Jusy: colours: violet, green. Printed lettering: Baby: colours: violet, green. Images of small birds: colours: green, violet. Images of a teddy bear sleeping on a crescent moon over a cloud: colour: green, violet. Geometric pattern along the sides: colours:green, violet. Geometric pattern - shaded circle: colour: violet. Pantones of dyes used: violet Pantone 266C, yellow Pantone 375C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece

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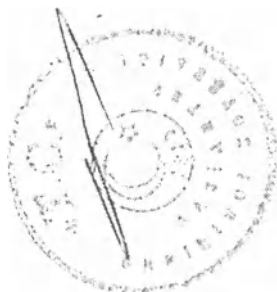
6.6.4 Table 17 below shows the linear dimensions, details of design, and structural features (hereinafter - "parameters") of Jusy Diapers, Baby Medical, size: «Maxi», 10/36/40 pcs. per pack.

Table 17. Parameters of Jusy Diapers, Baby Medical, size: «Maxi»

Sl.No.	Parameter	Value/ Description
1	SAP, g	6 ± 0.5
2	Fluffed pulp, g	12 ± 0.7
3	Product length, mm	475 ± 5
4	Product width (across the top of the back panel including side straps), mm	310 ± 5
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	410 ± 5
7	Width of absorbent core (across the top of the back panel), mm	120 ± 5
8	Width of absorbent core (in the centre), mm	95 ± 5
9	Dimensions of Velcro tapes (length x width), mm	$(60 \times 25) \pm 5$
10	Side barrier (length of elasticized tapes, non-stretched), mm	255 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: – Style of printed lettering: Jusy; colours: violet, green. – Printed lettering: Baby; colours: violet, green. – Images of small birds; colours: green, violet. – Images of a teddy bear sleeping on a crescent moon over a cloud: colour: green, violet. – Geometric pattern along the sides: colours: green, violet. – Geometric pattern - shaded circle: colour: violet. Pantones of dyes used: violet Pantone 266C, yellow Pantone 375C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

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Table 18. Parameters of Jusy Diapers, Baby Medical, size: «Junior»

Sl.No.	Parameter	Value/ Description
	including side straps), mm	
5	Product width (in the centre), mm	225 ± 5
6	Length of absorbent core, mm	450 ± 5
7	Width of absorbent core (across the top of the back panel), mm	120 ± 5
8	Width of absorbent core (in the centre), mm	100 ± 5
9	Dimensions of Velcro tapes (length × width), mm	(60 × 25) ± 5
10	Side barrier (length of elasticized tapes, non-stretched), mm	285 ± 5
11	Design of protective layer	Printed on the white background of the protective layer: Style of printed lettering: Jusy: colours: violet, green. Printed lettering: Baby: colours: violet, green. Images of small birds: colours: green, violet. Images of a teddy bear sleeping on a crescent moon over a cloud: colour: green, violet. Geometric pattern along the sides: colours: green, violet. Geometric pattern - shaded circle: colour: violet. Pantones of dyes used: violet Pantone 266C, yellow Pantone 375C.
12	Design of frontal tape	Transparent
13	Configuration of side straps	The front and rear side straps are part of one-piece design.

Jusy products contain no harmful substances that could affect the human health. Jusy Diapers, Baby Medical, contain hotmelt adhesive, polyethylene, polypropylene, nonwoven, polyester fiber, pulp, polyacrylate (SAP), elastic, green, and violet colourants.

A photographic image of the appearance of Jusy Diapers, Baby Medical, is shown in Schedule 3
Photographic image of the appearance of Diapers, Baby Medical.

7. Diapers, Baby Medical. Quality Parameters

Since manufacturers of the disposable hygiene products are obliged to care about the health and well being of the users, Sevinçler Sağlık Ürünleri is committed to the strict adherence of the following microbiological standards as shown in p. 7.1 - Microbiological Safety Parameters and p. 7.2 Organoleptic, Toxicological, Chemical Safety Parameters.

7.1 Microbiological Safety Parameters

Table 19 shows the microbiological safety parameters of Diapers, Baby Medical, manufactured by Sevinçler Sağlık Ürünleri San. ve Tic. A. S.

For the details of the microbiological safety test methods used to obtain the results, see p. 13.8.1

Table 19. Microbiological Safety Parameters of Diapers, Baby Medical, Manufacturer: Sevinçler Sağlık Ürünleri San. ve Tic. A. S.

Description	Value
Total Microbial Count, CFU	Max. 1 × 10 ⁴
Yeast-like moulds, per g	None
Bile-tolerant bacteria, per g	None

Table 18. Jusy Medikal Bebek Bezlerinin Parametreleri, boy: «Junior»

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7.2 Organoleptic, Toxicological, Chemical Safety Parameters

Diapers, Baby Medical, Manufacturer: Sevincler Sağlık Urunleri San. ve Tic., meet all safety standards as given in Table 20.

For the details of the safety test methods, used to obtain the results below, see p. 13.8.2 – p. 13.8.4.

Table 20. Organoleptic, Toxicological, and Chemical Safety Parameters of Diapers, Baby Medical, Manufacturer: Sevincler Sağlık Urunleri San. ve Tic. A. S.

Description	Value
Odour intensity from a diaper (aqueous extract), score, max.	7
Toxicity index, %, inclusive	70 – 120
pH value change, max	± 1.0

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7.4 Giggles Premium Diapers, Baby Medical. Quality Parameters

Table 22 shows the functional parameters that ensure the proper performance of Giggles Premium Diapers, Baby Medical.

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Table 22. Giggles Premium Diapers, Baby Medical. Quality Parameters

Description	Value					
Diaper weight, g	18 ± 3	19 ± 3	23 ± 3	27 ± 3	32 ± 3	35 ± 3
Full moisture absorption, g, min	120.0	160.0	240.0	270.0	300.0	330.0
Absorbing time, s, max.	3.0	3.0	3.0	3.0	3.0	3.0
Resorption, g, max.	0.4	0.4	6	14	16	16

7.5 Jusy Diapers, Baby Medical. Quality Parameters

Table 23 shows the functional parameters that ensure the proper performance of Jusy Diapers, Baby Medical.

For the details of the functional test methods used to obtain the results below, see p. 13.4 – 13.7.

Table 23. Jusy Diapers, Baby Medical. Quality Parameters

Description	Value				
Diaper Type and Size	Jusy «New Baby» (1)	Jusy «Mini» (2)	Jusy Midi (3)	Jusy «Maxi» (4)	Jusy «Junior» (5)
Diaper weight, g	18 ± 3	19 ± 3	23 ± 3	27 ± 3	32 ± 3
Full moisture absorption, g, min	120.0	160.0	240.0	270.0	280.0
Absorbing time, s, max.	3.0	3.0	3.0	3.0	3.0
Resorption, g, max.	0.4	0.4	6	14	16

7.6 No in-use ink set off is allowed if the label is printed on the product.

8. Labelling

8.1 Retail Package

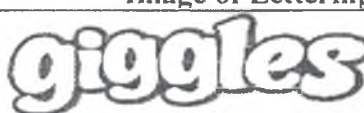
Package labelling of Diapers, Baby Medical, manufactured by Sevinçler Sağlık Ürünleri San. ve Tic. A. S. must be accurate, verifiable, and legible. Labels must be printed on the package, no ink set off is allowed.

Each retail package of the diapers is labelled with the following information:

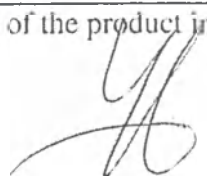
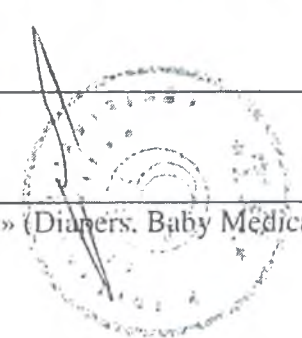
1) Front and rear views:

– Type of diaper in English. Images of letterings for different types are shown in Table 24.

Table 24. Images of Letterings for Different Types of Diapers

Sl. No.	Diaper Type	Image of Lettering to be included into Label
1	Giggles Diapers	 or 
2	Giggles Premium Diapers	
3	Jusy Diapers	

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Tablo 24. Resim bez cesitleri

Sl. No.	Diaper Type	Image of Lettering to be included into Label
1	Giggles bebek bezi	 veya 
2	Giggles Premium bebek bezi	
3	Jusy bebek bezi	

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Photographic image of a mother with a child; photographic image of a child for products size Junior+;
View of configuration of a product made as an image, see Fig. 3.

Fig. 3. Image of the Product Configuration

Symbol of the age group (size), child's weight range, diaper No., number of diapers per pack:

2) Top view:

- Type of diaper in English, images of letterings for different types are shown in Table 24.
- Description of the product in Russian: «Подгузники детские медицинские» (Diapers, Baby Medical);
- image of an arrow and text: "Open here" to indicate the recommended package opening spot;
- only for Giggles products: symbol of the age group (size), child's weight range, diaper No., number of diapers per pack; image of a mother with a child;

3) View from below:

- No special label is provided, as this part is reserved for seams;
- however, the labelling from the front and/ or the back of the package may continue into the bottom part of the package;

4) Left-side view 1 (to the right from the front view):

- List of ingredients;
- Batch no. containing the date of manufacture (DD/MM/YY);
- Expiry date;
- Manufacturer's logo:

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- Details of Product Manufacturer (name in English and Russian; Manufacturer's address in English including the country in English and Russia, tel. No., e-mail address, website, customer service tel. No.);

- Bar code;

- Manufacturer's country: text «Изготовлено в Турции» (Made in Turkey);

5) Left-side view 2 (to the right from the rear view):

- List of all sizes available for the product type in the form of a coloured chart (see Table 25). The image of the products included in the package is highlighted with a different colour.

Table 25. Image of the coloured size chart

Sl. No.	Diaper Type	Image of the coloured chart labelled on the retail package.
1	Giggles Diapers	

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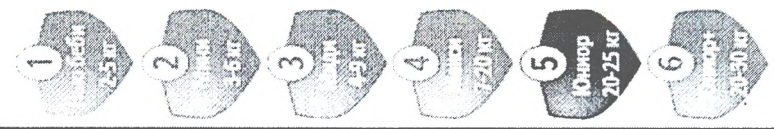
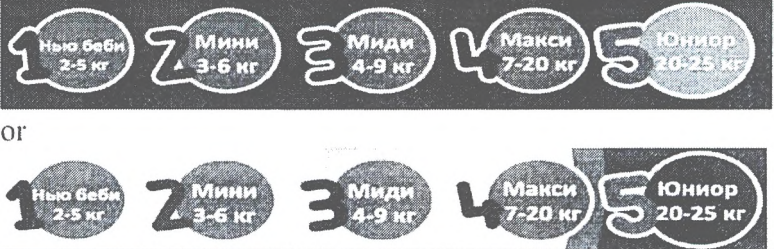
Sl. No.	Diaper Type	Image of the coloured chart labelled on the retail package.
2	Giggles Premium Diapers	
3	Jusy Diapers	

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- Instructions for use (diagrams and text):

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- Symbol "Do not re-use":
- Symbol "No litter":
- Symbol "Do not deposit into toilets":
- Symbol "Keep away from sunlight":
- Symbol "Keep dry":
- Symbol "Storage temperature range":
- Details of dealership in Russia (name, address, tel. No.):
- - Details of Registration in Russia: certificate No./ date of issue:

8) Additional graphic images required to clarify the label info:

- For Giggles Premium products, the front and rear parts of the package labelling contain a picture of stretchable side straps included in the product. See the scheme in Fig. 4.

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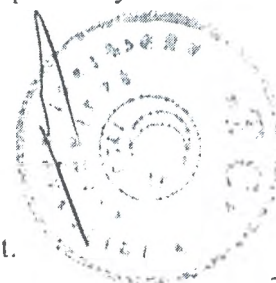
The labels must include the following handling-symbols:

- Symbol "Keep away from sunlight":
- Symbol "Keep dry".

The main signs must contain:

- Consignee's full or conventional name registered as appropriate;
- Destination including station or port of transit, where necessary;
- Serial no. of the unit within shipment/ total number of package units per shipment.

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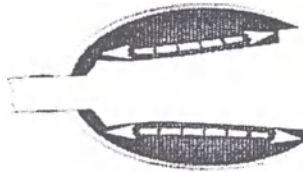
The explanatory text must contain the information included in the retail package labelling: Name of product (text in Russian: «подгузники детские медицинские» (Diapers, Baby Medical)), Product type (in English, must be in accordance with the lettering shown in Table 24), Symbol of age group (size), Number of products per pack (pcs.), Batch no. containing the date of manufacture (DD/MM/YY), Expiry date, details of registration in Russia: certificate No./ date of issue, Manufacturer's name, Consignor's address, Manufacturer's country (text «Изготовлено в Турции» (Made in Turkey)).

9. Packaging

9.1 Retail Package

The products in the amount established by the Manufacturer are placed in low density polyethylene packs for protection during shipping and storage.

See Table 26 for information regarding the number of products per pack.

Retail package labelling must be in Russian.

Package must be free from mechanical damage that throws the product surface open.

Table 26. No. of Diapers, Baby Medical, per pack

Type	Size	Pcs. per pack	Type	Size	Pcs. per pack
Giggles	«New Baby»	12	Giggles Premium	«New Baby»	56
	«Mini»	12		«Mini»	12
		44			40
		76			58
	«Midi»	11		«Midi»	11
		40			36
		68			48
	«Maxi»	10		«Maxi»	10
		36			32
		60			44
	«Junior»	8		«Junior»	8
		28			25
		46			36
Jusy	«New Baby»	12	Jusy	«Midi»	11
		40			40
		44		«Maxi»	10
	«Mini»	12			36
		40			40
		44		«Junior»	8
					28

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10. Disposal Information

Products manufactured by Sevinçler Sağlık Ürünleri San. ve Tic. A. S., meet the following requirements: No external defects are allowed in the products including mechanical damage (broken edges, cuts, damaged fasteners, etc.), stains of various origin, inclusions visible to a naked eye. The layers are glued together using a high-performance adhesive (see Schedule I. Materials), to ensure a good bond between the diaper layers (seams). The seams must be continuous.

- The printed symbols on the diapers must be clear, without blurring or spaces. No surface lifting or ink set off is allowed.

- The composition of Diapers, Baby Medical, must be in accordance with the description given in p. 6.1

- Composition of Diapers, Baby Medical.

- The configuration and different types of the products must be as described in p. 6.2 – Diapers, Baby Medical, Types.

- The linear dimensions, design, and structural features of different types of diapers must be as required in p. 6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features. p. 6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features , p. 6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features.

- The products contain no toxic substances that could affect the human health.

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- Do not use the products having external defects or mechanical damage (broken edges, cuts, damaged fasteners, etc.), stains of various origin, inclusions visible to a naked eye.

- Do not use the products having surface lifting or ink set off.

- Do not use the products if the package is broken.

- Do not use the products with ink set off. The printed symbols on the diapers must be clear, without blurring or spaces.

- Do not re-use the products. These products are non-reusable.

- Adhere to the instruction for use.

- Change when necessary in order to avoid diaper rash.

- - Do not use after the expiry date.

- - Keep away from open flame.

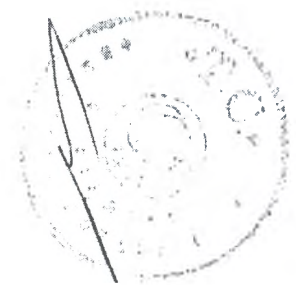
- Storage Temperature: +18°C - +25°C.

- Keep dry.

10.2 Disposal Information

- Discard after use.

- Depositing the used products into toilets is strictly prohibited.

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46

- After use, place the product into a container with a lid, to avoid spreading of bacteria and foul odour that can attract insects.
- Do not keep the used diapers at home for a long time.
- Use specialized anti-bacterial containers for the used diapers.

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In the manufacturing process, Sevinçler Sağlık Ürünleri San. ve Tic. A. S. does not use any water, so not water waste is generated. Besides, Sevinçler Sağlık Ürünleri San. ve Tic. A. S. does not generate any gaseous waste, therefore, there is no impact on the environment.

To preserve the environment, do not deposit the used products into toilets, use specialized waste containers instead.

12. Acceptance Criteria

12.1 The products are accepted for inspection provided the following conditions are met:

1. The package is intact, clean.
2. The product is not expired.

A batch is a specified amount of diapers of the same age group, composition, linear dimensions, design and style, made of the same materials and supported by the same quality certificate.

Every batch must come with a quality certificate containing the following details:

- Country of origin;
- Manufacturer's name;
- Product description;
- Age group (size), diaper No.;
- Batch no. containing the date of manufacture (DD/MM/YY);
- No. of products per batch;
- Expiry date.

12.2 The products must pass the following acceptance tests

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The tests are performed in accordance with the Quality Control System in place at Sevinçler Sağlık Ürünleri San. ve Tic. A. S., and the following quality parameters are checked:

- 1) p. 6.3 - Appearance of Diapers, Baby Medical;
- 2) p. 7.3 – Giggles Diapers, Baby Medical, Quality Parameters
or
p. 7.4 – Giggles Premium Diapers, Baby Medical, Quality Parameters
or
p. 7.5 – Jusy Diapers, Baby Medical, Quality Parameters Jusy
(depending of the product type);
- 3) p. 8.1 – Retail Package Labelling.

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Where the tests results do not meet the specification, even for one parameters, the test must be re-taken using a doubled amount of products from the same batch.

The results of such re-tests are final and applicable to the entire batch.

13. Test Methods

The Quality Control System in place at Sevinçler Sağlık Ürünleri San. ve Tic. A. S. includes the following stages:

13.1 In-line Quality Control

The quality control system in place at the Manufacturer's automatically detects and rejects the defective products. The fully automated manufacturing line includes an automatic check of presence of all parts of each diaper. If the product is found defective, the machine places the damaged diaper into a separate chamber to prevent the substandard products from getting into the standard batch.

13.2 Pre-packaging Inspection

To check the machine at the pre-packaging stage, the quality inspectors manually test the diapers every day of operation of the manufacturing line, picking the items out of the batch at random to check the appearance, linear dimensions, raw materials specifics, composition of the product, etc. After testing the item, the quality inspector picks another one.

13.3 Visual Inspection at Lab

The diapers are tested in accordance with the requirements set forth in p. 6.3 - Appearance of Diapers, Baby Medical, p. 8.1 - Retail Package Labelling, p. 9.1. - Retail Package the quality inspectors at the laboratory visually check the products selected for the acceptance tests as contemplated in p. 12.1, p. 12.2 (sampling method).

13.4 Product Weighing Method

The method is based on weighting the products using analytical scales with the accuracy of max. ± 0.1 g. Five diapers out of the sample set taken as per p. 12.1, p. 12.2 (sampling method) are weighed.

The test result is calculated as the arithmetic mean of five measurements.

13.5 Determining the Full Moisture Absorption

The test methods determines the amount of 0.9% sodium chloride solution (at (23 ± 2) °C) absorbed by the product after it was immersed in the solution for (30 ± 0.5) minutes and then centrifuged for (264 ± 1) seconds.

The full moisture absorption is determined for six diapers out of the sample set taken as per p. 12.1, p. 12.2 (sampling method).

Three diapers of six are selected. Each of three is weighed with the accuracy max. ± 0.01 g, then all three diapers are simultaneously immersed in a tank with NaCl solution so that they are fully covered liquid. After they absorb the solution, the diapers are placed in individual gauze bags. The bags are transferred to a centrifugal separator. The diapers are uniformly spread on the bottom of the machine so that their top sheet faces the machine walls. Upon separation, each diaper is taken out of its bag and weighed with the accuracy max. ± 0.01 g.

The full moisture absorption B, g of the product can be found as follows:

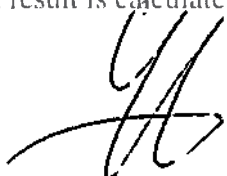
$$B = M_2 - M_1$$

where:

M_2 is weight of a wet diaper after centrifuging, g;

M_1 is weight of a dry diaper, g.

The test result is calculated as the arithmetic mean of six measurements.

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13.6 Determining the Absorption Time

The tests determines the time the products needs to fully absorb a specified amount of 0.9% sodium chloride solution applied on the product top sheet.

The full moisture absorption is determined for five diapers out of the sample set taken as per p. 12.1, p. 12.2 (sampling method).

The solution is poured through a funnel over a specified spot on the product. The solution must be at $(37 \pm 2) ^\circ\text{C}$.

Table 27 below specifies the amounts of 0.9% sodium chloride solution used to test different sizes of the products.

Table 27. Amounts of 0.9% sodium chloride solution used to test different sizes of the products

Description	Value					
Product	«New Baby» (1)	«Mini» (2)	«Midi» (3)	«Maxi» (4)	«Junior» (5)	«Junior+» (6)
Solution, cm^3	30 ± 1	30 ± 1	55 ± 2	80 ± 4	100 ± 5	100 ± 5

After the solution is poured, the stop-watch timer is turned on. This is the beginning of the test time. The test is over when is no solution is left on the diaper surface.

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13.7 Determining the Resorption

The test determines the amount of 0.9% solution separated from the product as a result of a weight load applied to it on the bottom backsheet side.

The resorption test is performed for five diapers out of the sample set taken as per p. 12.1, p. 12.2 (sampling method).

At least, five filters are placed in a pile, each pile is weighed with the accuracy within 0.01 g.

The product is stretched lengthwise until it is flat, the top sheet must be on top. A spot for the solution supply is chosen in the center of the absorbent core and marked with a felt-tip marker.

A specified amount of solution (see Table 29) is poured through a funnel over the chosen spot. The solution must be at $(37 \pm 2) ^\circ\text{C}$. The solution supply time per portion is 25-30 seconds.

After a portion of the solution is poured, the stop-watch timer is turned on. This is the beginning of the pause. The duration and number of pauses are given in Table 28.

Table 28. Amounts of 0.9% sodium chloride solution used to test different sizes of the products Duration and number of pauses.

Description	Value					
Product	«New Baby» (1)	«Mini» (2)	«Midi» (3)	«Maxi» (4)	«Junior» (5)	«Junior+» (6)
Portion 1 of solution, cm^3	30 ± 1	30 ± 1	55 ± 2	80 ± 4	100 ± 5	100 ± 5
Pause, s	300 ± 6	300 ± 6	300 ± 6	300 ± 6	300 ± 6	300 ± 6
Portion 2 of solution, cm^3	30 ± 1	30 ± 1	55 ± 2	80 ± 4	100 ± 5	100 ± 5
Pause, s	300 ± 6	300 ± 6	300 ± 6	300 ± 6	300 ± 6	300 ± 6
Portion 3 of solution, cm^3	30 ± 1	30 ± 1	55 ± 2	80 ± 4	100 ± 5	100 ± 5
Pause, s	300 ± 6	300 ± 6	300 ± 6	300 ± 6	300 ± 6	300 ± 6

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After the last pause, the product is turned over and placed with the top sheet on a pile of filters so that the solution supply spot touches the filters and the barrier elements are kept away from the top sheet. A (2000 ± 50) g weight is applied to the solution supply spot and left there for (15 ± 1) s. Then the pile of filters is weight.

The resorption «OC» , g, of the product can be found as follows:

$$OC = M_2 - M_1$$

where:

M_2 is weight of a pile of filters upon completion of the test, g;

M_1 is weight of a pile of filters before the test, g;

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13.8 Safety Parameters

The following safety parameters of the products are tested at the quality control center:

13.8.1 Microbiological Quality Control

Tests are performed to identify the total bacteria count and the dangerous / opportunistic pathogenic bacteria count of test specimens.

The tests are based on standard microbial culture techniques including inoculation of the product, dilution of the weighed amount in culture media followed by incubation under microbial growth conditions. Computer-aided rapid microbiological methods can be used.

Sampling

1% of packages of a batch are selected at random, and at least 30 pcs. products are taken out of this sample for purpose of acceptance tests.

Sterile instruments are used to prepare microbial test specimens.

Specimens are from the central part of the products are placed in a sterile vessel, mixed, and a representative test specimen is selected. The test specimen is weighed under sterile conditions

Test procedure

Total bacteria count: Using sterile instrument transfer 10 g of test specimen to 100 ml of phosphate-buffered sodium chloride peptone buffer solution and glass balls, shake carefully trying to avoid foaming, for 10 minutes.

Inoculation: agar plate; inoculated volume - 0.1 ml, culture media - QMA&OAMO.

Incubation: 14 days at 37 °C.

Results: Plate count - CFU/ plate, total bacteria count per g of test specimen - CFU/ plate x100

Microbial limits must be as follows: – 10^2 CFU; max TBC = 200.

Yeasts and moulds: Using sterile instruments select and transfer 1 g of test specimen in 100 ml of Sabouraud broth.

Inoculation: direct immersion in Sabouraud broth.

Incubation: 14 days at 25 °C.

Results:

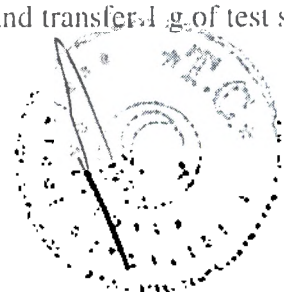
Attribute

No viable yeasts/ moulds are found in 1 g of test specimen: no visual growth (turbidity, sediment, film, coat) observed upon completion of incubation.

Viable colonies of yeasts/ moulds are found 1 g of test specimen: attributes of growth (turbidity, sediment, film, coat) observed

Enteric bacteria per 1 g of test specimen: Using sterile instruments select and transfer 1 g of test specimen in 100 ml of EC-broth.

Inoculation: direct immersion in EC-broth.

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Results:

Attribute

No enteric bacteria are found in 1 g of test specimen: no visual growth (diffusive turbidity, gas) observed upon completion of incubation.

Enteric bacteria are found in 1 g of test specimen: visual growth (diffusive turbidity, gas) observed upon completion of incubation.

Pathogenic staphylococci per 1 g of test specimen: Using sterile instruments select and transfer 1 g of test specimen in 100 ml of salt selective broth for staphylococci.

Inoculation: two-stage inoculation. Stage 1 - inoculation in broth, stage 2 - inoculation from broth to agar; inoculated volume for agar plate inoculation - 0.1 ml; culture media: salt selective broth for staphylococci, mannitol salt selective agar for staphylococci.

Incubation: broth - 72 hours at 37 °C; agar - 72 hours at 37 °C.

Results:

Attribute

No viable pathogenic staphylococci are found in 1 g of test specimen: no coagulase-positive mannitol fermenting staphylococci observed.

Viable pathogenic staphylococci are found in 1 g of test specimen: coagulase-positive mannitol fermenting staphylococci observed.

Pseudomonas aeruginosa per 1 g of test specimen: Using sterile instruments select and transfer 1 g of test specimen in 100 ml of trypticase-soy broth.

Inoculation: two-stage inoculation. Stage 1 - inoculation in broth, stage 2 - inoculation from broth to agar; inoculated volume for agar plate inoculation - 0.1 ml; culture media: trypticase-soy broth, Pseudomonas agar for pyocyanin.

Incubation: broth - 72 hours at 37 °C; agar - 72 hours at 37 °C.

Results:

Attribute

No viable Pseudomonas aeruginosa are found in 1 g of test specimen: no growth observed on Pseudomonas agar for pyocyanin.

Viable Pseudomonas aeruginosa are found in 1 g of test specimen: bacterial growth observed on Pseudomonas agar for pyocyanin.

13.8.2 Organoleptic Parameters

Sensory properties of extracts from the products result from the transfer of the product ingredients. These are important safety parameters of the product. Sensory properties are tested using a blind method. Tests are performed by a group of at least five inspectors who are able to clearly perceive odours. The odour intensity is determined in a blind test using an aqueous extract (distilled water). The extract is prepared by hot extraction of a test specimen performed at $(40 \pm 2) ^\circ\text{C}$ for 24 hours, at room temperature, and after heating up to 60 °C. The odour intensity from the test specimen is scored using standard techniques.

13.8.3 Toxicological Parameters.

The toxicological parameters are tested using the toxicity index method. The toxicity index is determined for an aqueous extract (distilled water). The extract is prepared by hot extraction of a test specimen performed at $(40 \pm 2) ^\circ\text{C}$ for 24 hours. The toxicity index is an integral indicator of hazardous substance exposure. The acute toxicity of the product extract is tested using a cell test object.

13.8.4 Chemical Parameters

The chemical parameters are tested using a pH value measurement method. The pH value of an aqueous extract (distilled water) is measured and compared to the quality control sample. The extract is prepared

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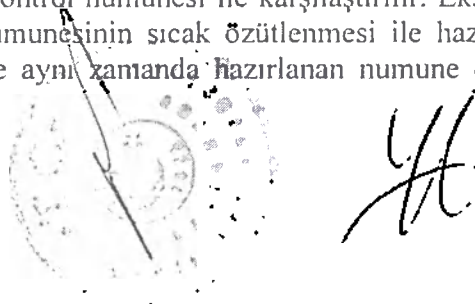
by hot extraction of a test specimen performed at $(40 \pm 2) ^\circ\text{C}$ for 24 hours. The quality control sample is distilled water without the specimen prepared at the same time as the extract from the test specimen. This integral indicator helps assess the total amount of migrants from the product. The pH values of the extracts for the product and QC samples is measured using potentiometers.

13.9 Determining the Ink Set off

The method consists in the visual control of the ink set off on the filter paper soaked in the solution at $(37 \pm 2) ^\circ\text{C}$ after it touches the prints on the product, and a $(2000 \pm 50)\text{g}$ weight load is applied for $(2.0 \pm 0.1)\text{h}$.

The ink set off is checked for five diapers out of the sample set taken as per p. 12.1, p. 12.2 (sampling method).

The product is considered fit if no traces of ink are found on the filter paper for three test specimens.

For details of specifications for these tests, see p. 7. Diapers, Baby Medical, Quality Parameters.

Therefore, the quality control department performs an important function in order to minimize the possibility of shipment of substandard, defective or failed products to the Customer.

14. Transportation, Storage, Disposal

14.1 Transport Rules

1. The transport must be clean, dry, without any odours.
2. The products can be shipped on or without pallets.
3. The temperature inside the transport must be from $-10 ^\circ\text{C}$ to $+25 ^\circ\text{C}$.

14.2 Storage Information:

Store the product in a dark well-ventilated place. Keep out of reach of children, away from heaters and flame. Protect against rain and moisture. At warehouses, store the products on pallets.

Storage Temperature: $+18 ^\circ\text{C}$ - $+25 ^\circ\text{C}$.

Discard after expiry date.

After opening, keep the package in dry places.

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15. Use

Diapers, Baby Medical. Instruction for Use.

1. Place a diaper underneath the baby so that the frontal tape is at the front, and the fasteners are at the back.
2. Make sure that the protective barriers fold inwards.
3. Secure the side tapes on the frontal tape using the fasteners, symmetrically on both sides.
4. You can open and close the diaper as many times as necessary.
5. After use, place the diaper in a specialized container. Do not flush.

16. Manufacturer's Warranty

Provided that they have been transported, stored, and used in a proper manner, the quality of the products will be in conformity with the Manufacturer's certificates for 4 (four) year from the date of manufacture..

17. Contacts

For any questions related to the quality of the medical device Diapers, Baby Medical, types:

– Giggles Diapers, Baby Medical, sizes: «New Baby», «Mini», «Midi», «Maxi», «Junior», «Junior+»

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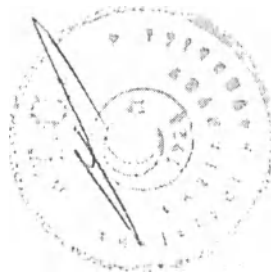
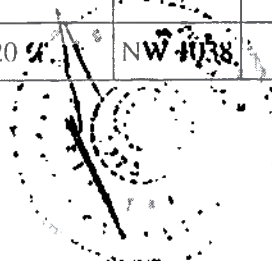
Sl.No.	Location	Description	Ingredients	Brand	Manufacturer
1	Top layer (1)	PP Spunbond Nonwoven	Light heat-bonded nonwoven fabric made of fine polypropylene threads (100% polypropylene), density (20°C) ~ 0.9 g/cm ³ (PP granules), melting point = 160°C, thermal destruction >300°C, colour: white (bleached CI 77891), no odour, not soluble in water.	GULSAN	GÜLSAN SENTETİK DOKUMA SANAYİ VE TİCARET A.Ş. Turkey
2	Adhesive to bond the layers	Hotmelt	Polymers 60 %, thermoplastic elastomer 20 %, mineral oil 17 %, additives 2.5 %, anti-oxidizing agent 0.5 %.	NW 1038 ZP	H.B. Fuller Deutschland GmbH/ Germany
3	Protective barriers against side leaks (2)	PP Spunbond Nonwoven	Light heat-bonded nonwoven fabric made of fine polypropylene threads (100% polypropylene), density (20°C) ~ 0.9 g/cm ³ (PP granules), melting point = 160°C, thermal destruction >300°C, colour: white (bleached CI 77891), no odour, not soluble in water.	GULSAN	GÜLSAN SENTETİK DOKUMA SANAYİ VE TİCARET A.Ş./Turkey
4	Adhesive to attach the elasticized tapes to the protective barriers	Hotmelt	Hydrogenated hydrocarbon resin 64%, thermoplastic rubber 15 %, paraffinic oils 15 %, resin-based alpha-methyl styrene 5.5 %, anti-oxidizing agent 0.5 %.	Lunatack D 7214 ZP	H.B. Fuller Deutschland GmbH/ Germany
5	Two elasticized bands arranged on the protective barrier	Spandex Fiber	Segmented polyurethane-polyurea > 95 %, original stabilizer max. 0.5%, residual solvent - dimethyl acetamide (CAS 127-19-5) < 0.6 %, matting material - Titanium dioxide (CAS 13463-67-7) max. 2%, lubricants - polymethylsiloxane (CAS 63148-62-9) max. 1%	Creora®	Hyosung Corporation/ Korea
6	Acquisition and distribution layer (ADL) (3)	Nonwoven Carded - Air Through Bonded	Nonwoven Carded - Air Through Bonded material made of polyester (PET). PET is a pure amorphous polymer, melting point = 250-260 °C, thermal destruction >280°C, flash point: = 390 - 410 °C, coloured with green dye (Pantone 375C) - CI 77288.	Akinal	Akinal Sentetik Tekstil San.ve Tic.A.Ş. / Turkey
7	Adhesive to bond the layers	Hotmelt	Polymers 60 %, thermoplastic elastomer 20 %, mineral oil 17 %, additives 2.5 %, anti-oxidizing agent 0.5 %.	NW 1038 ZP	H.B. Fuller Deutschland GmbH/ Germany
8	Top barrier sheet (4)	PP Spunbond Nonwoven	Light heat-bonded nonwoven fabric made of fine polypropylene threads (100% polypropylene), density (20°C) ~ 0.9 g/cm ³ (PP granules), melting point = 160°C, thermal destruction >300°C, colour: white (bleached CI 77891), no odour, not soluble in water.	GULSAN	GÜLSAN SENTETİK DOKUMA SANAYİ VE TİCARET A.Ş./Turkey
9	Adhesive to	Hotmelt	Polymers 60 %, thermoplastic elastomer 20 %	NW 1038 ZP	H.B. Fuller

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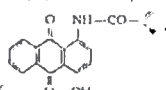
Sl.No.	Location	Description	Ingredients	Brand	Manufacturer
	bond the layers		mineral oil 17 %, additives 2.5 %, anti-oxidizing agent 0.5 %.	ZP	Deutschland GmbH/ Germany
10	Absorbent core (5)	Fluff Pulp	Raw fluffed pulp containing: cellulose (CAS 65996-61-4) 93%, water (CAS 7732-18-5) max. 7%	GP-S10	GP Cellulose GmbH Switzerland
		SAP (superabsorbent polymer)	Sodium polyacrylate (CAS 9003-04-7)	FAVOR SXM 9155	Evonik Nutrition & Care GmbH/ Germany
11	Adhesive to bond the layers	Hotmelt	Polymers 60 %, thermoplastic elastomer 20 %, mineral oil 17 %, additives 2.5 %, anti-oxidizing agent 0.5 %.	NW 1038 ZP	H.B. Fuller Deutschland GmbH/ Germany
12	Bottom backing sheet (6)	PP Spunbond Nonwoven	Light heat-bonded nonwoven fabric made of fine polypropylene threads (100% polypropylene), density (20°C) ~ 0.9 g/cm ³ (PP granules), melting point = 160°C, thermal destruction >300°C, colour: white (bleached CI 77891), no odour, not soluble in water.	GULSAN	GÜLSAN SENTETİK DOKUMA SANAYİ VE TİCARET A.Ş./ Turkey
13	Adhesive to bond the layers	Hotmelt	Polymers 60 %, thermoplastic elastomer 20 %, mineral oil 17 %, additives 2.5 %, anti-oxidizing agent 0.5 %.	NW 1038 ZP	H.B. Fuller Deutschland GmbH/ Germany
14	Protective layer (7)	Non breathable PE Film	Colour: white, no odour, melting point 111°C, thermal destruction 280°C, density(20°C) 0.924 g/cm ³ , not soluble in water. The images and lettering are applied to the outer surface using dyes designed for soft polyethylene, brand: Flexographic Printing Inks, Manufacturer: Sun Chemical Group/ UK. Ingredients: Pigment – (5-10) %, Butanon – (50-70) %, Ethanol – (10-30) %, 2-METHYLPROPAN-2-OL – < 1 %, Colourants (Pigments): Yellow – Nickel titanate (II) (NiTiO ₃); Purple - Cobalt ammonium phosphate (II) (CoNH ₄ PO ₄ ×H ₂ O); Pink – anthraquinone pink  (), Green - Chrome titanate (II) (CrTiO ₃).	GULSAN	GÜLSAN SENTETİK DOKUMA SANAYİ VE TİCARET A.Ş./ Turkey
15	Adhesive to bond the layers	Hotmelt	Polymers 60 %, thermoplastic elastomer 20 %, mineral oil 17 %, additives 2.5 %, anti-oxidizing agent 0.5 %.	NW 1038 ZP	H.B. Fuller Deutschland GmbH/

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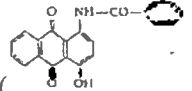
Sl.No.	Location	Description	Ingredients	Brand	Manufacturer
					Germany
16	Bottom backsheet (8)	pp Spunbond Nonwoven	Light heat-bonded nonwoven fabric made of fine polypropylene threads (100% polypropylene), density (20°C) ~ 0.9 g/cm ³ (PP granules), melting point = 160°C, thermal destruction >300°C, colour: white (bleached CI 77891), no odour, not soluble in water.	GULSAN	GÜLSAN SENTETİK DOKUMA SANAYİ VE TİCARET A.Ş. /Turkey
17	Adhesive to attach the elasticized tapes to the protective layer surface along the product sides	Hotmelt	Hydrogenated hydrocarbon resin 64%, thermoplastic rubber 15 %, paraffinic oils 15 %, resin-based alpha-methyl styrene 5.5%, anti-oxidizing agent 0.5%.	Lunatack D 7214 ZP	H.B. Fuller Deutschland GmbH/ Germany
18	Protective side bands - three elasticized bands arranged along the product sides (9)	Spandex Fiber	Segmented polyurethane-polyurea > 95 %, original stabilizer max. 0.5%, residual solvent - dimethyl acetamide (CAS 127-19-5) < 0.6 %, matting material - Titanium dioxide (CAS 13463-67-7) max. 2%, lubricants - polymethylsiloxane (CAS 63148-62-9) max. 1%	Creora®	Hyosung Corporation/ Korea
19	Adhesive to attach the side straps to the protective layer	Hotmelt	Hydrogenated hydrocarbon resin 64%, thermoplastic rubber 15 %, paraffinic oils 15 %, resin-based alpha-methyl styrene 5.5%, anti-oxidizing agent 0.5%.	Lunatack D 7214 ZP	H.B. Fuller Deutschland GmbH/ Germany
20	Side straps (1)	Side straps	Light heat-bonded nonwoven fabric made of fine polypropylene threads (100% polypropylene), density (20°C) ~ 0.9 g/cm ³ (PP granules), melting point = 160°C, thermal destruction >300°C, colour: white (bleached CI 77891), no odour, not soluble in water.	GULSAN	GÜLSAN SENTETİK DOKUMA SANAYİ VE TİCARET A.Ş./Turkey
21	Adhesive to attach the elastic side straps to the protective	Hotmelt	Polymers 60 %, thermoplastic elastomer 20 %, mineral oil 17 %, additives 2.5 %, anti-oxidizing agent 0.5 %.	NW 1038 ZP	H.B. Fuller Deutschland GmbH/ Germany

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Sl.No.	Location	Description	Ingredients	Brand	Manufacturer
	layer sandwiched between nonwoven				
22	Elastic side straps	Elastic Side straps	Ingredients: (40-50) Light heat-bonded nonwoven fabric made of fine polypropylene threads (100% polypropylene), density (20°C) ~ 0.9 g/cm³ (PP granules), melting point = 160°C, thermal destruction >300°C, colour: white (bleached CI 77891), no odour, not soluble in water. (40-50) % – Elastic Film: white elastic film made of thermoplastic elastomer semi-crystalline polyolefins (polyethylene, polypropylene, their combinations and copolymers), ethylene content = (60 – 90) % mol. α-olefin content ~ 1% mol.; brand Elastic Film Manufacturer: Mondi Gronau, Germany. (5-15) % - Hotmelt/adhesive to attach nonwoven to the elastic film surface: Hydrogenated hydrocarbon resin 64 %, thermoplastic rubber 15 %, paraffinic oils 15 %, resin-based alpha-methyl styrene 5.5 %, anti-oxidizing agent 0.5 %.	Nitto bento	Nitto Bento Bantçılık San. VE TIC. Inc., Turkey
23	Adhesive to bond between the side tapes with fasteners and side straps	Hotmelt	Hydrogenated hydrocarbon resin 64%, thermoplastic rubber 15%, paraffinic oils 15%, resin-based alpha-methyl styrene 5.5%, anti-oxidizing agent 0.5%.	Lunatack D 7214 ZP	H.B. Fuller Deutschland GmbH/ Germany
24	Velcro tapes. (10)	NW CLOSURE TAPES (NW (Y-bond)/	Ingredients: 20% - Light heat-bonded nonwoven fabric made of fine polypropylene threads (100% polypropylene), density (20°C) ~ 0.9 g/cm³ (PP granules), melting point = 160°C, thermal destruction >300°C, colour: white (bleached CI 77891), no odour, not soluble in water. 35% – BOPP Film (<u>Biaxially Oriented Polypropylene</u>)/brand: BOPP Film. Manufacturer: POLİNAS PLASTİK San. ve Tic. A.Ş./ Turkey 35 % – Pressure Sensitive Hot Melt adhesive – water-based polymer adhesive that bonds with a surface due to the pressure, forms a clear transparent film, viscosity: 100.0 - 1 000.0 mPa.s at 25°C, consists of acrylic copolymer ~ 45.8 %.	Nitto	NITTO BENTO BANTÇILIK SAN. ve TİC.A.Ş/ Turkey

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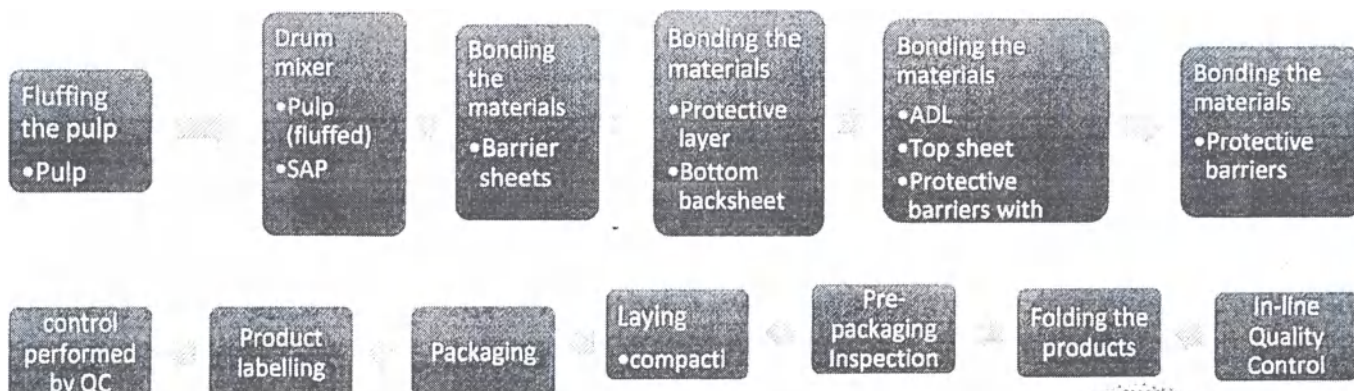
Sl.No.	Location	Description	Ingredients	Brand	Manufacturer
			emulsifier ~ 2.4 %, water ~ 50 %, neutralizing agent ~ 1.8 %; brand: Pressure Sensitive Hot Melt adhesive Nitto Bento. Manufacturer: NITTO BENTO BANTÇILIK SAN. ve TİC.A.Ş./Turkey. 10% – PP Hook-system consisting of a hook itself (PP 100 %) and substrate (PES non-woven (polyamid)); brand PP Hook. Manufacturer: Gottlieb Binder / Germany.		
25	Adhesive to bond the frontal tape to the top sheet	Hotmelt	Hydrogenated hydrocarbon resin 64%, thermoplastic rubber 15%, paraffinic oils 15%, resin-based alpha-methyl styrene 5.5%, anti-oxidizing agent 0.5%.	Lunatack D 7214 ZP	H.B. Fuller Deutschland GmbH/ Germany
26	Frontal tape	Mechanical Frontal Tapes	Ingredients:(45-50) % – LD polyethylene: density, g/cm -30.900-0.939; melting point, °C – (103-110); ultimate flexural strength, Pa (kgf/cm ²) –((117.6-196.07)· 105 (120-200)); shear strength, Pa (kgf/cm ²) – ((137.2-166.6)· 105 (140-170)); surface resistivity, Ohm -1015; 30-day water absorption, % - 0.02; dielectric constant at 1,010 Hz - (2.25-2.31); brittle point, max. °C – (from -100 to -120); thermal conductivity, W/(m·°C) –(0.36 – 0.40).(40-50) % – PES Loop system consisting of loops - PP 100%; substrate - PES non-woven (polyamide). < 1 % - PU Adhesive to bond the PES loop substrate with LD-polyethylene: mono polyurethane resin solution. brand: PU Adhesive. Manufacturer: COIM / Italy.	GÜLSAN	GÜLSAN SENTETİK DOKUMA SANAYİ VE TİCARET A.Ş./Turkey

19. Schedule 2 Manufacturing Process Flow Chart

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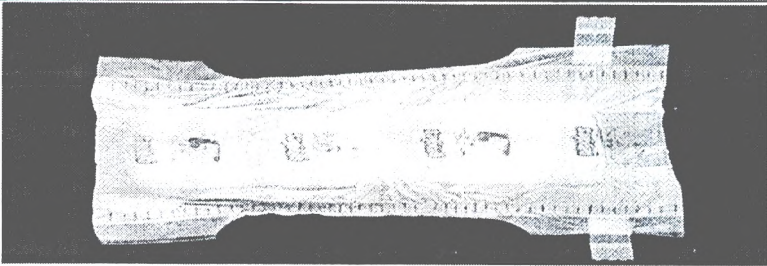
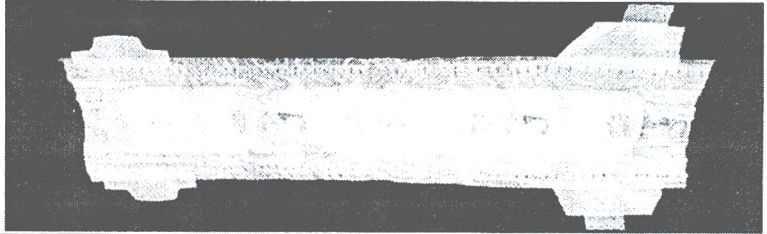
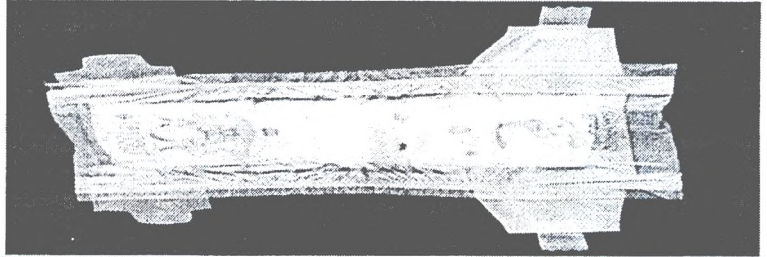
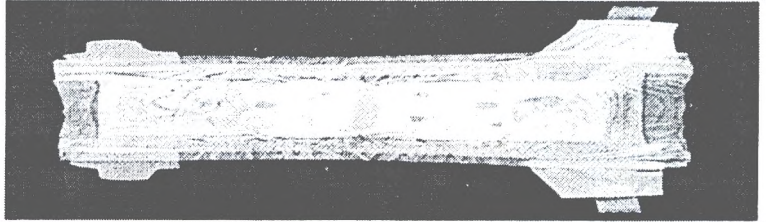
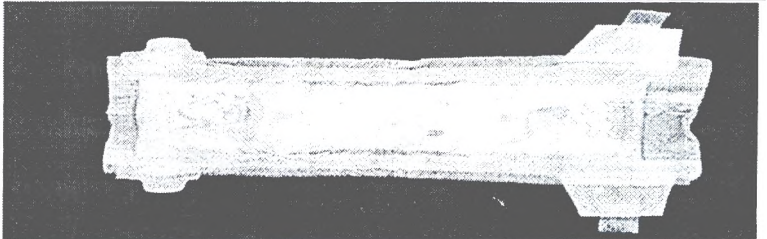
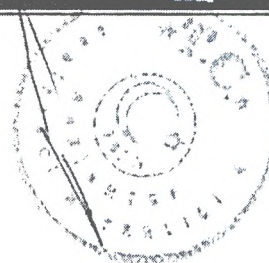
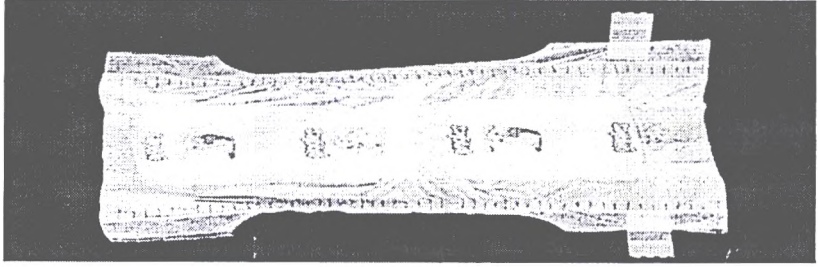
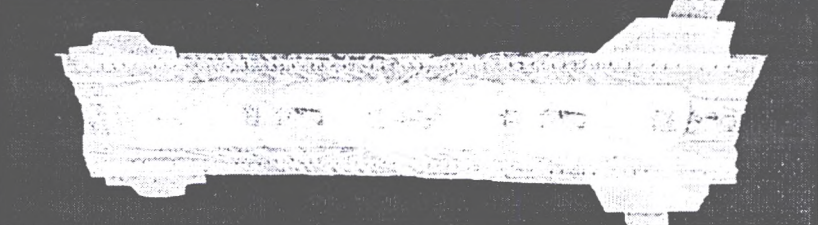
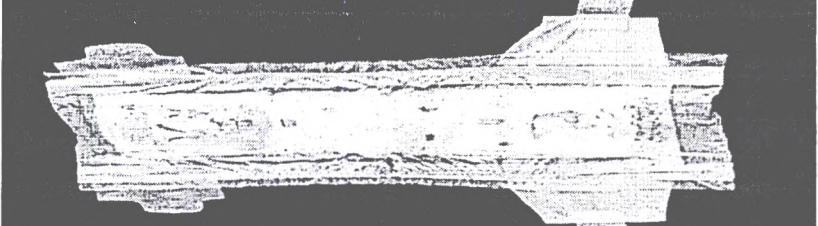
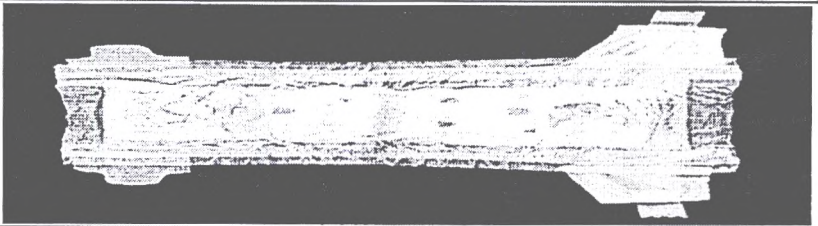
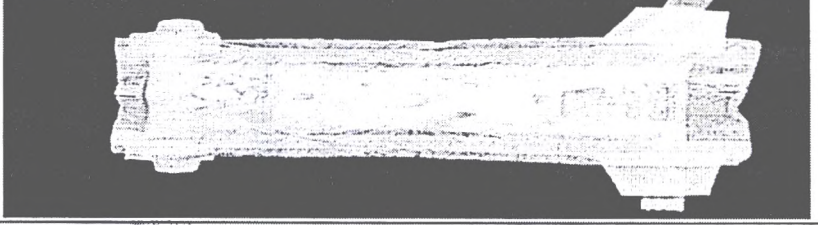
20. Schedule 3 Photographic images of the appearance of Diapers, Baby Medical.
Table 30. Photographic images of the appearance of Diapers, Baby Medical.

1. No.	Type	Photographic image of the appearance
1	2	3
1	Giggles Diapers. Baby Medical, sizes: «New Baby» «Mini» «Midi» «Maxi» «Junior» «Junior+»	
2	Giggles Premium Diapers, size «New Baby»	
3	Giggles Premium Diapers, size «Mini»	
4	Giggles Premium Diapers, size «Midi»	
5	Giggles Premium Diapers, size «Maxi»	

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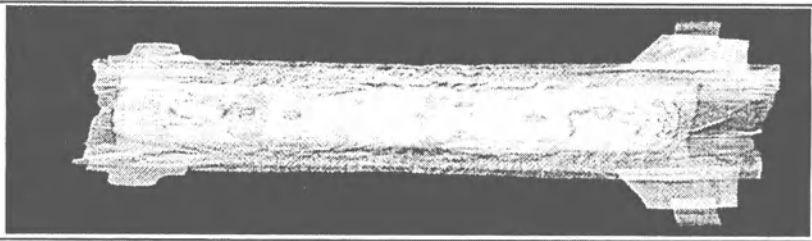
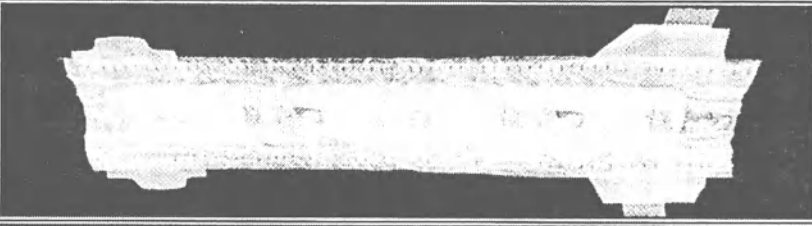
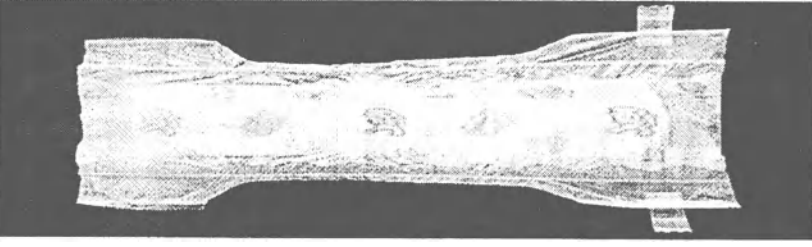
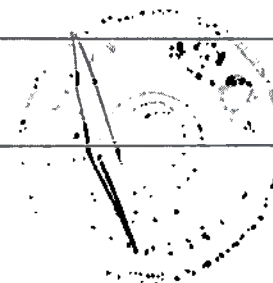
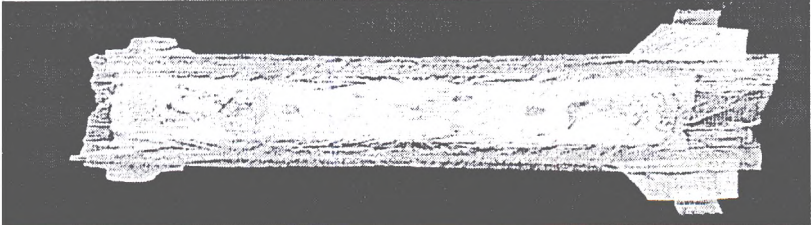
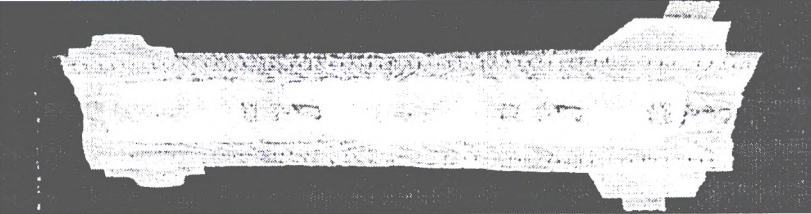
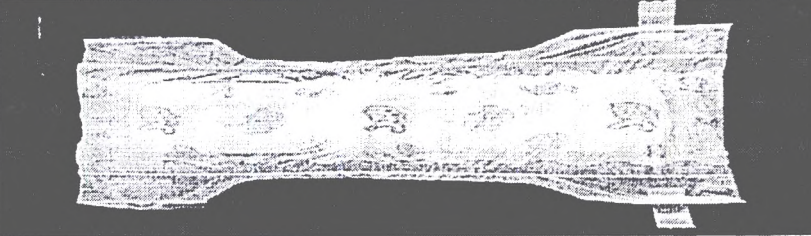
21. Schedule 4 The compliance of Extract from Technical Documentation with the requirements to the scope of the Manufacturer's technical documentation for medical devices as contemplated in Order No. 11H of the Ministry of Health Care of the Russian Federation dtd. 19 Jan 2017.

Table 31. The compliance of Extract from Technical Documentation with the requirements to the scope of information as contemplated in Order No. 11H

Sl.No.	Order No. 11H Section	Section of Extract
1	1) Product description, other information that identifies the medical device, e.g. model no., modifications;	1. Product Description 5. Classification
2	2) Intended use of Product and principles of operation	4.1 Intended Use of Product established by Manufacturer 4.2 Diapers, Baby Medical. Functional Principle
3	3) Indications and Contraindications for Use;	4.3 Indications for use 4.4 Contraindications and Possible Side Effects
4	4) Potential users of the Product;	4.5 Potential Users
5	5) Description of the main functional elements of the Product, that can include schemes, photographic images, drawings, charts, and other explanation	6. Specifications 7. Diapers, Baby Medical. Quality Parameters
6	6) Description of components (parts) of the Product (if any)	6.1 Composition of Diapers, Baby Medical
7	7) Description of non-medical accessories or products to be used in combination with the Product (if any);	N/A

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Table 31. The compliance of Extract from Technical Documentation with the requirements to the scope of information as contemplated in Order No. 11H

Sl.No.	Order No. 11H Section	Section of Extract
8	8) List and description of materials the Product contains, that contact, directly or indirectly, the patient's body (human body):	6.1 Composition of Diapers, Baby Medical 6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features. 6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features. 6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features. Schedule 1 Materials
9	9) Details of the Product Packaging and Labelling:	6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, parameters No.No.8-9 - Design; 6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, parameters No.No.8-9 - Design; 6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, parameters No.No.8-9 - Design; 8.1 Retail Package Labelling. 8.2 Shipping Package Labelling.
10	10) List of risks identified in the risk analysis process, description of risk management methods in order to lower the risks to the acceptable level (if any):	Manufacturer's Specification: Diapers, Baby Medical, Types: Giggles, Giggles Premium, Jusy Risk analysis procedure Risk analysis and evaluation chart. Risk analysis report.
11	11) Details of verification and validation of the medical device used to prove the compliance with the standards, including the following test results: a) laboratory tests performed by laboratories/ test centres; b) laboratory and/ or manufacturer's tests including those simulating real operational conditions; c) laboratory animal tests to validate the concept of the product:	The information included in the Product's File (not part of this document): 1. Test Certificate 13 24 02642 Patch Test Report Giggles Premium Diapers, Baby Medical, Laboratory: Quality Assurance and Control Systems Ltd. 2. Test Certificate 13 24 02643 Patch Test Report Giggles Diapers, Baby Medical Laboratory: Quality Assurance and Control Systems Ltd.
12	12) List of materials of animal and/ or human origin including details of their biological compatibility and safety, choice of sources (donors), sampling, processing, storage, handling (if any):	N/A
13	13) Details of tests, test certificates, analysis of test findings.	The information included in the Product's File (not part of this document): 1) Technical Test Data Evaluation Report for medical device _____ dtd. _____ including test schedule and certificates for the tests performed by Testing Laboratory for Medical Products (division of Federal Research & Clinical Center of Physical-Chemical Medicine, Federal Medical Biological Agency), Certificate of Accreditation No.

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Table 31. The compliance of Extract from Technical Documentation with the requirements to the scope of information as contemplated in Order No. 11H

Sl.No.	Order No. 11H Section	Section of Extract
		RA.RU.21MH25 dtd. 17 September 2015 r. Scope of Accreditation: Medical Devices); 2) Toxicological Data Evaluation Report for medical device _____ dtd. _____ including test schedule and certificates for the tests performed by Testing Laboratory for Medical Products (division of Federal Research & Clinical Center of Physical-Chemical Medicine, Federal Medical Biological Agency). Certificate of Accreditation No. RA.RU.21MH25 dtd. 17 September 2015 r. Scope of Accreditation: Medical Devices); -Clinical Data Sheet for clinical trials of medical device _____ dtd. _____ performed _____ by _____ _____ (Medical Practice License No. _____ dtd. _____).
14	14) references to prior modifications of the medical device, or similar modifications of current medical devices, where information on similar or prior modifications of the medical device was used to prove the compliance of the medical device with safety and efficacy standards;	Diapers, Baby Medical, manufactured by Sevinçler Sağlık Ürünleri San. ve Tic. A. Ş. are being registered in Russia for the first time. A list of data in regards to similar modifications of the medical device, is included in the Product's File (not part of this document): 1) Comparison with Similar Medical Devices
15	15) Details of the main design stages of the medical device, that can include schemes, photographic images, drawings, charts, and other explanations;	7. Diapers, Baby Medical. Quality Parameters 12. Acceptance Criteria. 13. Test Methods. Schedule 2 Manufacturing Process Flow Chart
16	16) Information from the records of the State Register of Medicines regarding the medical use of the contents of the medical device: - medicinal drug, including name (International Nonproprietary Name, or generic name, or chemical and invented name), Manufacturer's name, date and No. of Registration certificate; - active pharmaceutical ingredient, including name (International Nonproprietary Name, or generic name, or chemical and invented name), Manufacturer's name, date and No. of entry made in the State Register of Medicines);	N/A 

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Table 31. The compliance of Extract from Technical Documentation with the requirements to the scope of information as contemplated in Order No. 11H

Sl.No.	Order No. 11H Section	Section of Extract
17	17) Description of the product sterilization method, details of the sterilization process validation procedure (including the biological burden, pyrogenicity, sterilizing agent residue tests), details of the packaging process validation (where the medical device must be sterile as supplied);	The product is non-sterile and not intended for sterilization.
18	18) Details of development, design and validation of the software used in the product (where the medical device needs a software for proper operation and/ or use for intended purposes);	N/A
19	19) Maintenance and repair requirements of the medical device;	N/A
20	20) Product disposal procedure and conditions.	14.3 Disposal

22. Schedule 5 The compliance of this Extract from Technical Documentation with the requirements of the Russian national standard GOST P 52557 – 2011 Paper Children's Diapers. General Specifications.

Table 32. The compliance of this Extract from Technical Documentation with the requirements of the Russian national standard GOST P 52557 – 2011

Sl.No.	Section of GOST P 52557 – 2011	Section of Extract
1	1 Scope	4. Intended Use, Indications and Contraindications, Possible Side Effects, Potential Users
2	2 Referenced codes and standards	The information included in the Product's File (not part of this document): 1) List of codes and standards
3	4 Classification	5. Classification
4	5.2 Requirements to the composition of diapers	6.1 Composition of Diapers, Baby Medical
5	5.3 Configuration of diapers	6.2 Composition of Diapers, Baby Medical, Types
6	5.4 Design of diapers	6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, parameters No.No. 8-9 – Design ; 6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, parameters No.No. 8-9 – Design ; 6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, parameters No.No. 8-9 – Design .
7	5.5 Appearance Requirements	6.3 Appearance of Diapers, Baby Medical;
	5.6. The details of the composition, linear dimensions, configuration and design must be	6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features.

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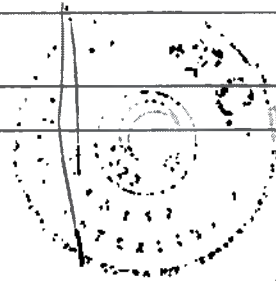
Table 32. The compliance of this Extract from Technical Documentation with the requirements of the Russian national standard GOST P 52557 – 2011

Sl.No.	Section of GOST P 52557 – 2011	Section of Extract
	included in technical documentation for the specific type of products or age.	6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features. 6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features.
8	5.7. The following materials are used in manufacture of the diapers	6.1 Composition of Diapers, Baby Medical 6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features. 6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features. 6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features. Schedule I Materials
9	5.8 The layers must be bonded using a heat treatment method, or a hotmelt adhesive, or any other methods that ensures a good bond between the diaper layers (seams). The seams must be continuous.	6.3 Appearance of Diapers, Baby Medical:
10	5.9 The details of raw materials used in manufacture of diapers must be included in technical documentation for specific types or groups of diapers.	Schedule I Materials
11	5.10 Odour intensity from a diaper (aqueous extract), must be score 1 (max).	7.2 Organoleptic, Toxicological, Chemical Safety Parameters
12	5.11 Quality parameters of diapers	7. Diapers Quality Parameters
13	5.12 Labelling	6.4 Giggles Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, parameters No.No.8-9 - Design: 6.5 Giggles Premium Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, parameters No.No.8-9 - Design: 6.6 Jusy Diapers, Baby Medical. Linear Dimensions, Design, and Structural Features, parameters No.No.8-9 - Design: 8.1 Retail Package Labelling. 8.2 Shipping Package Labelling.
14	5.13 Packaging	9. Packaging
15	6 Acceptance Criteria	12 Acceptance Criteria
16	7 Test Methods	13 Test Methods
	8 Transport and storage	14.1 Transport Rules 14.2 Storage of diapers
17	9 Manufacturer's Warranty	16. Manufacturer's Warranty

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