State of Minnesota

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SECRETARY OF STATE

APOSTILLE

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1. Country: United States of America

This Public Document: Documents for 3M Unitek

2. has been signed by Angela Maxcine Draper

3. acting in the capacity of Notary Public, State of Minnesota

4. bears the seal/stamp of Angela Maxcine Draper, Notary Public, State of Minnesota

CERTIFIED

5. at: St. Paul, Minnesota

6. Date: July 19, 2017

7. by: Secretary of State, State of Minnesota

8. File No: 9580763-10


9. Seal/Stamp:

10. Signature:

Steve Simon

Steve Simon
Secretary of State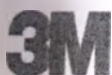

To Whom It May Concern:

In my capacity as Regulatory Affairs Manager, 3M Unitek, I certify that the attached English version of the Instructions for Use (IFU) for Transbond Adhesives, Kits and Refills with Accessories (Transbond Plus Color Change Adhesive, Transbond XT Adhesive, Transbond Supreme LV Adhesive, Transbond IDB Pre-Mix Chemical Cure Adhesive) intended for Russia is truthful and accurate.

Sincerely,

Mark Burville
Regulatory Affairs Manager
3M Unitek

18 JUL 2017
Date

STATE OF MINNESOTA)
COUNTY OF RAMSEY) ss

On July 18, 2017, before me, a Notary Public within and for said county, personally appeared Mark Burville, known by me to be the person described in and who executed the foregoing instrument and acknowledged that he executed the same as his own free act and deed.

Angela Maxcine Draper, Notary Public
My Commission Expires: January 31, 2019

Transbond™ Plus Color Change Adhesive

Instructions For Use




Indications: Products are intended for use in orthodontic treatment of malocclusions.

Intended For Use: This product is indicated for use in orthodontic appliance application and bonding for orthodontic treatment.

Note: Transbond™ Plus Color Change Adhesive is not intended to be used with other orthodontic applications where the cured adhesive is exposed and not covered by appliances.

Contraindications No known contraindications.

Scope of use: Orthodontics. Only for professional use.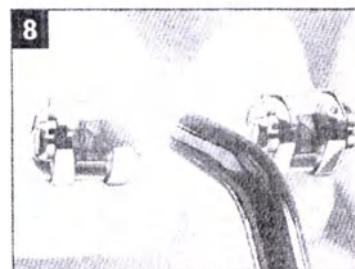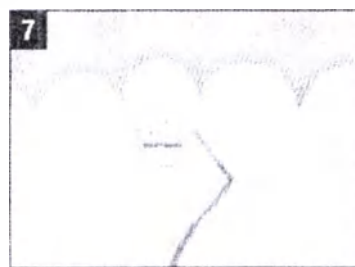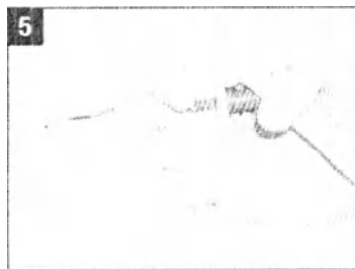

Warning: This product contains acrylate monomers. Acrylate monomers are known to produce allergic skin reactions in certain sensitive individuals. May cause eye and skin irritation.

Precautions: Avoid eye and skin contact. Wear gloves when handling this material.

First Aid:

Eye contact: Immediately flush with plenty of water. See a physician if irritation persists.

Skin contact: Wash affected area with soap and water. See a physician if irritation persists. See a physician if irritation persists.

Precautions: Porcelain Crowns or Facings: Care must be taken when bonding to porcelain crowns or facings as debonding may cause chipping, delamination or breakage of the crown. Do not bond to porcelain crowns that have thin facings or that appear to be compromised. Prepare the porcelain crown to be bonded by using a porcelain primer such as 3M™ ESPE™ RelyX™ Ceramic Primer. Follow instructions included with the porcelain primer.

Important: Replace syringe or capsule cap immediately after each use. Exposure to light will cause the color to fade and the adhesive to begin to cure if uncapped syringe or capsule is exposed to light.

Note: Transbond Plus Color Change Adhesive features a unique color-change adhesive that is pink upon extrusion from the syringe or capsule. The pink color may assist in positioning and/or adhesive flash clean-up. The adhesive color begins to fade under ambient light and completely fades after light-curing. Therefore brackets must be coated at chairside when the patient is ready for bonding. If brackets are coated with adhesive prior to tooth preparation color may begin to fade and the adhesive may begin to cure. Please note that the fading of the color is not an indication of cure.

Note: Indirect Bonding Using Transbond Plus Color Change Adhesive

- Because of the hydrophilic nature of the Transbond Plus Color Change Adhesive, it is not as effective for indirect bonding as a hydrophobic adhesive, such as Transbond™ XT Adhesive. However, if Transbond Plus adhesive is chosen to be used for indirect bonding, best results will be achieved when the stone model is thoroughly dried before placing the Transbond Plus adhesive coated brackets. After pouring the ortho stone into the impression, the stone model should be dried overnight in an oven at 110°F (43°C) or under ambient conditions.
- After removing the indirect bonding tray from the stone model, rinsing and drying, you may observe a cloudy white layer on the custom resin base. This white color is caused by a very thin surface layer of Transbond Plus adhesive resin. Subsequent microetching or cleaning (e.g. using a toothbrush) of the custom resin base will remove this layer and should provide adequate bond strength. This whitish layer only occurs during indirect bonding.
- It is strongly recommended that a secondary cure of the custom resin bases be performed by directly exposing the tray to a light source to achieve a complete cure.

Note that direct bonding of Transbond Plus Color Change Adhesive coated brackets is not affected by the hydrophilic properties of the adhesive.

A. Tooth Preparation

1. Isolate teeth using the Dry Field System or a combination of retractors, dri-angles and cotton rolls.
2. Prophyl teeth with an oil-free pumice or paste.
3. Rinse with water.
4. Air dry thoroughly using oil and moisture-free air source.

Figure 2.

B. Acid Etching

1. If using Transbond™ Plus Self Etching Primer, proceed to Option 3: Priming Surfaces with Transbond Plus Self Etching Primer steps below. Otherwise, continue with phosphoric acid etching.
2. Apply Unitek™ Etching Gel (REF 712- 039 or 712- 044) to tooth surfaces following the instructions provided with the etching gel delivery system. If using other gel etching systems, consult the manufacturer's instructions for proper technique and recommended etching times.
3. Rinse with water.
4. Air dry thoroughly using oil and moisture-free air source.

C. Priming Teeth

Option 1: Priming Surfaces with Transbond™ XT Primer

For detailed instructions on Transbond™ XT Primer, please refer to REF 011-519.

Option 2: Priming Surfaces with Transbond™ MIP Moisture Insensitive Primer

For detailed instructions on Transbond™ MIP Moisture Insensitive Primer, please refer to REF 011-563.

Option 3: Priming Surfaces with Transbond™ Plus Self Etching Primer

For detailed instructions on Transbond™ Plus Self Etching Primer, please refer to REF 011-581.

D. Adhesive Application from Capsules. Note: Do not apply adhesive onto brackets until patient is ready to be bonded.

1. To insert capsule into dispenser, open dispenser handle as wide as possible. Pull plunger toward opened handle. Place capsule tip into the barrel slot.

2. Seat the capsule in the barrel by pushing the capsule forward (away from the handle) as far as possible. Push plunger towards capsule. Close handle grip until plunger contacts capsule. Remove the cap from the capsule.

3. Dispense a small amount of Transbond PLUS Color Change Adhesive paste onto bracket base using a slow, steady pressure. When completed wipe capsule tip clean and replace cap.

4. To remove the used capsule, open the dispenser handle as wide as possible. Pull plunger away from capsule. Push the capsule toward the plunger. Turn dispenser upside down to drop capsule into hand.

E. Adhesive Application from Syringes

Note: Do not apply adhesive onto brackets until patient is ready to be bonded.

1. With the syringe apply a small amount of Transbond Plus Color Change Adhesive paste onto bracket base. Use sparingly. When completed wipe syringe tip clean and replace cap.

F. Placement and Curing

1. Immediately after applying adhesive, lightly place bracket onto tooth surface.

2. Adjust bracket to final position, press firmly to seat bracket.

Note: If final positioning will be delayed, cover patient's mouth with a dark colored mask or other article to avoid premature curing and fading of the pink color by ambient light.

3. Gently remove excess adhesive around appliance base without disturbing the bracket. The pink color aids in flash removal.

4. Hold the curing light stationary at a distance of 2-3 mm above interproximal contact for metal appliances and perpendicular to ceramic appliance surface.

A technique tip for faster light curing of metal brackets is to position the curing light guide interproximal to two brackets. However, in order for the bracket to fully cure, both sides of the bracket must be illuminated.

Precaution: Please follow manufacturer's directions for the handling, proper use and recommendations for eye protection when using a curing light.

Refer to the chart to determine the curing conditions for optimal bond strength. If your curing light does not appear on this chart, please refer to your light manufacturer's instructions for curing conditions.

Appliance with Transbond™ PLUS Color Change Adhesive	Ortholux™ LED Curing Light (App. 1000 mW/cm²) (LED)	Ortholux™ Luminous Curing Light (App. 1600 mW/cm²) (LED)
Metal Brackets	5 seconds mesial + 5 seconds distal	3 seconds mesial + 3 seconds distal
Ceramic Brackets	5 seconds through the bracket	3 seconds through the bracket
Bondable Buccal Tubes	10 seconds mesial + 10 seconds occlusal	6 seconds mesial + 6 seconds occlusal

5. Archwires can be placed immediately after bonding of last bracket.

G. Disinfection Information

Please refer to literature 011-650, "Reprocessing Instructions For Reusable Non-Sterile Devices" for cleaning and disinfection of 712-032, Adhesive Dispensing Gun.

H. Storage and Use

1. Do not expose materials to elevated temperatures or intense light. Bring to room temperature prior to use.
2. Do not store material in proximity to eugenol- containing products as this may inhibit the proper adhesive curing.
3. This system is designed to be used at room temperature (68-77°F, 20 -25°C). Storage between 35-80°F, 2-27°C.
4. Shelf-life at room temperature is per the expiration date on the kit box. Rotate inventory to optimize shelf life.
5. Light Sensitive. Do not expose uncured Transbond PLUS Color Change Adhesive to ambient light for extended periods of time. When not in use, store syringes with caps at room temperature.

Transport conditions

- Transportation by any covered transport, if storage conditions are maintained.
The ambient air humidity is not regulated.

Shelf life:

Shelf life at room temperature is 36 months from the date of manufacture.

Packaging content

I. Transbond Adhesives, kits:

1. Transbond Plus Color Change, Adhesive Capsule Kit with SEP:
 - 25 pouched capsules;
 - adhesive dispenser gun;
 - Transbond Plus SEP Kit (20 primer foil packs, 5 applicators)
2. Transbond Plus Color Change, Adhesive Syringe Kit with SEP:
 - 2 syringes;
 - Transbond Plus SEP Kit (20 primer foil packs, 5 applicators)
3. Transbond Plus Color Change, Adhesive Capsule Kit:
 - 25 Capsules;
 - adhesive dispenser gun.

II. Transbond Adhesive Refills:

1. Transbond Plus Color Change, Adhesive Syringe Refill (1, 2, 4 ea/Kt)
2. Transbond Plus Color Change, Adhesive Capsule Refill (2, 25 ea/Kt)

Packaging

Transbond Plus Color Change, Adhesive Capsule Kit: primary packaging - Nylon Adhesive Capsule, dimensions: Length 2.79 cm $\pm$ 0.13 cm, Diameter 0.77cm $\pm$ 0.01cm, secondary packaging - Adhesive Capsule Polyethylene and Aluminum Foil Pouch, dimensions: Length 8.9 $\pm$ 0.3cm, Width 15.2 $\pm$ 0.3cm, tertiary packaging - Adhesive Capsule Carton Box, dimensions: 11.9cm x 18.2cm x 3.9cm, final packaging -Low Density Polyethylene Bag, dimensions: 25.4cm $\pm$ 0.3cm x 20.32cm $\pm$ 0.3cm.

Transbond Plus Color Change, Adhesive Syringe Kit: primary packaging - Polypropylene Adhesive Syringe, dimensions: Length 12.9 cm $\pm$ 0.3 cm, secondary packaging - Adhesive Syringe Low Density Polyethylene Bag, dimensions: 12.7cm $\pm$ 0.6cm x 20.32cm $\pm$ 0.6cm, tertiary packaging - Low Density Polyethylene Bag, dimensions: 25.4cm $\pm$ 0.3cm x 20.32cm $\pm$ 0.3cm.

Transbond Plus Color Change, Adhesive Kit: primary packaging - Nylon Adhesive Capsule, dimensions: Length 2.79 cm $\pm$ 0.13 cm, Diameter 0.77cm $\pm$ 0.01cm, secondary packaging -

Adhesive Capsule Polyethylene and Aluminum Foil Pouch. dimensions: Length 8.9 ± 0.3 cm. Width 15.2 ± 0.3 cm. tertiary packaging - Adhesive Capsule Carton Box. dimensions: 11.9 cm x 18.2 cm x 3.9 cm.

Transbond Plus Color Change, Adhesive Capsule Refill (2, 25 ea/Kt): primary packaging - Nylon Adhesive Capsule. dimensions: Length 2.79 cm ± 0.13 cm. Diameter 0.77 cm ± 0.01 cm. secondary packaging - Adhesive Capsule Polyethylene and Aluminum Foil Bag. dimensions: Length 8.9 ± 0.3 cm. Width 15.2 ± 0.3 cm. tertiary packaging - Adhesive Capsule Low Density Polyethylene Bag. dimensions: 23.5 ± 1.3 cm x 12.7 ± 0.6 cm

Transbond Plus Color Change, Adhesive Syringe Refill (1, 2, 4 ea/Kt): primary packaging - Polypropylene Adhesive Syringe. dimensions: Length 12.9 cm ± 0.3 cm. secondary packaging - Adhesive Syringe Low Density Polyethylene Bag. dimensions: 1 ea bag dimensions - 12.7 ± 0.6 cm x 20.3 ± 0.6 cm. 2 ea bag dimensions - 12.7 cm ± 0.6 cm x 20.32 cm ± 0.6 cm. 4 ea bag dimensions - 15.2 ± 0.6 cm x 25.4 ± 0.6 cm.

Medicinal Substances There are no medicinal substances used in the device.

Biological Safety No materials of animal or human origin used in the device.

Requirements to medical product maintenance and repair: Not applicable.

Disinfection and cleaning: Please refer to literature 011-650. "Reprocessing Instructions For Reusable Non-Sterile Devices" for cleaning and disinfection of 712-032. Adhesive Dispensing Gun.

Labeling. Devices are supplied in packaging with labeling containing the following information: name, lot and ref number; manufacturer; package content; shelf life; as well as symbols - Consult instructions for use. CE Mark. Temperature limitation. Keep away from sunlight. Do not re-use. single use. Recyclable packaging. Low density polyethylene packaging.

Disposal. Packaging/contents should be disposed in accordance with the local/regional/national/international regulations. Medical products should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class B).

Warranty. 3M warrants this product will be free from defects in material and manufacture. 3M MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M sole obligation shall be repair or replacement of the 3M ESPE product.

The name and address of the manufacturer are provided below. 3M Unitek Corporation. 2724 South Peck Road. Monrovia, California, 91016, USA

Authorized representative in RF: 3M Russia CJSC. 121614. Moscow Krylatskaya Street, 17. Unit 3. tel: +7 (495) 784 7474 / fax: +7 (495) 784 7475, e-mail: 3mra@mmm.com

Transbond XT Adhesive

Instructions For Use

Indication: 3M Unitek products are intended for use in orthodontic treatment of malocclusions.

Intended use: This product is designed for direct bonding of ceramic orthodontic brackets and metal brackets.

Note: this product is not indicated for use with plastic (polycarbonate) brackets.

Contraindications

No known contraindications.

Scope of use

Orthodontics. Only for professional use.

Warning: This product contains acrylate monomers. Acrylate monomers are known to produce allergic skin reactions in certain sensitive individuals. May cause eye and skin irritation. Important: All resin-based primers that penetrate into the enamel rods may under certain conditions temporarily alter the appearance of the enamel. It does not discolor the enamel; rather it creates a contrast to the chalky appearance of the surrounding enamel.

Precautions: Avoid eye and skin contact. Wear gloves when handling this material. First Aid:

Eye contact: Immediately flush with plenty of water. See a physician if irritation persists.

Skin contact: Wash affected area with soap and water. See a physician if irritation persists.

Precaution: Porcelain Crowns or Facings: Care must be taken when bonding to porcelain crowns or facings as debonding may cause chipping, delamination or breakage of the crown. Do not bond to porcelain crowns that have thin facings or that appear to be compromised. Prepare the porcelain crown to be bonded by using a porcelain primer such as 3M™ ESPET™ RelyX™ Ceramic Primer. Follow instructions included with the porcelain primer.

A. Tooth Preparation

1. Isolate teeth using the Dry Field System or a combination of retractors, dri-angles and cotton rolls.
2. Prophyl teeth with an oil-free pumice or paste.
3. Rinse with water.
4. Air dry thoroughly using oil and moisture-free air source.

B. Acid Etching

1. If using Transbond™ Plus Self Etching Primer, proceed to Option 3: Priming Surfaces with Transbond Plus Self Etching Primer steps below. Otherwise, continue with phosphoric acid etching.
2. Apply Unitek™ Etching Gel (REF 712-039 or 712-044) to tooth surfaces following the instructions provided with the etching gel delivery system. If using other gel etching systems, consult the manufacturer's instructions for proper technique and recommended etching times.
3. Rinse with water.
4. Air dry thoroughly using oil and moisture-free air source.

C. Priming Teeth

Option 1: Priming Surfaces with Transbond™ XT Primer

1. Air dry tooth thoroughly.
2. Place small amount of Transbond™ XT Primer in well.
3. Apply thin uniform coat of primer on each tooth surface to be bonded. Note: Since Transbond XT primer acts as a wetting agent, only a very thin film of primer is necessary.

Option 2: Priming Surfaces with Transbond™ MIP Moisture Insensitive Primer

For detailed instructions on Transbond™ MIP Moisture Insensitive Primer, please refer to Instructions For Use REF 011-563.

Option 3: Priming Surfaces with Transbond™ Plus Self Etching Primer

D. Adhesive Application from Capsules

Note: Do not apply adhesive onto brackets until patient is ready to be bonded.

1. To insert capsule into dispenser, open dispenser handle as wide as possible. Pull plunger toward opened handle. Place capsule tip into the barrel slot.
2. Seat the capsule in the barrel by pushing the capsule forward (away from the handle) as far as possible. Push plunger towards capsule. Close handle grip until plunger contacts capsule. Remove the cap from the capsule.
3. Dispense a small amount of Transbond XT™ Adhesive paste onto bracket base using a slow, steady pressure. When completed wipe capsule tip clean and replace cap.
4. To remove the used capsule, open the dispenser handle as wide as possible. Pull plunger away from capsule. Push the capsule toward the plunger. Turn dispenser upside down to drop capsule into hand.

E. Adhesive Application from Syringes

Note: Do not apply adhesive onto brackets until patient is ready to be bonded.

With the syringe apply a small amount of Transbond XT adhesive paste onto bracket base. Use sparingly. When completed, wipe syringe tip clean and replace cap.

F. Placement and Curing

1. Immediately after applying adhesive, lightly place bracket onto tooth surface.
 2. Adjust bracket to final position, press firmly to seat bracket.
- Note:** If final positioning will be delayed, cover patient's mouth with a dark colored mask or other article to avoid premature curing by ambient light.
3. Gently remove excess adhesive from around the bracket base without disturbing the bracket.
 4. Hold the curing light stationary at a distance of 2-3 mm above interproximal contact for metal appliances and perpendicular to ceramic appliance surface.

A technique tip for faster light curing of metal brackets is to position the curing light guide interproximal to two brackets. However, in order for the bracket to fully cure, both sides of the bracket must be illuminated.

Precaution: Please follow manufacturer's directions for the handling, proper use and recommendations for eye protection when using a curing light.

Refer to the chart to determine the curing conditions for optimal bond strength. If your curing light does not appear on this chart, please refer to your light manufacturer's instructions for curing conditions.

Appliance with Transbond™ XT Adhesive	Ortholux™ LED Curing Light (App. 1000 mW/cm²) (LED)	Ortholux™ Luminous Curing Light (App. 1600 mW/cm²) (LED)
Metal Brackets	5 seconds mesial + 5 seconds distal	3 seconds mesial + 3 seconds distal
Ceramic Brackets	5 seconds through the bracket	3 seconds through the bracket
Bondable Buccal Tubes	10 seconds mesial + 10 seconds occlusal	6 seconds mesial + 6 seconds occlusal

5. Archwires may be placed immediately after curing the last bracket.

G. Disinfection Information

Transbond Adhesives, Kits and Refills with Accessories



Please refer to literature 011-650, "Reprocessing Instructions For Reusable Non-Sterile Devices" for cleaning and disinfection of 712-032. Adhesive Dispensing Gun.

Storage and Use

1. Do not expose materials to elevated temperatures or intense light. Bring to room temperature prior to use.
2. Do not store material in proximity to eugenol-containing products as this may inhibit the proper adhesive curing.
3. This system is designed to be used at room temperature (68-77°F, 20-25°C).
Storage between 35-80°F, 2-27°C
4. Shelf-life at room temperature is per the expiration date on the kit box. Rotate inventory to optimize shelf life.
5. Light Sensitive. Do not expose uncured Transbond XT Adhesive to ambient light for extended periods of time. When not in use, store syringes with caps at room temperature.

Transport conditions

Transportation by any covered transport, if storage conditions are maintained.
The ambient air humidity is not regulated.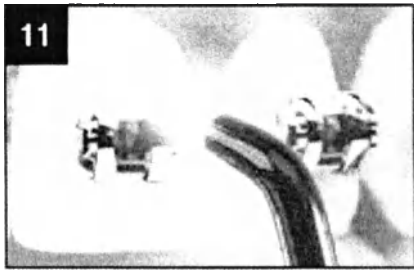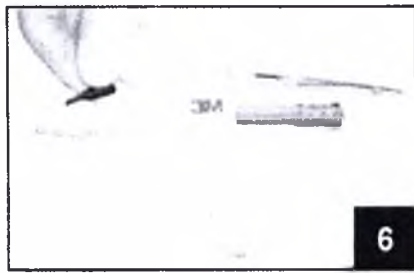

Shelf life

Shelf life at room temperature is 36 months from the date of manufacture.

Contents

Transbond XT Light Cure Adhesive Capsule Kit:

- Capsule (25 pcs.);
- Transbond XT Primer;
- Dispenser gun;
- Brush holder;
- Brush tips (60 pcs.).

Transbond XT Light Cure Adhesive Kit:

- Syringe (2 pcs.);
- Transbond XT Primer;
- Brush holder;
- Brush tips (60 pcs.).

Transbond XT Light Cure Adhesive Capsules (25 pieces per pack):

Transbond XT Light Cure Adhesive Syringes (2 or 4 pieces per pack)

Packaging

In Capsule Kit: capsule with adhesive: length 2.79 cm $\pm$ 0.13 cm, diameter 0.77 cm $\pm$ 0.01 cm.
Secondary packaging - box for capsules with adhesive: material: cardboard, dimensions 6.74 cm x 4.6 cm x 1.66 cm.

Tertiary packing - carton box material: cardboard, dimensions: 19.05 cm x 16.98 cm x 1.9 cm

In Capsules (25 pcs per pack): capsule with adhesive: length 2.79 cm $\pm$ 0.13 cm, diameter 0.77 cm $\pm$ 0.01 cm.

Secondary packaging - box for capsules with adhesive: material: cardboard, dimensions 6.74 cm x 4.6 cm x 1.66 cm.

In Kit: dispenser with adhesive: length: 12.9 cm $\pm$ 0.3 cm.

Secondary packaging-cardboard box: material: cardboard, dimensions 6.03 cm x 5.7 cm x 13.17 cm.

In Syringes (2 or 4 pcs per pack): syringe with adhesive: length: 12.9 cm $\pm$ 0.3 cm.

Secondary package - package: material: low density polyethylene, dimensions: 2 pcs - 12.7 cm $\pm$ 0.6 cm x 20.32 cm $\pm$ 0.6 cm, 4 pcs - 15.24 $\pm$ 0.6 cm x 25.4 $\pm$ 0.6 cm

Medicinal substances. The product does not use medicinal substances.

Biological safety. The product does not use materials of animal or human origin.

Requirements to the medical device maintenance and repair: Not applicable.

Disinfection and cleaning: Please refer to literature 011-650. "Reprocessing Instructions For Reusable Non-Sterile Devices" for cleaning and disinfection of 712-032. Adhesive Dispensing Gun.

Labelling. The products are supplied in a packaging with the labelling containing the following information: name, lot and reference number; name and address of the manufacturer; contents; shelf life; and symbols - Refer to Instructions for Use, EU compliance, storage / transportation temperature limitation, store away from sunlight, do not reuse, recyclable packaging, low density polyethylene packaging.

Disposal. Packaging/contents should be disposed in accordance with the local/regional/national/international regulations.

Medical devices should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class B).

Warranty

3M Company warrants this product will be free from defects in material and manufacture. 3M UNITEK MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is

Transbond Adhesives, Kits and Refills with Accessories

responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Company's sole obligation shall be repair or replacement of the 3M Company product.

Name and address of the manufacturer: 3M Unitek Corporation, 2724 South Peck Road, Monrovia, California, 91016, USA

Authorized representative on the territory of the Russian Federation:

ZAO "3M Russia" Russia 121614, Moscow, Krylatskaya str. bldg.17/3, Tel.:+7 (495) 784 7474

Fax: +7 (495) 784 7475. 3mra@mmm.com

Transbond Supreme LV Adhesive Kit

Instructions For Use

Indications: Products are intended for use in orthodontic treatment of malocclusions.

Contraindications: No known contraindications.

Scope of use: Orthodontics. Only for professional use.

Warning: The product described herein contains acrylate monomers. Acrylate monomers are known to produce allergic skin reactions in certain sensitive individuals. May cause eye and skin irritation.

Precautions: Avoid eye and skin contact. Wear gloves and eye protection when handling this material.

First aid:

Eye contact: Immediately flush with plenty of water. See a physician if irritation persists. Skin contact: Wash affected area with soap and water. See a physician if irritation persists. Note: The instructions for use must be kept for reference for the duration of product use.

Note: Protect the material from light and heat to prevent premature curing.

Precaution: Please follow manufacturer's directions on the handling, proper use and recommendations for eye protection, when using a curing light.

Precaution: Porcelain Crowns or Facings: Care must be taken when bonding to porcelain crowns or facings as debonding may cause chipping, delamination or breakage of the crown. Do not bond to porcelain crowns that have thin facings or that appear to be compromised. Prepare the porcelain crown to be bonded by using a porcelain primer such as 3M™ ESPET™ RelyX™ Ceramic Primer. Follow instructions included with the primer. Follow instructions included with the porcelain primer.

Indirect Tray Preparation: Follow your current laboratory procedure for preparing a stone model, positioning and bonding brackets onto the model, forming a custom resin base and preparing an indirect bonding transfer tray.

Tooth Preparation

1. Isolate teeth using the Dry Field System or a combination of retractors, dri-angles and cotton rolls.
2. Prophyl teeth with an oil-free pumice or paste.
3. Rinse with water.
4. Air dry thoroughly using oil and moisture-free air source.

Acid Etching

1. If using Transbond™ Plus Self Etching Primer, proceed to Option 3: Priming Surfaces with Transbond Plus Self Etching Primer steps below. Otherwise, continue with phosphoric acid etching.
2. Apply Unitek™ Etching Gel (REF 712- 039 or 712- 044) to tooth surfaces following the instructions provided with the etching gel delivery system. If using other gel etching systems, consult the manufacturer's instructions for proper technique and recommended etching times.
3. Rinse with water.
4. Air dry thoroughly using oil and moisture-free air source.

Priming Teeth

Option 1: Priming Surfaces with Transbond™ XT Primer. For detailed instructions on Transbond XT Primer, please refer to REF 011-519.

Option 2: Priming Surfaces with Transbond™ MIP Moisture Insensitive Primer. For detailed instruction on Transbond MIP Moisture Insensitive Primer, please refer to REF 011-563.

Option 3: Priming Surfaces with Transbond™ Plus Self Etching Primer. For detailed instructions on Transbond Plus Self Etching Primer, please refer to REF 011-581.

Adhesive Application and Bonding. After the teeth have been cleaned, isolated, etched and primed proceed with bonding.

1. Apply a small bead of Transbond™ Supreme LV Low Viscosity Light Cure Adhesive directly from the syringe via the applicator tip to the center of each custom resin base in the transfer tray. Spread the adhesive over the base using the dispensing tip.

Note: Use just enough adhesive to cover the base as thin as possible to minimize adhesive flash.

2. Seat the tray and while holding firmly in place, cure the mesial and distal sides of each bracket for 10 seconds with an Ortholux™ LED Curing Light. The tip of the light guide should touch the tray and be at a 90 degree angle to the tooth surface.

Note: If additional curing time is desired remove the hard tray then proceed with additional curing on the soft tray.

3. The tray can be removed immediately following light exposure. Remove the tray using a scaler to peel it from the lingual to buccal. Use extreme care when removing the tray from around the bracket wings and hooks.

4. Scale the excess resin from around the brackets and from the interproximal contacts. Use dental floss to check that all contacts are open.

5. Archwires can be immediately tied in.

Storage and Use

1. Do not expose materials to elevated temperatures or intense light. Bring to room temperature prior to use.

2. Do not store material in proximity to eugenol- containing products as this may inhibit the proper adhesive curing.

3. This system is designed to be used at room temperature (20°-25°C, 68°-77°F). Storage between 2°-27°C/35°-80°F. Light Sensitive.

4. Shelf-life at room temperature is per the expiration date on the kit box. Rotate inventory to optimize shelf life.

Transport conditions

Transportation by any covered transport, if storage conditions are maintained.

The ambient air humidity is not regulated.

Shelf life

Shelf life at room temperature is 24 months from the date of manufacture.

Packaging content

1. Transbond Adhesives, kits:

Transbond Supreme LV Adhesive Kit:

- 2 syringes:

- Metal Dispensing Tips (20/PK)

Packaging

Transbond Supreme LV Adhesive Kit: primary packaging - Polypropylene Adhesive Syringe, dimensions: Length 12.4 cm ± 0.7 cm, secondary packaging - Low Density Polyethylene Bag, dimensions: 12.7cm ±0.6cm x 20.32cm ±0.6cm

Medicinal Substances There are no medicinal substances used in the device.

Biological Safety No materials of animal or human origin used in the device.

Requirements to medical product maintenance and repair: Not applicable.

Disinfection and cleaning: Not applicable.

Labeling. Devices are supplied in packaging with labeling containing the following information: name, lot and ref number; manufacturer; package content; shelf life; as well as symbols - Consult instructions for use, CE Mark, Temperature limitation, Keep away from sunlight, Do not re-use, single use, Recyclable packaging, Low density polyethylene packaging.

Disposal. Packaging/contents should be disposed in accordance with the local/regional/national/international regulations. Medical products should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class B).

Warranty. 3M warrants this product will be free from defects in material and manufacture. 3M MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M sole obligation shall be repair or replacement of the 3M ESPE product.

The name and address of the manufacturer are provided below. 3M Unitek Corporation, 2724 South Peck Road, Monrovia, California, 91016, USA

Authorized representative in RF: 3M Russia CJSC, 121614, Moscow Krylatskaya Street, 17, Unit 3, tel: +7 (495) 784 7474 / fax: +7 (495) 784 7475, e-mail: 3mra@mmm.com

Transbond IDB Pre-Mix Chemical Cure Adhesive

Instructions For Use

Indications: Products are intended for use during orthodontic treatment of malocclusions.

Intended For Use: Transbond IDB Pre-Mix Chemical Cure Adhesive is indicated for indirect bonding only. The low viscosity of the adhesive does not allow sufficient bracket stability for standard direct bonding techniques.

Contraindications: No known contraindications.

Scope of use: Orthodontics. Only for professional use.

Warning: Transbond™ IDB Pre-Mix Chemical Cure Adhesive contains acrylate monomers. Acrylate monomers are known to produce allergic skin reactions in certain sensitive individuals. May cause eye and skin irritation.

Precautions: Avoid eye and skin contact. Wear gloves when handling this material.

First aid:

Eye contact: Immediately flush with plenty of water. See a physician if irritation persists.

Skin contact: Wash affected area with soap and water. See a physician if irritation persists.

Precaution: Porcelain Crowns or Facings: Care must be taken when bonding to porcelain crowns or facings as debonding may cause chipping, delamination or breakage of the crown. Do not bond to porcelain crowns that have thin facings or that appear to be compromised. Prepare the porcelain crown to be bonded by using a porcelain primer such as 3M™ ESPET™ RelyX™ Ceramic Primer. Follow instructions included with the primer. Follow instructions included with the porcelain primer.


Indirect Tray Preparation: Follow your current laboratory procedure for preparing a stone model, positioning and bonding brackets onto the model, forming custom resin bases and preparing the indirect tray.

Note: A hydrophobic adhesive, such as Transbond™ XT Adhesive or APC™ II Adhesive, should be used to create custom resin bases for indirect bonding.


Tooth Preparation:

Note: If indirect bonding with the Incognito™ Lingual Appliance System or another manufacturer's prefabricated indirect bonding tray, follow the manufacturer's recommendations for tooth preparation and proceed to acid etching.


1. Isolate teeth with the Dry Field System. (Figure 1)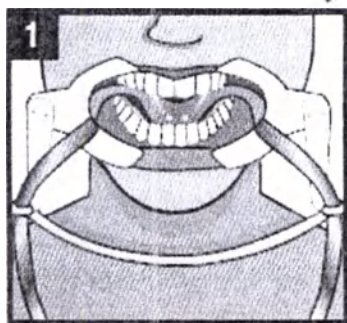


2. Prophyl teeth with an oil-free pumice or paste. (Figure 2)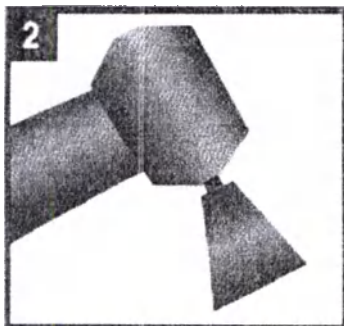

3. Rinse with water. (Figure 3)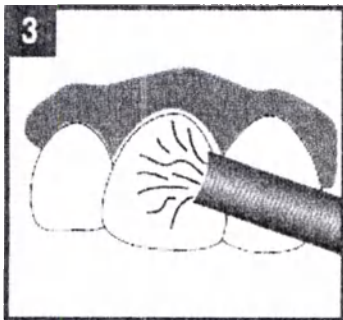

4. Air dry thoroughly using oil and moisture-free air source. (Figure 4)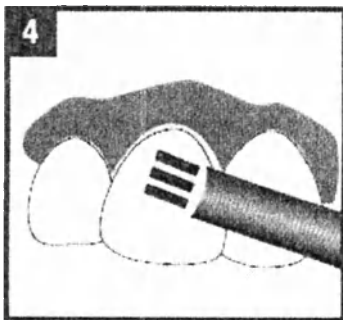


Acid Etching


Note: Transbond IDB Pre-Mix Chemical Cure Adhesive is not compatible with Transbond™ Plus Self Etching Primer. Teeth to be bonded using Transbond IDB Pre-Mix Adhesive should not be etched and primed using Transbond Plus Self Etching Primer rather follow the instructions below for Unitek™ Etching Gel.

Although the use of a primer is not necessary, Transbond™ IDB Pre-Mix Chemical Cure Adhesive is compatible with Transbond™ XT Primer and Transbond™ MIP Moisture Insensitive Primer. For detailed instructions on Transbond XT Primer, please refer to REF 011-519. For detailed instructions on Transbond MIP Moisture Insensitive Primer, please refer to Instructions For Use REF 011-563.


1. Apply Unitek Etching Gel (REF 712-039 or 712-044) to teeth surfaces following the instructions provided with the etching gel delivery system. (Figure 5)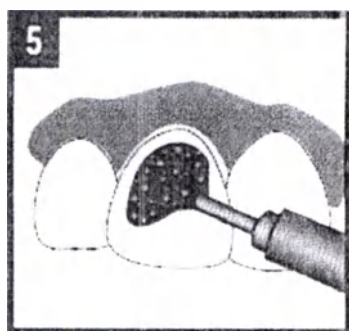

2. Rinse with water. (Figure 6)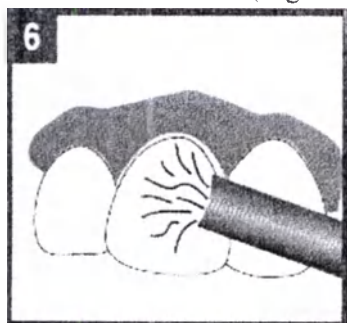

3. Air dry thoroughly using oil and moisture-free air source. (Figure 7)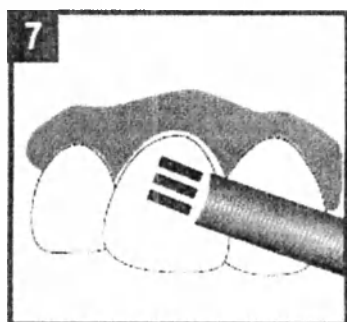


Adhesive Bonding


Note: The work and set times of Transbond™ IDB Pre-Mix Chemical Cure Adhesive are temperature dependant. At room temperature, the work time is 75 - 90 seconds and the set time is 100 - 120 seconds. If used directly from the refrigerator, the work time is 135 - 145 seconds and the set time is 170 - 190 seconds. The indirect bonding trays must be seated into the patient's mouth prior to the elapsed working time.

Note: The mixing well and applicator are disposable wells and intended for use with a single patient.


1. Dispense equal amounts Part A and Part B into a disposable well. (Figure 8)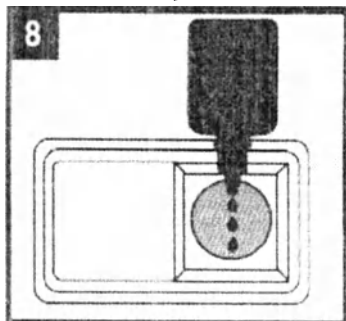



2. Using the disposable applicator brush, mix Part A and Part B together by stirring for 10 seconds. (Figure 9)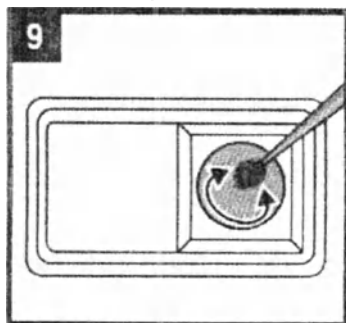


3. Apply a thin coat of the mixed adhesive to each custom resin base in the indirect bonding tray. (Figure 10)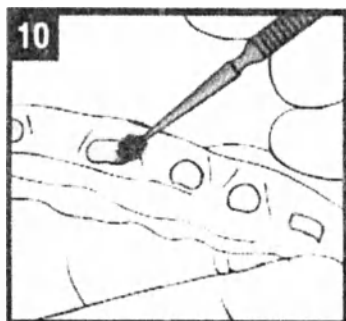

4. Apply a thin coat of the adhesive material to the tooth surface. (Figure 11A Labial Bonding, Figure 11B Lingual Bonding)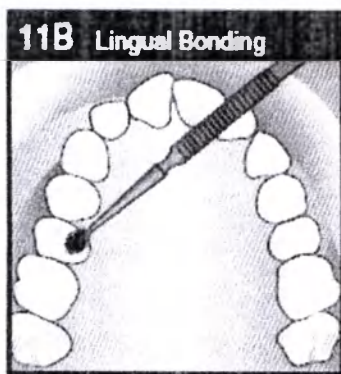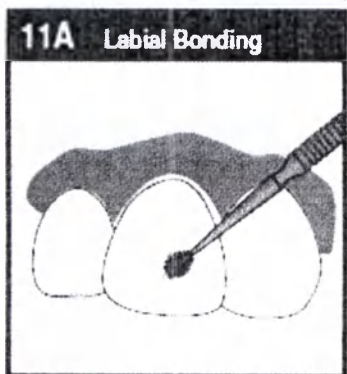


5. The tray is then seated and held in place for 3 minutes. Remove the tray after one additional minute. (Figure 12A Labial Bonding, Figure 12B Lingual Bonding)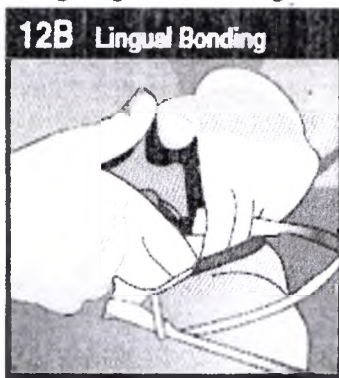


6. Repeat the steps for the opposing jaw.

Tray Removal


1. Remove the Dry Field System. (Figure 13) Isolate teeth with the Dry Field System. (Figure 13)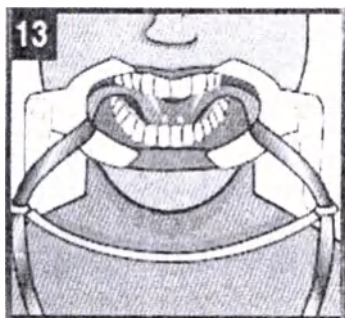

2. Remove the tray using a scaler to peel it from the lingual to buccal. Use extreme care when removing the tray from around the bracket wings and hooks. (Figure 14).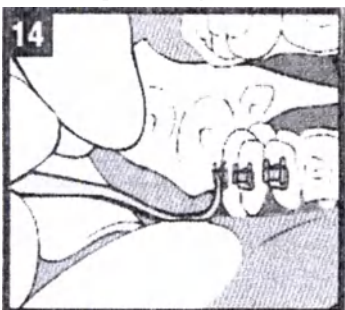

3. Scale the excess resin from around the brackets and from the interproximal contacts. Use dental floss to check that all contacts are open. (Figure 15)

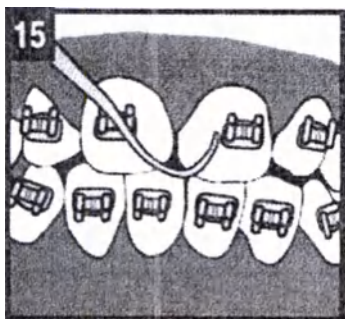4. Archwires can be immediately tied in.

Storage and Use

1. Refrigerate (35 - 45°F, 2 - 7°C) when not in use. Storage between 2°C-7°C/35°F-45°F.
2. Do not expose materials to elevated temperatures or intense light.
3. Remove the adhesive from the refrigerator immediately prior to dispensing.
4. Shelf-life when stored under refrigeration is per the expiration date on the bottle or kit box.

Transport conditions

Transportation by any covered transport, if storage conditions are maintained.
The ambient air humidity is not regulated.

Shelf life

Shelf life at recommended storage temperature is 24 months from the date of manufacture.

Packaging content

I. Transbond Adhesives, kits:

. Transbond IDB Pre-Mix Chemical Cure Adhesive Kit:

- 1 Bottle Part A
- 1 Bottle Part B
- 200 Applicators.
- 20 Disposable Wells.

II. Transbond Adhesive Refills:

1. Transbond IDB Pre-Mix Chemical Cure Adhesive bottle Part A.
2. Transbond IDB Pre-Mix Chemical Cure Adhesive bottle Part B.

Packaging

Transbond IDB Pre-Mix Chemical Cure Adhesive Kit: primary packaging - Part A and B Polyethylene Bottle, Polypropylene Cap, dimensions: Height 6.2cm± 0.1cm, Width 2.7cm ± 0.03cm, Cap Height 0.4cm ± 0.02cm, Cap Thread Pitch 2mm, Threading Length 13 mm, secondary packaging - Carton Box, dimensions: 23.3cm x 18.5cm x 4.1cm

Transbond IDB Pre-Mix Chemical Cure Adhesive Part A. Transbond IDB Pre-Mix Chemical Cure Adhesive Part B: primary packaging - Part A and B Polyethylene Bottle, Polypropylene Cap, dimensions: Height 6.2cm± 0.1cm, Width 2.7cm ± 0.03cm, Cap Height 0.4cm ± 0.02cm, Cap Thread Pitch 2mm, Threading Length 13 mm, secondary packaging - Low Density Polyethylene for each bottle, dimensions: 23.5 ± 1.3cm x 12.7 ± 0.6cm

Medicinal Substances There are no medicinal substances used in the device.

Biological Safety No materials of animal or human origin used in the device.

Requirements to medical product maintenance and repair: Not applicable.

Disinfection and cleaning: Not applicable.

Labeling. Devices are supplied in packaging with labeling containing the following information: name, lot and ref number; manufacturer; package content; shelf life; as well as symbols - Consult instructions for use, CE Mark, Temperature limitation. Keep away from sunlight. Do not re-use, single use, Recyclable packaging. Low density polyethylene packaging.

Disposal. Packaging/contents should be disposed in accordance with the local/regional/national/international regulations. Medical products should be disposed by the local public law organizations in accordance with SanPiN 2.1.7.2790-10 "Sanitary and Epidemiological Requirements for Medical Waste Treatment" waste Class B).

Warranty. 3M warrants this product will be free from defects in material and manufacture. 3M MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible

Transbond Adhesives, Kits and Refills with Accessories

for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M sole obligation shall be repair or replacement of the 3M ESPE product.

The name and address of the manufacturer are provided below. 3M Unitek Corporation, 2724 South Peck Road, Monrovia, California, 91016, USA

Authorized representative in RF: 3M Russia CJSC, 121614, Moscow Krylatskaya Street, 17, Unit 3, tel: +7 (495) 784 7474 / fax: +7 (495) 784 7475, e-mail: 3mra@mmm.com

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