To Whom It May Concern

**Registration documents for Ketac Cem Easymix
- Information for Use (IFU) -**

We, 3M Deutschland GmbH, Carl-Schurz-Str. 1, 41453 Neuss, herewith confirm that the following pages are an excerpt of the Information for Use.

Seefeld, February 28, 2017

Yours Sincerely,

3M Deutschland GmbH
3M Oral Care

Elfriede Zedler
Regulatory Affairs

Ketac Cem Easymix Glass Ionomer Luting Cement

Instructions for Use

ENGLISH

Product Description Ketac Cem Easymix is a luting cement for mixing by hand.

The instructions for use on the product concerned must be kept available for the length of time it is employed.

Indication:

- Cementation of inlays, onlays, crowns, and bridges made of metal, with metal frameworks, made of high-strength ceramic or composite, provided they are suitable for conventional cementation
- Cementation of crowns and bridges made of metal, with metal frameworks, made of high-strength ceramic or composite, provided they are suitable for conventional cementation, on implant abutments
- Cementation of posts and screws made of metal or high-strength ceramic, provided they are suitable for conventional cementation
- Cementation of orthodontic bands

Precautionary Measures

3M MSDSs can be obtained at www.mmm.com or from your local subsidiary.

Preparation

- To provide for optimal adhesion, carefully clean the surfaces of the tooth and restoration to be cemented.
- Blow the preparation dry in only 2-3 short intervals with water and oil-free air or dab dry with cotton pellets. Do not over-dry!

The preparation should be just so dry that the surface has a matt shiny appearance. Excessive drying with compressed air can negatively affect adhesion or result in post-operative sensitivity after seating the restoration.

- The surfaces of the restoration should be dried well with compressed air.
- Avoid further contamination by saliva.

Pulp Protection

The glass ionomer cement must not be applied directly to dentin - e. g. on inlays - situated close to the pulp or to the exposed pulp. Prior to taking the impression, cover areas close to the pulp using a hard-setting calcium hydroxide preparation.

Dosage

- Shake bottle to loosen powder.
- Screw off cap and replace by dispensing dropper.
- For cementing work, the standard powder-to-liquid mixing ratio is 3.8:1 w/w, this corresponding to 1 level spoonful of powder to two drops of liquid.
- Skim off spoon on plastic segment.
- Do not compress the powder.
- Dose powder and liquid next to each other on a mixing pad or glass slab.
 - Keep bottle upright while dispensing.
 - Crystallized liquid must not be allowed to form on the dispensing dropper.
- Carefully seal bottles after use. If the mix is too thin (due to insufficient dosing of powder) this may produce symptoms of pulpitis.

Mixing

Working of Ketac Cem Easymix should proceed at a room temperature of 20 - 25° C.

- Mix using a metal or plastic cement spatula.
- Add the powder to the liquid in one portion.
- On principle a sufficient quantity of powder should be added to the liquid to produce viscous consistency.
- The quantity of cement required for a crown should just about drip from the spatula.
- Continue to smooth out the paste until a homogeneous mixture is obtained.

Application

During application, water and saliva must be prevented from entering the working area.

Apply a thin coat of cement to the surface of the restoration to be cemented and to the preparation. Subsequently seat the restoration.

- Avoid overfilling the restoration. Applying hydrostatic pressure buildup in tightly fitting restorations may cause pulp discomfort.

Operating conditions

The following time scale applies at a room temperature of 23° C and 50% relative atmospheric humidity:

	min: sec
Mixing	0:30
Application incl. mixing	3:10
Setting from begin of mix	7:00

Higher temperatures shorten the working time, whereas lower temperatures will extend this period (e.g. mixing on a cooled glass slab). A higher amount of powder also brings about a shorter working time. Exceeding the working time causes loss of adhesion to enamel and dentine.

Removal of Excess

Remove excess using a Heidemann spatula and /or probe 6-8 min. after beginning of mix.

After Cementing

Carefully check the sulcus of the treated teeth and the surrounding areas; remove any cement residues still remaining. This is especially important when cementing to implant abutments.

Storage and Shelf Life

Keep the powder in a dry place away from damp.

Carefully reseal the powder bottle after each use.

Store the product at 15-25° C/59-77° F.

Do not use after the expiry date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Information valid as of July 2012

RUS: Authorised representative of the manufacturer in RF: ZAO "3M Russia", Russia 121614,

Moscow, Krylatskaya str. bldg. 17/3 T. +74957847474

3M Deutschland GmbH
Dental Products
Carl-Schurz-Str. 1
41453 Neuss - Germany

3M, ESPE and Ketac are trademarks of 3M or 3M Deutschland GmbH.
© 2014, 3M. All rights reserved.

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