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I, Celine Curran, Regulatory affairs specialist, Abbott Ireland Diagnostics Division, Finisklin Business Park, Sligo, Ireland, hereby certify that the attached documents that keep the number of pages indicated below are the Insert for Use. These documents are in a full accordance with actual manuals of manufacturer.

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Date: 27 May 2019

SLIGO CHAMBER OF COMMERCE (INC)

Celine Curran,
Regulatory affairs specialist,
Abbott Ireland Diagnostics Division,
Finisklin Business Park,
Finisklin,
Sligo,
Ireland

CERTIFIED CORRECT IN ALL GOOD FAITH

Cerelaine Courtney 27/05/19 **Authorised Official**

**Abbott Ireland Diagnostics Division
Finisklin Business Park
Sligo
County Sligo
Ireland**

**SLIGO CHAMBER OF COMMERCE
CERTIFIED CORRECT IN ALL GOOD FAITH**

Created August 2016.

NAME

Alinity i CMV IgM Controls (also referred to as CMV IgM Ctrls)

INTENDED USE

The Alinity i CMV IgM Controls are for the estimation of test precision and the detection of systematic analytical deviations of the Alinity i analyzer (reagents, calibrators and instrument) when used for the qualitative detection of IgM antibodies to Cytomegalovirus in human serum and plasma.

For additional information, refer to the Alinity i CMV IgM reagent package insert and the Alinity ci-series Operations Manual.

CONTENTS

CONTROL - contains recalcified human plasma.

CONTROL + contains recalcified human plasma reactive for anti-CMV IgM.

Preservatives: ProClin 950 and sodium azide.

The controls are at the following ranges and target concentrations:

		Anti-Cytomegalovirus IgM			
		CONC	RANGE		
		INDEX	INDEX		
Control	Quantity	Color	S/CO	S/CO	
CONTROL -	1 x 4.0 mL	Natural	-	0 - 0.60	
CONTROL +	1 x 4.0 mL	Natural	2.30	1.15 - 3.45	

NOTE: The insert ranges for the controls are not lot specific and represent the total range of values which may be generated throughout the life of the product. It is recommended that each laboratory establish its own means and acceptable ranges which should fall within the package insert ranges. Sources of variation that can be expected include:

- Calibration
- Control lot
- Reagent lot
- Calibrator lot
- Instrument

PRECAUTIONS

- **IVD**
- For *In Vitro* Diagnostic Use

Safety Precautions

-  **CAUTION:** This product contains human-sourced and/or potentially infectious components. Refer to the CONTENTS section of this package insert. No known test method can offer complete assurance that products derived from human sources or inactivated microorganisms will not transmit infection. Therefore, all human-sourced materials should be considered potentially infectious. It is recommended that these reagents and human specimens be handled in accordance with the OSHA Standard on Bloodborne Pathogens, Biosafety Level 2 or other appropriate biosafety practices should be used for materials that contain or are suspected of containing infectious agents.¹⁻⁴
- The human-sourced material used in the negative control is nonreactive for anti-CMV IgM, HBsAg, HIV-1 RNA or HIV-1 Ag, anti-HIV-1/HIV-2 and anti-HCV.
- The human-sourced material used in the positive control is reactive for anti-CMV IgM, and nonreactive for HBsAg, HIV-1 RNA or HIV-1 Ag, anti-HIV-1/HIV-2 and anti-HCV.

The following warnings and precautions apply to:

CONTROL - and **CONTROL +**

WARNING	Contains methylisothiazolones and sodium azide.
H317	May cause an allergic skin reaction.
EUH032	Contact with acids liberates very toxic gas.

Prevention

P261	Avoid breathing mist / vapors / spray.
P272	Contaminated work clothing should not be allowed out of the workplace.
P280	Wear protective gloves / protective clothing / eye protection.

Response

P302+P352	IF ON SKIN: Wash with plenty of water.
P333+P313	If skin irritation or rash occurs: Get medical advice / attention.
P362+P364	Take off contaminated clothing and wash it before reuse.

Disposal

P501	Dispose of contents / container in accordance with local regulations.
------	---

Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

For a detailed discussion of safety precautions during system operation, refer to the Alinity ci-series Operations Manual, Section 8.

PREPARATION FOR USE

- This product is liquid ready-to-use.
- This product may be used immediately after removal from 2 to 8°C storage.
- Prior to each use, mix by gentle inversion.

STORAGE

- Do not use past expiration date.

	Storage Temperature	Maximum Storage Time	Additional Storage Instructions
Unopened	2 to 8°C	Until expiration date	
Opened	2 to 8°C	Until expiration date	Store tightly capped. Return to refrigerated storage after use.

INSTRUMENT PROCEDURE

- To obtain the recommended volume requirements for the controls, hold the bottle vertically, and dispense 4 drops of the negative control and 4 drops of the positive control into each sample cup in the assigned position.
- For instructions on ordering and loading controls on the instrument, refer to the Alinity ci-series Operations Manual, Section 5.

INDICATIONS OF INSTABILITY OR DETERIORATION

Instability or deterioration should be suspected if there are precipitates, visible signs of leakage, turbidity, or if controls do not meet the appropriate package insert and/or Alinity ci-series Operations Manual criteria.

BIBLIOGRAPHY

1. US Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030, Bloodborne pathogens.

2. US Department of Health and Human Services. *Biosafety in Microbiological and Biomedical Laboratories*. 5th ed. Washington, DC: US Government Printing Office; December 2009.

3. World Health Organization. *Laboratory Biosafety Manual*. 3rd ed. Geneva: World Health Organization; 2004.

4. Clinical and Laboratory Standards Institute (CLSI). *Protection of Laboratory Workers From Occupationally Acquired Infections; Approved Guideline—Fourth Edition*. CLSI Document M29-A4. Wayne, PA: CLSI; 2014.

Note for number formatting:

- A space is used as thousands separator (example: 10 000 specimens).
- A period is used to separate the integer part from the fractional part of a number written in decimal form (example: 3.12%).

Key to Symbols

	Caution
	Consult instructions for use
	Manufacturer
	Temperature limitation
	Use by/Expiration date
	Control Number
	Concentration
	Contains Sodium Azide. Contact with acids liberates very toxic gas.
	Negative Control
	Positive Control
	Index
	In Vitro Diagnostic Medical Device
	Lot Number
	Product of Ireland
	Range
	List Number

Alinity is a trademark of Abbott Laboratories in various jurisdictions. All other trademarks are property of their respective owners.

 Abbott Ireland
Diagnostics Division
Finisklin Business Park
Sligo
Ireland
+353-71-9171712

Customer Service: Contact your local representative or find country-specific contact information on www.abbottdiagnostics.com

Created August 2016.

©2016 Abbott Laboratories

ADDENDUM

to the Instruction for Use Alinity i CMV IgM Controls for Russia

SHIPPING AND STORAGE CONDITIONS

Recommended shipping conditions for devices are 2-8°C. Medical devices must be stored in refrigerators/refrigerator chambers to ensure regulated temperature conditions with a daily temperature record.

Medical devices must be carried out in single-use thermal containers containing coolant elements, or in reusable thermo containers with automatically maintained temperature or in refrigerated trucks using thermal indicators.

Device is stable until the expiration date indicated on the package. Device is no longer suitable for use if shipping temperature limits have been violated. Ship in an any kind of enclosed carrier vehicle in accordance with shipping rules, applicable to the respective type of transport.

WARRANTY

Manufacturer guarantees that device remains stable until the expiration date when stored and handled as directed. Manufacturer set the maximum shelf life of

Alinity i CMV IgM Controls – 12 months from date of manufacturing

The dating of each Alinity i CMV IgM Reagent Kit, Calibrators, and Controls master lot is determined by the kit component with the shortest expiration dating.

Device mustn't be used if stored with deviations in manufacturer's requirements (GOST R 51088-2013).

MAINTENANCE

Not required.

REPAIRS

Not required.

LIST OF APPLICABLE STANDARDS

EN ISO 9001
ISO 13485
EN ISO 14971
EN 13641
EN 13612
EN ISO 15223-1
EN 980
EN ISO 18113-1
EN ISO 18113-2
EN ISO 18113-3
EN ISO 23640*
EN ISO 20776-1
EN 13975
EN ISO 17511

POTENTIAL USER

The Alinity i CMV IgM assay is intended for use in a variety of hospital clinical laboratory environments including, but not limited to; hospitals and commercial laboratories, whose personnel have been trained to use the Alinity i analyzer.

INFORMATION ABOUT PRESENCE OF DRUGS IN COMPOSITION OF MEDICAL DEVICE

Does not contain any drugs

STERILIZATION

Medical device is not required to be sterile.

INFORMATION ABOUT POSSIBILITY OF MULTIPLE APPLICATION

Medical device is intended for single usage

TECHNICAL CHARACTERISTICS

Density at 20 °C: 1,028 g/cm³

pH-value at 20 °C:

- Negative Control: 7,3
- Positive Control: 7,3

COMPATIBILITY RESTRICTIONS OF MEDICAL DEVICE

No available data.

MATERIALS OF ANIMAL/HUMAN ORIGIN

Materials of animal/human origin are tested and risks of using MD are indicated in SDS. Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

INDICATIONS

Not applicable.

CONTRAINDICATIONS

Not applicable.

QUALITY CONTROL PROCEDURES

The recommended control requirement for the Alinity i CMV IgM assay is that a single sample of each control level be tested once every 24 hours each day of use. Additional controls may be tested in accordance with local, state, and/or federal regulations or accreditation requirements and your laboratory's quality control policy.

To establish statistically-based control limits, each laboratory should establish its own concentration target and ranges for new control lots at each clinically relevant control level. This can be accomplished by assaying a minimum of 20 replicates over several (3-5) days and using the reported results to establish the expected average (target) and variability about this average (range) for the laboratory. Sources of variation that should be included in this study in order to be representative of future system performance include:

- Multiple stored calibrations
- Multiple reagent lots
- Multiple calibrator lots
- Multiple processing modules (if applicable)
- Data points collected at different times of the day Refer to published guidelines for information or general control recommendation, for example Clinical and Laboratory Standards Institute (CLSI) Document C24-A3 or other published guidelines, for general quality control recommendations.
- If more frequent control monitoring is required, follow the established quality control procedures for your laboratory.
- If quality control results do not meet the acceptance criteria defined by your laboratory, sample results may be suspect. Follow the established quality control procedures for your laboratory. Recalibration may be necessary. For troubleshooting information, refer to the Alinity ci-series Operations Manual, Section 10.
- Review quality control results and acceptance criteria following a change of reagent or calibrator lot.

ANALYTICAL CHARACTERISTICS

DESCRIPTION	PRODUCT REQUIREMENT
Within Laboratory (Total) Precision	• $\leq 10\%$ CV for positive samples within the range of 1 to 4 times the cutoff value, which is the medical decision point

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- US Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030, Bloodborne pathogens.
- US Department of Health and Human Services. *Biosafety in Microbiological and Biomedical Laboratories*. 5th ed. Washington, DC: US Government Printing Office; December 2009.
- World Health Organization. *Laboratory Biosafety Manual*. 3rd ed. Geneva: World Health Organization; 2004.
- Clinical and Laboratory Standards Institute (CLSI). *Protection of Laboratory Workers From Occupationally Acquired Infections; Approved Guideline—Fourth Edition*. CLSI Document M29-A4. Wayne, PA: CLSI; 2014.

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