Registered No. 2018 - 3414

NOTARIAL CERTIFICATE

Seowon Law & Notary Office Inc.

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Name

Title

Instruction for Use for Medical Device

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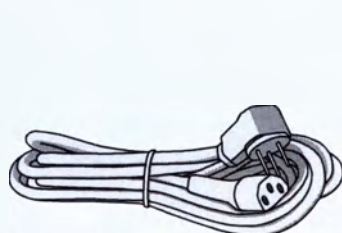
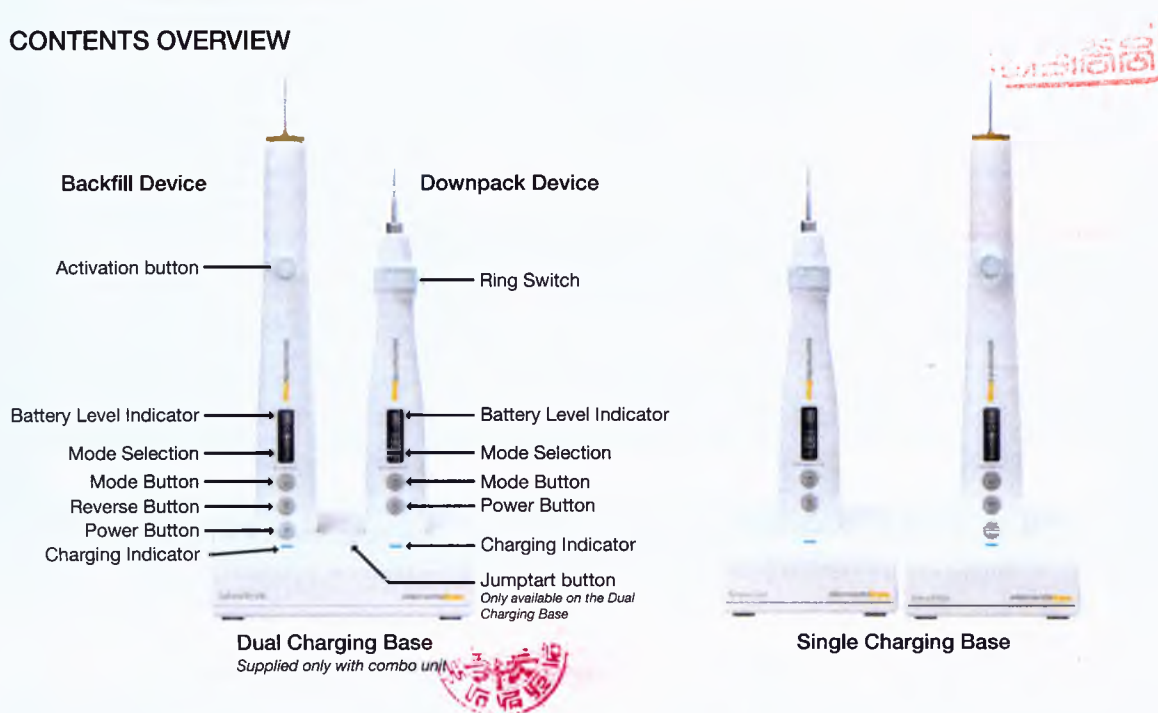
elementsTMfree


OBTURATION SYSTEM

The cordless Elements™ Free obturation system combines a breakthrough Downpack heat source with a world-class Backfill extruder. The Downpack device provides instantaneous heating and cooling of the heat plugger with precisely controlled temperature and timing—making it perfect for single-motion downpack obturation and creating post space. Utilizing single-use gutta percha needle cartridges, the ergonomic Backfill device has a motorized extruder system for precise temperature and speed control for a 3-dimensional fill when backfilling canals.


CONTENTS OVERVIEW


Power Cord

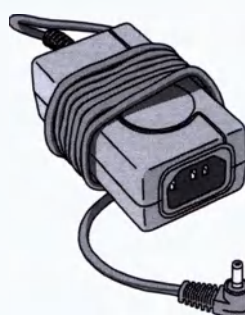Transformer

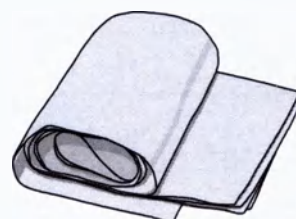
Sleeve Covers (5 PCS)

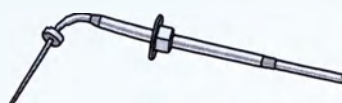Heated Plugger
Supplied only with Downpack and combo unit

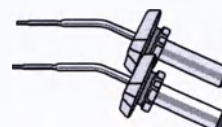Needle Bender
Supplied only with Backfill and combo unit

Backfill Cartridges (2 PCS)
Supplied only with Backfill and combo unit

⚠ WARNING

The System B tips and Backfill cartridges are designed to reach high temperatures. Inadvertent contact with patient and operator must be avoided to prevent accidental burns.

Handle empty extruder cartridge by plastic locknut only. Other portions could be hot and could result in serious burns.

Extruder cartridges are for single-patient use.

This unit should NOT be used on a patient with a pacemaker.

Only use the specified SybronEndo AC power supply and power cords.

Use of other accessories that are not authorized for use in connection with this device may cause malfunction and compromise patient safety.

No modification of this equipment is allowed.

⚠ CAUTION

This device has been investigated with regard to safety from electrical shock and fire hazard as well as electromagnetic compatibility (EMC). The device has not been investigated for other physiological effects. Please contact SybronEndo if you have any further questions regarding electrical safety or EMC.

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This device has been tested and found to comply with EMC limits for the Medical Device Directive 93/42/EEC. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation. The device generates radio frequency energy and, if not installed and used in accordance with these instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. If this device does cause harmful interference with other devices, which can be determined by turning the device off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving device.
- Increase the separation between devices.
- Connect the device into an outlet on a circuit different from that to which the other device(s) is connected.
- Consult the manufacturer for help.

To reduce the risk of electrical shock, do not remove cover. Refer servicing to qualified service personnel.

U.S. federal law restricts the sale of this device by or on the order of a healthcare professional.

Remove main plug from wall socket to isolate unit from main power.

Position device such that it can be easily unplugged from the wall.

Symbols

	Serial Number		European Representative
	Manufacturer		Corresponds to MDD 93/42 EEC Including EN 60601-1 and EN 60601-2
	Manufacturing Date		CSA mark with "C/US" indicator for certified products
	This way up		Caution/Warning
	Fragile, handle with care		Consult accompanying documents
	Keep away from rain		Type BF Patient Applied Part
	Temperature limits for storage and transportation		Class II Medical Electrical Equipment
	Humidity Limitation		Do no autoclave
	Pressure Limitation		Part Number
	Do not throw into trash		Caution: Hot!

GETTING STARTED

1. INITIAL SET-UP

- Remove the device(s), Charging Base, transformer and wall cords from their packaging.
- Connect the female end of the power cord into the transformer. Then plug the ends into the wall socket and the Charging Base.
- On the bottom of the device(s), make sure the switch is set to the "Unlocked" position.
- Place the handpiece(s) into the Charging Base, and confirm that they are firmly seated.

2. CHARGING THE UNITS

- Once the handpieces are firmly seated, the indication lights on the Charging Base will be either orange or blue.
- The light will be orange when the battery is charging and will turn blue when the battery is fully charged.

ORANGE Orange means the unit is charging **BLUE** Blue means the unit is fully charged

3. POWERING ON THE UNITS

- To power on the device(s), press the Power button "⏻".
- If you have the Dual Charging Base, and both units are seated firmly in the base, press the Jumpstart button "⏻" to power on both units at once.

4. SET TEMPERATURE MODE

- Devices come with the three preset temperature modes. The following table shows their function and default temperature settings:

Setting	Description	Default Temp.
GUTTA PERCHA	For use with Gutta Percha	200°C
SYNTHETIC	For use with synthetic filling materials	150°C
CUSTOM	For use as an additional preset.	200°C

- Press the Mode button to toggle to the appropriate preset.

For instructions on changing the default temperature for a preset, see "Changing Temperature Presets" in the ADVANCED SETTINGS section.

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DOWNPACK

1. INSTALLING THE TIP

- Index the hex nut to slide into one of six positions available.
- Install the tip until it is fully seated.

	CAUTION	
	Older System B tips without this hex nut or other manufacturer's tips may not operate correctly and may damage the unit.	

2. ACTIVATING THE DOWNPACK

Activate the handpiece by depressing and holding the Ring switch. The plugger will heat instantly and a light under the Ring switch will illuminate.

CAUTION:
Tip will be hot

	NOTE
	- To avoid overheating, the plugger will only remain heated up to a maximum of approximately 4 seconds regardless of how long the button is depressed. - Indicated temperature is the temperature the tip of the plugger reaches after activation in air. As the tip contacts other materials, its temperature may decrease depending on the nature of the contact.

	CAUTION
	The heat pluggers must be autoclaved between every patient to prevent cross-contamination. See the instructions in the CLEANING AND STERILIZATION section.

BACKFILL

1. INSERTING CARTRIDGE

- a. Insert the large end of the Elements Cartridge into the handpiece and rotate a quarter turn clockwise (CW) until it engages.

	CAUTION: HOT! When heated at the peak temperature for an extended period of time, plastic housing surface may exceed 85° C. Accordingly, it is recommended to turn the device off after 15 minutes of use and allow it to cool.
--	---

	CAUTION Elements Cartridges are SINGLE USE ONLY and should only be used on one patient. Re-use may lead to cross-contamination between patients.
--	--

2. HOW TO TELL WHEN THE BACKFILL IS READY FOR USE

- a. Once powered on, the Backfill unit will automatically begin to heat. The illuminated Activation button will indicate whether or not the device is ready for use.

Flashing Light:
Device is still heating and is NOT ready for use.

Steady Light:
Device is fully heated and ready for use.

3. ACTIVATING THE BACKFILL

Press & hold

- Manual Run Mode:**
a. Hold down the activation button to engage motor.
b. Release activation button to stop.

"Click" "Click"

- Continuous Run Mode (CR):**
a. Double-Click to engage CR mode.
b. Press button once to stop.

	NOTE Before placing the cartridge needle in the canal, activate the motor until material extrudes out of the needle. If the motor is stopped before the cartridge is empty, the plunger will retract slightly to prevent excess material from discharging from the needle. Once the plunger reaches the end of the stroke, the cartridge is empty and the plunger will automatically retract.
--	---

4. RETRACTING THE PLUNGER

- a. To retract the plunger before it reaches the end of its stroke, press the Reverse button. This will cause the plunger to fully retract to its "home" position.

5. REPLACING THE CARTRIDGE

- a. To replace a cartridge before it is completely empty, first ensure that the unit is fully heated, then press the Reverse button to retract the plunger.
b. Remove the cartridge by turning the nut 90° counterclockwise (CCW), then pull the cartridge out of the handpiece and dispose of properly.

	CAUTION: HOT! Handle empty cartridges by the plastic locknut only. Other portions could be hot and could result in a serious burn.
--	--

ADVANCED SETTINGS

1. CHANGING TEMPERATURE PRESETS

The unit has three different preset temperature modes. (See the table in Step 4 of the GETTING STARTED section)

- a. Press the Mode button to toggle to the temperature setting you would like to adjust.
b. Hold down the Mode button for approximately 2 seconds to enter Temperature Control Mode.
c. Press the Mode button repeatedly to change temperature.
d. Once desired temperature is reached, hold down the Mode button (approx. 2 seconds) again to exit Temperature Control Mode.

Hold 2 Seconds

	NOTE - When the temperature setting on the GUTTA PERCHA or SYNTHETIC modes is changed from the default setting, an asterisk "*" will appear next to the temperature. - On the Backfill device, only the CUSTOM temperature setting can be adjusted.
--	--

	CAUTION (BACKFILL ONLY) Keep temperature setting above 150° C. Extruding material at lower temperatures will create more stress on the motor, and may shorten the device's life.
--	--

2. ADJUSTING THE VOLUME

- a. Change the volume level by simultaneously holding down the Mode and Power buttons for approximately 2 seconds. This will open volume control mode.
b. Press the Mode button repeatedly to change the volume level.
c. Once you have set your desired level, exit volume control mode by repeating the first step (a) above.

Hold 2 Seconds

MAINTENANCE

1. CHARGING THE DEVICE(S)

The handpieces should be placed onto the Charging Base during and between uses. This will ensure that the batteries are maximally charged.

2. REPLACING THE BATTERY

Contact Customer Support if the battery appears to be malfunctioning. To replace battery:

- Remove screw on battery cover and remove battery.
- Install new battery in same orientation.
- Dispose of used battery per local regulations.

CAUTION

Only use SybronEndo batteries designated for this system. Use of other batteries may damage the device(s) or cause a malfunction.

3. POWER LOCK FEATURE

The device(s) can be locked in a power-off mode by sliding the switch on the base from the "unlock" position to the "locked" position (See Step 1 in GETTING STARTED).

CLEANING AND STERILIZATION

1. CLEANING THE DEVICE(S) AND CHARGING BASE

Clean all the surfaces with a cloth lightly moistened with isopropyl alcohol. To prevent cross-contamination, single-use barriers can be used on the hand pieces. SybronEndo recommends TotalCare's Pinnacle Syringe Sleeves (REF 3760).

CAUTION

- The electronic components should not be autoclaved as it will damage the circuitry.
- Do not spray devices with any liquids as it may damage the circuitry.
- Do not allow liquid to collect in the Charger base or come in contact with the connectors it may damage the circuitry.

2. CLEANING THE HEATED PLUGGERS

Clean with water and mild, nonabrasive detergent such as dishwashing liquid. Dry thoroughly. The heat pluggers must be autoclaved between every use. To avoid damaging the pluggers, place them in the System B tip stand before autoclaving. Operate sterilization cycle at 250°F (121°C) for 30 minutes or 275°F (134°C) for 6 minutes.

CAUTION

Do not place pluggers into an ultrasonic bath as it may damage the unit.

3. CLEANING THE CARTRIDGE NEEDLE BENDER

The cartridge needle bender should be wiped down with a cloth moistened with isopropyl alcohol. Sterilize by steam autoclave at 250°F (121°C) for 30 minutes or 275°F (134°C) for 6 minutes.

TROUBLESHOOTING GUIDE AND TECHNICAL SUPPORT

If a problem occurs with your device, refer to the guide below. If this guide does not resolve the problem, contact Customer Care at 800-346-3636 or 714-516-7979 (Available from 6:00am to 5:00pm PST). Outside of these hours, email us at customercare@sybrondental.com. Alternatively, contact your dealer or SybronEndo sales representative.

Problem	Possible Causes	Solution
Device(s) do not turn on	Lock switch on base of unit in "locked position"	Slide Power Lock switch to "unlocked" position. (See Step 1 in GETTING STARTED section)
	Battery out of charge	Place the devices on the Charging Base.
Charging base light(s) do not turn on	Base is not getting power	Ensure wall outlet has power. Check and adjust all connections: wall-outlet plug, power cord plug into transformer and the small plug into the Charging Base.
	Device(s) may not be fully seated	Gently wiggle the devices to ensure proper seat.
Heat plugger on Downpack unit is not heating	Heat plugger not seated into Downpack unit	See Step 1 in DOWNPACK section.
	Heat plugger tip is burned out	Replace tip.
	Battery is out of charge	Check battery level on display. If battery level indicator is low, place the device on the Charging Base.
Device(s) not heating	Battery is out of charge	Check battery level on display. If battery level indicator is low, place the device on the Charging Base.
Filling material/gutta percha do not extrude	Motor is stuck or plugged due to gutta percha in the unit.	Let the device heat up. Press Reverse button. DO NOT disassemble the device(s).
	Temperature not set correctly	Adjust temperature setting. See Step 4 in the GETTING STARTED section; See "Changing Temperature Presets" in the ADVANCED SETTINGS section.
	Cartridge is empty or defective	Replace cartridge.
Unable to remove used cartridge from Backfill device	Plunger inside the Backfill device is stuck.	Heat up the device. Press Reverse button. DO NOT disassemble the device.
The light on charger fails to turn from orange to blue	Battery may be near end-of-life	Wait at least 4 hours. If unit fails to reach full charge, replace battery. Only use SybronEndo battery.
Battery becomes exhausted quickly	Battery may be near end-of-life	Replace battery. Only use SybronEndo battery. Note: On the Backfill device, battery has just enough power for one procedure, roughly 15 minutes and one cartridge. It is recommended to place the device on the Charging Base between uses.

PARTS

Elements Free Obturation System	973-0500-TYPEX*
Downpack Unit	973-0502-TYPEX*
Backfill Unit	973-0504-TYPEX*
Dual Charger	973-0510
Single Charger	973-0512
Transformer	973-0515
Power Cord	973-0516-TYPEX*
Battery	973-0520

Heated Pluggers

Medium Large (ML)	952-0007
Medium (M)	952-0006
Fine Medium (FM)	952-0005
Fine (F)	952-0004
Extra-Fine (XF)	952-0031

Elements Cartridges

Gutta Percha, Medium Body, 23 Gauge	972-1002
Gutta Percha, Heavy Body, 23 Gauge	972-1005
Gutta Percha, Light Body, 25 Gauge	972-1003


SPECIFICATIONS

- Battery: 3.7 V, 1700 mAh Li-Ion, 6.3 Wh
- Adapter: Input: 100-240V~, 50/60 HZ, Output 9V = 2A
- Downpack: ~155mm x 28mm x 28mm (L x W x H), ~98 g
- Backfill: ~195mm x 28mm x 28 mm (L x W x H), ~155 g
- Device: IEC 601-1 Class II, Type BF Device
- Mode of Operation: Continuous Operation

*X designates plug type.

Type B		
Type F		
Type G		
Type I		
Type N		

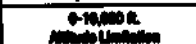CONDITIONS

Storage and Transport Conditions:	Conditions for use:
 Temperature	 Temperature
 Relative Humidity	 Relative Humidity
 Atmospheric Pressure	 Atmospheric Pressure
	 Altitude Limitation

WARRANTY

SybronEndo warrants the system (excluding batteries) to be free from defects in materials or workmanship for period of 2 years from the original date of purchase. The batteries are warrantied for a period of 6 months from the original date of purchase. If the system shows any defect within the warranty period that are not excluded from this warranty, SybronEndo shall, at its sole discretion, either replace or repair the device using suitable new or reconditioned parts. In the case other parts are used which constitutes an improvement, SybronEndo may, at its discretion, charge the customer for the additional cost of these parts. If the warranty claim provides to be justified, the product will be returned to the user freight prepaid. Warranty claims other than those indicated herein, are expressly excluded.

EXCLUSIONS

Damage and defects caused by the following conditions are not covered by the warranty:

- Improper handling/disassembly/modifying, neglect, or failure to operate the unit in compliance with the instructions given in this manual.
- Force majeure or any other condition that is beyond the control of SybronEndo.
- Damage caused by customer misuse or uses other than those specified.

DISCLAIMER

For safety reasons, this product should be used with accessories manufactured and sold by SybronEndo. Any use of non-authorized accessories or not following any of the instructions for use is done so at the operator's risk and voids the warranty. SybronEndo does not assume any responsibility for incorrect diagnosis due to operator error or equipment malfunction.

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Lee Dong-ho
attorney-in-fact of
META BIOMED CO., LTD.
PRESIDENT/ Suk Song Oh
appeared before me and
admitted said principal's
subscription to the attached
Instruction for Use for Medical Device,
Cordless dental elementsfree Obturation
System for root canal obturation

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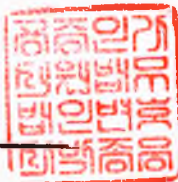
This is hereby attested on
this 3rd day of AUG, 2018
at this office.

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Seowon Law & Notary Office Inc.
Belong to Cheongju District Prosecutors' Office
84, Wonheong-ro, Seowon-gu,
Cheongju, Korea

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PARK IONG WOONG
Signature of the Notary Public

This office has been authorized
by the Minister of Justice, the
Republic of Korea, to act as
Notary Public since DEC, 3. 1996
under Law NO. 9416.

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NOTARIAL CERTIFICATE

Seowon Law & Notary Office Inc.

84, Wonheong-ro, Seowon-gu, Cheongju, Korea

TEL:(043) 288-7700 · FAX:(043) 288-7710

Approved by

Meta Biomed Co., Ltd, Korea

Date: August 3, 2018

Suk sang 

Name

Title

Supplement to the IFU for Medical Device
Cordless dental elementsfree Obturation System for root canal obturation

Only for using on the territory of Russian Federation. Provided on demand

Rev.1-18

Table of content

1. Name of the Medical Device	3
2. Description of Parts	4
3. Indications	6
4. Contraindications	6
5. Potential Complications/Side Effects	6
6. Technical Specification	7
7. Terms of Acceptance and Types of Test	8
8. Disposal Conditions	12
9. Applied Standards	12
10. Contact Information	13

Only for using on the territory of Russian Federation. Provided on demand

Rev.1-18

1. Name of the Medical Device

Cordless dental elementsfree Obturation System for root canal obturation

973-0500 TYPE F	elementsfree Obturation System: — Downpack Unit with Li-ion battery 3.7 V; — Backfill Unit with Li-ion battery 3.7 V; — Dual Charging Base; — Transformer; — Power Cord; — Sleeve Covers – 1 bag (5 pcs); — Heated Buchanan Plugger (Fine, .06 Taper); — Needle Bender Tool; — Gutta Percha Cartridges – 2 pcs; — Instruction for use.
973-0502 TYPE F	elementsfree Obturation Downpack: — Downpack Unit with Li-ion battery 3.7 V; — Single Charging Base; — Transformer; — Power Cord; — Sleeve Covers – 1 bag (5 pcs); — Heated Buchanan Plugger (Fine, .06 Taper); — Instruction for use.
973-0504 TYPE F	elementsfree Obturation Backfill — Backfill Unit with Li-ion battery 3.7 V; — Single Charging Base; — Transformer; — Power Cord; — Sleeve Covers – 1 bag (5 pcs); — Needle Bender Tool; — Gutta Percha Cartridges – 2 pcs; — Instruction for use.

Only for using on the territory of Russian Federation. Provided on demand

Rev.1-18

2. Description of Parts


The cordless elementsfree Obturation System combines a breakthrough Downpack heat source with a world- class Backfill extruder. The Downpack device provides instantaneous heating and cooling of the heat plugger with precisely controlled temperature and timing making it perfect for single-motion downpack obturation and creating post space. Utilizing single-use gutta percha needle cartridges, the ergonomic Backfill device has a motorized extruder system for precise temperature and speed control for a 3-dimensional fill when backfilling canals.

Contents overview


Only for using on the territory of Russian Federation. Provided on demand

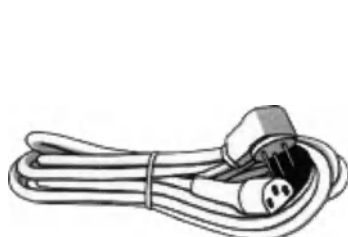
Rev.1-18

Power Cord

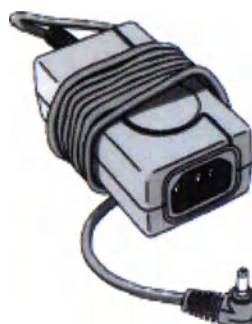transformer

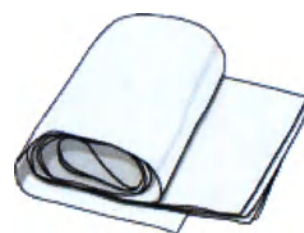sleeve Covers (5 PCs)

Heated Plugger
Supplied only with Downpack
and combo unit

needle Bender
Supplied only with Backfill
and combo unit

Backfill Cartridges (2 PCs)
Supplied only with Backfill
and combo unit

Power Cord

The power cord provides power to the device. The power cord is packaged in a polybag. There are various types to accommodate different global regions.

For Russia, it is applicable to use TYPE F.

	*X designates plug type.		
973-0516-TYPEB	Type B		
973-0516-TYPEF	Type F		
973-0516-TYPEG	Type G		
973-0516-TYPEI	Type I		
973-0516-TYPEN	Type N		

WARNING! To avoid risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

Transformer

The transformer is used in conjunction with the power cord to provide power to the device. It's packaged in a cardboard box.

Only for using on the territory of Russian Federation. Provided on demand

Sleeve Covers

The sleeves are used to cover the Downpack and Backfill handpieces to provide an additional level of protection against cross-contamination.

Heated Buchanan Plugger

The heated Buchanan plugger is used with the Downpack handpiece and is used to warm and condense the gutta percha in the canal. There are various taper sizes to accommodate various preparation sizes. The .06 (Fine) Heat Plugger is provided with the unit.

Additional sized heat pluggers are also available. These are size .04 (Extra Fine Taper), .08 (Fine/Medium Taper), .10 (Medium Taper), and .12 (Medium/Large Taper).

Needle Bending Tool

The Needle Bending Tool is used to bend the needle of the gutta percha cartridge to shape the needle to the fit the root canal anatomy.

Gutta Percha Cartridge

The Gutta Percha Cartridge is used with the Backfill Unit. The unit warms the gutta percha in the cartridge, which is then extruded directly into the root canal. There are various needle gauges and viscosities of the cartridge. These are 23 gauge (medium body), 23 gauge (heavy body) and 25 gauge (light body).

Gutta Percha Cartridges – 2 pcs., which are part of 2 delivery kits: elementsfree Obturation System (ref. 973-0500 TYPE F) and elementsfree Obturation Backfill (ref. 973-0504 TYPE F) are intended to informative/educational use only and shouldn't be used on human patients.

Warning:

- ⚠ For practical use, a clinician should order Gutta Percha Cartridges separately.
- ⚠ Extruder cartridges are for single-patient use.

3. Indications

elementsfree Obturation System is used for root canal obturation under the endodontic treatment in dental clinics.

For use by qualified and trained personnel only.

4. Contraindications

This unit should not be used on a patient with a pacemaker.

5. Potential Complications/Side Effects

Extruder cartridges are for single-patient use. Re-use may lead to cross-contamination between patients.

The Backfill cartridges are designed to reach high temperatures. Inadvertent contact with patient and operator must be avoided to prevent accidental burns. Handle empty extruder cartridge by plastic locknut only. Other portions

Only for using on the territory of Russian Federation. Provided on demand

could be hot and could result in serious burns.

6. Technical Specification

Mode of operation:	Continuous Operation
Electrical class in acc. with IEC 60601-1:	IEC 601-1 Class II Type BF Device
Backfill Unit:	
Working temperature range	40°C ~ 230°C
Heat time of gutta percha	Within 4 - 5 seconds at 40 °C Within 49 - 53 seconds at 230 °C
Downpack Unit:	
Working temperature range	140°C ~ 400°C
Heat time of gutta percha	< 1 second at 140 °C < 2 seconds at 400 °C
Adapter:	Input: 100-240V ~, 50/60 HZ 0,5A; Output: 9V, 2A
Battery:	3.7 V, 1700 mAh, Li-ion

These devices come with the three preset temperature modes. The following table shows their function and default temperature setting:

Setting	Descriptions	Units	Default temp.	Tolerance, °C
GUTTA PERCHA	For use with Gutta Percha	Backfill	200 °C	± 15
		Downpack	200 °C	± 25
SYNTETIC	For use with synthetic filling materials	Backfill	150 °C	± 15
		Downpack	150 °C	± 25
CUSTOM	For use as an additional preset	Backfill	200 °C	± 15
		Downpack	200 °C	± 25

Note:

- When the temperature setting on the GUTTA PERCHA or SYNTETIC modes is changed from the default setting, an asterisk "*" will appear next to the temperature.
- On the Backfill device, only the CUSTOM temperature setting can be adjusted.

Powering off device:

The unit will automatically shut down after approximately 5 minutes. To manually turn off the device, hold down the Power button for approximately 2 seconds. The unit will generate an audio signal to indicate it is powering down. Upon any power off of the Backfill, the motor will automatically retract for a few seconds to relieve the pressure on the cartridge.

Backfill loudness level, dB				
	Volume level (min)	Volume level (default)	Volume level (max)	Tolerance, %
Power ON/OFF, Ready	38	68	84	± 5

Only for using on the territory of Russian Federation. Provided on demand

Rev.1-18

Reserve	73	± 5
Extrusion	55	± 5

Downpack loudness level, dB				
	Volume level (min)	Volume level (default)	Volume level (max)	Tolerance, %
Power ON/OFF, Activate	28	58	74	± 5

Force for installation and removal of cartridges, N			
	Installation	Removal	Tolerance, %
Backfill	1.1	0.64	± 15

Technical parameters of Gutta Percha Cartridges:

PN	Size	Nominal needle outside diameter		Nominal needle inside diameter		Nominal wall thickness	
	Gauge	mm	tolerance (mm)	mm	tolerance (mm)	mm	tolerance (mm)
972-1005	23	0.6414	±0.0064	0.337	±0.019	0.1524	±0.0064

Density	2,7 g/cm ³ ± 5%
Radio-opacity	7,03 mm/Al ± 5%
Shore hardness	55-60
Fusion temperature	64-91 °C
Mollifying temperature	50-80 °C
Disintegration temperature	300 °C ± 5%
Film thickness	≤ 50 micron
Solubility	< 3%
Curing time	Full cure of material in the canals occurs in 5 – 10 minutes after the termination of heated elements exposure.
Biocompatibility	Neutral to tooth tissue. Resistant to dental pulp fluid, no damaged and no resolved.
Usage conditions	from 32 °C to 42 °C (intrusive)

7. Terms of Acceptance and Types of Test

- SW Validation (version 1.0.3, class risk A in accordance with standard IEC 62304) – Backfill unit
- SW Validation (version 1.0.3, class risk A in accordance with standard IEC 62304) – Downpack unit

Only for using on the territory of Russian Federation. Provided on demand

Rev.1-18

- Compensated/Non-compensated Temperature Characteristics – Backfill unit
- Temperature Characteristics – Downpack unit
- Cytotoxicity – Heated Buchanan Pluggers
 - Considered Nontoxic
- Cytotoxicity – Gutta Percha Cartridge
 - Considered Nontoxic
- Cytotoxicity – Gutta Percha
 - Considered Nontoxic

Quality inspections

Products inspected through in-coming products inspection per internal quality system requirements and sampling frequencies.

Guidance and manufacturer's declaration - electromagnetic emissions

The Elements Free is intended for use in the electromagnetic environment specified below. The customer or the user of the Elements Free should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The Elements Free uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The Elements Free is suitable for use in all establishments other than domestic, and may be used in domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes, provided the following warning is heeded: Warning: This Elements Free is intended for use by healthcare professionals only. This equipment/system may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as re-orienting or relocating the Elements Free or shielding the location.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacturer's declaration - electromagnetic immunity

The Elements Free is intended for use in the electromagnetic environment specified below. The customer or the user of the Elements Free should assure that it is used in such an environment.			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV Contact ±8 kV air	±6 kV Contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.

Only for using on the territory of Russian Federation. Provided on demand

Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) to line(s) ±2 kV line(s) to earth	±1 kV line(s) to line(s) ±2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines	< 5 % U_T (> 95 % dip in U_T) for 0.5 cycle 40 % U_T (60 % dip in U_T) for 5 cycle 70 % U_T (30 % dip in U_T) for 25 cycle <5 % U_T (>95 % dip in U_T) for 5 s	< 5 % U_T (> 95 % dip in U_T) for 0.5 cycle 40 % U_T (60 % dip in U_T) for 5 cycle 70 % U_T (30 % dip in U_T) for 25 cycle <5 % U_T (>95 % dip in U_T) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Elements Free image intensifier requires continued operation during power main interruptions. It is recommended that the Elements Free image intensifier be powered from an uninterruptible power supply.
Power frequency (50 Hz and 60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable mobile RF communications equipment should be used no closer to any part of the Elements Free, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1,2\sqrt{P}$ $d = 1,2\sqrt{P}$ 80 MHz to 800 MHz $d = 2,3\sqrt{P}$ 800 MHz to 2.5 GHz Where P is the maximum output power rating of the
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	

Only for using on the territory of Russian Federation. Provided on demand

Rev.1-18

			transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range^b. Interference may occur in the vicinity of equipment marked with the following symbol: 
--	--	--	--

Note 1: U_T is the a.c. mains voltage prior to application of the test level.

Note 2: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 3: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Elements Free is used exceeds the applicable RF compliance level above, the Elements Free should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Elements Free.
- b. Over the frequency range 150 kHz to 80MHz, field strengths should be less than 3 V/m.

The Elements Free is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Elements Free can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Elements Free as recommended below, according to the maximum output power of the communications equipment.

Recommended separation distances between portable and mobile RF communications equipment and the Elements Free.

The Elements Free is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Elements Free can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Elements Free as recommended below, according to the maximum output power of the communications equipment.

Only for using on the territory of Russian Federation. Provided on demand

Rev.1-18

Rated maximum output power of transmitter [W]	Separation distance according to frequency of transmitter [m]		
	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

8. Disposal Conditions

elementsfree Obturation System is not a consumable and does not produce medical waste during its usage.

When you plan to dispose of the elementsfree Obturation System, please return it to SybronEndo for environmentally sound recycling. Please contact your sales representative for details. The Waste Electrical and Electronic Equipment (WEEE) Directive forbids the disposal of waste electrical and electronic equipment as unsorted municipal waste, requiring that it be collected and recycled or disposed of separately.

In Russian Federation, all medical devices should be disposed by local public organizations in accordance with SanPin 2.1.7.2790-10 "Sanitary-and-epidemiological requirements to medical wastes management" or other national and local regulations.

9. Applied Standards

MDD 93/42/EEC	EU Medical Device Directive
EN 980:2008	Graphical Symbols for Use in the Labelling of Medical Devices
EN 1041:2008	Information supplied by the manufacturer of medical devices
EN 13485: 2012	Medical Devices, Quality management systems
EN 14971: 2012	Medical devices. Application to risk management to medical devices
IEC 60601-1: 2005	Electrical Safety Testing, Medical Devices
IEC 60601-1-2:2005	Electromagnetic Compatibility Testing, Medical Devices
IEC 62304:2006	Medical device software – Software life cycle processes

Only for using on the territory of Russian Federation. Provided on demand

Rev.1-18

10. Contact Information

Manufactured for:

Ormco Corporation also trading as SybronEndo
1332, S. Lone Hill Avenue, Glendora, CA, 91740, USA
Tel: 1-800-854-17-41

Manufacturer:

META BIOMED CO., LTD,
270, Osongsaengmyeong 1-ro, Osong-eup, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, Korea.
Tel: 043-218-1981

Manufacturing site:

META SYSTEMS CO., LTD, also doing business as S-Denti Co., Ltd., 55, Galmachi-ro 281 beon-gil, Jungwon-gu, Seongnam-si, Gyeonggi-do, 13209, Korea.

Authorized representative in Russia:

KaVo Dental Russland LLC
64 Malookhtinsky avenue, Building "B", premises 26H, 195112 Saint Petersburg, Russia.
Tel: 7(812)324-42-12

For Technical Support in Russia, please contact:

KaVo Dental Russland LLC
64 Malookhtinsky avenue, Building "B", premises 26H, Saint Petersburg, Russia
Tel. 7(812)324-42-12
Direct one's claims to:
E-mail: kavo@kavodental.ru

Only for using on the territory of Russian Federation. Provided on demand

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Lee Dong-ho
attorney-in-fact of
META BIOMED CO., LTD.
PRESIDENT/ Suk Song Oh
appeared before me and
admitted said principal's
subscription to the attached
Supplement to the IFU for Medical Device,
Cordless dental elementsfree Obturation
System for root canal obturation

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