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Founding capital: HRK 118.472.000,00 (paid in full) / Issued shares no. : 1184720 / Share nominal value: HRK 100,00

President of the Management Board: Ivo Usmiani / Executive Director: Mislav Vučić

Personal identification number: 20950636972, VAT number: HR20950636972 / Tel+385 51 546 124 / Fax+385 51 546 124/e.mail: jgl@jgl.hr / [www.jgl.hr](http://www.jgl.hr)

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### Description:

The concentration of salts in isotonic solutions is close to the salt composition of fluids of the human body, due to which there is a sparing effect on the nasal mucosa, the water-ion balance is normalized, which, in turn, reduces swelling in the nasal cavity. Irrigation with isotonic solutions significantly reduces the number of microbial antigens, thereby reducing the microbial load on the nasal mucosa.

A jet of the solution under pressure thoroughly washes all parts of the nasal cavity and nasopharynx, actively cleansing them of excess mucus, viruses, bacteria, foreign particles, including allergens, dense exudate (crusts) and without disturbing the biocenosis of the mucous membrane.

The use of saline solutions for nasopharyngeal lavage is recommended as a non-specific therapy for acute viral respiratory infections.

Isotonic solution of sea water, when applied to the mucous membrane, ensures its normal physiological state, helps to liquefy mucus and normalize its production in the goblet cells of the mucous membrane.

- The microelements that are part of the solution improve the function of the ciliated epithelium, have an anti-inflammatory, cleansing, stimulating, restorative effect on the mucous membrane of the nasal cavity.
- Reduces swelling of the nasal mucosa and nasopharynx;
- Reduces the risk of local complications and accelerates healing processes after operations in the nasal cavity and paranasal sinuses (removal of adenoids, polyps, septoplasty, etc.);
- Reduces the duration of rhinitis symptoms;
- Increases the therapeutic efficacy of drugs applied to the nasal mucosa (vasoconstrictors, hormones, inhaled corticosteroids, antiallergic drugs, etc.);
- Reduces the need for the use of topical nasal medications.
- Thins nasal mucus and normalizes its production.

When the nozzle is pressed, the solution is distributed in the form of an aerosol.

A special "anatomical" nozzle with a rounded applicator makes it easy to use in infants, young children and adults. A special corrugated surface with a limiter makes it possible to carry out uniform intensive spraying of the "shower" type, and a tight connection with the nozzle prevents the solution from leaking out. The spray system is specially designed to provide the most effective rinsing while the procedure is comfortable for the patient.

Modern packaging technology avoids contact between propellant and seawater solution. In addition, the cylinder design provides reliable protection of the product from a microbiological point of view;

AGENCIJA ZA ENKIGI LABORATORIJ d.d. (JGL d.d.), Šviben 20, 51000 Rieka, Croatia; E-mail: [info@jgl.hr](mailto:info@jgl.hr); Tel: +385 (0)51 40004561; Fax: +385 (0)51 40004562

OTP BANKA d.d., Dalmatinski trg 61, 21000 Split, Croatia; IBAN: HR4024070011000160179; E-mail: [info@otp.hr](mailto:info@otp.hr); Fax: +385 (0)21 3300000  
ErsteBank d.d., Jadranski trg 1a, 51000 Rieka, Croatia; IBAN: HR4024070011000160179; E-mail: [info@erste.hr](mailto:info@erste.hr); Fax: +385 (0)51 40004562  
PRVI DVA BANKA ZAGREB d.d., Radnička cesta 50, 10000 Zagreb, Croatia; IBAN: HR4024070011000160179; E-mail: [info@prvi.hr](mailto:info@prvi.hr); Fax: +385 (0)1 48100000  
ZAGREBAČKA BANKA d.d., Trg bana Josipa Jelačića 10, 10000 Zagreb, Croatia; IBAN: HR4024070011000160179; E-mail: [info@zabanka.hr](mailto:info@zabanka.hr); Fax: +385 (0)1 48100000

Founding capital: 10.415.325,00 EUR (paid in full) / Number of issued shares: 1.255.025; Shareholding price: 1,00 / 1,00  
President of the Management Board: Ivo Usmiani (Executive Director: Miroslav Vukobratović)  
Personal ident. number: 20950636972 / VAT no.: HR20950636972 / Tel: +385 (0)51 40004561 / Fax: +385 (0)51 40004562; E-mail: [info@jgl.hr](mailto:info@jgl.hr); [www.jgl.hr](http://www.jgl.hr)

2. Insert the balloon tip into the right nostril.

3. Press the spray bottle and hold it for a few seconds, thus rinsing the right half of the nose.

4. Remove any residual fluid from the nasal cavity (help the child blow his nose or use an aspirator).

**Product for nose rinsing and irrigation** “Aqua Maris® Norm. Intensive rinsing” is used in children from 6 months of age and adults.

1. Nasal irrigation in children and adults is carried out in an upright position (sitting or standing). In this case, the head should be located straight and slightly tilted forward. Make sure that the head is not thrown back during rinsing. Insert the tip of the balloon into the right half of the nose.

2. Press the spray bottle and hold it for a few seconds, thus rinsing the right side of the nose.

**BIT BANKA** d.d., Dalmatinskega rata 61, 21000 Split, Croatia; IBAN HR94487000110000000000000000  
Erste Steiermärkische Bank d.d., Jadranski trg 3a, 51000 Rijeka, Croatia; IBAN HR30440000110000000000000000  
**KUPURENICA BANCA ZAGREB**, Radnička cesta 80, 10000 Zagreb, Croatia; IBAN HR30340000110000000000000000  
**ZABR BACKA BANKA** t.d.b., Trg bana Josipa Jelencića 10, 10000 Zagreb, Croatia; IBAN HR30340000110000000000000000

- 

The product is compatible with all products used to treat a runny nose.

No contraindications have been identified.

It does not affect the speed of reaction when driving vehicles and working with other mechanisms.

It can be used during pregnancy and lactation.

When used in accordance with the instructions, no side effects were detected.

3 years. Do not use after the expiration date.

50 ml, 125 ml or 150 ml of solution in a metal cylinder under pressure. One cylinder with a nozzle and a plastic cap is placed in a cardboard box together with an instruction for use.

**Intended purposes:**

The medical device is intended to be applied to the nasal mucosa for effective rinsing and removal of discharge.

**Description:**

The concentration of salts in isotonic solutions is close to the salt composition of fluids of the human body, due to which there is a sparing effect on the nasal mucosa, the water-ion balance is normalized, which, in turn, reduces swelling in the nasal cavity. Irrigation with isotonic solutions significantly reduces the number of microbial antigens, thereby reducing the microbial load on the nasal mucosa.

A jet of the solution under pressure thoroughly washes all parts of the nasal cavity and nasopharynx, actively cleansing them of excess mucus, viruses, bacteria, foreign particles, including allergens, dense exudate (crusts) and without disturbing the biocenosis of the mucous membrane.

The use of saline solutions for nasopharyngeal lavage is recommended as a non-specific therapy for acute viral respiratory infections.

Isotonic solution of sea water, when applied to the mucous membrane, ensures its normal physiological state, helps to liquefy mucus and normalize its production in the goblet cells of the mucous membrane.

- The microelements that are part of the solution improve the function of the ciliated epithelium, have an anti-inflammatory, cleansing, stimulating, restorative effect on the mucous membrane of the nasal cavity.
- Reduces swelling of the nasal mucosa and nasopharynx;
- Reduces the risk of local complications and accelerates healing processes after operations in the nasal cavity and paranasal sinuses (removal of adenoids, polyps, septoplasty, etc.);
- Reduces the duration of rhinitis symptoms;
- Increases the therapeutic efficacy of drugs applied to the nasal mucosa (vasoconstrictors, hormones, inhaled corticosteroids, antiallergic drugs, etc.);
- Reduces the need for the use of topical nasal medications.
- Thins nasal mucus and normalizes its production.

When the nozzle is pressed, the solution is distributed in the form of an aerosol.

A special "anatomical" nozzle with a rounded applicator makes it easy to use in infants, young children and adults. A special corrugated surface with a limiter makes it possible to carry out

uniform intensive spraying of the "shower" type, and a tight connection with the nozzle prevents the solution from leaking out. The spray system is specially designed to provide the most effective rinsing while the procedure is comfortable for the patient.

Modern packaging technology avoids contact between propellant and seawater solution. In addition, the balloon design provides reliable protection of the product from a microbiological point of view; makes it possible to use up all the amount of solution available in the bottle; It provides convenient use, as the product can be used in all positions.

The pressure in the balloon of the Aqua Maris® Norm. Intensive rinsing" reaches a value of  $8.0 \cdot 10^5$  Pa, which contributes to effective rinsing of the nasal passages along the entire length. The pressure in the balloon "Aqua Maris® Baby. Intensive rinsing" does not exceed  $7.2 \times 10^5$  Pa, providing a delicate rinsing of the baby's nose with a "soft shower".

Aqua Maris® Intensive Rinsing is used in medical institutions and at home.

The medical device has no pharmacological, immunological, genetic or metabolic effects on the human body.

## Indications:

- Complex treatment of acute and chronic inflammatory diseases of the nasal cavity, paranasal sinuses and nasopharynx (infectious, allergic):
  - acute and chronic rhinitis;
  - acute and chronic sinusitis;
  - acute and chronic adenoiditis;
  - allergic and vasomotor rhinitis;
- Complex treatment of viral respiratory infections.
- Daily use during viral respiratory infections and influenza epidemics.
- Nasal cavity care:
  - after surgical interventions in the nasal cavity;
  - removal of microorganisms, foreign particles, allergens, dust, pollen;
  - preparation of the nasal mucosa for the use of medicines;
  - long-term therapy with topical corticosteroids.

## How to use:

Intranasal.

1-2 injections into each nasal passage several times a day (2-6 times a day) or more if necessary. The period of application is not limited.

JADRAN (JALENSKI) LABORATORIJE d.d. (JGL d.d.) Šumen 20, 51000 Rijeka, Croatia; E-mail: [info@jgl.hr](mailto:info@jgl.hr); Tel.: +385 (0)51 311 05 40; Fax: +385 (0)51 311 05 41; Company ID: 346004561

OTR BANKA d.d., Dalmatinska cesta 61, 21000 Šibenik, Croatia; IBAN: HR9424070001101010101; SWIFT: OTBHR33K  
ErsteSteiermärkische Bank d.d., Jadranski trg 3a, 51000 Rijeka, Croatia; IBAN: HR9424070001101010101; SWIFT: ERSTHR33  
PRIVREDAJNA BANKA ZAGREB d.d., Radnička cesta 50, 10000 Zagreb, Croatia; IBAN: HR93140000110101010101; SWIFT: PRVZHR33  
ZAGREBAČKA BANKA d.d., Trg bana Josipa Jelačića 10, 10000 Zagreb, Croatia; IBAN: HR93140000110101010101; SWIFT: ZABZHR33

Founding capital: 10 315 325.00 EUR (paid in full) / Number of issued shares: 1 255 025 / Share nominal value: 100.00 HRK  
President of the Management Board: Ivo Ušumani / Executive Director: Miroslav Vučković  
Personal identification no.: 20950636972 / VAT no.: HR20950636972 / Tel.: +385 (0)51 311 05 40; Fax: +385 (0)51 311 05 41; E-mail: [info@jgl.hr](mailto:info@jgl.hr); [www.jgl.hr](http://www.jgl.hr)

## Instructions for use:

**Product for nose rinsing and irrigation "Aqua Maris® Baby. Intensive rinsing"** is used in infants from 3 months of age.

1. Nasal rinsing in a young child is carried out in an upright position, optimally - in the arms of an adult (if the child already knows how to sit). At the same time, the child's head should be located straight and slightly tilted forward. Make sure that the child does not throw his head back during rinsing.

2. Insert the balloon tip into the right nostril.

3. Press the spray bottle and hold it for a few seconds, thus rinsing the right half of the nose.

4. Remove any residual fluid from the nasal cavity (help the child blow his nose or use an aspirator).

**Product for nose rinsing and irrigation "Aqua Maris® Norm. Intensive rinsing"** is used in children from 6 months of age and adults.

JADRAN - GALENSKI LABORATORIJI d.d. (JGL d.d.) Šušunjska 20, 51000 Rijeka, Croatia, t: 051 386 1111, f: 051 386 1112, e: info@jgl.hr, www.jgl.hr, OIB: 640004561

OTP BANKA d.d. Opatovinskih rta 61, 21000 Split, Croatia, t: 021 386 1111, f: 021 386 1112, e: info@otp.hr, www.otp.hr  
ErsteSteiermarkische Bank d.d. Jartrantski trg 1a, 51000 Rijeka, Croatia, t: 051 386 1111, f: 051 386 1112, e: info@esb.hr, www.esb.hr  
PRIVREDNA BANKA ZAGREB d.d. Radnička cesta 50, 10000 Zagreb, Croatia, t: 01 386 1111, f: 01 386 1112, e: info@privredna.hr, www.privredna.hr  
ZAGREBAČKA BANKA d.d. Trg bana Josipa Jelačića 10, 10000 Zagreb, Croatia, t: 01 386 1111, f: 01 386 1112, e: info@zabanka.hr, www.zabanka.hr

Registered Capital: 25.315.325,00 EUR (paid in full) / Number of issued shares: 1.255.125 / Share capital: 100% / 100%  
President of the Management Board: Ivo Usmiani / Executive Director: Miroslav Vukobratović  
Personal identification no.: 20950636972 / VAT no.: HR20950636972 / Tel.: +385 (0) 51 386 1111, fax: +385 (0) 51 386 1112, e: info@jgl.hr, www.jgl.hr

1. Nasal irrigation in children and adults is carried out in an upright position (sitting or standing). In this case, the head should be located straight and slightly tilted forward. Make sure that the head is not thrown back during rinsing. Insert the tip of the balloon into the right half of the nose.

2. Press the spray bottle and hold it for a few seconds, thus rinsing the right side of the nose.

3. Remove any residual flushing fluid by blowing your nose or using a nasal aspirator (if necessary, in children).

Perform a similar procedure with the left half of the nose.

Repeat the procedure if necessary.

The medical device is compatible with all products used to treat a runny nose.

### Contraindications:

No contraindications have been identified.

**Ability to affect the speed of reaction when driving vehicles or working with other mechanisms:**

It does not affect the speed of reaction when driving vehicles and working with other mechanisms.

**Use during pregnancy and lactation:**

It can be used during pregnancy and lactation.

**Side effects:**

When used in accordance with the instructions, no side effects were detected.

### Shelf life:

3 years. Do not use after the expiration date.

**Dosage form and packaging:**

50 ml, 125 ml or 150 ml of solution in a metal balloon under pressure. One balloon with a nozzle and a plastic cap is placed in a cardboard box together with an instruction for use.

### Precautions:

Pressurized vial: May explode when exposed to heat. Do not pierce or burn even after use. Keep away from heat, hot surfaces, sparks, open flames, or any other source of ignition. No smoking. Protect from sunlight. Do not expose to temperatures above 50 °C.

**Long-term storage and transportation conditions:**

The medical device is transported by all modes of transport in covered vehicles in accordance with the rules for the transportation of goods applicable to each type of transport. During transportation, the medical device is resistant to air and climatic factors, as well as to mechanical stress.

Long-term storage and transportation are carried out at a humidity of 45-80%, a pressure of 84-106.7 kPa and a temperature of +1 to +50°C.

**Storage (operation) conditions:**

Store in a dry place, out of the reach of children, at a temperature of  $+25 \pm 10^\circ\text{C}$ , humidity of 45-80%, pressure of 84-106.7 kPa.

### Utilization:

Product for nose rinsing and irrigation "Aqua Maris® Intensive rinsing" does not have a negative impact on the environment, so no special measures are required to dispose of the solution.

**Waste hazard class:**

Waste class A according to SanPiN 2.1.3684-21.

- intended purpose and indications;
- how to use;
- contraindications;
- the volume of the medical device;
- storage conditions;
- batch number;
- date of production;
- shelf life "Годен до ...";
- information about the registration certificate number.

Consumer packaging does not contain symbols, all information is written in words.

The markings on the transport box contain the following information:

- name of the medical device;
- the number of consumer packages in the transport box;
- date of production;
- shelf life;
- country of origin;
- barcodes;
- Manipulation signs:

|                                                                                     |                         |
|-------------------------------------------------------------------------------------|-------------------------|
|  | Temperature restriction |
|  | Top                     |
|  | Fragile. Watch out      |

## List of standards that the medical device complies with:

The medical device complies with all applicable requirements of Directive 93/42/EEC.

Harmonized standards, national standards and other regulatory documents applied:

|                     |                                                                                                                                   |
|---------------------|-----------------------------------------------------------------------------------------------------------------------------------|
| ISO 9001:2015       | Quality Management Systems                                                                                                        |
| EN ISO 13485:2016   | Medical devices – Quality Management Systems – Regulatory Requirements                                                            |
| EN ISO 15223-1:2016 | Medical devices. Symbols used for marking on medical devices, labels and accompanying documentation. Part 1. General requirements |

JADRAN JAGRE NISKI LABORATORIJE d.d. (JGL d.d.) Šumić 20, 51000 Rijeka, Croatia (E-mail: [info@jgl.hr](mailto:info@jgl.hr), [med@jgl.hr](mailto:med@jgl.hr)) Tel: +386 (0)51 2445045

GRF BANKA d.d. Opatovskingska 61, 21000 Split, Croatia (BAN: HRBA4407000177) [www.grf.hr](http://www.grf.hr)  
 Erste Steiermärkische Bank d.d. Jadranski trg 3a, 51000 Rijeka, Croatia (BAN: HRBA4407000177) [www.erste.hr](http://www.erste.hr)  
 PRIVREDNA BANKA ZAGREB d.d. Radnička cesta 50, 10000 Zagreb, Croatia (BAN: HRBA4407000177) [www.pzb.hr](http://www.pzb.hr)  
 ZAGREBAČKA BANKA d.d. Trg bana Josipa Jelačića 10, 10000 Zagreb, Croatia (BAN: HRBA4407000177) [www.zabanka.hr](http://www.zabanka.hr)

Founding capital: 10.315.325,00 EUR (paid in full) / Number of issued shares: 1.255.025 / Share nominal value: 8,33 EUR

President of the Management Board: Ivo Ušumani / Executive Director: Miroslav Vuk

Personal identification no.: 20950636972 / VAT no.: HR20950636972 (EU) • HRB 51144 • OIB 61144 • HRB 51144 • HRB 51144 • HRB 51144 • [www.jgl.hr](http://www.jgl.hr)

|                      |                                                                               |
|----------------------|-------------------------------------------------------------------------------|
| EN 1041:2008+A1:2013 | Information provided by the manufacturer of medical devices                   |
| EN ISO 14971:2012    | Medical devices - Application of a risk management system to medical devices. |
| EN ISO 14644:2015    | Clean rooms and associated controlled environments.                           |

## Information about the materials used in the medical device:

Tables 1-7 provide information on the materials from which the bag with the valve system, nozzles, and cap are made. The data applies to all variants of the medical device. Table 8 shows data on the materials of the vial contents.

*Table 1. Information about the materials of the bag with a valve system (manufactured by Aptar Radolfzell GmbH, Germany).*

| Component of the vial          | Material                                             | Manufacturer                                                     |
|--------------------------------|------------------------------------------------------|------------------------------------------------------------------|
| Sack                           | Foil (PET/ALU/OPA /CPP)<br>POLIALUVEL® AGGRESS 51489 | WIPF AG                                                          |
| Corps                          | Polypropylene HJ325MO                                | BOREALIS AG                                                      |
| Spring                         | Stahl 1.4310/ AISI 302                               | - RESSORT HUON<br>DUBOIS S.A.<br>- SEKONA                        |
| Valve                          | Polyoxymethylene KEPITAL F25-03                      | KOREA ENGINEERING<br>PLASTICS CO., LTD.                          |
|                                | Dye white 50 SWS13-POM (SWS13-POM Weißbatch 50)      | GRAPHS                                                           |
| Coating valve*                 | Aluminium EN AW-5754/VP 16675                        | - THOMAS,<br>- EUSCHER                                           |
| Gasket*                        | Rubber 8BU1941                                       | THE JOINT FRANCAIS<br>S.N.C. (HUTCHINSON®<br>group of companies) |
|                                | Elastomer J 175-01                                   | GLOBUS                                                           |
| Gasket under the coating valve | Nitrile Butadiene Rubber NITRILE 450                 | APTAR FRANCE SAS                                                 |
| Tape*                          | Paper (does not have a stamp)                        | CCL LABEL AG                                                     |

\*There is no contact with the solution before/after use.

(AJURAN - GALT Nisk LABORATORIJ) d.d. (jgl d.d.), Šuljin 20, 51000 Rijeka, Croatia, E-mail: info@ajuran.hr, Web: www.ajuran.hr, OIB: 640004561

OTP BANKA d.d. (Dalmatinski trg 61, 21000 Split, Croatia), IBAN: HR844017001100016019, SWIFT: OTPLHR33  
Erste Steiermärkische Bank d.d. (Jadranski trg 3a, 51000 Rijeka, Croatia), IBAN: HR80420001100016019, SWIFT: ERSTHR33  
PRIVREDNA BANKA ZAGREB d.d. (Radnička cesta 50, 10000 Zagreb, Croatia), IBAN: HR30300001601900016019, SWIFT: ZAGREB33  
ZAGREBAČKA BANKA d.d. (Trg bana Josipa Jelačića 10, 10000 Zagreb, Croatia), IBAN: HR30300001601900016019, SWIFT: ZAGREB33

Founding capital: 16.315.325,00 EUR (paid in full) / Number of issued shares: 1.275.000 / Share price: 12,80 EUR / Shareholder:  
President of the Management Board: Ivo Ujman / Executive Director: Marko Vukobratović  
Personal identification no.: 20950636972 / VAT no.: HR20950636972 / Tax identification no.: 63163436972 / Web: [www.jgl.hr](http://www.jgl.hr)

*Table 2. Information about the materials of the bag with a valve system (manufactured by Coster Tecnologie Speciali S.p.A., Italy).*

| <i><b>Component of the vial</b></i> | <i><b>Material</b></i>                             | <i><b>Manufacturer</b></i>     |
|-------------------------------------|----------------------------------------------------|--------------------------------|
| Sack                                | Foil (PET 12/AL 8,5/PA 15/LLD PE 76) VELANI 406862 | GOGLIO                         |
|                                     | Foil (PET 12/AL 9/PA 15/LLD PE 80) 420010/0        | AMCOR FLEXIBLE                 |
|                                     | Foil (PET 12/AL 9/PA 15/LLD PE 80) YV 8108001      | SAFTA                          |
| Corps                               | High Density Polyethylene PURELL ACP 6541A         | LYONOELLBASELL                 |
| Spring                              | Steel AISI 316                                     | ADDA                           |
| Valve                               | Polybutyleneterephthalate VALOX 325F               | SABIC                          |
| Coating valve*                      | Aluminium EN AW-5754                               | -THOMAS GMBH<br>- EUSHER       |
| Gasket*                             | Chlorobutyl 8 BU 1941<br>Chlorobutyl J 175/ 01     | - THE FRENCH JOINT<br>- GLOBUS |
| Gasket under the coated valve       | Nitrile Butadiene Rubber KA 7001                   | ACTION TECHNOLOGY              |

*\*There is no contact with the solution before/after use.*

Table 3. Information about the materials of the nozzle with a limiting ring and the nozzle without a limiting ring (manufactured by Aptar Radolfzell GmbH, Germany).

| <i>Component of the vial</i>       | <i>Material</i>                   | <i>Manufacturer</i> |
|------------------------------------|-----------------------------------|---------------------|
| Inner and outer part of the nozzle | High-density polyethylene MG6941  | BOREALIS            |
|                                    | White dye 3.5% FK23260            | BRUNO PETER AG      |
| Insert                             | High-density polyethylene MG7547S | BOREALIS            |

*Table 4. Information about the materials of the nozzle with a limiting ring (manufactured by Coster Tecnologie Speciali S.p.A., Italy).*

| <i>Component of the vial</i> | <i>Material</i>                                                        | <i>Manufacturer</i>            |
|------------------------------|------------------------------------------------------------------------|--------------------------------|
| Inside of the nozzle         | Polypropylene Random Copolymer<br>( - Moplen RP 340 N;<br>- RE 420 MO) | - LYONDELLBASELL<br>- BOREALIS |
| Outer part of the nozzle     |                                                                        |                                |

OTF BANKA d.d. Orlowskijska cesta 61, 21000 Split, Croatia, IBAN: HR64240700011191416119, SWIFT: OTFCRO22  
ErsteSteiermärkische Bank d.d. Jadranski trg 3a, 51000 Rijeka, Croatia, IBAN: HR64240700011191416119, SWIFT: ERSTHR33  
PRIVREDNA BANKA ZAGREB d.d. Radnička cesta 80, 10000 Zagreb, Croatia, IBAN: HR64240700011191416119, SWIFT: ZGHR33  
ZAGREBAČKA BANKA d.d. Trg bana Josipa Jelačića 10, 10000 Zagreb, Croatia, IBAN: HR64240700011191416119, SWIFT: ZGHR33

President of the Management Board: Ion Usmiani / Executive Director: Mihail Gheorghe

|        |                                         |        |
|--------|-----------------------------------------|--------|
| Insert | Polyoxymethylene Hostaform<br>C27021XAP | TICONA |
|--------|-----------------------------------------|--------|

**Table 5. Information about the materials of the nozzle without a limiting ring (manufactured by Coster Tecnologie Speciali S.p.A., Italy).**

| <b>Component of the vial</b> | <b>Material</b>                                                        | <b>Manufacturer</b>                           |
|------------------------------|------------------------------------------------------------------------|-----------------------------------------------|
| Inside of the nozzle         | Polypropylene Random Copolymer<br>( - Moplen RP 340 N;<br>- RE 420 MO) | - LYONDELLBASELL<br>- BOREALIS                |
| Insert                       | Polyoxymethylene Hostaform<br>C27021XAP                                | TICONA                                        |
| Outer part of the nozzle     | Polypropylene Random Copolymer<br>( - Moplen RP 2380;<br>- PPR 10232)  | - LYONDELLBASELL<br>- TOTAL<br>PETROCHEMICALS |

**Table 6. Information about the materials of the universal nozzle (manufactured by Aptar Radolfzell GmbH, Germany).**

| <b>Component of the vial</b>       | <b>Material</b>                              | <b>Manufacturer</b>     |
|------------------------------------|----------------------------------------------|-------------------------|
| Outer and inner part of the nozzle | High-density polyethylene (BORMED HE7541-PH) | BOREALIS                |
|                                    | White dye 3.0% (FIBAPLAST WEISS PE 11307407) | KARL FINKE GMBH & CO KG |
| Insert                             | High-density polyethylene (BORMED HE7541-PH) | BOREALIS                |
| Cap                                | Polypropylene Pural HP373P                   | LyondellBasell          |

Nozzles with and without limiting rings have caps manufactured by *Coster Tecnologie Speciali S.p.A., Italy* or *TEHNOPLAST, Croatia*. Information about the materials from which the caps are made is indicated in Table 7.

**Table 7. Cap material data (excluding cap data for universal nozzle).**

| <b>Manufacturing company</b>             | <b>Material</b>         | <b>Manufacturer</b> |
|------------------------------------------|-------------------------|---------------------|
| Coster Tecnologie Speciali S.p.A., Italy | Polypropylene RJ-370-MO | BOREALIS            |
| TEHNOPLAST, Croatia                      | Polypropylene RJ-470-MO | BOREALIS            |

JADRAN – GALENSKI LABORATORIJ d.d. (JGL d.d.), Šibenik 20, 51000 Rijeka, Croatia / Company identification: Rijeka, Croatia, OIB: 640004561

OTK. BANKA d.d. Dominskog rata 61, 21000 Split, Croatia / IBAN: HR1441030111000160190001  
Erste Steiermärkische Bank d.d. Jadranski trg 3a, 51000 Rijeka, Croatia / IBAN: HR1441030111000160190001  
PRIVREDNA BANKA ZAGREB d.d., Radnička cesta 50, 10000 Zagreb, Croatia / IBAN: HR20240300012001200120012001  
ZAGREBAČKA BANKA d.d., Trg bana Josipa Jelačića 10, 10000 Zagreb, Croatia / IBAN: HR20240300012001200120012001

Founding capital: 16.415.325,00 EUR (paid in full) / Number of issued shares: 1.245.000 / Share market value: 13,21 EUR  
President of the Management Board: Ivo Usimani / Executive Director: Miroslav Vilić  
Personal identification no.: 20950636972 / VAT no.: HR20950636972 / Tel.: +385 (0)51 351 104 / E-mail: info@jgl.hr / [www.jgl.hr](http://www.jgl.hr)

**Bottle** – Aluminium EN AW1050A, manufactured by Linhardt GmbH & Co.KG; Aluminium EN AW 1070A, manufactured by Nussbaum Frankenberg GmbH.

**Secondary packaging** is PAP 21 cardboard.

*Table 8. Data on the materials of the vial contents*

| <i><b>Name of the material</b></i> | <i><b>Composition per 100 ml</b></i>                                               | <i><b>Manufacturer</b></i> |
|------------------------------------|------------------------------------------------------------------------------------|----------------------------|
| Sea water solution                 | Sea water 27.1 ± 3.0 ml (Certificate of Analysis No S-0254/2022)                   | JGL d.d., Croatia          |
|                                    | Purified water up to 100 ml (Certificate of Analysis No 678/2022) complies with EF | JGL d.d., Croatia          |

For the production of the Product for nose rinsing and irrigation “Aqua Maris® Intensive rinsing”, a natural component is used, namely sea water. Sea water is extracted in the northern part of the Adriatic Sea (coordinates - latitude 44° 58' 12" and longitude 14° 38' 0"). The permissible deviation within a radius of 250 meters from the specified location. Sea water is taken in the middle layers of the sea, at a depth of 20-30 meters, throughout the year using a system that includes mechanical filters (pore size approx. 100 µm), tubes, a pump and a tank.

### Characteristics of the solution contained in the medical device

(applicable for all variants)

Table 9. Characteristics of the solution.

| <i>Nº</i> | <i>Parameter</i>       | <i>Meaning</i>                                                                                                                                                                                                                                                                                                                                                                           |
|-----------|------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 1.        | Appearance             | Clear colorless liquid.                                                                                                                                                                                                                                                                                                                                                                  |
| 2.        | pH                     | 6,0-9,0                                                                                                                                                                                                                                                                                                                                                                                  |
| 3.        | Osmolality             | 0.270-0.310 osmol/kg                                                                                                                                                                                                                                                                                                                                                                     |
| 4.        | MICROBIOLOGICAL PURITY | <ul style="list-style-type: none"> <li>- Total aerobic microorganisms (TAMC): no more than <math>10^2</math> CFU/ml</li> <li>- Total number of yeasts and molds (TYMC): no more than <math>10^1</math> CFU/ml</li> <li>- Staphylococcus aureus: absent in 1 ml</li> <li>- Pseudomonas aeruginosa: absent in 1 ml</li> <li>- Bile-resistant enterobacteriaceae: absent in 1 ml</li> </ul> |
| 5         | Density of solution    | 1.005 g/ml                                                                                                                                                                                                                                                                                                                                                                               |

## Specifications of balloon components

Table 10. Technical characteristics of the balloon Linhardt GmbH & Co. KG (Germany) without nozzle

| Parameters of the balloon manufactured by Linhardt GmbH & Co. KG (Germany) without a nozzle (Fig. 1)           | Meaning                                 |                  |                  |
|----------------------------------------------------------------------------------------------------------------|-----------------------------------------|------------------|------------------|
|                                                                                                                | 50 ml                                   | 125 ml           | 150 ml           |
| Overall Height                                                                                                 | 104.75 – 105.25 mm                      | 149.7 – 150.3 mm | 141.7 – 142.3 mm |
| Vial bottom diameter                                                                                           | 34.8 – 35.2 mm                          | 44.7 – 45.3 mm   | 52.7 – 53.3 mm   |
| Outer diameter of the throat                                                                                   | 31.1 – 31.5 mm                          | 31.1 – 31.5 mm   | 31.1 – 31.5 mm   |
| Wall thickness                                                                                                 | 0.31 – 0.35 mm                          | 0.30 – 0.34 mm   | 0.36 – 0.40 mm   |
| Internal pressure resistance:<br>- internal pressure that does not cause permanent deformation of the balloon; | Not less than 1.2 MPa                   |                  |                  |
| - Destructive internal pressure                                                                                | At least 1.44 MPa                       |                  |                  |
| Surface quality                                                                                                | Smooth, free of contaminants and cracks |                  |                  |

Table 11. Technical characteristics of the balloon Nussbaum Frankenberg GmbH (Germany) without nozzle

| Parameters of the balloon manufactured by Nussbaum Frankenberg GmbH (Germany) without a nozzle (Fig. 2) | Meaning               |                  |                  |
|---------------------------------------------------------------------------------------------------------|-----------------------|------------------|------------------|
|                                                                                                         | 50 ml                 | 125 ml           | 150 ml           |
| Overall Height                                                                                          | 104.6 – 105.4 mm      | 149.6 – 150.4 mm | 141.6 – 142.8 mm |
| Vial bottom diameter                                                                                    | 34.7 – 35.3 mm        | 44.7 – 45.3 mm   | 52.7 – 53.3 mm   |
| Outer diameter of the throat                                                                            | 31.1 – 31.5 mm        | 31.1 – 31.5 mm   | 31.1 – 31.5 mm   |
| Wall thickness <sup>2</sup>                                                                             | 0.38 – 0.41 mm        | 0.34 – 0.48 mm   | 0.38 – 0.50 mm   |
| Internal pressure resistance:                                                                           | Not less than 1.8 MPa |                  |                  |

<sup>2</sup> \*- Wall thickness of pure aluminum bottle

JADRAN-GALENSKI LABORATORIJI d.d. (JGL d.d.), Švinske 20, 51000 Rijeka, Croatia / Čimbeničeva 100/1, Rijeka, Croatia, HR-51000, OIB: 640004561

OTP BANKA d.d., Domaljevskegata 61, 21000 Split, Croatia / IBAN: HR8424070001100016219, SWIFT: OTFHR23X  
ErsteSteiermärkische Bank d.d., Jadranski trg 3a, 51000 Rijeka, Croatia / IBAN: HR3424070001100016219, SWIFT: ERSTHR33  
PRIJEDNA BANKA ZAGREB d.d., Radnička cesta 50, 10000 Zagreb, Croatia / IBAN: HR2623400391101072000, SWIFT: PRIBHR33  
ZAGREBAČKA BANKA d.d., Trg bana Josipa Jelačića 10, 10000 Zagreb, Croatia / IBAN: HR3023600001101072000, SWIFT: ZABZHR33

Founding capital: 16 315 325.00 EUR (paid in full) / Number of issued shares: 1,255,025 / share nominal value: 13.00 HRK  
President of the Management Board: Ivo Usmani / Executive Director: Miroslav Vrdoljak  
Personal identification no.: 20950636972 / VAT no.: HR20950636972 / Tel.: +385 91 680 700 / Fax: +385 91 680 123 / E-mail: info@jgl.hr, www.jgl.hr

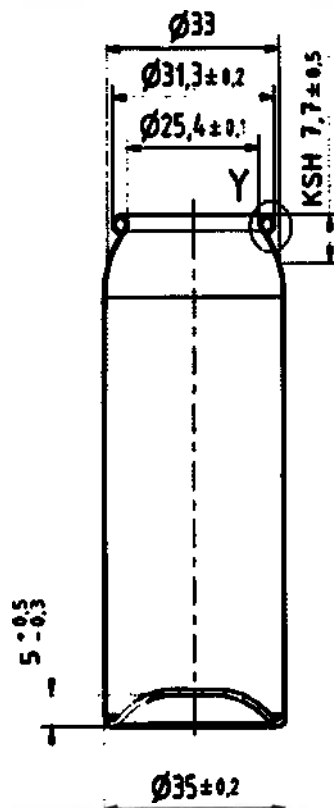
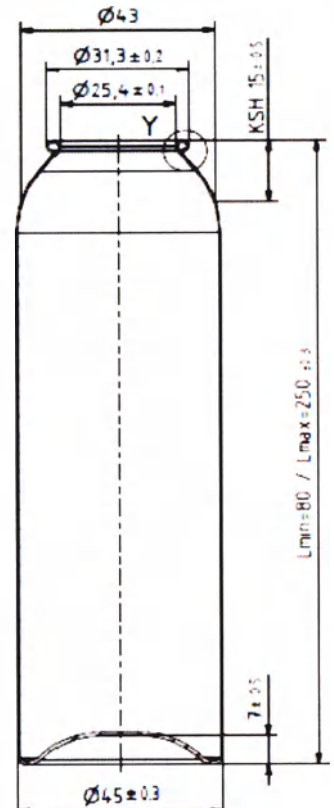
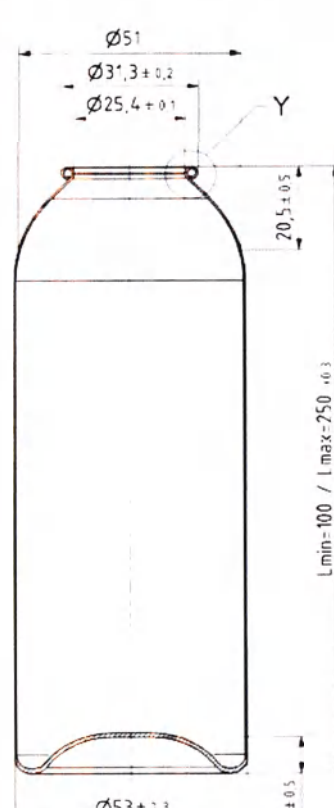
|                                                                                                                                                                       |                                                                                     |                                                                                      |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------|
| <ul style="list-style-type: none"><li>- internal pressure that does not cause permanent deformation of the balloon;</li><li>- Destructive internal pressure</li></ul> | At least 2.16 MPa                                                                   |                                                                                      |
| Surface quality                                                                                                                                                       | Smooth, free of contaminants and cracks                                             |                                                                                      |
|                                                                                     |  |  |
| Balloon, 50 ml                                                                                                                                                        | Balloon, 125 ml                                                                     | Balloon, 150 ml                                                                      |

Fig. 1 Technical drawing of the balloon manufactured by Linhardt GmbH & Co. KG (Germany) without a nozzle.

Fig. 1 Technical drawing of the balloon manufactured by Linhardt GmbH & Co. KG (Germany) without a nozzle.

JADRAN-GALENSKI LABORATORIJE d.d. (JGL d.d.) Šuljin 20, 51000 Rijeka, Croatia / Contact: E-mail: [info@jgl.hr](mailto:info@jgl.hr) / Tel: +385 (0)51 340004561

OTF BANKA d.d. Dalmatinskog rata 61, 21000 Split, Croatia / IBAN: HR944070001100016019 / Nalog: 001 / HR3X  
 ErsteSteiermarkische Bank d.d. Jadranski trg 3a, 51000 Rijeka, Croatia / IBAN: HR3424120001100028110 / Nalog: 001 / HR3X  
 PRIVREDNA BANKA ZAGREB d.d. Radnička cesta 50, 10000 Zagreb, Croatia / IBAN: HR9023400091100157300 / Nalog: 001 / HR3X  
 ZAGREBAČKA BANKA d.d. Trg bana Josipa Jelačića 10, 10000 Zagreb, Croatia / IBAN: HR582360000110018100 / Nalog: 001 / HR3X

Founding capital: 16.315.325,00 EUR (paid in full) / Number of issued shares: 1.255.025 / Share nominal value: 1,000,00 HRK  
 President of the Management Board: Ivo Usmanj / Executive Director: Mladen Vučić  
 Personal identification no.: 20950636972 / VAT no.: HR20950636972 / Tel: +385 51 6887000 / Fax: +385 51 6887000 / E-mail: [info@jgl.hr](mailto:info@jgl.hr) / [www.jgl.hr](http://www.jgl.hr)

|                                                                                                                                                                                                                          |                  |                  |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------|------------------|
| Valve Outer Diameter                                                                                                                                                                                                     | 32.42 – 32.58 mm | 32.61 – 32.89 mm |
| Valve Inner Diameter                                                                                                                                                                                                     | 25.07 – 25.23 mm | 25.05 – 25.25 mm |
| Parameters of the bag with valve system, 150 ml                                                                                                                                                                          |                  |                  |
| Parameter                                                                                                                                                                                                                | Meaning          |                  |
|                                                                                                                                                                                                                          | Coster           | Aptar            |
| Length                                                                                                                                                                                                                   | 123-125mm        | 115-117 mm       |
| Width                                                                                                                                                                                                                    | 99-101mm         | 96-98 mm         |
| Valve Outer Diameter                                                                                                                                                                                                     | 32.42-32.58 mm   | 32.61-32.89 mm   |
| Valve Inner Diameter                                                                                                                                                                                                     | 25.07-25.23mm    | 25.05-25.25mm    |
| Parameters of the nozzle for irrigation in the product for nose rinsing and irrigation “Aqua Maris® Baby. Intensive rinsing”                                                                                             |                  |                  |
| Parameter                                                                                                                                                                                                                | Meaning          |                  |
|                                                                                                                                                                                                                          | Coster           | Aptar            |
| Length                                                                                                                                                                                                                   | 39.5 – 40.3 mm   | 39.7 – 40.5 mm   |
| Width                                                                                                                                                                                                                    | 23.1 – 23.7 mm   | 23.5 – 24.1 mm   |
| Spray intensity                                                                                                                                                                                                          | 0,17-0,27 g/s    | 0,17-0,27 g/s    |
| Dispersion of spraying                                                                                                                                                                                                   | 20-60 µm         | 20-60 µm         |
| Parameters of the nozzle for irrigation in the product for nose rinsing and irrigation “Aqua Maris® Norm. Intensive rinsing”                                                                                             |                  |                  |
| Parameter                                                                                                                                                                                                                | Meaning          |                  |
|                                                                                                                                                                                                                          | Coster           | Aptar            |
| Length                                                                                                                                                                                                                   | 39.5 – 40.3 mm   | 39.6 – 40.4 mm   |
| Width                                                                                                                                                                                                                    | 23.1 – 23.7 mm   | 23.5 – 24.1 mm   |
| Spray intensity                                                                                                                                                                                                          | 0,47-1,25 g/s    | 0,47-1,25 g/s    |
| Dispersion of spraying                                                                                                                                                                                                   | 20-60 µm         | 20-60 µm         |
| Parameters of the universal nozzle for irrigation in the product for nose rinsing and irrigation “Aqua Maris® Norm. Intensive rinsing” and product for nose rinsing and irrigation “Aqua Maris® Baby. Intensive rinsing” |                  |                  |
| Parameter                                                                                                                                                                                                                | Meaning          |                  |
|                                                                                                                                                                                                                          | Coster           | Aptar            |
| Length                                                                                                                                                                                                                   | 37.95 – 38.75 mm |                  |
| Width                                                                                                                                                                                                                    | 34.1 – 35.1 mm   |                  |
| Spray intensity                                                                                                                                                                                                          | 0,47-1,25 g/s    |                  |
| Dispersion of spraying                                                                                                                                                                                                   | 20-60 µm         |                  |
| Cap Parameters                                                                                                                                                                                                           |                  |                  |
| Parameter                                                                                                                                                                                                                | Meaning          |                  |
|                                                                                                                                                                                                                          | Coster           | Tehnoplast Aptar |

JADRAN-GALENSKI LABORATORIJI d.d. (JGL d.d.), Svrlin 20, 51000 Rijeka, Croatia; Commercial register in Rijeka register 5100-HL; OIB no. 640004561

DTP BANKA d.d., Domjovinskog rata 61, 21000 Split, Croatia; IBAN: HRB424070001100516219; SWIFT: DTPJHR2X  
 Erste Steiermärkische Bank d.d., Jadranski trg 3a, 51000 Rijeka, Croatia; IBAN: HR3424070061100026110; SWIFT: ERSIHR33  
 PRIVREDNA BANKA ZAGREB d.d., Radnička cesta 50, 10000 Zagreb, Croatia; IBAN: HR6623400091111257380; SWIFT: RBZCZ333  
 ZAGREBAČKA BANKA d.d., Trg bana Josipa Jelačića 10, 10000 Zagreb, Croatia; IBAN: HR5623600071111081200; SWIFT: ZABAHR3X

Founding capital: 16 315 325.00 EUR (paid in full) / Number of issued shares: 1 255 025 / Share nominal value: 13.00 EUR  
 President of the Management Board: Ivo Usmarić / Executive Director: Mladen Vojak  
 Personal identification no.: 20950636972 / VAT no.: HR20950636972 / Tel.: +385 51 660 700 / Fax: +385 51 660 122 / E-mail: [info@jgl.hr](mailto:info@jgl.hr) / [www.jgl.hr](http://www.jgl.hr)

|        |              |                  |                |
|--------|--------------|------------------|----------------|
| Length | 46-48 mm     | 46.2 – 47.8 mm   | 41.2 – 42.2 mm |
| Width  | 33.7-35.7 mm | 33.55 – 35.15 mm | 34.1 – 35.1 mm |

Table 13. General characteristics

|                                                                                                   |                                      |
|---------------------------------------------------------------------------------------------------|--------------------------------------|
| <i>Weight of the solution in the vial:</i>                                                        |                                      |
| - 50 ml;                                                                                          | 49,1 – 51,1 g                        |
| - 125 ml;                                                                                         | 122,5 – 127,5 g                      |
| - 150 ml.                                                                                         | 147,0 – 153,0 g                      |
| Pressure required to trigger the spray (for all nozzle for irrigation types)                      | from 19 to 33 H                      |
| Pressure in the Product for nose rinsing and irrigation "Aqua Maris® Baby. Intensive rinsing"     | $6,0 \cdot 10^5 - 7,2 \cdot 10^5$ Pa |
| The pressure in the Product for nose rinsing and irrigation "Aqua Maris® Norm. Intensive rinsing" | $6,0 \cdot 10^5 - 8,0 \cdot 10^5$ Pa |

## Information about the manufacturer and the organization accepting claims:

### Manufacturer:

JADRAN-GALENSKI LABORATORIJ d.d., Svilno 20, 51000 Rijeka, Croatia

### Place of production:

JADRAN-GALENSKI LABORATORIJ d.d., Svilno 20, 51000 Rijeka, Croatia

## Claims to the quality of the product should be sent to the Limited Liability Company "JADRAN":

119330, Moscow, Lomonosovsky prospekt, 38, office VII, tel./fax: +7(499)143-33-71.

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Founding capital: 16.315.325,00 EUR (paid in full) / Number of issued shares: 1.255.025 / share nominal value: 13,00 EUR  
President of the Management Board: Ivo Usman / Executive Director: Miroslav Vukobratović  
Personal identification no.: 20950636972 / VAT no.: HR20950636972 / Tel.: +385 (0)51 70011 / Fax: +385 (0)51 70012 / E-mail: [info@jgl.hr](mailto:info@jgl.hr) / [www.jgl.hr](http://www.jgl.hr)

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