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	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 1 of 14

Instructions for use

Read this instruction manual carefully before using the product.

Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions:

YRN-PL29-01, YRN-PL30-02, YRN-PL29-02, YRN-PL27-03, YRN-PL27-02, YRN-PL26-05, YRN-PL26-03, YRT-PL26-02, YRS-PL26-03, YRS-PL27-02, YRS-PL27-05, YRDS-PL26-03, YRPN-PL21-40-L, YRPN-PL23-40-L, YRPN-PL21-80-L, YRPN-PL23-80-L, YRN30-01, YRN30-02, YRN29-02, YRN27-03, YRN27-02, YRN26-05, YRN26-03, YRS29-02, YRS26-03, YRS27-02, YRT26-03, YRDS26-03, YRPN19-43-L, YRPN19-91, YRPN19-96-L, YRPN18-126-L, YRDN-SC-AN-01, YRDN-FC-AN-01

1. YRN-PL29-01 includes:

- PLA thread
- General needle 29G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

2. YRN-PL30-02 includes:

- PLA thread
- General needle 30G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

3. YRN-PL29-02 includes:

- PLA thread
- General needle 29G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

4. YRN-PL27-03 includes:

- PLA thread
- General needle 27G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

5. YRN-PL27-02 includes:

- PLA thread
- General needle 27G,
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

6. YRN-PL26-05 includes:

- PLA thread
- General needle 26G
- Sponge (thread fixation to the needle)

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 2 of 14

- Protective cap
 - Instructions for use
7. YRN-PL26-03 includes:
- PLA thread
 - General needle 26G
 - Sponge (thread fixation to the needle)
 - Protective cap
 - Instructions for use
8. YRT-PL26-02 includes:
- PLA thread
 - General needle 26G
 - Sponge (thread fixation to the needle)
 - Protective cap
 - Instructions for use
9. YRS-PL26-03 includes:
- PLA thread
 - General needle 26G
 - Sponge (thread fixation to the needle)
 - Protective cap
 - Instructions for use
10. YRS-PL27-02 includes:
- PLA thread
 - General needle 27G
 - Sponge (thread fixation to the needle)
 - Protective cap
 - Instructions for use
11. YRS-PL27-05 includes:
- PLA thread
 - General needle 27G
 - Sponge (thread fixation to the needle)
 - Protective cap
 - Instructions for use
12. YRDS-PL26-03 includes:
- PLA thread
 - General needle 26G
 - Sponge (thread fixation to the needle)
 - Protective cap
 - Instructions for use
13. YRPN-PL21-40-L includes:
- PLA thread
 - General "L" blunt needle 21G
 - Sponge (thread fixation to the needle)
 - Protective cap
 - Instructions for use

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 3 of 14

14. YRPN-PL23-40-L includes:

- PLA thread
- General "L" blunt needle 23G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

15. YRPN-PL21-80-L includes:

- PLA thread
- General "L" blunt needle 21G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

16. YRPN-PL23-80-L includes:

- PLA thread
- General "L" blunt needle 23G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

17. YRN30-01 includes:

- PDO thread
- General needle 30G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

18. YRN30-02 includes:

- PDO thread
- General needle 30G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

19. YRN29-02 includes:

- PDO thread
- General needle 29G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

20. YRN27-03 includes:

- PDO thread
- General needle 27G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 4 of 14

21. YRN27-02 includes:

- PDO thread
- General needle 27G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

22. YRN26-05 includes:

- PDO thread
- General needle 26G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

23. YRN26-03 includes:

- PDO thread
- General needle 26G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

24. YRS29-02 includes:

- PDO thread
- General needle 29G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

25. YRS26-03 includes:

- PDO thread
- General needle 26G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

26. YRS27-02 includes:

- PDO thread
- General needle 27G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

27. YRT26-03 includes:

- PDO thread
- General needle 26G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 5 of 14

28. YRDS26-03 includes:

- PDO thread
- General needle 26G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

29. YRPN19-43-L includes:

- PDO thread
- General "L" blunt needle 19G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

30. YRPN19-91- includes:

- PDO thread
- General sharp needle 19G
- Protective cap
- Instructions for use

31. YRPN19-96-L includes:

- PDO thread
- General "L" blunt needle 19G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

32. YRPN18-126-L includes:

- PDO thread
- General "L" blunt needle 18G
- Sponge (thread fixation to the needle)
- Protective cap
- Instructions for use

33. YRDN-SC-AN-01 includes:

- PDO thread
- Solid general needle 21G
- Case
- Fixations for a thread with the needle in a case
- Instructions for use

34. YRDN-FC-AN-01 includes:

- PDO thread
- Solid general needle 21G
- Case
- Fixations for a thread with the needle in a case
- Instructions for use

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 6 of 14

(Hereinafter, the PDO thread, the PLA thread)

1. Medical device manufacturer's data

Company name: Yurim Medical Co., Ltd.

Registration address:

303, 310, 312-ho (KU Biomedical Research Center), 125, Osongsaengmyeong 2-ro, Osong-eup, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, Korea, Korea

Address of the production place:

501-ho, 7, Gangni 1-gil, Ochang-eup, Cheongwon-gu, Cheongju-si, Chungcheongbuk-do, Korea, Korea

Tel. +82-43-231-9788

e-mail: yyss9787@hanmail.net

3. Medical device developer's data

Company name: Yurim Medical Co., Ltd.

Registration address:

303, 310, 312-ho (KU Biomedical Research Center), 125, Osongsaengmyeong 2-ro, Osong-eup, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, Korea, Korea

Tel. +82-43-231-9788

e-mail: yyss9787@hanmail.net

4. Information on the authorized representative in the Russian Federation

Company name: LLC E+

Registration address: Russian Federation, Moscow, 105062, Mashkova str. 24, apt. 10

5. Intended performance and consumers

The devices are designed for implantation into soft facial and body tissues to correct age-related gravitational changes within the subcutaneous fat layer, superficial muscular-aponeurotic complex, and some facial muscles.

Potential consumers:

patients with age-related changes in the skin of face and body with a reduced level of elastic components of the skin, early manifestations of menopause, consequences of transferred skin diseases, pronounced greasiness, age-related pigmentation.

6. Terms of use

VICTORIA threads are intended for subcutaneous insertion to a patient by qualified medical personnel that underwent special training. The procedure should be carried out in accordance with the appropriate aseptic conditions under hospital conditions in the conditions of health facilities.

7. Indications

- minor gravitational changes in face and neck
- pronounced nasolabial folds, vertical frontal folds
- purse-string wrinkles and horizontal wrinkles of the forehead
- inadequate skin turgor of different parts of the face and body
- sagging tissues due to an abnormality of elasticity

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 7 of 14

If general somatic contraindications are absent, it is possible and desirable to perform RF-lifting to enhance the effect. Products are well combined with other types of rejuvenation (biorevitalization, contour plasty, hardware methods, etc.) and complement each other.

8. Contraindications

- acute infectious diseases (ARVI, influenza, etc.)
- exacerbation of chronic diseases
- disorders of the immune system
- allergies and dermatoses
- oncological diseases
- blood coagulability disorders
- pregnancy
- hypersensitivity to the thread material

If general somatic contraindications are absent, it is possible and desirable to perform RF-lifting to enhance the effect. Products are well combined with other types of rejuvenation (biorevitalization, contour plasty, hardware methods, etc.) and complement each other.

9. Possible side effects when using the device

Local reactions: hematoma, pain, local edema, condensation, erythema, telangiectasia, nodules, granulomas, very rare injectable induced skin reactions, such as mild pain, a sense of pulling and pressure, burning, pruritis.

Infectious complications: extremely rare due to breaches of antiseptic requirements, in some cases the infection occurs at the site of the threads' insertion

The development of hypersensitivity to the threads material is possible

10. Product Features

VICTORIA threads is a sterile synthetic PDO and PLA absorbable suture for single use with a carrier needle for fixation for implantation into soft facial and body tissues to correct age-related gravitational changes within the subcutaneous fat layer, superficial muscular-aponeurotic complex, some mimic muscles.

Polydioxanone composing the thread is a completely biodegradable inert sterile polymer without antigenic and pyrogenic properties. In the process of biodegradation, it decomposes completely into water and CO₂ and causes a minimal local tissue reaction. Decay products are removed through the urine and respiration. It was found that the complete absorption takes about 180 days. As a final result, a very thin frame of fibers of its own collagen is formed, which later fulfills a maintaining function, providing a long lifting - the effect lasts up to 12 months.

Polylactic acid (Poly-L-Lactic Acid) composing the thread is a biosynthetic completely degradable biologically inert sterile polymer without antigenic and pyrogenic features. It decomposes completely into water and carbon dioxide during its biodegradation and causes a minimal local tissue reaction. Decay products are removed through the urine and respiration. It was found that the complete absorption takes about 240 days. As a final result, a very thin frame of fibers of its own collagen is formed, which later fulfills a maintaining function, providing a long lifting - the effect lasts up to 12 months.

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 8 of 14

Depending on the degree of potential risk of medical use, the device "VICTORIA PDO and PLA sterile absorbable threads for lifting tissues on carrier needles of different sizes, hardware versions" refers to Class 3, in accordance with European Directive MDD93 / 42 / EEC as amended 07/47 / EEC Annex IX, Rule 7.

Model	Needle sizes		Suture size	
	Gauge / outer diameter needle (B) (mm)	Needle length (A) (mm) / weight* (g / pcs)	USP size / diameter suture (mm)	Suture length (mm): not less than 98%
YRN-PL29-01	29G (0,324 ~ 0,351)	25 (+ 1,5 / -2,5) / 0,179	6-0 (0,070-0,099)	30
YRN-PL29-02		38 (+ 1,5 / -2,5) / 0,187		60
YRN29-02	29G (0,324 ~ 0,351)	38(+1.5/-2.5) / 0,187	6-0 (0,070-0,099)	60
YRS29-02		38(+1.5/-2.5) / 0,187		60
YRN-PL30-02	30G (0,298 ~ 0,320)	38 (+ 1,5 / -2,5) / 0,182	6-0 (0,070-0,099)	60
YRN30-01		25(+1.5/-2.5) / 0,182		30
YRN30-02		38(+1.5/-2.5) / 0,182		60
YRS-PL27-02	27G (0,400 ~ 0,420)	50 (+ 1,5 / -2,5) / 0,195	5-0 (0,100-0,149)	80
YRN-PL27-02		50 (+ 1,5 / -2,5) / 0,195		80
YRN-PL27-03		60 (+ 1,5 / -2,5) / 0,195		100
YRS-PL27-05		90 (+ 1,5 / -2,5) / 0,195		160
YRN27-03		60(+1.5/-2.5)/ 0,195		100
YRN27-02		50(+1.5/-2.5) / 0,195		80
YRS27-02		50(+1.5/-2.5) / 0,195		80
YRN-PL26-05	26G (0,440 ~ 0,470)	90 (+ 1,5 / -2,5) / 0,225	5-0 (0,100-0,149)	160
YRN-PL26-03		60 (+ 1,5 / -2,5) / 0,225		100
YRS-PL26-03		60 (+ 1,5 / -2,5) / 0,225		100
YRN26-05		90(+1.5/-2.5) / 0,225		160
YRN26-03		60(+1.5/-2.5) / 0,225		100
YRS26-03		60(+1.5/-2.5) / 0,225		100
YRT-PL26-02	26G (0,440 ~ 0,470)	50 (+ 1,5 / -2,5) / 0,225	6-0 (0,070-0,099)	80
YRDS-PL26-03		60(+1.5/-2.5)/0,225	6-0 (0,070-0,099)	100
YRPN-PL21-40-L	21 G (0,800-0,830)	90 (+ 1,5 / -2,5) / 0,32	2-0 (0,300-0,349)	150
YRPN-PL21-80-L		90 (+ 1,5 / -2,5) / 0,32		160

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 9 of 14

YRPN-PL23-80-L	23G (0,600 ~ 0,673)	90 (+ 1,5 / -2,5) / 0,25	3-0 (0,200-0,249)	150
YRPN19-43-L	19G (1,020-1,100)	100(+1.5/-2.5) / 0,385	0-0(0,350-0,399)	205
YRPN19-91		100 (+1.5/-2.5) / 0,385		150
YRPN19-96-L	19G (1,020-1,100)	100(+1.5/-2.5) / 0,385	1-0 (0,400-0,499)	150
YRPN18-126-L	18G (1,020-1,100)	100(+1.5/-2.5) / 0,391		185
YRDN-SC-AN-01	21 G (0,800-0,830)	110(+1.5/-2.5) / 0,61		420
YRDN-FC-AN-01		110(+1.5/-2.5) / 0,61		420
YRPN-PL23-40-L	21 G (0,800-0,830)	90 (+ 1,5 / -2,5) / 0,25	3-0 (0,200-0,249)	150
YRT26-03	26G (0,440 ~ 0,470)	60(+1.5/-2.5) / 0,225	7-0 (0,050-0,069)	100
YRDS26-03	26G (0,440 ~ 0,470)	60(+1.5/-2.5) / 0,225	7-0 (0,050-0,069)	100

10. Completeness

- Smooth PDO thread with a general needle, models:

YRN30-01, YRN30-02, YRN29-02, YRN27-03, YRN27-02, YRN26-05, YRN26-03, YRS29-02, YRS26-03, YRS27-02, YRT26-03, YRDS26-03,

- Cog PDO thread with blunt general "L" or "45°" needle, models: YRPN19-43-L, YRPN18-126-L, YRPN19-96-L

- Smooth PLA thread with a general needle, models:

YRN-PL29-01, YRN-PL30-02, YRN-PL29-02, YRN-PL27-03, YRS-PL27-05, YRN-PL27-02, YRN-PL26-05, YRN-PL26-03, YRT-PL26-02, YRS-PL26-03, YRS-PL27-02, YRDS-PL26-03

- Cog PLA thread with blunt "L" general needle, models: YRPN-PL21-80-L, YRPN-PL23-80-L, YRPN-PL21-40-L, YRPN-PL23-40-L

- Cog PDO thread with two sharp (solid) general needles, models: YRDN-SC-AN-01, YRDN-FC-AN-01

№	Part name	Function
1	PDO thread / PLA thread	for implantation into soft facial and body tissues within the subcutaneous fat layer, superficial muscular-aponeurotic complex, some facial muscles
2	General needle	for inserting the thread within the subcutaneous fat layer, superficial muscle-aponeurotic complex, some facial muscles
3	Sponge (thread fixation to the needle)	To fix suture to the needle
4	Protective cap	To protect the carrier needle with absorbable sterile thread
5	Instructions for use	For understanding how to use this product.

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 10 of 14

№	Part name	Function
1	PDO thread	for implantation into soft facial and body tissues within the subcutaneous fat layer, superficial muscular-aponeurotic complex, some facial muscles
2	Solid general needle	for inserting the thread within the subcutaneous fat layer, superficial muscle-aponeurotic complex, some facial muscles
3	Case	To store the suture with needle
4	Fixations for a thread with the needle in a case	To fix the thread with a needle to the case
5	Instructions for use	For understanding how to use this product.

- Cog PDO thread with a sharp general needle, model:
YRPN19-91

1	Actions before using	for implantation into soft facial and body tissues within the subcutaneous fat layer, superficial muscular-aponeurotic complex, some facial muscles
2	General needle	for inserting the thread within the subcutaneous fat layer, superficial muscle-aponeurotic complex, some facial muscles
3	Protective cap	To protect the carrier needle with absorbable sterile thread
4	Instructions for use	For understanding how to use this product

- Before using the product, the patient's condition should be stable
- Disinfect the area where the procedure will be carried out, for example, with a cotton tampon
- Prepare a single use product "Sterile suture" in the sterile packaging
- Check that the model is proper
- Check expiration date
- Check the integrity of the package
- Apply only after you become familiar with the instructions

12. Sterilization

This medical device is sterilized using ethylene oxide gas (refer to the report on the approval of the ethylene oxide sterilization process No. YRM-Eo-1302), in accordance with:

ISO 11135-1:2007 "Sterilization of health care products — Ethylene oxide — Part 1: Requirements for development, validation and routine control of a sterilization process for medical devices"

The storage life of sterilization for unopened, undamaged packaging is 2 years.

13. How to use the product

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 11 of 14

1. Before performing the medical procedure, calm the patient and explain the rules of conduct during the procedure. Before using the device, the patient's state should be stable
2. Before opening the sterile package of the device, check its integrity, damage absence, and the expiration date.
3. Determine the correct size of the device according to the type of procedure.
4. Release the device from the packaging.
5. Before insertion of the device, fulfill an antiseptic treatment of the therapy area. Disinfect the area to be treated, for example, with a cotton swab with alcohol
6. Perform an application anesthesia.
7. After the application anesthesia, carefully insert the carrier needle with the thread along the intended trajectory for the entire length, until the end of the suture is completely moved, instead of biting the skin with the needle.
8. Then remove the carrier needle, and leave the thread in the tissues.

9. Apply an aseptic bandage to the injured area of the skin. Needle entry points heal quickly and leave no traces. Threads are thin enough, so, sufficiently light that does not lead to their slipping. For the procedure, flexible needles of hyperfine diameter are used, therefore, traces of impact are minimal. Due to the creation of a thread framework in problem areas, a momentary lifting effect is manifested. The patient immediately after the procedure returns to the usual mode of life.

No infiltration anesthesia required for the procedure. Thus, a large volume of infiltration, the risk of an allergic reaction to an anesthetic and massive hematomas are completely excluded.

The procedure is less traumatic than in the case of operations for lifting the skin. No cuts on the skin required.

Do not use the product if it was opened.

Because the product is intended for a single use, it should not be reused.

14. Recycling for the Russian Federation

Dispose of the used product in a safe place.

The used materials are disposed of according to the specified hazard class of medical waste in accordance with Sanitary Regulations and Standards, 2.1.7.2790-10 for class B - epidemiological hazardous waste contaminated with blood and other biological fluids.

Devices that are not used for their intended purpose due to expiring and other reasons are of class A as epidemiological safe waste and are disposed of as household waste.

Standard precautions should be followed when using needles. Do not re-sterilize.

15. Warning

Prevent applying a suture to the following patients:

- a) Patients with malnutrition
- b) Debilitated patients
- c) Patients with hypertrophic scar formation
- d) Patients who receive anticoagulants or platelet aggregation inhibitors (e.g., ACC)
- e) Patients with infected or inflamed skin areas
- f) Patients with autoimmune diseases undergoing immunotherapy
- g) Patients with hypersensitivity to the device
- h) Pregnant and lactating women
- i) Patients under 18

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 12 of 14

16. Precautions

- This device should be used by doctors only
- Do not use the product after the expiration date
- Do not use the product if the packaging is damaged.
- This product should only be used for its intended purpose.
- If the area to be treated is inflamed or infected, the product should not be used.

17. Storage conditions

- Store the product in a clean and dry place.
 - Avoid direct sunlight and high humidity.
- Temperature: $+2^{\circ} \sim +25^{\circ} \text{C}$.
Relative humidity: 10 ~ 80%.
Atmospheric pressure: 500 ~ 1060 hPa.

18. Transportation conditions

Transportation can be carried out by any means of transport in compliance with the requirements for storage:
Temperature: $+2^{\circ} \sim +25^{\circ} \text{C}$.
Relative humidity: 10 ~ 80%.
Atmospheric pressure: 500 ~ 1060 hPa.

19. Terms of use

Medical devices in operation are resistant to the influence of climatic factors under the temperature condition $+32^{\circ}\text{C} \sim +42^{\circ}$.

20. FORBIDDEN!

- Stop taking anticoagulants (e.g., aspirin, marukumar, etc.) a week before the operation
- Stop taking alcohol and smoking before and after operation
- Unexpected indispositions, such as cold or fever, can cause problems with anesthesia; it is necessary to inform the attending physician on such indispositions.

21. Expiration date

Expiration date: 2 years from the date of sterilization
After the expiration date, this medical product must be disposed.

The term of absorption: polydioxanone: complete absorption takes approximately 180 days. As a result, a very thin frame of fibers of its own collagen is formed, which performs a maintaining function providing a long lifting — the effect lasts up to 12 months (360 days).

Poly-L-Lactic Acid: complete absorption takes approximately 240 days. As a final result, a very thin frame of fibers of its own collagen is formed, which later fulfills a maintaining function, providing a long lifting — the effect lasts up to 12 months.

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 13 of 14

22. Warranty statement

The manufacturer guarantees the performance of the medical device for 2 years from the moment of sterilization provided compliance with the rules of transportation, storage, and operation.

23. Description of the symbols

Description of the label symbols on the box and blister

Symbol used	Description	Symbol used	Description
	Symbol "Manufacturer"		Symbol "Temperature range" Storage conditions of medical devices
	Symbol "Keep away from sunlight"		Symbol "Do not reuse"
	Protect from moisture		Symbol "Ethylene oxide sterilization"
	Symbol "Manufacturing standard"		Symbol "Do not re-sterilize"
	Symbol "Caution. Refer to the instructions for use"		Symbol "Do not use the product that packaging is damaged"
	Символ «Верх»		

this symbol means important information. If the device has this symbol, it refers to important information contained in the accompanying documents. This symbol means "Caution"

24. Information on the circumstances when a consumer should consult a health professional:

- 1) During preparation for treatment, to determine the methods and types of threads.
- 2) If complications occur and in two, four and eight weeks after the procedure for regular consultation.

25. Contents and frequency of maintenance:

No maintenance required.

Complaint bureau of the authorized representative of the manufacturer's company in the Russian Federation - LLC E+, Mashkov str. 24, apt. 10, Moscow, 105062

	REPORT ON THE STRUCTURE OF THE INSTRUCTIONS FOR USE	Report No.	YRM-IFU-1602
	Polydioxanone (PDO) and polylactic acid (PLA) sterile absorbable threads VICTORIA for lifting tissues on carrier needles of different sizes, versions	Page	Page 14 of 14

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I swear that the attached translation is
true to the original.

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Registered No. 2018-534

Notarial Certificate

AN HONG LAN----- personally
appeared before me, confirmed that
the attached translation is true to the
original and subscribed his(her) name.

This is hereby attested on this
26th day of Apr. 2018 at this office.

Name of the office

TONGIL LAW & NOTARY OFFICE

Belong to Uijeongbu

District Prosecutor's Office

Address of the office

1187 Jungang-ro, Ilsandong-gu,
Goyang-Si, Gyeonggi-Do, Korea

Shim Jong Shin

Signature of the Notary Public

SHIM JONG SHIN

This office has been authorized by the
Minister of Justice, the Republic of
Korea, to act as Notary Public Since
7, Feb. 2015 Under Law No.9416.

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