Label (case)
Sterile lancets for single use MediCare 30G
(For the handling on the territory of the Russian Federation)

Marketing on the package:

- Painless: Tri-beveled needle
- Safe: Sterile. The lancet is sterilized by ethylene oxide

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The package content:

1. Lancets in a carton packaging
2. User's manual

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Transportation of the packed product is carried out by all kinds of covered vehicles in accordance with the working regulations for this mode of transport.

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Quantity in the package: 200 pieces

Shelf life: 5 years from the date of sterilization. Do not use after the expiration date.

Registration certificate FSZ № **dated**

CERTIFIED TRUE COPY

Victor Koh Koon Hor

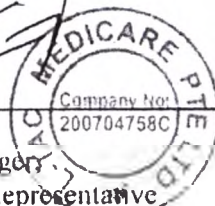
Manufacturer	Concerning the handling of the medical device on the territory of the Russian Federation contact the authorized representative
L-Tac MediCare Pte Ltd 25 Mandai Estate #07-11 Innovation Place Tower 1 Singapore 729930 Tel: +65 6334 2273 Fax: +65- 6863 8190 Manufacturer's address: Shandong Lianfa Medical Plastic Products Co., Ltd Shuangshan Sanjian road, 250200, Zhangqiu, Jinan city, Shandong, China	105043 Formula-FR LLC, Moscow, Pervomayskaya street , 35/18 Tel.: 8 926 208 71 61

ⓧ **IMPORTANT!** Only for single use and individual use

LOT CE 0123

Amos Sim
 Amos Sim
 Business manager
 Management Representative

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Sterile lancets for single use MediCare 30G

User's manual

(Concerning the handling of the medical device on the territory of the Russian Federation)

Purpose: For lancing of a fingertip or other body regions in order to obtain a drop of blood, which subsequently will be used to determine the blood glucose level.

They are intended for use as a part of Blood Glucose Monitoring System MediCare-Lis Model BG-101.

Device description

The lancet is a sterile flat blade, enclosed in a plastic body. The cap and the lancet body are made of HDPE, LDPE.

The needle is of stainless steel, the grade is AISI 304. The high-quality medical steel is used in the production of needles which allows producing the continuous sharpening of needles without edge defects and notches on the needle. This continuous sharpening makes the needle body smooth and the puncture minimally painful. The used needle diameter 30G (0.3 mm) allows to adjust the volume of the obtained blood drop with the help of the sharpening form and the changing of the angle.

Technical characteristics.

The technical characteristics of the lancets are presented in the Table 1.

Table 1.

Needle gauge	The depth of the puncture	Recommendations for use	Needle form
30G	1,8 mm	When a small amount of capillary blood is needed.	Tri-beveled needle point

The lancet length is 32 mm $\pm$ 1 mm; the width is 6,5 mm $\pm$ 0,5 mm; the thickness is 6,2 mm $\pm$ 0,5 mm.

The size of the needle's cap is 9 mm $\pm$ 0,5 mm.

The needle length is 3,3 mm $\pm$ 0,3 mm.

The diameter of the 30G needle: (0,298-0,320) mm.

The needle is tightly fixed in the body.

The lancet is sterilized by ethylene oxide.

The lancet weight is 0,55g.

Preparation for the operation, operation order of the device

1. Remove carefully the protective cap.
2. Insert the lancet in the lancing device.

Precautionary measures:

- Do not use the lancet if the protective cap is damaged or missing;
- Before using the lancet, wash your hands with warm water and soap, then rinse thoroughly and dry;
- Disinfect the intended puncture site with an alcohol wipe;
- Use the tight-fitting snap caps for the safe disposal of lancets.

Storage and transportation conditions: Keep out of children's reach at a temperature of 10 °C to 40 °C at a relative humidity 10-85 %.

CERTIFIED TRUE COPY

Victor Koh

Transportation of the packed product is carried out by all kinds of covered vehicle in accordance with the working regulations for this mode of transport.

Utilization conditions: The used products are Class B waste and disposed of in accordance with SanPiP 2.1.7.2790-10 "Sanitary - epidemiological requirements for the treatment of medical waste (approved by Decision of the Chief State Sanitary Physician of the Russian Federation of December 9, 2010 №163).

Shelf life: 5 years from the date of sterilization. **Do not use after the expiration date.**
Registration certificate FSZ № **dated**

Manufacturer	Concerning the handling of the medical device on the territory of the Russian Federation contact the authorized representative
L-Tac MediCare Pte Ltd 25 Mandai Estate #07-11 Innovation Place Tower 1 Singapore 729930 Tel: +65 6334 2273 Fax: +65- 6863 8190 Manufacturer's address:  Shandong Lianfa Medical Plastic Products Co., Ltd Shuangshan Sanjian road, 250200, Zhangqiu, Jinan city, Shandong, China	105043 Formula-FR LLC, Moscow, Pervomayskaya street , 35/18 Tel.: 8 926 208 71 61

ⓧ IMPORTANT! Only for single use and individual use

CE 0123

LOT

Amos Sim
Business manager
Management Representative

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