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GIOVANNI OGNA & FIGLI S.R.L.
Via Figini 41 – 20835 Muggio (MB) Italy
Phone: +390392782954, Fax: +390392782958
e-mail: ognalab@ognalaboratori.it, www.ognalaboratori.it

Exploitation documentation (instructions for use)

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e-mail: ognalab@ognalaboratori.it, www.ognalaboratori.it

Exploitation documentation (instructions for use)

Dental material for treating root canals Niclor 5 in a glass vial

1. Manufacturer:

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Via Figini 41 – 20835 Muggio (MB) Italy

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Manufacture address: Via Figini 41 – 20835 Muggio (MB) Italy

2. Specification of the medical device:

Glass vial 250 ml.

3. Composition:

NaOCl titrated to 5% of active chlorine

4. Description:

Niclor 5 is a ready for use stabilized and purified solution of NaOCl titrated to 5% of active chlorine. Niclor 5 should be used for irrigation of root canal system during endodontic treatment using a syringe and an appropriate endodontic needle. Niclor 5 effectively dissolves necrotic and organic components of the smear layer which appears during instrumentation of the root canal. This provides an effective irrigation.

5. Properties:

Effective dilution of necrotic and organic debris of pulp and smear layer (during root canal instrumentation). Effective irrigation of root canal system.

6. Instructions for use and preparing the material:

Open the primary package by tearing the aluminum cover and internal rubber cover. Take the solution into the syringe and start irrigation of the root canal with the appropriate endodontic needle. Do not insert the needle too deep on the root canal in order to avoid the excessive apical pressure and as a sequence – leading out of apex. After finishing the irrigation take the solution out of the canal. You can use Niclor 5 during the instrumentation (by alternating Niclor 5 with an instrument) and also by itself as an independent procedure to irrigate passively (for

example after instrumentation). After irrigation with Niclor 5 continue the root canal treating according to your protocol (for example EDTA to delete the smear layer) and/or start the next endodontic procedures (for example drying and final obturation – according to your protocol).

7. Application:

Dental material Niclor 5 should be used only in dental practice.

8. Indications:

all kinds of diseases of root canal system

9. Contraindications:

Don't use in case of open apex of the tooth

10. Precautions/safety requirements:

while working with the material doctor should use the specialized protecting medical clothes, gloves or glasses. For external use. Not for injection. Not for oral usage. Keep in place out of children. Don't use after expiration date.

11. Side effects:

Material can irritate skin and eyes

12. Shelf life / Transportation, exploitation and storage conditions:

The shelf life of the product depends on the Surrounding temperature:

Storage at temperature 25C: 24 months

Storage at temperature 45 C: 6 months

The indicated expiration date on the box of the product supposes the corresponding storage conditions (25C). After opening the material should be used before expiration dated mentioned on the package if the user follows the storage conditions.

Relative humidity: from 10 to 80 %

13. Guaranties of the manufacturer:

If the user follows the requirements for transportation, storage and application, the manufacturer guaranties the quality of the medical device as follows: effective dilution of necrotic and organic components of the smear layer during the root canal instrumentation. Minimally allowed concentration of NaOCl – 2,5%.

14. Technical characteristics:

Solution base: Water

Ph index: 12,5

Organoleptic characteristics:

Color: transparent yellow

Smell: classic smell of chlorine

Density: 1.0952 g/ml

15. Information about utilization (destruction):

Niclor 5 does not present an epidemiological, toxicological or radiation hazard, does not have a negative impact on the environment, so there is no need to disinfect / neutralize this medical device for utilization purposes. The utilization / destruction of this medical device in the territory of the Russian Federation must be carried out in accordance with the rules for the utilization of medical waste that are valid on the territory of the Russian Federation, taking into account its epidemiological, toxicological and radiation safety.

16. Authorized representative of the manufacturer on the territory of Russian Federation:

DENTALINK LTD

INN 7709854799

KPP 773401001

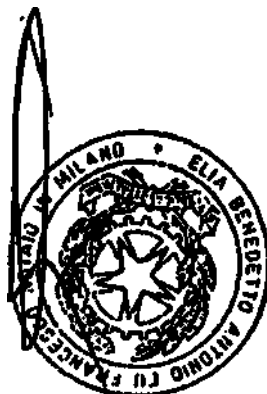
Address: 123592 Moscow, Kulakova st, 20 building 1A, Unit XVII, room 64, floor 5

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