WSM

WEAR SAFE (MALAYSIA) SDN BHD

(Company № 204396-X)

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Signed by,

Mr. Lawrence Lim
Marketing General Manager

Date: 27/01/2015

Surgical sterile latex powder free disposable gloves «iNtouch»

Instruction Manual

CONTENT

1	INTENDED USE	3
2	INDICATIONS FOR USE	3
3	CONTRA-INDICATIONS FOR USE	3
4	SIDE EFFECTS	3
5	DIRECTIONS FOR USE	3
6	SPECIFICATIONS	6
7	DISPOSAL AND ENVIRONMENTAL PROTECTION	8
8	TRANSPORTATION AND STORAGE	8
9	PACKAGE CONTENTS	8
10	MARKING	8
11	MANUFACTURER WARRANTY	9
12	MANUFACTURER INFORMATION	9
13	RESPONSIBLE SUPPLIER	9

1. INTENDED USE

The current Instruction Manual applies to Surgical sterile latex powder free disposable gloves «iNtouch» (hereinafter – gloves), are designed for medical technician and patient protection against cross contamination during surgical operations.

Conditions of use: medical and health care institutions.

Surgical sterile latex powder free disposable gloves «iNtouch» are made to be the following models: (see Annex 1)

2. INDICATIONS FOR USE:

Carrying out of the different procedures during maintaining surgical operations in the universal medical treatment and preventive care institutions.

3. CONTRA-INDICATIONS FOR USE

Dermatitis, allergic reactions to latex.

4. SIDE EFFECTS

No

5. DIRECTIONS FOR USE:

5.1 General provisions

The product is intended as a hand protective device to protect healthcare personnel hands from contamination and to decrease transmission of potentially pathogenic microorganisms.

The proper technique for removing and donning of the gloves is necessary, it prevents contamination of medical professionals hands.

If the integrity of the glove is broken (for example, if it is punctured), it should be replaced as soon as possible and hands must be processed with antiseptic agent.

Using of the additional pair of gloves reduces the damage rate in the 2-4 ratio when working with sharp instruments.


5.2 Pre-starting procedures:

5.2.1 Before gloves using make sure of package integrity. Then open the outer pack. The inner pack of the gloves is sterile. It is necessary to perform hand hygiene before the inner pack opening, picking up and donning of the gloves.

5.3 Procedure of work with device:

5.3.1 Donning of the gloves requires special techniques in order to avoid contamination during donning process. There is two donning techniques for gloves: open and closed. The main thing when using any technique is to avoid contamination of its outer surface.

5.3.2 Open donning steps

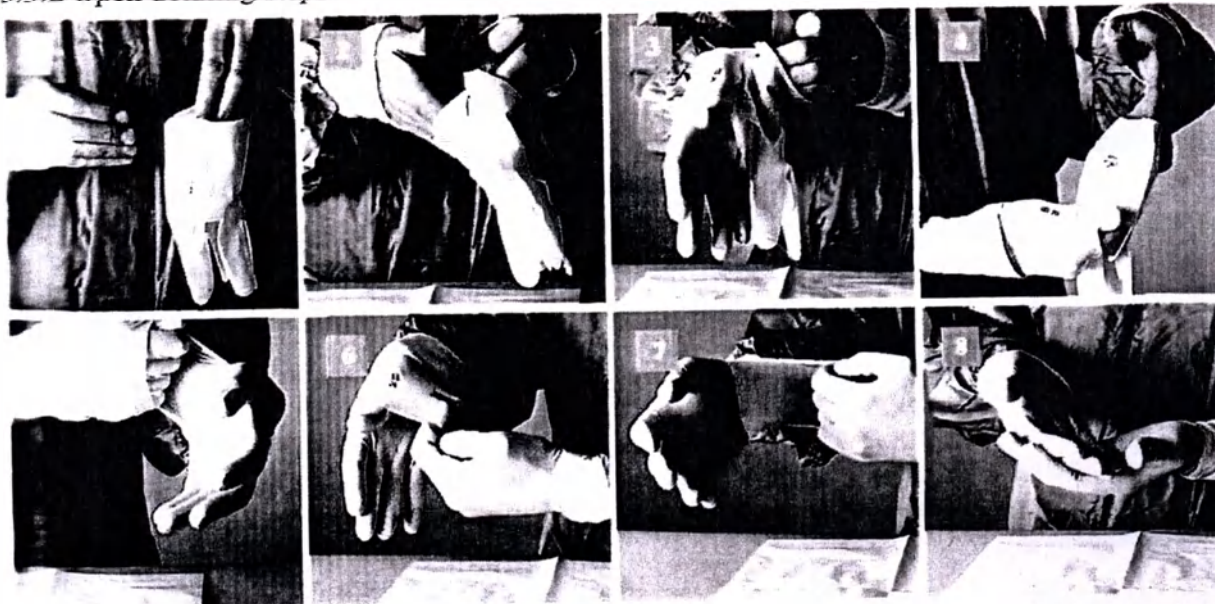- Peel open the package and pick up both gloves with your left fingertips on the inside of the left glove so that the fingertip of the thumb is inside the right glove. Hold the gloves vertically together. Ensure the fingers are in contact with internal surface of the gloves;
- Slide your right hand into the right glove whilst maintaining cuff grip with your left hand;
- Pull on the folded glove cuffs until the right hand is in the glove;
- Place your gloved right fingers under the left glove cuff and slide your left hand into the glove;
- With all four fingers, pull up the left glove cuff over the gown cuff;
- Pull with all four fingers on the glove cuff to cover the gown cuff for comfortable use;

5.3.3 Closed donning steps

- Peel open the package before handwashing. The inner pack is sterile; in order to open the package and pick up the gloves it is necessary to use sterile gown;
- With your gown covering your fingers, use your right hand to remove the left glove. Hold your left hand palm up, fingers straight. Lay the glove on your left wrist with the fingers of the glove pointing toward you and right thumb is on the wrist. Grip the cuff with your left thumb;

- Place your right thumb inside the top cuff edge. Make a fist with your right hand and stretch the glove over your fingertips;
- Keeping your left fingers straight and holding the cuff with right hand, pull down the glove with four fingers of right hand until your hands fills the glove;
- Repeat the above procedure to don the other glove. That is: use your gloved left hand to lay the right glove on your right wrist with the fingers of the glove pointing toward you and thumb of the glove is on the wrist. Grip the cuff with your right thumb;
- Slide your left thumb inside the top of the cuff, make a fist, and stretch the cuff over your fingertips;
- Keeping your right fingers straight and covering the hand with the cuff of your sterile gown, pull down the glove with four fingers of left hand until your hands fills the glove;
- Now with both hands covered you can attend to finer adjustment of both gloves without risk of contamination.
- To prevent glove/gown separation ensure sufficient length of your gown sleeves and sufficient length of the gown cuff down to the base of your palm and absence of folds on the glove cuff.

5.3.4 Glove Removal

- Grab one glove with the fingers of other hand at the wrist
- pull the glove away and hold it with other hand
- Then place your bare fingers of ungloved hand inside the cuff on the other hand and pull it away in a similar manner as first glove.
- turn inside out the glove and use it as the bag for other glove.
- Put the gloves in dedicated container. Then perform the hand hygiene


6. SPECIFICATIONS

6.1 Sizes of the gloves describe in the Annex 2.


6.1.1 Gloves comply with the palm width and the palm length standards as well as wall thickness, (please see table 1).

Table 1 - Sizes of the surgical gloves

Description	Size							
	5,5	6	6,5	7	7,5	8	8,5	9
Width W by size (pic.1), mm	72±4	70±10	80±10	85±10	95±10	110±10	120±10	130±10
Length L, not less (pic. 1), mm	250	260	260	270	270	270	280	280
Thickness, not less (pic.2), mm								
- of finger	0,08							
- of palm	0,08							
- of cuff	0,08							

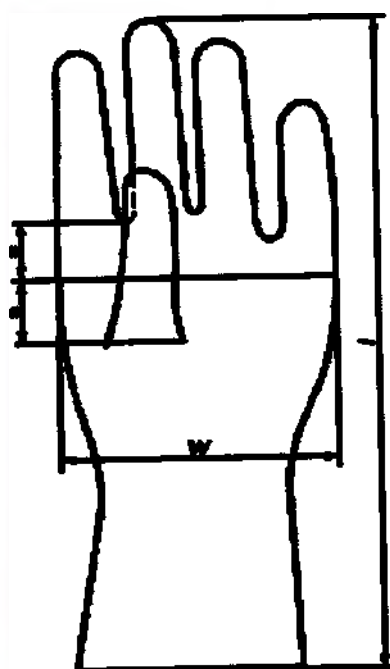Picture 1 - Measurement points of the glove width and length

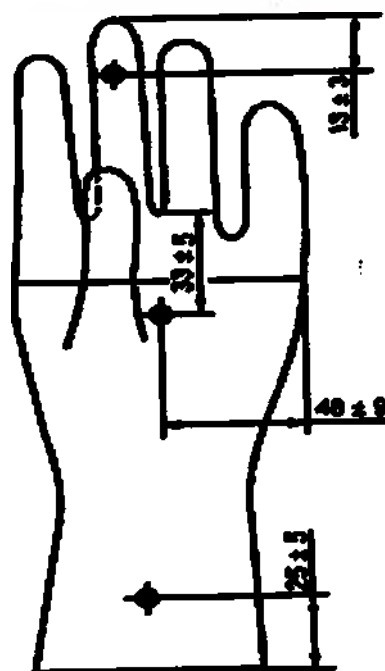Picture 2 - Measurement points of the glove wall thickness

6.2 Strength characteristics of the surgical gloves.

6.2.1 Tensile characteristics of the surgical gloves should comply with Table 2.

Table 2- Tensile characteristics of the surgical gloves.

Characteristic	Value
1 Nominal strength before accelerated aging, MPa, not less	18
2 Nominal strength after accelerated aging, MPa, not less	14
3 percentage elongation before accelerated aging, %, not less	650
4 percentage elongation after accelerated aging, %, not less	500

6.3 Gloves are manufactured from materials shown at Table 3.

Table 3 Materials (Raw materials), which have a contact with healthcare personnel and (or) patient				
No	Name	Used material		Manufacturer
1	Powder free textured fingertips latex surgical beaded cuff anatomic gloves Intouch Eco: INTW0955, INTW0960, INTW0965, INTW0970, INTW0975, INTW0980, INTW0985, INTW0990	REVULTEX LR brand Natural rubber latex		WEAR SAFE (MALAYSIA) SDN. BHD.
2	Powder free textured fingertips latex surgical beaded cuff anatomic gloves with polymer coating Intouch Powder Free: INTW0555, INTW0560, INTW0565, INTW0570, INTW0575, INTW0580, INTW0585, INTW0590	glove	REVULTEX LR brand Natural rubber latex	WEAR SAFE (MALAYSIA) SDN. BHD.
		Polymeric coated	P300 brand PLA (poylactide)	
3	Powder free textured fingertips latex surgical beaded cuff anatomic chlorinated gloves Intouch Powder Free: INTW0455, INTW0460, INTW0465, INTW0470, INTW0475, INTW0480, INTW0485, INTW0490	REVULTEX LR brand Natural rubber latex		WEAR SAFE (MALAYSIA) SDN. BHD.
4	Powder free textured fingertips latex surgical beaded cuff anatomic gloves with ultrapure surface Intouch MPXX: INTW1155, INTW1160, INTW1165, INTW1170, INTW1175, INTW1180, INTW1185, INTW1190	REVULTEX LR brand Natural rubber latex		WEAR SAFE (MALAYSIA) SDN. BHD.
5	Powder free textured with thin film on fingertips latex surgical beaded cuff anatomic gloves Intouch Micro: INTW1255, INTW1260, INTW1265, INTW1270, INTW1275, INTW1280, INTW1285, INTW1290	REVULTEX LR brand Natural rubber latex		WEAR SAFE (MALAYSIA) SDN. BHD.
6	Powder free textured fingertips polymer coated latex surgical beaded cuff anatomic gloves Intouch Slide: INTW0355, INTW0360, INTW0365, INTW0370, INTW0375, INTW0380, INTW0385, INTW0390	glove	REVULTEX LR brand Natural rubber latex	WEAR SAFE (MALAYSIA) SDN. BHD.
		Polymeric coated	P300 brand PLA (poylactide)	
7	Powder free textured fingertips latex surgical beaded cuff anatomic gloves for extra protection Intouch Spot: INTW0755, INTW0760, INTW0765, INTW0770, INTW0775, INTW0780, INTW0785, INTW0790	glove	REVULTEX LR brand Natural rubber latex	WEAR SAFE (MALAYSIA) SDN. BHD.
		blue colorant	VS brand indulin	

6.4 Features

This product has the following features:

- sterile;
- textured fingertips;
- powder free: polymeric coated – PLA (poylactide) or chlorinated;
- Single use.

The texture of the outer surface can improve the capture of the medical instrument. In order to avoid sticking of the gloves, we conduct modification of the surface using chlorine. Useful dyes, non-toxic coatings. Agents PLA (poylactide) used for the surface treatment biodegradable.

6.5 Chemicals permeability

Code Letter	Chemicals	Typical Breakthrough time
A	Methanol (CAS № 7664-93-9)	Before 15 minutes
B	Acetone (CAS № 67-64-1)	After 5 minutes
C	Acetonitrile (CAS № 75-05-08)	Before 15 minutes
D	Dichloromethane (CAS № 75-09-2)	Before 15 minutes
E	Carbon disulfide (CAS № 75-15-0)	After 240 minutes

F	Toluene (CAS № 108-88-3)	Before 5 minutes
G	Diethylamine (CAS № 109-89-7)	Before 30 minutes
H	Tetrahydrofuran (CAS № 109-99-9)	Before 15 minutes
I	Ethyl acetate (CAS № 141-78-6)	Before 45 minutes
J	n-Heptane (CAS № 142-85-5)	Before 15 minutes
K	Sodium Hydroxide 40% (CAS № 1310-73-2)	After 5 minutes
L	Sulfuric 96% (CAS № 7664-93-9)	Before 5 minutes
-	Formaldehyde 15% (CAS № 50-00-0)	Before 15 minutes

7. DISPOSAL AND ENVIRONMENTAL PROTECTION

Gloves must be assembled and transferred to a special licensed organizations engaged in the disposal of medical waste. Disposal should be carry out in accordance with local legislation current standards and requirements.

8. TRANSPORTATION AND STORAGE.

8.1 The transportation of the gloves should be carry out by any means of transport in covered vehicles in accordance with rules of cargo transportation valid for this transport.

8.2. If transported at temperature below zero, the gloves should be stored at storage conditions within 1 day before unpacking.

8.3 Gloves should be stored packed in rooms at temperature from +5 °C up to +37 °C and relative humidity not above 80%.

During storage gloves should be protected from direct sunlight, should be placed at the distance 1 m from heating appliances, should not be impacted by oil, gasoline, kerosene (and other organic solvents), as well as other corrosive substances.

9 PACKAGE CONTENTS

#	Description	Quantity, pc.
1	Surgical sterile latex powder free disposable gloves «iNtouch», *	1 pair
2	Instruction manual	1
3	Warranty	1

*- the customer can choose models, the list see in Annex 1

10. MARKING

10.1 The symbols used in marking.

- Serial number

- Lot No:

(number)

- " SINGLE USE ONLY "

- " Manufacturing date "

(year/month)

- " Best by ... "

(year/month)

- "STERILE UNLESS PACKAGE IS OPENED OR DAMAGED "

- "The product is made of natural latex rubber, it may cause allergy ".

- "Irradiation sterilization "

11. MANUFACTURER WARRANTY

The manufacturer guarantees compliance of the gloves to manual requirements if the operating, transportation and storage conditions are satisfied by a customer.
Guaranteed shelf life for the gloves is 3 years from the day of manufacturing.

12. MANUFACTURER INFORMATION:

WEAR SAFE (MALAYSIA) SDN. BHD., Lot 5068 & 5069, Batu 4 ½, Jalan Meru, 41050 Klang, Selangor Darul Ehsan. Malaysia. Tel.: 03-3392 1388, E-MAIL info@wearsafe.com.my.

13. RESPONSIBLE SUPPLIER:

Limited Liability Company "GLOMAKO" (GLOMAKO LLC), 196128, Russia, Saint-Petersburg, Varshavskaya street, house 5, building 2A, office 604-A Telephone +7 (812)6358898

Annex 1

I. Surgical sterile latex powder free disposable gloves iNtouch, possible types:

1. Powder free textured fingertips latex surgical beaded cuff anatomic gloves Intouch Eco: INTW0955, INTW0960, INTW0965, INTW0970, INTW0975, INTW0980, INTW0985, INTW0990;

2. Powder free textured fingertips latex surgical beaded cuff anatomic gloves with polymer coating Intouch Powder Free: INTW0555, INTW0560, INTW0565, INTW0570, INTW0575, INTW0580, INTW0585, INTW0590;

3. Powder free textured fingertips latex surgical beaded cuff anatomic chlorinated gloves Intouch Powder Free: INTW0455, INTW0460, INTW0465, INTW0470, INTW0475, INTW0480, INTW0485, INTW0490;

4. Powder free textured fingertips latex surgical beaded cuff anatomic gloves with ultrapure surface Intouch MPXX: INTW1155, INTW1160, INTW1165, INTW1170, INTW1175, INTW1180, INTW1185, INTW1190;

5. Powder free textured with thin film on fingertips latex surgical beaded cuff anatomic gloves Intouch Micro: INTW1255, INTW1260, INTW1265, INTW1270, INTW1275, INTW1280, INTW1285, INTW1290;

6. Powder free textured fingertips polymer coated latex surgical beaded cuff anatomic gloves Intouch Slide: INTW0355, INTW0360, INTW0365, INTW0370, INTW0375, INTW0380, INTW0385, INTW0390;

7. Powder free textured fingertips latex surgical beaded cuff anatomic gloves for extra protection Intouch Spot: INTW0755, INTW0760, INTW0765, INTW0770, INTW0775, INTW0780, INTW0785, INTW0790.

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