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We, OMRON HEALTHCARE Co., Ltd., 53, Kunotsubo, Terado-cho, Muko, KYOTO, 617-0002 JAPAN submit following technical documents to our authorized representative ComplectService Co Ltd, mansard, bldg. 10, 4 Solnechnogorskaya st., Moscow 125413, Russian Federation (power of attorney dated March 20, 2018), for making changes in the documents of the registration dossier № РД-2692/47063 from 18.12.2013 of a medical device: Peak Flow Meter OMRON PFM20 in the Federal Service for Supervision in the Sphere of Health Care (Roszdravnadzor):

- Instruction manual for Peak Flow Meter OMRON PFM20 of 2 pages.
- Warranty card Peak Flow Meter OMRON PFM20 of 16 pages.

We hereby certify that the submitted documentation developed in Russian language, in accordance with the laws of the Russian Federation.

We confirm the accuracy of the provided information.

4 May, 2018

OMRON HEALTHCARE EUROPE BV

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PO BOX 2000 NL 2140 GL Hoofddorp

TEL +31-23 5544700

FAX +31-23 5544701

André van Gils
President and CEO
OMRON Healthcare Europe B.V.

Single Patient Use




Instructions for Use

IM-PFM20-05-10/2017

Standard Range Peak Flow Meter
BS EN ISO 23747 Standard
(EU Scale)
Universal Range 60 - 800 LPM

OMRON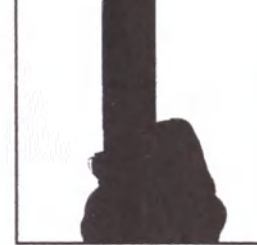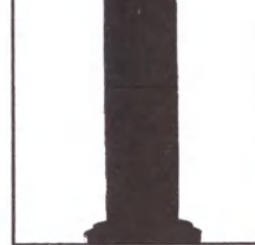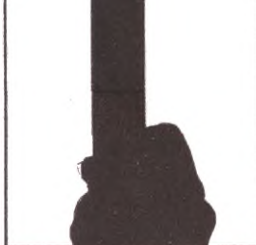

EN

Your OMRON PFM20 Peak Flow Meter will help you to monitor your asthma by measuring your peak flow. This leaflet explains how to use and care for your peak flow meter and how to record the readings. With careful use the OMRON PFM20 would be expected to last for two years or approx. 2,000 uses.

Medical Purpose:

The OMRON PFM20 Peak Flow Meter is a device used to determine lung function by measuring the maximum rate at which air is exhaled from the lungs. This is also referred to as the Peak Expiratory Flow (PEF) or more simply, the Peak Flow.

Intended Use:

This product is intended for use in a medical facility, such as a Hospital, clinic or doctor's office and in a room of general household.

Intended User:

Adults and children age 3* and above (with the help of their caregivers) who wish to monitor their lung function.

* Windmill trainer designed to encourage children

Care and Use of Your Peak Flow Meter

This peak flow meter does not require maintenance (other than cleaning) or calibration for the two years following its first use. At the end of two years, it should be replaced. When not in use this device should be kept in a clean, dry area away from any hazardous substances.

It is important to keep your peak flow meter clean when in use. It should be cleaned at least once every six months and thoroughly if it has not been used for some time. The mouthpiece is detachable and should be disinfected before each use using a proprietary disinfectant solution.

△ Store in a clean dry place.

The OMRON PFM20 peak flow meter is designed for single patient use.

△ **Caution:** Fungal growths may occur if the instrument has not been cleaned for some time.

To personalize the peak flow meter using the zone markers, refer to section "Using the zone monitoring system".

How to use your OMRON PFM20 peak flow meter

1 If required (e.g. for children), insert the extra mouthpiece into the meter, if not already fitted. Ensure the pointer is at the bottom of the scale (nearest the mouthpiece).

2 Hold the peak flow meter by the handle, ensuring that your fingers are clear of the scale and slot.

△ Do not obstruct the holes at the end of the peak flow meter, or the vents at the side of the mouthpiece.

3 Stand up if possible. Take a deep breath. Place the peak flow meter between the lips and hold horizontally, closing the lips around the mouthpiece. Blow as hard and as fast as you can.

△ Please note that any coughing or spitting into the meter will adversely affect your readings and should be avoided.

4 Note the number on the scale indicated by the pointer on a piece of paper. If unusual readings are obtained and the meter does not appear broken - seek advice from your healthcare professional.

5 Return the pointer to the bottom of the scale (nearest the mouthpiece) and repeat the procedure twice more to obtain three readings. Mark the highest of the three readings on your Peak Flow Diary provided overleaf (Refer to 'How to record your peak flow').

Cleaning Instructions
(for Single Patient Use)

Immerse the peak flow meter in warm (but not hot) mild detergent solution for 2-3 minutes (maximum 5 minutes). Agitate the meter to ensure thorough cleaning. Rinse in clean warm water and shake gently to remove any excess water.

Allow to dry thoroughly before using again.

Disposal Note:

The OMRON PFM20 peak flow meter can be disposed of as normal household waste.

Technical

Product code	
Product of	
Model:	
Operating	
Storage at	
Accuracy	
Max. resist	
Weight:	
Dimension	
Durable p	
Package	

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Peak flow meters	
Peak flow meter	
PFM20	
Temperature: 30 to 35°C (86 to 95°F)	
Humidity: 30 to 75%RH (not condensed)	
Pressure: 850 - 1060hPa	
Report rate: 10 to 50°C (23 to 122°F)	
Humidity: 10 to 95%RH (not condensed)	
Flow rate: Better than 10% or 10L/min.	
Flow rate: 0.36 kPa/L/s	
Weight: 44g	
Dimensions: 44 (W) x 139 (H) x 35 (D) mm	
Life span: 2 years	
Contents: 1 Adult / Child PFM20 Peak Flow Meter complete with optional zone markers, universal mouthpiece and instructions with Peak Flow Diary.	

conforms to BS EN ISO 23747 and the NAEP guidelines.

How to Record Your Peak Flow

It is important to keep a record of your peak flow. Three readings should be taken twice a day, first thing in the morning and at bedtime unless otherwise directed by your healthcare professional.

With a dot record the highest of the three readings on the Peak Flow Diary, both in the morning and at bedtime, every day.

By joining the dots (see example, overleaf) you can see how your peak flow varies from day to day.

Make sure to measure and note your peak flow if you suddenly become wheezy or breathless at any time of the day or night.

Using the Zone Monitoring System

The use of a 3 zone monitoring system set up like traffic lights uses coloured plastic clips (green, yellow, red) as zone markers and has proven useful to patients monitoring their asthma with peak flow readings.

Once your green, yellow and

red zones have been determined, record their values in the Treatment Action Plan overleaf.

Positioning the Markers

Set the zone markers on your meter to correspond with the values*. Space has also been provided overleaf to record the actions to be taken, from your Treatment Action Plan, based on your peak flow readings.

The general principle of the zone monitoring system is as follows:

Zone	Condition	Reading (% of best reading)	Action
Green	Go	80-100%	Go ahead with normal activities.
Yellow	Caution	50-80%	Use caution in your activities. Refer to your treatment plan for actions to be taken.
Red	STOP	Less than 50%	Medical alert! Seek immediate medical attention.

These would normally be set by your Doctor or Nurse

* For example, if you and your physician or other licensed healthcare professional determine your individual best reading is 500 LPM, then your Green Zone is 400-500 LPM (80-100%). Set the green marker at 500. Your Yellow Zone is 250-400 LPM (50-80%). Set the yellow marker at 400. Your Red Zone is 0-250 (less than 50%). Set the red marker at 250.

Caution

Do not attempt to remove the coloured clips once they have been attached to the OMRON PFM20. They are designed to stay in their specified position and can not be moved.

This device is not a toy.

Danger of small parts.
Choking hazard.

Troubleshooting

Should you take your peak flow measurement and get an unusually low reading, it could

mean that your peak flow meter is broken or that the reading is accurate showing your asthma is getting worse.

Check to make sure that the meter is not broken by sliding the pointer along the slot, making sure that it moves freely. If you feel comfortable that you are following the instructions correctly and that

your meter is not broken, contact your healthcare professional for advice.

No matter what your peak flow readings are, if you have symptoms such as chest tightness, shortness of breath, coughing or wheezing you should contact your healthcare professional for advice.

Symbols

	Consult 'Instructions for use'
	Caution
	Manufacturer
CE 0197	This product complies with the essential requirements of the medical devices directive (93/42/EEC). Compliance has been verified by UK notified body per 0197 (SGS United Kingdom Ltd).

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President and CEO

OMRON Healthcare B.V.

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OMRON HEALTHCARE EUROPE BV
Scorpius 33

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Mr Cindy Mei Siang Lim LLM, substitute of Mr Charles François Tasseron LLM, civil-law notary at Haarlemmermeer (the Netherlands) certifies that she has established the identity of the aforementioned person(s) according to the legal requirements and confirms the authenticity of the above signature(s) according to article 52 section 2 of the Dutch Notary-Law

This statement explicitly does not contain any judgement regarding the contents of the document or the competence of the aforementioned person(s)

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