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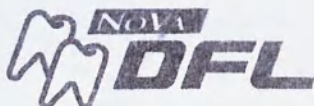
Mauro - d

Mauro Krajcer
Diretor P & D
CPF: 236.515.557-04

Mauro Dagoberto Krajcer
"DFL Industria e Comercio S.A."
Managing Director

2º OFÍCIO

12th November 2019

DFL Industria e Comercio S.A.

Est do Guerengue, 2059, 22.713-001, Jacarepaguá, Rio de Janeiro, Brazil

Name of the document: INSTRUCTIONS FOR USE

Number: DFL-MAN-3A/2 (version for the Russian Federation)

DENTAL RESTORATIVE UNIVERSAL MATERIALS NATURAL, IN SEPARATE PACKAGES

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1. PRODUCT NAME:

Dental restorative universal materials Natural, in separate packages:

1. Dental material **Natural Shade**, in syringe, shade: Enamel A1- 1 pc
2. Dental material **Natural Shade**, in syringe, shade: Enamel A2- 1 pc
3. Dental material **Natural Shade**, in syringe, shade: Enamel A3- 1 pc
4. Dental material **Natural Shade**, in syringe, shade: Enamel A3.5- 1 pc
5. Dental material **Natural Shade** in syringe, shade: Enamel A4- 1 pc
6. Dental material **Natural Shade** in syringe, shade: Enamel B1- 1 pc
7. Dental material **Natural Shade** in syringe, shade: Enamel B2- 1 pc
8. Dental material **Natural Shade** in syringe, shade: Enamel B3 - 1 pc
9. Dental material **Natural Shade** in syringe, shade: Enamel C2- 1 pc
10. Dental material **Natural Shade** in syringe, shade: Enamel C3- 1 pc
11. Dental material **Natural Shade** in syringe, shade: Enamel C4 - 1 pc
12. Dental material **Natural Shade** in syringe, shade: Enamel D3- 1 pc
13. Dental material **Natural Shade** in syringe, shade: Enamel CL Bleach- 1 pc
14. Dental material **Natural Shade** in syringe, shade: Incisal - 1 pc
15. Dental material **Natural Shade** in syringe, shade: Dentin A2 - 1 pc
16. Dental material **Natural Shade** in syringe, shade: Dentin A3- 1 pc
17. Dental material **Natural Shade** in syringe, shade: Dentin A3.5- 1 pc
18. Dental material **Natural Shade** in syringe, shade: Dentin B2 - 1 pc
19. Dental material **Natural Shade** in syringe, shade: Dentin C2- 1 pc
20. Dental material **Natural Shade** in syringe, shade: Dentin Universal- 1 pc
21. Dental material **Natural Shade** in syringe, shade: Enamel A1- 2 pcs
22. Dental material **Natural Shade** in syringe, shade: Enamel A2 - 2 pcs
23. Dental material **Natural Shade** in syringe, shade: Enamel A3- 2 pcs
24. Dental material **Natural Shade** in syringe, shade: Enamel A3.5- 2 pcs
25. Dental material **Natural Z**, in syringe, shade: Enamel A1- 1 pc
26. Dental material **Natural Z**, in syringe, shade: Enamel A2- 1 pc
27. Dental material **Natural Z**, in syringe, shade: Enamel A3- 1 pc
28. Dental material **Natural Z**, in syringe, shade: Enamel A3.5- 1 pc
29. Dental material **Natural Z**, in syringe, shade: Incisal - 1 pc
30. Dental material **Natural Z**, in syringe, shade: Dentin A2- 1 pc
31. Dental material **Natural Z**, in syringe, shade: Dentin Universal- 1 pc
32. Dental material **Natural Ortho** in syringe- 1 pc

Developing, manufacturing and distribution of the product meet the requirements of ISO 13485:2016/NS-EN ISO 13485:2016, registration number – 216851-2017-AQ-BRA-NA-PS Rev. 1.0.

Product code according to the All-Russian Classifier of Products OK 034-2014: OKPD 2 32.50.11.000.

Class, depending on the degree of potential risk of medical use in accordance with the nomenclature classifier of medical devices approved by the order of the Ministry of Health of the Russian Federation date June 6, 2012 No. 4n - 2a.

The product is non-sterile, reusable.

2. PURPOSE:

Materials dental restorative universal **Natural**, produced by "DFL Industry of E Comercio S.A" (DFL Industria e Comercio S.A., Brazil) – in individual packages of consumables used for the photopolymerization of composite resins to fill the cavity of the tooth or repair damaged tooth tissues. This product is intended for multiple use.

Natural Shade is used in aesthetic restorations. Classes I and II, Classes III, IV and V; Making direct veneers, closing diastems. Fixation of dental bars (splints). **Natural Ortho** is designed to fix metal, plastic, ceramic (porcelain) braces on teeth, for subsequent restoration.

Natural Z is used in aesthetic restorations and restorations. Classes I and II, Classes III, IV and V; Making direct veneers, closing diastems. Fixation of dental bars (splints).

3. FIELD OF APPLICATION: Dentistry. For professional use in conditions of health facilities.

4. INDICATIONS FOR USE OF MEDICAL EQUIPMENT:

Natural Shade is indicated for:

- restorations of cavities I, II, III, IV and V classes;
- restores direct veneeres;
- diastemas closure, core preparation;
- periodontal splinting with fibers.

Natural Ortho is indicated for:

- fixing metal, plastic, ceramic (porcelain) braces, for their subsequent restoration.

Natural Z is indicated for:

- restorations of classes I, II, III, IV and V;
- restores direct veneeres;
- diastemas closure, core preparation;
- periodontal splinting with fibers.

5. CONTRAINDICATIONS:

Dental materials **Natural Shade** is contraindicated in patients with an allergic reaction to acrylates or other components with the same composition.

Dental material **Natural Ortho**, is contraindicated in patients whose body exhibits an allergic reaction to photopolymerized monomers based on composites.

Dental material **Natural Z**, contraindicated in patients whose body exhibits an allergic reaction to composite plastics based on polymerized monomers.

If an allergic reaction occurs, seek medical attention, remove the product and discontinue use.

6. Instructions for use

6.1. Application of Natural Shade

Natural Shade is used together with adhesive and acid gel:

1 Selection of color

- 1.1. Perform prophylaxis. The teeth should be clean and moisturized.
- 1.2. Select color, preferably under natural light.

2. Preparation of the cavity

- 2.1. Perform preparation for adhesive restorations.
- 2.2. Make bevel, if necessary.
- 2.3. If necessary, proceed with pulp protection by using a calcium hydroxide and/or Glass Ionomer.

3. Application acid gel

- 3.1. Performing the isolation of the area to be treated, ensuring that the operating field is free of moisture .
- 3.2. Apply the acid conditioner by 30 seconds in enamel and 15 seconds in the dentin.
- 3.3. Rinse and remove excess moisture avoiding dehydrating the dentin.
- 3.4. The surface can not be contaminated. In the event of contamination with saliva, repeat the etching.

4. Application of adhesive

- 4.1. If necessary, use transparent matrix band with interproximal support wooden wedges.
- 4.2. Apply a layer of dentin on adhesive.
- 4.3. Rubbing for 5 seconds, wait 20 seconds and gently applying a stream of air for 5 to 10 seconds.
- 4.4. Apply a second layer of adhesive.
- 4.5. After waiting 20 seconds, light cure the adhesive for 20 seconds.

5. Application NATURAL SHADE

- 5.1. Dispense with a spatula a small amount of **NATURAL SHADE** on the mixing pad.
- 5.2. Apply **NATURAL SHADE** in the cavity in increments up to 2 mm.
- 5.3. Light cure each increment for 20 seconds.
- 5.4. Finish with a light cure of 40 seconds.

6. Finishing

- 6.1. Perform finishing with diamond burs;

7. Check the occlusion

6.2. Application of Natural Ortho

Natural Ortho is used in conjunction with adhesive and acid gel:

INSTRUCTIONS FOR USE:

- 1- Clean the enamel surface with oil and fluor-free prophylactic paste. Wash and dry carefully using an oil-free jet of air.
- 2- Isolate the tooth. Apply the etching gel on the areas where the brackets are to be fixed.
- 3- Wait for 40 seconds (up to 90 seconds for deciduous teeth)
- 4- Wash and isolate the tooth again and then dry carefully with an oil-free jet of air. The area conditioned by the acid should appear whitish. If not, repeat the conditioning process using acid gel for an additional 20 seconds, then rinse and dry.
- 5- Isolate the tooth again and apply a thin layer of the adhesive on the conditioned area. Next, apply **NATURAL ORTHO** to the bracket base. Place the bracket on the tooth and press it firmly against

the enamel surface. Remove the excess around the bracket base, being careful to not displace the bracket.

- 6- Photo cure each bracket for 40 seconds using a good quality, odontologic photo curing unit (470 nm wave length), rotating the light around the bracket. It is important to periodically test the light equipment using a radiometer.

Note: Plastic brackets require special adhesives.

The instructions of the manufacturer should be followed for the conditioning and silanization of the ceramic or porcelain brackets.

6.3. Application of Natural Z

Natural Z is used in conjunction with adhesive and acid gel:

1 Selection of shade

- 1.1. Perform prophylaxis, without fluorine. The teeth should be clean and moisturized.
- 1.2. Select shade, preferably under natural light.

2. Preparation of the cavity

- 2.1. Perform preparation for adhesive restorations.
- 2.2. Make bevel, if necessary.
- 2.3. If necessary, proceed with pulp protection by using a calcium hydroxide and/or Glass Ionomer.

3. Application acid gel

- 3.1. Perform the isolation of the area to be treated, ensuring that the operating field is free of moisture.
- 3.2. Apply the acid conditioner by 30 seconds in enamel and 15 seconds in the dentin.
- 3.3. Rinse and remove moisture excess avoiding dentin dehydration.
- 3.4. The surface cannot be contaminated. In the event of contamination with saliva, repeat the etching.

4. Application of adhesive

- 4.1. If necessary, use transparent matrix band with interproximal support wooden wedges.
- 4.2. Apply a layer of dentin on adhesive.
- 4.3. Rubbing for 5 seconds, wait 20 seconds and gently applying a stream of air for 5 to 10 seconds.
- 4.4. Apply a second layer of adhesive.
- 4.5. After waiting 20 seconds, light cure the adhesive for 20 seconds.

5. Application NATURAL Z

- 5.1. Dispense with a spatula a small amount of **NATURAL Z** on the mixing pad.
- 5.2. Apply **NATURAL Z** in the cavity in increments up to 2 mm.
- 5.3. Light cure each increment for 20 seconds.
- 5.4. Finish with a light cure of 40 seconds.

6. Finishing

- 6.1. Perform finishing with diamond burs;

7. Check the occlusion

WARNING:

- Use gloves when handling materials.
- Avoid prolonged contact with soft tissue or skin. In case of contact, immediately remove the material and rinse with water.
- Avoid contact with eyes. In the event of such a situation, immediately flush eyes with water and seek medical help.
- Avoid contact with eugenol -based products that can slow the polymerization of composite plastics.
- After use, the materials should be stored closed in their original packaging and in accordance with the storage conditions described below, otherwise the materials will lose their properties.

-Do not use waste after using materials. They should be disposed of in accordance with the current legislation of the country.

-Keep out of the reach of children.

-Dental materials should be used only by dentists and for the purposes described in the instructions for use.

DO NOT APPLY TO THE FOLLOWING CASES:

- the package is damaged or opened;
- the product has been damaged;
- the product is used by unqualified persons;
- the product is used after the expiry date;

7. INTERACTIONS WITH OTHER MEDICAL PRODUCTS:

The product is compatible with all restorative materials, such as conditioners, light-curing composites, components of restorative materials and polymer cements.

8. POSSIBLE COMPLICATIONS AND ADVERSE EFFECTS:

Recurrences and complications of tooth decay, polymerization shrinkage, postoperative sensitivity – discomfort, unstable connection with the dentine and, as a result edge slit, besides possibility of discharge of residual unreacted monomers as a result of incomplete polymerization, may promote development of allergic reaction, stimulate bacterial growth among restoration, may cause complications of tooth decay (pulp necrosis, periodontal disease).

9. TECHNICAL CHARACTERISTICS:

9.1 Technical Data for Natural Shade

1. At visual inspection, the paste should be uniform, without foreign inclusions.
2. Flexural strength for Natural Shade ≥ 80 MPa.
3. The value of N, which characterizes the bending elasticity, should not be greater than the bending strength S, MPa.
4. Diametrical strength or strength with a diametral fracture of not less than 34 MPa.
5. Water absorption is not more than $40 \mu\text{g} / \text{mm}^3$.
6. Water solubility not more than $5 \mu\text{g} / \text{mm}^3$.
7. The sensitivity to ambient lighting is not more than 60 s.
8. Polymerization time is not more than 40 seconds.
9. The depth of hardening is not less than 2 mm.
10. X-ray contrast should be no more than the density of the film's blackening at the image of the aluminum disk.
11. Full deviation of color characteristics in CIELAB color coordinates, not more than 2.5.
12. Transparency coefficient, 30-60%.
13. The ability to polish: a polished sample must have a clean surface with a mirror finish.
14. Adhesive strength in connection with hard tooth tissues, not less than 7 MPa.
15. Dental material Natural Shade must be wear-resistant while impact of a toothbrush according to ISO/TR 14569-1:2007. Density of dental material: $(1,180 \pm 0,005) \text{ g/sm}^3$;

Technical Data According to ISO 4049:

Average particle size:0.7 μm

Range of particle size:0.04–2.8 μm

Percentage by volume of total inorganic filler:60-64 %

Percentage by weight of total inorganic filler:75-79 %

9.2 Technical Data for Natural Ortho

1. At visual inspection, the paste should be uniform, without foreign inclusions.
2. The bending strength for Natural Ortho is not more than 50 MPa.
3. The value of N, which characterizes the bending elasticity, should not be greater than the bending strength S, MPa.
4. Diametrical strength or strength with a diametral fracture of not less than 34 MPa.
5. Water absorption is not more than $40 \mu\text{g} / \text{mm}^3$.
6. Water solubility not more than $5 \mu\text{g} / \text{mm}^3$.
7. The sensitivity to ambient lighting is not more than 60 s.
8. Polymerization time is not more than 20 seconds.
9. The depth of hardening is not less than 2 mm.
10. X-ray contrast should be no more than the density of the film's blackening at the image of the aluminum disk.
11. Full deviation of color characteristics in CIELAB color coordinates, not more than 2.5.
12. Transparency coefficient, 30-60%.
13. The ability to polish: a polished sample must have a clean surface with a mirror finish.
14. Adhesive strength in connection with hard tooth tissues, not less than 7 MPa.
15. The film thickness should be 25-35 microns.
16. Dental material Natural Ortho must be wear-resistant while impact of a toothbrush according to ISO/TR 14569-1:2007. Density of dental material: $(1,180 \pm 0,005) \text{ g/sm}^3$;

Technical data for Natural Ortho according to ISO 4049:

Average particle size	6-7 microns.
Range of Particle size	not more than 20 microns
Percentage by volume of total inorganic	The material is comprised of 3 different inorganic fillers of different densities. Therefore, percentage by volume of total inorganic filler cannot be accurately calculated.
Percentage by weight of total inorganic filler	$\cong 75\%$

9.3 Technical data for Natural Z

1. At visual inspection, the paste should be uniform, without foreign inclusions.
2. Bending strength for Natural Z ≥ 80 MPa.
3. The value of N, which characterizes the bending elasticity, should not be greater than the bending strength S, MPa.
4. Diametrical strength or strength with a diametral fracture of not less than 34 MPa.
5. Water absorption is not more than $40 \mu\text{g} / \text{mm}^3$.
6. Water solubility not more than $5 \mu\text{g} / \text{mm}^3$.
7. The sensitivity to ambient lighting is not more than 60 s.
8. Polymerization time is not more than 40 seconds.
9. The depth of hardening is not less than 2 mm.
10. X-ray contrast should be no more than the density of the film's blackening at the image of the aluminum disk.
11. Full deviation of color characteristics in CIELAB color coordinates, not more than 2.5.

12. Transparency coefficient, 30-60%.
13. The ability to polish: a polished sample must have a clean surface with a mirror finish.
14. Adhesive strength in connection with hard tooth tissues, not less than 7 MPa.
15. Dental material Natural Z must be wear-resistant while impact of a toothbrush according to ISO/TR 14569-1:2007. Density of dental material: $(1,180 \pm 0,005) \text{ g/sm}^3$;

Technical data for Natural Z according to ISO 4049:

Average partical size	0.7 μm
Range of partical size	0.01-4,5 μm
Percentage by volume of total inorganic filler	62-63%
Percentage by weight of total inorganic filler	77-78%

10. SET OF THE PRODUCT DELIVERY:

Dental restorative universal materials Natural, in separate packages:

1. Dental material **Natural Shade**, in syringe, shade: Enamel A1- 1 pc
2. Dental material **Natural Shade**, in syringe, shade: Enamel A2- 1 pc
3. Dental material **Natural Shade**, in syringe, shade: Enamel A3- 1 pc
4. Dental material **Natural Shade**, in syringe, shade: Enamel A3.5- 1 pc
5. Dental material **Natural Shade in syringe**, shade: Enamel A4- 1 pc
6. Dental material **Natural Shade in syringe**, shade: Enamel B1- 1 pc
7. Dental material **Natural Shade in syringe**, shade: Enamel B2- 1 pc
8. Dental material **Natural Shade in syringe**, shade: Enamel B3 - 1 pc
9. Dental material **Natural Shade in syringe**, shade: Enamel C2- 1 pc
10. Dental material **Natural Shade in syringe**, shade: Enamel C3- 1 pc
11. Dental material **Natural Shade in syringe**, shade: Enamel C4 - 1 pc
12. Dental material **Natural Shade in syringe**, shade: Enamel D3- 1 pc
13. Dental material **Natural Shade in syringe**, shade: Enamel CL Bleach- 1 pc
14. Dental material **Natural Shade in syringe**, shade: Incisal - 1 pc
15. Dental material **Natural Shade in syringe**, shade: Dentin A2 - 1 pc
16. Dental material **Natural Shade in syringe**, shade: Dentin A3- 1 pc
17. Dental material **Natural Shade in syringe**, shade: Dentin A3.5- 1 pc
18. Dental material **Natural Shade in syringe**, shade: Dentin B2 - 1 pc
19. Dental material **Natural Shade in syringe**, shade: Dentin C2- 1 pc
20. Dental material **Natural Shade in syringe**, shade: Dentin Universal- 1 pc
21. Dental material **Natural Shade in syringe**, shade: Enamel A1- 2 pcs
22. Dental material **Natural Shade in syringe**, shade: Enamel A2 - 2 pcs
23. Dental material **Natural Shade in syringe**, shade: Enamel A3- 2 pcs
24. Dental material **Natural Shade in syringe**, shade: Enamel A3.5- 2 pcs
25. Dental material **Natural Z**, in syringe, shade: Enamel A1- 1 pc
26. Dental material **Natural Z**, in syringe, shade: Enamel A2- 1 pc
27. Dental material **Natural Z**, in syringe, shade: Enamel A3- 1 pc
28. Dental material **Natural Z**, in syringe, shade: Enamel A3.5- 1 pc
29. Dental material **Natural Z**, in syringe, shade: Incisal - 1 pc
30. Dental material **Natural Z**, in syringe, shade: Dentin A2- 1 pc
31. Dental material **Natural Z**, in syringe, shade: Dentin Universal- 1 pc

32. Dental material **Natural Ortho** in syringe – 1 pc

Note: Instruction for use is attached to each delivery kit.

11. PACKAGING:

Natural Shade:

- The package (Refil) contains 1 syringe, 4 g. **Natural Shade**.
- The package (Refil) contains 2 syringes, 4 g. **Natural Shade**.

Natural Z:

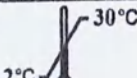
- The package (Refil) contains 1 syringe, 2 g **Natural Z**.

Natural Ortho:

- The package (Refil) contains 1 syringe containing 4 g.– **Natural Ortho**.

12. SYMBOLS USED ON A LABEL:

The marking is carried out in accordance with the labeling requirements set out in Annex I of the EU Medical Products Directive and the use of graphic symbols in marking conforms to EN980: 2008.

Symbol	Meaning
	- Batch number
	- Watch out! Refer to the accompanying documentation (instructions for use).
	- Avoid exposure to sunlight.
	- Avoid from moisture.
	- Protect from the effects of atmospheric ozone.
	- Protect from heating and radioactive sources radiation.
	- Do not use if the packaging is damaged.
	- Temperature range of storage.

	- Date of manufacture.
	- Use before (year, month).
	- Information about the manufacturer.
	- Information about the representative of the manufacturer.

Depending on the properties of the materials, additional information is provided on the label:

"Polymer dental material for fixation" / "polymer dental restorative material";

"Radiopaque";

"To restore occlusal surfaces"

13. STORAGE OF THE PRODUCT

Shelf life is 2 years from the date of manufacture.

14. STORAGE AND TRANSPORTATION CONDITIONS:

Materials packed in transport packaging are transported by all types of covered vehicles according to the rules for the carriage of goods acting on each mode of transport in accordance with the rules in force on this mode of transport.

Resistance to mechanical influences during transportation: products must be resistant to vibrational loads during transportation in the frequency range 10-55 Hz with a travel amplitude of 0.35 mm and shock loads with a peak acceleration of 10 g and a shock acceleration duration of 16 ms.

The materials are stored in the packaging of the manufacturer in the warehouses of the supplier and the consumer in a clean and dry room at temperatures from +2 ° C to + 30 ° C, away from direct sunlight and exposure to moisture, far from sources of ignition. The presence in the air of vapors of acids, alkalis and other aggressive impurities is not allowed. Keep out of the reach of children.

15. DISPOSAL CONSIDERATIONS AND DISPOSAL:

Materials are to be destroyed as medical waste in accordance with the legislation in force in the territory of the consumer's country. (Products with expired shelf life must be destroyed as Class A medical waste in accordance with SanPin 2.1.7.2790-10.).

16. MANUFACTURER:

DFL Industria e Comercio S.A., Brazil
Est do Guerengue, 2059, 22.713-001, Jacarepaguá, Rio de Janeiro, Brazil
CEP: 22713-002 - CNPJ:33.112.665/0001-46
Tel.: 55-21-2445-6766 - Fax: 55-21-3342-4009
www.dfl.com.br

17. ADDRESS OF PLACE OF MANUFACTURE OF MEDICAL DEVICE:

Est do Guerengue, 2059, 22713-002, Rio De Janeiro, Brazil

18. AUTHORIZED REPRESENTATIVE ON THE TERRITORY OF THE RUSSIAN FEDERATION:

"HIRATRADE", Ltd

Address: Gogolya, 20a, Vladimir, 600000, Russia

Tel.: (4922) 32-60-24

e-mail: mail@hiratrade.ru

19. WARRANTY LIABILITIES:

The manufacturer guarantees the conformity of the materials of the dental restorative universal Natural, in sets and in separate packages, with accessories to all the requirements of the technical file, subject to the conditions of transportation and storage specified in the regulatory document.

Claims and complaints to the quality of products in the territory of the Russian Federation are taken to the address of the authorized representative of the manufacturer in the territory of the Russian Federation.

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