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WeldOne™ Unpacking Instructions

Unpacking

Check the shipping box for visible damage upon arrival.
If you find any damage please contact your freight forwarder immediately.

Open the cardboard box.


Take out the following:

User's Manual in five languages

Unpacking instructions including list of components

A safety sign to be placed near the instrument

Repair Request form of the manufacturer
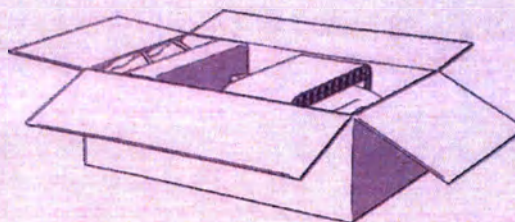


Follow instructions for use!

Scope of delivery

Take out all the accessories:

Product designation	DENTSPLY Implants Ref.
WeldOne™ welding unit	47-4710
Welding clamp	47-4711
Set of replacement electrodes	47-4712
Clamp connecting cable	47-4713
Pedal switch	47-4714
Mains cable	47-4715
Electrode replacement tool	47-4716
Clamp retainer	47-4717
Fuses	47-4718

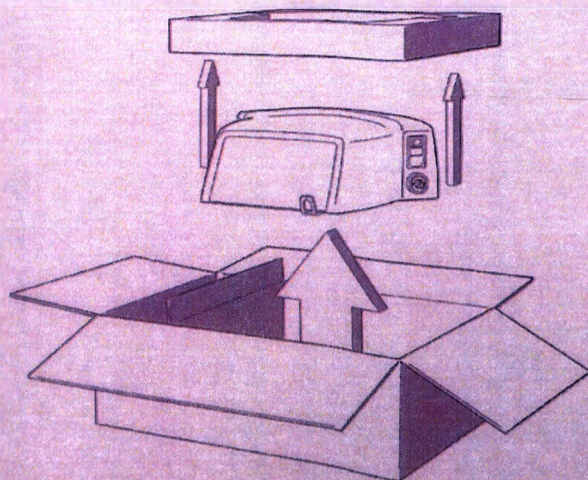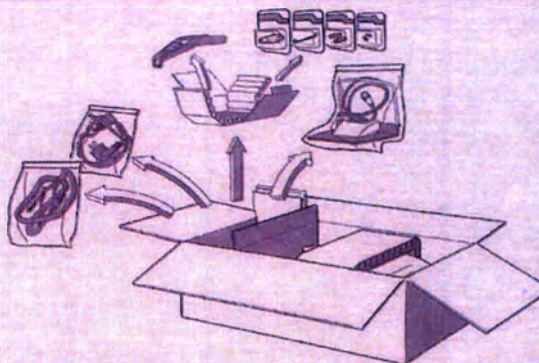Remove the padding material, which is placed on top of the welding unit.

Remove the formed foam padding that surrounds the welding unit.

PLEASE NOTE:

The weight of the welding unit is approx. 8.6 kg!

Lift out the welding unit.

Keep the original shipping box and all packaging and padding material for a future safe transport of the welding unit (e.g. repair).

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WARNING:

Not to be used on or by wearers of pacemakers!

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IMPORTANT!

Read instructions for use prior to first use!

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Please complete the form:

FOR CUSTOMER			
Name		City	
Company name		State	
Address		Zip Code	
		Phone	
		Fax	
		Email	

DEVICE INFORMATION	
Model/Name	Serial Number

DESCRIPTION OF THE PROBLEM	
Describe the problem and its frequency: Company name	Frequently ☐
	Occasionally ☐
	After prolonged use ☐
	Power on ☐
	Other ☐

WARRANTY		
Is this a warranty request?	YES ☐	NO ☐
Date of purchase	Attach the bill of sale or proof of purchase	
The request for warranty repair is subject to approval of the Studioemme after checking if the warranty terms are applicable to the fault.		

INSTRUCTIONS:

- Complete this form, sign it and send it to Studioemme: aftersales@studioemme.it or Fax +39 051 6417511
- Wait the answer to your e-mail or your fax with further instruction and, if applicable, with the RETURN CODE.
- If the repair will last more than 5 days (excluding shipping time) you can request the loaner welding unit. For additional information about the conditions and the costs please contact Studioemme.

Optional: Please complete the form:		
Do you need the loaner welding unit during the repair?	YES ☐	NO ☐
Date:	Signature:	

RESERVED FOR STUDIOEMME		
SHIP THE DEVICE YES ☐ NO ☐	Ship the device to this address:	Further instructions:
RETURN CODE		Use the following carrier:

If you get the RETURN CODE:

- Use the original packing and secure your equipment carefully in a box with suitable packing material to ensure proper protection during shipment.

Studioemme cannot be held responsible for equipment that is damaged or lost in shipping. Therefore, we strongly advise that you follow these instructions carefully.
Studioemme will not collect shipments for a fee or without the RETURN CODE.

- Include this form and any applicable warranty documentation in the box and clearly record the RETURN CODE on the delivery note.
- Use the carrier, provided above (see the table), that provides tracking information for the package.
- If the repair is out of warranty shipping expense will be charged to the customer.
- Ship your package to the location provided above (see the table).
- You will receive a confirmation receipt and, if applicable, an estimate of duration of the repair and the costs. If you do not have an email address, the confirmation will be sent via Mail.

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