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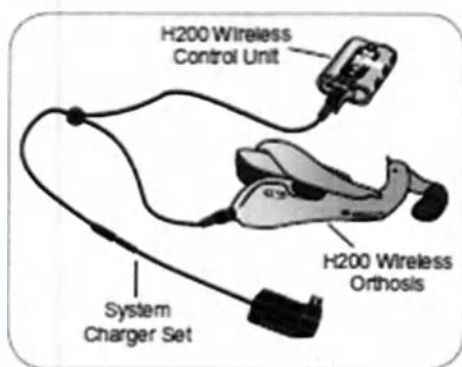
See User's
Guide

Charging the H200 Wireless System

Connect the System Charger Set to the H200 Wireless Orthosis and the H200 Wireless Control Unit simultaneously. Charge the H200 Wireless System daily.

Control Unit Charging Displays

	Flashes Yellow	Low Battery
	Rotating Green Circle	Charging
	Horizontal Green Line	Fully Charged

Orthosis Charging Displays

	Status Light Flashes Yellow	Low Battery
	Status Light Flashes Yellow and Green	Charging
	Status Light Steady Green	Fully Charged

Putting on the H200 Wireless Orthosis

- 1 Wet and attach the cloth electrodes.
- 2 Position the Orthosis on a stable surface, and put your hand into the spiral end. The Thenar (with attached electrode) should rest at the base of your thumb.
- 3 Position the wrist bridge comfortably on the back of your wrist.
- 4 Place your forearm in the Orthosis flexor support.
- 5 Close the extensor wing.
- 6 Fasten the wrist strap, if appropriate.

H200 Wireless Orthosis Displays

System On Status Light Flashes Green	Stimulation On Status Light Flashes Yellow	Stimulation Paused Status Light Steady Yellow	Faulty Electrode Contact Status Light Flashes Red	Hardware/Software Error; Charging Error Status Light Steady Red

Worldwide Corporate Office
Bioness Inc.
25100 Rye Canyon Loop
Valencia, CA 91355
Telephone: 800-211-9136 or 661-211-9136
Email: info@bioness.com
Website: www.bioness.com

Manufactured by:
Bioness Neurostimulation Ltd.
8 Hanagar St.
Hod Hasharon 4501309
Israel

European Authorized Representative
Bioness Europe B.V.
Stationweg 41
3331 LR Zevenaar, The Netherlands
Telephone: +31 78 625 6088
Website: www.bioness.com

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NESS H200 Wireless Clinician's Reference Card

Fitting the H200 Wireless Orthosis

- 1 Select the appropriate Orthosis size and then attach the appropriate Thenar.
- 2 Attach the appropriate Wrist Insert.
- 3 Don the Orthosis and check between the FPL #5 electrode and the wrist. If there is a gap, attach and tighten the Orthosis Wrist Strap.
- 4 If there is still a gap, doff the Orthosis and attach the FPL Panel.
- 5 Select and attach an Extensor Fitting Panel that will elicit the desired hand/finger movement. Start with fitting panel A.
- 6 Select and attach a Flexor Fitting Panel that will elicit the desired hand/finger movement. Start with fitting panel A.
- 7 Wet all of the cloth electrodes.
- 8 Attach the cloth electrodes to the electrode bases.
- 9 Don the Orthosis.
- 10 Adjust stimulation parameters.

Adjusting Stimulation Parameters

- 1 Log into the H200 Wireless Software. Select a patient record and press Open, or press New to create a patient record.
- 2 From the Programs Menu, press Stim. Parameters.
- 3 Adjust stimulation parameters for the extensors and flexors. Press Start to turn on stimulation. Press Stop to stop stimulation. Deactivate the trigger button on the Orthosis, if appropriate. Press Done to return to the Programs menu.

For a complete list of all contraindications, warnings, adverse reactions, precautions, and care instructions please review the Clinician's Guide found at www.bioness.com

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All Rights Reserved