Apparatus for physical therapy Lumicell Wave 6 comprising

Manufacturer: Silhouet-tone Corporation, Canada

Manufacturer's address: 2185 Michelin street, Laval, Quebec H7L 5B8, Canada

Phone: +18663409456

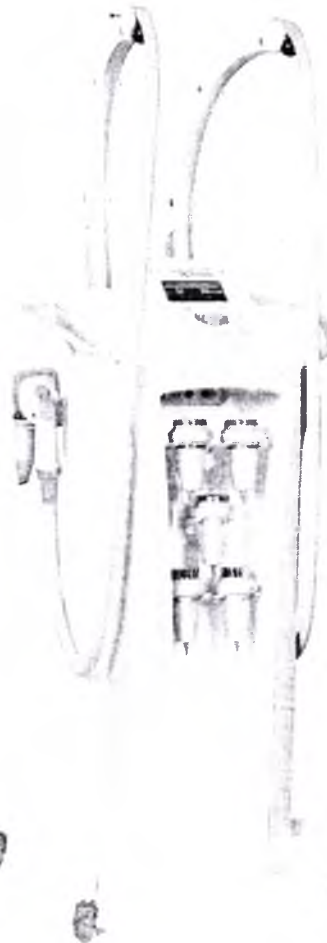
E-mail: info@silhouettone.com

Authorized representative: Skin Science, Limited Liability Company

Address: 125 Varshavskoye Highway, building 1, section 11, 1st floor, room 1, office 39, 117587, Moscow, Russia.

Phone: +7 (495)987-38-68

E-mail: info@hydrafacial.ru

User Manual Version 1

I certified the signature of Geneviève St-Jean for Silhouet-Tone.

I HAVE SIGNED: *Geneviève St-Jean*

Solemnly declared before me on the third (3) day of January 2019.

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CONTENTS

PRECAUTIONARY STATEMENTS	2
LUMICELL WAVE 6 ASSEMBLY INSTRUCTIONS	3
01. INTRODUCTION	6
02. CONTRAINDICATIONS	7
03. LUMICELL WAVE 6	10
04. TECHNOLOGIES USED IN LUMICELL WAVE 6	12
05. WHAT WE NEED TO KNOW ABOUT CELLULITE	16
06. DETAILS OF PROCEDURES	18
07. LUMICELL WAVE 6 – HAND-PIECES	19
08. MAINTENANCE OF THE APPARATUS	21
09. DESCRIPTION OF THE APPARATUS	23
10. SELECTION OF PARAMETERS	25
11. PRESET PROGRAMS	29
12. CARRYING OUT PROCEDURES	34
13. MASSAGE TECHNIQUES FOR DIFFERENT AREAS	35
14. GUIDELINES FOR PATIENTS DURING TREATMENT	39
15. APPLICATION OF THE APPARATUS AFTER LIPOSUCTION	41
16. NEEDLE-FREE MESOTHERAPY	44
17. PATIENT CHART	46
18. CLEANING FILTERS	50
19. COMPLETE SET OF LUMICELL WAVE 6	51
20. FURTHER INFORMATION AND SPECIFICATION	52

All rights reserved. No part of this copyrighted work can be reproduced or used in any form by any means – graphic, electronic, mechanical, including copying, recording, and storing information in electronic form – without permission from Silhouet-Tone Ltd.

©Silhouet-Tone Ltd 2008

PRECAUTIONARY STATEMENTS

Read the following instructions before using the apparatus.

1. Use the apparatus only for its intended purpose and only in the context of health care facilities.
2. Never use the apparatus if it has a damaged wire, does not work properly, or if it has been damaged.
3. Keep the wire away from heated surfaces.
4. The apparatus is not intended for home use and can be used only by qualified professionals.
5. The apparatus must be serviced by SILHOUET-TONE service centers or its regional representatives.
6. The apparatus must be plugged into an outlet with an earth loop. In the event of a short circuit, earthing reduces the risk of electric shock by providing electrical current drain. The apparatus is supplied with earthing wire and plug.

CAUTION: INCORRECT USE OF THE EARTHED PLUG MAY RESULT IN ELECTRIC SHOCK

If it is necessary to replace the wire or plug, do not connect the earthed wire to the port with the flat contact. The insulated green wire is the earthed wire. Consult the electrician or service representative if the earthing instructions are not completely clear or the device is not properly earthed.

7. No adapter may be used with the device. Do not alter the plug if it does not fit the socket. All installations must be made by an electrician. If the apparatus needs reconnecting, all operations shall be carried out by qualified personnel.
8. If an extension wire is needed, use a three-core wire with a triple plug and a socket that is suitable for the plug installed in the apparatus. Replace the wire if damaged.
9. Do not disassemble the apparatus. Maintenance must be performed at a service center.
10. The apparatus is intended for use in the electromagnetic environment, the specification in Tables I to 4 of the Declaration of Electromagnetic Compatibility, in the Appendix. The specialist and the operator must ensure that the system will be used in this particular environment.

Warning: to avoid fire, replace the fuses only with fuses of the same type and rating as the used ones.

SAVE THESE INSTRUCTIONS

LUMICELL WAVE 6 ASSEMBLY INSTRUCTIONS

Step 1:

Insert the connecting pipe holder into the flange with the sleeve at the top of the apparatus.

Step 2: Insert the connecting pipes for PPM to connectors.

Make sure the latch is locked.

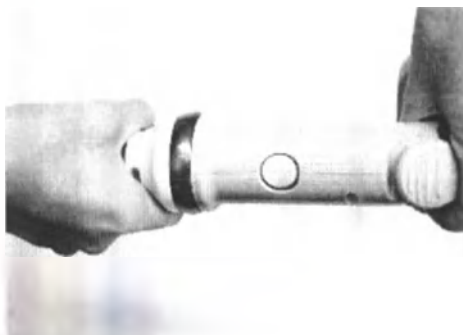
Step 3: Insert the connecting pipe for PPM into the special holder and make sure that there is enough play for rotation.

The connecting pipe must enter the holder with some pressing applied.

Step 4: Insert the opposite end of the connecting pipe into one of the hand-pieces for PPM.


Make sure the latch is locked.


Step 5: Insert the hand-piece for PPM into one of the holders, located on the sides of the apparatus.

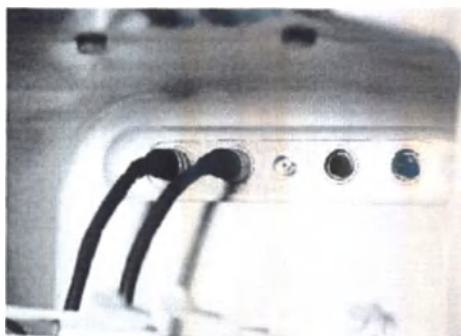
Repeat steps 1-5 to connect the second hose.

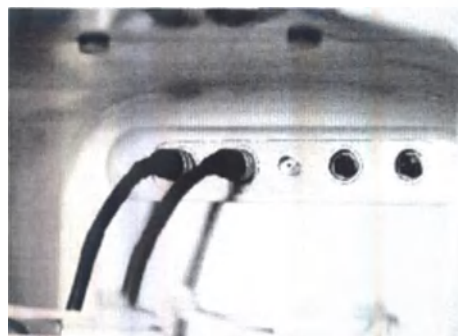
Step 6: Connect the hand-pieces for radio-frequency therapy into two ports on the left.

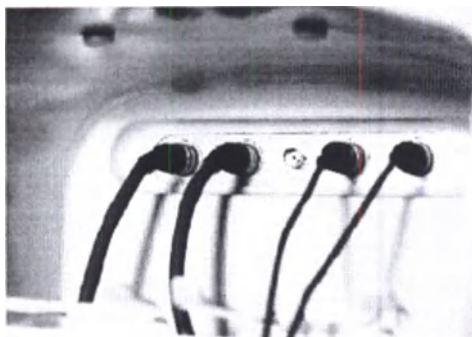
Adjust connectors and screws after the connection.

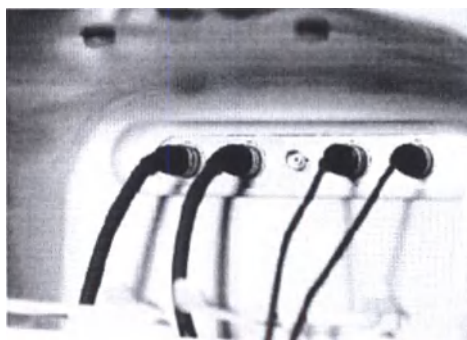Step 7: Connect the hand-pieces for needle-free mesotherapy into two ports on their right.

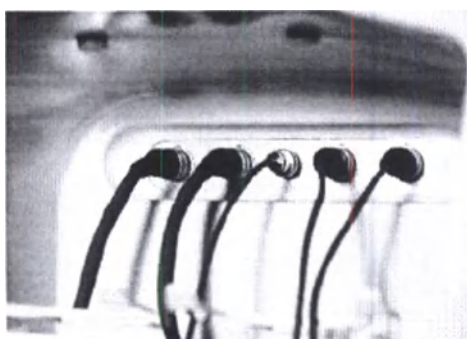Adjust connectors and screws after the connection.

Step 8: Connect the hand-piece for ultrasonic cavitation into the port in the middle.

Adjust connectors and screws after the connection.

Step 9: Place the hand-pieces into the special holder for hand-pieces.

01. INTRODUCTION

Cellulite is a common and disturbing physiological condition characteristic of post-pubertal women. Clinically, it is determined by the uneven contour of the skin or the formation of depressions on the skin, especially in the thighs and buttocks. The cause of it is the deposition of excess adipose tissue in fibrous septae: the skin irregularity is proportional to the subcutaneous fat in the papillary dermis. The fibrous septae divide the fat into individual clusters, while the vertical muscles are attached to the deep fascia, which leads to the formation of irregularities on the skin surface.

The etiology of cellulite is unknown, but it is determined by the amount of fat in the body and the nature of the fibrous septae. The amount of fat in the woman's body is more (27%) than in the man's body (15%) at a proportional body composition. This difference is due to the density of hormone receptor placement and their sensitivity in different parts of the body. In addition, insulin accelerates the lipolysis and facilitates the glucose intake into adipose cells. Histological analysis of fibrous septae in cellulite has also showed the presence of inflammation, which is the sign of the presence of local edema with a subsequent increase in the amount of adipose tissue. Moreover, a functional defect of lymph secretion is the cause of fluid buildup within fibrous septae, which slows down the blood circulation and, thus, increases the formation of fat. Changes in capillary dynamics lead to an increase in the amount of proteins and the release of fluid, which further slows down the blood circulation and leads to acid intoxication, ultimately leading to sclerosis, interstitial fibrosis, decrease in venous and lymphatic drainage. Such a chain reaction leads to the deformation of adipose cells and uneven skin color characteristic of cellulite.

Unfortunately, dieting and exercise cannot prevent the formation of cellulite.

(extract from the Journal of Cosmetic Therapy. 2005; 7: 81-85

Article by Tina S. Alster and Elizabeth L. Tanzi

Washington Institute of Dermatology, Washington, USA)

02. CONTRAINDICATIONS

Contraindications to procedures using hand-pieces:

Contraindications to photo-pneumatic massage:

- venous thrombosis
- phlebitis
- varicose veins
- cancer
- HIV (AIDS);
- pregnancy or lactation
- cardiac failure
- uncontrolled high blood pressure
- impaired skin integrity
- dermatological diseases in the acute stage
- burns
- tendinitis
- bursitis
- lymphadenitis
- gastric or duodenal ulcer
- blood-clotting disorder, taking anticoagulants
- herniae
- gynecological disorders (fibroids, ovarian disorders, cervical erosion)
- menstruation
- gynecological disorders
- hepatic failure
- renal failure

Never use the apparatus for breast massage.

Avoid areas with dilated veins.

Contraindications to vacuum therapy:

- venous thrombosis
- phlebitis
- varicose veins
- cancer
- HIV (AIDS)
- pregnancy or lactation
- cardiac failure
- uncontrolled high blood pressure
- impaired skin integrity
- dermatological diseases in the acute stage
- burns
- tendinitis

- bursitis
- lymphadenitis
- gastric or duodenal ulcer
- blood-clotting disorder, taking anticoagulants
- herniae
- menstruation
- gynecological disorders
- hepatic failure
- renal failure.

Never use the apparatus for breast massage.

Avoid areas with dilated veins.

Contraindications to procedures of needle-free mesotherapy:

- presence of cardiac pacemaker
- presence of metallic prostheses and implants
- individual electrical sensitivity
- uncontrolled high blood pressure
- atrial fibrillation
- pregnancy or lactation
- thrombophlebitis
- epilepsy
- impaired skin integrity
- dermatological diseases in the acute stage
- cancer.

Contraindications to procedures using ultrasonic cavitation:

- hepatitis and hepatic failure
- renal failure
- cholelithiasis
- nephrolithiasis
- thrombophlebitis
- osteoporosis
- presence of cardiac pacemaker
- presence of metallic prostheses and implants
- uncontrolled high blood pressure
- blood-clotting disorder and taking anticoagulants
- autoimmune diseases and diabetes mellitus type I
- atrial fibrillation
- pregnancy or lactation
- gynecological disorders (fibroids, ovarian disorders, cervical erosion)
- menstruation
- impaired skin integrity
- dermatological diseases in the acute stage
- cancer.

Contraindications to procedures using radio-frequency therapy:

- hepatic failure
- renal failure
- thrombophlebitis
- presence of cardiac pacemaker
- presence of metallic prostheses and implants
- uncontrolled high blood pressure
- blood-clotting disorder and taking anticoagulants
- autoimmune diseases and diabetes mellitus type I
- atrial fibrillation
- pregnancy or lactation
- gynecological disorders (fibroids, ovarian disorders, cervical erosion)
- menstruation
- impaired skin integrity
- dermatological diseases in the acute stage
- cancer.

LUMICELL WAVE 6

After studying the results of medical research on cellulite, we concluded that cellulite is a multifaceted phenomenon that requires different technological approaches to implement a successful treatment and achieve an optimal result. The latest technological advances and scientific research enable us to combine carefully selected mechanical, thermal and optical methods in order to achieve the most efficient, fast and sustainable result.

Cellulite is inherently associated with deep changes. It may be deeper or more superficial. Lumicell Wave 6 is able to cope with different stages of cellulite by heating the skin at different levels of depth.

During the first stage of treatment, a pulsating vacuum massage separates the fibrous septae. While suction increases the blood circulation, reduces swelling in the intrapore space, the vacuum nozzle heats and softens the subcutaneous fat layer with infrared radiation, making it more pliable. The thermal treatment with a 900 nm infrared ray is followed by ultrasound, which increases the temperature in the deeper layers of skin. Ultrasound with a frequency of 1 MHz accelerates lipolysis, thus reducing the volume of adipose cells. At the end of treatment, the use of bipolar radio frequency increases the formation of collagen and restores the elasticity of the tissue, stopping the further formation of cellulite. All three methods should be used consistently for optimal results.

Lumicell Wave 6 is an exceptionally efficient, non-traumatic system specially designed for cellulite treatment and body contouring. Used during treatment:

- infrared photo-pneumatic massage (wavelength of the infrared radiation is 900 nm);
- ultrasound with a frequency of 1 MHz;
- bipolar radio-frequency therapy with a frequency of 450 kHz.

Combining technological approaches in Lumicell Wave 6 allows you to carry out a safe and effective treatment that keeps a stable result. The photo-pneumatic massage improves the blood circulation and weakens the connective tissue around fat deposits, which is the cause of uneven skin contour. The **infrared** radiation heats the skin from the inside, making it more pliable and susceptible to the rotation of the attachment, straightening fibrous septae without damaging them. The **ultrasound** following all that distributes heat in the layers of fat, weakening the stability of the membranes of adipose cells, which leads to a decrease in fat deposits. It also enhances the blood circulation and lymph flow, eventually reducing the circumference and volume of localized fat deposits.

In the final stage of treatment, the **radio-frequency** therapy leads to the formation of collagen and improves the structure of the skin by selective heating of the skin proteins, creating a controlled inflammatory response and initiating the formation of collagen. Lumicell Wave 6 offers a wide range of treatment options suitable for various body shapes and stages of cellulite, and also allows creating an individual approach to each patient.

Lumicell Wave 6 also offers **needle-free mesotherapy**. The needle-free mesotherapy is used for transdermal effects, providing natural destruction of fat deposits.

Lumicell Wave 6 is ideal for its intended use in health care facilities.

Lumicell Wave 6 is a multimodal cosmetological system for the treatment of skin defects and rejuvenation

Lumicell Wave 6 is designed to provide an exceptionally effective treatment with the help of methods aimed not only at improving the beauty, but also at the general condition and complete relaxation of the body. The intensive treatment eliminates muscle and venous tension and unleashes vitality, improving the blood circulation, lymph drainage and cell nourishment.

04 TECHNOLOGIES USED IN LUMICELL WAVE 6

Infrared radiation

The electromagnetic waves between the visible light and microwaves are called infrared waves. The length of infrared waves is 700-3000 nm. Further, the infrared waves are divided into short, medium and long ones:

- IR-A: 700-1400 nm (short)
- IR-B: 1400-3000 nm (medium length)
- IR-C: 3000 nm-1 mm (long)

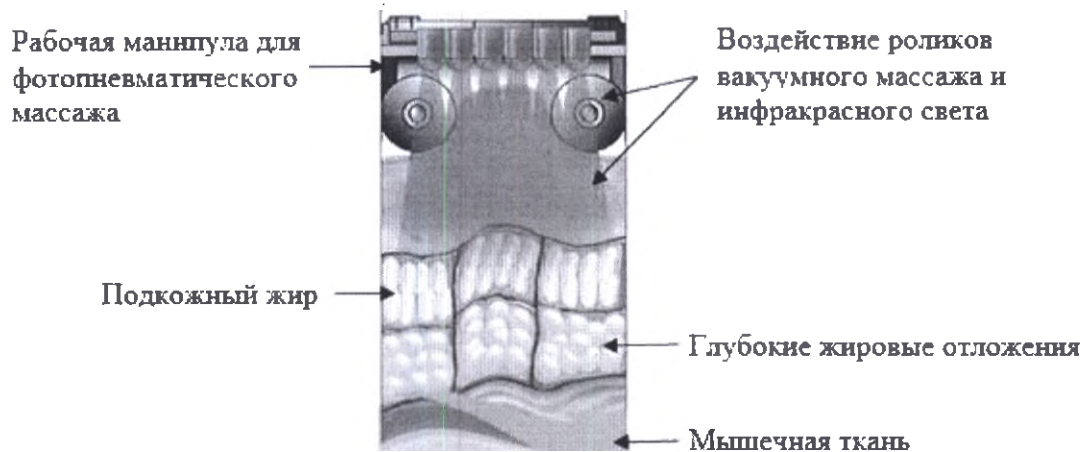
In the spectrum of the electromagnetic radiation, the infrared rays are longer than red ones (red rays are the second in length). This type of radiation is invisible to the naked eye, but distinguishable with the help of special cameras that transform the infrared radiation into the visible light. The infrared radiation is a form of energy that heats objects by converting them, without heating the air between the source of radiation and the object. The infrared radiation is also called infrared energy (IR).

The infrared radiation is able to penetrate deep into tissues – up to 30-40 mm – and transfers heat to the subcutaneous layers.

The infrared radiation and its therapeutic effect has been studied for over 20 years and is now incredibly popular. The infrared therapy improves overall physical condition due to recognized general positive effect. Studies have shown more than 100 benefits of the infrared therapy, such as improving the blood circulation, weight control, detoxifying the body and reducing cellulite.

The effect of the infrared photo-pneumatic massage

The hand-piece for the photo-pneumatic massage Lumicell Wave 6, emitting infrared light, allows for a deep contour massage of tissues throughout the body.


Effect:

- exfoliating fibrous septae
- promoting lipolysis
- reducing swelling in the intrapore space
- improving blood circulation and lymph drainage
- reducing the "orange peel" effect

1MHz Ultrasound

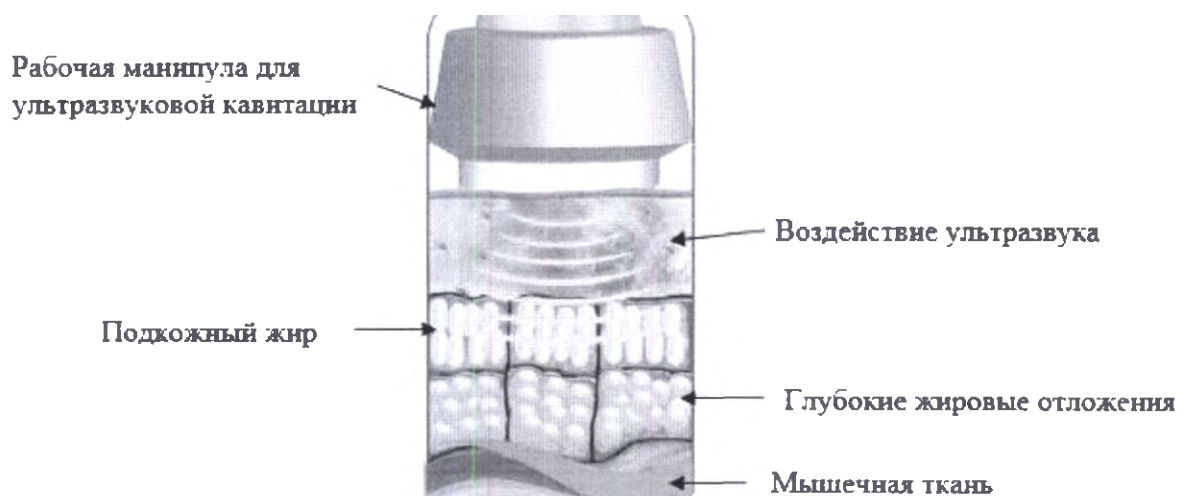Ultrasound is a mechanical vibration at a frequency higher than the audible one. In clinical conditions, ultrasound with a frequency of 1-3 MHz is usually used with a power of 20 to 100%. At less than 100%, continuous pulsing ultrasound is usually used, while at 100% power – long continuous ultrasound is used.


Effect of ultrasound on adipose fat

Dr. Asuko Sasaki from the Japanese Department of Preventive Medicine (Jikei University, School of Medicine), etc. Tetsuro Shuu (President, Tokyo Obesity Research Institute) have demonstrated the effect of 1 MHz ultrasound on the lipolysis. It is known that catecholamines such as epinephrine and norepinephrine stimulate the lipolysis, reducing the accumulation of triglycerides in adipose tissue. "Procedures using the ultrasound stimulate the lipolysis in adipose tissue by increasing the release of norepinephrine over the area of exposure; therefore, the exposed part of the body can lose weight. Based on these results, we investigated the effect obtained with a combination of ultrasound exposure to subcutaneous thigh fat and physical exercise. Men were divided into three groups: (group A) 10 min exposure at 1 MHz and 10 min walk, (group B) 10 min exposure at 500 kHz and 10 min walk, and (group C) 10 min walk without exposure. The ultrasound procedures and walks took place every day for 10 days. The procedures were performed on the inner thigh of the right leg, leaving the left leg as control. We found that the amount of subcutaneous fat on the right inner thigh was significantly reduced compared with the left leg, which was not subjected to the procedures. The body weight before and after the experiment has not changed."


Effect of bipolar radio waves on the collagen production

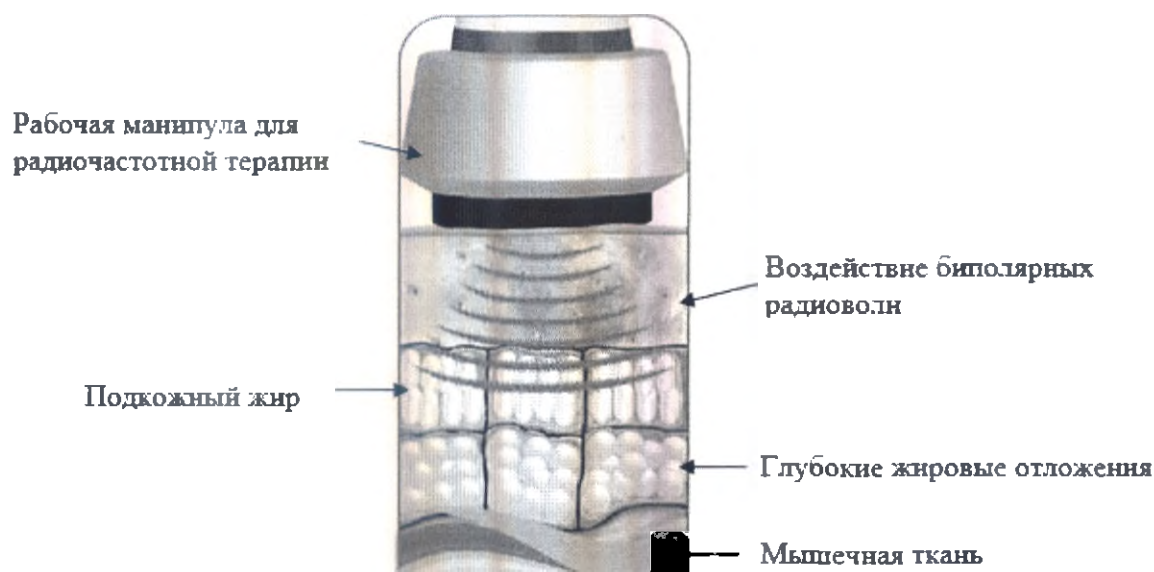Bipolar radio waves are an amazing and new non-invasive technology that has two main advantages – tightening skin and reducing body fat deposits. Radio waves are electromagnetic energy with a wavelength slightly higher than that of microwaves. They heat the skin and subcutaneous tissue without harming the skin. Heat stimulates the production of collagen, making the skin more toned and smooth. In addition, the bipolar radio waves improve the blood circulation and lymph drainage, and, in combination with the ultrasound, thermally accelerates the lipolysis in adipose cells.

Opportunities with needle-free mesotherapy.

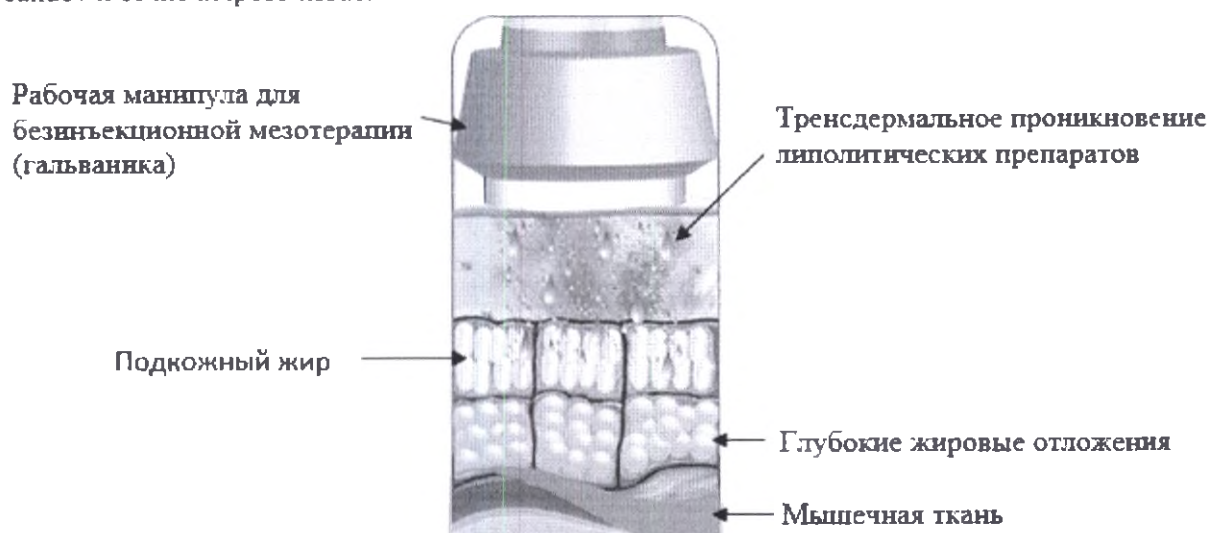The needle-free mesotherapy is an alternative way to deliver active substances to the deeper skin layers. Non-invasive, painless transdermal administration (6 cm penetration depth) of lipolytic products enriched with fat-burning components. Active components significantly accelerate the process of natural breakdown of the adipose tissue.

05 WHAT WE NEED TO KNOW ABOUT CELLULITE

It is believed that about 95% of women have a certain grade of cellulite. Hereditary background, lack of physical activity, unbalanced nutrition, stress, lifestyle –all these factors contribute to the development of cellulite. Skin care professionals know that in order to find a new way to treat cellulite, it is necessary first to understand the dynamics of its formation.

Cellulite – or lipoatrophy – is defined as an imbalance between lipogenesis (formation and deposition of fat) and lipolysis (fat burning). This imbalance leads to hypertrophy of adipose cells, which blocks the access of oxygen to the adipose tissue of the hypodermis, triggering a physiological chain reaction, leading to the characteristic signs of cellulite: the formation of depressions on skin. Other consequences are moisture retention in the tissues, blockage of the lymphatic vessels, increased venous pressure, blood circulation disorder, excessive lymph viscosity, which makes adipose cells absorbing fat and modeling the skin. This is how cellulite appears.

Women are more susceptible to the formation of cellulite than men, as they have an unequal distribution of adipose cells and a high content of adipose cells in the upper skin layers, especially in the thighs and buttocks. The structure of men's skin is very different from women's: adipose cells are equally distributed and cellulite does not occur * 1. The difference in structures is triggered by a high level of estrogen in women * 2.

*1 Pierard, Am. J. Dermo-pathology, 2000, vol. 22 (1): 34-37)

*2 (Gruber, Gyn. Endocrinology., 2002)

The main causes of cellulite:

1) Hormonal changes – the level of estrogen varies considerably under the influence of various conditions:

- puberty
- pregnancy
- menopause

2) Hereditary background.

3) Ineffective diets that result in metabolic disorders, thus causing overweight instead of reducing it.

4) Overweight.

Human body and hormonal systems

Hormones are produced by the glands. They enter the bloodstream, from where they go directly into the tissue. The human body is constituted in such a way that allows the hormone-producing glands to perform their important function. Their main purpose is to provide the balance of the body, regardless of external influence factors.

In the case of cellulite, we can divide the hormonal activity into two groups:

- fat loss activity
- fat spread activity.

Alpha- and beta-receptors

The adipose cells contain two different types of hormone receptors – alpha and beta. Alpha-receptors allow free fatty acids from the blood to attach to adipose cells as fats (lipogenesis). Beta-receptors use the fat from adipose cells as energy for muscles or thermoregulation (lipolysis).

Infrared radiation, ultrasound and radio waves stimulate the beta-receptor fat-burning activity, increasing the temperature in the surrounding tissues and simulating a "fever" reaction. Beta-receptors trigger the process of lipolysis and use fat to regulate body temperature. Physical exercises also leads to the release of fat, allowing the use of fat as muscle energy.

Grades of Cellulite (Melvin L. Elson, Doctor Nashville)

Dr. Elson identifies five grades of cellulite:

- **Grade I:** the skin looks normal, but the "orange peel" effect appears when the skin is compressed;
- **Grade II:** the "orange peel" is noticeable without compression;
- **Grade III:** horizontal indentations and ripples without looseness;
- **Grade IV:** skin looseness in addition to indentations and ripples;
- **Grade V:** skin folds in addition to the symptoms of the previous grade: the deformation becomes clearly visible, with obvious vascular problems. This is the latest grade of cellulite.

06. DETAILS OF PROCEDURES

Before starting the procedures:

- It is important to understand that Lumicell Wave 6 treatment is not a cure for any kind of disease.
- In case of a disease, the procedures must be approved by the attending physician.
- All patients must complete and sign an agreement before starting treatment.
- If the patient is taking medication, it is important to ensure that there are no side effects or contraindications. This information can be obtained from the attending physician or pharmacist.
- The procedures can be performed on healthy skin only. If skin lesions or dermatitis are visible, such areas must not be subject to the procedure.
- The treatment of the lower back in women during menstruation may increase the amount of menstrual flow: this side effect is completely innocuous. If it still disturbs the patient, it is possible to reschedule the session.

Precautions

- Use the apparatus only for its intended purpose and only in the context of health care facilities.
- Never use the apparatus if it has a damaged wire, does not work properly, or if it has been damaged.
- Keep the wire away from heated surfaces.
- The apparatus is not intended for home use and can be used only by qualified professionals.
- The apparatus must be serviced by SILHOUET-TONE service centers or its regional representatives.
- The apparatus must be plugged into an outlet with an earth loop. In the event of a short circuit, earthing reduces the risk of electric shock by providing electrical current drain. The apparatus is supplied with earthing wire and plug.

CAUTION: INCORRECT USE OF THE EARTHED PLUG MAY RESULT IN ELECTRIC SHOCK

If it is necessary to replace the wire or plug, do not connect the earthed wire to the port with the flat contact. The insulated green wire is the earthed wire. Consult the electrician or service representative if the earthing instructions are not completely clear or the device is not properly earthed.

- No adapter may be used with the device. Do not alter the plug if it does not fit the socket. All installations must be made by an electrician. If the apparatus needs reconnecting, all operations shall be carried out by qualified personnel.
- If an extension wire is needed, use a three-core wire with a triple plug and a socket that is suitable for the plug installed in the apparatus. Replace the wire if damaged.
- Do not disassemble the apparatus. Maintenance must be performed at a service center.

Warning: to avoid fire, replace the fuses only with fuses of the same type and rating as the used ones.

Precautions when using the apparatus after liposuction

- Lumicell Wave 6 must not be used until the edema subsides and the compression bandage is removed, which usually takes 6 to 8 weeks.
- It is possible to begin treatment earlier (3-4 weeks after surgery), but only with the permission of the surgeon.
- To achieve the best result, the whole body surface should be treated.
- Avoid the areas with cuts. Go around them.
- It is strongly recommended to receive a certificate from the doctor and attach it to the chart of the patient who has been or will be undergoing the liposuction procedure.

Treatment options:

- Lumicell Wave 6 offers a wide range of treatments.
- Procedures can be carried out individually or in combination with a wrap, thermal baths or balneotherapy.
- The results of cellulite treatment with Lumicell Wave 6 are the result of continuously repeated procedures.

- The regularity of procedures leads to the best results: the treatment should take place twice a week with a three-day interval.
- The course of cellulite treatment usually takes from 8 to 12 sessions, depending on the age of the patient and the severity of the problem (cellulite grade).
- Each session takes from 35 to 60 minutes.
- A refresher course of treatment after a month break supports the result.

Preparation for the procedure:

- Cover the couch with a disposable towel or sheet.
- Make sure that the couch is well covered, as the patient will roll over during the procedure.
- Room temperature should be comfortable.
- Lighting should be soft for better relaxation.
- **Peeling should be done before the first procedure** to remove dead cells and provide better absorption of the infrared radiation.

07. LUMICELL WAVE 6 – HAND-PIECES

The complete set Lumicell Wave 6 uncludes:

Two hand-pieces for photo-pneumatic massage, which are specially designed for body treatments, while facial massage is carried out using **the hand-piece for vacuum therapy**. Their ergonomic design is suitable for both left and right hands, simulating the continuation of the hands of a massage therapist. It is important for a massage therapist to use both hands during a photo-pneumatic massage procedure: one hand on the massager, the other on the client's skin to control the effect of the procedure on the skin and muscle tissue. This constant contact will also provide the client with more complete relaxation.

In addition to facial massage, the hand-piece for vacuum therapy can be used to massage small areas of the body, such as hands, nape, or calves.

The hand-pieces are color-coded to make it easier to identify their purposes.

Hand-piece for ultrasonic cavitation (yellow marking) is used to reproduce sound waves that affect adipose tissue and stimulate the release and breakdown of fat.

Two hand-piece for radio-frequency therapy (purple marking) are used to heat the skin, form collagen and tighten the skin at a time.

Two hand-pieces for needle-free mesotherapy (pink marking). Using these hand-pieces allows the subcutaneous administration of products that stimulate lipolysis.

Two hand-pieces for photo-pneumatic massage:

- roller lengths are 50 and 38 mm;
- they perform vacuum infrared photo-pneumatic massage

Hand-piece for vacuum therapy

Hand-piece for ultrasonic cavitation

Two hand-pieces for radio-frequency therapy

Two hand-pieces for needle-free mesotherapy

ATTENTION: REGULAR MAINTENANCE OF THE ATTACHMENTS AND ELECTRODES OF THE APPARATUS ENSURES IT CORRECT AND LONG-TERM FUNCTIONALITY

08. MAINTENANCE OF THE APPARATUS

Regular maintenance of the Lumicell Wave 6 will ensure its correct and long-lasting functionality.

Maintenance of small and large attachments to hand-pieces for photo-pneumatic massage and small and large attachments to hand-pieces for vacuum therapy

- Treat the rollers and the outer surface with antibacterial soap or a weak disinfecting solution (for example, chlorhexidine), wash and place into a UV sterilizer for 20 minutes.
- This simple hygiene procedure will prevent the transmission of skin diseases, such as mycosis, that cannot be diagnosed visually.

Please note: only attachments to hand-pieces should be treated in this way, but not electrical connections.

Hand-pieces for photo-pneumatic massage

- The hand-pieces for photo-pneumatic massage should be treated after each use, as the residual oils can permanently damage the contacts.
- Disinfect hand-pieces for PPM with 70% alcohol wipes or a weak disinfectant solution (for example, chlorhexidine) using a soft cloth.
- In order to avoid short-circuiting or damage to the LEDs, the hand-pieces must be completely dry before connecting after treatment.

Hand-piece for ultrasonic cavitation, radio-frequency therapy and needle-free mesotherapy:

They must be cleaned after each procedure, as the remaining gel can damage the components of the hand-pieces. Remove the remaining gel with a paper towel and use 70% alcohol wipes or a weak disinfecting solution (for example, chlorhexidine).

ATTENTION: NEVER IMMERSE HAND-PIECES INTO LIQUID

Surface of Lumaticell Wave 6

- From time to time, clean the surface of the apparatus with a damp cloth using a small amount of soap solution.
- Never spray any product on any part of the apparatus.
- Do not pour liquids onto the apparatus.
- Do not use harsh cleaners or solvents.

Oil suction filters.

On the rear part of the apparatus, there are two control filters collecting oil particles that accidentally get into the apparatus during the vacuum massage. They prevent massage oil from entering the Lumaticell Wave 6 pump. Daily check of filters is recommended.

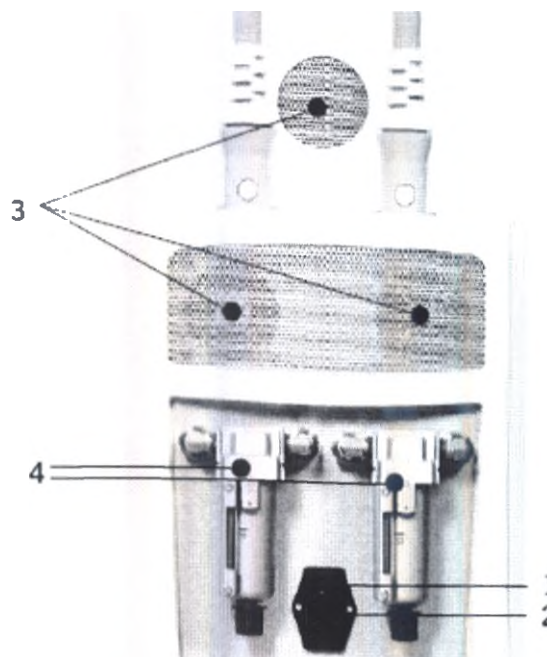
ATTENTION: CHECK FILTERS PERIODICALLY AND CLEAN IN CASE OF FILLING UP TO THE MARK.

Non-observance of these rules can lead to failure of the entire filtering system. Using thick massage oils reduces the expected lifetime of the filters. Using a small amount of oil extends the lifetime of filters.

DESCRIPTION OF THE APPARATUS

Rear part

- 1) Apparatus on/off toggle
- 2) Power cable connection port
- 3) Air vents
- 4) Oil suction filters


Control panel description

- 1) PPM on/off button 1
- 2) Start/Stop
- 3) Navigation

- 4) Radio-frequency cavitation on/off
- 5) PPM on/off button 2;
- 6) Menu

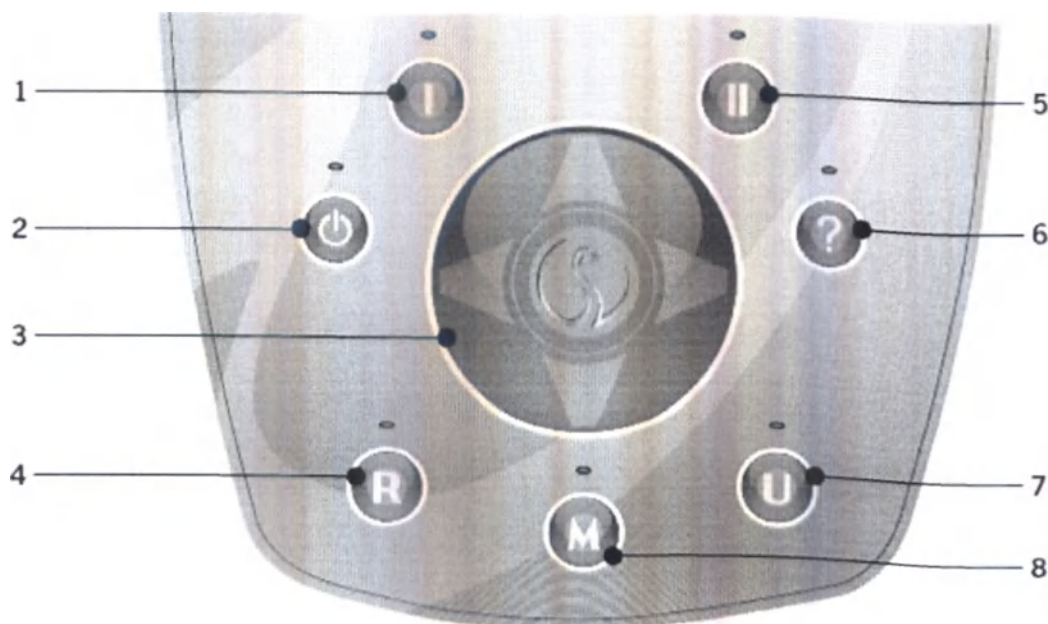
- 7) Ultrasonic cavitation on/off;
- 8) Needle-free mesotherapy on/off.

Navigation

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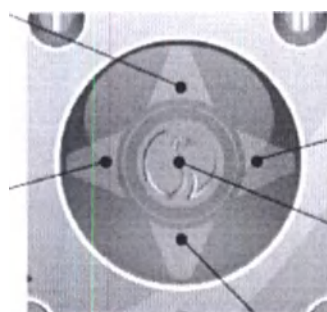Up/Down buttons

To scroll and increase/decrease the intensity

Left/Right buttons

To move screens and activate windows

Section 1: procedures

Section 2: vacuum adjustment

Section 3: intensity settings

Enter button

To confirm the selection and entry of data

0 SELECTION OF PARAMETERS

Start

1. **Turn on** the apparatus using the toggle switch located on the rear panel of the apparatus.
2. Silhouet-Tone logo will be displayed.
3. Wait a few seconds until the apparatus warms up.
4. Anti-cellulite treatment options are preset by default: Cellulite – Grade I – Intensity 3 – Time 30 minutes.
5. Use the **START/STOP** button to start or end the procedure.
6. If you want to use other settings, click the "?" button
7. Use the navigation to select a program and press **ENTER** to confirm your selection.

Program settings:

Pressing the "?" button gives access to three programs:

CELLUTE – CONTOURING – DRAINAGE

Double-pressing the "?" button gives access to three other programs:

LANGUAGE – SOUND – WARRANTY

Language options:

English/French
/Spanish/German

To change the language:

- * Use the "?" button
- * Select the **LANGUAGE** option using the navigation arrows.
- * Press the **ENTER** button.
- * Select language (**ENGLISH**) using the arrows
- * Confirm the selection by pressing the central button (**ENTER**). The selected language will be memorized
- * To return to the main menu, press the "?" button

Sound options:

- Use the "?" button
- Select **SOUND** using the arrows.
- Confirm your selection buy pressing the **ENTER** button
- Select the sound level with the **UP/DOWN** arrows
- Confirm your choice with the **ENTER** button. The selected settings will be memorized
- To return to the main menu, press the "?" button

Waranty

Lumicell Wave 6 warranty period is 1 year.

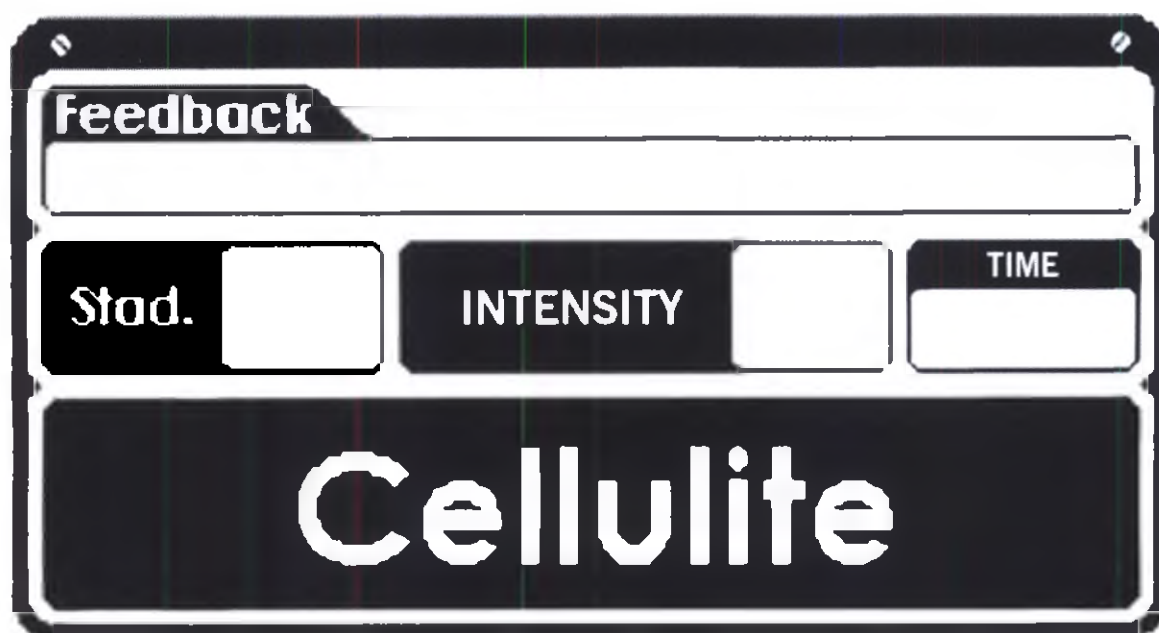
When the apparatus is turned on, service hours are automatically recorded in the memory of the apparatus.

ATTENTION: Service hour meter runs when the apparatus is on. IF NOT USED, TURN OFF THE APPARATUS USING THE ON/OFF BUTTON ON THE REAR PART OF THE APPARATUS.

To check the number of service hours:

- * Use the "?" button
- * Select the **WARRANTY** option using the navigation arrows
- * Press the **ENTER** button.
- * Total number of service hours will be displayed
- * To return to the main menu, press the "?" button

OPERATION SCREEN

Programs

Settings are displayed 3 seconds after the apparatus is turned on. Anti-cellulite treatment options are preset by default: **Cellulite – Grade I – Intensity 3 – Time 30 minutes.**

These settings are displayed every time in the procedure mode; all settings of the procedures can be changed depending on the needs of the patient.

Selection of parameters | 10

The operation screen has 3 sections

SECTION 1

Displays selected programs: **Cellulite – Body Contouring – Drainage**

SECTION 2

This section displays the sub-sections of the programs:

- Cellulite program: **Grade 1, Grade 2, Grade 3, Grade 4**. The screen will display the possibility to select the intensity for the specified grade of cellulite.
- Body contouring program: **Sculpt 1, Sculpt 2, Sculpt 3**. The possibility to select the proposed options
- Drainage program: **Intensity**. A single vacuum massage rhythm is available to stimulate the lymph drainage.

SECTION 3

Displays the time of the procedure.

Anti-cellulite program

- Turn on the apparatus using the toggle switch located on the rear panel.
- Anti-cellulite treatment options are preset by default: Cellulite – Grade 1 – Intensity 3 – Time 30 minutes.
- To change settings, use the navigation buttons.
- Press the **START/STOP** button to start the procedure.
- When you hear that the apparatus is on, you can start the procedure.
- To change the intensity settings, use the navigation arrows, select the intensity and confirm the selection using the **ENTER** button. The light will turn on in the intensity window.
- Use the **UP/DOWN** arrows to increase or decrease the intensity.

Body contouring program

- To select a program, press the "?" button
- To confirm the selected program, use the **ENTER** button; the program will appear on your screen.
- Press the **START/STOP** button to start the procedure

Drainage program

- To select a program, press the "?" button
- To confirm the selected program, use the **ENTER** button; the program will appear on your screen.
- Press the **START/STOP** button to start the procedure.

Selection of parameters | 10

Ultrasonic cavitation function

- To access the program settings, use the "U" button
- 25% intensity is automatically set to 10 minutes. The intensity can be increased according to the amount of body fat (50, 75 or 100%). The time of the procedure can be increased or decreased, depending on the exposed area. Do not exceed 5 minutes per area.
- Press the **START/STOP** button to start the procedure
- To avoid overheating and damage to the ultrasonic hand-pieces, turn off the apparatus immediately after completion of the procedure.

Radio-frequency therapy function

- To access the program settings, use the “**R**” button
- 20% intensity is automatically set to 10 minutes. The intensity can be set at 10-100% and can be increased in flabby areas that require compaction. The time of the procedure can be increased or decreased, depending on the exposed area. Always start with the lowest settings and do not exceed the exposure time for more than 5 minutes per area.
- Press the **START/STOP** button to start the procedure.
- To avoid overheating and damage to the hand-pieces for radio-frequency therapy, turn off the apparatus immediately after the procedure.

needle-free mesotherapy function

- To access the program settings, use the “**M**” button
- The intensity of 1000 microamps is automatically set to 15 minutes.
- The intensity can be increased to treat areas with thicker skin or reduced in sensitive areas.
- Do not exceed 15-minute duration of the procedure on each leg.
- Press the **START/STOP** button to start the procedure.

PRESET PROGRAMS

For easy settings, preset programs can be selected. In any case, these installed parameters can be considered as recommended. They can be adjusted depending on the patient's needs.

Selection of 3 programs:

Anti-cellulite program

It is based on the patient's cellulite grade and the complexity of the associated problems.

Body contouring program

This program focuses on persistent body fat in "non-cellulite" areas such as waist, sides, and abdomen. It is aimed at stimulating the reduction of adipose cells (lipolysis).

Drainage program

To treat patients with circulatory problems, varicose veins, edema and postoperative patients.

Anti-cellulite program selection

Lumicell Wave 6 offers a choice of 4 preset programs aimed at treatment of various cellulite grades.

Suction adjustment is set according to two criteria:

- Cellulite grade
- Patient sensitivity

Grade 1: suitable for the elimination of "orange peel" in case of painful cellulite, vein and blood circulation disorder in the legs. These settings should be used initially to determine the sensitivity of the client and to prepare tissues for more intense exposure.

Grade 2: to eliminate fibrotic folds that cause the formation of cavities and indentations on the skin.

Grade 3: for deep cellulite, fixed on the subcutaneous layers, in addition to cavities and indentations.

Grade 4: for the last grade of cellulite characterized by the symptoms mentioned above, as well as skin looseness with wave-like folds.

Selection of programs that sculpt the contour of the body

Lumicell Wave 6 offers 3 preset vacuum massage programs focused on various levels of localization of the body fat.

Suction adjustment is set according to two criteria:

- Body fat level
- Patient sensitivity

Sculpt 1: suitable for body fat in the abdomen. This mode should be used during the first procedure to determine the sensitivity of the patient and to prepare tissues for a stronger treatment.

Sculpt 2: to eliminate body fat in the sensitive thigh areas.

Sculpt 3: for more persistent body fat in insensitive areas.

Selection of the vacuum massage intensity

Lumicell Wave 6 has 10 levels of intensity settings for each program, allowing you to personalize the settings. It is important to consider the needs of the patient. Procedures must be comfortable and painless.

Timer

- There is a countdown in real time.
- To set up the preset procedure time:
- Use the **UP/DOWN** arrows to select the time window.
Confirm your choice by pressing the **ENTER** button.
Use the **UP/DOWN** arrows to set the desired procedure time.
- The UP button increases the procedure time by 5 minutes.
The DOWN button decreases the procedure time by 1 minute.

The infrared photo-pneumatic massage is used for:

- tissue heating and preparation for more intensive procedures;
- desensitization;
- reducing pain in case of stagnant cellulite;
- reducing adipose tissue;
- intense deep tissue massage;
- muscle relaxation and stress reduction during spa treatments.

Techniques of infrared photo-pneumatic massage

There are five basic techniques for anti-cellulite and contouring procedures. If you are a massage therapist, you can also use your own ones.

Longitudinal: improves blood circulation, promotes lymph drainage.

Semicircular: gentle movements for the abdominal area.

Transversal: for separation of the fibrous septae causing the formation of indentations.

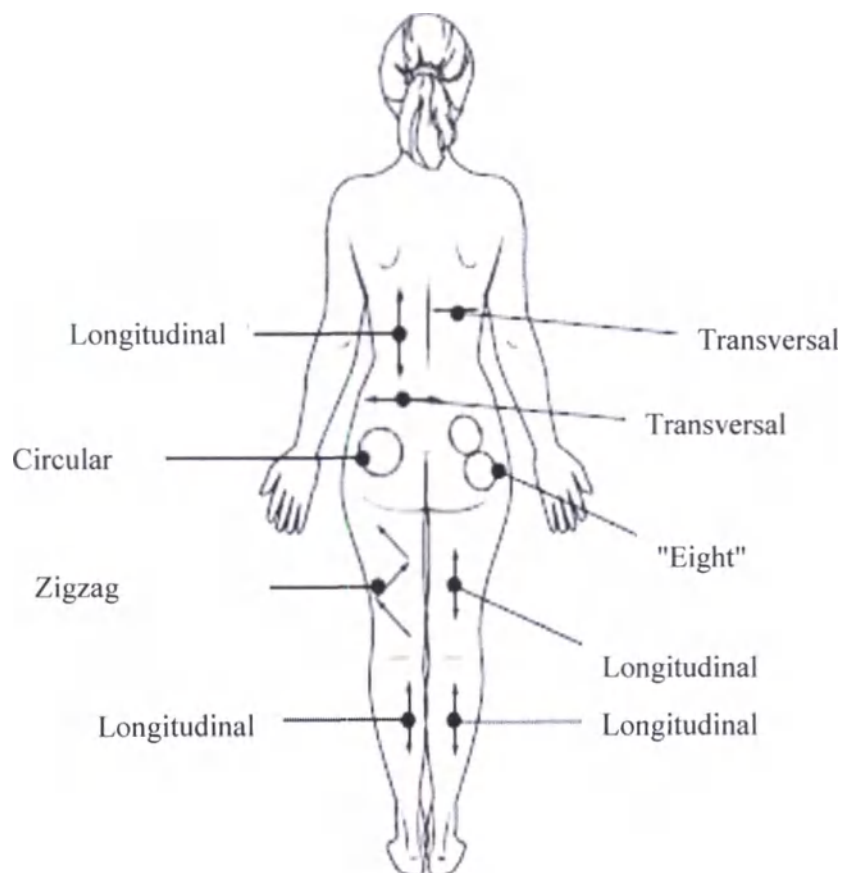
Zigzag: softens cellulite tissue and eliminates fibrotic folds.

"Eight": intensive treatment of large areas of body fat, such as the buttocks.

Five basic techniques can be applied to any part of the body


Always take into account the area of the body that is affected and the sensitivity of the patient. The procedure must be comfortable and painless.

Overall body treatment techniques

- These techniques can be used individually or in various combinations on different parts of the body.
- The appropriate attachment can be selected depending on the desired procedure and the shape of the client's body.

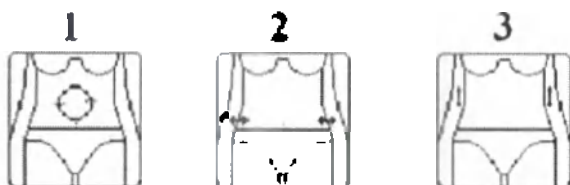Abdomen

Legs

Buttocks

Back

Arms

CARRYING OUT PROCEDURES

The course of treatment:

8 to 10 or more sessions (depending on the severity of the problem) twice a week required.

Time: 35 – 60 minutes (depending on the treated area)


- Before the procedure, peeling of the whole body is recommended.
- Use oil for photo-pneumatic massage.
- Use the hand-piece and attachment for photo-pneumatic massage, (30 minutes or less when working on a small area).
- Use a warm, damp towel to remove any excess oil before the ultrasound cavitation procedure.
- Use gel for ultrasound procedures.
- Move the hand-piece for ultrasonic cavitation with slow circular and linear movements, focusing on the areas of cellulite and fat deposits (no longer than 5 minutes per area). Cleaning the surface of the gel after the procedure is not necessary.
- Use gel for RF-procedures and hand-pieces for radio-frequency procedure for slow massage of body parts (no longer than 5 minutes per area). To avoid pain, the hand-pieces must be tightly positioned to the skin surface. Never position the attachments anglewise.
- Use a warm, damp towel to remove gel.
- Complete the procedure with a light hand massage using a slimming or skin tightening cream.
- It is recommended to rest for 5 minutes after the procedure.
- While the patient is resting, use this time to give recommendations to the client.

| MASSAGE TECHNIQUES FOR DIFFERENT AREAS

Abdomen

Step 1:

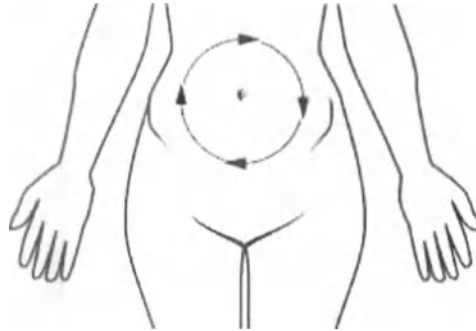- Work in this area, using the lowest intensity, in semi-circular movements.
- Repeat 3 times, guiding the hand-piece over and under the navel.

Step 2:

- Using the longitudinal technique, massage the chest area.

Step 3:

- Using the transverse technique, massage the waist.

Legs

Step 1:

- Massage the inner and outer surface of the thighs with upward movements.

Step 2:

- Repeat the same movement in the front part.

Step 3:

- Use the longitudinal and zigzag techniques.

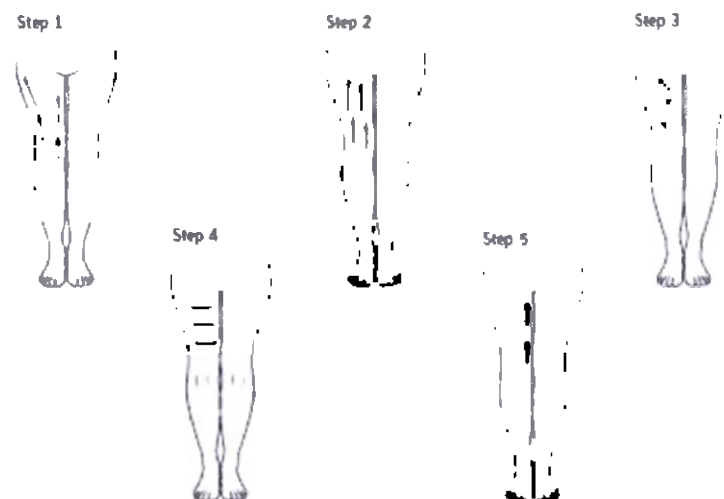
Step 4:

- Use the transverse technique on the thighs.

Step 5:

- Complete the procedure with vibrating movements on the inner thigh.

Repeat all the procedures from step 1 to step 5 for each leg.


Deep subdermal (subcutaneous) photo-pneumatic massage

The patient lies on the side, the lower leg is straight, the upper one is bent. This position is the most convenient and allows the patient to tolerate the procedure better in case of using more powerful vacuum settings.


Step 1:

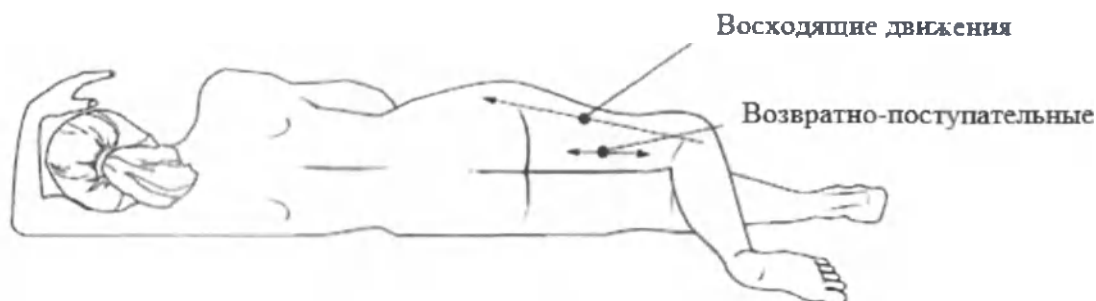- Start with slow upward movements, followed by reciprocating ones.

Step 2:

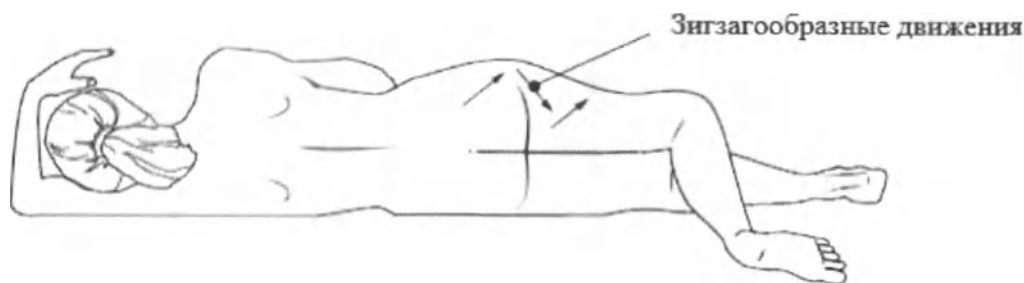- Use the zigzag technique.


Step 3:

- Use the semicircular technique on the buttocks.

Step 4:

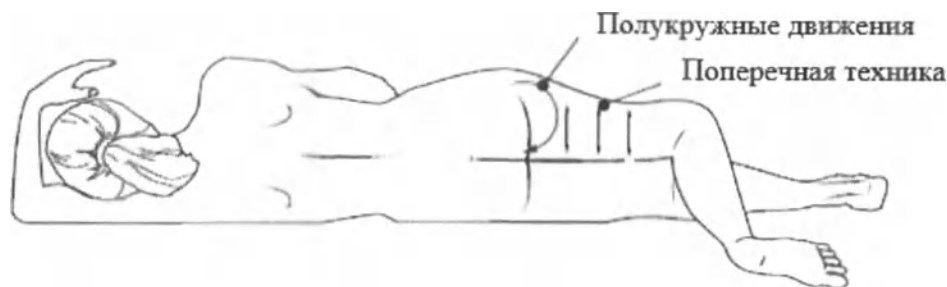- Massage the thighs using the transverse technique.

Rear part of the legs

Patient in the prone position. Use oil for photo-pneumatic massage.

Step 1:

- Start with wide strokes, driving blood to the heart.

Step 2:

- Use the reciprocating movement while massaging the thighs.

Step 3:

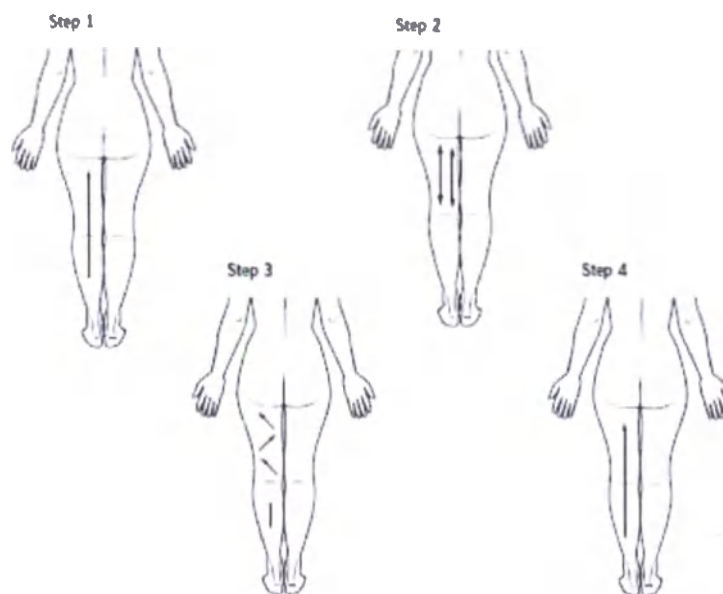
- Work in the transverse and zigzag techniques.

Step 4:

- Use upward movements, starting from the bottom to the upper thigh.

Do the same on the other leg.

After completing the procedure, cover the patient and allow him/her to lie on his/her back for about 5 minutes.

11 RECOMMENDATIONS TO PATIENTS DURING TREATMENT

For proper effect of any kind of treatment, it is important that the patient is aware of all the factors that can help achieve and maintain an optimum result.

ESSENTIAL MEASURES TO REDUCE CELLULITE

Lumicell Wave 6 is the apparatus that can help you to get rid of cellulite. But you can help yourself to achieve better results. Lumicell Wave 6 is the primary means of achieving results. In addition to the fact that many clients notice satisfactory changes only during the course treatment, they can also enhance and improve these results by following a few simple rules.

Use a bath glove

Use a bath glove or loofah for your daily shower. You can also use a body massage brush before or after shower. These actions improve the blood circulation and lymph drainage, and are essential in preventing cellulite. Pay particular attention to areas that are prone to fat deposits, such as the thighs, hips, buttocks, and abdomen.

Exercise regularly

Physical exercise is an important part of healthy lifestyle. It stimulates beta-receptors to convert fats into energy (through lipolysis). Exercise also improves metabolism and helps to burn calories. It is not necessary to be vigorous to be useful: active walking is the best way to start regular exercise.

DIETARY RECOMMENDATIONS

Cut down on sugar

Eliminate sweets from the diet (drinks, pastries, sweets, etc.). The presence of sugar in the diet provokes lipogenesis (fat deposition), increasing the volume of adipose cells. Alpha-receptors in adipose cells need sugar to convert it to fatty acids and body fat.

Develop healthy eating habits

Eat more fruit and vegetables; eat less red meat and replace it with chicken and fish; Whole grains and vegetables (beans) are an important part of the diet and help to cleanse and detoxify the body.

Cut down on saturated fat

The consumption of saturated fat is associated with cancer and heart disease. Chicken and fish are the best dietary choice. When cooking, give preference to steaming or boiling, rather than frying in oil. Replace butter with oil.

Drink plenty of water

Water is an important component of cellulite treatment, as it supports optimal hydration. Drink 8 to 12 glasses of water per day. This will help the effective functioning of the body and remove toxins from the body.

Include fiber in your diet

Foods rich in fiber is an important element of healthy digestion and weight loss. Complex carbohydrates are part of whole grains and vegetables. Fruits and vegetables are best eaten raw, baked or boiled, so as not to lose the vitamins they contain.

Cut down on salt

Excess sodium in the diet can lead to moisture retention, especially with a tendency to swell (legs and arms).

ATTENTION: A copy of these recommendations should be provided to each patient.

15 APPLICATION OF THE APPARATUS AFTER LIPOSUCTION

General information on liposuction

- In the postoperative period, most patients experience painful sensations. Some areas are more painful. For example, the abdominal area is more sensitive than the thighs and buttocks.
- Skin abrasions, edema and swelling after liposuction are normal.
- The recovery time depends on the area of impact.
- Patients must wear slimming clothing for four weeks, 24 hours a day. Its helps to reduce swelling and sculpts the contour of the body.
- The recovery time is different for different patients and depends on the postoperative care. It is usually safe to return to normal sports activity when the patient is comfortable, usually 4-6 weeks.

ATTENTION: Liposuction does not alter the appearance of cellulite. Although it leads to a decrease in the circumference, the waviness and indentations on the skin remain.

Liposuction procedure

The incisions for catheters are very small – 3-4 mm. Each hole is sutured with an invisible stitch. A small bandage is applied to the incisions. After that, the patient wears a slimming bandage that controls edema and swelling.

Problems

After removing the bandage, the tissues are hard and hilly; the skin is unable to hold them in place and folds are formed in the areas where fat has been removed. These areas need daily massaging.

Treatment with Lumaticell Wave 6

Advantages of use before surgery

It is recommended to use **six Lumaticell Wave 6 modes, including infrared photo-pneumatic massage or radio waves**. This will increase the skin elasticity, reduce the recovery time and improve the visible effects of liposuction.

- Follow the general guidelines (see previous pages).
- Use radio-frequency therapy to stimulate collagen production.

After surgery

Lumicell Wave 6 has a **drainage program** for massaging delicate tissues to improve the condition of the body after liposuction. This program will improve the postoperative results of treatment and reduce the need for repeated supporting surgical treatment.

The ultrasonic cavitation contributes to the fastest resorption of edema. Make sure you use enough gel for the ultrasound procedures. Use 20% intensity and exposure time from 1 to 2 minutes.

If the patient experiences pain during exposure, the intensity should be reduced and the procedure stopped.

Ultrasound stimulates fibroblasts to produce collagen, which speeds up the healing process. Connective tissues become more plastic and smooth.

Precautions:

- **Lumicell Wave 6 must not be used until the edema subsides and the compression bandage is removed, which usually takes 6 to 8 weeks.**
- It is possible to begin treatment earlier (3-4 weeks after surgery), but only with the permission of the surgeon.
- To achieve the best result, the whole body surface should be treated.
- **Avoid the areas with incisions.** Go around them.
- It is strongly recommended to receive a certificate from the doctor and attach it to the chart of the patient who has been or will be undergoing the liposuction procedure.

Frequency of procedures

It is recommended to **carry out a course consisting of 10-12 sessions.**

Procedures should be held **twice a week.**

1 or 2 sessions per month as continuation treatment.

Frequency settings

The first six sessions are drainage massage. The intensity must be very low. **Always be guided by the preferences and sensitivity of the patient. Never hurt: treatment should be comfortable.** If the patient feels pain during the procedure, the intensity of the photo-pneumatic massage must be reduced.

After liposuction

Use the anti-cellulite program at the lowest intensity level. The best results are achieved by using a softer vacuum massager – swelling subsides faster.

Record the results and intensity level of each session.

The intensity level can be increased during the procedure, taking into account the patient's comfort.

You can use the highest intensity level, usually during the 8th session, paying attention to the patient's condition.

- Remember that treatment must be comfortable and enjoyable.
- You can start using ultrasound procedures from the 9th session to improve the mechanism of cell renewal and speed up recovery.

16. NEEDLE-FREE MESOTHERAPY

The possibility of carrying out the procedure of needle-free mesotherapy with the apparatus Lumicell Wave 6

A new method to administer products into the mesoderm using the electroporation technology.

Numerous benefits:

- The method does not require the use of injection needles.
- No pain or side effects.
- Fast impact on a large area.
- It reduces the interval between treatments and contributes to the rapid achievement of the effect.

Developed in France more than 50 years ago, the mesotherapy has been successfully used to reduce cellulite. This process requires weekly injections for 8–10 weeks. Unfortunately, the distribution of the method is limited, since injections cause discomfort, uncomfortable burning and swelling. The most common patient complaint is the number of injections used in mesotherapy. Due to painful injections, the level of refusal to perform subsequent procedures is high (80-90%).

The function of mesotherapeutic effects in Lumicell Wave 6 is a needle-free medicine electrophoresis procedure. It allows 99% of the medicinal substance to penetrate to a depth of 1 cm and approximately 65% – to a depth of 6 cm. Whereas conventional electrophoresis provides penetration of only 10% of the substance to a depth of 1 cm.

How does it work?

- The needle-free mesotherapy function transforms the direct current into a complex wave using the three-phase polarity.
- The flow is repeated every 3.5 seconds to ensure better penetration and to improve the results of treatment. By electroporation, this process allows more substance to pass through the cell membrane.
- In the course of treatment, both positive and negative charges are used in order to ensure the passage of all the components of the product.
- The exposure time is about 30 minutes (15 minutes per each leg).

Carrying out the procedure

- Clean the exposed area with a fat-free product.
Use mesotherapy gel.
Select a program using the **"M"** button.
- The intensity of 1000 microamps is automatically preset. You can increase the intensity up to 3000 microamps when working on thicker areas, or lower it on sensitive areas.
- Using attachments for needle-free mesotherapy, perform slow movements and apply slight pressure.
When working on dense areas, work on each one for 15 minutes.
Complete the procedure using a tightening or slimming cream.
- For best results, it is recommended to hold 10 sessions, twice a week.
One session per month is recommended to maintain the result.

7 PATIENT CHART

PATIENT CHART: _____

Section to be filled in by the client

Personal information

Surname: _____ Name: _____
 Date of birth: _____ Sex: ☐ F ☐ M
 Address: _____
 Postcode: _____ Phone: _____
 Employer: _____
 By recommendation: _____

General Information/Lifestyle

Height: _____ Weight: _____
 Clothing size: _____ Desired weight: _____
 Number of pregnancies: _____ Diet: ☐ Yes ☐ None
 Contraception: ☐ Pills ☐ IUD ☐ Menopause
 Have you taken anti-cellulite procedures before? ☐ Yes ☐ None
 What? _____ When? _____
 How many meals a day do you take? _____
 Do you exercise? ☐ Yes ☐ None
 What kind of exercise? _____
 How many glasses of water a day do you drink? _____
 Do you use special body care products? ☐ Yes ☐ None
 Do you smoke? ☐ Yes ☐ None

Lumicell Wave procedures are absolutely safe for most people, but you must take precautions in case of certain diseases or conditions.

General contraindications

- ☐ Pregnancy/Lactation ☐ Thrombophlebitis
☐ Cancer/HIV ☐ Skin Damage/Inflammation
☐ Uncontrolled high blood pressure

Contraindications to the use of vacuum massage

- ☐ Inflammation of lymph nodes ☐ Thrombophlebitis
☐ Gastric or duodenal ulcer ☐ Cardiac failure
☐ Prominent varicose veins

Contraindications to ultrasound, radio frequency and electroporation

- ☐ Cardiac pacemaker ☐ Adrenal failure
☐ Bone prostheses ☐ Nephrolithiasis
☐ Cholelithiasis

More detailed information

Menstruation before the procedure: Tell your doctor.

IUDs: abdominal massage is not recommended.

One month after Caesarean section: with the permission of the doctor.

Blood circulation or vascular disorders:

*medical examination is required.*Taking anticoagulants: *medical examination is required.*

Current medication:

Signature

"I confirm the absence of contraindications and the correctness of the information"


--

*Section to be filled in by the professional***Conclusion**

Cellulite	☐ Orange peel	☐ Unevenness	☐ Local body fat
Blood circulation and lymph flow	☐ Varicose veins	☐ Spider veins	☐ Edema
Other changes	☐ Heavy legs		
	☐ Elasticity/Tone	☐ Stretch marks	☐ Others

Cellulite grade
☐ Grade 1 ☐ Grade 2 ☐ Grade 3 ☐ Grade 4
Further Information

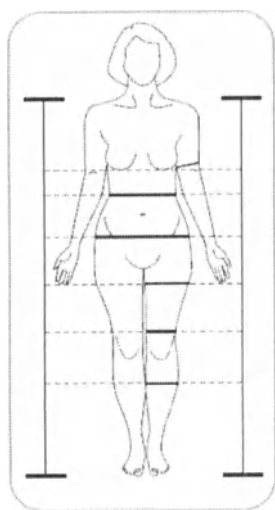Measurements + Photos 1 cm color background 2 meters distance no face. Optional.

	First procedure	Last procedure	Difference
Dates			
Weight	kg	kg	
Arms	cm	cm	
Waist	cm	cm	
Upper thigh	cm	cm	
Lower thigh	cm	cm	
Knees	cm	cm	
Lower legs	cm	cm	

Course and treatment regimen	
Problem areas	

Programs and intensity

Procedures	Cellulite		Sculpt		Drainage	EP	US	RF
	Stage	Intensity	Stage	Intensity	Intensity	mA	%	%
1								
4								
7								
10								
13								
16								


Procedures	Dates	Doctor	Observation data
1			
2			
3			
4			
5			
6			
7			
8			
9			
10			
11			
12			
13			
14			
15			

Results: Client's and professional's comments

18. CLEANING FILTERS

There are 2 methods to clean the filters.

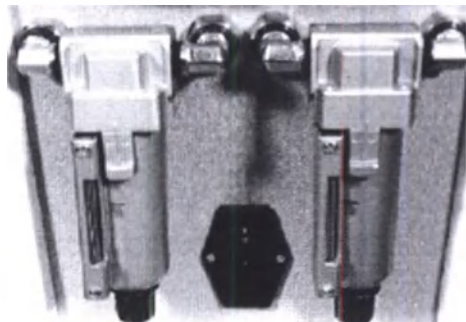Method 1:

This is what the filters on the rear part of the device look like. Each filter has a "maximum fill" level.

Filters must be cleaned as soon as the oil level reaches Max.

Press the button and rotate the filter. The marks must match (see the picture). Remove the filter by pulling it down.

To install the filter, repeat the procedure in the reverse order.

Method 2:

Turn the valve clockwise to drain the oil. Drain the oil and turn the valve in the opposite direction.

19. COMPLETE SET OF LUMICELL WAVE 6

Number	Code	Description
2	411369	Hand-piece for radio-frequency therapy
1	411371	Hand-piece for ultrasonic cavitation
2	411368	Hand-piece for needle-free mesotherapy
2	411377	Hand-pieces for photo-pneumatic massage
1	411376	Small attachment to the hand-piece for photo-pneumatic massage
1	411375	Large attachment to the hand-piece for photo-pneumatic massage
2	411373	Connecting pipes for 411377 and their holders
1	411378	Hand-piece for vacuum therapy
1	411382	Small attachment to the hand-piece for vacuum therapy
1	411383	Large attachment to the hand-piece for vacuum therapy
4	412283	Casters
1		Network cable

* Please check the availability of all accessories upon receipt.

21.1 The hand-pieces are color-coded to make it easier to identify their purposes

Hand-piece for ultrasonic cavitation – yellow marking.

Two hand-piece for radio-frequency therapy – purple marking.

Two hand-pieces for needle-free mesotherapy – pink marking.


21.2 Symbols on the Apparatus

Symbol	Meaning
	Special disposal. Individual assembly
	Refer to the instructions.
	Caution! Voltage
	BF type operating parts

21.3 Handling signs on transport packaging

Symbol	Meaning
	Manufacturer
	Up
	Handle with care
	Protect from moisture

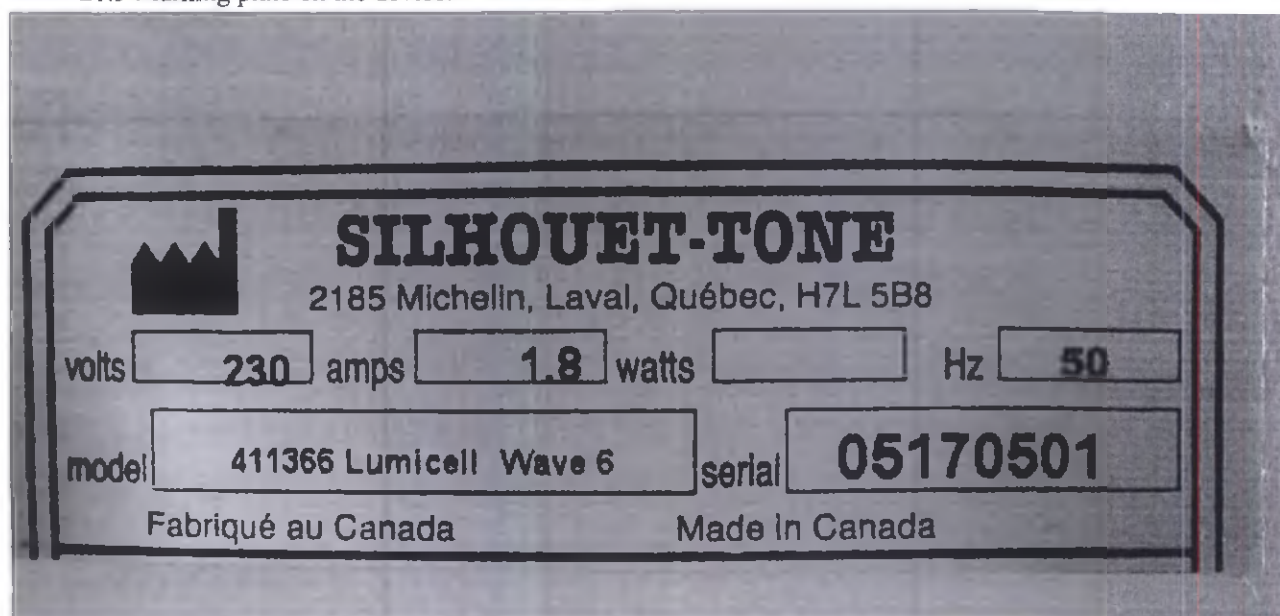21.3 Marking plate on the device:

21.4 Marking on the device for consumers in the Russian Federation:

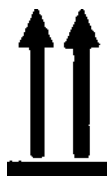
- name and trademark of the manufacturer;
- manufacturer's address;
- apparatus name and model designation;
- name and address of the authorized representative in the Russian Federation;
- device manufacture date;
- device number;
- nominal voltage, current and frequency;
- country of origin.

21.5 The following shall be marked on the shipping container:

- name or trademark of the manufacturer;
- manufacturer's address;
- apparatus name or model designation;
- apparatus bar-code
- year and month of packaging;
- European Compliance mark;
- sign "Up";
- transportation and storage conditions (in symbols);
- legend: "APPAREIL FRAGILE. NE PAS LAISSER TOMBER. DELIVATE INSTRUMENTS. DON'T DROP"

- Marking is applied on a paper label with an adhesive backing.


The shipping container must have handling symbols with the following meaning: "Up", "Temperature limits from -15°C to $+50^{\circ}\text{C}$ ", "Permissible humidity from 0 to 80%", "Atmospheric pressure limit from 700 to 1060 kPa"


Up

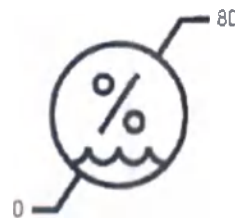Temperature limits from -15°C to $+50^{\circ}\text{C}$

Permissible humidity from 0 to 80%

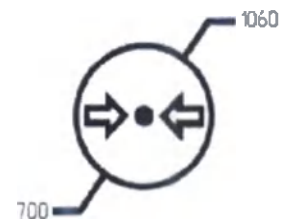Atmospheric pressure limit from 700 to 1060 kPa

Handling symbol are applied on a label with an adhesive backing.

Designation of storage conditions and other additional inscriptions should be applied on the container, free from shipping markings.

Transportation requirements | 22

Transport in the original packaging of the manufacturer only.

Transportation temperature: from -15°C to $+50^{\circ}\text{C}$ (if the refrigerant is drained), relative humidity from 0 to 80% without condensation, atmospheric pressure from 500 to 1060 kPa

Rapid temperature changes should be avoided due to possible condensation.

It is allowed to use the device only after temperature equalization.

The packaging is disposed at the collection points for household waste.

Disposal of Lamicell Wave 6 and accessories requires special precautions and it must be taken to appropriate disposal sites separately from normal household waste. To prevent environmental pollution, all waste generated during the disposal of the device must be collected and disposed according to the established procedure in the country of use, in compliance with the applicable requirements for Class A medical waste handling.

The authorized representative of the manufacturer in the Russian Federation – LLC Skin Science, 125, Varshavskoye Highway, building 1, section 11, 1st floor, room 1, office 39, 117587, Moscow, Russia. E-mail: info@hydrafacial.ru (tel. 8 (495) 987-38-68) – carries out a separate collection of old equipment aimed in order to prevent potential damage to the environment and health, and recycle second hand devices, if necessary.

When this symbol, representing a crossed-out garbage container, is on the device, it means that the device is subject to European Directive 2002/96/EC and must not be mixed with normal household waste.

10. FURTHER INFORMATION AND SPECIFICATION

1. This apparatus must be connected to the power supply through a surge protection unit and earthed.
2. Lemicell Wave 6 is approved by UL, CSA (NRTL) and CE certificates.
3. If you have any questions about the operation of this equipment, you can contact Silhouet-Tone or your regional distributor.
4. Lemicell Wave 6 equipment must be serviced and used by certified professionals only. The Silhouet-Tone Company is not responsible for any harm or accidents due to misuse of the equipment.
5. The use of accessories, transducers and cables that are not listed, with the exception of transducers and cables supplied by the manufacturer of the medical device as replacement parts for internal components, may lead to an increase in electromagnetic emission or a reduction in the device immunity.
6. The medical device should not be used in close proximity or in conjunction with other equipment and, if such use is necessary, verification of the normal functioning of the product in this configuration should be carried out.

The main specifications are shown in Table 1.

Table 1

Item Name	Parameter	Value
Main unit	Voltage	220/230 V
	Frequency	50 Hz
	Safety device	2A SB
	Alternating current	2A
	Dimensions:	
	Width	58.4 cm
	Height	167 cm
	Depth	58.4
	Unit weight	57 kg
	Electronic parts	Microprocessor controlled central board
When operating in the photo-pneumatic massage mode	Wavelength of the infrared radiation	890 nm
When operating in the vacuum therapy mode	Vacuum level	from 1 to 10 un. (1 un. = 6.772 kPa)
	Maximum vacuum level	67.727 kPa
	Minimum vacuum level	6.772 kPa
	Vacuum mode	Continuous
When operating in the ultrasonic cavitation mode	Intensity	from 25 to 100 %
	Pitch	25 %
	Ultrasonic frequency	1 MHz
When operating in the radio-frequency therapy mode	Intensity	from 20 to 100 %
	Pitch	20 %
	Frequency	450 kHz
When operating in the needle-free mesotherapy mode	Intensity	From 1000 μ A to 4000 μ A
	Pitch	100 μ A
	Frequency	450 kHz
Filters	Trademark	-
	Volume	105 ml
	Dimensions	diameter – 45 mm, height – 133
	Weight	232 g

Item Name	Parameter	Value
	Number	2
Hand-piece for photo-pneumatic massage	Dimensions	length – 83 mm, width – 78 mm, height – 80 mm.
	Weight	230 g/1 pc.
	Number	2
Hand-piece for vacuum therapy	Dimensions	length – 126 mm, diameter – 32 mm
	Weight	83 g/1 pc.
	Number	2
Hand-piece for ultrasonic cavitation	Dimensions	length – 133 mm, diameter – 63 mm
	Weight	450 g/1 pc.
	Number	1
Hand-piece for radio-frequency therapy	Dimensions	length – 133 mm, diameter – 63 mm
	Weight	585 g/1 pc.
	Number	2
Hand-pieces for needle-free mesotherapy	Dimensions	length – 143 mm, diameter – 58 mm
	Weight	470g/1 pc.
	Number	2
Total sound power level		Max 50 dB
Moving force		Max 150 N
Brake pedal force		Max 50 N
Safe duration of procedures		from 35 min to 60 min
needle-free mesotherapy mode:	Current	From 1000 μ A to 4000 μ A (pace 100)
	Pitch	100 μ A
	Frequency	450 kHz
	Penetration depth	from 1 to 4 cm

Appendix – Electromagnetic Compatibility Declaration.

Table 2

Manufacturer's instruction and declaration – electromagnetic emission		
Apparatus for physical therapy Lumicell Wave 6 is intended for use in the electromagnetic environment as defined below. The buyer or the user of the Apparatus for physical therapy Lumicell Wave 6 should ensure their use in the specified electromagnetic environment		
Electromagnetic Emission Test	Conformity	Electromagnetic environment - instructions
Radio interference by GOST R 51318.11-2006 (CISPR 11-2004)	Group 1	The apparatus uses radio frequency energy only to perform internal functions. RF emission level is low and probably will not lead to functional disruptions in the proximity of electronic equipment. The apparatus has a structural protection from the effects of external EM fields. The apparatus is suitable for use in all institutions that are directly connected to the public low-voltage power supply network used for domestic purposes.
Radio interference by GOST R 51318.11-2006 (CISPR 11-2004)	Class B	
Harmonic components of current consumption in accordance with GOST 30804.3.2-2013 (IEC 61000-3-2: 2005)	Class A	
Voltage fluctuations and flicker by GOST 30804.3.3-2013 (IEC 61000-3-3: 2008).	Conform	

Table 3

Manufacturer's instruction and declaration – noise immunity			
Apparatus for physical therapy Lumicell Wave 6 is intended for use in the electromagnetic environment as defined below. The buyer or the user of the apparatus for physical therapy Lumicell Wave 6 should ensure their use in the specified electromagnetic environment.			
Noise immunity test	Test level according to IEC 60601	Compliance Level	Electromagnetic environment - instructions
Electrostatic discharge (ESD) by GOST 30804.4.2-2013 (IEC 61000-4-2:2008)	±6 kW – contact discharge ±8 kW – air discharge	± 6 Kw contact discharge ± 4, Kw air discharge	Floors should be of wood, concrete or ceramic tiles or covered with an antistatic material. Service personnel should wear clothes that do not accumulate electrostatic discharge. Relative humidity should be at least 30%.
Electrical fast transient burst by GOST 30804.4.4-2013 (IEC 61000-4-4:2004)	±2 kW for power lines ±1 kW for input/output lines	±2 kW for power lines ± 1 kW for input/output lines	The quality of electrical energy in the network in accordance with the typical conditions of commercial or hospital environment

Microsecond high energy impulse noise by GOST P 51317.4.5-99 (IEC 61000-4-5:96)	± 1 kW when supplying noise by the "wire-to-wire" scheme ± 2 kW when supplying noise by the "wire-to-earth" scheme	± 1 kW when supplying noise by the "wire-to-wire" scheme ± 2 kW when supplying noise by the "wire-to-earth" scheme	The quality of electrical energy in the power supply network in accordance with the typical conditions of commercial or hospital environment
Failures, interruptions and voltage changes in the input power lines by GOST 30804.4.11-2013 (IEC 61000-4-11:2004)	$<5\% U_H$ (Voltage dip $>95\% U_H$) during 0.5 period $40\% U_H$ (Voltage dip $60\% U_H$) during 5 periods $70\% U_H$ (Voltage dip $30\% U_H$) during 25 periods $0\% U_H$ (Voltage dip $>95\% U_H$) during 250 periods	$<5\% U_H$ (Voltage dip $>95\% U_H$) during 0.5 period $40\% U_H$ (Voltage dip $60\% U_H$) during 5 periods $70\% U_H$ (Voltage dip $30\% U_H$) during 25 periods $0\% U_H$ (Voltage dip $>95\% U_H$) during 250 periods	The quality of electrical energy in the network in accordance with the typical conditions of commercial or hospital environment. If the user of the apparatus for physical therapy Lumicell Wave 6 needs to ensure continuous operation under conditions of possible interruptions of the network voltage, it is recommended to supply power from an uninterruptible power supply or battery
Magnetic field of industrial frequency according to GOST R 50648-94 (IEC 1000-4-8-93)	3 A/m	3 A/m	Levels of the magnetic field of industrial frequency should be provided in accordance with the typical conditions of commercial or hospital environment.

Note – U_H – the voltage level of the electrical network until test exposure.

Table 3

Manufacturer's instruction and declaration – noise immunity			
Apparatus for physical therapy Lumicell Wave 6 is intended for use in the electromagnetic environment as defined below. The buyer or the user of the Apparatus for physical therapy Lumicell Wave 6 should ensure their use in the specified electromagnetic environment			
EM security test	Test parameters according to IEC 60601	Compliance Level	Electromagnetic Security - Instruction
			Portable and mobile RF communication equipment should be used no closer to any part of the device, including cables, than the recommended distance, which is calculated from the equation that takes into account the frequency of the transmitter. recommended distance

Conductive interference induced by radio-frequency electromagnetic fields according to GOST R 51317.4.6-99 (IEC 61000-4-6-96)	3 V (root-mean-square value) within the range 150 kHz to 80 MHz	3 V	
Radiated radio-frequency electromagnetic field by GOST 30804.4.3-2013 (IEC 61000-4-3:2006)	3 V/m within the range of 80 MHz to 2.5 GHz	3 V/m.	$d = 1.11 \sqrt{P}$ $d = 0.95 \sqrt{P} \quad 80 \text{ MHz} - 800 \text{ MHz}$ $d = 1.89 \sqrt{P} \quad 800 \text{ MHz} - 2.5 \text{ GHz}$ Where "P" is the maximum output power of the transmitter in watts (W) according to the manufacturer, and "d" is the recommended distance in meters from the transmitter (m). The field density of stationary RF transmitters, as determined by the electromagnetic survey of the object, ^a Must be under the level of compatibility in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1: At 80 MHz and 800 MHz, the frequency range is higher.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic wave propagation is affected by absorption and reflection from structures, objects and people.

^a The field density of stationary transmitters such as base stations for radio (cellular/cordless) phones and land mobile radio stations, amateur radio, AM and FM radio and television broadcasting cannot be based theoretically accurate. In order to assess the electromagnetic environment and the conditions of influence of stationary radio transmitters, take into account the location of the electromagnetic survey. If the measured field density on site where the apparatus is used exceeds the permissible RF level specified above, it is necessary to examine the performance of the apparatus to ensure that it operates in the normal way. If you notice irregularities in the work, additional measures may be necessary, for example, reorientation of the apparatus location.

^b In the frequency range from 150 kHz to 80 MHz, the field density should be under 3 V/m.

Table 4

Recommended values of the separation distance between portable and mobile radio frequency communication equipment and the apparatus for physical therapy Lumicell Wave 6

The apparatus for physical therapy Lumicell Wave 6 is intended for use in the electromagnetic environment in which the levels of radiated interference are monitored. The buyer or the user of the apparatus for physical therapy Lumicell Wave 6 can avoid the effects of electromagnetic interference by ensuring minimal spatial separation between portable and mobile radio frequency communication means (transmitters) and the apparatus for physical therapy Lumicell Wave 6 as recommended below, taking into account the maximum output power of the communication means.

Nominal maximum transmitter output power, W	Separation distance depending on the transmitter frequency, m		
	150 kHz – 80 MHz $d = 1.2 \sqrt{P}$	80 MHz – 800 MHz $d = 1.2 \sqrt{P}$	800 MHz – 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	0.11	0.09	0.19
0.1	0.35	0.30	0.60
1	1.11	0.95	1.89
10	3.51	2.99	5.98
100	11.11	9.46	18.92

For transmitters designed for maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the formula applicable to the transmitter frequency, where P is the maximum transmitter output power in watts (W) depending on the manufacturer's transmitter.

NOTE 1: At 80 MHz and 800 MHz, separation distance for higher frequency range is applied.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic wave propagation is affected by absorption and reflection from structures, objects and people.

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