WRP Asia Pacific Sdn Bhd

147817V

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
Email customer_wrp@wrpworld.com

Website www.wrpworld.com

Instruction for use of Medical Product

Examination latex gloves TET-A-TET[®] sterile disposable powder
free, produced by WRP Asia Pacific SDN. BHD., Malaysia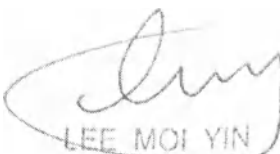

LEE MOI YIN
NOTARY PUBLIC
PETALING JAYA
SELANGOR
MALAYSIA

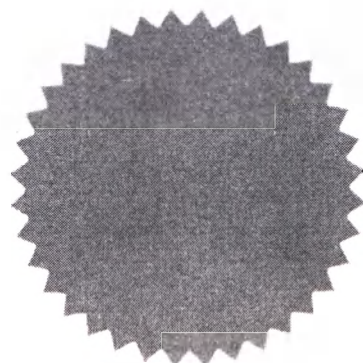
Table of Contents

1.	Name of a Medical Product	3
2.	Field of Application of a Medical Product	3
3.	Application of a Medical Product	3
4.	Description of a Medical Product	6
5.	Technical characteristics	7
6.	Storage and Transportation Conditions	12
7.	Conditions for the Use	12
8.	Packaging	13
9.	Labelling	13
10.	Contents of Delivery	14
11.	Warranty and Shelf Life	14
12.	Procedure for Disposal and Destruction of a Medical Product	14
13.	Information about the Producer and Manufacturer of a Medical Product	14
14.	Responsible Supplier	14
	Annex	15

1. Name of a Medical Product

Examination latex gloves HPL A-HPL – sterile disposable powder free, produced by WRP A.S. (in the SLOVAK REPUBLIC), BHD., Malaysia

2. Field of Application

2.1 Application

The gloves are intended for single use as a protective barrier when worn by medical personnel during the examination/treatment of a patient or for other sanitary purposes.

2.2 Field of application

Medical treatment and preventive care institutions of a wide profile.

3. Application of a Medical product

3.1 Indications for use

The gloves are intended for single use in diagnostic procedures, therapeutic procedures, medical research, which require compliance of patient- medical personnel protection from cross infection

Attention! Medical gloves do not guarantee 100 % hands protection from puncture
Gloves are indicated for single use.

3.2 Contradictions:

- allergic reaction on latex

3.3 Side effects:

Gloves manufactured from natural rubber latex can cause an allergic reaction

Due to the use of gloves medical personnel can be exposed to the following types of adverse effects: non-allergic contact dermatitis, allergic contact dermatitis and immediate type allergic reactions, followed by chronic skin itching and systemic allergies

Allergic reactions caused by the use of gloves are often associated with latex allergy, i.e. hypersensitivity sensitization of medical personnel to chemicals in latex which is the most popular material for the production of medical gloves

3.4 Rules for utilization of a Medical Product

Each medical worker, knowing the peculiarities of job, must determine the requirements to the gloves

taking into consideration:

- real manufacturing risks (assessment of biological risks, assessment of mechanical risks, assessment of chemical risks);
- sanitary and epidemiological conditions (necessity of sterility);
- features of manipulations (types of manipulations and procedures performed, physical effort, the duration of work in gloves, the need to preserve tactile sensitivity, and so forth);
- conditions of the skin on hands of medical personnel, allergological anamnesis of the worker and the patient and a number of other factors

The use of gloves does not replace hand hygiene. Hand treatment should be carried out in accordance with the manipulation mode

- It is necessary to choose the right size gloves
- It is necessary to choose the gloves in accordance to the medical procedure – they should be easily worn and should not impede movements or squeeze hands
- When putting on gloves, no significant efforts should be made
- When putting on gloves, the installed equipment should be used, which will ensure their sterility
- Before putting on gloves for performing medical procedures, it is necessary to prepare a workplace – ready to use sterile tools and consumables should rest on the manipulation table. It is necessary to ensure their availability without compromising the sterility of the hands
- The use of an extra pair of gloves reduces the frequency of glove damage by 2-4 times when working with sharp tools
- Do not use gloves with damaged packaging, as their sterility may be impaired
- After removing the gloves, wash hands with soap and treat them with a dermal antiseptic


3.5 Application of a Medical Product

Before using gloves, you must ensure the integrity of the package. Then open the secondary packaging. The primary glove pack is sterile. In order to open the primary packaging, get gloves and start using them for the intended purpose, it is necessary to treat the hands with an antiseptic.

3.5.1 Putting on gloves

When putting on gloves, it is recommended to follow the techniques described below to avoid contamination of their external surfaces during the donning process. There are two techniques for putting on gloves: open and closed. With any technique of putting on gloves, the most important thing is not to allow contamination of their outer surface.


Steps of putting on gloves by open technique

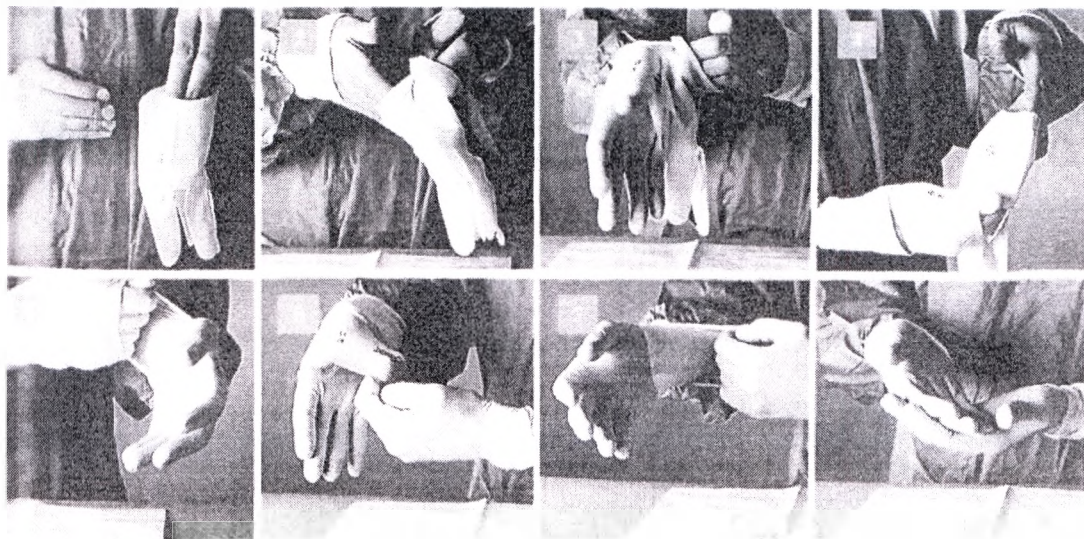- Open the sterile package and take both gloves with the fingertips of the left hand from the inside of the left glove so that the tip of the thumb is inside the right glove and keep them together in an upright position. Make sure that the fingers touch only the inner surface of the gloves.
- Pass the gloves with your right hand into the cuff, holding the gloves with your left hand.
- Tighten the folded glove until the right hand is in the glove.
- Lower the fingers of the right hand in the glove into the cuff of the left glove and tighten it on the arm.
- Using four fingers, tighten the cuff of the left glove over the cuff of the medical gown.
- Place the fingers of the left hand in the glove under the folded cuff of the glove and pull it onto the cuff of the medical gown.
- Tighten the cuff with four fingers and finally fix the gloves, ensuring their comfortable use.

Steps of putting on gloves by closed technique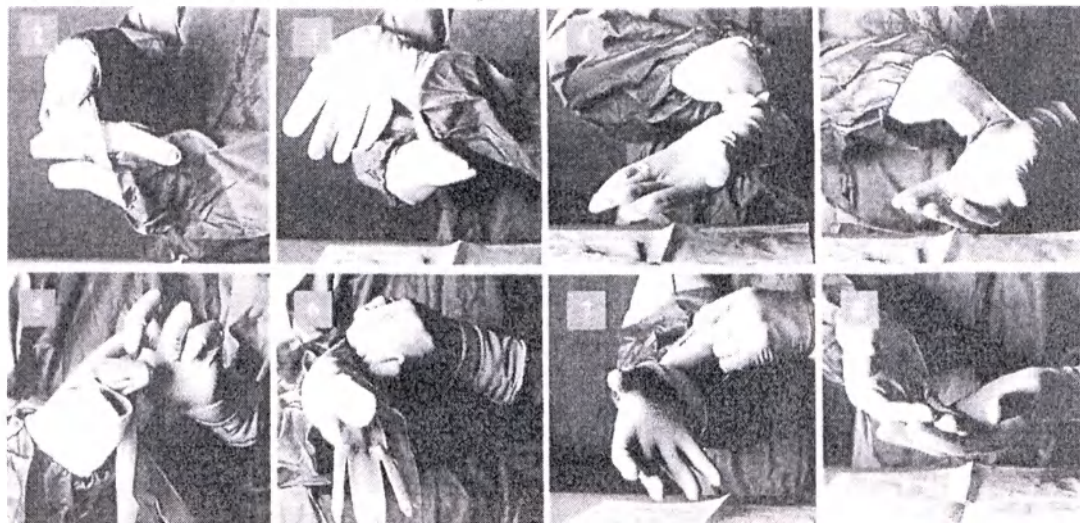


- Open the secondary packaging before washing hands.
- Pulling the sleeves of the dressing gown on your hands, pull out your left glove with your right hand. Hold your hand palm up with straight fingers. Put the glove on your left wrist in such a way that its fingers are pointed towards you, and the thumb of the glove is on the wrist. Press the cuff with the thumb of the left hand.
- Place the thumb inside the cuff from the edge. Squeeze your right hand into a fist and pull the glove over your fingers.
- Having straightened the fingers of your left hand and holding the cuff with your right hand, tighten the glove with the four fingers of your right hand until it is put on the fingers of your left hand.
- Repeat the procedure with another glove. Particularly: with your left hand in the glove put the right glove on the wrist of the right hand so that fingers are directed towards you, and the thumb of the glove is on the wrist. Press the cuff with the thumb of the right hand.
- Place the left thumb in the cuff from the edge. Squeeze your right hand into a fist and pull the glove over your fingers.
- Having straightened the fingers of the right hand and covering the hand with the cuff of the medical sterile dressing gown, use four fingers of the left hand to tighten the glove until it is put on the fingers of the right hand.
- Now that gloves are worn on both hands, they can be fitted on without the risk of contamination.
- To avoid slipping of the gloves from the cuffs of the dressing gown, it is necessary to ensure sufficient length of the sleeves of the medical gown and sufficient length of the cuffs to the base of the palm, as well as the absence of folds of the cuff of the gloves.

3.5.2 Requirements to the work in gloves

- It is necessary to change gloves when going from "dirty manipulation" to "clean", as well as from one patient to another.
- It is necessary to change gloves:
 - after detecting a visible defect;
 - after a possible or actual puncture with a needle, suture material, bone, etc.;
 - If the glove begins to swell, stretch, stick or not seated on the hand as a result of the absorption of liquids or lubricants by the material.
- The gloves are not treated during manipulation. If necessary, the gloves should be replaced.
- Torn or leaking gloves are not suitable for use. If one glove is damaged, you must immediately replace both, because you cannot remove one glove without contaminating the other.
- Remove gloves very carefully, avoiding contamination of the skin of hands and splashing of liquids.
- Treatment of disposable latex gloves with alcohol-containing solutions is not recommended, since these solutions help to weaken the protective layer of gloves.
- Strictly observe the safety measures when working with sharp objects and tools.

3.6 Taking off the gloves

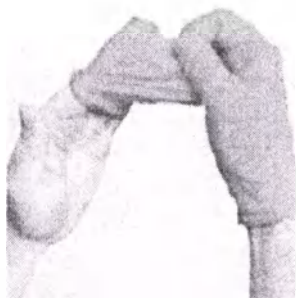It is recommended to remove gloves according to the following algorithm:

- Grasp the glove in the palm base area with the fingers of the other hand.
- Pull the glove and hold it with the other hand.
- Then place your fingers in the glove cuff on the other hand and pull it like the first.
- Turn the glove inside out, use it as a bag for another glove.
- Place the gloves in the container intended for them. Then conduct hygienic treatment of hands.

4. Product Description

4.1 Gloves are made from natural rubber latex.

4.2 There are following types of gloves depending on the coating:

- powder-free textured on fingers gloves;
- powder-free polymer coated textured on fingers gloves;
- powder-free double-chlorinated textured on fingers gloves;
- powder-free single-chlorinated textured on fingers gloves;
- powder-free textured on fingers gloves with a pH of 5.5;
- powder-free gloves, textured surface;
- powder-free polymer coated gloves, textured surface;
- powder-free double chlorinated gloves, textured surface;
- powder-free single chlorinated gloves, textured surface;
- powder-free gloves with a pH of 5.5, textured surface.

4.3 The textured area of textured products includes fingers or an entire surface of the glove. Fingers or fingers and palm on both sides of the glove are textured.

4.4 The polymer coating offers additional strength to gloves, gives protection from the effect of natural latex proteins, makes it easy to put gloves on, and prevents gloves from sticking together.

The gloves can be subjected to chlorine gas treatment (chlorination) by immersion in a chlorine solution. There are two kinds of chlorination: single and double (on both sides) chlorination.

4.5 The gloves can be made in different colors.

4.6 The gloves can be made with cuffs, and with or without beaded cuffs.

4.7 There are seven standard sizes of gloves:

- Extra small (XS)
- Small (S)
- Medium (M)
- Large (L)
- Extra large (XL)
- Extra extra large (XXL)
- Extra extra extra large (XXXL)

4.8 The products are sterile.

4.9. The products are intended for single use only.

4.10 Gloves are classified as Class IIa according to classification by the potential application risk.

5. Technical characteristics

5.1 Width, length and thickness of gloves depending on their size:

- Width, length and thickness of examination latex gloves TET-A-TET[®] Special Exam Latex sterile disposable powder free, textured on fingers, beaded cuff "Standard" shall be as specified in Table 1.

Table 1. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	≤80	250±10	Finger thickness – 0.16±0.05, Palm thickness – 0.14±0.05, Cuff thickness – 0.13±0.05
Small (S)	80±10	250±10	
Medium (M)	95±10	250±10	
Large (L)	111±10	250±10	
Extra large (XL)	115±5	250±10	
Extra extra large (XXL)	123±5	250±10	
Extra extra extra large (XXXL)	130±5	250±10	

- Width, length and thickness of examination latex gloves TET-A-TET[®] Special Exam Latex Polymer sterile disposable powder free, textured on fingers, with polymer coating, beaded cuff "Standard" shall be as specified in Table 2

Table 2. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	≤80	250±10	Finger thickness – 0.16±0.05, Palm thickness – 0.14±0.05, Cuff thickness – 0.13±0.05
Small (S)	80±10	250±10	
Medium (M)	95±10	250±10	
Large (L)	111±10	250±10	
Extra large (XL)	115±5	250±10	
Extra extra large (XXL)	123±5	250±10	
Extra extra extra large (XXXL)	130±5	250±10	

- Width, length and thickness of examination latex gloves TET-A-TET[®] Special Exam Latex Text sterile disposable powder free, textured on fingers, double chlorinated, beaded cuff "Standard" shall be as specified in Table 3.

Table 3. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	≤80	250±10	Finger thickness – 0.16±0.05, Palm thickness – 0.14±0.05, Cuff thickness – 0.13±0.05
Small (S)	80±10	250±10	
Medium (M)	95±10	250±10	
Large (L)	111±10	250±10	
Extra large (XL)	115±5	250±10	
Extra extra large (XXL)	123±5	250±10	
Extra extra extra large (XXXL)	130±5	250±10	

- Width, length and thickness of examination latex gloves TET-A-TET[®] Special Exam Latex Single Chlor latex sterile disposable powder free, textured on fingers, single chlorinated, beaded cuff "Standard" shall be as specified in Table 4.

Table 4. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	≤80	250±10	Finger thickness – 0.16±0.05

Small (S)	80 ± 10	250 ± 10	Palm thickness – 0.6 ± 0.05 Cuff thickness – 0.5 ± 0.05
Medium (M)	95 ± 10	250 ± 10	
Large (L)	111 ± 10	250 ± 10	Finger thickness – 0.5 ± 0.05 Palm thickness – 0.6 ± 0.05 Cuff thickness – 0.5 ± 0.05
Extra large (XL)	115 ± 5	250 ± 10	
Extra extra large (XXL)	123 ± 5	250 ± 10	
Extra extra extra large (XXXL)	130 ± 5	250 ± 10	

Width, length and thickness of examination latex gloves HEI-A-HE-F Special Exam Latex pH 5.5 “Standard” shall be as specified in Table 5.

Table 5. Width, length and thickness of gloves.

Size	Width W, (Fig. 1), mm	Length L, (Fig. 1), mm	Thickness at the points shown in (Fig. 2), mm
Extra small (XS)	80	250 ± 10	Finger thickness – 0.5 ± 0.05 Palm thickness – 0.6 ± 0.05 Cuff thickness – 0.5 ± 0.05
Small (S)	80 ± 10	250 ± 10	
Medium (M)	95 ± 10	250 ± 10	
Large (L)	111 ± 10	250 ± 10	
Extra large (XL)	115 ± 5	250 ± 10	Finger thickness – 0.5 ± 0.05 Palm thickness – 0.6 ± 0.05 Cuff thickness – 0.5 ± 0.05
Extra extra large (XXL)	123 ± 5	250 ± 10	
Extra extra extra large (XXXL)	130 ± 5	250 ± 10	

Width, length and thickness of examination latex gloves HEI-A-HE-F Special Exam Latex pH 5.5 “Standard” shall be as specified in Table 6.

Table 6. Width, length and thickness of gloves.

Size	Width W, (Fig. 1), mm	Length L, (Fig. 1), mm	Thickness at the points shown in (Fig. 2), mm
Extra small (XS)	80	250 ± 10	Finger thickness – 0.5 ± 0.05 Palm thickness – 0.6 ± 0.05 Cuff thickness – 0.5 ± 0.05
Small (S)	80 ± 10	250 ± 10	
Medium (M)	95 ± 10	250 ± 10	
Large (L)	111 ± 10	250 ± 10	
Extra large (XL)	115 ± 5	250 ± 10	Finger thickness – 0.5 ± 0.05 Palm thickness – 0.6 ± 0.05 Cuff thickness – 0.5 ± 0.05
Extra extra large (XXL)	123 ± 5	250 ± 10	
Extra extra extra large (XXXL)	130 ± 5	250 ± 10	

Width, length and thickness of examination latex gloves HEI-A-HE-F Special Exam Latex (no sterile) disposable powder free, textured on fingers, beaded cuff, high risk “Standard” shall be as specified in Table 7.

Table 7. Width, length and thickness of gloves.

Size	Width W, (Fig. 1), mm	Length L, (Fig. 1), mm	Thickness at the points shown in (Fig. 2), mm
Extra small (XS)	80	250 ± 10	Finger thickness – 0.5 ± 0.05 Palm thickness – 0.6 ± 0.05 Cuff thickness – 0.5 ± 0.05
Small (S)	80 ± 10	250 ± 10	
Medium (M)	95 ± 10	250 ± 10	
Large (L)	111 ± 10	250 ± 10	
Extra large (XL)	115 ± 5	250 ± 10	Finger thickness – 0.5 ± 0.05 Palm thickness – 0.6 ± 0.05 Cuff thickness – 0.5 ± 0.05
Extra extra large (XXL)	123 ± 5	250 ± 10	
Extra extra extra large (XXXL)	130 ± 5	250 ± 10	

- Width, length and thickness of examination latex gloves TET-A-TET[®] Special Exam Latex Long sterile disposable powder free, textured on fingers, beaded cuff, extended cuff "Standard" shall be as specified in Table 8.

Table 8. Width, length and thickness of gloves.

Size	Width W, (Fig. 1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	≤80	290±10	Finger thickness – 0.20±0.05, Palm thickness – 0.17±0.05, Cuff thickness – 0.12±0.05.
Small (S)	80±10	290±10	
Medium (M)	95±10	290±10	
Large (L)	111±10	290±10	
Extra large (XL)	115±5	290±10	
Extra extra large (XXL)	123±5	290±10	
Extra extra extra large (XXXL)	130±5	290±10	

- Width, length and thickness of examination latex gloves TET-A-TET[®] Special Exam Latex sterile disposable powder free, textured surface, without bead "Extra" shall be as specified in Table 9.

Table 9. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	≤80	250±10	Finger thickness – 0.17±0.05, Palm thickness – 0.16±0.05, Cuff thickness – 0.13±0.02.
Small (S)	80±10	250±10	
Medium (M)	95±10	250±10	
Large (L)	111±10	250±10	
Extra large (XL)	115±5	250±10	
Extra extra large (XXL)	123±5	250±10	
Extra extra extra large (XXXL)	130±5	250±10	

- Width, length and thickness of examination latex gloves TET-A-TET[®] Special Exam Latex Polymer sterile disposable powder free, textured surface, with polymer coating, beaded cuff "Extra" shall be as specified in Table 10.

Table 10. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	≤80	250±10	Finger thickness – 0.17±0.05, Palm thickness – 0.16±0.05, Cuff thickness – 0.13±0.02.
Small (S)	80±10	250±10	
Medium (M)	95±10	250±10	
Large (L)	111±10	250±10	
Extra large (XL)	115±5	250±10	
Extra extra large (XXL)	123±5	250±10	
Extra extra extra large (XXXL)	130±5	250±10	

- Width, length and thickness of examination latex gloves TET-A-TET[®] Special Exam Latex Text sterile disposable powder free textured surface, double chlorinated, beaded cuff "Extra" shall be as specified in Table 11.

Table 11. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	≤80	250±10	Finger thickness – 0.17±0.05, Palm thickness – 0.16±0.05.
Small (S)	80±10	250±10	

Medium (M)	95±10	250±10	Cuff thickness – 0.13±0.02.
Large (L)	111±10	250±10	
Extra large (XL)	115±5	250±10	
Extra extra large (XXL)	123±5	250±10	
Extra extra extra large (XXXL)	130±5	250±10	

- Width, length and thickness of examination latex gloves TET-A-TET® Special Exam Latex Single Chlor sterile disposable powder free, textured surface, single chlorinated, beaded cuff “Extra” shall be as specified in Table 12.

Table 12. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	≤80	250±10	Finger thickness – 0.17±0.05. Palm thickness – 0.16±0.05. Cuff thickness – 0.13±0.02.
Small (S)	80±10	250±10	
Medium (M)	95±10	250±10	
Large (L)	111±10	250±10	
Extra large (XL)	115±5	250±10	
Extra extra large (XXL)	123±5	250±10	
Extra extra extra large (XXXL)	130±5	250±10	

- Width, length and thickness of examination latex gloves TET-A-TET® Special Exam Latex pH 5.5 sterile disposable powder free, textured surface, beaded cuff, with pH 5.5 “Extra” shall be as specified in Table 13.

Table 13. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	≤80	250±10	Finger thickness – 0.17±0.05. Palm thickness – 0.16±0.05. Cuff thickness – 0.13±0.02.
Small (S)	80±10	250±10	
Medium (M)	95±10	250±10	
Large (L)	111±10	250±10	
Extra large (XL)	115±5	250±10	
Extra extra large (XXL)	123±5	250±10	
Extra extra extra large (XXXL)	130±5	250±10	


- Width, length and thickness of examination latex gloves TET-A-TET® Special Exam Latex Micro sterile disposable powder free, textured surface, beaded cuff, increased tactile sensitivity “Extra” shall be as specified in Table 14.

Table 14. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	≤80	250±10	Finger thickness – 0.15±0.04. Palm thickness – 0.13±0.02. Cuff thickness – 0.12±0.01.
Small (S)	80±10	250±10	
Medium (M)	95±10	250±10	
Large (L)	111±10	250±10	
Extra large (XL)	115±5	250±10	
Extra extra large (XXL)	123±5	250±10	
Extra extra extra large (XXXL)	130±5	250±10	

Fig. 1. Measurement points for width and length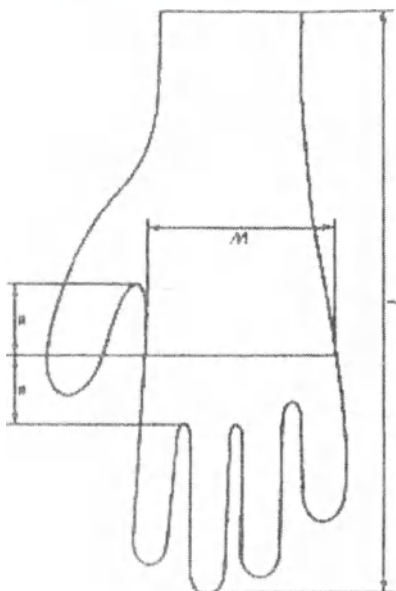

Size	Width W, (Fig. 1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	<80	290±10	Finger thickness - 0.22 ± 0.05, Palm thickness - 0.19 ± 0.05, Cuff thickness - 0.14 ± 0.05
Small (S)	80±10	290±10	
Medium (M)	95±10	290±10	
Large (L)	111±10	290±10	
Extra large (XL)	115±5	290±10	
Extra extra large (XXL)	123±5	290±10	
Extra extra extra large (XXXL)	130±5	290±10	


Table 16. Width, length and thickness of gloves.

- Width, length and thickness of examination latex gloves TET-A-TET Special Exam Latex Long sterile disposable powder free, textured surface, beaded cuff, extended cuff "Extra" shall be as specified in Table 16.

Size	Width W, (Fig. 1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
Extra small (XS)	<80	250±10	Finger thickness - 0.30 ± 0.05, Palm thickness - 0.25 ± 0.05, Cuff thickness - 0.19 ± 0.05
Small (S)	80±10	250±10	
Medium (M)	95±10	250±10	
Large (L)	111±10	250±10	
Extra large (XL)	115±5	250±10	
Extra extra large (XXL)	123±5	250±10	
Extra extra extra large (XXXL)	130±5	250±10	

Table 15. Width, length and thickness of gloves.

- Width, length and thickness of examination latex gloves TET-A-TET Special Exam Latex Ortho sterile disposable powder free, textured surface, beaded cuff, high risk "Extra" shall be as specified in Table 15.

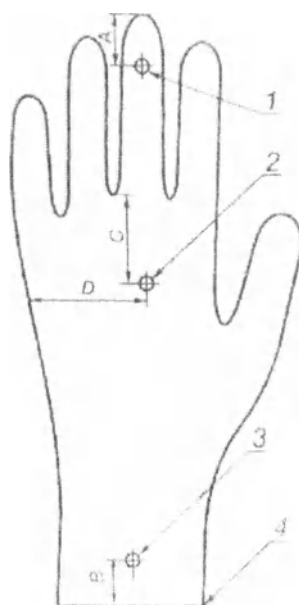1: fingertip; 2: palm; 3: cuff; 4: edge of the cuff.
 $A = (13 \pm 3)$ mm; $B = (25 \pm 5)$ mm; $C = (33 \pm 5)$ mm; $D = (48 \pm 9)$ mm.
 Fig. 2: Points for measuring thickness of the glove

5.2 The required thickness of the cuff is 1.40 ± 1.9 mm.

5.3 Strength characteristics

Strength characteristics

5.3.1 "Standard" glove material shall have the following strength characteristics.

5.3.1.1 Minimum force at break before accelerated ageing is 7 N.

5.3.1.2 Minimum elongation at break before accelerated ageing is 650%.

5.3.1.3 Minimum force at break after accelerated ageing is 6 N.

5.3.1.4 Minimum elongation at break after accelerated ageing is 500%.

5.3.2 "Extra" glove material shall have the following strength characteristics.

5.3.2.1 Minimum force at break before accelerated ageing is 9 N.

5.3.2.2 Minimum elongation at break before accelerated ageing is 750%.

5.3.2.3 Minimum force at break after accelerated ageing is 6 N.

5.3.2.4 Minimum elongation at break after accelerated ageing is 600%.

5.3.3 "Standard" sensitive touch glove material shall have the following strength characteristics.

5.3.3.1 Minimum force at break before accelerated ageing is 7 N.

5.3.3.2 Minimum elongation at break before accelerated ageing is 650 %.

5.3.3.3 Minimum force at break after accelerated ageing is 6 N.

5.3.3.4 Minimum elongation at break after accelerated ageing is 500 %.

5.3.4 "Extra" sensitive touch glove material shall have the following strength characteristics.

5.3.4.1 Minimum force at break before accelerated ageing is 7 N.

5.3.4.2 Minimum elongation at break before accelerated ageing is 650 %.

5.3.4.3 Minimum force at break after accelerated ageing is 6 N.

5.3.4.4 Minimum elongation at break after accelerated ageing is 500 %.

5.3.5 "Standard" high-strength glove material shall have the following strength characteristics.

5.3.5.1 Minimum force at break before accelerated ageing is 9 N.

5.3.5.2 Minimum elongation at break before accelerated ageing is 750 %.

5.3.5.3 Minimum force at break after accelerated ageing is 6 N.

5.3.5.4 Minimum elongation at break after accelerated ageing is 600 %.

5.3.6 "Extra" high-strength glove material shall have the following strength characteristics.

5.3.6.1 Minimum force at break before accelerated ageing is 9 N.

5.3.6.2 Minimum elongation at break before accelerated ageing is 750 %.

5.3.6.3 Minimum force at break after accelerated ageing is 6 N.

5.3.6.4 Minimum elongation at break after accelerated ageing is 700 %.

5.7 Gloves are produced from.

5.7.1 Latex compound:

- Natural rubber latex (CAS 9006-04-6) > 90 %
- Sulfur (CAS 7704-34-9) 1.0-2.0 %
- Zinc oxide (CAS 1314-13-2) 1.0-2.0 %
- Zinc dibutyl dithiocarbamate (ZDBC) (CAS 136-23-2) > 0.5 %
- Zinc diethyl dithiocarbamate (ZDEC) (CAS 14324-55-1) > 0.5 %
- Solvent additive (CAS 9008-63-3) < 1%

5.7.2 The following coloring agents are used in the production of gloves

- white: Titanium Dioxide;
- green: Farsperse Green PG704;
- blue: Farsperse Blue MPL 1442;
- pink: Farsperse Violet PV193;
- black: Farsperse Black PB708RF;
- orange: Farsperse ORANGE PO132E;
- yellow: Farsperse Yellow PY123MA.

5.7.3 The polymer N-Uthane 67 is used for internal polymer coating of gloves.

5.7.4 N-Uthane 58 is used for coating with pH 5.5.

6. Storage and Transportation Conditions

6.1 Transportation of the product can be carried out using all kinds of transport in accordance with the transport regulations applicable to the specific kind of transport in the original factory transport packages (cardboard boxes). The used kind of transport shall be protected from direct sunlight.

The gloves are supposed to retain performance characteristics when used in the following environment conditions: temperature and relative humidity:

- temperature — from +10 to +35 °C;
- relative humidity — up to 80 % at a temperature of 25 °C.

The gloves in transport packaging must be resistant to the following climatic effects:

- temperature — from -30 to +50 °C;
- relative humidity — up to 98 % at a temperature of 35 °C without condensation.

6.2. The gloves shall be stored in a well-ventilated room away from heat sources, direct sunlight, light sources and electrical equipment. Storage temperatures below 10 °C can cause condensation. If the storage temperature drops below 10 °C, warm up the gloves to 25–30 °C before use, otherwise it may be difficult to put them on due to the loss of elasticity.

6.3 The gloves must be stored in a dry place, away from direct sunlight and high humidity at a temperature of 10 °C to 30 °C.

6.4 The gloves must be stored away from ignition sources.

7. Conditions for the Use of a Medical Product

The gloves are used under the following environmental conditions:

- temperature — from +10 to +35 °C; relative humidity — up to 80 % at a temperature of +25 °C.

8. Packaging

8.1 Packaging should provide protection against mechanical and climatic factors during transportation and storage, and also ensure the full use of the carrying capacity (capacity) of vehicles and the convenience of carrying out loading and unloading operations.

The packaging must be sealed. The sterility of the packaging is ensured by compliance with the requirements of ISO 11607-1.

8.2 The packaging of products must comply with the requirements of the manufacturer, as well as conditions of the contract between the manufacturer and an authorized representative.

Internal package

A pair of gloves is packed into an internal package, which shall be big enough to ensure that the glove can be easily put on when opening it.

Individual package

In order to ensure the sterility of the product a pair of gloves is packed into an individual package (single package) together with the internal package.

Consumer (multiple) package

Gloves in individual packages are packed into cardboard boxes. Consumer (outer) package protects the gloves from external mechanical damage and allows to determine size of the gloves visually.

Transport package

Gloves in consumer packages are packed into corrugated cardboard boxes. Cardboard boxes must be hard enough to ensure integrity and sterility of the products during transportation and storage.


8.3 Package dimensions and quantity of products

Internal package dimensions, (H x W), ±5 mm	Individual packaging type	Individual package dimensions, (H x W), ±5 mm	Q-ty of gloves in consumer package	Consumer package dimensions, (L x W x H), ±5 mm	Q-ty of consumer packages per one transport package	Transport package dimensions, (L x W x H), ±5 mm
440x270	Paper Paper	270 x 150	50 pairs	260 x 130 x 200	8 pcs	530 x 270 x 410
400x260	Thermoform	160 x 132	50 pairs	131 x 131 x 210	8 pcs	545 x 275 x 255

9. Labelling

9.1. Symbols used for labelling:

- batch number;
- "Disposable"
- "Sterilized using radiation"
- "The product contains natural latex"
- "Non-toxic materials"
- "Date of manufacturing"
- "Shelf-life..."
- "Temperature range"
- "Avoid exposure to sunlight"
- "Avoid exposure to water"
- "Conformity mark symbol"
- "Acceptable quality level"

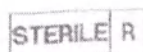(year/month)

(year/month)

AQL 1.5

10. Content of Delivery

No.	Name	Quantity
I	Examination latex gloves TET-A-TET* sterile disposable powder free produced by WRP Asia Pacific SDN. BHD., Malaysia*	1 individual package
II	Instructions for use	1 pc

* – type of gloves is selected from among the presented ones; see Annex.

11. Warranty and Shelf Life

The manufacturer guarantees that the product will have all the characteristics specified in this document during its entire shelf life (5 years), provided that the integrity of the packaging is preserved and conditions of storage, transportation and use comply with the instructions for use.

12. Procedure for Disposal and Destruction of a Medical Product

- Discard gloves in accordance with SanPiN 2.1.7.2790 (Sanitary Rules and Norms)

- The system of collection, temporary storage and transportation of medical waste should include the following stages:
 - collection of waste by organizations performing medical and/or pharmaceutical activities;
 - movement of waste from the units and their temporary storage on the territory of the waste-generating organization;
 - disinfection/deactivation of waste;
 - transportation of waste from the territory of the waste-generating organization;
 - dumping/disposal of medical waste.
- The method of chemical disinfection of waste at the place of its formation is used as a mandatory temporary measure in the absence of an area intended for the processing of medical waste in medical and/or pharmaceutical organizations, or in the absence of a centralized system for the disposal of medical waste on this territory.
- The choice of safe methods of disinfection/deactivation depends on the capabilities and profile of the medical organization, availability of equipment for disinfection/deactivation of waste, as well as on the method of disinfection/deactivation of waste used in this organization (burning, transportation to landfills, disposal).
- Collection, temporary storage and disposal of waste should be carried out in accordance with the scheme for medical waste handling used in this medical and/or pharmaceutical organization.

13. Information about the Producer and Manufacturer of a Medical Product

WRP AsiaPacific SDN. BHD. (WRP AsiaPacific SDN. BHD.)

Address: Lot 1, Jalan 3, Kawasan Perusahaan Bandar Baru Salak Tinggi, 43900 Sepang, Selangor Darul Ehsan, Malaysia

Production site:

WRP AsiaPacific SDN. BHD. (WRP AsiaPacific SDN. BHD.)

Address: Lot 1, Jalan 3, Kawasan Perusahaan Bandar Baru Salak Tinggi, 43900 Sepang, Selangor Darul Ehsan, Malaysia


14. Responsible Supplier

For all questions about the medical product "Examination latex gloves TET-A-TET[®] non-sterile disposable powder free" produced by WRP Asia Pacific SDN. BHD., Malaysia, please contact the authorized manufacturer representative in the Russian Federation "Volenta" LLC, registered and mail address: 61 Vyborgskaya nab., lit. A, St. Petersburg, 197342 Tel: 8 (921) 582-78-23, E-mail: info@volenta-spb.com.

Annex

Examination latex gloves TET-A-TET[®] sterile disposable powder free, produced by WRP Asia Pacific SDN. BHD., Malaysia, types of products:

1. Examination latex gloves TET-A-TET[®] Special Exam Latex sterile disposable powder free, textured on fingers, beaded cuff "Standard", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.
2. Examination latex gloves TET-A-TET[®] Special Exam Latex Polymer sterile disposable powder free, textured on fingers, with polymer coating, beaded cuff "Standard", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.
3. Examination latex gloves TET-A-TET[®] Special Exam Latex Text sterile disposable powder free, textured on fingers, double chlorinated, beaded cuff "Standard", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.
4. Examination latex gloves TET-A-TET[®] Special Exam Latex Single Chlor latex sterile disposable powder free, textured on fingers, single chlorinated, beaded cuff "Standard", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.
5. Examination latex gloves TET-A-TET[®] Special Exam Latex pH5.5 sterile disposable powder free, textured on fingers, beaded cuff, with pH 5.5 "Standard", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.

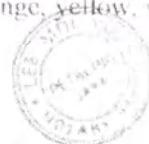6. Examination latex gloves TET-A-TET® Special Exam Latex Micro sterile disposable powder free, textured on fingers, beaded cuff, increased tactile sensitivity "Standard", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.

7. Examination latex gloves TET-A-TET® Special Exam Latex Ortho sterile disposable powder free, textured on fingers, beaded cuff, high risk "Standard", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.

8. Examination latex gloves TET-A-TET® Special Exam Latex Long sterile disposable powder free, textured on fingers, beaded cuff, extended cuff "Standard", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.

9. Examination latex gloves TET-A-TET® Special Exam Latex sterile disposable powder free, textured surface, without bead "Extra", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.

10. Examination latex gloves TET-A-TET® Special Exam Latex Polymer sterile disposable powder free, textured surface, with polymer coating, beaded cuff "Extra", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.

11. Examination latex gloves TET-A-TET® Special Exam Latex Text sterile disposable powder free textured surface, double chlorinated, beaded cuff "Extra", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.

12. Examination latex gloves TET-A-TET® Special Exam Latex Single Chlor sterile disposable powder free, textured surface, single chlorinated, beaded cuff "Extra", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.

13. Examination latex gloves TET-A-TET® Special Exam Latex pH5.5 sterile disposable powder free, textured surface, beaded cuff, with pH5.5 "Extra", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.

14. Examination latex gloves TET-A-TET® Special Exam Latex Micro sterile disposable powder free, textured surface, beaded cuff, increased tactile sensitivity "Extra", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.

15. Examination latex gloves TET-A-TET® Special Exam Latex Ortho sterile disposable powder free, textured surface, beaded cuff, high risk "Extra", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.

16. Examination latex gloves TET-A-TET® Special Exam Latex Long sterile disposable powder free, textured surface, beaded cuff, extended cuff "Extra", colour: white, green, blue, pink, black, orange, yellow, sizes: XS, S, M, L, XL, XXL, XXXL.

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