WRP Asia Pacific Sdn Bhd

1 4 7 8 1 7 V

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
Email customer_wrp@wrpworld.com

Website www.wrpworld.com

Instruction for use of Medical Product

**Surgical nitrile sterile disposable powder free gloves TET-A-TET[®]
with textured surface, produced by WRP Asia Pacific SDN. BHD.,
Malaysia**
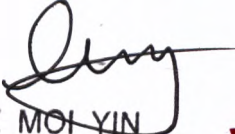
LEE MOI YIN
NOTARY PUBLIC
PETALING JAYA
SELANGOR
MALAYSIA

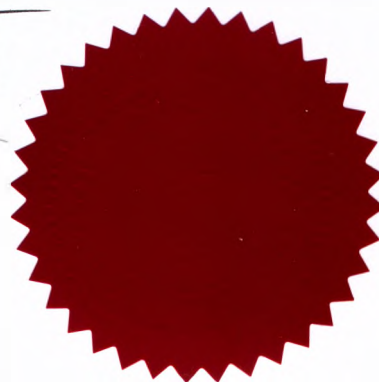
Table of Contents

1.	Name of a Medical Product	3
2.	Field of Application of a Medical Product	3
3.	Application of a Medical Product	3
4.	Description of a Medical Product	6
5.	Technical characteristics	7
6.	Storage and Transportation Conditions	15
7.	Conditions for the Use	16
8.	Packaging	16
9.	Labelling	18
10.	Contents of Delivery	18
11.	Warranty and Shelf Life	19
12.	Procedure for Disposal and Destruction of a Medical Product	19
13.	Information about the Producer and the Manufacturer of a Medical Product	19
14.	Responsible Supplier	19
	Annex	20

1. Name of a Medical Product

Surgical nitrile sterile disposable powder free gloves TET-A-TET® with textured surface, produced by WRP Asia Pacific SDN. BHD., Malaysia

2. Field of Application

2.1. Application

Purpose of a Medical Product determined by the manufacturer:

– the gloves are intended for single-use as a protective barrier when worn by medical personnel in the surgical field.

2.2. Field of Application

Treatment and prophylactic institutions.

3. Application of a Medical product

3.1. Indications for use:

Surgical sterile medical gloves should be used:

- During any surgical intervention, during large and small surgical operations, during small surgical operations on the skin and other manipulations associated with penetration into the tissues under the skin, into the mucous membranes;

- For protection of medical personnel and patients during the manipulations related to access to sterile areas of the body (vessels, genitourinary tract, tracheobronchial tree), rectal, vaginal examination, contact with mucous membranes, etc.;

- when preparing solutions for the total parenteral insertion;

- when covering a sterile table and laying out instruments and endoscopes on a sterile sheet after sterilization;

- for the protection of medical personnel and patients, when there is a risk of infection from direct contact with the patient (its biological area), or in case of contact with mucous membranes, damaged skin, blood or other biological substrates that are potentially or clearly contaminated by microorganisms.

When choosing the gloves for maximum comfortable use of the product, it is recommended to consider their size.

Attention! Medical gloves do not guarantee 100 % hands protection from puncture.

Gloves are indicated for single use.

3.2. Contradictions

Dermatitis

3.3. Side effects

No

3.4. Rules for utilization of a Medical Product

- Gloves should be put on at every approach associated with penetration into subcutaneous tissues, into mucous membranes, during manipulation in sterile boxes and within sterile areas, materials and instruments, when there is a contact with blood, other biological substrates and objects in order to reduce the risk of contamination by microorganisms.
- The use of gloves does not replace hand hygiene. Hand treatment should be carried out in accordance with the manipulation mode.
- It is necessary to choose the right size gloves.
- It is necessary to choose the gloves in accordance to the medical procedure – they should be easily worn and should not impede movements or squeeze hands.
- When putting on gloves, no significant efforts should be made
- Special techniques of putting on gloves should be used, that will ensure their sterility.
- Before putting on gloves for performing medical procedures, it is necessary to prepare a workplace - ready-to-use sterile tools and consumables should rest on the manipulation table. It is necessary to ensure their availability without compromising the sterility of the hands.
- The use of an extra pair of gloves reduces the frequency of glove damage by 2-4 times when working with sharp tools.
- Do not use gloves with damaged packaging, as their sterility may be impaired.
- After removing the gloves, wash hands with soap and treat them with a dermal antiseptic.

3.5. Application of a Medical Product

Before using gloves, you must ensure the integrity of the package. Then open the secondary packaging. The primary glove pack is sterile. In order to open the primary packaging, get gloves and start using them for the intended purpose, it is necessary to treat the hands with an antiseptic.

3.5.1. Putting on gloves


When putting on gloves, it is recommended to follow the techniques described below to avoid contamination of their external surfaces during the donning process. There are two techniques for putting on gloves: open and closed. With any technique of putting on gloves, the most important thing is not to allow contamination of their outer surface.

Steps of putting on gloves by open technique

- Open the sterile package and take both gloves with the fingertips of the left hand from the inside of the left glove so that the tip of the thumb is inside the right glove and keep them together in an upright position. Make sure that the fingers touch only the inner surface of the gloves.
- Pass the gloves with your right hand into the cuff, holding the gloves with your left hand.
- Tighten the folded glove until the right hand is in the glove.
- Lower the fingers of the right hand in the glove into the cuff of the left glove and tighten it on the arm.
- Using four fingers, tighten the cuff of the left glove over the cuff of the medical gown.
- Place the fingers of the left hand in the glove under the folded cuff of the glove and pull it onto the cuff of the medical gown.
- Tighten the cuff with four fingers and finally fix the gloves, ensuring their comfortable use.

Steps of putting on gloves by closed technique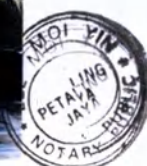

- Open the secondary packaging before washing hands. The primary packaging is sterile. Before opening the package and removing gloves, you must wear a sterile surgical gown.
- Pulling the sleeves of the dressing gown on your hands, pull out your left glove with your right hand. Hold your hand palm up with straight fingers. Put the glove on your left wrist in such a way that its fingers are pointed towards you, and the thumb of the glove is on the wrist. Press the cuff with the thumb of the left hand.
- Place the thumb inside the cuff from the edge. Squeeze your right hand into a fist and pull the glove over your fingers.
- Having straightened the fingers of your left hand and holding the cuff with your right hand, tighten the glove with the four fingers of your right hand until it is put on the fingers of your left hand.
- Repeat the procedure with another glove. Particularly: with your left hand in the glove put the right glove on the wrist of the right hand so that fingers are directed towards you, and the thumb of the glove is on the wrist. Press the cuff with the thumb of the right hand.
- Place the left thumb in the cuff from the edge. Squeeze your right hand into a fist and pull the glove over your fingers.
- Having straightened the fingers of the right hand and covering the hand with the cuff of the medical sterile dressing gown, use four fingers of the left hand to tighten the glove until it is put on the fingers of the right hand.
- Now that gloves are worn on both hands, they can fitted on be without the risk of contamination.
- To avoid slipping of the gloves from the cuffs of the dressing gown, it is necessary to ensure sufficient length of the sleeves of the medical gown and sufficient length of the cuffs to the base of the palm, as well as the absence of folds of the cuff of the gloves.

3.5.2. Requirements to the work in gloves

- It is necessary to change gloves when going from "dirty manipulation" to "clean", as well as from one patient to another.
- It is necessary to change gloves:
 - after detecting a visible defect;
 - after a possible or actual puncture with a needle, suture material, bone, etc.;
 - immediately after direct contact with uncured methyl methacrylate (bone cement);
 - after an accidental electric shock in the arm by the electrosurgical instrument;
 - If the glove begins to swell, stretch, stick or not seated on the hand as a result of the absorption of liquids or lubricants by the material
- The gloves are not treated during manipulation. If necessary, the gloves should be replaced.
- Torn or leaking gloves are not suitable for use. If one glove is damaged, you must immediately replace both, because you can not remove one glove without contaminating the other.
- Remove gloves very carefully, avoiding contamination of the skin of hands and splashing of liquids.
- Treatment of disposable latex gloves with alcohol-containing solutions is not recommended, since these solutions help to weaken the protective layer of gloves.
- Strictly observe the safety measures when working with sharp objects and tools.

3.5.3. Taking off the gloves

It is recommended to remove gloves according to the following algorithm:

- Wash your hands in gloves in a special solution to wash away blood or other biological fluids.
- Take one of the gloves behind the cuff and pull it out partially. The glove will turn inside out. It is important to hold the first glove on the hand before removing the second glove to avoid contact of the outer surface of the glove with the skin of the hands.
- Leaving the first glove on the fingers, take the second glove near the cuff and pull it over the edge. The glove will turn inside out. It is important to keep the second glove partially on the hand to avoid contact of the outer surface of the glove with the skin of the hands.
- At the same time, remove both gloves, gently touching only the inner surface of the gloves with your hands.
- Turning the glove inside out, use it as a bag for another glove.

- Because surgical gloves are only recommended for single use, remove them properly. Wash hands immediately after removing gloves, as gloves may have punctures or cuts that leave you at risk of contamination with contaminated blood and other body fluids.

4. Product Description

4.1 Gloves are made from synthetic nitrile.

4.2 There are following types of gloves depending on the coating:

- powder-free polymer coated gloves with textured surface;
- powder-free double-chlorinated gloves with textured surface.

4.3 Textured is the entire surface of a glove (fingers and palm).

4.4 The polymer coating offers additional strength to gloves, makes it significantly easier to put them on, and also prevents gloves from sticking together.

The gloves can be subjected to chlorine gas treatment (chlorination) by immersion in a chlorine solution. Double chlorination (on both sides).

4.5 The gloves can be made in different colours.

4.6 The gloves can be made with or without beaded cuffs.

4.7 There are ten standard sizes of gloves:

- size 5.0;
- size 5.5;
- size 6.0;
- size 6.5;
- size 7.0;
- size 7.5;
- size 8.0;
- size 8.5;
- size 9.0;
- size 9.5.

4.8 The products are sterile.

4.9 Gloves containing two pairs are used for protection of medical personnel and patients during a situation of a high infectious risk. This type of gloves is produced in two colors. Not only they ensure reliable protection due to their high density, but also, they have an indicator system: during the contact with blood or rinsing fluid in a cut, puncture or mechanically damaged surface there is appeared a contrasting spot.

Packaging of gloves for use in the double puncture indicator system: two pairs of gloves in sterile packaging. The inner pair of gloves is green. The outer pair of gloves is white. The inner pair of gloves is ½ size smaller than the outer pair.

4.10 The products are intended for single use only.

4.11 Gloves are classified as Class IIa according to classification by the potential application risk.

5. Technical characteristics

5.1 Width, length and thickness of gloves depending on their size:

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile sterile disposable powder free with textured surface, with polymer coating, beaded cuff "Standard" shall be as specified in Table 1.

Table 1. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.16±0.01. Palm thickness – 0.15±0.01. Cuff thickness – 0.14±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Beadless sterile disposable powder free with textured surface, with polymer coating, without bead "Standard" shall be as specified in Table 2.

Table 2. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.16±0.01. Palm thickness – 0.15±0.01. Cuff thickness – 0.14±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Double Chlor sterile disposable powder free with textured surface, double chlorinated, beaded cuff "Standard" shall be as specified in Table 3.

Table 3. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.17±0.01. Palm thickness – 0.16±0.01. Cuff thickness – 0.15±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Micro sterile disposable powder free with textured surface, with polymer coating, increased tactile sensitivity, applicable for microsurgery, beaded cuff "Standard" shall be as specified in Table 4.

Table 4. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.15±0.01. Palm thickness – 0.14±0.01. Cuff thickness – 0.14±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Micro Beadless sterile disposable powder free with textured surface, with polymer coating, increased tactile sensitivity, applicable for microsurgery, without bead “Standard” shall be as specified in Table 5.

Table 5. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.15±0.01. Palm thickness – 0.14±0.01. Cuff thickness – 0.14±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Ortho sterile disposable powder free with textured surface, with polymer coating, high risk, applicable for orthopedics and traumatology, beaded cuff “Standard” shall be as specified in Table 6.

Table 6. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.18±0.01 Palm thickness – 0.15±0.01. Cuff thickness – 0.14±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Ortho Beadless sterile disposable powder free with textured surface, with polymer coating, high risk, applicable for orthopedics and traumatology, without bead “Standard” shall be as specified in Table 7.

Table 7. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.18±0.01 Palm thickness – 0.15±0.01. Cuff thickness – 0.14±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Dual sterile disposable powder free with textured surface, with polymer coating, high risk, for puncture indication, 2 pairs in an individual packing: internal pair of gloves is green with beaded cuff; external pair of gloves is white with beaded cuff “Standard”, shall be as specified in Table 8.

Table 8. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Inner glove: Finger thickness – 0.15±0.01. Palm thickness – 0.15±0.01. Cuff thickness – 0.14±0.01. Outer glove: Finger thickness – 0.16±0.01. Palm thickness – 0.16±0.01. Cuff thickness – 0.14±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Dual Beadless sterile disposable powder free with textured surface, with polymer coating, high risk, for puncture indication, 2 pairs in an individual packing: internal pair of gloves is green without beaded cuff; external pair of gloves is white without beaded cuff “Standard”, shall be as specified in Table 9.

Table 9. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Inner glove: Finger thickness – 0.15±0.01. Palm thickness – 0.15±0.01. Cuff thickness – 0.14±0.01. Outer glove: Finger thickness – 0.16±0.01. Palm thickness – 0.16±0.01. Cuff thickness – 0.14±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Gyno sterile disposable powder free with textured surface, with polymer coating, extended cuff, beaded cuff “Standard” shall be as specified in Table 10.

Table 10. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	485±10	Finger thickness – 0.16±0.01. Palm thickness – 0.16±0.01. Cuff thickness – 0.14±0.01.
5.5	70±6	485±10	
6.0	76±6	485±10	
6.5	83±6	485±10	
7.0	89±6	485±10	
7.5	95±6	485±10	
8.0	102±6	485±10	
8.5	108±6	485±10	
9.0	114±6	485±10	
9.5	121±6	485±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Gyno Beadless sterile disposable powder free with textured surface, with polymer coating, extended cuff, without bead “Standard” shall be as specified in Table 11.

Table 11. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	485±10	Finger thickness – 0.16±0.01. Palm thickness – 0.16±0.01. Cuff thickness – 0.14±0.01.
5.5	70±6	485±10	
6.0	76±6	485±10	
6.5	83±6	485±10	
7.0	89±6	485±10	
7.5	95±6	485±10	
8.0	102±6	485±10	
8.5	108±6	485±10	
9.0	114±6	485±10	
9.5	121±6	485±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile sterile disposable powder free with textured surface, with polymer coating, beaded cuff “Extra” shall be as specified in Table 12.

Table 12. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.17±0.01. Palm thickness – 0.16±0.01. Cuff thickness – 0.15±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Beadless sterile disposable powder free with textured surface, with polymer coating, without bead “Extra” shall be as specified in Table 13.

Table 13. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.17±0.01. Palm thickness – 0.16±0.01. Cuff thickness – 0.15±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Double Chlor sterile disposable powder free with textured surface, double chlorinated, beaded cuff “Extra” shall be as specified in Table 14.

Table 14. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.18±0.01. Palm thickness – 0.17±0.01. Cuff thickness – 0.16±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Micro sterile disposable powder free with textured surface, with polymer coating, increased tactile sensitivity, applicable for microsurgery, beaded cuff “Extra” shall be as specified in Table 15.

Table 15. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.16±0.01. Palm thickness – 0.15±0.01. Cuff thickness – 0.15±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Micro Beadless sterile disposable powder free with textured surface, with polymer coating, increased tactile sensitivity, applicable for microsurgery, without bead “Extra” shall be as specified in Table 16.

Table 16. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.16±0.01. Palm thickness – 0.15±0.01. Cuff thickness – 0.15±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Ortho sterile disposable powder free with textured surface, with polymer coating, high risk, applicable for orthopedics and traumatology, beaded cuff “Extra” shall be as specified in Table 17.

Table 17. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.20±0.02 Palm thickness – 0.17±0.02. Cuff thickness – 0.15±0.02.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Ortho Beadless sterile disposable powder free with textured surface, with polymer coating, high risk, applicable for orthopedics and traumatology, without bead “Extra” shall be as specified in Table 18.

Table 18. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Finger thickness – 0.20±0.02 Palm thickness – 0.17±0.02. Cuff thickness – 0.15±0.02.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Dual sterile disposable powder free with textured surface, with polymer coating, high risk, for puncture indication, 2 pairs in an individual packing: internal pair of gloves is green with beaded cuff; external pair of gloves is white beaded cuff “Extra”, shall be as specified in Table 19.

Table 19. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Inner glove: Finger thickness – 0.16±0.01. Palm thickness – 0.16±0.01. Cuff thickness – 0.15±0.01. Outer glove: Finger thickness – 0.17±0.01. Palm thickness – 0.17±0.01. Cuff thickness – 0.15±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Dual Beadless sterile disposable powder free with textured surface, with polymer coating, high risk, for puncture indication, 2 pairs in an individual packing: internal pair of gloves is green without beaded cuff; external pair of gloves is white without beaded cuff “Extra”, shall be as specified in Table 20.

Table 20. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	290±10	Inner glove: Finger thickness – 0.16±0.01. Palm thickness – 0.16±0.01. Cuff thickness – 0.15±0.01. Outer glove: Finger thickness – 0.17±0.01. Palm thickness – 0.17±0.01. Cuff thickness – 0.15±0.01.
5.5	70±6	290±10	
6.0	76±6	290±10	
6.5	83±6	290±10	
7.0	89±6	290±10	
7.5	95±6	290±10	
8.0	102±6	290±10	
8.5	108±6	290±10	
9.0	114±6	290±10	
9.5	121±6	290±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Gyno sterile disposable powder free with textured surface, with polymer coating, extended cuff, beaded cuff “Extra” shall be as specified in Table 21.

Table 21. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	485±10	Finger thickness – 0.17±0.01. Palm thickness – 0.17±0.01. Cuff thickness – 0.15±0.01.
5.5	70±6	485±10	
6.0	76±6	485±10	
6.5	83±6	485±10	
7.0	89±6	485±10	
7.5	95±6	485±10	
8.0	102±6	485±10	
8.5	108±6	485±10	
9.0	114±6	485±10	
9.5	121±6	485±10	

- Width, length and thickness of surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Gyno Beadless sterile disposable powder free with textured surface, with polymer coating, extended cuff, without beaded cuff “Extra” shall be as specified in Table 22.

Table 22. Width, length and thickness of gloves.

Size	Width W, (Fig.1), mm	Length L, (Fig. 1), mm	Thickness (at the points shown in Fig. 2), mm
5.0	67±4	485±10	Finger thickness – 0.17±0.01. Palm thickness – 0.17±0.01. Cuff thickness – 0.15±0.01.
5.5	70±6	485±10	
6.0	76±6	485±10	
6.5	83±6	485±10	
7.0	89±6	485±10	
7.5	95±6	485±10	
8.0	102±6	485±10	
8.5	108±6	485±10	
9.0	114±6	485±10	
9.5	121±6	485±10	

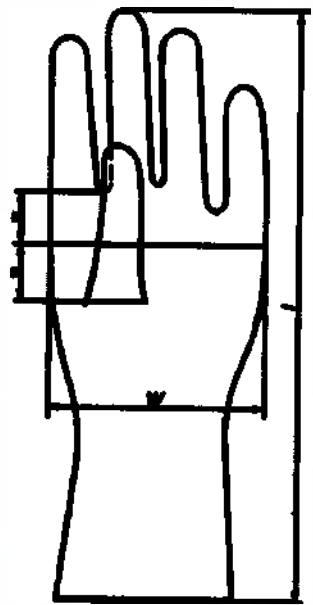
Fig. 1: Measurement points for width and length

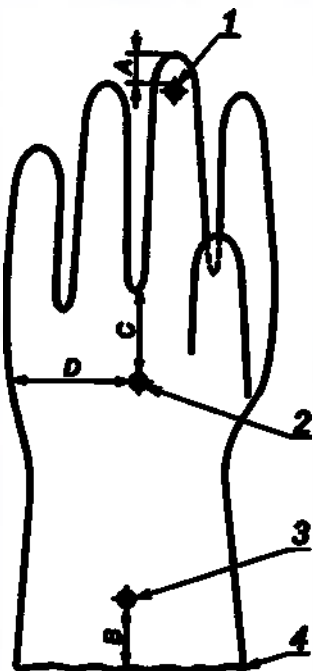
Fig. 2: Points for measuring thickness of the glove

1: fingertip; 2: palm; 3: cuff; 4: edge of the cuff;
A = (13±3) mm; B = (25±5) mm; C = (33±5) mm; D = (48±9) mm.

- 5.2 The required thickness of the cuff is 1.9÷2.1 mm.
- 5.3 Strength characteristics
- 5.3.1 "Standard" glove material shall have the following strength characteristics.
- 5.3.1.1 Minimum force at break before accelerated ageing is 9 N.
- 5.3.1.2 Minimum elongation at break before accelerated ageing is 600 %.
- 5.3.1.3 Minimum force at break after accelerated ageing is 9 N.
- 5.3.1.4 Minimum elongation at break after accelerated ageing is 550 %.
- 5.3.1.5 The effort necessary to achieve elongation by 300 % before accelerated aging should be no more than 3.0 N.
- 5.3.2 "Extra" glove material shall have the following strength characteristics.
- 5.3.2.1 Minimum force at break before accelerated ageing is 9 N.
- 5.3.2.2 Minimum elongation at break before accelerated ageing is 600 %.
- 5.3.2.3 Minimum force at break after accelerated ageing is 9 N.
- 5.3.2.4 Minimum elongation at break after accelerated ageing is 550 %.
- 5.3.2.5 The effort necessary to achieve elongation by 300 % before accelerated aging should be no more than 3.0 N.
- 5.3.3 "Standard" sensitive touch glove material shall have the following strength characteristics.
- 5.3.3.1 Minimum force at break before accelerated ageing is 9 N.
- 5.3.3.2 Minimum elongation at break before accelerated ageing is 600 %.
- 5.3.3.3 Minimum force at break after accelerated ageing is 9 N.
- 5.3.3.4 Minimum elongation at break after accelerated ageing is 550 %.
- 5.3.3.5 The effort necessary to achieve elongation by 300 % before accelerated aging should be no more than 3.0 N.
- 5.3.4 "Extra" sensitive touch glove material shall have the following strength characteristics.
- 5.3.4.1 Minimum force at break before accelerated ageing is 9 N.
- 5.3.4.2 Minimum elongation at break before accelerated ageing is 600 %.
- 5.3.4.3 Minimum force at break after accelerated ageing is 9 N.
- 5.3.4.4 Minimum elongation at break after accelerated ageing is 550 %.
- 5.3.4.5 The effort necessary to achieve elongation by 300% before accelerated aging should be no more than 3.0 N.
- 5.3.5 "Standard" high-strength glove material shall have the following strength characteristics.
- 5.3.5.1 Minimum force at break before accelerated ageing is 9 N.
- 5.3.5.2 Minimum elongation at break before accelerated ageing is 600 %.
- 5.3.5.3 Minimum force at break after accelerated ageing is 9 N.
- 5.3.5.4 Minimum elongation at break after accelerated ageing is 550 %.
- 5.3.5.5 The effort necessary to achieve elongation by 300% before accelerated aging should be no more than 3.0 N.
- 5.3.6 "Extra" high-strength glove material shall have the following strength characteristics.
- 5.3.6.1 Minimum force at break before accelerated ageing is 9 N.
- 5.3.6.2 Minimum elongation at break before accelerated ageing is 600 %.
- 5.3.6.3 Minimum force at break after accelerated ageing is 9 N.
- 5.3.6.4 Minimum elongation at break after accelerated ageing is 550 %.
- 5.3.6.5 The effort necessary to achieve elongation by 300 % before accelerated aging should be no more than 3.0 N.
- 5.4 The amount of residual powdering substance may not exceed 2 mg per glove.
- 5.5 The gloves are sterilized using gamma radiation.
- 5.6 Gloves are produced from:
- 5.6.1 Nitrile compound:
- NBR Synthetic Latex (CAS 9003-18-3) $\geq 70,40$ %
 - Sulfur (CAS 7704-34-9) 1.0-2.0 %
 - Zinc oxide (CAS 1314-13-2) 1.0-2.0 %
 - Zinc dibutyl dithiocarbamate (ZDBC) (CAS 136-23-2) > 0.5 %
 - Zinc diethyl dithiocarbamate (ZDEC) (CAS 14324-55-1) > 0.5 %
 - Solvent additive (CAS 9008-63-3) < 1 %
- 5.6.2 The following colouring agents are used in the production of gloves:
- White: Titanium Dioxide;
 - Green: Farsperse Green PG704;
 - Blue: Farsperse Blue MPL 1442;
 - Brown: Farsperse Brown PB510.
- 5.6.3 The polymer N-Uthane 67 is used for internal polymer coating of gloves.

6. Storage and Transportation Conditions

6.1 Transportation of the product can be carried out using all kinds of transport in accordance with the transport regulations applicable to the specific kind of transport in the original factory transport packages (cardboard boxes) in transport protected from direct sunlight.

The gloves are supposed to retain performance characteristics when used in the following environment conditions: temperature and relative humidity:

temperature — from +10 to +35 °C;

relative humidity — up to 80 % at a temperature of 25 °C.

The gloves in transport packaging must be resistant to the following climatic effects:

temperature — from -30 to +50 °C;

relative humidity — up to 98 % at a temperature of 35 °C without condensation.

6.2 The gloves shall be stored in a well-ventilated room away from heat sources, direct sunlight, light sources and electrical equipment. Storage temperatures below 10 °C can cause condensation. If the storage temperature drops below 10 °C, warm up the gloves to 25–30 °C before use, otherwise it may be difficult to put them on due to the loss of elasticity.

6.3 The gloves must be stored in a dry place, away from direct sunlight and high humidity at a temperature of 10 °C to 30 °C.

6.4 The gloves must be stored away from ignition sources.

7. Conditions for the Use of a Medical Product

The gloves are used under the following environmental conditions:

temperature — from +10 to +35 °C; relative humidity — up to 80 % at a temperature of +25 °C.

8. Packaging

8.1 The package must provide protection of the product against mechanical and climatic factors during its transportation and storage.

The package must be sealed. The sterility of the package is ensured by compliance with the requirements of ISO 11607-1.

8.2 The packaging of products must comply with the requirements of the manufacturer, as well as conditions of the contract between the manufacturer and an authorized representative.

Internal package

A pair of gloves is packed into an internal package, which shall be big enough to ensure that the glove can be easily put on when opening it.

Individual package

In order to ensure the sterility of the product a pair of gloves is packed into an individual package (single package) together with the internal package.

Consumer (multiple) package

Gloves in individual packages are packed into cardboard boxes. Consumer (outer) package protects the gloves from external mechanical damage and allows to determine size of the gloves visually.

Transport package

Gloves in consumer packages are packed into corrugated cardboard boxes. Cardboard boxes must be hard enough to ensure integrity and sterility of the products during transportation and storage.

8.3 Package dimensions and quantity of the following products:

8.3.1 Package dimensions and quantity for following types of gloves:

– Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile sterile disposable powder free with textured surface, with polymer coating, beaded cuff “Standard”, colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

– Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Beadless sterile disposable powder free with textured surface, with polymer coating, without bead “Standard”, colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

– Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Double Chlor sterile disposable powder free with textured surface, double chlorinated, beaded cuff “Standard”, colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

– Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Micro sterile disposable powder free with textured surface, with polymer coating, increased tactile sensitivity, applicable for microsurgery, beaded cuff “Standard”, colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

– Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Micro Beadless sterile disposable powder free with textured surface, with polymer coating, increased tactile sensitivity, applicable for microsurgery, without bead “Standard”, colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.


– Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Ortho sterile disposable powder free with textured surface, with polymer coating, high risk, applicable for traumatology and orthopedics, beaded cuff “Standard”, colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Ortho Beadless sterile disposable powder free with textured surface, with polymer coating, high risk, applicable for traumatology and orthopedics, without bead “Standard”, colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile sterile disposable powder free with textured surface, with polymer coating, beaded cuff “Extra”, colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Beadless sterile disposable powder free with textured surface, with polymer coating, without bead “Extra”, colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Double Chlor sterile disposable powder free with textured surface, double chlorinated, beaded cuff “Extra”, colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Micro sterile disposable powder free with textured surface, with polymer coating, increased tactile sensitivity, applicable for microsurgery, beaded cuff “Extra”, colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Micro Beadless sterile disposable powder free with textured surface, with polymer coating, increased tactile sensitivity, applicable for microsurgery, without bead “Extra”, colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Ortho sterile disposable powder free with textured surface, with polymer coating, high risk, applicable for traumatology and orthopedics, beaded cuff “Extra”, colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Ortho Beadless sterile disposable powder free with textured surface, with polymer coating, high risk, applicable for traumatology and orthopedics, without bead “Extra”, colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

Internal package dimensions, (H x W), ±5 mm	Individual packaging type	Individual package dimensions, (H x W), ±5 mm	Q-ty of gloves in consumer package	Consumer package dimensions, (L x W x H), ±5 mm	Q-ty of consumer packages per one transport package	Transport package dimensions, (L x W x H), ±5 mm
440x270	Paper Paper	270 x 150	50 pairs	260 x 130 x 200	8 pcs	530 x 270 x 410
400x260	Thermoform	160 x 132	50 pairs	131x 131 x 240	8 pcs	545 x 275 x 255

8.3.2 Package dimensions and quantity for following types of gloves::

- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Dual sterile disposable powder free with textured surface, with polymer coating, high risk, for puncture indication, 2 pairs in an individual packing: internal pair of gloves is green with beaded cuff, external pair of gloves is white with beaded cuff “Standard”, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Dual Beadless sterile disposable powder free with textured surface, with polymer coating, high risk, for puncture indication, 2 pairs in an individual packing: internal pair of gloves is green without beaded cuff; external pair of gloves is white without beaded cuff “Standard”, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Gyno sterile disposable powder free with textured surface, with polymer coating, with extended cuff, beaded cuff “Standard”, colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Gyno Beadless sterile disposable powder free with textured surface, with polymer coating, with extended cuff, without bead “Standard”, colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Dual sterile disposable powder free with textured surface, with polymer coating, high risk, for puncture indication, 2 pairs in an individual packing: internal pair of gloves is green with beaded cuff, external pair of gloves is white with beaded cuff “Extra”, sizes 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
- Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Dual Beadless sterile disposable powder free with textured surface, with polymer coating, high risk, for puncture indication, 2 pairs in an individual packing: internal pair of gloves is green without beaded cuff; external pair of gloves is white without beaded cuff “Extra”, sizes 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

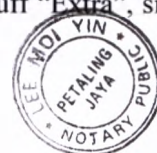– Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Gyno sterile disposable powder free with textured surface, with polymer coating, with extended cuff, beaded cuff “Extra”, colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

– Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Gyno Beadless sterile disposable powder free with textured surface, with polymer coating, with extended cuff, without bead “Extra”, colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

Internal package dimensions, (H x W), ±5 mm	Individual packaging type	Individual package dimensions, (H x W), ±5 mm	Q-ty of gloves in consumer package	Consumer package dimensions, (L x W x H), ±5 mm	Q-ty of consumer packages per one transport package	Transport package dimensions, (L x W x H), ±5 mm
440x270	Paper Paper	360 x 150	25 pairs	360 x 150 x 115	4 pcs.	370 x 310 x 240
400x260	Thermoform	180 x 132	25 pairs	131 x 131 x 200	4 pcs.	270 x 275 x 210

9. Labelling

9.1 Symbols used for labelling:.

- batch number:

- "Disposable"

- "Sterilized using radiation"

"This product does not contain natural latex"

- "Non-toxic materials"

- "Date of manufacturing"

(year/month)

- "Shelf life ..."

(year/month)

- "Temperature range"

- "Avoid exposure to sunlight"

- "Avoid exposure to water"

- "Conformity mark symbol"

- "Acceptable quality level"

AQL 1.5

10. Contents of delivery

No.	Name	Quantity
I	Surgical nitrile sterile disposable powder free gloves TET-A-TET® with textured surface, produced by WRP Asia Pacific SDN. BHD., Malaysia*	1 individual package
II	Instructions for use	1 pc

* – type of gloves is selected by the Customer from among the presented ones: ref. Annex

11. Warranty and Shelf Life

The manufacturer guarantees that the product will have all the characteristics specified in this document during its entire shelf life (5 years), provided that the integrity of the packaging is preserved and conditions of storage, transportation and use comply with the instructions for use.

12. Procedure for Disposal and Destruction of a Medical Product

- The system of collection, temporary storage and transportation of medical waste should include the following stages:
 - collection of waste by organizations performing medical and/or pharmaceutical activities;
 - movement of waste from the units and their temporary storage on the territory of the waste-generating organization;
 - disinfection/deactivation of waste;
 - transportation of waste from the territory of the waste-generating organization;
 - dumping/disposal of medical waste.
- The method of chemical disinfection of waste at the place of its formation is used as a mandatory temporary measure in the absence of an area intended for the processing of medical waste in medical and/or pharmaceutical organizations, or in the absence of a centralized system for the disposal of medical waste on this territory.
- The choice of safe methods of disinfection/deactivation depends on the capabilities and profile of the medical organization, availability of equipment for disinfection/deactivation of waste, as well as on the method of disinfection/deactivation of waste used in this organization (burning, transportation to landfills, disposal).
- Collection, temporary storage and disposal of waste should be carried out in accordance with the scheme for medical waste handling used in this medical and/or pharmaceutical organization.

13. Information about the Producer and Manufacturer of a Medical Product

Manufacturer:

WRP AsiaPacific SDN. BHD. (WRP AsiaPacific SDN. BHD.)

Address: Lot 1, Jalan 3, Kawasan Perusahaan Bandar Baru Salak Tinggi, 43900 Sepang, Selangor Darul Ehsan, Malaysia

Production site:

WRP AsiaPacific SDN. BHD. (WRP AsiaPacific SDN. BHD.)

Address: Lot 1, Jalan 3, Kawasan Perusahaan Bandar Baru Salak Tinggi, 43900 Sepang, Selangor Darul Ehsan, Malaysia

14. Responsible Supplier

For all questions about the medical product "Surgical latex gloves disposable powder free TET-A-TET®", with textured surface, produced by WRP Asia Pacific SDN. BHD., Malaysia", please contact the authorized manufacturer representative in the Russian Federation "Volenta" LLC, registered and mail address: 61 Vyborgskaya nab., lit. A, St. Petersburg, 197342 Tel: 8 (921) 582-78-23. E-mail: info@volenta-spb.com.

Annex

Surgical nitrile sterile disposable powder free gloves TET-A-TET® with textured surface, produced by WRP Asia Pacific SDN. BHD., Malaysia, types of products:

1. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile sterile disposable powder free with textured surface, with polymer coating, beaded cuff "Standard", colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
2. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Beadless sterile disposable powder free with textured surface, with polymer coating, without bead "Standard", colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
3. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Double Chlor sterile disposable powder free with textured surface, double chlorinated, beaded cuff "Standard", colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
4. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Micro sterile disposable powder free with textured surface, with polymer coating, increased tactile sensitivity, applicable for microsurgery, beaded cuff "Standard", colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
5. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Micro Beadless sterile disposable powder free with textured surface, with polymer coating, increased tactile sensitivity, applicable for microsurgery, without bead "Standard", colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
6. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Ortho sterile disposable powder free with textured surface, with polymer coating, high risk, applicable for traumatology and orthopedics, beaded cuff "Standard", colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
7. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Ortho Beadless sterile disposable powder free with textured surface, with polymer coating, high risk, applicable for traumatology and orthopedics, without bead "Standard", colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
8. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Dual sterile disposable powder free with textured surface, with polymer coating, high risk, for puncture indication, 2 pairs in an individual packing: internal pair of gloves is green with beaded cuff, external pair of gloves is white with beaded cuff "Standard", sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
9. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Dual Beadless sterile disposable powder free with textured surface, with polymer coating, high risk, for puncture indication, 2 pairs in an individual packing: internal pair of gloves is green without beaded cuff; external pair of gloves is white without beaded cuff "Standard", sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
10. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Gyno sterile disposable powder free with textured surface, with polymer coating, with extended cuff, beaded cuff "Standard", colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
11. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Gyno Beadless sterile disposable powder free with textured surface, with polymer coating, with extended cuff, without bead "Standard", colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
12. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile sterile disposable powder free with textured surface, with polymer coating, beaded cuff "Extra", colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
13. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Beadless sterile disposable powder free with textured surface, with polymer coating, without bead "Extra", colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
14. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Double Chlor sterile disposable powder free with textured surface, double chlorinated, beaded cuff "Extra", colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
15. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Micro sterile disposable powder free with textured surface, with polymer coating, increased tactile sensitivity, applicable for microsurgery, beaded cuff "Extra", colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
16. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Micro Beadless sterile disposable powder free with textured surface, with polymer coating, increased tactile sensitivity, applicable for microsurgery, without bead "Extra", colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.
17. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Ortho sterile disposable powder free with textured surface, with polymer coating, high risk, applicable for traumatology and orthopedics, beaded cuff "Extra", colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

18. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Ortho Beadless sterile disposable powder free with textured surface, with polymer coating, high risk, applicable for traumatology and orthopedics, without bead "Extra", colour: white, green, blue, brown, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

19. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Dual sterile disposable powder free with textured surface, with polymer coating, high risk, for puncture indication, 2 pairs in an individual packing: internal pair of gloves is green with beaded cuff, external pair of gloves is white with beaded cuff "Extra", sizes 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

20. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Dual Beadless sterile disposable powder free with textured surface, with polymer coating, high risk, for puncture indication, 2 pairs in an individual packing: internal pair of gloves is green without beaded cuff; external pair of gloves is white without beaded cuff "Extra", sizes 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

21. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Gyno sterile disposable powder free with textured surface, with polymer coating, with extended cuff, beaded cuff "Extra", colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

22. Surgical nitrile gloves TET-A-TET® SurgiClassic Nitrile Gyno Beadless sterile disposable powder free with textured surface, with polymer coating, with extended cuff, without bead "Extra", colour: white, green, blue, sizes: 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5.

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