Germany

LeMaitre Vascular GmbH
Otto-Volger-Str. 5a/b
65843 Sulzbach/Ts.

Tel: +49-(0)6196-659230
Fax: +49-(0)6196-527072

NASDAQ: LMAT
www.lemaitre.com

France • LeMaitre Vascular SAS
Tel: +33-(0)344 260041 • Fax: +33-(0)344 260731
Italy • LeMaitre Vascular S.r.l.
Tel: +39-02 9884851 • Fax: +39-02 98497059

Spain • LeMaitre Vascular Spain S.L.
Tel: +34-(0)91 5030040 • Fax: +34-(0)91 1849871

Switzerland • LeMaitre Vascular Switzerland GmbH
Tel: 0800 561761 • Fax: +41-(0)415 608236

Sweden • LeMaitre Vascular GmbH Norden Filial
Tel: +46-35 105620 • Fax: +46-35 105625

Japan • LeMaitre Vascular, GK
Tel: +81-(0)3-5215-5681 • Fax: +81-(0)3-5215-5682

Canada • LeMaitre Vascular ULC
Tel: +1-905 673-2266-0 • Fax: +1-905 673-2223

United States • LeMaitre Vascular, Inc.
Tel: +1-781 221-2266 • Fax: +1-781 221-2223

Vein Stripper InvisiGrip

Instructions for Use

Vein Stripper InvisiGrip

12/2011

Germany

LeMaitre Vascular GmbH
Otto-Volger-Str. 5a/b
65843 Sulzbach/Ts.

Tel: +49-(0)6196-659230
Fax: +49-(0)6196-527072

NASDAQ: LMAT
www.lemaitre.com

France • LeMaitre Vascular SAS

Tel: +33-(0)344 260041 • Fax: +33-(0)344 260731

Italy • LeMaitre Vascular S.r.l.

Tel: +39-02 9884851 • Fax: +39-02 98497059

Spain • LeMaitre Vascular Spain S.L.

Tel: +34-(0)91 5030040 • Fax: +34-(0)91 1849871

Switzerland • LeMaitre Vascular Switzerland GmbH

Tel: 0800 561761 • Fax: +41-(0)415 608236

Sweden • LeMaitre Vascular GmbH Norden Filial

Tel: +46-35 105620 • Fax: +46-35 105625

Japan • LeMaitre Vascular, GK

Tel: +81-(0)3-5215-5681 • Fax: +81-(0)3-5215-5682

Canada • LeMaitre Vascular ULC

Tel: +1-905 673-2266-0 • Fax: +1-905 673-2223

United States • LeMaitre Vascular, Inc.

Tel: +1-781 221-2266 • Fax: +1-781 221-2223

112

Vein Stripper InvisiGrip

Vein Stripper InvisiGrip: effective length 60 cm, diameter 7F, nosepiece 5 mm.

STERILE EO 

T-handle

Description

Vein Stripper InvisiGrip designed for the removal of vena saphena magna (VSM) by invasive extirpation from the knee to the groin without a distal incision.

InvisiGrip consists of the following parts:

Vein Stripper InvisiGrip - effective length 60 cm, the nosepiece diameter of 5 mm, connected to a catheter - 7 F.

Warning

Do not perform an extirpation through a tourniquet or other stopper. The cable may be damaged.

Sterility

This product is intended FOR SINGLE USE ONLY, DO NOT REUSE. Use aseptic technique at every step of the procedure. The company «LeMaitre Vascular» is not responsible for any product that was sterilized repeatedly.

The content is sterile unless package is opened or damaged.

Germany

LeMaitre Vascular GmbH
Otto-Volger-Str. 5a/b
65843 Sulzbach/Ts.

Tel: +49-(0)6196-659230
Fax: +49-(0)6196-527072

NASDAQ: LMAT
www.lemaitre.com

France • LeMaitre Vascular SAS

Tel: +33-(0)344 260041 • Fax: +33-(0)344 260731

Italy • LeMaitre Vascular S.r.l.

Tel: +39-02 9884851 • Fax: +39-02 98497059

Spain • LeMaitre Vascular Spain S.L.

Tel: +34-(0)91 5030040 • Fax: +34-(0)91 1849871

Switzerland • LeMaitre Vascular Switzerland GmbH

Tel: 0800 561761 • Fax: +41-(0)415 608236

Sweden • LeMaitre Vascular GmbH Norden Filial

Tel: +46-35 105620 • Fax: +46-35 105625

Japan • LeMaitre Vascular, GK

Tel: +81-(0)3-5215-5681 • Fax: +81-(0)3-5215-5682

Canada • LeMaitre Vascular ULC

Tel: +1-905 673-2266-0 • Fax: +1-905 673-2223

United States • LeMaitre Vascular, Inc.

Tel: +1-781 221-2266 • Fax: +1-781 221-2223

Contraindications / Precautions

- Do not use for patients with weakened immune systems, and other patients who are expected failed wound healing.
- Infections
- Patients with active deep venous thrombosis (DVT).


The main application

The procedure of "invisible capture" using the device InvisiGrip

1. Perform the skin incision in the groin area above the saphenofemoral junction. Suture the vena saphena magna and all side branches in the anastomosis, including subfascial branches.
2. After ligation of the vena saphena magna lock Vein Stripper in the closed position, as shown in Figure 1, then insert the Vein Stripper nosepiece into vena saphena. Slide the Vein Stripper nosepiece through vena saphena magna to the level of the knee articulation.

To close the device hold down T-shaped handle and extend the base of T-shaped handle distally. Turn the base of T-shaped handle in the direction of the symbol "closed" to align the marks.

FIGURE 1: The handle and nosepiece in the CLOSED position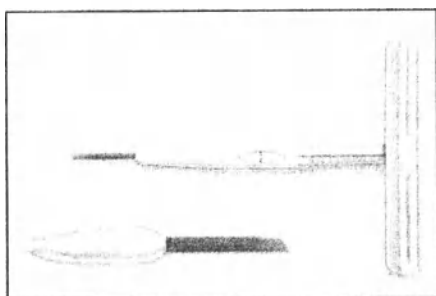


3. Insert the nosepiece of InvisiGrip into vena saphena at the safenofemorale junction and slide it to a near distance to the knee. The point of Vein Stripper nosepiece / sheath should be localized by the tubercle, which shall be formed under the skin. If there is no tubercle, the nosepiece and the sheath can be found by palpation of the upper part of the device.

4. When Vein Stripper is at the desired position, open InvisiGrip, separating the nosepiece from the sheath and expose pinch-blades as shown in Figure 2.


To open the device hold T-shaped handle down, gently pull the base of T-shaped handle and turn it in the direction of the symbol "open" to align the marks and unlock the device InvisiGrip. To open the pinch-blades move the base of T-shaped handle to the T-shaped handle.

FIGURE 1: The handle and nosepiece in the OPEN position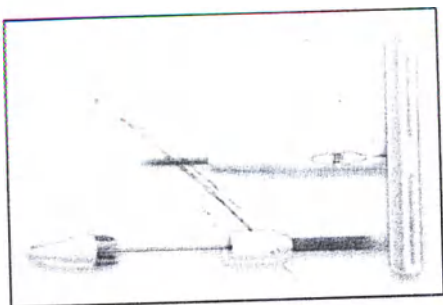


5. Press the open head of the device by gently sliding the T-shaped handle forward and backward until you feel resistance, as shown in Figure 3. Resistance is caused by spikes that are fixed into the wall of the vein.

FIGURE 3: Press the open head.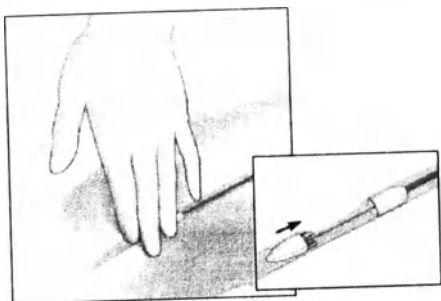


7. Holding the T-handle down, slowly turn the T-handle and the catheter around 360°. So additional spikes will act, which will provide a firm capture of the vein wall, as shown in Figure

FIGURE 4: Turn the handle around 360°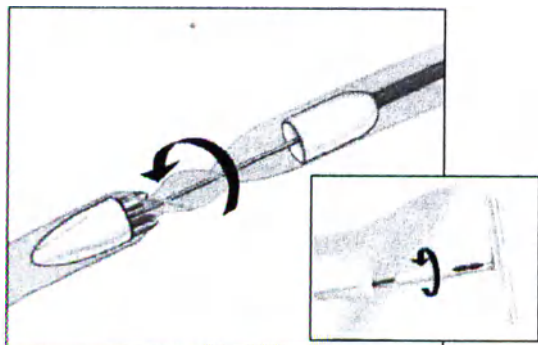

9. Holding the T-shaped handle down, slide the T-shaped handle distally and rotate the T-shaped handle to the symbol **CLOSED**, while markings are aligned, as shown in Figure 5. So the device is closed and the vein is captured.


Figure 5: Close the device.

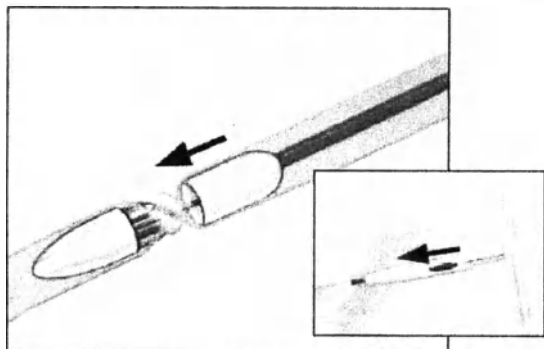11. Slowly and steady pull the device by holding the T-handle to start operate invaginated vein, as shown in Figure 6. Continue to pull and fully bring invaginated vein out.

FIGURE 6: Pull slowly.

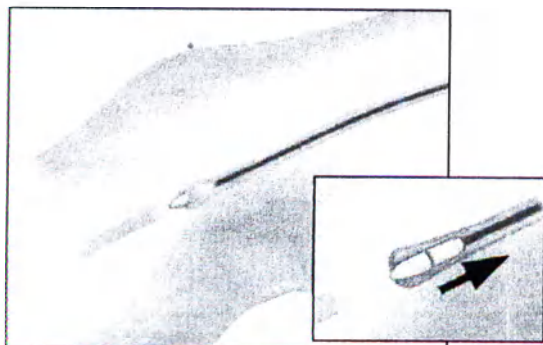13. When the vein becomes visible, replace the pulling with Vein Stripper by the straight pulling keeping the trunk of the invaginated vein using tweezers forceps or by hand.

If vein lacerates, the device can be reintroduced in the flank and moved a little proximally to the approximate location of the rupture. Then repeat the procedure until the extirpation of the whole vein will be performed.

CAUTION: Federal law (USA) restricts the sale of the device to private physicians or on the order of a physician.

Complications

As at other phlebectomy procedures complications can arise. These can include damage to the saphenous nerve, ecchymoses, seromas, leg swelling, post-operative bleeding (hematoma), infection, and thromboembolic disorders.

Labeling in accordance with ISO 15223:2000 and BS EN 980:2003.

Packaging

Vein stripper InvisiGrip is packaged in individual packaging made of a polymeric material that ensures the preservation of sterility. Number of packaging units in one box is 5.

Following symbols used for packaging:

- NAME TITLE
- MANUFACTURER
- REFERENCE NUMBER. BATCH NUMBER
- STERILE. PRODUCT STERILIZED WITH ETHYLENE OXIDE.
- SHELF LIFE
- FOR SINGLE USE ONLY
- SEE PACKAGE INSERT
- DO NOT USE IF THE PACKAGE IS DAMAGED
- SEE THE INSTRUCTION BEFORE USAGE
- RX ONLY
- EFFECTIVE LENGTH AND DIAMETER
- CE MARKING

Storage

The product should be stored in dry place away from bright light. Avoid contact with chemicals.

Shelf life

Assured shelf life of Vein stripper InvisiGrip is 5 years from the date of sterilization.

Disposal

Unused products with undamaged packaging disposed as waste Class A.

Used products that have come into contact with blood and other body fluids, as well as products with damaged packaging disposed as waste Class B.

Transportation

Transport by covered vehicles of all kinds in accordance with the shipping rules applicable to the transport of this

Warranty

Vein stripper InvisiGrip is supplied sterile. Product is sterilized with ethylene oxide. Sterility is assured if the packaging is not opened or damaged.

In the case of any defects in the period of warranty duties the owner of the device should send his claims on the following address: Angiomed Ltd, 125009, Moscow, Tverskaya st., 9/17, bld. 7, tel. +7(495) 940-65-71

		Distributed By:	#	Ø OD	— cm —	Rx only	
English	Symbol explanation	Distributor	Quantity	External diameter	Effective length	Caution: U.S. Federal and other law restricts this device to sale by or on the order of a physician.	Do not use if the package is opened or damaged.

Peter Gebauer
President International
LeMaitre Vascular

LeMaitre
VASCULAR

LeMaitre
VASCULAR

Peripheral Vision™

LeMaitre Vascular GmbH
Otto-Volger-Str. 5a/b
65843 Sulzbach/Ts.

Tel: +49-(0)6196-659230
Fax: +49-(0)6196-527072

NASDAQ: LMAT
www.lemaitre.com

119
Tel: +33-(0)344 260041 · Fax: +33-(0)344 260731

Italy · LeMaitre Vascular S.r.l.

Tel: +39-02 9884851 · Fax: +39-02 98497059

Spain · LeMaitre Vascular Spain S.L.

Tel: +34-(0)91 5030040 · Fax: +34-(0)91 1849871

Switzerland · LeMaitre Vascular Switzerland GmbH

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