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Operational documentation
Lubricant for intimate use Contex

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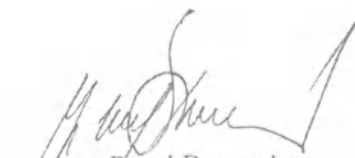
V Brně dne 04.10.2018

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Instruction for use¹
of medical device «Lubricant for intimate use Contex» in variations, legal manufacturer
«Altermed Corporation s.r.o.»

MEDICAL DEVICE NAME

Lubricant for intimate use Contex in variations:

1. Lubricant for intimate use Contex Wave;
2. Lubricant for intimate use Contex Romantic
3. Lubricant for intimate use Contex Flash
4. Lubricant for intimate use Contex Long Love
5. Lubricant for intimate use Contex Strong
6. Lubricant for intimate use Contex Green

Registration certificate number № _____

LEGAL MANUFACTURER

Altermed Corporation s.r.o. Pekarska 2997/36A, 74601 Opava, Czech Republic

Manufacturing site:

Cosmonde a.s. Dusikova 795/7, Brno, Czech Republic

Device description

Lubricant for intimate use Contex are medical devices with intended purpose to ease the unpleasant and painful sensations during intercourse with insufficient vaginal humectation, which leads for additional pleasure and comfort.

These lubricants are applied to the intimate areas prior to sexual intercourse or as required. The products are suitable for vaginal and anal sex, as indicated on instructions for use.

Lubricant ease the discomfort that may occur when natural lubrication is missing.

Gel lubricant for intimate use Contex Flash contains natural essential oils, creating a warming effect, to improve the sensation during sexual intercourse with insufficient moistening of the vaginal mucosa, which leads for additional comfort and pleasure. Produced on water-basis. Does not contain alcohol. Intended for adults. Does not destroy latex. Helps to prevent friction.

Gel lubricant for intimate use Contex Green contains green tea extract, to improve the sensation during sexual intercourse with insufficient moistening of the vaginal mucosa, which leads for additional comfort and pleasure. Produced on water-basis. Does not contain alcohol. Intended for adults. Does not destroy latex. Helps to prevent friction.

Gel lubricant for intimate use Contex Long Love contains menthol, which causes a sensation of cold receptors and can reduce sensitivity that can extend your pleasure to improve the sensation during sexual intercourse with insufficient moistening of the vaginal mucosa, which leads for additional

¹ The corresponding information included on the bottle of medical device

comfort and pleasure. Produced on water-basis. Does not contain alcohol. Intended for adults. Does not destroy latex. Helps to prevent friction.

Gel lubricant for intimate use Contex Romantic contains a sensual aroma of strawberries to improve the sensation during sexual intercourse with insufficient moistening of the vaginal mucosa, which leads for additional comfort and pleasure. Produced on water-basis. Does not contain alcohol. Intended for adults. Does not destroy latex. Helps to prevent friction.

Gel lubricant for intimate use Contex Strong contains aloe vera extract. To improve the sensation during sexual intercourse with insufficient moistening of the vaginal mucosa, which leads for additional comfort and pleasure. Produced on water-basis. Does not contain alcohol. Intended for adults. Does not destroy latex. Helps to prevent friction.

Gel lubricant for intimate use Contex Wave is moisturizing gel, which contains panthenol (pro-vitamin B5) to improve the sensation during sexual intercourse with insufficient moistening of the vaginal mucosa, which leads for additional comfort and pleasure. Produced on water-basis. Does not contain alcohol. Intended for adults. Does not destroy latex. Helps to prevent friction.

Lubricant composition

Lubricant composition

Device name	Chemical compound/raw material name
Lubricant for intimate use Contex Green	Purified water
	Glycerin
	Cellulose Gum
	Carbomer
	Camelia Sinensis (Green Tea) Leaf Extract
	Sodium Hydroxide
	Propylene Glycol, Methylparaben, Propylparaben
	Aroma Lime/Lemongrass
Lubricant for intimate use Contex Strong	Purified water
	Glycerin
	Cellulose Gum
	Carbomer
	Aloe Barbadensis Leaf Extract
	Sodium Hydroxide
	Propylene Glycol, Methylparaben, Propylparaben
	Aroma Mango
Lubricant for intimate use Contex Long Love	Purified water
	Glycerin
	Cellulose Gum
	Carbomer
	Menthol
	Sodium Hydroxide
	Propylene Glycol, Methylparaben, Propylparaben
	Aroma Melon
Lubricant for intimate use Contex Flash	Purified water
	Glycerine
	Trideceth-9, PEG-5 Ethylhexanoate,

	PEG-40 Hydrogenated Castor Oil
	Cellulose Gum
	Carbomer
	Sodium Hydroxide
	Eugenia Caryophyllus Leaf Oil
	Cinnamon Oil
	Turpentine
	Propylene Glycol, Methylparaben, Propylparaben
Lubricant for intimate use Context Romantic	Purified water
	Glycerin
	Cellulose Gum
	Carbomer
	Sodium Hydroxide
	Propylene Glycol, Methylparaben, Propylparaben
	Aroma Wild Strawberry Flavour
Lubricant for intimate use Context Wave	Purified water
	Glycerin
	Panthenol
	Cellulose Gum
	Carbomer
	Sodium Hydroxide
	Propylene Glycol, Methylparaben, Propylparaben
	Aroma Wild Fruit/Vanilla

MEDICAL DEVICE APPLICATION

Lubricants for intimate use Context are medical devices with intended purpose to ease the unpleasant and painful sensations during intercourse with insufficient vaginal mucosal humectation

PRECAUTIONS FOR USE AND POSSIBLE SIDE EFFECTS

Avoid contact with eyes. Do not apply on damaged skin. Do not use in case of container failure. Do not use in cases of hypersensitivity to some of the product ingredients. Use only as indicated. In case of accidental ingestion, seek medical advice. Do not use the product after the expiry date indicated on the bottom of packaging. Keep out of the reach of children. Do not expose to direct sunlight. In case of occurrence of adverse effects please contact immediately doctor.

Description and application.

To open for the first time: use a twisting motion along the perforation following the direction of the arrows - for 100 ml bottle; open cap – 30 ml pack. Open the bottle or tuba and squeeze out by pressing on the dispenser or the tuba. Apply the gel before and during sexual intercourse on intimate areas, or externally on a fitted condom* always in amounts as needed. After sexual intercourse, it is desirable to wash off the gel with warm water.

* Lubricant for intimate use Contex in variations are compatible with NRL condoms..

STORAGE AND SHELF LIFE

Storage

Store at the following temperature 5°C – 25°C, out of reach of children and away from direct sunlight.

Shelf- life:

1. Lubricant for intimate use Contex Wave - 3 years
2. Lubricant for intimate use Contex Romantic -3 years
3. Lubricant for intimate use Contex Flash -3 years
4. Lubricant for intimate use Contex Long Love - 3 years
5. Lubricant for intimate use Contex Strong -3 years
6. Lubricant for intimate use Contex Green -3 years

Labelling on the bottle

Labeling symbols

Symbol	Interpretation
	Name: Batch number
	Name: EXP - Use by
	Name: Do not litter!
	Name: before using read instructions
	Name: PP (Polypropylene)

Additional information can be printed on the individual package, such as trade name, importer, authorized representative, advertising information, including reference to international standards

CONSUMER COMPLAINTS

In the event of consumer complaints regarding the device, contact with authorized representative in Russia Federation «Reckitt Benckiser Healthcare LLC»

Legal address 115114 Moscow, Shluzovaya emb.,4, 3rd floor.

Phone: +7 (495) 961 25 65; fax: +7 (495) 961 25 66.

Telephone Hotline (free of charge in Russia): 8-800-200-82-20

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